INSTRUCTIONS FOR USE OF A MEDICAL PRODUCT

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults,

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1. Main Characteristics

Description: Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults is an aerosol product in pressurized metal container with internal protective anti-corrosion coating, with anatomy nose sprayer and plastic cap.

It contains a clear colorless and odorless with slightly salty taste, which is a sterile seawater solution (100 ml contain 30-33 ml of natural sea water) with isotonic NaCl concentration of 8,7-10,2 g/l and all active substances and microelements of seawater: K, Mg, Na, Ca, Cl, Se, I, Zn, Cu, Fe, Ag.

The product doesn't contain any conservants. It is not a drug.

Designed for babies from 6 months and adults.

2. Technical Specifications

Technical and functional characteristics of the product are provided in the table below:

Name of characteristic	Description
Product description and design	Aerosol in pressurized metal container with internal protective anti-corrosion coating.. A four-layer bag with content and a special nonreturn valve maintaining sterility of the content during the whole term of use by preventing backflow of content and penetration of ambient air into the bag is placed inside the container.
Package materials	Container – aluminum (ALU 8); bag with nonreturn valve: four-layer bag – polypropylene (PP 75)-polyamide (OPA 15)-aluminum foil (ALU 8) – polyester (PETP 12); valve body – polypropylene (HJ 325MO); valve: piston – polyacetal resin (POM Kepital F2503); spring – stainless steel (AISI 316); inner liner – natural rubber (DB 219); external liner – butyl (U 133); valve cup – aluminum (ALU 8); nose piece – polyethylene (PE HE7541); cap – polypropylene (PP RF830MO).
Product form	Nasal spray
Size (mm) h-high; d-diameter	Balloon (h-148±1; d-45±1), cap (h-33±1; d-33±1), diffuser (h-38±1; d-34±1)
Balloon volume (ml)	125
Weight with cap (g)	169±5
Description of contents	Clear colorless and odorless liquid with a slight salty taste, being a sterile sea water solution (100 ml contain 30-33 ml of natural sea water), with isotonic NaCl concentration
Contents composition	natural sea water 30 %, purified water 70 %
pH	6.0-8.5
NaCl content (g/l)	8.7-10.2
Storage conditions	Keep in a dry place protected from direct sunlight at +5 °C to +25 °C. Keep out of the reach of children.
Sterility	Sterile. Irradiation sterilization. Maintains sterility during the whole period of use.
The quantity of extracted substances volume (ml)	> 120.0

Relative density at 20 ° C (density solution) ml	1.000 – 1.010
Source of raw materials for the aerosol, of which the manufacturer allows the preparation of a medical device	Nitrogen, Sea water from the North of Brittany on the Pink Granite Coast (Cote de Granit Rose), in the Cotes d'Armor department, in Pleumeur-Bodou, on the island "Ile Grande" (22560), more than 22 meters deep. Purified water
Maximum pressure for unclosing of aerosol valve	should not exceed 40 N (4 kp)
Allowable steam leakage through seal of nipple	should not exceed 0,1±0,05 cm ³ during 6 hours under pressure (1,20*0,05) MPa
Stability to axial force of compression (opening/closing)	Aerosol valve should withstand axial force of compression not less 150 N (15 kp) during 500 work cycles (opening/closing)
Spray head's nozzle pressure	Spray head's nozzle should withstand pressure without falling out: Plastic nozzle - (1,00±0,05) MPa
Force of tube removal off pocket shank	Not less 60 N (6 kp). Allowable gap between tube's back and pocket - no more 0,8 mm.
Internal burst pressure	1,5 MPa
Withstand the internal pressure of balloon without causing permanent deformation	1,2 MPa;
The material of internal protective anti-corrosion coating	Internal coating: SISTIAGA® 1420-14 Aluminum Container Coating is a pigmented epoxy phenolic based lacquer for use in monobloc aerosol cans. The material has protective and anti-corrosion effect. The coating is fully cured and solid. The value of electrical current flowing through the coating is not more than 20 mA
Outer balloon coating	The outer balloon coating is chemically resistant and resistant to hot water. Mark of outer balloon coating's material is a paint SWS13-POM WeiZbatch 50, Item-No. 91-01449, manufacturer is GRAFE Color Batch GmbH, Germany.

3. Intended use

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults is intended for the prevention and treatment of acute respiratory viral infections, influenza, acute rhinitis (including allergic), sinusitis, during exacerbation of chronic rhinitis, dryness of the mucosa of the nasal cavity (subtropical and atrophic rhinitis), preparation of the nasal cavity to the application of drugs, and also for the daily hygiene of the nasal cavity.

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults has the following properties:

- thoroughly flush all parts of the nasal cavity and nasal pharynx actively clearing them from bacteria, viruses, allergens, crusts, mucus, dust particles;
- induce a beneficial effect on the nasal mucosa and nasal pharynx, reduces swelling of the nasal mucosa and nasal pharynx, helps to restore nasal breath;
- induce the moisturizing effect, relieves irritation;
- reduces the risk of complications after surgery in the nasal cavity and paranasal sinuses;
- helps to maintain normal physiological condition of the nasal mucosa and nasal pharynx;
- enhances the therapeutic efficacy of drugs applied to the nasal mucosa, reduces the need for their use and side effects.

Conditions of use: for individual use and in health care institutions.

4. Indications and contraindications:

Indications for use:

Prevention and treatment of acute and chronic inflammatory diseases of the nasal cavity, paranasal sinuses and nasal pharynx (infectious, allergic, atrophic):

- acute and chronic rhinitis (including allergic and vasomotor),
- acute and chronic sinusitis (including antritis),
- acute and chronic adenoiditis,

Prevention and treatment of acute respiratory viral infections and influenza,

Preparation of the nasal cavity to the application of drugs,

Prevention of colds during epidemics,

Daily hygiene of the nasal cavity and nasal pharynx,

Hygiene after surgery in the nasal cavity, nasal pharynx and paranasal sinuses.

Contraindications:

Individual intolerance to the components of the container contents.

Special instructions:

Pregnancy and breastfeeding are not contraindications.

Side effects are not identified.

5. Preparation for work and operation of the product

For infants from 6 months to 2 years:

The nasal lavage is performed in the supine position (Fig.1):

- Turn the child's head to one side.
- Insert the can's nozzle into the nasal passage on the top. Rinse the nasal cavity for several seconds.
- Remove residual rinsing liquid with an aspirator. Repeat the procedure if necessary.
- Perform the procedure with another nasal passage.

Fig.1

For children older than 2 years and adults:

The nasal lavage for children older than 2 years and adults (Fig.2):

- Turn the head on one side.
- Insert can's nozzle into the nasal passage on the top. Rinse the nasal cavity for several seconds.
- Remove residual rinsing liquid with an aspirator/ for a child to blow his nose. Repeat the procedure if necessary.
- Perform the procedure with another nasal passage.

Fig.2

For daily hygiene:

Flush each nasal passage 2-3 times a day (morning and/or evening).

Rheum:

Flush each nasal passage 4-6 times a day (frequently if it is necessary).

To Prevent The Common Cold:

Flush each nasal passage 3-4 times a day.

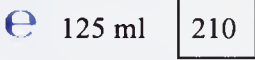
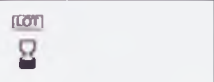
6. Sanitization indications: process the nose sprayer by solution of ethyl alcohol before application of the medical device.

7. Packaging and labelling

Metal container with internal protective anti-corrosion coating, with anatomy nose sprayer and plastic cap in cartoon box with leaflet describing the usage of MD.

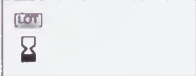
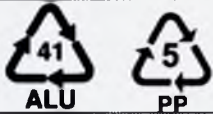
Marking

Cartoon box

Marking	Meaning
	Net weight 125 ml
	Refer to the instructions for use
 Fabriqué par : Yslab 2, rue Félix Le Dantec 29000 Quimper - France	Name and the address of manufacturer
	Code of products' certification in the European Union
	Batch code
	Expiry date (indicated on the box's bottom)
	Has been sterilized
	Packing from cartoon
	Self recycling (do not contaminate the environment)
	Non-toxic material
Warning: Pressurized container: may explode when heated. Keep away from heat, sparks, open flames, hot surfaces. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 ° C. Keep away from children. Store at temperatures between +5 ° C to +25 ° C.	The Unified Mark of products on the market of states - members of the Customs Union
	Features of use and storage

Metal container

Marking	Meaning
 125 ml 	Net weight 125 ml
 Fabriqué par : Yslab 2, rue Félix Le Dantec 29000 Quimper - France	Name and the address of manufacturer
	Manufacture date
 0459	Code of products' certification in the European Union
	Batch code Expiry date (indicated on the box's bottom)
	Has been sterilized
	Packing from aluminum (container) and plastic (nozzle and cap)
	Self recycling (do not contaminate the environment)
	Non-toxic material
	The Unified Mark of products on the market of states - members of the Customs Union
Warning: Pressurized container: may explode when heated. Keep away from heat, sparks, open flames, hot surfaces. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 ° C. Keep away from children. Store at temperatures between +5 ° C to +25 ° C.	Features of use and storage

It is also prescribed on the metal container:

- The name and address of the manufacturer, the country;
- The product name in English and Russian languages;
- The indications of the product;
- The application mode;
- Supply conditions from the pharmacies;
- Number of the registration certificate

On the cartoon box:

- The name and address of the manufacturer, the country;
- The product name in English and Russian languages;
- The indications of the product;
- The application mode;
- Supply conditions from the pharmacies;
- Address for the claims submission;
- Number of the registration certificate

8. Rules of storage, transportation, waste disposal.

Shelf life of the product – 3 years. Use before the date labeled on the package.

Store the product at the temperature from +5°C to +25°C, in a dry place protected from direct sunlight.

Keep out of the reach of children.

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults can delivered by any kind of transport in covered vehicles in accordance with the current rules of this mode of transport at a temperature within the range of from +1 °C to +40 °C.

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults refers to Class A of medical waste classification in accordance with SanPiN's sanitary - epidemiological rules and norms 2.1.7.2790-10.

Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» at the end of shelf life is disposed with other industrial waste in accordance with applicable national legislation.

9. Manufacturing organization, warranty

The product is manufactured by YS LAB, Le Forum, 2 rue Felix Le Dantec, QUIMPER 29000, France.

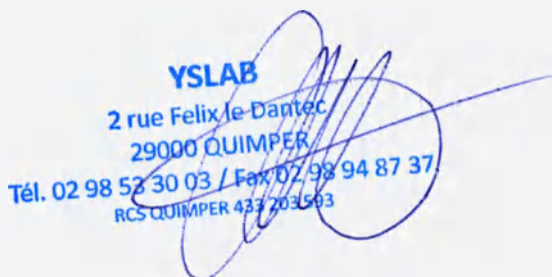
Quality of Cure for nose cavity lavage «NATUMA Aquastandard 125 ml» for children and adults and production facilities of YS LAB, France comply with all international quality standards, which is confirmed by the ISO 13485:2012 certificate №12656 Revision 5 and declaration of compliance for the product.

The manufacturer guarantees the quality of the product during the shelf life.

On the territory of the Russian Federation all the claims to quality must be send to FarmaStar Co., Ltd., 104005, Moscow, 2-ya Baumanskaya street, building 7, str.1A, tel. 8 495 969 21 84.

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