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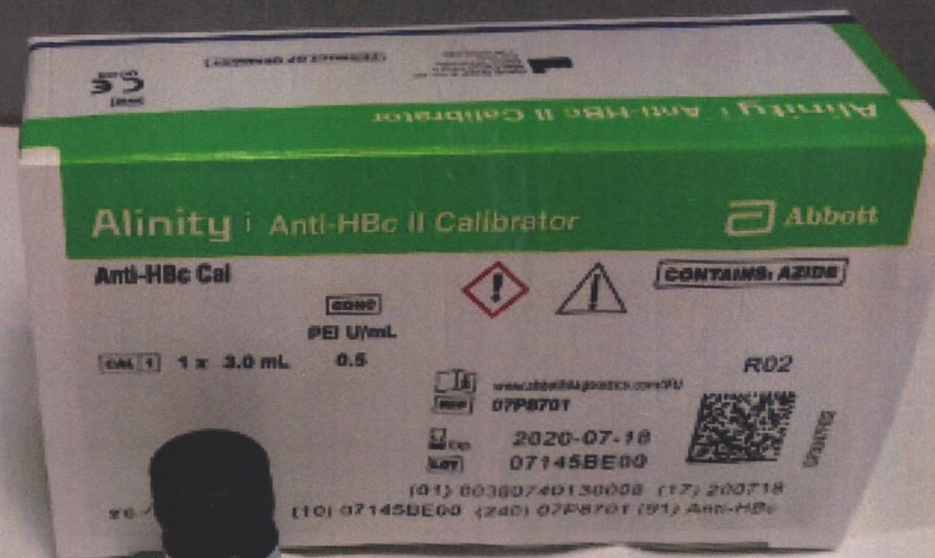
Created August 2015

NAME

Alinity i Anti-HBc II Calibrator (also referred to as Anti-HBc Cal)

INTENDED USE

The Alinity i Anti-HBc II Calibrator 1 is for the calibration of the Alinity i analyzer when used for the quantitative detection of antibody to hepatitis B core antigen (anti-HBc) in human serum and plasma. For additional information, refer to the Alinity i Anti-HBc II reagent package insert and the Alinity ci-series Operations Manual.


CONTENTS

1 calibrated human plasma and dyes. The calibrator

en
Anti-HBc
07P8701
071212R02
57P870

The following warnings and precautions apply to [CAL 1]

WARNING	Contains methylisothiazolone and sodium azide
H317	May cause an allergic skin reaction
EUH032	Contact with acids liberates very toxic gas.
Prevention	
P261	Avoid breathing mist / vapors / spray
P272	Contaminated work clothing should not be allowed out of the workplace.
	Wear protective gloves / protective clothing / eye

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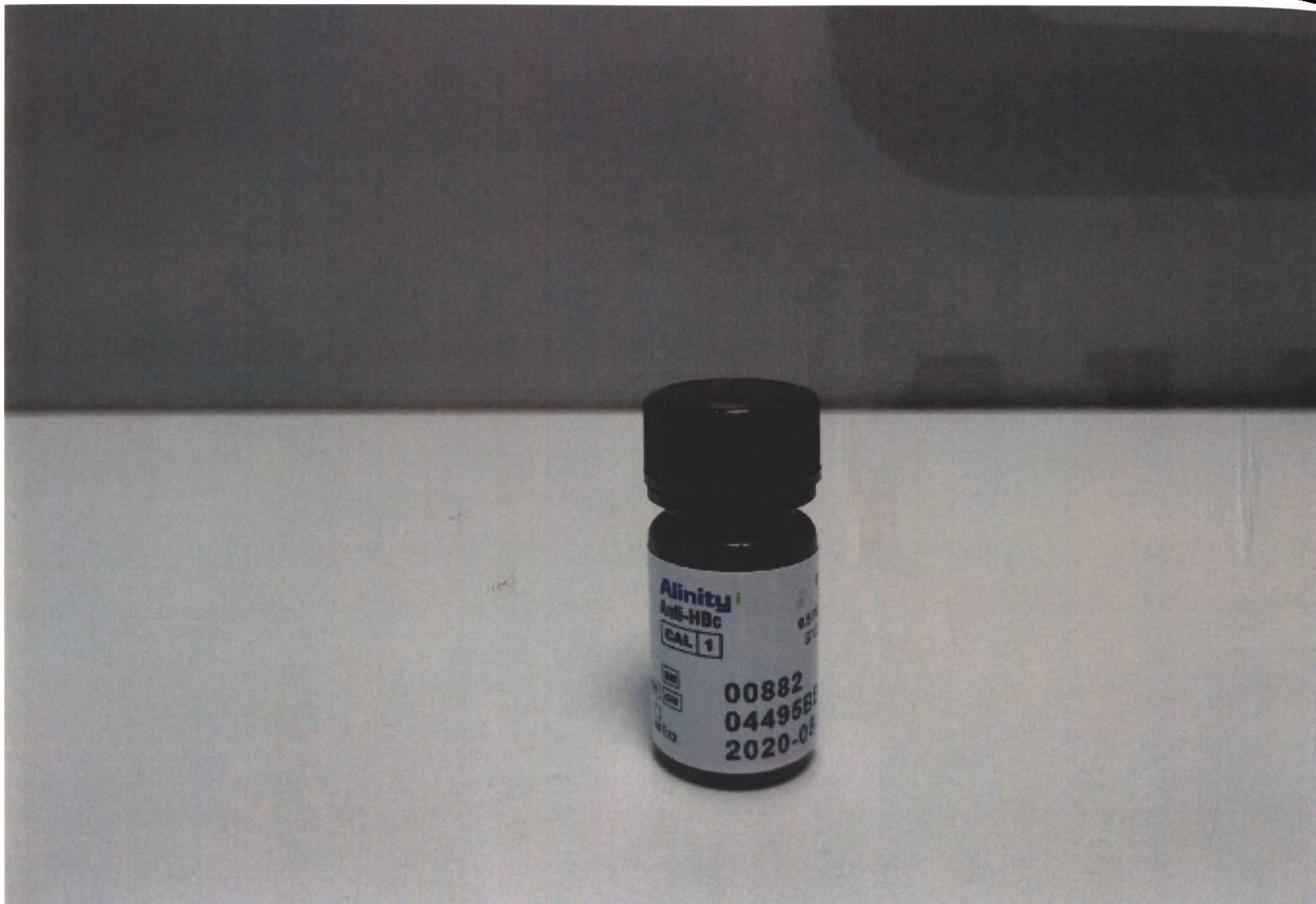The label layout includes the following elements:

- Top Right:** A temperature control diagram showing a vertical line with a horizontal bar at the top labeled '2°C' and a horizontal bar at the bottom labeled '8°C'.
- Left Side (Vertical Text):** '070347905' and 'www.abbottdiagnostics.com/IFU'.
- Center:** 'LOT' with a box icon, 'REF' with a box icon, and '07P8701'.
- Bottom Center:** 'CAL 1' in a box, '1 x 3.0 mL', and '0.5'.
- Bottom Left:** 'CONTAINS AZIDE' in a box, a warning triangle with an exclamation mark, and a red diamond warning symbol.
- Bottom Right:** 'Anti-HBc Cal' and 'CONC'.
- Green Bar:** A large green bar containing the Abbott logo, 'Alinity i Anti-HBc II Calibrator', and 'Alinity i Anti-HBc II Calibrator'.
- Bottom Right (Small):** 'Anti-HBc Cal' and a temperature control diagram with '2°C' and '8°C'.
- Bottom Left (Small):** 'Abbott GmbH', 'Max-Planck-Ring 2', '65205 Wiesbaden', 'Germany', '+49-6122-580', and 'PRODUCT OF GERMANY'.
- Bottom Center (Small):** 'IVD' and 'CE 0123'.

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