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COMPOSITION

- Cross-linked hyaluronic acid (BDDE) 19 mg
- Phosphate buffer, pH 7.2, and mannitol qs 1g
- 1 ml pre-filled syringe

DESCRIPTION

DESIRIAL® is a sterile, non-pyrogenic, resorbable, cross-linked hyaluronic acid gel of non-animal origin, with physiological pH and osmolarity. It is packaged in 1ml pre-filled syringes and sterilised by autoclaving.

DESIRIAL® is a single-use product.

Each box contains two 1 ml syringes of DESIRIAL®, 2 x 30G ½ and 2 x 27G ½ sterile, single-use needles, an instruction for use and labels with the batch number, one of which should be given to the patient and the other attached to the doctor's patient file.

INDICATIONS

DESIRIAL® is an injectable gel indicated in:

- the rehydration of vaginal and vestibular mucous membranes,
- slight hypotrophy of the vulvar labia majora and the pubis, by intradermal injections.

MODE OF ACTION

The product is injected in the tissue where it will restore the tone and tension of the area of connective tissue. The hydration and volume capacity originate from the ability of cross-linked hyaluronic acid to bind water.

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Depending on the characteristics of the treated area, the patient and the depth of the injection, one to two injection sessions are necessary for best results, as the persistence of the product varies from patient to patient (from 6 to 12 months post-treatment based on clinical studies results). Regular retouching sessions can help maintain the desired correction.

CONTRAINDICATIONS

DESIRIAL® must not be used:

- For injections other than into the vulvar labia majora, the mons veneris or the vaginal and vestibular mucous membranes
- In clitoral areas and in the labia minora
- In areas too close to the urethra (no injection in the ventral wall of the vagina) or within the urethra
- In patients with a tendency for developing hypertrophic scars
- In patients with a known hypersensitivity to one of the ingredients (hyaluronic acid, mannitol)
- In pregnant or breastfeeding women or in children
- In areas presenting clinical signs of inflammation and/or infection of bacterial, fungal (e.g. thrush) or viral (e.g. papilloma, herpes) origin and/or in patients following treatment for one of these pathologies
- In blood vessels

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DESIRIAL[®] must be injected by legally authorised practitioners with a good anatomical knowledge of the site to be treated and with experience using similar filling products injection techniques.

- Before carrying out the treatment, it is essential:
 - to review the patient's medical history together with the patient
 - to explain the indications and expected results with DESIRIAL[®] to her, and to verify that the patient's expectations can be fulfilled
 - to explain the contraindications, the precautions for use, the potential adverse events linked to the treatment as well as the "Advice to patients"
- Adhere to the rules for asepsis and handling related to this type of procedure.
- If stored in the refrigerator, bring the product to room temperature before injection.
- A preliminary anaesthesia (topical or lidocaine injection) may be used; this is the responsibility of the doctor.
- The practitioner will select an appropriate needle for the area to be treated, the type of correction desired and the preferred injection technique.

The needle must be firmly screwed onto the Luer Lock tip of the syringe.

- Injections in the vaginal and vestibular mucous membranes
The injections are carried out in the vestibule and on the posterior and lateral walls of the vagina, 2 to 3 cm after the introitus using for example the multi-puncture technique.

- Injections in the Labia Majora
The injections are carried out along the longitudinal axis of the labia majora by linear retro-tracing techniques, either fan or lattice.

- Inject slowly.
- If injected too superficially, skin discolouration or slight indurations may appear or the correction may be uneven.
- The dose required to obtain an optimum correction is determined by the doctor. Do not overcorrect.
- Do not use an excessive quantity of DESIRIAL[®].
- It is advised not to use more than 2.0 ml per treated area in a single session, and not to inject more than 20 ml of cross-linked hyaluronic acid per person per year. The safety of injecting greater amounts has not been established.
- If the needle is blocked, do not increase the pressure required for the injection and replace the needle.
- After the injection, massage the treated area well to optimise the evenness of the correction.
- A follow-up consultation a few days after the injection is advised.

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The practitioner must inform the patient of possible side effects that may occur following the injection of DESIRIAL®, especially:

- Inflammatory reactions such as redness, oedema or erythema that may be combined with itching and/or pain at the injection site and which generally disappear after a few days.
- Induration(s) or nodules, colouration or discolouration in the injected area.
- Slight bleeding at the injection site.
- Post-injection pain.
- Haematomas.
- Difficulties urinating.
- Rare cases of necrosis, abscess, granulomas and hypersensitivity after injections of hyaluronic acid into the face have been described in the literature. The patient must be informed of this.
- In patients with a severe predisposition to allergies, dermatological disease, problems with haemostasis or inflammatory disease or if the precautions for use are not adhered to, the incidence of adverse events may be increased.
- Any of the aforementioned side effects that persist for more than a week or the appearance of any other adverse effect must be notified to the doctor by the patient. The doctor must remedy this using an appropriate treatment and notify the distributor or the manufacturer about the problem as soon as possible.

STORAGE – EXPIRY DATE

- DESIRIAL® must be used before the expiry date shown on the packaging and the syringe.
- DESIRIAL® must be stored between 2°C and 25°C, away from frost and light.
- Non-adherence to the storage conditions could alter the performance of the product.

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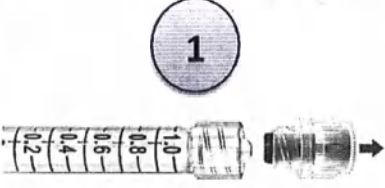
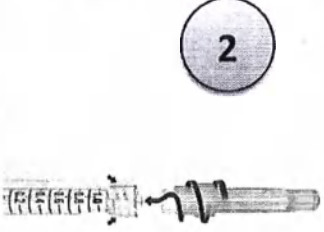
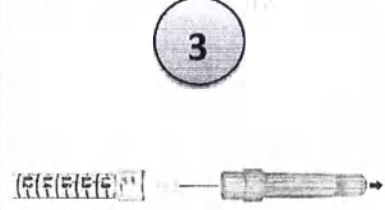
Desirial

EN

			
EN	1. Remove the end cap of the syringe carefully.	2. Hold the ring at the end of the syringe and firmly insert the hub of the needle into the Luer-lock end of the syringe, by screwing until you feel some strong counter pressure .	3. Remove the needle cap.

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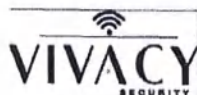
Sterilised using steam or dry heat. Syringe contents have been sterilised using moist heat.

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Injection system with a bi-material technology and patented ergonomics.

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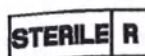
Patented cross-linking technology.

Laboratoires VIVACY
252 rue Douglas Engelbart
Archamps Technopole
74160 ARCHAMPS - FRANCE
Tel. : +33 450 317 181
Fax. : +33 450 317 191
contact@vivacy.fr

The CE marking was obtained in compliance with Directive 93/42/CEE relating to medical devices. Desirial® obtained CE marking in 2011. 0344 corresponds to the number of the Notified Body.

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20

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Certified copy
Sonia BEY
Quality & RA Director

UJMA Aline,
Vu exclusivement pour
certification matérielle de la signature
de (Seen exclusively to certify the
above signature) SONIA BEY

Desirial® Plus EN

COMPOSITION

- Cross-linked hyaluronic acid (BDDE) 21 mg
- Phosphate buffer, pH 7.2, and mannitol qs 1 g
- 2 ml pre-filled glass syringe

LABORATOIRES VIVACY

252 Rue Douglas Engelbart
Archamps Technopole
74160 ARCHAMPS

Tel: 04 50 31 71 81 - Fax 04 50 31 71 91

Siret 498 485 275 00057 - Code APE 2120Z

N° TVA FR 47 498 485 275

SAS AU CAPITAL DE 1 744 200 €

Email : accueil@vivacy.fr

DESCRIPTION

DESIRIAL® PLUS is a sterile, non-pyrogenic, resorbable, cross-linked hyaluronic acid gel of non-animal origin, with physiological pH and osmolarity. It is packaged in 2ml pre-filled glass syringes and sterilised by autoclaving.

DESIRIAL® PLUS is a single-use product.

Each box contains one 2 ml syringe of DESIRIAL® PLUS, 2 x 27G ½ sterile, single-use needles, 1 x 19G needle (for the cannula pre-hole), 1 x 18G cannula, an instruction for use and labels with the batch number, one of which should be given to the patient and the other attached to the doctor's patient file.

INDICATIONS

DESIRIAL® PLUS is an injectable gel indicated for moderate hypotrophy, severe hypotrophy or atrophy of the vulvar labia majora, by subcutaneous injections.

MODE OF ACTION

The product is injected in the tissue where it will restore volume, to rehydrate and to add tone and tension to the area of connective tissue. The volume capacity originates from the gel and the ability of cross-linked hyaluronic acid to bind water.

The product will be resorbed slowly over time.

Depending on the characteristics of the treated area, the patient and the depth of the injection, one to two injection sessions are necessary for best results, as the persistence of the product varies from patient to patient (from 6 to 12 months post-treatment based on clinical studies results).. Regular retouching sessions can help maintain the desired correction.

CONTRAINDICATIONS

DESIRIAL® PLUS must not be used:

- For injections other than into the subcutaneous areas of the vulvar labia majora
- In clitoral areas and in the labia minora
- In areas too close to the urethra (no injection in the ventral wall of the vagina) or within the urethra
- In patients with a tendency for developing hypertrophic scars
- In patients with a known hypersensitivity to one of the ingredients (hyaluronic acid, mannitol)
- In pregnant or breastfeeding women or in children
- In areas presenting clinical signs of inflammation and/or infection of bacterial, fungal (e.g. thrush) or viral (e.g. papilloma, herpes) origin and/or in patients following treatment for one of these pathologies
- In blood vessels

PRECAUTIONS FOR USE

- It is strongly recommended that the free and informed consent of the patient is obtained prior to treatment.

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de (Seen exclusively to certify the

above signature) SONIA BEY

A thorough pre-injection examination must be performed to eliminate the possibility of infection (fungal, viral or bacterial) or vulvo-vaginal inflammation.

- If the patient has a history of genital herpes, there is a risk that needle punctures may trigger a new episode of genital herpes.
- Do not inject close to menstruation.
- The current rules for asepsis for injections must be rigorously observed.
- Do not use quaternary ammonium derivatives during disinfection as there is a risk of irritating the genital mucous membranes.
- Do not inject into blood vessels or muscles. Any accidental intravascular injection may cause vascular occlusion which may result in rare but serious complications such as necrosis of the skin/underlying tissues.
- Ensure that the needle has penetrated correctly before injection.
- Use the cannula/needles supplied with DESIRIAL® PLUS, as the combination of these two devices has been validated.
- For patients:
 - having a history of or currently having an autoimmune disease
 - having a history of or currently having cancer in areas close to the injection site,

the practitioner must decide whether to inject on a case-by-case basis, depending on the nature of the illness as well as the associated treatment. In addition, he/she must ensure special monitoring of such patients, in particular offering them a double test beforehand and not injecting if the disease is active.

- Patients with a history of streptococcal disease such as recurrent sore throat or acute rheumatic fever must undergo a double test before any injection. In the case of acute rheumatic fever with cardiac localisation, injection is not recommended.
- It is recommended that injectable gels in the DESIRIAL® line are not injected into an area that has been corrected by injectable gels other than those in the DESIRIAL® line, as no clinical data is available.
- If there is increased pain during the injection, stop the procedure and withdraw the needle or cannula.
- Single-use product. Do not re-use.
An injectable gel must only be used for a single patient to prevent any risk of cross-contamination.
- Inject immediately after opening; discard after injection. Cannot be re-sterilised in any circumstances.
- As soon as the injection is finished, discard the syringe and the residual product and discard the needles/cannula in an appropriate container. An unused medical product is classified as Class A waste, used in Class B and must be disposed of by local public law organizations in accordance with SanPin 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of the territories of urban and rural settlements, for water objects, drinking water and drinking water supply of the population, atmospheric air, soils, residential premises, operation of industrial, public premises, organization and conduct sanitary and anti-epidemic (preventive) measures"
- Do not use if the packaging is open or damaged.
- Do not mix with other products.
- The composition of this product is compatible with fields used for magnetic resonance imaging.

DOSAGE – MODE OF ADMINISTRATION

DESIRIAL® PLUS must be injected by legally authorised practitioners with a good anatomical knowledge of the site to be treated and with experience using similar filling products injection techniques.

- Before carrying out the treatment, it is essential:

to review the patient's medical history together with the patient

to explain the indications and expected results with DESIRIAL® PLUS to her, and to verify that the patient's expectations can be fulfilled

- to explain the contraindications, the precautions for use, the potential adverse events linked to the treatment as well as the "Advice to patients"

- Adhere to the rules for asepsis and handling related to this type of procedure.
- If stored in the refrigerator, bring the product to room temperature before injection.
- Anaesthesia is strongly advised, under the responsibility of the doctor.
- The injections are performed with the 18G cannula or with the 27G ½ needles.
- The needle/cannula must be firmly screwed onto the Luer Lock tip of the syringe.
- The injection is carried out sub-cutaneously along the longitudinal axis of the labium majorum using a linear retro-tracing technique.
- Inject slowly.
- If injected too superficially, skin discolouration or slight indurations may appear or the correction may be uneven
- The dose required to obtain an optimum correction is determined by the doctor. Do not overcorrect.
- Do not use an excessive quantity of DESIRIAL® PLUS.
- It is advised not to use more than 4.0 mL per labium majus in a single session, and not to inject more than 20 ml of cross-linked hyaluronic acid per person per year. The safety of injecting greater amounts has not been established.
- If the needle/cannula is blocked, do not increase the pressure required for the injection and replace the needle/cannula.
- After the injection, massage the treated area well to optimise the evenness of the correction.
- A follow-up consultation a few days after the injection is advised.

ADVICE TO PATIENTS

- Avoid taking aspirin, NSAIDs, platelet aggregation inhibiting drugs, anticoagulants or Vitamin C in the week preceding the injection.
- Slight bleeding may be observed after the injection.
- Notify your doctor if you are undergoing treatment with anticoagulants or if you are suffering from a haemostatic disorder (increased risk of haematomas).
- After the injection, avoid going to wet locations (swimming pools, gyms, steam rooms, jacuzzis, etc.) for 10 days.
- Avoid any sexual activity for 5 days.
- Avoid any activities that put pressure on the treated area (cycling, horseback riding, etc.) for several days.

ADVERSE EFFECTS

The practitioner must inform the patient of possible side effects that may occur following the injection of DESIRIAL® PLUS, especially:

- Inflammatory reactions such as redness, oedema or erythema that may be combined with itching and/or pain at the injection site and which generally disappear after a few days.
- Induration(s) or nodules, colouration or discolouration in the injected area.
- Slight bleeding at the injection site.
- Post-injection pain.
- Local mobility of the injectable gel
- Haematomas.

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- Rare cases of necrosis, abscess, granulomas and hypersensitivity after injections of hyaluronic acid into the face have been described in the literature. The patient must be informed of this.
- In patients with a severe predisposition to allergies, dermatological disease, problems with haemostasis or inflammatory disease or if the precautions for use are not adhered to, the incidence of adverse events may be increased.
- Any of the aforementioned side effects that persist for more than a week or the appearance of any other adverse effect must be notified to the doctor by the patient. The doctor must remedy this using an appropriate treatment and notify the distributor or the manufacturer about the problem as soon as possible.

STORAGE – EXPIRY DATE

- DESIRIAL® PLUS must be used before the expiry date shown on the packaging and the syringe.
- DESIRIAL® PLUS Must be stored between 2°C and 25°C, away from frost and light.
- Non-adherence to the storage conditions could alter the performance of the product.

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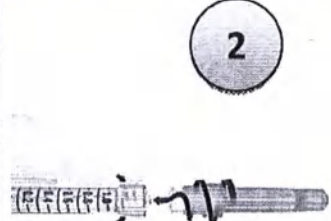
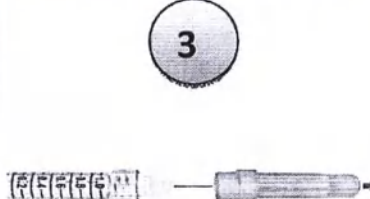
de (Seen exclusively to certify the

above signature) SONIA BEY

			
EN	1. Remove the end cap of the syringe carefully.	2. Hold the ring at the end of the syringe and firmly insert the hub of the needle into the Luer-lock end of the syringe, by screwing until you feel some strong counter pressure.	3. Remove the needle cap.

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Caution: Consult the instruction for use.

Temperature limits. Store between 2°C and 25°C (35.6° F and 77° F).

Expiry date. Use before the date shown.

Keep away from sunlight.

Batch code.

Fragile; handle with care.

Do not reuse.

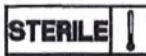
Do not use if package is damaged.


Sterile fluid pathway. Syringe contents have been sterilised using moist heat.

Manufacturer.

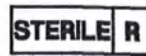Sterilised using steam or dry heat. Syringe contents have been sterilised using moist heat

Sterilised using irradiation. Needles/cannulas have been sterilised using irradiation.

Sterilised using ethylene oxide. Needles/cannulas have been sterilised using ethylene oxide.

UPX-Like TECHNOLOGY® Patented cross-linking technology

RFID-based remote authentication system for VIVACY products (Radio Frequency Identification).

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0344

The CE marking was obtained in compliance with Directive 93/42/CEE relating to medical devices. Desirial® Plus obtained CE marking in 2011. 0344 corresponds to the number of the Notified Body.

Certified copy
Sonia BEY
Quality & RA Director

LABORATOIRES VIVACY
252 Rue Douglas Engelbart
Archamps Technopole
74160 ARCHAMPS
Tel. 04 50 31 71 81 - Fax 04 50 31 71 91
Siret 498 485 275 00057 - Code APE 2120Z
N° TVA FR 47 498 485 275
SAS AU CAPITAL DE 1 744 200 €

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