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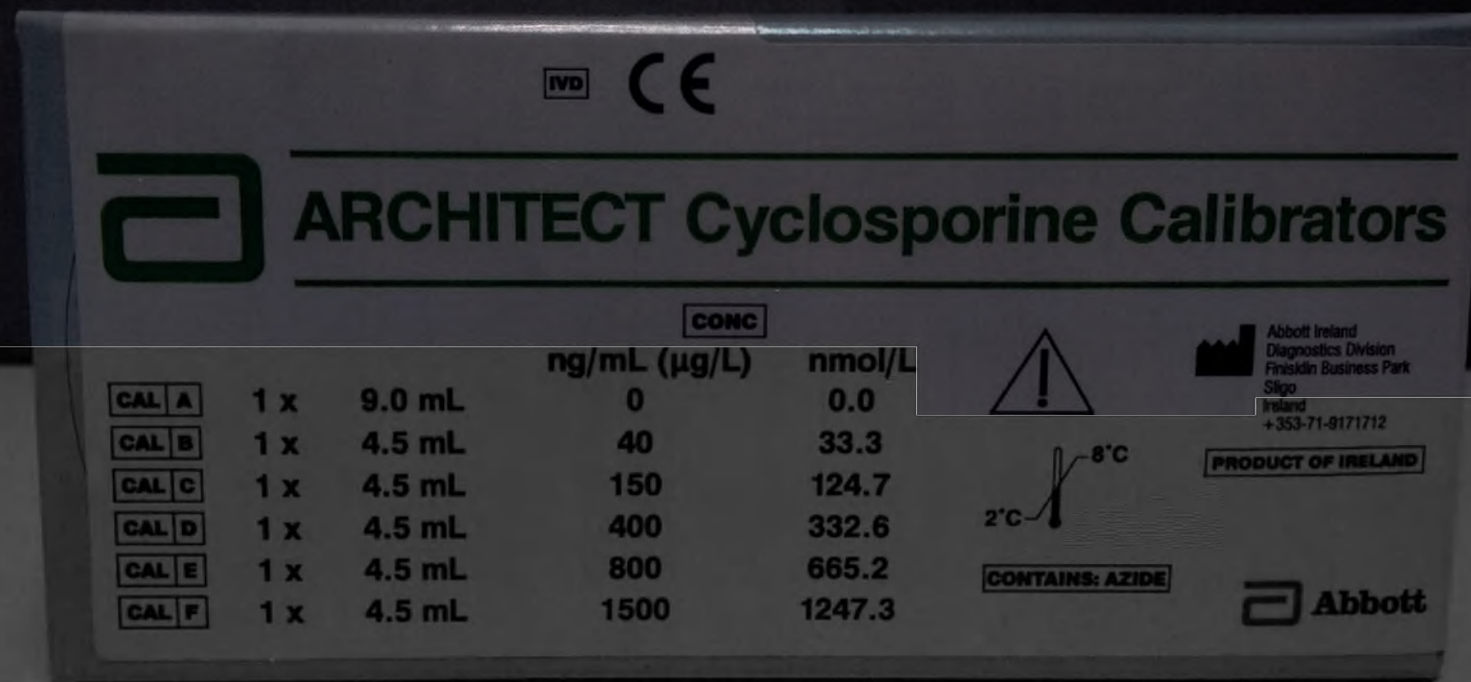
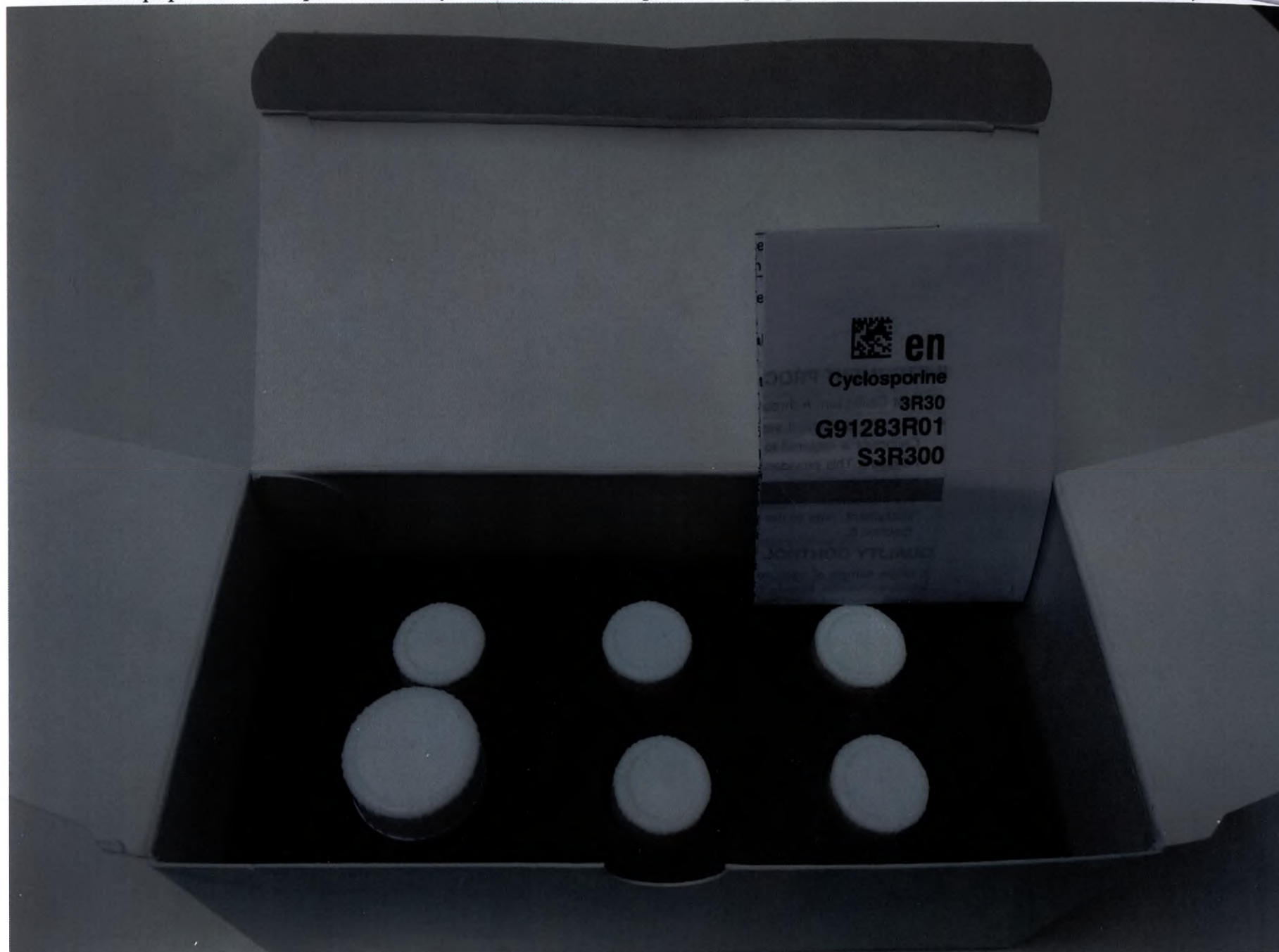
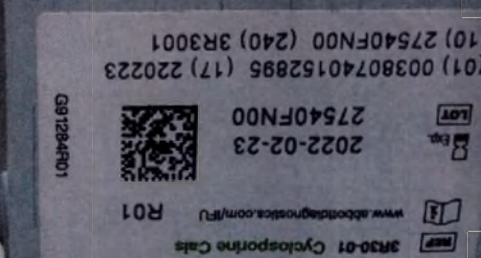
ARCHITECT Cyclosporine Calibrators

		ng/mL (µg/L)	nmol/L
CAL A	1 x 9.0 mL	0	0.0
CAL B	1 x 4.5 mL	40	33.3
CAL C	1 x 4.5 mL	150	124.7
CAL D	1 x 4.5 mL	400	332.6
CAL E	1 x 4.5 mL	800	665.2
CAL F	1 x 4.5 mL	1500	1247.3

CAUTION: This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other precautions should be used for materials that

Abbott

ARCHITECT Cyclosporine Calibrators

Created June 2019.

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

For laboratory professional use only.

NAME

ARCHITECT Cyclosporine Calibrators (also referred to as Cyclosporine Cals)

INTENDED USE

The ARCHITECT Cyclosporine Calibrators are for the calibration of the ARCHITECT Cyclosporine Calibrator.

Safety Precautions

CAUTION: This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other precautions should be used for materials that

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