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Approved by:

Dr. Elke Barsties

Manager Regulatory Affairs

Date: 12. December 2013

Only for professional dental use.

INSTRUCTION FOR USE

AH Temp root canal dressing

manufactured by

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany

1. Filling dental material AH Temp is an aqueous, 12,0 pH calcium-hydroxide based temporary root canal dressing for direct application into the root canal.

- calcium hydroxide has antibacterial effect, stimulates apex fixation and promotes periapical healing; antibacterial properties of the AH Temp paste are equal to conventional calcium hydroxide dressing;
- composition of AH Temp paste provides proven compatibility with AH Plus root canal sealants;
- reduced desiccation of AH Temp paste leads to reduced blocking of the cannula during application and storage;
- easy navigation through root canal with ultra-thin, long and preangulated AH Temp cannula;
- excellent radioopacity and X-Ray visibility

Delivery forms: syringe contains 0,75 ml AH Temp paste.

AH Temp cannulas used for direct placement of the paste into root canal. Single-use, non-sterile: angulation 45°, working length - 25 mm, outer diameter 0,41 mm (nominal), inner diameter 0,31 mm (nominal) with silicone stopper.

Finger grip is used for more convenient syringe's grip during the procedure as it limits plunger's movement.

2. Ingredients

Calcium hydroxide, polyethylene glycol, calcium tungstate, zirconium oxide, water.

3. Indications.

Intermediate, inter-appointment root canal dressing.

4. Contraindications.

None known.

5. Warnings.

The material contains calcium hydroxide which is basic and which may cause burns of skin, eyes and oral mucosa.

Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.

Avoid skin contact to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If irritation of skin or rash occurs, seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Avoid swallowing to prevent inflammation. If swallowed, immediately rinse out mouth and then drink plenty of water. Seek medical attention.

6. Precautions.

This product is intended to be used only as specifically outlined in this Instruction for Use.

Use protective measures for the dental team and patients such as glasses & gloves in accordance with local best practice. Use adequate isolation, such as rubber dam as contact with saliva, blood and sulcus fluid during application may reduce the efficacy of the dressing.

Tightly close syringes immediately after use with original cap.

Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.

Deliver paste with AH Temp cannulas staying at least 2 mm short of the apex, always keeping the cannula loose in the canal. Don't deliver with plastic tips which can occlude tight in the canal.

The AH Temp syringe and the finger grip cannot be reprocessed. To prevent AH Temp syringe and finger grip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory the the syringe and finger grip are handled with clean/disinfected gloves.

Cannulas AH Temp are intended for single-use. Utilize them right after use. Do not re-use in order to avoid cross-contamination.

7. Adverse reactions.

Eye contact: irritation and possible cornea damage.

Skin contact: irritation; reddish rashes may be seen on the skin.

Contact with Mucous Membranes: inflammation (see Warnings)

8. Storage conditions, shelf-life.

AH Temp must be stored between +2°C and +28°C.

Use the product at room temperature.

Do not freeze.

Keep out of direct sunlight and protect from moisture.

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

Syringes must be tightly closed between procedures.

Shelf-life of AH Temp – 2 years.

Shelf-life of cannulas AH Temp – 3 years.

Shelf-life of Finger Grip – 3 years.

Do not use after expiration date.

Do not use syringe AH Temp 3 months after the first usage.

9. Step-by-step instruction.

9.1. Preparation of the root canal

Prior to the application of the material prepare, clean and dry the root canals to be filled using **appropriate** endodontic techniques.

9.2. Application.

Overfilling.

Damage of periapical tissue.

- Do not place too much material to prevent overfilling of the root canal.
- If overfilling occurs the material is usually tolerated very well by the surrounding tissue. If however material is pressed into the mandibular canal, immediately apply state of care measures.

Clogging.

Separation of cannula from syringe.

- Do not allow material to dry inside.
- Do not reuse steel cannulas.
- Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.
- Do not use syringe, once removed from pouch after 3 months

1. Remove cap from the end of the syringe.

2. To assure free flow of material from syringe, express a small amount onto a mixing pad.

3. Attach disposable cannula over the protective barrier to the end of the syringe.

4. Turn cannula clockwise until fully seated, then tug on cannula to assure locking into syringe collar.

5. Attach finger grip with wings pointing towards front of syringe.

6. To assure free flow of material from the cannula extrude a small amount onto the mixing pad

Excessive force.

- Apply slow and steady pressure on the syringe.
- Always make sure cannula is properly connected to Luer-Lock before application. The material should extrude easily.

7. If paste does not flow freely, remove syringe from patient field and check for obstruction.

8. Insert cannula short of the apex. Apply paste slowly withdrawing cannula from the root canal.

9. Remove surplus of paste, place cotton pellets and or temporary restoration to tightly seal the coronal openings of the root canals.

10. Right after use remove finger grip.

12. Detach disposable cannula by turning the cannula counter-clockwise.

13. Replace the cap on the syringe head with clean/disinfected gloves.

Tightly sealing of the endodontic cavity using temporary restorative material is essential in order to avoid the contamination of the root canal system between the appointments.

9.3. Removal.

To achieve optimal disinfection of the root canal, calcium hydroxide root canal dressings should remain between seven days and four weeks in the root canal. If used as a long-term root canal dressings, the dressing should be changed every three months.

1. Prior to obturation of the root canal with AH Plus, AH Plus Jet or AH 26 root canal sealant and root canal points clean AH Temp residues from canal walls with root canal preparation instruments and rinse with sterile water.

2. Perform a final rinse with sterile water, dry and obturate.

9.4. Hygiene during the procedure.

Syringe AH Temp and finger grip.

- Cannot be reprocessed, so it is necessary to avoid their contamination with body fluids.
- To prevent the AH Temp syringe and finger grip from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the syringe and finger grip are handled with clean/disinfected gloves. Do not reuse syringe/finger grip if contaminated.

Steel cannulas – single use.

Right after use, carefully remove cannula.

Do not clean, disinfect or reuse steel cannulas.

Clogging.

Separation of cannula from syringe.

- Do not allow material to dry inside.
- Do not reuse steel cannulas
- Do not use excessive force in order to prevent damage to the steel cannulas.
- Do not use syringe, once removed from pouch after 3 months.

10. Transportation conditions.

Transportation conditions standards, at temperature from +10°C to +30°C during period not longer than 14 days. Small deviations from these temperatures are allowed.

11. Environment protection requirements.

Material does not have negative impact on environment during its processing cycle.

12. Utilization.

AH Temp syringes, finger grips, steel cannulas should be utilized in accordance with protocols of medical establishment and local legislation. There are no special requirements on utilization.

13. Manufacturer's guarantee.

Manufacturer guarantees quality of the material providing that transportation and storage conditions are observed and material is used strictly in accordance with the Instruction for Use.

All quality related questions should be addressed to Manufacturer's Notified Representative.

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