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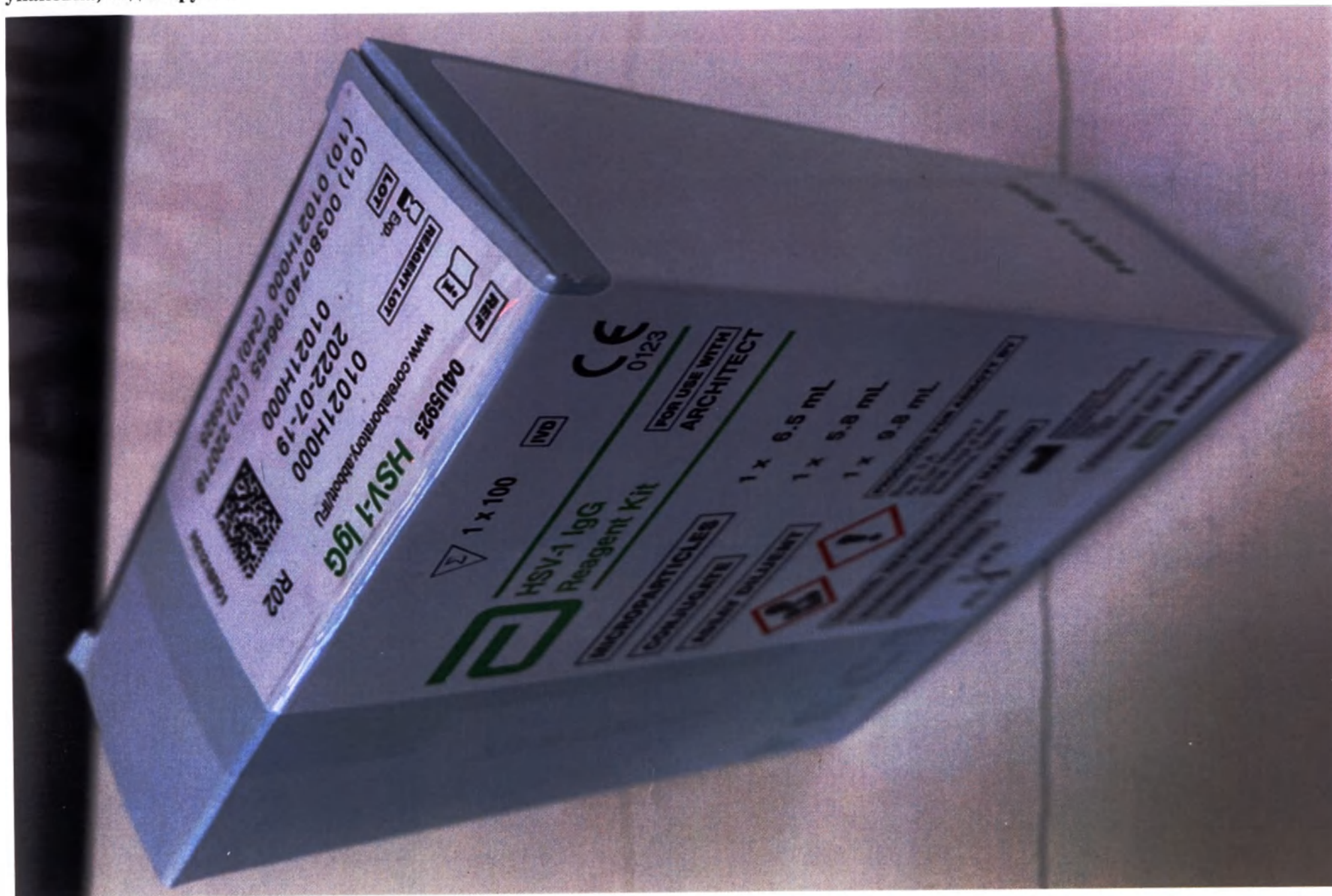
FOR USE WITH
ARCHITECT

Revised September 2021.

Instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from these instructions.

For laboratory professional use only.

■ NAME

HSV-1 IgG

■ INTENDED USE

The HSV-1 IgG assay is a chemiluminescent microparticle immunoassay (CMIA) used for the qualitative determination of specific IgG antibodies to herpes simplex virus type 1 (HSV-1) in human serum and plasma on the ARCHITECT i System.

The HSV-1 IgG assay is to be used for testing sexually active adults or expectant mothers to aid in the presumptive diagnosis of HSV-1 infection. The HSV-1 IgG assay is to be used as an aid in the determination of immune status and as an aid in the diagnosis of HSV infection in individuals including expectant mothers. The predictive value of a positive or negative result depends on the prevalence of HSV-1 infection in the population and the pre-test likelihood of HSV-1 infection.

NOTE: The performance of the HSV-1 IgG assay has not been established for use in the pediatric population, for neonatal screening, or for testing immunocompromised or immunosuppressed

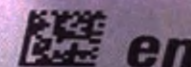
immunocompromised, and neonatal herpes.⁴ Neonatal herpes, a potentially fatal infection, is one of the most serious complications of genital herpes. A woman may transmit the infection to her child during pregnancy, or the infant may be infected during childbirth or shortly after birth. Women who acquire genital herpes close to the time of delivery or who are shedding virus during delivery, increase the infant's risk for developing neonatal herpes.⁵

Clinical diagnosis based on the characteristic lesions (orofacial or genital) may be confirmed with laboratory testing. However, definitive diagnosis can be made by type-specific serology, culture, polymerase chain reaction, and antigen detection.⁴ Type-specific serology tests can help with the identification and diagnosis of HSV infection.⁵

■ BIOLOGICAL PRINCIPLES OF THE PROCEDURE

This assay is an automated, two-step immunoassay for the qualitative determination of specific IgG antibodies to HSV-1 in human serum and plasma using chemiluminescent microparticle immunoassay (CMIA) technology.

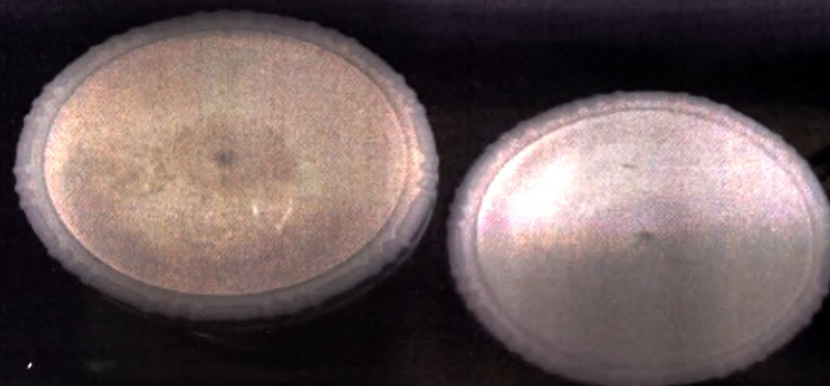
Sample, HSV-1 specific recombinant gG1 antigen coated paramagnetic microparticles, and assay diluent are combined and incubated. The IgG antibodies to HSV-1 (HSV-1 IgG) present in the sample bind to the HSV-1 specific recombinant gG1 antigen coated microparticles. The mixture is washed. Anti-human IgG acridinium-

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