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BAUSCH + LOMB

See better. Live better.

1400 North Goodman Street
Rochester, NY 14609
585.338.6000
www.bausch.com

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Multi-purpose solutions ReNu® with accessories, versions:

1. Multi-purpose solution ReNu® MultiPlus 60 ml, 120 ml, 240 ml, 360 ml.

2. Multi-purpose solution ReNu® MPS 120 ml, 240 ml, 360 ml.

Accessories: 1. Contact lens containers

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Legal Manufacturer: Bausch & Lomb Incorporated, USA

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
Official Regulatory Authorization
Bausch & Lomb Incorporated
1400 North Goodman Street
Rochester, NY 14609

State of New Jersey

County of Essex

Subscribed and sworn to (or affirmed) before me on this 11 day of Nov, 2024, by Marissa Danna, proved to me on the basis of satisfactory evidence to be the person(s) who appeared before me.
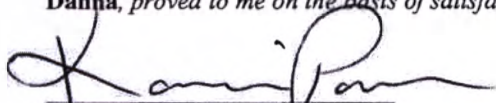
Signature of Notary Public

KAMAKSHI PAREKH
Commission # 50222559
Notary Public, State of New Jersey
My Commission Expires
June 03, 2029

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I hereby certify accuracy, correctness and reliability of this document translated into Russian

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RU_1628

BAUSCH+LOMB

OPERATIONAL DOCUMENTATION (instruction for use)

- I. Multi-purpose solutions ReNu® with accessories, versions:**
 - 1. Multi-purpose solution ReNu® MultiPlus 60 ml, 120 ml, 240 ml, 360 ml.**
 - 2. Multi-purpose solution ReNu® MPS 120 ml, 240 ml, 360 ml.**
- II. Accessories: 1. Contact lens containers.**

PLEASE READ CAREFULLY AND SAVE THIS INSERT FOR FUTURE REFERENCE.

Multi-purpose solution ReNu® MPS

Multi-Purpose sterile solution for the care of soft contact lenses ReNu® MPS with a container for storing contact lenses

Multi-purpose solutions are intended for: daily cleaning, rinsing, chemical (not heat) disinfection and storage of soft (hydrophilic) contact lenses including silicone hydrogels.

DIRECTIONS

For daily cleaning, disinfection follow these simple steps.

STEP 1: Place 3 drops of ReNu® MPS Multi-Purpose Solution on each side of the lens surface and gently rub for 20 seconds.

STEP 2: Thoroughly rinse each side of the lens with ReNu® MPS Multi-Purpose Solution.

STEP 3: Place cleaned contact lens in the lens case and fill with fresh ReNu® MPS Multi-Purpose Solution. Leave the lenses in the solution for at least 4 hours. Remember to always use fresh solution. Your lenses are now ready to wear. No saline rinse necessary. If any debris remains on contacts, rinse with ReNu® MPS Multi-Purpose Solution prior to insertion.

Always follow the instructions of the ophthalmologist. Based upon your individual tear chemistry and lens-wearing schedule, your ophthalmologist may recommend additional products or procedures.

INFORMATION ABOUT POTENTIAL CONSUMERS OF A MEDICAL DEVICE

Multi-Purpose Contact Lens Care Solutions are intended for anyone who needs to clean, disinfect, rinse and store of soft contact lenses, regardless of age, nationality or gender, as recommended by an ophthalmologist.

STORAGE

You may store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

CONTENTS

DYMED (polyaminopropyl biguanid), Boric Acid, Sodium Borate, Edetate Disodium, Sodium Chloride, Poloxamine, Hydrochloric Acid, Sodium Hydroxide, Purified Water.

ACTION

Cleans, loosens and removes accumulations of film, deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and rewets lenses before insertion.

INDICATIONS FOR USE

ReNu® MPS Multi-Purpose Solution is indicated for use in the daily cleaning, rinsing, and storage of day and extended wear contact lenses, and chemical (not heat) disinfection according to the recommended by an ophthalmologist.

CONTRAINDICATIONS

If you are allergic to any ingredient in this product, do not use.

WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN CORNEAL INFECTION AND/OR ULCER AND LEAD TO LOSS OF VISION (BLINDNESS).

It is essential that you follow your ophthalmologist's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision (blindness). Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your ophthalmologist. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, and / or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that contact lens wearers see their ophthalmologist twice each year, or if directed, more frequently.

SOAKING AND STORING YOUR LENSES

- You should fill your lens case with fresh solution every time you store your lenses, and never "top-off" or re-use solution. You should discard your solution immediately after your lenses have been removed from the lens case.
- You should not expose or store your lenses in or rinse your lens case with any water, such as tap, bottled or distilled, or with any non-sterile solution.

CARE OF THE LENSES CASE

- Before first use, thoroughly rinse the case and lids with a disinfectant solution for contact lenses.
- Clean and rinse the lens case with multipurpose solution each time you remove your lenses from the case, then air dry. The lens case can be turned upside down while air-drying to allow excess solution to run off.
- Change your lens case at least every three months.

WARNING:

- Failure to empty the solution from the lens case after each use, "refill" the solution, or use plain water to clean the lenses can lead to contamination, subsequent eye injury, and potential loss of vision.
- Follow the label's recommended times for wiping and rinsing your lenses exactly to properly disinfect your lenses and reduce the risk of lens contamination. If you reduce the time you wipe or rinse your lenses, there is a risk that residual dirt will remain on the lenses.

- Never use water, saline, or lubricating drops to disinfect lenses. These solutions do not provide adequate lens disinfection.

Failure to use the recommended disinfectant may result in serious infection, loss of vision, or blindness.

- Do not store lenses in water or any non-sterile solution, and do not wash the lens case with these products. Use only fresh universal solution to avoid contaminating your lenses or lens case. Use of a non-sterile solution may result in serious infection, loss of vision, or blindness.
- Using the solution after the expiration date may result in contamination of the solution and serious infection, loss of vision or blindness.

IMPORTANT SAFETY INFORMATION

- Always follow the instruction for use. Violation of the instructions may lead to blurring.
- Visit your eye care professional regularly.
- Always wash and dry your hands before handling lenses.
- Do not use tap water, bottled water, or saliva to clean the lens or lens container.
- Use only fresh solution to clean and disinfect contact lenses.
- After each disinfection cycle, always pour the remaining solution out of the container.
- Saline or moisturizing drops do not disinfect lenses.
- Always replace solutions and lenses according to the recommendations.
- To avoid contamination, do not touch the vial tip to any surface.
- Close the vial with a cap after use.
- Do not use any eye medications with this solution unless directed by a doctor.
- Do not use the solution together with heat (thermal) disinfection.

PRECAUTIONS

- When not in use, keep bottle tightly closed.
- Use the solution before the expiration date printed on the package and vial.
- Dispose of remaining solution within 90 days of opening the vial.
- Keep the solution out of the reach of children
- Store solution at room temperature

IMPORTANT INFORMATION

If irritation or excessive tearing occurs, persists or worsens, or if vision deteriorates, stop using the solution and contact your ophthalmologist immediately.

Undesirable reactions (problems and ways to solve them):

You should be aware that the following problems may occur:

- Tingling, burning, itching (irritation), or other pain in the eye
- Wearing the lens is less comfortable compared to the first time you put it on
- Abnormal sensation in the eye (eg, foreign body sensation, scratched area)
- Excessive watering (lacrimation) or unusual discharge from the eye
- Eye redness
- Reduced visual acuity (poor visual acuity)
- Blurred vision, iridescent circles or halos around objects

- Sensitivity to light (photophobia)
- Dry eyes

If you notice any of the above, remove your lenses immediately.

If the discomfort or problem disappears, carefully inspect the lens.

- If the lens is damaged in any way, do not wear it again. Place the lens in a storage container and see an ophthalmologist.
- If there is dirt, an eyelash, or other foreign body on the lens, or the problem disappears and the lens appears intact, thoroughly clean, rinse, and disinfect the lens, then put it back on.

If the problem persists, remove the lens immediately and consult an ophthalmologist.

The occurrence of any of the above problems may indicate the presence of a serious disease, such as infection, corneal ulcer, neovascularization or iritis.

Seek immediate medical attention to determine the problem and promptly begin treatment to avoid serious eye damage. When visiting an ophthalmologist to determine a potential eye infection, bring lenses, solutions, and a lens case with you to the appointment for a possible bacteriological examination in order to more accurately determine the type of all microorganisms.

Proper hygiene procedures for lens care:

- Always wash and rinse your hands before handling and putting on lenses.
- Clean, rinse, and disinfect your lenses each time you remove them.
- Always put the lens on and off the same eye first to avoid confusion.
- After each use, empty the lens case, rinse it with fresh multi-purpose solution and allow to air dry.

TRANSPORTATION CONDITIONS

At temperatures from 1°C to 40°C. Do not freeze

Modes of transport: Any type of land, air, river transport approved for the carriage of goods.

STORAGE AND USE CONDITIONS

Store and use the solution at a temperature not exceeding 25°C (at room temperature).

Do not freeze.

Use before the expiration date marked on the carton and bottle. Discard any remaining solution 90 days after opening.

Note: With daily use, this solution lasts for 90 days. Keep out of reach of children. Keep the bottle tightly closed when not in use.

EXPIRY DATE

2 years.

Do not use the solution after being opened for 90 days.

If you don't use contact lenses, you can store them in a closed container for up to 30 days.

DELOVERY SET

Multi-purpose solution is offered for sale in sterile plastic bottles of 120 ml, 240 ml and 360 ml.

Carton packaging with one bottle inside, a container for contact lenses and instructions for use.

THE PROCEDURE AND CONDITIONS FOR THE DISPOSAL OR DESTRUCTION OF A MEDICAL DEVICE

To prevent harm to the environment, it is necessary to comply with safety requirements, storage conditions, transportation and operation. The product must be disposed of by local public law organizations, in accordance with the requirements of local authorities and the recycling regulations in your area. For the Russian Federation, recycling is carried out in accordance with SanPiN 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of territories of urban and rural settlements, for water bodies, drinking water and drinking water supply, atmospheric air, soils, residential premises, operation of industrial, public premises, organization and carrying out sanitary and anti-epidemic (preventive) measures. The product is classified as class A medical waste (epidemiologically safe waste, similar in composition to solid household waste).


SYMBOLS

®/TM are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated or its affiliates

Attention! Refer to instructions for use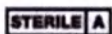

Sterile, manufactured under aseptic conditions

Name and address of manufacturer


Lot number


Best before: (YYYY MM)

Green Dot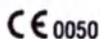

0050 confirms compliance with EU regulations

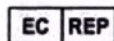authorized representative in the EU

The "CTP" may be marked to the packaging if necessary.

Manufacturing country.

Manufacturing site with address

Marketing authorization No. XXXXX/XXXXXX, date of issue.

Authorized representative of the manufacturer in the Russian Federation, address, telephone/fax.

www.bausch.ru www.ultralinzi.ru

Bausch & Lomb Incorporated, 1400 North Goodman Street, Rochester, NY 14609, USA /

Made in Italy

Manufacturer: Bausch & Lomb - Iom SpA, Via Pasubio 34, Macherio, Monza Brianza 20846, Italy

**AUTHORIZED PERSON OF THE MANUFACTURER IN THE TERRITORY OF THE
RUSSIAN FEDERATION**

Bausch Health LLC

Claims for quality in the Russian Federation should be sent to:

Russia, 115093, Moscow, ext. ter. Danilovsky Municipal District, 7 Pavlovskaya St., bldg. 1, premises
1N, tel/fax +7-495-510-28-79, Russia.Claims@bausch.com

Date of revision of the instruction: XXXX-XX

BAUSCH+LOMB

OPERATIONAL DOCUMENTATION (instruction for use)

- I. Multi-purpose solutions ReNu® with accessories, versions:**
 - 1. Multi-purpose solution ReNu® MultiPlus 60 ml, 120 ml, 240 ml, 360 ml.**
 - 2. Multi-purpose solution ReNu® MPS 120 ml, 240 ml, 360 ml.**
- II. Accessories: 1. Contact lens containers.**

PLEASE READ CAREFULLY AND SAVE THIS INSERT FOR FUTURE REFERENCE.

Multi-purpose solution ReNu® MultiPlus

Multi-Purpose sterile solution for the care of soft contact lenses ReNu® MultiPlus with a container for storing contact lenses

Multi-purpose solutions are intended for: daily cleaning, removal of protein deposits, rinsing, chemical (not heat) disinfection and storage of soft (hydrophilic) contact lenses including silicone hydrogels.

DIRECTIONS

For daily cleaning, disinfection and removal of protein deposits, follow these simple steps.

STEP 1: Place 3 drops of ReNu® MultiPlus Multi-Purpose Solution on each side of the lens surface and gently rub for 20 seconds.

STEP 2: Thoroughly rinse each side of the lens for 5 seconds with ReNu® MultiPlus Multi-Purpose Solution.

STEP 3: Place cleaned contact lens in the lens case and fill with fresh ReNu® MultiPlus Multi-Purpose Solution. Leave the lenses in the solution for at least 4 hours. Remember to always use fresh solution. Your lenses are now ready to wear. No saline rinse necessary. If any debris remains on contacts, rinse with ReNu® Multi-Purpose Solution prior to insertion.

Always follow the instructions of the ophthalmologist. Based upon your individual tear chemistry and lens-wearing schedule, your ophthalmologist may recommend additional products or procedures.

INFORMATION ABOUT POTENTIAL CONSUMERS OF A MEDICAL DEVICE

Multi-Purpose Contact Lens Care Solutions are intended for anyone who needs to clean, disinfect, remove of proteins deposits, rinse and store of soft contact lenses, regardless of age, nationality or gender, as recommended by an ophthalmologist.

STORAGE

You may store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

CONTENTS

DYMED (polyaminopropyl biguanid), Boric Acid, Sodium Borate, Edetate Disodium, Sodium Chloride, Poloxamine, HYDRANATE (hydroxyalkyl phosphonate), Hydrochloric Acid, Sodium Hydroxide, Purified Water.

ACTION

Cleans, loosens and removes accumulations of film, protein, other deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and wets lenses before insertion.

INDICATIONS FOR USE

ReNu® MultiPlus Multi-Purpose Solution is indicated for use in the daily cleaning, removing protein deposits, rinsing, chemical (not heat) disinfection and storage of soft (hydrophilic) contact lenses as recommended by an ophthalmologist.

CONTRAINDICATIONS

If you are allergic to any ingredient in this product, do not use.

WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN CORNEAL INFECTION AND/OR ULCER AND LEAD TO LOSS OF VISION (BLINDNESS).

It is essential that you follow your ophthalmologist's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision (blindness). Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your ophthalmologist. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, and / or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that contact lens wearers see their ophthalmologist twice each year, or if directed, more frequently.

SOAKING AND STORING YOUR LENSES

- You should fill your lens case with fresh solution every time you store your lenses, and never "top-off" or re-use solution. You should discard your solution immediately after your lenses have been removed from the lens case.
- You should not expose or store your lenses in or rinse your lens case with any water, such as tap, bottled or distilled, or with any non-sterile solution.

CARE OF THE LENSES CASE

- Before first use, thoroughly rinse the case and lids with a disinfectant solution for contact lenses.
- Clean and rinse the lens case with multipurpose solution each time you remove your lenses from the case, then air dry. The lens case can be turned upside down while air-drying to allow excess solution to run off.
- Change your lens case at least every three months.

WARNING:

- Failure to empty the solution from the lens case after each use, "refill" the solution, or use plain water to clean the lenses can lead to contamination, subsequent eye injury, and potential loss of vision.
- Follow the label's recommended times for wiping and rinsing your lenses exactly to properly disinfect your lenses and reduce the risk of lens contamination. If you reduce the time you wipe or rinse your lenses, there is a risk that residual dirt will remain on the lenses.
- Never use water, saline, or lubricating drops to disinfect lenses. These solutions do not provide adequate lens disinfection.

Failure to use the recommended disinfectant may result in serious infection, loss of vision, or blindness.

- Do not store lenses in water or any non-sterile solution, and do not wash the lens case with these products. Use only fresh universal solution to avoid contaminating your lenses or lens case. Use of a non-sterile solution may result in serious infection, loss of vision, or blindness.
- Using the solution after the expiration date may result in contamination of the solution and serious infection, loss of vision or blindness.

IMPORTANT SAFETY INFORMATION

- Always follow the instruction for use. Violation of the instructions may lead to blurring.
- Visit your eye care professional regularly.
- Always wash and dry your hands before handling lenses.
- Do not use tap water, bottled water, or saliva to clean the lens or lens container.
- Use only fresh solution to clean and disinfect contact lenses.
- After each disinfection cycle, always pour the remaining solution out of the container.
- Saline or moisturizing drops do not disinfect lenses.
- Always replace solutions and lenses according to the recommendations.
- To avoid contamination, do not touch the vial tip to any surface.
- Close the vial with a cap after use.
- Do not use any eye medications with this solution unless directed by a doctor.
- Do not use the solution together with heat (thermal) disinfection.

PRECAUTIONS

- When not in use, keep bottle tightly closed.
- Use the solution before the expiration date printed on the package and vial.
- Dispose of remaining solution within 90 days of opening the vial.
- Keep the solution out of the reach of children
- Store solution at room temperature

IMPORTANT INFORMATION

If irritation or excessive tearing occurs, persists or worsens, or if vision deteriorates, stop using the solution and contact your ophthalmologist immediately.

Undesirable reactions (problems and ways to solve them):

You should be aware that the following problems may occur:

- Tingling, burning, itching (irritation), or other pain in the eye

- Wearing the lens is less comfortable compared to the first time you put it on
- Abnormal sensation in the eye (eg, foreign body sensation, scratched area)
- Excessive watering (lacrimation) or unusual discharge from the eye
- Eye redness
- Reduced visual acuity (poor visual acuity)
- Blurred vision, iridescent circles or halos around objects
- Sensitivity to light (photophobia)
- Dry eyes

If you notice any of the above, remove your lenses immediately.

If the discomfort or problem disappears, carefully inspect the lens.

• If the lens is damaged in any way, do not wear it again. Place the lens in a storage container and see an ophthalmologist.

• If there is dirt, an eyelash, or other foreign body on the lens, or the problem disappears and the lens appears intact, thoroughly clean, rinse, and disinfect the lens, then put it back on.

If the problem persists, remove the lens immediately and consult an ophthalmologist.

The occurrence of any of the above problems may indicate the presence of a serious disease, such as infection, corneal ulcer, neovascularization or iritis.

Seek immediate medical attention to determine the problem and promptly begin treatment to avoid serious eye damage. When visiting an ophthalmologist to determine a potential eye infection, bring lenses, solutions, and a lens case with you to the appointment for a possible bacteriological examination in order to more accurately determine the type of all microorganisms.

Proper hygiene procedures for lens care:

- Always wash and rinse your hands before handling and putting on lenses.
- Clean, rinse, and disinfect your lenses each time you remove them.
- Always put the lens on and off the same eye first to avoid confusion.
- After each use, empty the lens case, rinse it with fresh multi-purpose solution and allow to air dry.

TRANSPORTATION CONDITIONS

At temperatures from 1°C to 40°C. Do not freeze

Modes of transport: Any type of land, air, river transport approved for the carriage of goods.

STORAGE AND USE CONDITIONS

Store and use the solution at a temperature not exceeding 25°C (at room temperature).

Do not freeze.

Use before the expiration date marked on the carton and bottle. Discard any remaining solution 90 days after opening.

Note: With daily use, this solution lasts for 90 days. Keep out of reach of children. Keep the bottle tightly closed when not in use.

EXPIRY DATE

2 years.

Do not use the solution after being opened for 90 days.

If you don't use contact lenses, you can store them in a closed container for up to 30 days.

DELOVERY SET

Multi-purpose solution is offered for sale in sterile plastic bottles of 60 ml, 120 ml, 240 ml and 360 ml.

Carton packaging with one bottle inside, a container for contact lenses and instructions for use.

THE PROCEDURE AND CONDITIONS FOR THE DISPOSAL OR DESTRUCTION OF A MEDICAL DEVICE

To prevent harm to the environment, it is necessary to comply with safety requirements, storage conditions, transportation and operation. The product must be disposed of by local public law organizations, in accordance with the requirements of local authorities and the recycling regulations in your area. For the Russian Federation, recycling is carried out in accordance with SanPiN 2.1.3684-21 "Sanitary and epidemiological requirements for the maintenance of territories of urban and rural settlements, for water bodies, drinking water and drinking water supply, atmospheric air, soils, residential premises, operation of industrial, public premises, organization and carrying out sanitary and anti-epidemic (preventive) measures. The product is classified as class A medical waste (epidemiologically safe waste, similar in composition to solid household waste).


SYMBOLS

®/TM are trademarks of Bausch & Lomb Incorporated or its affiliates

© Bausch & Lomb Incorporated or its affiliates

Attention! Refer to instructions for use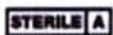

Sterile, manufactured under aseptic conditions

Name and address of manufacturer


Lot number


Best before: (YYYY MM)

Green Dot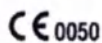

confirms compliance with EU regulations

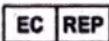authorized representative in the EU

The "CTP" may be marked to the packaging if necessary.

Manufacturing country.

Manufacturing site with address

Marketing authorization No. XXXXX/XXXXXX, date of issue.

Authorized representative of the manufacturer in the Russian Federation, address, telephone/fax.

www.bausch.ru www.ultralinz.ru

Bausch & Lomb Incorporated, 1400 North Goodman Street, Rochester, NY 14609, USA /
Made in Italy
Manufacturer: Bausch & Lomb - Iom SpA, Via Pasubio 34, Macherio, Monza Brianza 20846, Italy

**AUTHORIZED PERSON OF THE MANUFACTURER IN THE TERRITORY OF THE
RUSSIAN FEDERATION**

Bausch Health LLC

Claims for quality in the Russian Federation should be sent to:

Russia, 115093, Moscow, ext. ter. Danilovsky Municipal District, 7 Pavlovskaya St., bldg. 1, premises
1N, tel/fax +7-495-510-28-79, Russia.Claims@bausch.com

Date of revision of the instruction: XXXX-XX

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