«APPROVE»

CEO
EASTDENT CO.,LTD

Hyonsu Kim

«26» August 2020 r.

EASTDENT Co.,Ltd.

Hyon Su. Kim

H. S. Kim / President

**INSTRUCTION FOR USE
FOR A MEDICAL DEVICE
«Knitted gingival retraction cord Smartcord».**

1.MANUFACTURER

EASTDENT CO.,LTD, 601-1,Sogong B/D, 64, Gugal-ro, Giheung-gu, Yongin-si, Gyeonggi-do, Republic of Korea

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2. INFORMATION ABOUT THE REPRESENTATIVE IN RUSSIA

The Company "HIRATRADE" Ltd (In Russian ООО СП «ХИРАТРЕЙД», Gogolya, 20A, Vladimir, 600000 Russia

Tel.: +7 4922 42-05-32

E-mail: info@hiratrade.ru

3. PRODUCT NAME

Gingival retraction cord Smartcord.

4. OPTIONS

1. Dental thread for gingival retraction Smartcord 04;
2. Dental thread for gingival retraction Smartcord 03;
3. Dental thread for gum retraction Smartcord 02;
4. Dental thread for gum retraction Smartcord 0;
5. Dental thread for gum retraction Smartcord 1;
6. Dental thread for gum retraction Smartcord 2;
7. Dental thread for gingival retraction Smartcord X 04;
8. Dental thread for gum retraction Smartcord X 03;
9. Dental thread for gingival retraction Smartcord X 02;
10. Dental thread for gum retraction Smartcord X 0;
11. Dental thread for gum retraction Smartcord X 1;
12. Dental suture for gingival retraction Smartcord X 2.

5.CLASSIFICATION

Risk class: 2 a.

6. PURPOSE

For the temporary gingival retraction and hemostasis of the gingival margin.

7. INDICATION FOR USE

- bite defects of any severity;
- removal of destroyed fragments of the jaw row;
- gingival therapy;
- in the process of anesthesia to enhance the point effect of the drug;
- with professional whitening in a clinic;
- in the production of veneers and crowns;
- when removing stony deposits using ultrasonic influence;;
- as a manipulation that stops bleeding that occurs during therapy, surgical actions of a doctor.

8. CONTRAINDICATIONS

- Acute gingivitis or acute necrotizing ulceritic gingivitis.
- Periodontitis and periodontal disease.
- Patients with an allergy to Aluminum Chloride Hexahydrate

- Coagulation disorders.
- Arrhythmia and tachycardia.
- Viral forms of hepatitis.
- HIV infection, tuberculosis.
- Diabetes.
- Malignant tumor.

9. POSSIBLE SIDE EFFECTS

Nonexistent.

10. PRECAUTIONS

For professional use only

For single use only

After finishing the procedure, make sure to remove the cord from the sulcus

The package must be closed immediately after use.

11. POTENTIAL CONSUMER

Qualified medical personnel

12. APPLICATION

Dentistry.

13. A DESCRIPTION OF THE PRODUCTS, APPEARANCE

Product Name	Appearance	Description
Smartcord ⁰⁴		a tube with a lid, inside which is a spool of thread, knitted from 4 threads without impregnation (2 of each color). The cap of the lid is equipped with a knife for cutting the required length.

Smartcord 0³	 	
Smartcord 0²	 	
Smartcord 0	 	

Smartcord 1		
Smartcord 2		
Smartcord X 0 ⁴		a tube with a lid, inside which is a coil of thread (with impregnation), knitted from 4 threads without impregnation (2 of each color). The cap of the lid is equipped with a knife for cutting the required length.
Smartcord X 0 ³		

		
Smartcord X 0 ²	 	
Smartcord X 0	 	
Smartcord X 1	 	

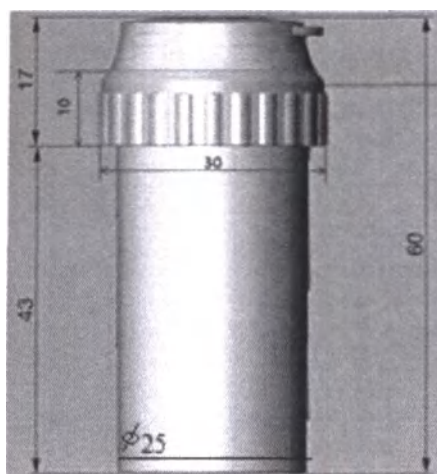
Smartcord X 2		
---------------	---	--

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2. REQUIREMENTS FOR THE CONSTRUCTION

- Products must be resistant to mechanical and climatic influences during operation, transportation and storage.
- Individual consumer packaging must be sealed.
- The cutter must be securely fastened.
- Metal parts must be made of corrosion-resistant materials.
- Materials whose safety has been confirmed in accordance with the established procedure must be used in the manufacture.
- Products must be resistant to the effects of secretions of body tissues that they come into contact with during operation.
- The shelf life must be at least 3 years.

15. RAW MATERIALS

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
Smartcord 0 ⁴	Black	Sunzol Black NWR liq.	99 % cotton+1% coloring	100% cotton (HJ Cotton 140)	Hanjung Union Corporation, Korea
	Blue	Mix: Sunzol Blue RS, Blue –light blue G133%			
Smartcord 0 ³	Black	Sunzol 4 Black NWR liq.	99 % cotton+1% coloring	100% cotton (HJ Cotton 100)	Hanjung Union Corporation, Korea

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
	Gray	Mix: Sunfix yellow S3R 150%, Sunfron blue SN-R, Sunfix red S3B 150%			
Smartcord 0 ²	Brown	Mix: Sunfix yellow S3R 150%, Sunfix Navy blue SBF Spec, Sunfix red S3B 150%	99 % cotton+1% coloring	100% cotton(HJ Cotton 80)	Hanjung Union Corporation, Korea
	Yellow	Mix: Sunfix yellow S3R 150%, Sunfix yellow S4GL, Sunfron blue SN-R.			
Smartcord 0	Purple	Mix: Sunzol Blue RS, Sunfix red S3B 150%.	99 % cotton+1% coloring	100% cotton (HJ Cotton 60)	Hanjung Union Corporation, Korea

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
	Light-purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.			
Smartcord 1	Blue	Mix: Sunzol blue RS, Sunzol light-blue G 133%.	99 % cotton+1% coloring	100% cotton (HJ Cotton 40)	Hanjung Union Corporation, Korea
	Light-blue	Mix: Sunfix yellow S3R 150%, Sunzol blue RS, Sunzol light-blue G 133%.			
Smartcord 2	Green	Mix: Sunfix yellow S4GL, Sunzol blue RS, Sunzol light-blue G 133%.	99 % cotton+1% coloring	100% cotton (HJ Cotton 30)	Hanjung Union Corporation, Korea
	Light-green	Mix: Sunfix yellow S3R 150%, Sunfix yellow S4GL, Sunzol light-blue G 133%.			

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
	Light-purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.			
Smartcord 1	Blue	Mix: Sunzol blue RS, Sunzol light-blue G 133%.	99 % cotton+1% coloring	100% cotton (HJ Cotton 40)	Hanjung Union Corporation, Korea
	Light-blue	Mix: Sunfix yellow S3R 150%, Sunzol blue RS, Sunzol light-blue G 133%.			
Smartcord 2	Green	Mix: Sunfix yellow S4GL, Sunzol blue RS, Sunzol light-blue G 133%.	99 % cotton+1% coloring	100% cotton (HJ Cotton 30)	Hanjung Union Corporation, Korea
	Light-green	Mix: Sunfix yellow S3R 150%, Sunfix yellow S4GL, Sunzol light-blue G 133%.			

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
Smartcord X 0 ⁴	Black	Sunzol Black NWR liq.	cotton 83,59%, coloring 0,84%, Impregnation: Alumnum chloride hexahydrate 15,57%.	100% cotton (HJ Cotton 140)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Blue	Mix: Sunzol blue RS, Sunzol, light-blue G 133%.	cotton 84,29%, coloring 0,85%, Impregnation: Alumnum chloride hexahydrate 14,86%.		Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
Smartcord X 0 ³	Black	Sunzol black NWR liq.	cotton 83,59%, coloring 0,84%, Impregnation: Alumnum chloride hexahydrate 15,57%.	100% cotton (HJ Cotton 100)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Gray	Mix: Sunfix yellow S3R 150%, Sunfron blue SN-R, Sunfix red S3B 150%			

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
Smartcord X 0 ²	Brown	Mix: Sunfix yellow S3R 150%, Sunfix Navy blue SBF Spec, Sunfix red S3B 150%	Cotton 83,4%, coloring 0,85%, Impregnation: Alumnum chloride hexahydrate 15,75%.	100% cotton (HJ Cotton 80)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Yellow	Mix: Sunfix yellow S3R 150%, Sunfix yellow S4GL, Sunfron blue SN-R.			
Smartcord X 0	Purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.	Cotton 83,4%, coloring 0,85%, Impregnation: Alumnum chloride hexahydrate 15,75%.	100% cotton (HJ Cotton 60)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Light-purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.			
Smartcord X 1	Blue	Mix: Sunzol blue RS,	cotton 84,29%, coloring	100% cotton (HJ	Hanjung Union Corporat

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
Smartcord X 0 ²	Brown	Mix: Sunfix yellow S3R 150%, Sunfix Navy blue SBF Spec, Sunfix red S3B 150%	Cotton 83,4%, coloring 0,85%, Impregnation: Alumnium chloride hexahydrate 15,75%.	100% cotton (HJ Cotton 80)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Yellow	Mix: Sunfix yellow S3R 150%, Sunfix yellow S4GL, Sunfron blue SN-R.			
Smartcord X 0	Purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.	Cotton 83,4%, coloring 0,85%, Impregnation: Alumnium chloride hexahydrate 15,75%.	100% cotton (HJ Cotton 60)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Light-purple	Mix: Sunzol blue RS, Sunfix red S3B 150%.			
Smartcord X 1	Blue	Mix: Sunzol blue RS,	cotton 84,29%, coloring	100% cotton (HJ	Hanjung Union Corporat

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
		Sunzol Turquoise blue G 133%.	0,85%, Impregnation: Alumnium chloride hexahydrate 14,86%.	Cotton 40)	ion, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Light-blue	Mix: Sunfix yellow S3R 150%, Sunzol blue RS, Sunzol Turquoise blueG 133%.			
Smartcord X 2	Green	Mix: Sunfix yellow S4GL, Sunzol blue RS, Sunzol Turquoise blue G 133%.	cotton 83,93%, coloring 0,85%, Impregnation: Alumnium chloride hexahydrate 15,22%.	100% cotton (HJ Cotton 30)	Hanjung Union Corporation, Korea Daejung Chemicals&Metals Co.ltd, Korea
	Light-green	Mix: Sunfix yellow S4GL, Sunzol blue RS, Sunzol Turquoise blue G			

Product name	Color combination	Coloring agent	percent ratio of the components	Yarn	Manufacturer
		133%.			
Aluminum chloride Hexahydrate concentration: 0.47±0.31 mg / cm (1.2±0.8 mg / inch)					

16. GURATION

Gingival retraction cord Smartcord of one option – 1 pc.

Instruction for use – 1 pc.

17.CONDITIONS OF USE

Use the product at a temperature of +10°From to +35°C and relative humidity of 75% at 15°C and 100% at 25°C.

18. STORAGE AND TRANSPORTATION CONDITIONS

Products are transported by all types of transport, in covered vehicles.

Storage conditions for products in the manufacturer's packaging: store in a dry, dark place at a temperature of +1 to +30 °C.

19. GUARANTEE Shelf life: 3 years. After opening the package, the thread can be used until the expiration date.

Shelf life: 3 years. After opening the package, the thread can be used until the expiration date.

20. REQUIREMENTS FOR ENVIRONMENTAL PROTECTION WHEN USING A MEDICAL DEVICE

Products at room temperature do not emit toxic substances into the environment and do not have a harmful effect on the human body in direct contact.

21. PACKAGING

Primary packaging-tube:

1) container: butadiene and styrene copolymer, grade K, manufactured by Chevron Phillips Chemical Company LLC, USA

2) cover: polyethylene, SÊETEC brand manufactured by LG Chem Ltd., Korea

3) bobbin: low-density polyethylene, Pipe brand manufactured by LG Chem Ltd., Korea

4) Bobbin limiter: polycarbonate resin, LUPOY brand manufactured by LG Chem Ltd., Korea

5) Knife-stainless steel, grade 304, manufactured by Posco, Korea for versions: dental cord for gingival retraction Smartcord 04; dental cord for gingival retraction Smartcord 03; dental cord for gingival retraction Smartcord 02;

Dental cord for gingival retraction Smartcord 0; dental cord for gingival retraction Smartcord 1; dental cord for gingival retraction Smartcord 2,
 - zirconium, manufactured by DAEMYUNG CHEMICAL, Korea for versions: dental cord for gingival retraction Smartcord X 04; dental Thread for gum retraction Smartcord X 03;
 Dental cord for gingival retraction Smartcord X 02; dental cord for gingival retraction Smartcord X 0; dental cord for gingival retraction Smartcord X 1; dental cord for gingival retraction Smartcord X 2).

Group

Box made of corrugated cardboard (composition: cellulose 92%, water 8%), produced by Boxro, Korea.

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Box made of corrugated cardboard (composition: cellulose 92%, water 8%), produced by Boxro, Korea.

Capacity: 600 PCs. (6 group packages of 100 pieces).

Overall size, LxWxH, cm: 35x35x45 (10% tolerance).

22.LABELS

Manufacturer's consumer label

Smartcord 0⁴

Smartcord 0⁴

Knitted Retraction Cord

The finest 120inch/305cm

Non impregnated

EASTDENT

cotton 100%

Bobbin inside

Easy cut

CE

CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional.

EASTDENT CO., LTD
 64 Gugaero, Giheunggu, Yongin-si, Korea
 Am Pflaushof 12, 22393, Hamburg, Germany
 EDSLE.Rev2, 07/2019

www.east.dent.com

Made in Korea

make it SMART

8 809370 410363

Smartcord 0³

Smartcord 0³

Knitted Retraction Cord

The finest 120inch/305cm

Non impregnated

EASTDENT

cotton 100%

Bobbin inside

Easy cut

CE

CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional.

EASTDENT CO., LTD
 64 Gugaero, Giheunggu, Yongin-si, Korea
 Am Pflaushof 12, 22393, Hamburg, Germany
 EDSLE.Rev2, 07/2019

www.east.dent.com

Made in Korea

make it SMART

8 809370 410059

Smartcord 0²

Knitted retraction cord from 100% cotton, non-impregnated
Indicated for gingival retraction
Before taking an impression and placing resin restorations near the marginal gingiva
Close the cap after use and store in a dry place and away from direct light
Patent No. 1197687
120inch(305cm)

CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional

EASTDENT CO., LTD
64 Gugaj-ro, Giheung-gu, Yongin-si, Korea
BIOGERI
Am Pfladshof 12, 22393 Hamburg, Germany
EDSLE,Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea
make it SMART

8 809370 410066

Smartcord 0

Knitted retraction cord from 100% cotton, non-impregnated
Indicated for gingival retraction
Before taking an impression and placing resin restorations near the marginal gingiva
Close the cap after use and store in a dry place and away from direct light
Patent No. 1197687
120inch(305cm)


CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional

EASTDENT CO., LTD
64 Gugaj-ro, Giheung-gu, Yongin-si, Korea
BIOGERI
Am Pfladshof 12, 22393 Hamburg, Germany
EDSLE,Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea
make it SMART

8 809370 410073

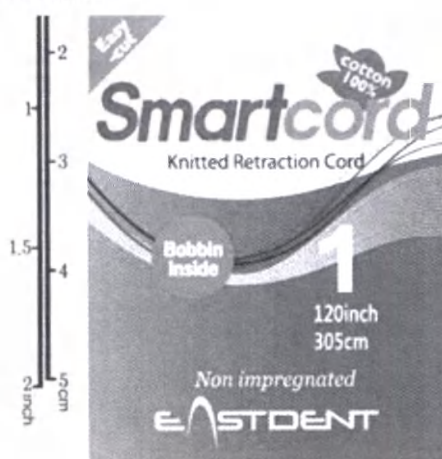Smartcord 1

Knitted retraction cord from 100% cotton, non-impregnated
Indicated for gingival retraction
Before taking an impression and placing resin restorations near the marginal gingiva
Close the cap after use and store in a dry place and away from direct light
Patent No. 1197687
120inch(305cm)

CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional

EASTDENT CO., LTD
64 Gugaj-ro, Giheung-gu, Yongin-si, Korea
BIOGERI
Am Pfladshof 12, 22393 Hamburg, Germany
EDSLE,Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea
make it SMART

8 809370 410080

Smartcord 2

Knitted retraction cord from 100% cotton, non-impregnated

Indicated for gingival retraction. Before taking an impression and placing resin restorations near the marginal gingiva.

Close the cap after use and store in a dry place and away from direct light. Patent No. 1197667

120inch(305cm)

CAUTION: U.S. Federal Law restricts this device to the sale by or on the order of a dental professional.

EASTDENT CO., LTD
64 Gugak-ro, Giheung-gu, Yongin-si, Korea

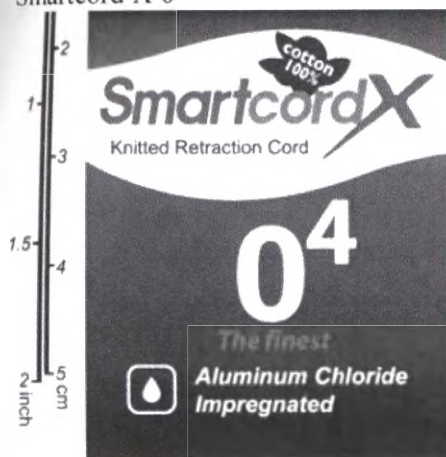
BIOGERI
Am Pfleishof 12, 22393 Hamburg, Germany
EDSLE_Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea

8 809370 410097

Smartcord X 0⁴

Smartcord X is made of 100% cotton yarn impregnated with Aluminum Chloride Hexahydrate 1.2 ± 0.8 mg/inch

100 inch / 254cm

EASTDENT CO., LTD
64 Gugak-ro, Giheung-gu, Yongin-si, Korea

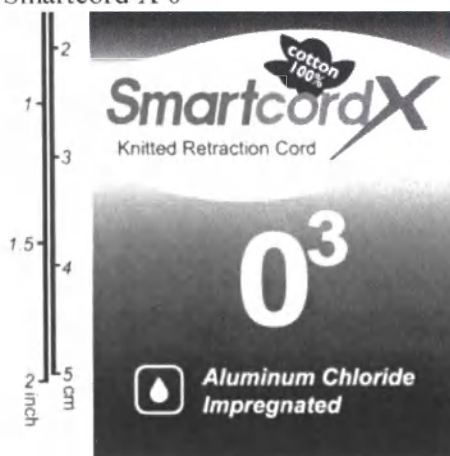
BIOGERI
Am Pfleishof 12, 22393 Hamburg, Germany
EDXLE_Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea

8 809370 410332

Smartcord X 0³

Smartcord X is made of 100% cotton yarn impregnated with Aluminum Chloride Hexahydrate 1.2 ± 0.8 mg/inch

100inch / 254cm

EASTDENT CO., LTD
64 Gugak-ro, Giheung-gu, Yongin-si, Korea

BIOGERI
Am Pfleishof 12, 22393 Hamburg, Germany
EDXLE_Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea

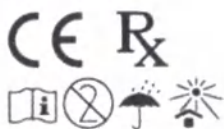
8 809370 410219

Smartcord X 0²

Smartcord X is made of
100% cotton yarn
impregnated with
Aluminum Chloride Hexahydrate
1.2 ± 0.8 mg/inch
100inch / 254cm

EASTDENT
www.east-dent.com
Made in Korea

Manufacturer
EASTDENT CO., LTD
64 Gugal-ro, Giheung-gu,
Yongin-si, Korea

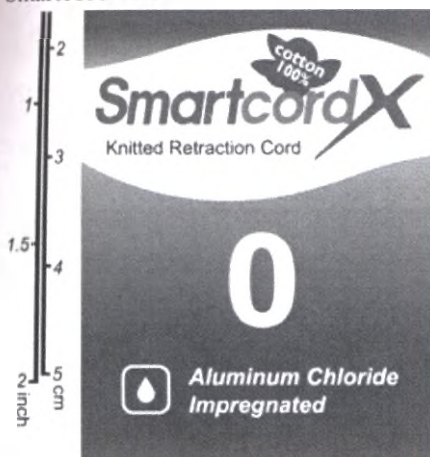
BIOGERI
Am Pfeilshof 12, 22393
Hamburg, Germany
EDXLE.Rev2, 07/2019

LOT


8 809370 410226

Smartcord X 0

Smartcord X is made of
100% cotton yarn
impregnated with
Aluminum Chloride Hexahydrate
1.2 ± 0.8 mg/inch
100inch / 254cm

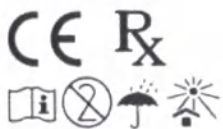EASTDENT
www.east-dent.com
Made in Korea

Manufacturer
EASTDENT CO., LTD
64 Gugal-ro, Giheung-gu,
Yongin-si, Korea

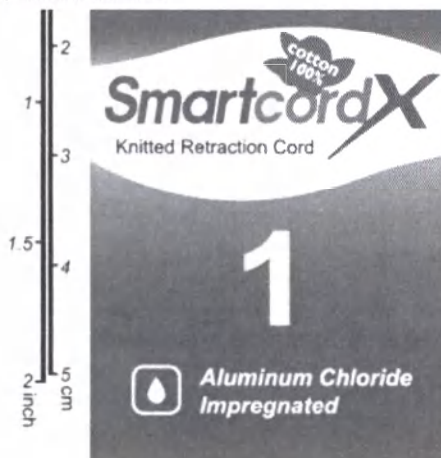
BIOGERI
Am Pfeilshof 12, 22393
Hamburg, Germany
EDXLE.Rev2, 07/2019

LOT

8 809370 410233

Smartcord X 1

Smartcord X is made of
100% cotton yarn
impregnated with
Aluminum Chloride Hexahydrate
1.2 ± 0.8 mg/inch
100inch / 254cm

EASTDENT
www.east-dent.com
Made in Korea

Manufacturer
EASTDENT CO., LTD
64 Gugal-ro, Giheung-gu,
Yongin-si, Korea

BIOGERI
Am Pfeilshof 12, 22393
Hamburg, Germany
EDXLE.Rev2, 07/2019

LOT

8 809370 410240

Smartcord X 2

Smartcord X is made of 100% cotton yarn impregnated with Aluminum Chloride Hexahydrate 1.2 ± 0.8 mg/inch
100 inch / 254cm

SmartcordX
Knitted Retraction Cord

2

Aluminum Chloride Impregnated

CE Rx

Manufactured by
EASTDENT CO., LTD
64 Guguk-ro, Giheung-gu,
Yongin-si, Korea

Am Pfeilshof 12, 22393
Hamburg, Germany

EDXLE.Rev2, 07/2019

EASTDENT
www.east-dent.com
Made in Korea

LOT

8 809370 410257

Group -top view

--side view

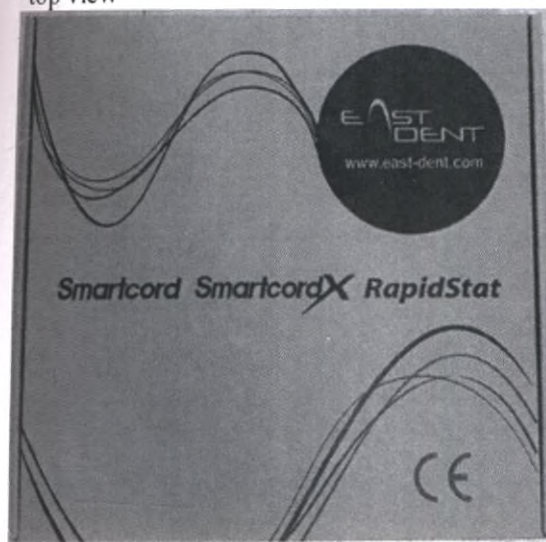
-side view

Smartcord X 2

Group -top view

--side view

-side view

Transport
-side view

-side view

Tables in Russian for Russian Federation

Stickers

1. consumer

- e.g. Smartcord 0⁴

Smartcord 0⁴

Manufacturer: Eastdent Co., Ltd. Republic Of Korea
EASTDENT CO., LTD. 601-1, Sogong B/D. 64, Gugal-ro, Giheung-gu
Yongin-si, Gyeonggi-do, Republic of Korea

Complete set:

dental cord for gingival retraction Smartcord 0-1 PC.; instructions – 1 PC.

RU: №

See the colors on the packaging. The expiration date is indicated on the package.

Dispose of in accordance with the current legislation of the Russian Federation.

Authorized organization for filing claims on the territory of the Russian Federation: JV

HIRATRADE LLC, tel.: +7(4922) 42-05-32

Made in the Republic of Korea

Length
305 cm

- e.g. Smartcord X 0⁴

Smartcord X 0⁴

Manufacturer: Eastdent Co., Ltd. Republic Of Korea
EASTDENT CO., LTD. 601-1, Sogong B/D. 64, Gugal-ro, Giheung-gu
Yongin-si, Gyeonggi-do, Republic of Korea

Complete set: dental cord for gingival retraction Smartcord X 0⁴ – 1 PC.; instructions – 1 PC.

RU: №

Concentration of aluminum chloride Hexahydrate: 0.47±0.31 mg / cm

for Shades, see the package. The expiration date is indicated on the package.

Dispose of in accordance with the current legislation of the Russian Federation.

Authorized organization for filing claims on the territory of the Russian Federation: JV

HIRATRADE LLC, tel.: +7(4922) 42-05-32

Made in the Republic of Korea

Length
305 cm

Group

-e.g. Smartcord 0⁴

Smartcord 0⁴

Manufacturer: Eastdent Co., Ltd, Republic Of Korea
 EASTDENT CO., LTD, 601-1, Sogong B/D, 64, Gugal-ro, Giheung-gu
 Yongin-si, Gyeonggi-do, Republic of Korea
 Complete set: dental cord for gingival retraction Smartcord 0-1 PC.; instructions – 1 PC.
 RU: № _____
 See the colors on the packaging. The expiration date is indicated on the package.
 Dispose of in accordance with the current legislation of the Russian Federation.
 Authorized organization for filing claims on the territory of the Russian Federation: JV
 HIRATRADE LLC, tel.: +7(4922) 42-05-32
 Made in the Republic of Korea
 Quantity: 100 pc.

Length
 305 cm

Non -toxic

Smartcord X 0⁴

Smartcord X 0⁴

Manufacturer: Eastdent Co., Ltd, Republic Of Korea
 EASTDENT CO., LTD, 601-1, Sogong B/D, 64, Gugal-ro, Giheung-gu
 Yongin-si, Gyeonggi-do, Republic of Korea
 Complete set: dental cord for gingival retraction Smartcord X 0⁴ – 1 PC.; instructions – 1
 PC.
 RU: № _____
 See the colors on the packaging. The expiration date is indicated on the package.
 Dispose of in accordance with the current legislation of the Russian Federation.
 Authorized organization for filing claims on the territory of the Russian Federation: JV
 HIRATRADE LLC, tel.: +7(4922) 42-05-32
 Made in the Republic of Korea
 Quantity: 100 pc.

Length
 305 cm

Non -toxic

Transport

-e.g. Smartcord 0⁴

Smartcord 0⁴

Manufacturer: Eastdent Co., Ltd, Republic Of Korea
EASTDENT CO., LTD, 601-1, Sogong B/D, 64, Gugal-ro, Giheung-gu
Yongin-si, Gyeonggi-do, Republic of Korea
RU: № _____

Dispose of in accordance with the current legislation of the Russian Federation.

Authorized organization for filing claims on the territory of the Russian Federation: JV

HIRATRADE LLC, tel.: +7(4922) 42-05-32

Made in the Republic of Korea

Quantity: 600 pc.

Non-toxic

-e.g. Smartcord X 0⁴

Smartcord X 0⁴

Manufacturer: Eastdent Co., Ltd, Republic Of Korea
EASTDENT CO., LTD, 601-1, Sogong B/D, 64, Gugal-ro, Giheung-gu
Yongin-si, Gyeonggi-do, Republic of Korea
RU: № _____

Dispose of in accordance with the current legislation of the Russian Federation.

Authorized organization for filing claims on the territory of the Russian Federation: JV

HIRATRADE LLC, tel.: +7(4922) 42-05-32

Made in the Republic of Korea

Quantity: 600 pc.


Non-toxic

Symbols and their meanings:

	Authorized representative in the European Community		Manufacturer
	Date of issue		Do not reuse
	Batch code		Non sterile
	Keep away from sunlight		Keep dry
	Temperature limit		Consult instructions for use

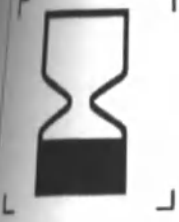	Use-by date		CE Mark
	Caution: Federal law restricts this device to sale by or on the order of a dental professional.		Barcode
	The presence of impregnation: Aluminum chloride hexahydrate		Do not use if the package is damaged
	Top		Non sterile
	Handle with care		

23. MAINTENANCE AND REPAIR

Not maintainable.

24. PROCEDURE

1. Check the conditions of gingival tissue (e.g. depth or thickness)
2. Select a cord with an appropriate diameter,
3. Cut the cord to the required length with EasyCut and SmartRuler
4. Insert the cord into the sulcus by means of a retraction cord packer
5. Leave the cord about 5 minutes in the sulcus
6. Remove the cord just before taking the impression. Slightly moisten the cord to make its removal easier and reduce gingival hemorrhage.

25. INFORMATION ABOUT STERILITY

Non sterile

26. UTILIZATION

Disposed of in accordance with medical practice and regional or state regulations.

27. REFERENCES

- 93/42/EEC amended by Directive 2007/47/EC Concerning medical devices
EN ISO 15223-1[2016] Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements
EN 1041[2008] Information supplied by the manufacturer of medical devices
EN ISO 14971[2012] Medical devices – Application of risk management to medical devices
EN ISO 10993-1[2009] Biological evaluation of medical devices – Part 1: Evaluation and Testing
EN ISO 10993-5[2009] Biological evaluation of medical devices – Part 5: Tests for in vitro cytotoxicity
EN ISO 10993-10[2013] Biological evaluation of medical devices – Part 10: Tests for irritation and delayed-type hypersensitivity
EN ISO 13485[2016] Medical devices - Quality management systems –Requirements for regulatory purposes

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Easy cut

cotton 100%

Smartcord
Knitted Retraction Cord

Bobbin Inside

2
120inch
305cm

Non impregnated

EASTDENT

Knitted retraction cord from 100% cotton, non-impregnated
Indicated for gingival retraction
Before taking an impression and
placing resin restorations near the
marginal gingiva
Close the cap after use and store in
a dry place and away from direct light
Patent No. 1197667
120inch(305cm)

CE

CAUTION: U.S. Federal Law restricts
this device to the sale by or on the
order of a dental professional

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cotton 100%

Smartcord X
Knitted Retraction Cord

The finest

04
100 inch / 254cm

Aluminum Chloride
Impregnated

Smartcord X is made of
100% cotton yarn
impregnated with
Aluminum Chloride Hexahydrate
1.2 ± 0.8 mg/inch

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INSTRUCTION FOR USE
(brief)

FOR A MEDICAL DEVICE
«Knitted retraction cord Smartcord».

Instruction for use (brief)
Knitted retraction cord Smartcord

MANUFACTURER

EASTDENT CO., LTD , 601-1, Sogong B/D, 64, Gugal-ro, Giheung-gu, Yongin-si, Gyeonggi-do, Republic of Korea
Tel.: (82)31-693-5767
Fax: (82) 31-693-5768

INFORMATION ABOUT THE REPRESENTATIVE IN RUSSIA

The Company "HIRATRADE" Ltd (In Russian ООО СП «ХИПАТРЕЙД»), Gogolya, 20A, Vladimir, 600000 Russia
Tel.: +7 4922 42-05-32
E-mail: info@hiratrade.ru

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CONTRAINDICATIONS

- Acute gingivitis or acute necrotizing ulceritic gingivitis.
- Periodontitis and periodontal disease.
- Patients with an allergy to Aluminum Chloride Hexahydrate
- Coagulation disorders.
- Arrhythmia and tachycardia.
- Viral forms of hepatitis.
- HIV infection, tuberculosis.
- Diabetes.
- Malignant tumor.

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CONDITIONS OF USE

Use the product at a temperature of +10°C from to +35°C and relative humidity of 75% at 15°C and 100% at 25°C.

STORAGE AND TRANSPORTATION CONDITIONS

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Pictograms

	Authorized representative in the European Community		Manufacturer		Use-by date		CE Mark
	Date of issue		Do not reuse		Caution: Federal law restricts this device to sale by or on the order of a dental professional.		Barcode
	Batch code		Non sterile		The presence of impregnation: Aluminum chloride hexahydrate		Do not use if the package is damaged
	Keep away from sunlight		Keep dry		Temperature limit		Consult instructions for use
	Top		Fragile. Handle with care		Handle with care		

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