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UP007, UP008, UP009, UP010

2 Scope of supply

UP007/UP008: 2x pump tubes in UP013
4x membranes in UP014

UP009/UP010: 2x pump tubes in UP013

3 Intended use

Irrigation – the tubing set serves to transport and, where applicable, pump irrigation fluid to the irrigation instrument for endoscopic/hysteroscopic/laparoscopic/arthroscopic/urological purposes.

Irrigation and suction – the tubing set serves to transport and, where applicable, pump irrigation fluid to the irrigation instrument or the suctioned off fluid from the instrument to a suction container for laparoscopic/hysteroscopic purposes.

Suction – the tubing set serves to transport and, where applicable, pump suctioned off fluid from the suction instrument to the suction container (for lithotripsy/hysteroscopic and urological purposes).

3.1 Indications

The use of these tubing sets does not depend on particular indications. They serve as accessories and are used whenever fluids need to be transported, or whenever the associated KARL STORZ medical device is used.

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UP010

1 Article numbers
UP007, UP008, UP009, UP010

2 Scope of supply
UP007/UP008: 2x pump tubes in UP013,
4x membranes in UP014
UP009/UP010: 2x pump tubes in UP013

3 Intended use
Irrigation - the tubing set serves to transport and, where applicable, pump irrigation fluid to the irrigation instrument for endoscopic/hysteroscopic/laparoscopic/arthroscopic/urological purposes.
Irrigation and suction - the tubing set serves to transport and, where applicable, pump irrigation fluid to the irrigation instrument or the suctioned off fluid from the suction instrument to a suction container for laparoscopic/hysteroscopic purposes.
Suction - the tubing set serves to transport and, where applicable, pump suctioned off fluid from the suction instrument to the suction container (for lithotripsy/hysteroscopic and urological purposes).

3.1 Indications

The use of these tubing sets does not depend on particular indications. They serve as accessories and are used whenever fluids need to be transported or whenever the associated KARL STORZ medical device is used.

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