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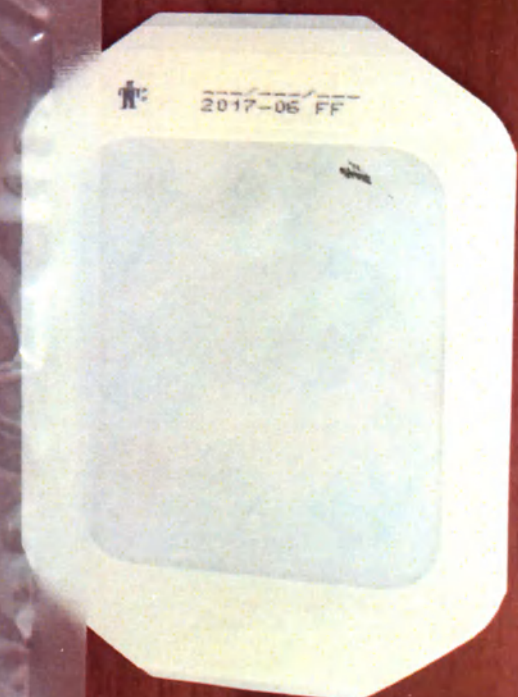
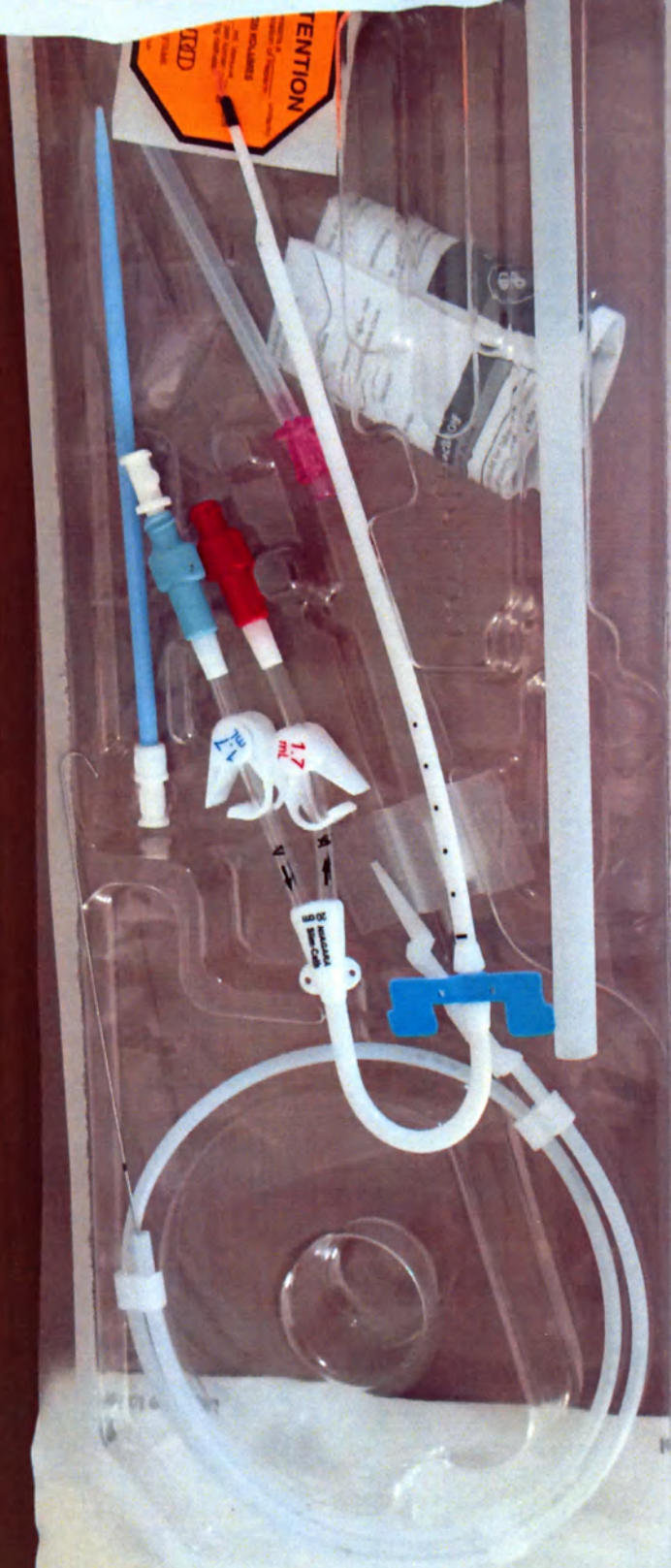
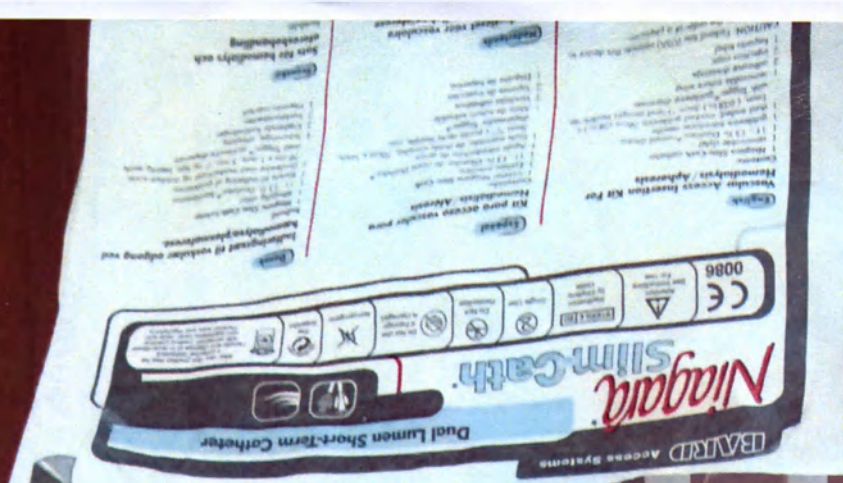
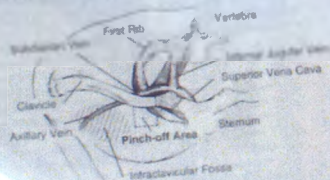
CONTRAINDICATIONS

- The catheter is intended for short-term vascular access only and is not to be used for any purpose other than indicated in these instructions. The catheter must not be left in the femoral vein longer than three days. To maintain peak performance, it is recommended that subclavian and jugular catheters be replaced after four weeks.

WARNINGS

- SUBCLAVIAN ONLY: Pinch-off Prevention:** Percutaneous insertion of the catheter must be made into the axillary-subclavian vein at the junction of the outer and middle third of the clavicle lateral to the thoracic outlet. The catheter must be inserted into the subclavian vein medially, because the catheter can lead to the subclavian vein medially, because the catheter can lead to compression of the catheter between the first rib and clavicle and can lead to damage or fracture and embolism of the catheter. Fluoroscopic or radiographic confirmation of catheter placement can be helpful in demonstrating that the catheter is not being pinched by the first rib and clavicle.

Signs of Pinch-off

- Clinical:**
- Difficulty with blood withdrawal
 - Resistance to infusion of fluids
 - Fluoroscopic position changes required for infusion of fluids or blood withdrawal

Radiologic Signs of Pinch-Off		
Grade	Severity	Recommended Action
Grade 0	No Distortion	No action
Grade 1	Distortion present without luminal compromise	Fluoroscopic x-ray should be taken to monitor progression of pinch-off to grade 2
Grade 2	Distortion present with luminal compromise	Replace catheter

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