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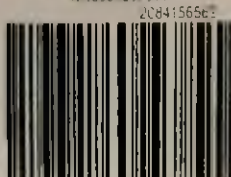
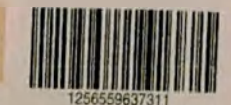
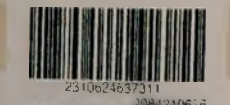
Precautions and warnings

For in vitro diagnostic use

Exercise the normal precautions required for handling all laboratory reagents

Disposal of all waste material should be in accordance with local guidelines

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Prevention:

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

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The calibrators have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg (HBsAg II Cal1) and antibodies to HCV and HIV.

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The serum containing HBsAg (HBsAg II Cal2) was inactivated using β -propiolactone and UV-radiation

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Method Sheet download

The box at the top of this document shows the product code together with the respective document version

Please go to dialog.roche.com to download this document

The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge

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English

Precautions and warnings

For in vitro diagnostic use.
Exercise the normal precautions required for handling all laboratory reagents.
Disposal of all waste material should be in accordance with local guidelines.
Safety data sheet available for professional user on request.
This lot contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

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Prevention:

P261 Avoid breathing dust/fume/gas/mist/vapours/spray

P272 Contaminated work clothing should not be allowed out of the workplace

P280 Wear protective gloves

Response:

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention

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Disposal:

P501 Dispose of contents/container to an approved waste disposal plant.

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The calibrators have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg (HBsAgII2 Cal1) only and antibodies to HCV and HIV

The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

The serum containing HBsAg (HBsAgII2 Cal2) was inactivated using β -propiolactone and UV-radiation

However, as no inactivation or testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls)

Method Sheet download

The box at the top of this document shows the product code together with the respective document version.

Please go to dialog.roche.com to download this document.

The website displays either the version listed here or, if available, any later version that is also valid for this product code. If you have no internet access please contact your local Roche affiliate to obtain the document free of charge.

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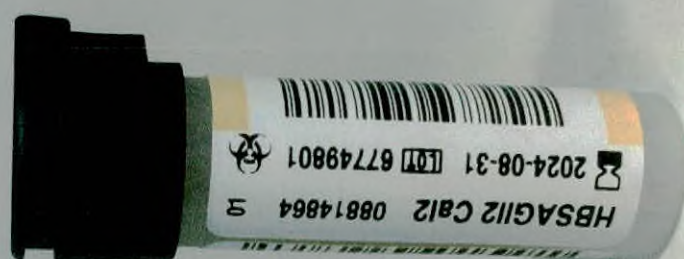
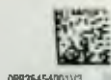
Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents. Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

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P362 + P364 Take off contaminated clothing and wash it before reuse

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Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

All human material should be considered potentially infectious

The calibrators have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg (HBsAgII2 Cal1) only) and antibodies to HCV and HIV.

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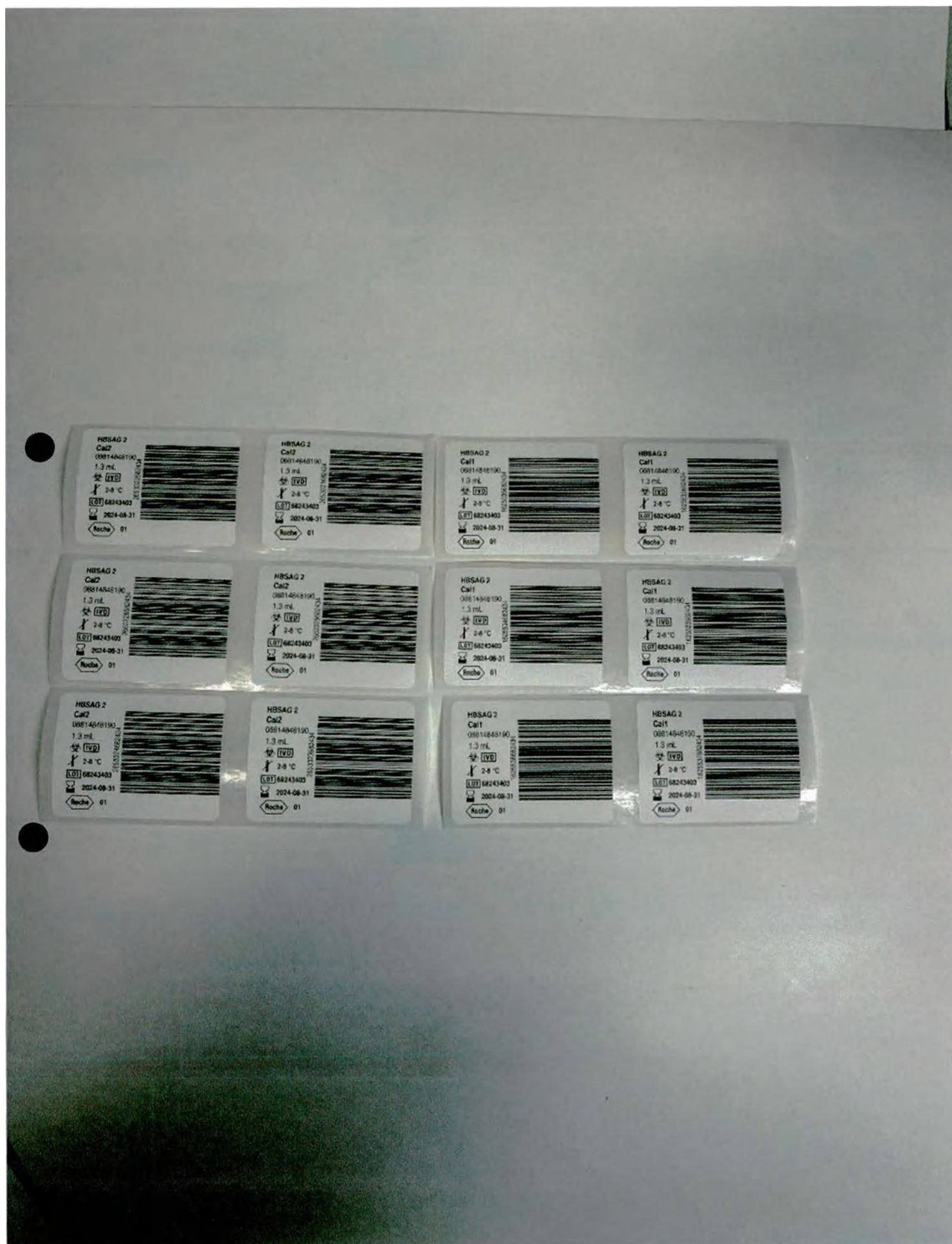
English

Precautions and warnings

For in vitro diagnostic use.
Exercise the normal precautions required for handling all laboratory reagents.
Disposal of all waste material should be in accordance with local guidelines.
Safety data sheet available for professional user on request.
For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008

Warning

H317 May cause an allergic skin reaction

Prevention:

P261 Avoid breathing dust/fume/gas/mist/vapour/spray

P272 Contaminated work clothing should not be allowed out of the workplace.

P280 Wear protective gloves

Response:

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

P362 + P364 Take off contaminated clothing and wash it before reuse.

Disposal:

P501 Dispose of contents/container to an approved waste disposal plant

Product safety labeling follows EU GHS guidance

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

All human material should be considered potentially infectious.

The calibrators have been prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg (HBsAg 2 Cal1 only) and antibodies to HCV and HIV.

The testing methods used have been approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

The serum containing HBsAg (HBsAg 2 Cal2) was inactivated using β -propiolactone and UV-radiation.

However, as no inactivation or testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Method Sheet download

The product code together with the respective document version (if applicable) is listed in the download information boxes listed above. Please go to the website listed in the relevant box to download this document. The website displays either the version listed here or, if available, any later version.

2022-02, V 4

HBsAg II cobas®

This is also valid for this product code. If you have no internet access, please contact your local Roche affiliate to obtain the document free of charge.

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