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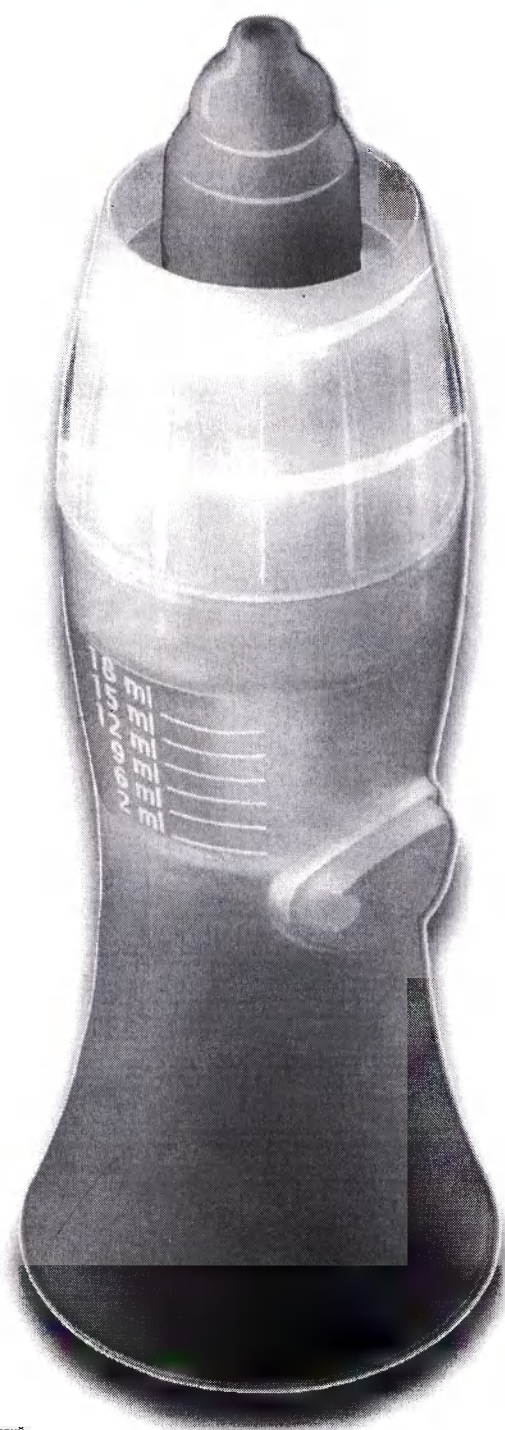
We are pleased you have chosen one of our products and thank you for placing your trust in us. Our aim is to completely satisfy our consumers by offering them innovative products for the treatment of respiratory diseases.

Please note that the complete range of MED2000 products is contained in the website www.med2000.it.

The R1-model Rinomed nasal irrigator is the fastest and most effective way to wash and treat with medication the nasal cavities and nasopharynx. It can be used with saline, isotonic, hypertonic solutions, etc., thermal water or medicines.

Application: otolaryngology. Intended use: prevention and treatment of ENT diseases through irrigation and the washing of the nasal cavity and nasopharynx.

The R1-model Rinomed unit sprays around 10 ml of solution in about 2 minutes. Read these instructions carefully and keep them for future reference.

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PLEASE READ THESE GUIDELINES BEFORE USE

ATTENTION!

- Carefully read the instructions contained in this booklet and keep in a safe place for future reference.
- Keep the nebulizer chamber out of reach of children or unqualified people without supervision and take special care during use.
- Keep packaging components (cardboard box, plastic bags, etc.) out of reach of children.
- Do not use essential oils. Oily medications and substances can damage the plastic components.
- Only use the nebulizer chamber for aerosol inhalation with medications prescribed by a doctor.
- Do not use fittings and accessories not recommended by the manufacturer.
- Do not use the nebulizer chamber, if after unpacking, there are any signs of damage that may have been caused during transportation.
- For assistance, please contact Med 2000 Srl or your nearest authorised dealer.
- The accessories are only for personal use.
- After treatment, do not leave any residues of medication in the nebulizer chamber.
- Treatment should always be prescribed by a doctor who will give indications on the type of medication to use, the doses and the best administration methods for the patient.
- The manufacturer, assembler or distributor shall be held responsible for the safety of the product if, and only if:
 - The assembly technique, inspection and repairs were performed by authorised personnel.
 - The user and maintenance instructions have been followed.
 - The manufacturer is not responsible for damages caused by improper, incorrect or unreasonable use.

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INSTRUCTIONS FOR USE

- Choose the nasal adaptor **1** according to the age of the patient.
- Pour the solution in the lower part of the nebulizer chamber **5**.
- Insert the pisper **4** in the lower part **7**.
- Close the nebulizer chamber by screwing the top part **2**.
- Connect the extension tube **8** to the lower part of the nebulizer chamber and attach the other end of the tube to the connector of the aerosol.
- Place the tube in the tube holder **9** so that it does not tip over.
- At this point, turn the unit on and press the dispensing button **6** to activate the nebulizer chamber.
- The therapy must be performed with the patient sitting comfortably in an upright position.
- Insert the R1-model Rinomed nasal irrigator into a nostril and proceed with the treatment, alternating the nostrils every 15/20 seconds for better efficacy. Before treatment with medication, treat with physiological water.
- When the mist starts disappearing from the nebulizer chamber the treatment is done.
- At this point, remove the upper part of the nebulizer chamber **2**, which is held in place by the O-ring **3**, with the pisper nozzle **4** and shake any excess medicine collected inside.
- After the treatment, rinse thoroughly and sterilise the accessories as described below.

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INSTRUCTIONS FOR USE

CLEANING

After use, disassemble the parts of the nasal irrigator and wash them in warm water and mild detergent.

Rinse carefully. Remove any excess water with a cloth and leave to dry in a clean place.

Choose one of the following sterilisation methods:

- 1) Immerse in a solution of water and vinegar (three parts water and one part vinegar). Rinse thoroughly with clean water.*
- 2) Immerse in a special sterilising solution, following the manufacturer's instructions.*

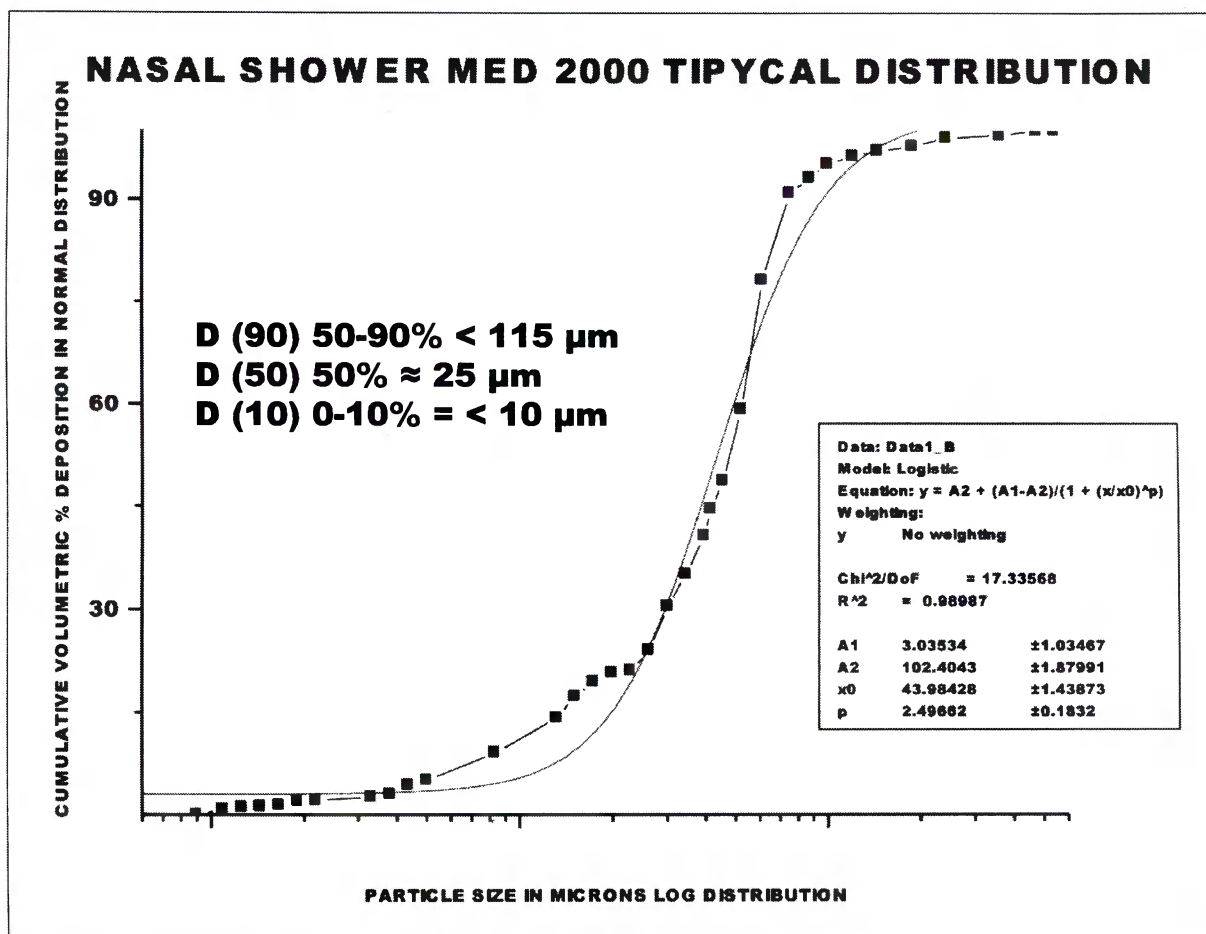
Dry and reassemble the irrigator making reference to the assembly diagram.

Each patient is recommended to use the irrigator Rinomed R1 for a maximum of 200 treatments.

IMPORTANT !!

Microbial contamination:

To prevent any risk of infection, personal use of the irrigator is advisable.

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Rinomed nasal shower ensures under a correct use a high particle flow with a cloud of nebulization with 90% of the particles having an aerodynamic size above (>10 microns). The median aerodynamic distribution obtained is usually around 23-26 microns. The latter distribution with more than 95% of the aerosolized particles above 8 microns guarantee that practically no particles are able to pass the Oropharyngeal tract and to reach the bronchial tract. With this kind of distribution patterns a local deposition is granted with minimal potential presence of systematic effect.

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This is a medical device for home use and must be used with a doctor's prescription.

- It can be used with saline, isotonic, hypertonic solutions, etc., thermal water or medicines normally found in pharmacies. A homemade solution can also be made to wash the nose daily by dissolving a teaspoon of salt in half a litre of warm water; in the case of specific diseases such as cystic fibrosis or immediately after surgery, it is recommended to use sterilised water or boiled water.

- The R1-model Rinomed unit can be combined with all Med2000 aerosol products.

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WARNINGS

The nebulizer chamber must be used with an air compressor conforming to the current laws. Table 1 shows the recommended flow and pressure values.

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The R1-model Rinomed nasal irrigator is classified as a Class IIa medical device and is CE0434 certified, as it conforms to the 93/42/EEC Medical Device Directive and the UNI EN 13544-1:2009 standard.

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*The **WARRANTY** certificate is only valid when the sales receipt is attached*

WARRANTY TERMS

- *The warranty covers any manufacturing or material defects, provided that the parts have not been tampered with by the user or unauthorised personnel.*
- *This warranty does not include:
accessories, components and parts subject to normal wear. Damage caused by incorrect use of the appliance, accidental damage such as knocks, falls, transportation, and however from causes not attributable to the manufacturer.*
- *In case of failure, please contact Med 2000 srl or your nearest authorised dealer, attaching the warranty certificate, sales receipt or purchase invoice; otherwise, the warranty will be void and the amount of the assistance charged.*

- *When returning the unit for repair, make sure to maintain the configuration of the original kit. Any changes to the inside of the packaging made from the time of purchase will affect the validity of the warranty.*
- *Any transport costs are to be paid by the customer.*
- *The manufacturer is not liable for any warranty extensions granted by third parties.*

ATTENTION: this is a medical device and therefore, for hygienic purposes, must not be replaced under any circumstance.

ATTENTION: the warranty is only valid if accompanied by a sales receipt or invoice.

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Med 2000 S.r.l.