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CERTIFICATE

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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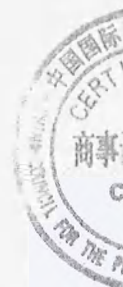
THIS IS TO CERTIFY THAT: the seal of LAKONG MEDICAL DEVICES CO., LTD. on the annexed DOCUMENT is genuine.

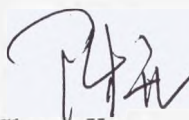
China Council for the Promotion
of International Trade

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Authorized
Signature:

Chen Yao

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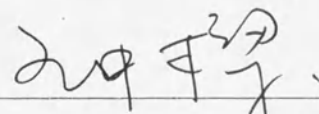
(Date: Sep. 03, 2019)

证明书签字网址 Website for verifying the certificate: <http://www.rzccpit.com/validate.html>

Approved

FEB. 14, 2019

President

Mr. Wang Zhongliang
Lakong Medical Devices Co Ltd., China
Shichong Industrial Park, Shupai, Dongguan, 523345, Guangdong, China

tel.: +86-(769)-81807881;

fax: +86-(769)-81807111

INSTRUCTION FOR USE
OF THE MEDICAL DEVICE:

Disposable dental microapplicator Clean+Safe[®]

1. Name of the medical device:

Disposable dental microapplicator Clean+Safe[®], versions:

Disposable dental microapplicator Clean+Safe[®], Regular, green color
Disposable dental microapplicator Clean+Safe[®], Regular, orange color
Disposable dental microapplicator Clean+Safe[®], Fine, yellow color
Disposable dental microapplicator Clean+Safe[®], Fine, blue color
Disposable dental microapplicator Clean+Safe[®], Superfine, white color
Disposable dental microapplicator Clean+Safe[®], Superfine, pink color
Disposable dental microapplicator Clean+Safe[®], X-Slim, black color

2. Information about the designer, the manufacturer and the authorized representative:

Designer: Lakong Medical Devices Co. Ltd.

Legal address: Shichong Industrial Park, Shipai, Dongguan, 523345, Guangdong, China

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Manufacturer: Lakong Medical Devices Co. Ltd.

Legal address: Shichong Industrial Park, Shipai, Dongguan, 523345, Guangdong, China

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Fax: +86-(769)-81807111

Web-site: <http://lakong.medincn.com>

The authorized representative in the Russian Federation:

“PROTECO”, JSC, legal and postal address: 192102, Saint-Petersburg, Salova street 57, Building 1, Letter “I”. Tel.: +7 (812) 635-88-90. E-mail: info@protecodent.ru.

3. Application of the medical device, potential consumers

Disposable dental microapplicator Clean+Safe[®] (hereinafter referred to as microapplicator, medical device, product), is intended for application of dental substances and materials on teeth of a patient.

Application:

– Dentistry.

Potential users of the medical device: dentists.

4. Contraindications, side effects :

- patient’s allergy to any component of the microapplicator;
- impossibility to observe the established procedure of application.

Side effects: none.

Instruction for use of the medical device:

Disposable dental microapplicator Clean+Safe[®]

The medical device is non-sterile, single-use. Repeated use of microapplicators may result in infection. Reuse is prohibited.

5. Construction and functional specification of the medical device.

The microapplicator is a dental hand tool in the form of a stick (handle), on the bending part of which there is a tip (head) of various diameters (Fig. 1). On the surface of the tip (head) the flock in the form of a brush is applied.

1. Fiber brushes (flock) do not adsorb liquids, solutions, varnishes, allowing to apply the entire amount of the taken material.
2. The tip, bent at different angles, allows to apply materials even in hard-to-reach areas.
3. Different color and changeable handle length (if necessary, it can be shortened by breaking off its lower part) eliminate the risk of cross-infection.
4. Microapplicator X-Slim has an even thinner elongated head, which makes possible to use it for agents or materials application into endodontic channels.

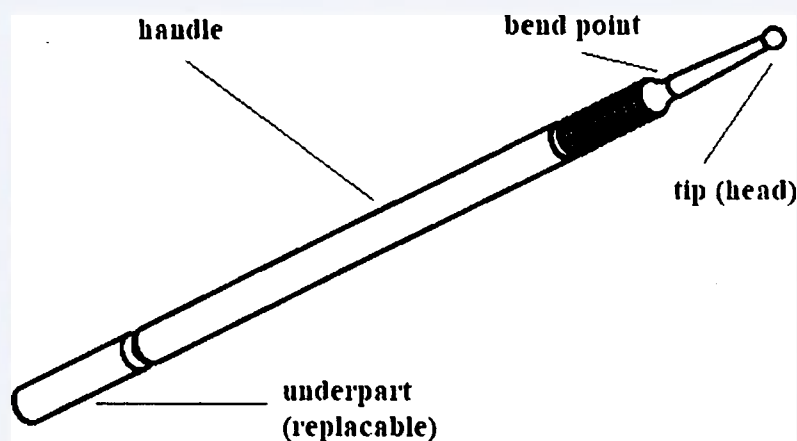
Figure 1. Construction of the microapplicator

6. Application procedure

- Choose a microapplicator of the required size. Turn carefully the feeder-cap and take out a microapplicator. Bend the microapplicator into the desired angle if necessary (it is not recommended to repeatedly bend the microapplicator).
- Take the dental material that is necessary to apply and place it (required amount) on the special mixing pad (or any other device of the same use)
- Using the microapplicator, take the necessary amount of the dental material (microapplicator's fibers retain the dental material but do not absorb it)
- Apply the dental material
- The microapplicator is intended for single use and must not be reused.

The medical device is non-sterile, single-use. Repeated use of microapplicators may result in infection. Reuse is prohibited.

Technical specification

Model	Length (l), mm	Length of the bottom part l ₁ , mm	Diameter of the handle (D), mm	Weight, g	Diameter of the tip d, mm	Diameter of the tip (head) with flock, mm	Diameter d ₁ , mm	Length of the tip (head) without flock, (l ₂), max., mm
Regular	102±2	16±0.1	3.2±0.1	0.70±0,2	1.7±0.12	2,7 ± 0,12	1.7±0.1	1.5
Fine				0.69±0,2	1.0±0.12	2,0 ± 0,12	1.7±0.1	1.5
Superfine				0.68±0,2	0.5±0.12	1,5 ± 0,12	1.7±0.1	1.5
X-Slim				0.68±0,2	0.5±0.12	1,5 ± 0,12	1.5±0.1	2.0

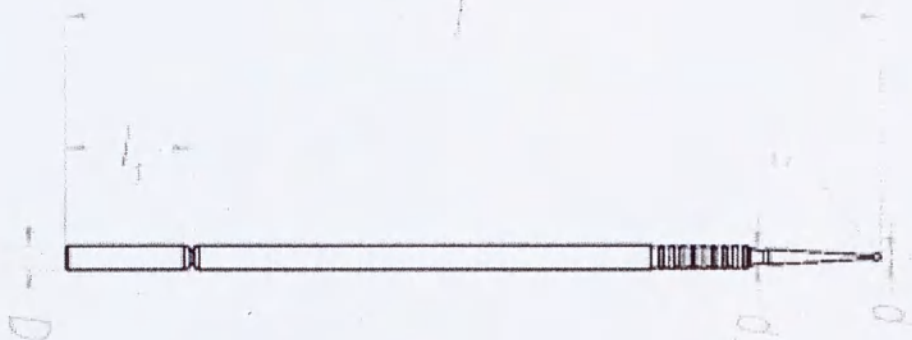
Figure 2. General view of the microapplicator

8. Operating, transportation and storage conditions

8.1. Keep in a dry well-ventilated room, avoid exposure to direct sunlight.

Storage conditions

Ambient temperature	-20 - +50°C
Relative humidity	10 to 80%
Air pressure	500 to 1060 hPa

8.2. Transportation conditions

Transportation of products packaged in shipping containers must be carried out by all types of closed vehicles in compliance with the precautions specified on the shipping container and in accordance with the rules for the carriage of goods in force for this type of transport.

Ambient temperature	-20 - +50°C
Relative humidity	10 to 80%
Air pressure	500 to 1060 hPa

8.3. Operational environment.

Operational temperature from +32°C to +42°C

Scope of supply

No.	Name	Quantity
I	Disposable dental microapplicator Clean+Safe® *	4 tubes of 100 pcs
II	Instruction for use	1 pc.

* – select the needed version within the range of presented variants:

Regular: green, orange

Fine: yellow, blue

Superfine: white, pink

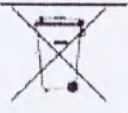
X-Slim: black

10. Shelf life

Shelf life: unlimited, provided that the storage and transportation requirements are observed.

11. Symbols and images used in labeling of the medical device and their description

- Symbols used in labeling:

СИМВОЛ	Описание	СИМВОЛ	Описание
	Lot number		Temperature range
	Non-sterile		CE marking
	For single use only		Dispose separately from household waste
	Catalogue No.		Symbol of conformance to the Russian standard GOST R

12. Procedures of disposal and destruction of the medical device

The medical device is to be disposed according to the rules of collection, record-keeping and disposal established by the competent federal executive body.

The medical device falls to category of epidemiologically hazardous medical waste: materials and instruments, objects contaminated with blood and/or other biological fluids.

Consumer package, defective device refer to epidemiologically safe waste, similar in composition to household waste.

List of international regulatory documents/standards applicable to the medical device

ISO 9001: 2015, ISO 13485:2016, EN 1041:2013, EN ISO 14971:2007, ISO 15223-1:2016, ISO 10993-1:2009, ISO 10993-2:2006, ISO 10993-5:2009, ISO 10993-10:2010, ISO 10993-12:2012.

14. Requirements for maintenance and technical repair of the medical device.

Products are easy to use, intended for single use; the medical devices do not require special training or special skills from the user. Do not require the maintenance and technical repair.

15. Warranty liabilities.

Lakong Medical Devices Co., Ltd., China guarantees that this product has been manufactured and packaged with the due care prescribed in EU Directive 93/42 / EEC. Lakong Medical Devices Co., Ltd, China, has EU Certificates assuring that the quality management system of Lakong Medical Devices Co., Ltd, China, complies with the requirements of the Annex VII of the Directive 93/42/EEC, ISO 13485:2016.

Lakong Medical Devices Co., Ltd, China, will replace any packaging, in case of detection of defects during manufacturing and packaging (evidence is provided), and reports of such defects to Lakong Medical Devices Co., Ltd , China, or its authorized representative.

The user is responsible for determining the suitability of the product for use. Lakong Medical Devices Co., Ltd, China, and its distributors are not responsible for any harm or damage of any nature incurred directly or indirectly when using the product.

If the product has become defective within the warranty period, your exclusive option and the sole responsibility of Lakong Medical Devices Co., Ltd, China, should be to correct or replace the product. Product liability is terminated if any damage has occurred due to transportation, storage or improper handling by the user, as well as other factors beyond the control of the manufacturer.

This Instruction for use was issued in June 2018.

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