align

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VP, Global Regulatory Affairs & Quality Assurance, Legal, Corporate Affairs & Regulatory

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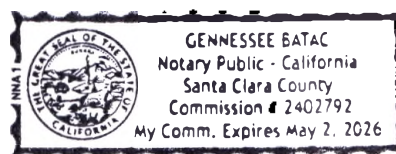
**Instruction for use
(For patients)**

**«Invisalign removable orthodontic aligners for the correction of malocclusion in sets,
with accessories»**

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personally appeared STEPHANIE FITS
Name(s) of Signer(s)

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I certify under PENALTY OF PERJURY under the laws of the State of California that the foregoing paragraph is true and correct.

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Description of Attached Document

Title or Type of Document: RU cover letter HUI's patient

Document Date: Aug. 25, 2022 Number of Pages: 1

Signer(s) Other Than Named Above: _____

Capacity(ies) Claimed by Signer(s)

Signer's Name: _____

☐ Corporate Officer – Title(s): _____

☐ Partner – ☐ Limited ☐ General

☐ Individual ☐ Attorney in Fact

☐ Trustee ☐ Guardian or Conservator

☐ Other: _____

Signer is Representing: _____

Signer's Name: _____

☐ Corporate Officer – Title(s): _____

☐ Partner – ☐ Limited ☐ General

☐ Individual ☐ Attorney in Fact

☐ Trustee ☐ Guardian or Conservator

☐ Other: _____

Signer is Representing: _____

/Logo: invisalign/

**Use and care instruction
for your Invisalign aligners**

Align Technology, Inc.
2820 Orchard Parkway
San Jose, CA 95134
USA

Align Technology B.V.
Zekeringstraat 41
1014 BV Amsterdam
The Netherlands

Patent www.aligntech.com/patents

0344

© 2019 Align Technology, Inc. Align, Invisalign, ClinCheck and iTero, among others, are trademarks and/or service marks of Align Technology, Inc. or one of its subsidiaries or affiliated companies and may be registered in the U.S. and/or other countries.

AW207020 Rev. B (2020-01) Rx only

Authorized representative in Russia and the CIS

Align Technology LLC
117105, Moscow, Varshavskoe Shosse 9, Building 1, Floor 3,
Premise II, Room 1
Telephone: +7 495 995 8854

/Logo: align/

/Logotype: invisalign/

Use and care instructions

for your Invisalign aligners

Wearing and using your Invisalign aligners

Here are some tips to help ensure proper use and avoid damaging your aligners.

Always remember to

1. Wear your aligners per your doctor's instructions, usually 20–22 hours per day.
2. Wash your hands thoroughly with soap and water before handling your aligners.
3. Handle only ONE aligner at a time.
4. Rinse your aligners when removing them from the packaging.

Use proper technique described below when inserting and removing aligners to avoid damaging aligners.

Aligner insertion

1. Prior to each insertion and before bed, inspect aligners for cracks and deformation. Severe cracks or deformities should be reported to the doctor immediately. Aligners with cracks the size of the height of the tooth should not be used.

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2. Make sure you have the proper aligner—the upper for your top teeth and the lower for your bottom teeth.
3. To help avoid confusion, each aligner is engraved with your unique case number, a "U" for upper and an "L" for lower, followed by the stage number. (Figure 1)

Figure 1

4. You may insert either the upper or the lower aligner first. When inserting each aligner, gently push the aligners over your front teeth. Then apply equal pressure, using your fingertips, to the tops of your left and right molars (back teeth) until the aligner snaps into place. (Figure 2)

Figure 2

5. Do NOT bite your aligners into position. This may damage them. For aligners with precision wings (figures 3a and 3b, precision wings circled), do NOT bite down on precision wings. The lower precision wings should be positioned in front of the upper precision wings when the aligners are in the mouth.

Figure 3a

Figure 3b

Note: If you experience sharp pain or significant discomfort, discontinue the use of the aligners and contact your doctor.

Aligner removal

1. On one side of your mouth, use your fingertip on the inside of your back molar to slowly pull the aligner from your molars. (Figure 4)
2. Repeat this process on the other side of your mouth before trying to completely remove the aligner. (Figure 5)
3. Once aligner is disengaged from the molars on both sides of your mouth, you should be able to slowly work your way forward gently prying the aligner away from your teeth with your fingertips. (Figure 6)

Figure 4

Figure 5

Figure 6

Note:

- Immediately rinse aligner with water, shake off excess water, and store your aligners in the protective case provided with your starter kit.
- Aligners are intended to only be removed for eating, drinking, smoking, and for oral care. Unnecessary and excessive removal/insertion of aligners with precision wings may prematurely damage and/or break the aligners.
- Take care in removing your aligners, especially if multiple attachments are being used.
- Do NOT use excessive force to bend or twist an aligner to get it off.
- Do NOT use any sharp object to remove your aligners.
- Consult with your doctor if your aligners are extremely difficult to remove.

Daily care and maintenance of your Invisalign aligners

1. Clean your aligners prior to each insertion. Use a soft bristle toothbrush with water and a small amount of toothpaste. You may find it easiest to clean the outside of your aligners by brushing them while they are still on your teeth (Figure 7), then remove your aligners to clean the inside surfaces (Figure 8).

Figure 7

Figure 8

Note: Be sure to rinse each aligner thoroughly with water after each cleaning. We also recommend that you use the Invisalign Cleaning Crystals once a week, or as needed.

2. Prior to each insertion and before bed, inspect aligners for cracks and deformation. Severe cracks or deformities should be reported to the doctor immediately. Aligners with cracks the size of the height of the tooth should not be used.

Note: Parents or guardians should inspect aligners of pediatric patients.

3. Do NOT use denture cleaners to clean aligners. Do NOT soak them in mouthwash. These products can damage the surface of the aligner, causing it to become dull and more visible.

Proper oral hygiene

1. Remove your aligners for eating and drinking (You do not need to remove your aligners to drink cool water.)
2. Brush and floss your teeth after each meal or snack prior to re-inserting your aligners. If you do not have access to your cleaning system or toothbrush, you can simply rinse your mouth, and then clean your aligners by holding them under warm running water. It is not the best way to clean, but it works in a pinch. Be sure to thoroughly clean your aligners at your earliest convenience.
3. If you have any questions regarding hygiene techniques, please consult your doctor.
4. Regular dental checkups and cleaning are recommended for the continued health of your teeth and gums.

Storing your Invisalign aligners

Aligner storage case(s) are included in your starter kit. We recommend that you store your Invisalign aligners in a case when they are not in your mouth (Figures 9 and 10). This will help protect them from loss and damage. Keep aligner cases clean by hand washing them with soap and water. Do not wash them in dish washing machines.

Figure 9

Figure 10

Always keep the most recently used aligners. If your current aligner is lost or broken, your doctor may recommend that you temporarily go back one stage while a replacement is being made.

Note: Keep all of your older aligners in a clean plastic bag, or as instructed by your doctor. Keep them out of reach of small children and pets.

Frequently asked questions and answers

What if I lose or break an Invisalign aligner?

In the event that an aligner is lost or broken, you should immediately inform your doctor. Your doctor will probably tell you to start wearing your last set or next set of aligners immediately. He or she will possibly order you a new set of aligners to replace the ones you just lost, which should arrive in a few days.

For patients undergoing mandibular advancement, why do some of my aligners have "wings" on them?

The "wings" on the sides of aligners are called precision wings, and are especially designed to position the lower jaw forward to help correct the bite.

Will the treatment be painful?

Most people experience tooth soreness for a few days after starting each new stage. This is normal. It is a sign that the Invisalign aligners are working, moving your teeth to their final destination. This

soreness should gradually go away a couple of days after inserting the new aligner in the series. If it does not, promptly call your doctor.

Will wearing the Invisalign aligners affect my speech?

Like all orthodontic treatments, the aligners may temporarily affect your speech, and you may have a slight lisp for a day or two. However, as your tongue gets used to having aligners in your mouth, any lisp or minor speech impediment caused by your aligners should disappear.

What should I do if my new Invisalign aligner does not snap onto my teeth?

Minor discrepancies between the new aligner and the current tooth position are normal, since the teeth need time to conform to the new aligner position. In the event of significant problems with aligner fit, inform your doctor.

Are there restrictions on what I can eat?

In general, no. Unlike traditional orthodontics, you can usually eat and drink whatever you desire because you remove your aligners while eating. Thus, there is no need to restrict your consumption of any of your favorite foods and snacks, unless otherwise instructed by your doctor.

Is it OK to drink hot or cold beverages while wearing Invisalign aligners?

Except for cool water, we recommend that you do not drink while wearing the aligners. This is to avoid formation of cavities and stains, or warping of the aligners with hot drinks and hot water.

Can I chew gum while wearing Invisalign aligners?

NO. Gum will stick to the aligners. We recommend removing your aligners for all snacks and meals.

Will smoking or chewing tobacco stain the aligners?

We discourage tobacco use while wearing aligners because of the possibility of aligner discoloration.

Why do some of my aligners have bumps or ridges on them?

Depending on your specific treatment, some of the movements may require either "attachments" or "ridges" to help the aligner grip the teeth. These bumps, or wells, are where the aligner grips the attachment the doctor places on your teeth. The attachments are actually small pieces of composite the doctor affixes to your teeth that are then gripped by bumps on your aligners. The ridges are slim indentations in your aligners. Your doctor will use one or both of these features to attempt to achieve the desired movement.

What if I lose or break an attachment?

In the event that an attachment is lost or broken, you should immediately contact your doctor.

Note: If you have additional questions after reading this pamphlet, please consult with your doctor.

Warning: In rare instances, some patients may be allergic to the plastic aligner material. In such cases, discontinue use and consult a health care professional immediately. Align Technology must also be notified. Orthodontic appliances, or parts thereof, may be accidentally swallowed or aspirated and may be harmful.

For professional dental use – Rx only

Symbol glossary

Symbol	Title	Explanatory text	Standard reference
	Consult instructions for use	Indicates the need for the user to consult the instructions for use.	EN ISO 15223-1*; 5.4.3
	Date of manufacture	Indicates the date when the medical device was manufactured.	EN ISO 15223-1*; 5.1.3
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.	EN ISO 15223-1*; 5.4.4
	Patient number	Indicates a unique number associated with an individual patient.	EN ISO 15223-1*; 5.7.1
	Manufacturer	Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC.	EN ISO 15223-1*; 5.1.1
	Authorized representative in the European Community	Indicates the authorized representative in the European Community.	EN ISO 15223-1*; 5.1.2
Rx only	Prescription Use Only	Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.	US Code of Federal Regulations, Title 21, Part 801.109(b)(1)
 0344	CE marking Conformance Européenne Notified Body Reference No. 0344	Signifies European technical conformity.	MDD 93/42/EEC Articles 4,11,12,17, Annex II

*Medical devices — Symbols to be used with medical device labeling and information to be supplied — Part 1: General requirements.

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2820 Orchard Parkway, San Jose, CA 95134, United States of America

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Instruction for use

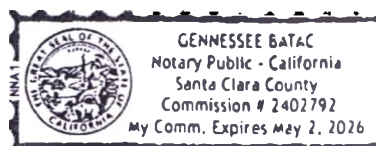
(For doctors)

«Invisalign removable orthodontic aligners for the correction of malocclusion in sets, with accessories»

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CALIFORNIA ACKNOWLEDGMENT

CIVIL CODE § 1189

A notary public or other officer completing this certificate verifies only the identity of the individual who signed the document to which this certificate is attached, and not the truthfulness, accuracy, or validity of that document.

State of California

County of santa clara }

On Aug. 25, 2022 before me, Genessee Batac, Notary Public
Date Here Insert Name and Title of the Officer

personally appeared Stephanie Hitts
Name(s) of Signer(s)

who proved to me on the basis of satisfactory evidence to be the person(s) whose name(s) is/are subscribed to the within instrument and acknowledged to me that he/she/they executed the same in his/her/their authorized capacity(ies), and that by his/her/their signature(s) on the instrument the person(s), or the entity upon behalf of which the person(s) acted, executed the instrument.

i certify under PENALTY OF PERJURY under the laws of the State of California that the foregoing paragraph is true and correct.

WITNESS my hand and official seal.

Place Notary Seal and/or Stamp Above

Signature [Signature]
Signature of Notary Public

OPTIONAL

Completing this information can deter alteration of the document or fraudulent reattachment of this form to an unintended document.

Description of Attached Document

Title or Type of Document: PH CARD LETTER HUGO DOCTOR

Document Date: Aug. 25, 2022 Number of Pages: 1

Signer(s) Other Than Named Above: _____

Capacity(ies) Claimed by Signer(s)

Signer's Name: _____

☐ Corporate Officer – Title(s): _____

☐ Partner – ☐ Limited ☐ General

☐ Individual ☐ Attorney in Fact

☐ Trustee ☐ Guardian or Conservator

☐ Other: _____

Signer is Representing: _____

Signer's Name: _____

☐ Corporate Officer – Title(s): _____

☐ Partner – ☐ Limited ☐ General

☐ Individual ☐ Attorney in Fact

☐ Trustee ☐ Guardian or Conservator

☐ Other: _____

Signer is Representing: _____

/Logo: invisalign/

Invisalign System Instructions for Use

Device Description

The Invisalign System consists of a series of doctor-prescribed, customized, thin, clear plastic removable orthodontic appliances (aligners) that gently move the patient's teeth in small increments and may also position the mandible forward from their original state to a more optimal, treated state.

Indications for Use

The Invisalign System is indicated for orthodontic treatment of malocclusion.

Contraindications

The Invisalign System is contraindicated for use in patients with the following condition:

- Active periodontal disease.

Warnings

- In rare instances, some patients may be allergic to the plastic aligner material.
- In rare instances, patients with hereditary angioedema (HAE), a genetic disorder, may experience rapid local swelling of subcutaneous tissues including the larynx. HAE may be triggered by mild stimuli including dental procedures.
- In both cases, discontinue aligner use and consult a health care professional immediately. Align Technology must also be notified.
- Orthodontic appliances, or parts thereof, may be accidentally swallowed or aspirated and may be harmful.

Precautions

- Aligners are intended to only be removed for eating, drinking, and for oral care. Unnecessary and excessive removal and insertion of aligners with precision wings may prematurely damage and/or break the aligners.
- To avoid aligner damage, instruct patients to use proper techniques to insert and remove aligners.
- For aligners with precision wings, biting directly on the precision wings or positioning them on top of each other may increase a risk of aligner deformation or cracking.
- Keep aligners away from hot water and harsh chemicals.
- Keep aligners out of reach of young children and pets.
- Always store aligners in a cool, dry place.
- Treatment of severe open bite, severe overjet, mixed dentition and/or skeletally narrow jaw may require supplemental treatment in addition to aligner treatment.
- Skeletal expanders, bonded auxiliaries, elastics, and other fixed and/or removable orthodontic appliances may be needed for more complicated treatment plans where aligners alone may not be adequate to achieve the desired outcome.
- Oral surgery may be necessary to correct severe crowding or jaw imbalances. If oral surgery is required, risks associated with anesthesia and proper healing must be taken into account prior to treatment.
- Atypically-shaped, erupting and/or missing teeth may affect aligner adaptation and may affect the ability to achieve the desired treatment results.
- Existing dental restorations (e.g., crowns, bridges) may become dislodged and require re-cementation or, in some instances, replacement.
- Aligners are not effective in the movement of dental implants.
- Certain medical conditions and use of certain medications may affect orthodontic treatment outcomes.
- Dental tenderness may be experienced following the initial aligner placement and after switching to each new aligner in the series.
- The aligners may temporarily affect the speech and may result in a slight lisp, although any speech impediment caused by the aligners should disappear within one or two weeks.
- Failure to wear the appliances for the required number of hours per day and/or not using the product as directed by your doctor can lengthen the treatment time and affect the ability to achieve the desired results.
- Gums, cheeks, or lips may be scratched or irritated by the product.
- Attachments may be temporarily bonded to one or more teeth during the course of treatment to facilitate aligner retention and/or tooth movement.
- Attachments may fall off and require replacement.
- All attachments should be removed at the end of treatment.
- Risks associated with interproximal reduction, which may be prescribed to create space for tooth movement as part of treatment must be taken into account.
- Tooth decay, periodontal disease, and permanent markings from stains and decalcification may occur if patients do not brush and floss their teeth properly during treatment or if they consume foods or beverages containing sugar while wearing aligners.
- The bite may change throughout the course of treatment and may result in temporary patient discomfort.
- At the end of the treatment, the bite may require adjustment (occlusal adjustment) by the doctor.
- Teeth may shift position after treatment. Constant wearing of retainers at the end of treatment should reduce this tendency.
- All teeth should be at least partially covered by retainers to avoid supra-eruption of teeth.
- Temporary increase in salivation or mouth dryness may also be caused.
- A tooth that has been previously traumatized, or significantly restored may be aggravated. In rare instances the useful life of the tooth may be reduced, the tooth may require additional dental treatment such as endodontic and/or additional restorative work and the tooth may be lost.
- The health of the bone and gums, which support the teeth, may be impaired or aggravated.
- The length of the roots of the teeth may be shortened during orthodontic treatment, which may become a threat to the longevity of the teeth.
- In rare cases, problems with the temporomandibular joint (TMJ) may occur, causing joint pain, headaches or ear problems.

Directions for use

1. Rinse new aligners with cold water prior to using them.
2. If tooth extraction has been performed as a part of treatment, retain the space while the area is healing. Attachments should be placed only after complete healing of the extraction site.
3. Review the "Use and care instructions for your Invisalign aligners" brochure with your patient.
4. Each set of aligners is designed to be replaced after 1–2 weeks when worn 20–22 hours a day or at doctor's discretion.
5. Seat aligner No. 1 on the patient's teeth, then visually inspect soft tissues for pinching (impingement). If necessary, trim the aligners. After trimming, polish the aligners using the EZ-Trim™ Polishing Wheels (or similar product).
6. Precision wings are specifically designed for moving the lower jaw forward while simultaneously correcting malocclusion. It is important to make sure that the precision wings are in the proper engaged position (lower wings in front of upper wings) when the aligners are worn.
7. Complete any attachment placement and perform any necessary interproximal reduction (IPR) procedures using the Invisalign Interproximal Reduction Kit (or similar product).
8. Add pontics to the aligners, if applicable, using the Invisalign Pontic System (or similar product).
9. Provide the patient with Invisalign aligner storage cases. Instruct patient to store aligners in a case whenever not in use to protect them from loss or damage.
10. During each visit, inspect teeth for tight interproximal contacts and, if necessary, perform IPR procedures as prescribed. Also, check that the aligner matches the current treatment stage and fits snugly to the teeth, and, if necessary, review the ClinCheck setup to monitor the treatment progress.
11. Dispense the next set of aligner stages. The patient should be given enough aligners to last them until next appointment.
12. In case additional aligners are needed, new impressions may be required.
13. At the end of treatment, retainer wear is highly recommended.
14. Additional information can be found in the Invisalign training materials.

Daily care and maintenance of your Invisalign Clear Aligners

1. Prior to each insertion and before going to bed, inspect aligners for cracks and deformation. Inform your dentist immediately if major cracks or deformations are found. Aligners with cracks the size of the height of the tooth should not be used.

Note: Parent or guardian should inspect aligners of pediatric patients.

2. Aligners should be cleaned prior to each insertion using a soft bristle toothbrush with water and a small amount of toothpaste.

Note: Rinse aligners thoroughly with water after each cleaning. Additionally, we recommend to use Invisalign cleaning products for aligners cleansing once a week, or as needed.

3. DO NOT use denture cleansers to clean your aligners. DO NOT soak aligners in mouthwash. These products can damage the surface of the aligners, causing them to become dull and more visible.
4. Aligner storage case(s) are included in the starter kit. Invisalign aligners should be stored in a case when not in use. Aligners with precision wings should be stored in the appropriate aligner case that is designed with more space to accommodate this product.
5. Aligner cases may be washed manually with soap and water. Do not use dishwashers to clean the cases.

Proper oral hygiene

1. Aligners should be removed for eating and drinking (Patients do not need to remove aligners to drink cool water.)
2. Patients should brush and floss their teeth after each meal or snack and prior to re-inserting their aligners. If they do not have access to Invisalign cleaning system or a toothbrush, they can simply rinse their mouth, and then clean the aligners by holding them under warm running water. Aligners should be thoroughly cleaned at the patient's earliest convenience.
3. If the patient has any questions regarding hygiene techniques, they should consult with their doctor.
4. Regular dental checkups and cleaning are recommended for the continued health of a patient's teeth and gums.

Package Contents

Each aligner is cleaned and packaged prior to shipment. Each aligner contains the Invisalign logo and a number, which identifies the patient ID number and treatment stage.

Symbol glossary

Symbol	Title	Explanation	Standard reference
	Consult Instructions for use	Indicates the need for the user to consult the instructions for use	EN ISO 15223-1*; 5.4.3
	Date of Manufacture	Indicates the date when the medical device was manufactured.	EN ISO 15223-1*; 5.1.3
	Caution!	Indicates the need for the user to consult the instructions for use for important cautionary information, such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself	EN ISO 15223-1*; 5.4.4
	Patient Number	Indicates a unique number associated with an individual patient	EN ISO 15223-1*; 5.7.1
	Manufacturer	Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC.	EN ISO 15223-1*; 5.1.1
	Authorized EU Representative	Indicates the authorized representative in the European Union	EN ISO 15223-1*; 5.1.2
Rx only	Prescription Use Only	Caution: US Federal law restricts this device to sale by or on the order of a licensed dental professional.	US Code of Federal Regulations, Title 21, Part 801.109(b)(1)

*Medical devices — Symbols to be used with medical device labeling and information to be supplied — Part 1: General requirements

/Logo: align/

Align Technology, Inc.
2820 Orchard Parkway
San Jose, CA 95134
USA

Align Technology B.V.
Zekeringstraat 41
1014 BV Amsterdam
The Netherlands

Patent www.aligntech.com/patents

Authorized representative in Russia and the CIS
Align Technology LLC
117105, Moscow, Varshavskoe Shosse 9, Building 1, Floor 3,
Premise II, Room 1
Telephone: +7 495 995 8854

© 2019 Align Technology, Inc. Align, Invisalign, ClinCheck and iTero, among others, are the trademarks and/or service marks of Align Technology, Inc. or one of its subsidiary or affiliated companies, and may be registered in the USA and/or other countries.

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AG-3236