
38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016 JAPAN
Phone: +81 3-5817-8665 Fax: +81 3-3835-2265

Date: 27th of February, 2023
«Approved by»

Hideki Kazama
President
Tokuyama Dental Corporation

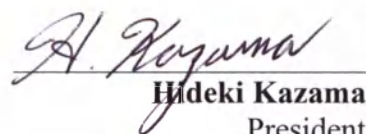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This product is certified as a medical device in the European Union under the Medical Device Directive 93/42/EEC by SGS CE1639, exclusively for the indication(s) shown in the IFU of the product. Other non-medical uses ascribed to this product are not within the scope of CE certification, and users should be aware product performance and/or safety has not been evaluated by SGS for those purposes.

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Outline

Tokuyama Universal Bond II

Tokuyama Universal Bond II can be adhere to any materials as well as teeth without additional primers.

ESTECER II PLUS

ESTECER II PLUS is a dual-cure(light and/or self-cure), radiopaque, adhesive resin cement system with excellent handling, esthetic and adhesive properties to teeth and all prosthetic materials.

SHADE AVAILABLE

Restoration

metals/alloys (precious and non-precious)
glass/oxide ceramics
(porcelain, zirconia and alumina)
resin materials including inorganic filler
(composite materials)


Restoration

Tooth

Teeth (metal cores/resin cores)

Even if the adherent surface includes materials other than tooth structure such as ceramics, porcelain, composite, and/or metal, **no extra primers are needed.**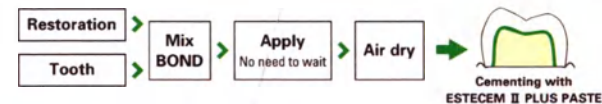

For Tokuyama Universal Bond II, the mixing state can be visually checked by its color change.

•Etching of the tooth structure (optional)

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The PASTE and Tokuyama Universal Bond II should be brought to room temperature before using.

STORAGE - Store the ESTECM II PLUS KIT, PASTE under refrigeration at temperatures between 0 to 10°C (32 to 50°F)
- Store the Tokuyama Universal Bond II at temperatures between 0 to 25°C (32 to 77°F)

If the syringe is not brought up to room temperature, you will find it difficult to squeeze out the PASTE.
If the bottle is not brought up to room temperature, you will find it difficult to dispense one drop of liquid.

How to use

ESTECM II PLUS KIT provides two types of mixing wells.

Rubber

Working time within 1 minute of dispensing

For repeated use

The cured Tokuyama Universal Bond II can be removed by inverting the rubber.

Disposable mixing well


Working time within 3 minutes of dispensing

For single use

Example of use


How to attach the mixing tip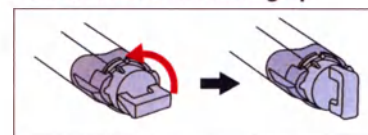


Turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves in the syringe


Holding the base of the cap, remove the cap by twisting and pressing downward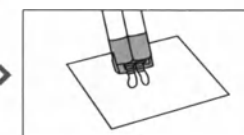

Before each use, squeeze out a small quantity of PASTE A and B, making sure equal amounts are being dispensed through the two outlets of the syringe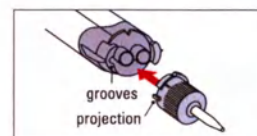

Attach a mixing tip to the syringe aligning the projections on the mixing tip with the grooves in the syringe as closely as possible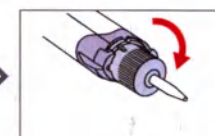

Turn the mixing tip 1/4 of a turn clockwise to lock it in place


Restoration pre-treatment overview

Restoration preparation


- *1
- **Ceramics other than porcelain, and composite materials**
Roughen the interior of the restoration by sandblasting (0.1 to 0.2MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion.
 - **Porcelain**
DO NOT roughen the surface; Prepare the interior of the restoration, providing a clean and fresh surface for adhesion.
 - **Metal restorations**
Roughen the area by sandblasting (0.3 to 0.5MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion.

Cores(resin/metal) preparation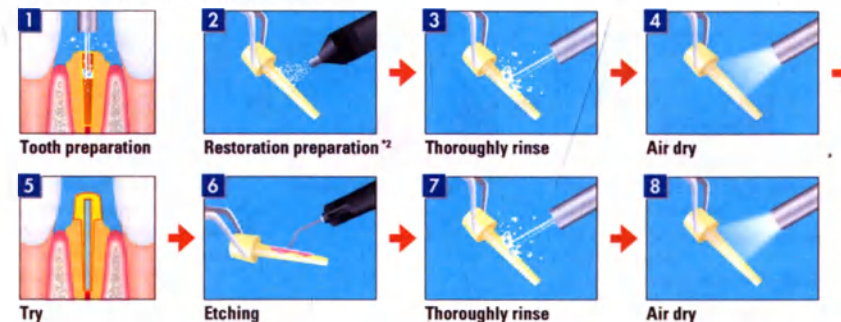


- *2
- **Resin cores/glass-fiber posts cores**
Roughen the interior of the restoration by sandblasting (0.1 to 0.2MPa), air abrasion.
 - **Metal cores**
Roughen the area by sandblasting (0.3 to 0.5MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion.

Cementation of Indirect restorations (metals/alloys/glass/porcelain/zirconia/alumina /composite materials)

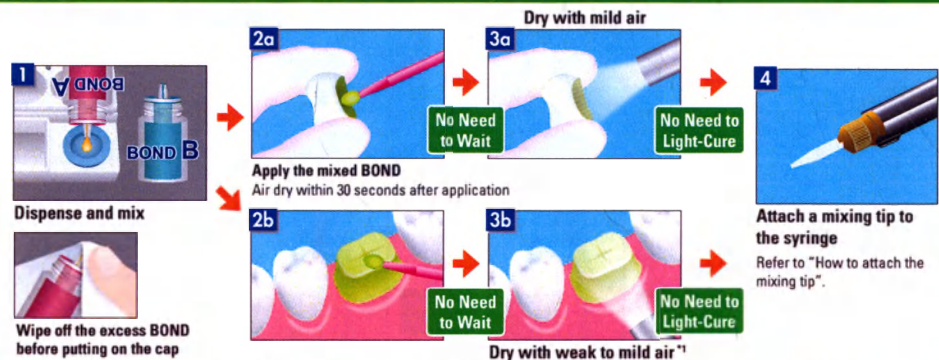
^{*1} Apply weak air continuously to the BOND surface until the runny BOND stays in the same position without any movement. Finish by using mild air to the surface.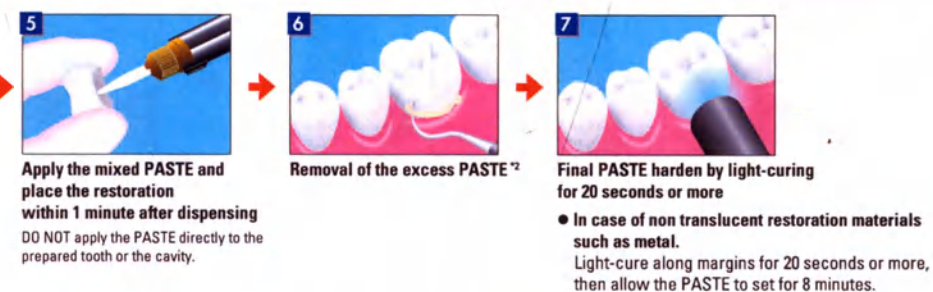

^{*2} • When using the light-cure method
Light-cure the excess PASTE with a dental curing light for 2 to 4 seconds and remove the gel state of the excess PASTE.
• When using the self-cure method
Remove the gel state of the excess PASTE within 1 to 3.5 minutes after seating.

Cementation of cores (metal or resin cores/glass-fiber posts)

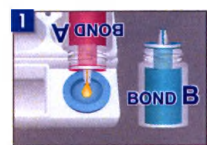Dispense and mix

Wipe off the excess BOND before putting on the cap

Apply the mixed BOND


Air dry within 30 seconds after application

*1

Dry with mild air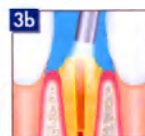

Dry with weak to mild air


No Need to Wait

No Need to Light-Cure


Attach a mixing tip to the syringe

Refer to "How to attach the mixing tip".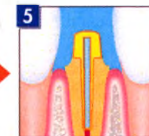


Apply the mixed PASTE and place the restoration^{*2}

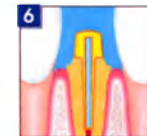Removal of the excess PASTE^{*3}

Final PASTE harden by light-curing for 20 seconds or more

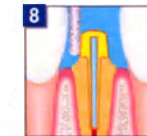Abutment tooth formation

• In case of non translucent restoration materials such as metal. Light-cure along margins for 20 seconds or more, then allow the PASTE to set for 8 minutes.

*2 If the PASTE is applied directly into the cavity, place the core or post into the cavity within 30 seconds after application of the PASTE.

*3 • When using the light-cure method

Light-cure the excess PASTE with a dental curing light for 2 to 4 seconds and remove the gel state of the excess PASTE.

• When using the self-cure method

Remove the gel state of the excess PASTE within 1 to 3.5 minutes after seating.

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This public document

2. has been signed by KATO Tomohiro

3. acting in the capacity of Notary of the Tokyo Legal Affairs Bureau

4. bears the seal/stamp of KATO Tomohiro, Notary

Certified

5. at Tokyo

6. MAR. 03. 2023

7. by the Ministry of Foreign Affairs

8. 23- NO 014468

9. Seal/stamp:

10. Signature

HAMAMOTO Hiroki

For the Minister for Foreign Affairs

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On this 3rd day of March 2023 , Tomoko MIFUNE personally appeared before me, KATO Tomohiro , a notary in and for Tokyo Legal Affairs Bureau , with satisfactory evidence of her identification and of her being the agent of Hideki KAZAMA, President of Tokuyama Dental Corporation, in accordance with the Japanese Notary Act, stated that the principal , said Hideki KAZAMA , had acknowledged to have affixed his signature and seal to the attached document.

In witness whereof , I set my hand and seal .

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1. Medical device name

Dental cement "ESTECCEM II PLUS KIT":

1. Dental cement ESTECCEM II PLUS (PASTE A (2.3 mL (4.7 g) and PASTE B (2.3 mL (4.7 g)) in syringe – 1 pc.
2. Tokuyama Universal Bond II BOND A (bottle 5 mL) – 1 pc.
3. Tokuyama Universal Bond II BOND B (bottle 4 mL) – 1 pc.
4. Cement Mixing Tip – 15 pcs.
5. MINI MIXING TIP – 5 pcs.
6. NOZZLE FOR MINI MIXING TIP – 5 pcs.
7. DISPOSABLE APPLICATOR (FINE) – 25 pcs.
8. MIXING WELL with rubber cup – 1 pc.
9. 1-Dimple Disposable Mixing Well – 5 pcs.
10. Instructions for Use – 1 pc.
11. Clinical Guide – 1 pc.

Hereinafter referred to as material, device, cement, ESTECCEM II PLUS, PASTE A, PASTE B, Bond A, Bond B.

2. Information about the manufacturer, designer and authorized representative

Designer

Tokuyama Dental Corporation
38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan
Tel.: +81-3-3835-7201
URL: <http://www.tokuyama-dental.com>

Manufacturer

Tokuyama Dental Corporation
38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan
Tel.: +81-3-3835-7201
URL: <http://www.tokuyama-dental.com>

Manufacturing site

Tokuyama Dental Corporation Kashima Factory
26, Sunayama, Kamisu-shi, Ibaraki 314-0255, Japan

Authorized representative of the manufacturer in the Russian Federation

PROTECO JSC

Legal address: 192102, St. Petersburg, 57 Salova str., bldg. 1, lit. I.

Business address: 196128, St. Petersburg, 5 Varshavskaya str., bldg. 2, office 401, Business Center Varshavsky

Tel.: +7 (812) 635-88-90; Email: info@protecodent.ru

3. Intended use

This device is intended for cementation of dental prosthetic materials such as metals, ceramics and composite materials to tooth structure or in case of their repair.

Instruction for use for the medical device:

Dental cement "ESTECCEM II PLUS KIT"

4. Potential consumer, scope of application

Potential consumers of a medical device: dentists.

Scope of application: dentistry.

5. Indications, contraindications and possible side effects associated with the use of the medical device as directed

Indications

- Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia and alumina), metals/alloys (precious and non-precious) and resin materials including inorganic filler (composite materials).
- Repair of fractured porcelain fused to metal crowns and all ceramic restorations.
- Cementation of veneers.
- Cementation of adhesion bridges.
- Cementation of metal or resin cores, metal or glass-fiber posts.

Contraindications:

This device contains methacrylic monomers, peroxide, organic solvents and acids. DO NOT use this device for patients allergic to or hypersensitive to methacrylic monomers, related monomers, peroxide, organic solvents and acids.


Possible side effects: allergic reactions to or hypersensitive to methacrylic and similar monomers, peroxide, organic solvents and acids or any other ingredients.

6. Description of the device, principle of operation of the medical device

The kit consists of dental cement ESTECM II PLUS, which is a cement paste based on a composite (PASTE A and B) and a primer for preparation of the hard tooth tissues and prosthetic materials Tokuyama Universal Bond II (BOND A and BOND B). Dental cement ESTECM II PLUS (PASTE A and PASTE B) is a dual-curing radiopaque material (cured by chemical reaction and/or photopolymerization (dual-curing). When Cement PASTE (A and B) is cured on the adhesive surface between the tooth and the prosthetic material, adhesion occurs.

Adhesive (primer) Tokuyama Universal Bond II is a two-component (Bond A and Bond B) self-curing dental adhesive system that promotes adhesion of ESTECM II PLUS to tooth structure, and also enhances the adhesion of ESTECM II PLUS to all prosthetic materials.

Tokuyama Universal Bond II is a solvent-thinned monomer liquid and consists of two fluid types (BOND A and BOND B). BOND B contains a polymerization initiator. After mixing these two liquids, then applying the resulting mixture to the adherend surface and removing the solvent contained

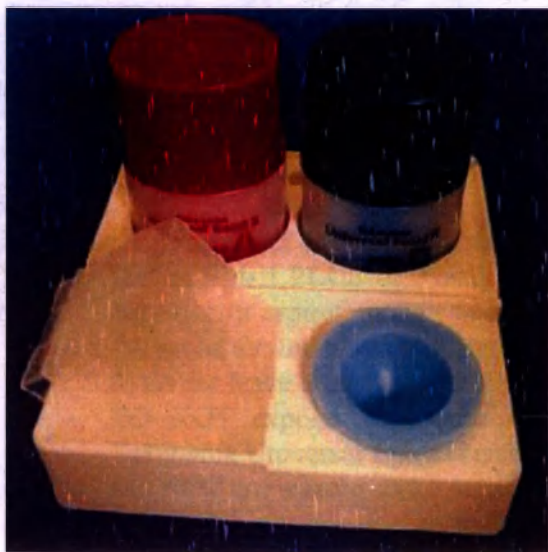Instruction for use for the medical device:

Dental cement " ESTECM II PLUS KIT"

in this device by blowing with air, polymerization of the monomers occurs during the polymerization reaction, which ultimately leads to the formation of a polymer layer from this device on the adherend surface.

Other components included in the kit: cement mixing tip, mini mixing tip, nozzle for mini mixing tip, disposable applicators (Fine), mixing well with rubber cup, 1-dimple disposable mixing well for dispensing, mixing and applying of materials. Mixing well is a plastic platform with contoured cells and rubber cup for material mixing, in which bottles with Tokuyama Universal Bond II, 1-Dimple disposable mixing well (5 pcs) are placed.

7. Precautions

- Use Tokuyama Universal Bond II as primer for ESTECER II PLUS cement. DO NOT use other primers or adhesives as primer for adhesive surfaces.
- DO NOT USE ESTECER II PLUS for any purpose other than those specified in this manual. Use ESTECER II PLUS only as described below.
- ESTECER II PLUS is designed for sale and use by licensed dental care professionals only. This material is not designed for use by non-dental care professionals.
- DO NOT USE the kit if the safety seals are broken or appear to have been tampered with.
- If ESTECER II PLUS causes an allergic reaction or oversensitivity, discontinue its use immediately.
- Use examination gloves (polymer, vinyl or latex) at all times when handling ESTECER II PLUS to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If ESTECER II PLUS (PASTE A and B) or Tokuyama Universal Bond II comes in contact with examination gloves, remove and dispose of the gloves, and wash your hands thoroughly with water as soon as possible.
- Avoid contact of ESTECER II PLUS or Tokuyama Universal Bond II with eyes, mucousal membrane, skin and clothing.
 - If ESTECER II PLUS or Tokuyama Universal Bond II comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If ESTECER II PLUS or Tokuyama Universal Bond II comes in contact with the mucousal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is seated. Affected areas may whiten from protein coagulation; however, such whitening should disappear within 24 hours. If such whitening does not disappear within 24 hours, the patient should be advised to contact a physician immediately.
 - If ESTECER II PLUS or Tokuyama Universal Bond II comes into contact with the skin or clothing, immediately saturate the area with an alcohol-soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- ESTECER II PLUS or Tokuyama Universal Bond II should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- To avoid the unintentional ingestion of ESTECER II PLUS or Tokuyama Universal Bond II, do not leave it unsupervised within the reach of patients and children.
- DO NOT expose Tokuyama Universal Bond II or its vapor to open flame because Tokuyama Universal Bond II is flammable.
- DO NOT use mixing wells supplied by other manufacturers.
- To avoid cross infection or lowering bond strength, DO NOT reuse the disposable applicator and the disposable mixing well. Clean the mixing well thoroughly with alcohol (not less than 70%) after each use.

Instruction for use for the medical device:

Dental cement "ESTECER II PLUS KIT"

- DO NOT use ESTECCEM II PLUS directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended. DO NOT use EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit curing of Tokuyama Universal Bond II.
- Protective shields, goggles or glasses should be worn at all times when using a light-curing unit.
- Be aware that when a restoration is chipped from misaligned occlusion or bruxism (clenching, grinding or tapping), the repaired restoration may chip again.
- DO NOT mix ESTECCEM II PLUS or Tokuyama Universal Bond II with other brands of paste, primers or adhesives.
- ESTECCEM II PLUS or Tokuyama Universal Bond II should be brought to room temperature before using. After removing device from refrigerator, allow it to sit for 20 minutes or until it reaches room temperature. Return ESTECCEM II PLUS to the refrigerator after use.
- DO NOT use after the expiration date on the syringe/bottle/package.

PRECAUTIONS FOR ESTECCEM II PLUS

- Use the Cement Mixing Tip or Mini Mixing Tip included in this kit when extruding mixed ESTECCEM II PLUS.
- Use a clean instrument or explorer to apply or remove ESTECCEM II PLUS.
- Remove the mixing-tip from the auto-mixing syringe after each use to avoid unexpected hardening of ESTECCEM II PLUS. Wipe the excess paste from the tip of syringe with using a clean cloth or tissue.

PRECAUTIONS FOR Tokuyama Universal Bond II

- The bottles of Tokuyama Universal Bond II should not be opened immediately after taking out of the refrigerator to avoid splashing the liquid.
- Use the Mixing well with rubber cup or 1-Dimple Disposable Mixing Well well provided when dispensing and mixing Tokuyama Universal Bond II. If Tokuyama Universal Bond II is not dispensed into the rubber mixing, it may be difficult to remove cured Tokuyama Universal Bond II from mixing well. The cured Tokuyama Universal Bond II can be removed by inverting the rubber.
- Some materials and medications (hemostatic materials) inhibit the setting/adhesion of Tokuyama Universal Bond II.
- DO NOT USE products containing:
 - eugenol,
 - aluminum chloride,
 - ferrous sulfate,
 - calcium sulfate,
 - aluminum sulfate,
 - diamine silver fluoride [molecular formula: $\text{Ag}(\text{NH}_3)_2\text{F}$]
- Tokuyama Universal Bond II does not adhere to tooth structure immediately after application of materials containing:
 - hydrogen peroxide (oxidol),
 - sodium hypochlorite
- We advise avoiding the use of these materials on the day of cementation, however these materials can be used for root canal treatment as the inhibition caused by these materials usually evaporates within five days.

- Make sure to hold the bottles vertically when dispensing Tokuyama Universal Bond II A and B. Do not hold the bottle diagonally or horizontally when dispensing: this could cause Tokuyama Universal Bond II to flow back into the nozzle and become contaminated.

8. Application procedure

1. Tooth preparation

Remove contamination from the tooth surface such as plaque, temporary cement residue, oil from materials to test crown fit, oil mist from a hand-piece, saliva, blood and exudates fluids by brushing with flour pumice, ultrasonic scaling, or cleaning the tooth surface with alcohol (not less than 70%). Rinse thoroughly and dry the surface.

- If the adherent surface involves uncut enamel, apply etching agent to the uncut enamel. See section "Selective-enamel-etch technique".
- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.

2. Etching of the tooth structure (optional)

a) Selective-enamel-etch technique

Apply phosphoric acid gel only to the enamel surrounding the margin of prepared cavity and leave phosphoric acid gel in place for 10-15 seconds. Rinse the etched surface thoroughly (at least 15 seconds) with water, and then dry with mild air.

b) Total etch technique

Apply phosphoric acid gel onto the prepared enamel first, and then the dentin. The etchant should be left to react on the enamel and dentin for 10-15 seconds. Rinse thoroughly (at least 15 seconds) with water and dry with mild air or with cotton pellets; do not overdry.

3. Preparation for restoration

Follow the procedure shown below, depending on the material to be restored.

1) Ceramics, other than porcelain, and composite materials

Roughen the interior of the restoration by sandblasting (0.1 to 0.2 MPa), air abrasion, or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Rinse and dry the surface thoroughly.

2) Porcelain

DO NOT roughen the surface. Prepare the interior of the restoration, providing a clean and fresh surface for adhesion. Thoroughly rinse and dry the surface.

3) Metal restorations

Roughen the area by sandblasting (0.3 to 0.5 MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

- When using the sandblaster or air abrasion, use at least 30-50 micron grid of alumina powder.
- If unable to roughen or polish the adhesive surface, etch the area for adhesion with phosphoric acid then thoroughly rinse and dry.
- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol (not less than 70%) soaking. After adjustment is complete, clean the restoration.
 - 1) Gypsum and investment.
 - 2) Silicone oil from fit check material.
 - 3) Oil mist from headpiece.
 - 4) Plaque, tartar, saliva, blood and exudate fluids.
 - 5) Temporary restoration and temporary cement.

Instruction for use for the medical device:

Dental cement "ESTECEM II PLUS KIT"

- If necessary, protect the surface not being bonded with metal separator, paraffin film, Vaseline, etc.

4. Dispensing of Tokuyama Universal Bond II

Allow the Tokuyama Universal Bond II to reach room temperature before using. Open the bottle by removing the cap and dispense one drop of each Tokuyama Universal Bond II A and B into mixing well with rubber cup or disposable mixing well. Mix thoroughly with disposable applicator until the mixed bonding agent turns green.

- Dispense Tokuyama Universal Bond II B IMMEDIATELY after dispensing Tokuyama Universal Bond II A. Tokuyama Universal Bond II A may turn pink due to solvent volatilization if allowed to sit too long. If Bond A turns pink, dispose of it and re-dispense fresh Tokuyama Universal Bond II A and B.
- Complete the application within 1 minute of dispensing when using the mixing well since Tokuyama Universal Bond II contains volatile solvents. After mixing, the color of Tokuyama Universal Bond II changes gradually based on the reaction of the catalyst of Tokuyama Universal Bond II, this is normal and does not indicate any problems with Tokuyama Universal Bond II.
- Complete the application within 3 minutes of dispensing when using the disposable mixing well.
- DO NOT touch the nozzle to the mixing well when dispensing. DO NOT interchange the bottle caps of Tokuyama Universal Bond II A and B. If Tokuyama Universal Bond II A and B come into contact during dispensing, Tokuyama Universal Bond II may cure in the nozzle.
- Wipe off excess of Tokuyama Universal Bond II from the tip of the tip of the nozzle and the inside of the cap before closing.
- Replace the bottle caps of Tokuyama Universal Bond II A and B tightly immediately after dispensing.
- Do not over tighten the bottle caps.

5. Pre-treatment of the restoration with Tokuyama Universal Bond II

1) Application

Apply the mixed Tokuyama Universal Bond II to the surface to be bonded using the disposable applicator.

- Make sure Tokuyama Universal Bond II covers all surfaces, where ESTECM II PLUS will be applied.
- DO NOT over apply. One application of Tokuyama Universal Bond II is sufficient. Too thick layer may compromise the fit of the restoration.
- DO NOT allow saliva, blood, oil, water, etc. to come in contact with the surface to be bonded. If the treated surface becomes contaminated, rinse thoroughly with water, dry it and reapply fresh Tokuyama Universal Bond II.
- DO NOT wait too long after the application of Tokuyama Universal Bond II. Air dry within 30 seconds after application to ensure the appropriate film thickness.

1) Air dry

Using an oil-free air/water syringe, apply mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

Make sure to dry the surface after Tokuyama Universal Bond II is applied. Incomplete drying may result in poor adhesion.

6. Tooth (Adherent surface) pretreatment with Tokuyama Universal Bond II

1) Application

Instruction for use for the medical device:

Dental cement " ESTECM II PLUS KIT"

Apply the mixed Tokuyama Universal Bond II to the surface to be bonded using the disposable applicator.

- Make sure Tokuyama Universal Bond II covers all surfaces, where ESTECM II PLUS will be applied. No extra primers are required even if the adherent surface includes materials other than tooth structure, such as ceramics, porcelain, composite and/or metals.
- Dispensed Tokuyama Universal Bond II can be used for both the restoration and tooth pretreatment if used within 1 minute using the mixing well or within 3 minutes if using the disposable mixing well.
- DO NOT USE dispensed Tokuyama Universal Bond II if the time after dispensing is over 1 minute (mixing well) or over 3 minutes (disposable mixing well), due to the possibility of the poor coating properties or insufficient adhesion. If the time after dispensing exceeds the above limit (1 minute or 3 minutes), dispense again one drop each of Tokuyama Universal Bond II A and B into the mixing well or disposable mixing well and mix thoroughly with the disposable applicator.
- If the Tokuyama Universal Bond II pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or with a new disposable applicator.
- Be careful not to allow excess Tokuyama Universal Bond II to flow into the subgingival. If Tokuyama Universal Bond II flows into the subgingival, rinse and dry the area thoroughly.
- DO NOT wait too long after applying Tokuyama Universal Bond II; this will lead to the possibility of the restoration not seating properly with cementation. Air dry within 30 seconds after application to ensure the appropriate film thickness.

2) Air Dry

Using an oil-free air/water syringe, apply weak air continuously to the Tokuyama Universal Bond II surface until the runny Tokuyama Universal Bond II stays in the same position without any movement. Finish by using mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

- If saliva, blood, or other fluids contaminate the applied Tokuyama Universal Bond II, thoroughly rinse with water, dry and re-apply fresh Tokuyama Universal Bond II.
- DO NOT rinse the applied Tokuyama Universal Bond II with water except in unintentional contamination.
- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.

7. Cementation and final bonding

1) Preparing the syringe and mixing tip

Remove the cap from the auto-mix syringe before each use and squeeze out a small amount of paste A and B, making sure the same amounts of paste are being dispensed through the two outlets of the syringe.

- How to remove the cap: turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves in the syringe. Holding the base of the cap, remove the cap by twisting and pressing downward.
- DO NOT allow ESTECM II PLUS PASTE A to come into contact with PASTE B, as there is a possibility of hardening of the ESTECM II PLUS at the tip of the syringe. Wipe the excess ESTECM II PLUS from the tip of the syringe with a clean cloth or tissue before using a new mixing tip.
- There is a possibility of poor polymerization if equal amounts of ESTECM II PLUS PASTE A and B are not dispensed. Before attaching the mixing tip to the syringe,

Instruction for use for the medical device:

Dental cement "ESTECM II PLUS KIT"

make sure that equal amounts are being dispensed through the two outlets of the syringe.

- How to attach the mixing tip: Align the projections on the adapter as closely as possible with the grooves in the syringe and turn the mixing tip 1/4 of a turn clockwise to lock it in place.
- Remove the mixing tip from the automix syringe after use and replace the cap of the automix syringe.

2) Cementation

a) Cementation of Restoration

Apply a Cement Mixing Tip to the syringe, squeeze the ESTECER II PLUS (A and B) from the syringe and apply the mixed ESTECER II PLUS to the inner surface of the restoration. Place the restoration on the tooth with increased firm pressure. Any excess ESTECER II PLUS at the margin can be removed with the brush tip.

- DO NOT APPLY ESTECER II PLUS directly to the prepared tooth or cavity treated with Tokuyama Universal Bond II to avoid incorrect seating of the restoration.
- Seat the restoration as soon as possible after application of ESTECER II PLUS (within 1 minute after dispensing) to keep the appropriate film thickness, if not there is a possibility that the restoration will not seat properly with cementation.

b) Placing the core or post

Attach a Mini Mixing Tip to the syringe and the nozzle for mini mixing tip to the syringe for application in the root canal. Apply the mixed paste over the entire adherent surface of the core or post, or the entire tooth surface in the cavity. Quickly place the core or post into the cavity, shaking gently to prevent air bubbles from entering the root canals.

- If the paste is applied directly to the cavity, place the core or post into the cavity within 30 seconds after application of ESTECER II PLUS.
- DO NOT USE Lentulo spirals to insert cement into the root canal, as this can accelerate setting excessively.

8. Removal of Excess ESTECER II PLUS

There are two methods to remove excess ESTECER II PLUS.

a) When using the light-cure method

Light-cure the excess ESTECER II PLUS with a dental curing light for 2-4 seconds once the restoration is seated. The gel state excess ESTECER II PLUS can be easily removed.

- Excess light curing will lead to difficult removal of the excess cement.
- Tokuyama Universal Bond II layer may appear fluorescent under the dental curing light irradiation.

b) When using the self-cure method

The gel state of the excess ESTECER II PLUS can be easily removed within 1 to 3.5 minutes after seating.

- Waiting more than 3.5 minutes after seating will lead to difficult removal of the excess cement.
- The setting time of ESTECER II PLUS may change (approximately 20 seconds at the maximum) depending on the oral temperature in each clinical case.

9. Final Adjustment

1) Final curing of ESTECER II PLUS

- a) When using transparent restorative materials such as ceramics or composites, irradiate the light to the ESTECER II PLUS on the seated restoration for 20 seconds or more. When the adherent area is too large to be completed by a single light-cure, multiple cures are recommended.

Instruction for use for the medical device:

Dental cement "ESTECER II PLUS KIT"

- Before using the light-curing unit, make sure that it provides sufficient intensity ($>500 \text{ mW/cm}^2$). Note that using a cracked light guide will lower the intensity.
 - b) In case of non translucent restorative materials such as metal, light-cure along margins for 20 seconds or more and then allow ESTECM II PLUS to set for 8 minutes, with the patient applying normal occlusal pressure to the restoration.
- 2) Finishing
- Check patient's bite, if necessary. Re-polish the restoration with the material of choice.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Specifications are subject to change without notice. When the product specification changes, the instructions and precautions may also change.

9. Functional characteristics.

Table 1. Specification of ESTECM II PLUS

Test item	Specification
Film thickness	Maximum 50 μm
Working time	The material shall be capable of forming a thin layer at 60 seconds after completion of mixing; during its formation there shall be no detectable change in its homogeneity
Setting time	Maximum 10 minutes
Flexural strength	Minimum 50 MPa
Water sorption	Maximum 40 $\mu\text{g/mm}^3$
Solubility	Maximum 7.5 $\mu\text{g/mm}^3$
Color stability	No more than a slight color change shall be observed after 7 days water sorption and after 24 hours of light irradiation
Radio-opacity	The radio-opacity of the set material shall be equal to or greater than that of the same thickness of aluminum
Appearance	Homologous mixed paste

Table 2. Specification of Tokuyama Universal Bond II

Test item	Specification	
Appearance	BOND A	Translucent liquid, yellowish - orangish
	BOND B	Transparent - translucent liquid, bluish - blue-greenish
Working time	Minimum 90 seconds	
Bond strength	To the tooth: To enamel: minimum 12 MPa To dentine: minimum 10 MPa To the prosthesis: To metal: minimum 12 MPa To ceramics (porcelain, ceramics and composite resin): minimum 10 MPa	
Coat property	Form the uniform thickness film	

Instruction for use for the medical device:

Dental cement "ESTECM II PLUS KIT"

10. List of standards that the medical device comply with

EN ISO 10993-1, EN ISO 10993-18, ISO 7405, EN ISO 13485, EN ISO 14971, EN 1041, EN 1641, EN ISO 15223-1, ISO/TS 11405, ISO 4049

The latest versions of reference standards should be used: the organization should maintain and update the list of reference standards as a record of its own management system in compliance with ISO 13485.

11. Interpretation of symbols used in the labeling of consumer and transport packaging

Table 3. Symbols and description.

Symbol	Description	Symbol	Description
	Manufacturer		Lot number
	Catalog number		Expiration date
	Possibility for the occurrence of skin allergic reaction		Caution
	Consult instructions for use		Highly flammable liquid and vapour

12. Requirements for the maintenance and repair of a medical device

Devices do not need maintenance and repair.

13. Sterilization, cleaning and disinfection

The medical device is supplied non-sterile and is not intended to be sterilized before use.

Disposable mixing well, disposable applicator, nozzle for mini mixing tip are single-use accessories and should not be reused according to the instructions for use.

After use, mixing well with rubber cup should be rinsed with an antimicrobial liquid, such as alcohol (not less than 70%). Alcohol is effective in eliminating most bacteria and fungi.

14. Conditions of transportation, storage and operation

Transportation of devices packed in shipping containers should be carried out by all types of closed vehicles in compliance with the precautions indicated on the shipping container and in accordance with the rules for the carriage of goods in force for this type of transport.

Kit shipping conditions:

- Temperature – from 0 to +10 ° C;
- Relative humidity: 10 – 100 %;
- Atmospheric pressure: 800 ~ 1,060 hPa.

Instruction for use for the medical device:

Dental cement “ ESTECM II PLUS KIT”

Store in the refrigerator. Keep away from heat, direct sunlight, sparks and open flames.

Kit storage conditions:

- Temperature – from 0 to +10 °C;
- Relative humidity: 10 –70%;
- Atmospheric pressure: 900 ~ 1,060 hPa.

Indoor conditions when using a medical device

- Temperature from +18 °C to +30 °C

15. Shelf life

The shelf life of the device ESTECM II PLUS PASTE (PASTE A and PASTE B) is 2 years at a temperature of 0-10 °C.

The shelf life of Tokuyama Universal Bond II is 2 years at the temperature 0 °C to 25 °C.

16. Procedure for the disposal and destruction of a medical device

The medical device Dental cement “ESTECM II PLUS KIT” depending on the degree of their epidemiological, toxicological and radiation hazard, is classified as epidemiologically safe waste, similar in composition to solid domestic waste (SDW). However, it is recommended to dispose of them separately as industrial waste through specialized organizations licensed for this type of activity.

The following should be done for the safe disposal of product residues: remove unused ESTECM II PLUS cement (PASTE A and PASTE B) from the syringe, mix and allow to harden; absorb the adhesive Tokuyama Universal Bond II with a neutral material (gauze, cotton wool).

The devices should be collected and handed over to a specially licensed medical waste disposal organization. Waste transportation from institutions to the place of disposal is carried out by specialized companies.

Collection of non-hazardous waste is carried out in reusable containers or disposable bags (with the exception of bags in the color accepted for hazardous waste). Disposable bags are placed inside reusable containers. Containers for collecting waste should be marked and installed on a special site (indoor). Reusable containers after emptying should be washed and disinfected. The procedure for washing and disinfecting reusable containers is determined in accordance with the waste management scheme in each specific organization.

Medical devices should be disposed of in accordance with local medical procedures and local, state, and federal regulations. The choice of methods for the safe disposal and destruction of products depends on the capacity and profile of the medical organization, the availability of facilities for the disinfection/disposal of waste, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

Instruction for use for the medical device:

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Registration No. 566 of 2021

Certificate of Acknowledgment of Notary

On this 21st day of December 2021, Tomoko MIFUNE personally appeared before me, KATO Tomohiro, a notary in and for Tokyo Legal Affairs Bureau, with satisfactory evidence of her identification and of her being the agent of Hideki KAZAMA, President of Tokuyama Dental Corporation, in accordance with the Japanese Notary Act, stated that the principal, said Hideki KAZAMA, had acknowledged to have affixed his signature and seal to the attached document.

In witness whereof, I set my hand and seal.

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