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Read instructions
before operation.

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5255802105 Clinical Procedure
direct and indirect restorations that can be used with Self-etch, Selective-etch and Total-etch techniques. As a universal adhesive, Tokuyama Universal Bond II has been designed to be fully compatible with light-cured, self-cured and dual-cured composite materials such as glass-ionomer cements, acrylic resin and composite resin to indirect restorative materials such as glass-ceramics (porcelain), oxide-ceramics (zirconia and alumina), metals (precious and non-precious) and resin materials including inorganic filler (Bisphenol A di(2-hydroxypropoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), 2-Hydroxyethyl methacrylate (HEMA), MTU-6 (thiouracil monomer), Silane coupling agent, Peroxide, Borate catalyst, Acetone, Ethanol and Purified water.

INDICATIONS

- Direct anterior and posterior restorations with light-curing, dual-curing, and self-curing
- Note: Certain substances may react with the examination gloves, remove and dispose of the gloves when combined with light-cure, dual-cure or self-cure cements
- Bonding of denture resin to metal base, clasp or attachment
 - Repair of denture with metal base, clasp or attachment
 - Bonding of opaque resin to a metal base in the fabrication of resin-faced crowns

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«Tokuyama Dental Corporation»