mindray

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Toxoplasma gondii IgG (CLIA)

Order Information

Catalog No.	Package Size
105-012524-00	2×50 tests (calibrators not included)
105-012525-00	2×100 tests (calibrators not included)
105-012527-00	2×50 tests (calibrators included)
105-012528-00	2×100 tests (calibrators included)

Intended Use

The CL-series Toxoplasma gondii IgG assay is a Chemiluminescent Immunoassay (CLIA) for the quantitative detection of Toxoplasma gondii specific IgG in human serum and plasma. The results of this assay can be used as an aid in the diagnosis of Toxoplasma gondii infection and in the assessment of immune status of individuals including pregnant women.

Summary

Toxoplasmosis is a world spread zoonotic infectious caused by Toxoplasma gondii. The infection can be transmitted through oral intake of food containing Toxoplasma gondii, direct contact with soil or water contaminated by oocysts, or vertical transmission of fetus through placenta^[1].

There are no obvious clinical symptoms and signs in healthy individuals infected with Toxoplasma gondii. However, in patients with immune deficiency or suppression, this parasite may invade various organs and lead to serious diseases, such as Toxoplasmosis encephalopathy, ophthalmopathy and liver damage.

If the infection occurs in pregnancy, Toxoplasma gondii can infect the fetus through placenta. In general, the later the pregnant period when pregnant women are infected, the higher the fetal infection rate, and the lighter clinical symptoms. In the first trimester, the fetal infection rate is 10-20%, which often causes spontaneous abortion. In the second trimester, the fetal infection rate increases and most of the fetuses suffer from stillbirth, premature delivery or severe damage of central nervous system. In the third trimester, the fetal infection rate

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Intended Use

The CL-series *Toxoplasma gondii* IgG assay is a Chemiluminescent Immunoassay (CLIA) for the quantitative detection of *Toxoplasma gondii* specific IgG in human serum and plasma. The results of this assay can be used as an aid in the diagnosis of *Toxoplasma gondii* infection and in the assessment of immune status of individuals including pregnant women.

Summary

Toxoplasmosis is a world spread zoonotic infectious caused by *Toxoplasma gondii*. The infection can be transmitted through oral intake of food containing *Toxoplasma gondii*, direct contact with soil or water contaminated by oocysts, or vertical transmission of fetus through placenta^[1].

There are no obvious clinical symptoms and signs in healthy individuals infected with *Toxoplasma gondii*. However, in patients with immune deficiency or suppression, this parasite may invade various organs and lead to serious diseases, such as Toxoplasmosis encephalopathy, ophthalmopathy and liver damage.

If the infection occurs in pregnancy, *Toxoplasma gondii* can infect the fetus through placenta. In general, the later the pregnant period when pregnant women are infected, the higher the fetal infection rate, and the lighter clinical symptoms. In the first trimester, the fetal infection rate is 10-20%, which often causes spontaneous abortion. In the second trimester, the fetal infection rate increases and most of the fetuses suffer from stillbirth, premature delivery or severe damage of central nervous system. In the third trimester, the fetal infection rate

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