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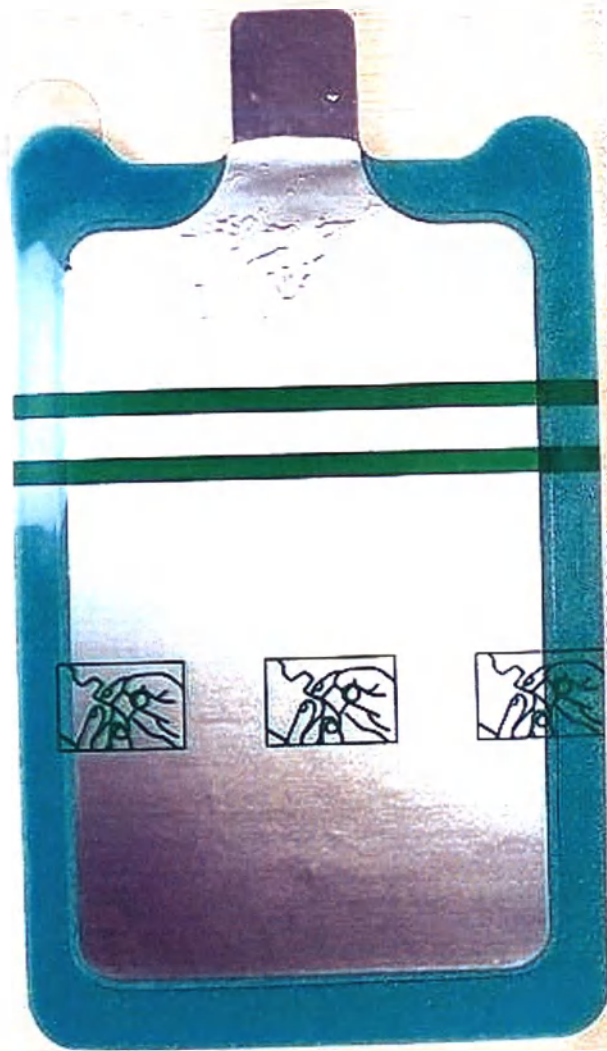
# DISPERSIVE ELECTRODE ELECTROSURGICAL PADS

Model No.: GBS-Dm1030a

Adult Solid

Adult (>13.6kg)

GBS BAISHENG MEDICAL CO., LTD.

## DISPERSIVE ELECTRODE

### Indications for Use

The device is used to match with the high frequency electrosurgical unit for which it is intended according to international standards. Its output power should not exceed 400 ± 20%W and output frequency cannot be less than 300kHz.

### Contraindications

1. Do not fold the electrosurgical pad when you use it. Especially the tongue of the clamp should not be folded over the angle of 90°, in case the metal film will be broken.
2. Make sure to use it with original matching clamps and cables. Otherwise, it may cause loose contact because of the mismatching.
3. Do not put the cable and the clamp on the patient body or under combustibles.
4. This device must be used within 24 hours after you open the package. Stick it on the patient body as soon as possible after you.
5. Reuse of the device is prohibited.

### Instruction for safe use

1. Clean the patient contact area.
2. Tear the package, take out the electrosurgical pad.
3. Select a smooth, well-vascularized, muscular area as the contact site that allows electrosurgical pad to skin in full contact. Avoid place over bony prominences, metal prostheses or scar tissue.
4. The contact area must be clean, dry and free of hair. Remove the hair at contact area.
5. Hold the tongue with the plate clamp, the bare metal foot should be all placed in the clamp, which can stand the pulling force. Contact the other end of the cable with output hole on high frequency equipment.

If instructions are not strictly followed, there is a risk for or even the occurrence of a shock.

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21.12.2021

20.12.2024

BAISHENG MEDICAL CO., LTD.  
Address: No. 11 Fusheng Road, Xihu District, Jiangmen City, Guangdong, China

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EC REP Polina GmbH