"APPROVED"

Ontex BVBA

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<http://legalweb.diplomatie.be>

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Exploitation documentation

DIRECTIONS FOR USE OF THE MEDICAL PRODUCT

Medical absorbing products «iD»: adult pants and adult diapers in the following dimensions (see Paragraph 1 on page 2)

I, the undersigned, Henri Van den Bossche, notary holding residence at Buggenhout, confirm that the signature on this document is the signature of Mr. Bart Waterschoot and is well known by me.

This confirmation of the signature only concerns its authenticity and does not imply a certification by the notary of the capacity or the powers of the aforementioned signatory.

Done on the 19th of January 2017 at Buggenhout.

«Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium
2016

Ontex bvba
Registered Office
Genthof 5
B-9255 Buggenhout

Headquarters Ontex bvba, Korte Keppestraat 21, B-9320 Aalst (Erembodegem)
T +32 (0)53 333 600 F +32 (0)53 790 342 www.ontexglobal.com
RPR Dendermonde BTW BE 0419.457.296 KBC IBAN BE65 7310 1366 9796 BIC KRED8EBB

1. Name of the medical product (hereinafter "MP")

Medical absorbing products «iD» in the following dimensions (hereinafter "Products"):

1. Adult pants «iD» XS (Extra Small)
2. Adult pants «iD» S (Small)
3. Adult pants «iD» M (Medium)
4. Adult pants «iD» L (Large)
5. Adult pants «iD» XL (Extra Large)
6. Adult diapers «iD» XS (Extra Small)
7. Adult diapers «iD» S (Small)
8. Adult diapers «iD» M (Medium)
9. Adult diapers «iD» L (Large)
10. Adult diapers «iD» XL (Extra Large)

2. Information on the medical product

Name of the operating company: Ontex BVBA

Registered address of the company: «Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium.

Name of the manufacturing companies:

1. «Ontex BVBA», Belgium.
2. «Ontex CZ s.r.o.», Czech Republic.
3. «Ontex Health Care France SA», France.
4. «Ontex BVBA Eeklo», Belgium.
5. «Ontex Tüketim Ürünleri Sanayi Ve Ticaret Anonim Sirketi», Turkey.
6. LLC «Ontex RU», Russia.
7. LLC «Ontex RU», Russia.

Registered address of the company:

1. Genthof 5, 9255 Buggenhout, Belgium.
2. Turnov, Vesecko cp. 491, PSC 51101, Czech Republic.
3. ZA Artoipole Monchy-le-Preux 62118 Biache-Saint-Vaast, France.
4. Korte Moeie 53, 9900 Eeklo, Belgium.
5. Selimpasa Mah. 6093 Sokak No. 78 Silivri, Istanbul, Turkey.
6. 142438, Moscow Region, Noginsk district, 58 km of Highway Moscow - N. Novgorod, Industrial site №1, Russia.
7. 142400, Moscow Region, Noginsk district, 56km+900m of Highway M-7 «Volga», h. 1, bld. 3, Russia.

Company profile of the manufacturer:



Production, sale and distribution of hygienic and medical products both under own brands and retail brands.

Authorised representative of the manufacturer: LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.


3. Description of the product

Medical absorbing products «iD» of the following versions (see Paragraph 1 on page 2) – single use.



3.1 Description of the product structure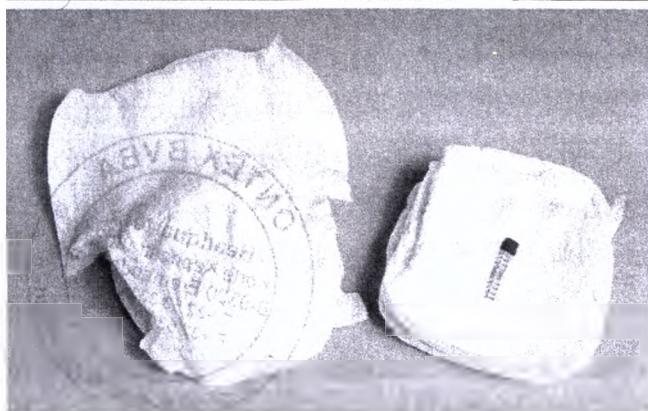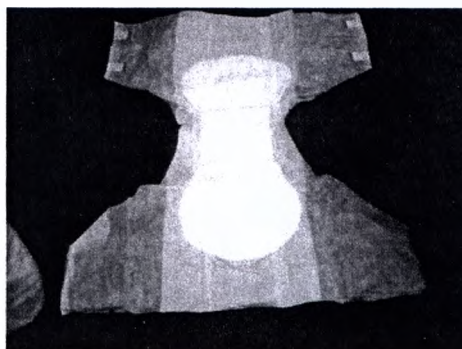

■ Example of the adult pants

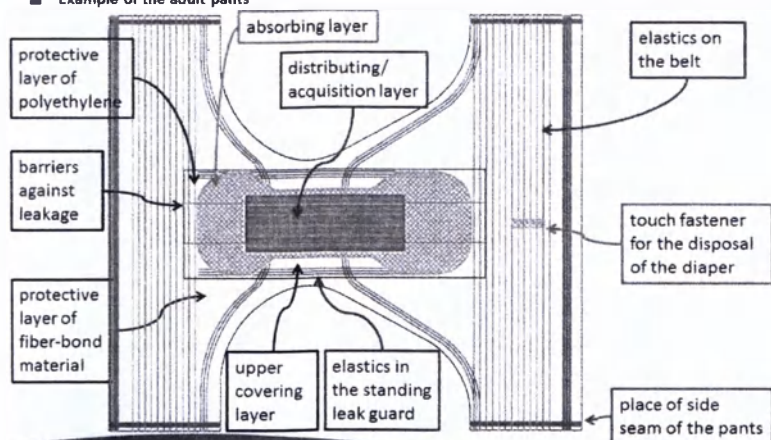■ Example of the adult diapers

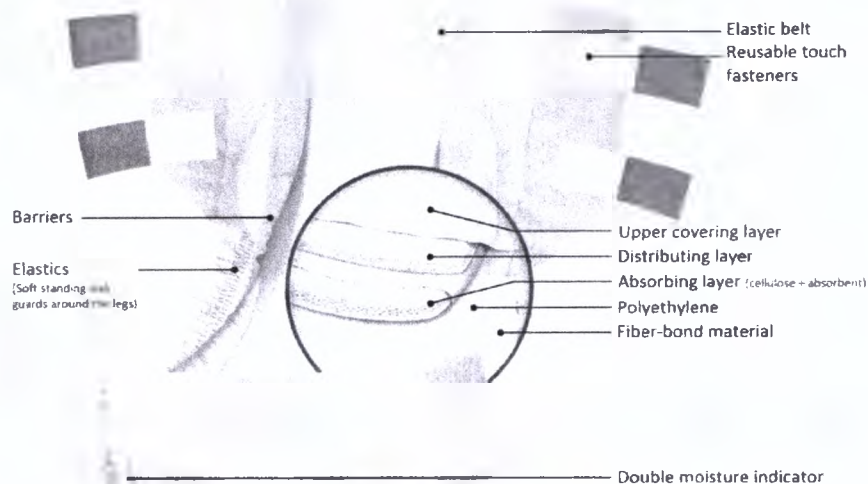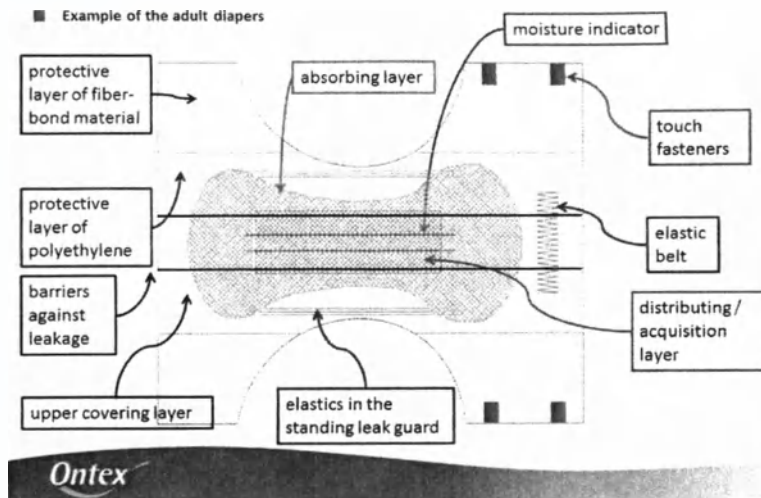Products contain the following layers:

- The upper covering layer;
 - The distribution/Acquisition layer;
 - The absorbing layer;
 - The protective layer or backsheet
 - The touch fastener from the both sides of the Product;
 - The standing leak guards (barriers and side arc-like frills) fixed with the elastics.
- The number of the elastics in the barrier should be more than one;
- Elastic belt.

3.2 Designation of the product given by the producer

Products are intended for the patients with incontinence.

Products protect the clothing and pants from soaking and contamination by bioeffluents.

3.3 Indication for use of the products

Products are used in case of incontinence of light, moderate and severe form.

3.4 Contra indications for use of the products:

- dermatopathy of externalia (dermatitis, eczema)
- idiosyncrasy for the materials of the products, expressed hypersensitivity reactions

3.5 Side effects of the products

Side effects of the product were not identified by the manufacturer.

3.6 Scope of application of the product

Products are used at general healthcare institutions as well as domiciliary (in everyday life).

3.7 Use instructions of the product

Before application of the product one should:

- wash hands before and after the change of the product;
- change the product every 3-4 hours or more often, if the product soaks quicker;
- touch fasteners should not disturb the belly and legs, that is normal blood circulation should not be disturbed;
- do not tear the used product, don't wash it, don't throw to the sewage line;
- do not cut or change the form of the product because it may influence its effectiveness;
- wash genitals or clean the intimate zone with the help of special napkins for intimate hygiene at each change of the product.

3.8 Methods of product application:

- Unwrap the product;
- Turn the lying patient on the side, put the narrower part of the diaper between the patient's legs. Then lay the patient on the back in such way that the rare wider part of the diaper was under the clunis.
- Fix the product with the help of touch fasteners from the both sides of the diaper or wear like the underwear (as pants);
- Only intended for single use;
- Wrap the used product into the paper napkin and throw to the dustbin.

3.9 Completeness

Products are supplied completed in accordance with the engineering and technical documents and order requirements. Completeness of the product delivery should comply with the information specified in the Table No 1.

Table No 1. Completeness of the delivery

No	Name	Number (pcs.)
1.	Adult diapers «iD» or adult pants «iD»	8, 10, 12, 14, 16, 18, 20, 22, 24, 26, 28, 30

4. Technical characteristics of the medical product

4.1 Size of the products


Size of the products (diapers and pants) should comply with the sizes specified in the Table No 2 and Table No 3.

Table No 2. Sizes of diapers for adults

Name of the size	Sizes in accordance with the identifying signs				
	Extra Small (XS)	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	40-70	50-95	70-130	100-160	120-175
Length/Width of the diaper *, mm	610X390	685X490	805X660	970X790	950X805
Length of the absorbing layer *, mm	510	550	605	720	720
Width of the absorbing layer in the front part*, mm	225	300	290	280	305
Width of the absorbing layer in the rare part *, mm	250	315	290	280	325
Width of the absorbing layer in the middle part *, mm	150	165	160	170	190
Total width of the touch fasteners*, mm	60	60	60	60	70
Distance between the upper and lower touch fastener*, mm	78	130	118	130	148
Width of the inner frills*, mm	35	40	35	40	45
Length/width of the elastic belt*, cm	19X4	19X4	19X4	19X4	19X4

*acceptable variations on target indicators specified in the table are of $\pm 10\%$

Table No 3. Sizes of pants for adults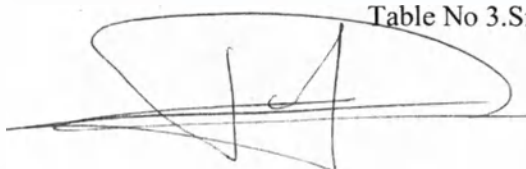

Name of the size of the products	Sizes					Width of the inner frills*
	Waist line*, cm	Length/Width of the pants*, mm	Absorbing layer (length/width)*, mm	Width of absorbing layer in the middle*	Width of absorbing layer in front and rare parts*	
XS	50-70	630X175	510X100	150 mm	225 mm	35 mm
S	60-90	800X582	480X190	165 mm	300 mm	40 mm
M	80-120	830X728	480X190	160 mm	290 mm	35 mm
L	100-145	900X811	480X190	170 mm	280 mm	40 mm
XL	130-170	900x925	400x190	190 mm	305 mm	45 mm


*acceptable variations on target indicators specified in the table are of +-10%

4.2 Materials of the products

Materials specified in the Table No 4 are used for manufacturing of the products.

Table No 4. Materials of the product

Layer of the products	Materials of the products
Upper covering layer	Hydrophilic fiber-bonded material of polypropylene under trademark “Regent” produced by LLC «First Chemical Corporation», Russia
Distributing layer	Fiber-bonded material of polypropylene under trademarks «DRY WEB TDL2», «DRY WEB T28», «DRY WEB TDL3 G», «DRY WEB T19», «DRY WEB T12G» produced by «TWE Meulebeke BVBA», Belgium
Absorbing layer	Cellulose bleached without chlorine under trademark «GP Cellulose» produced by «GP Cellulose LLP», USA Polymer superabsorbent under trademark «LG Chem GS-3070HD» produced «LG Chem, Ltd», the Republic of Korea
Protective layer	Polyethylene film under trademark «Hyfol PP Plus», produced by «RKW SE», Germany. Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol Ros», Russia.
Touch fastener	Self-adhesive plastic band (polypropylene) under trademark «AVERY DENNISON F5 2400 Wave C» produced by «AVERY DENNISON Belgie b.v.b.a», Belgium
Standing leak guard	Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol

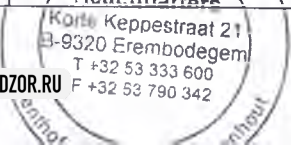	Ros», Russia.
Elastic belt	Plastic band (polystirol) under trademark "3M EBL Soft", produced by "3M Deutschland GmbH", Germany
Elastics	Elastomer polyurethane thread under trademark "Spandex" produced by «HYOSUNG SPANDEX (JIAXING) CO.LTD», the Republic of Korea.
Moisture indicator	Hot melt glue (moisture indicator) under trademark «Swift Melt 1525» produced by «H.B. Fuller Deutschland GmbH», Germany
Glue	Hot-melt adhesive: Hot melt glue «Bostik» on the base of styrole copolymer under trademarks «H2465-04», «H4256» produced by «Bostik Nederland B.V.», Netherlands.

Layers (seams) of the products should be fastened tightly with the help of thermal processing or hot-melt glue, or other method providing the tightness of the glued layers and seams of the diapers. Seams should be smooth.

4.3 Weight and absorptive capacity of the product

Weight and functional performance of the product (diapers or pants) are specified in the Table No 5,6

Table No 5. Weight and absorptive capacity of the products

Name of the size of the products	Specification of the diapers for adults			Specification of the pants for adults		
	Total weight, g	Absorptive capacity*, g, not less than	Minimal weight of the absorbent, g	Total weight, g	Absorptive capacity*, g, not less than	Minimal weight of the absorbent, g
XS	50	1000	5	70.6	1300	5
S	79.5	1300	5	66	1320	5
M	86	1450	10	70.5	1450	10
L	96	1800	8	75	1500	8
XL	134	2000	6	80.5	1700	6

*acceptable variations on target indicators specified in the table are of $\pm 10\%$

Table No 6. Functional performance of the products

Name of the size of the products	Specification of the diapers for adults			Specification of the pants for adults		
	Rewet, g	Absorptive capacity after	Absorption time, cm ³ /s	Rewet, g	Absorptive capacity after	Absorption time, cm ³ /s

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*acceptable variations on target indicators specified in the table are of $\pm 10\%$

5. Classification of the medical product

In accordance with the degree of potential risk of the application in medical purposes medical product "Adult diapers and adult pants «iD» in the following versions (see Paragraph 1 on page 2)" belong to the 1 class according to Annex 2 of the Order No 4n "On approval of nomenclature classification of the medical products" as of June 06, 2012 of the Ministry of Health of the Russian Federation with subsequent amendments.


Type of contact with human body: direct contact

6. Marking of the products

Ready-made products are marked on the package or on the labels; place of marking application is specified in the design documents.

Marking of the separate tare contains:

- The name of the manufacturing company and (or) its trademark;
- The address of the manufacturing company, including the country of origin;
- The name of the products;
- The size of the product;
- The number of products in the package;
- The date of manufacturing of the products;
- The expiry date;
- The usage instruction;
- The designation of the product;
- Contra indications for use of the products;
- Storage regulations;
- The information on disposal of the products;
- The bar code of the product;
- The information about registration;
- The information on the single use of the products;
- The name of the importing company on the territory of the Russian Federation;
- The address of the importing company on the territory of the Russian Federation;

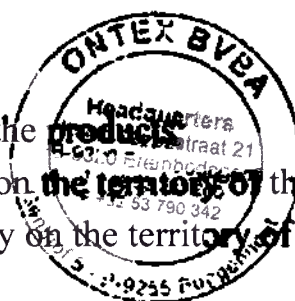It is allowed to mark other data including information about consumer properties and use of the products.

Data are printed or embossed that ensure their readability and preservation (stability) during the storage, transportation, handling and intended usage of the products.

Marking of the transportation tare with the following information:

- the name of the products;
- the article of the products;
- the number of the products in the transportation tare;
- the name and the address of the manufacturing company;
- the name and the address of the company authorized by the manufacturer, of the importer;
- the bar code of the transportation tare;
- the date of manufacturing of the products (month, year);
- the expiry date;
- conditions of storage, transportation, recycling possibility;
- icons and symbols according to the legislation requirements of the consignee country.

7. Packaging

Several pieces of products are packed into the bags made of polyethylene which protects the products in transportation and storage. Seams in the bags of polyethylene should be welded.

Products of one group, structure, linear sizes, technical and decorative version are packed in one bag.

Package should provide the protection of the products from the contamination, mechanical damage, weather impact in transportation and storage as well as the most complete usage of vehicle capacity (tonnage) and easy performance of loading-unloading operations.

Boxes of corrugated cardboard and other packing materials ensuring the protection of the products may be used as transportation tare.

It is allowed to use the other packing materials including those that are manufactured by the manufacturing company which provide the necessary durability and protection of the products in transportation and storage.

8. Transportation and storage

Products are transported by all kinds of vehicles in accordance with the cargo transportation rules operating on the transport of given type. Material is not hazardous at any kind of transportation.

Ready-made products should be stored in sheltered storage facilities. Storage conditions should exclude the impact of moisture and aggressive environment on the products.

To preserve the quality product should be preserved in original package at the room temperature from +5 °C to +25 °C and relative humidity no more than 70 % in childproof and moisture protected sites. Don't expose the products to direct sunlight.

9. Safety requirements

Work with products does not require special safety measures.

Structure of the products does not contain materials which are hazardous for the health of the person during the operation.

Products are explosive safe, inflammable when exposed to the open fire without explosions.

In case of fire materials of the products and package should be extinguished with foam, extinguishing powder, carbon dioxide, water, sand.

10. Requirements for environment protection

Requirements for environment protection during the application of the product

Products do not release toxic substances to the environment and do not impact (at direct contact) the health of the person and the environment during the transportation, storage and operation. Products are explosive safe, inflammable when exposed to the open fire without explosions. Special environment protective measures from harmful influence are not required during the operation.

Requirements for environment protection and disposal of the products

The main possible harmful impact on the environment is the contamination of the atmosphere, soils and waters of the inhabited areas as a result of:

- unorganized waste landfilling on the territory of manufacturing company or outside its territory;
- random landfilling in unspecified places.

Products and materials used for their manufacture do not impose risk for the health of people and for the environment both during the operation and after its completion.

11. Disposal

It is allowed to dispose waste and materials during the manufacture by contract with company which has the required licence. (To dispose with the help of the method approved by the local legislation).

The product can be disposed as household waste after its usage. It is not allowed to throw the used products into the toilet.

12. Warranty liabilities

Manufacturer assures the compliance of the products with the requirements of the product specifications and guarantees the replacement of the products in case of nonconformity.

Warranty period for the product is 5 years from the moment of manufacture/packing. Date of manufacture is specified with news ticker on the side of the package. After the expiration date this medical product should be disposed.

13. Information about reclamations

If defects of the products appear during warranty period as well as at primary acceptance of the products, the owner of the product should send a reclamation to the address of the selling organization – request for the replacement according to the form No 1.

Form No 1.

The address where the representative of the company carrying out warranty liabilities should arrive, phone number;		
Date	Short description of the defect	Date of the sending of the reclamation

Send the reclamations to the address:

LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.
Postal address: LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.
Tel.: 8 495 646 92 21; website: www.ontex.ru

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