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[Handwritten signature]

КОПИЯ
COOK MEDICAL EUROPE LTD.

O'Halloran Road,
National Technology Park
Limerick, Ireland.

Company Registration No. 492272
VAT Registration No. 577923416

INSTRUCTION FOR USE

Savary-Gilliard Dilator Sets for dilating strictures in the gastrointestinal tract
SGD-100-1; SGD-100-2; SGD-70-1; SGD-70-2

INTENDED USE

This device is used over a pre-positioned Savary-Gilliard wire guide for dilation of esophageal strictures.

CONSTRUCTION

Set SGD-100-1 includes Savary-Gilliard dilators, diameter 5; 7; 9; 11; 12.8; 14; 15 mm, length 100 cm, 7 pieces, Savary-Gilliard Wire Guide, length 200 cm, Washing adapter.


Set SGD-100-2 includes Savary-Gilliard dilators, diameter 5; 6; 7; 8; 9; 10; 11; 12; 12.8; 14; 15; 16; 17; 18; 19; 20 mm, length 100 cm, 16 pieces, Savary-Gilliard Wire Guide, length 200 cm, Washing adapter.

Set SGD-70-1 includes Savary-Gilliard dilators, diameter 5; 7; 9; 11; 12.8; 14; 15 mm, length 100 cm, 7 pieces, Savary-Gilliard Wire Guide, length 200 cm, Washing adapter.

Set SGD-70-2 includes Savary-Gilliard dilators, diameter 5; 6; 7; 8; 9; 10; 11; 12; 12.8; 14; 15; 16; 17; 18; 19; 20 mm, length 100 cm, 16 pieces, Savary-Gilliard Wire Guide, length 200 cm, Washing adapter.

Product	Construction
Savary-Gilliard Dilator Sets	

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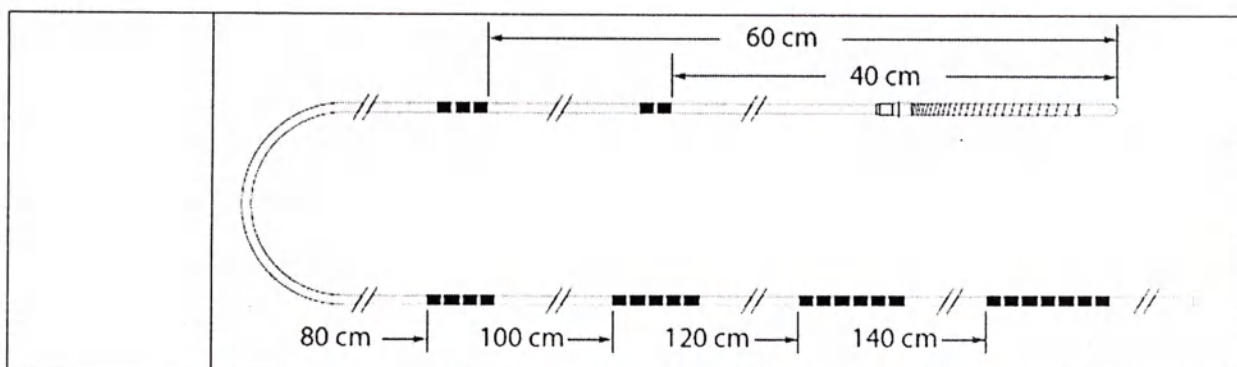NOTES

This device is supplied non-sterile and is reusable if the device integrity is intact.

Do not use this device for any purpose other than stated intended use.

If the package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.

Store in a dry location, away from temperature extremes.

Use of this device restricted to a trained healthcare professional.

CONTRAINDICATIONS

Contraindications include those specific to upper GI endoscopy.

Contraindications to dilation include, but are not limited to: uncooperative patient; asymptomatic strictures; inability to advance the dilator through the strictured area; coagulopathy; known or suspected perforation; severe inflammation or scarring near the dilation site, recent myocardial infarction, active ulcer and severe cervical arthritis.

POTENTIAL COMPLICATIONS

Potential complications associated with upper gastrointestinal endoscopy and esophageal dilation include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

PRECAUTIONS

Reusability of a device depends, in large part, on care of device by user. Factors involved in prolonging life of this device include, but are not limited to: thorough cleaning following instructions included in this booklet.

TECHNICAL PARAMETERS

Product Code/ Model	Dilator Diameter (MM) / (FR)	Dilator Length (CM)	Wire Guide Length (CM)
------------------------	---------------------------------	------------------------	---------------------------

Product Code/ Model	Dilator Diameter (MM) / (FR)	Dilator Length (CM)	Wire Guide Length (CM)
SGD-100-1	5/15, 7/21, 9/27, 11/33, 12.8/38, 14/42, 15/45	100	200
SGD-100-2	5/15, 6/18, 7/21, 8/24, 9/27, 10/30, 11/33, 12/36, 12.8/38, 14/42, 15/45, 16/48, 17/51, 18/54, 19/57, 20/60,	100	200
SGD-70-1	5/15, 7/21, 9/27, 11/33, 12.8/38, 14/42, 15/45	70	200
SGD-70-2	5/15, 6/18, 7/21, 8/24, 9/27, 10/30, 11/33, 12/36, 12.8/38, 14/42, 15/45, 16/48, 17/51, 18/54, 19/57, 20/60,	70	200

INTENDED USE/INDICATION FOR USE

Savary-Gilliard Dilators and Wire Guides are used for dilation of gastrointestinal strictures. Wire guides are used to introduce the Savary-Gilliard Dilator.

Savary cleaning brush is used to clean the dilator after procedure.

USAGE CONDITIONS

Endovascular introducers are used in the patient care medical institutions.

CONTRAINDICATIONS

Contraindications include those specific to upper GI endoscopy. Those to dilation include, but are not limited to: uncooperative patient; asymptomatic strictures, inability to advance the dilator through strictured area; coagulopathy; known or suspected perforation; severe inflammation or scarring near dilation site, recent myocardial infarction, active ulcer and severe cervical arthritis.

INSTRUCTIONS FOR USE

Savary-Gilliard Dilators and Wire Guides are used for dilation of gastrointestinal strictures. Proceed with esophageal dilation. Upon completion, remove dilator and clean dilator following cleaning instructions immediately after use of device per IFU guidelines.*

INSTRUCTIONS FOR USE Savary-Gilliard Dilator

1. Perform screening endoscopy to identify strictured area.
2. Introduce a Savary-Gilliard Wire Guide, floppy tip first, into accessory channel of endoscope and advance until it is endoscopically visualized well beyond tip of scope.
3. When wire guide is in position well beyond strictured area, slowly begin to withdraw endoscope in 5-10 cm increments while simultaneously advancing wire guide in 5-10 cm increments to ensure wire guide remains in position. **Caution:** Continuous fluoroscopic monitoring of wire guide is essential in order to ensure it remains in proper position.
4. When endoscope is removed completely, fluoroscopically confirm that wire guide has not been displaced. **Note:** Markings on distal portion of wire guide may also be utilized to determine wire guide position. Markings occur in 20 cm increments from 40 to 140 cm. (See fig. 1)
5. Generously lubricate dilator and advance it over pre-positioned wire guide to strictured area.
6. Proceed with esophageal dilation. **Note:** Successive dilations can be accomplished by repeating steps 5-7.
7. Upon completion of esophageal dilation, remove dilator and wire guide from patient.
8. Clean and prepare dilator for future use following cleaning instructions below. During cleaning, inspect integrity and function of device to determine advisability of reuse. If kinks, bends or breaks exist, do not use. Dispose of device per institutional guidelines for biohazardous medical waste.

CLEANING INSTRUCTIONS

Caution: Savary-Gilliard Dilators cannot be autoclaved or gas sterilized. Savary-Gilliard Dilator case should be washed thoroughly with soap and water then wiped dry with a lint-free cloth or air-dried.

Note: Betadine is not recommended for use on Savary-Gilliard Dilators.

1. Immediately after use, soak dilator in a solution of warm water and enzymatic cleaning solution. **Note:** Refer to enzymatic cleaning solution instructions for recommended duration of cleaning.
2. Scrub external surfaces of dilator with warm water and enzymatic solution. Using a Savary Cleaning Brush (sold separately), brush wire guide lumen of dilator.
3. Rinse dilator thoroughly with clean, running water. Attach suction to tapered end of dilator or utilize flushing adapter to suction water through wire guide lumen.
4. Wipe external portions of dilator dry with a lint-free cloth. Use medical grade (oil-free) forced air to dry wire guide lumen of dilator.
5. Proceed with high-level disinfection of dilator.

HIGH LEVEL DISINFECTION/STERILIZATION

A list of disinfectant solutions for use with this device is as follows:

- Glutaraldehyde
- Ortho-phthalaldehyde (OPA)
- Peracetic acid

- Hydrogen peroxide

1. Completely immerse dilator in a disinfectant. **Note:** Refer to disinfectant solution instructions for duration of immersion required for high level disinfection or sterilization. If difficulty is encountered with solution penetrating lumen, we recommend attaching suction to tapered end of dilator or utilize a flushing adaptor. **Note:** Discoloration of dilator may occur following use of these solutions. This is not harmful to device.
2. Rinse dilator thoroughly with clean, running water, utilizing above method for thoroughly rinsing lumen. Attach suction to tapered end of dilator or utilize flushing adapter to suction water through wire guide lumen. **Note:** Proper rinsing with clean water and complete drying of dilator is necessary following use of these solutions.
3. Dry external portions of dilator completely with a lint-free cloth. Use medical grade (oil-free) forced air to dry wire guide lumen.

INSTRUCTIONS FOR USE Savary-Gilliard Wire Guide

1. Perform screening endoscopy and identify strictured area.
2. Introduce device, floppy tip first, into accessory channel of endoscope and advance until it is endoscopically visualized well beyond tip of scope. Example: Esophageal strictures in an unoperated GI tract would require wire guide placement in distal body or antrum of stomach.
3. When wire guide is in position well beyond strictured area, slowly begin to withdraw endoscope in 5-10 cm increments while simultaneously advancing wire guide in 5-10 cm increments to ensure wire guide remains in position. Caution: Continuous fluoroscopic monitoring of wire guide is essential in order to ensure it remains in proper position.
4. When endoscope is removed completely, fluoroscopically confirm that wire guide has not been displaced. Note: Markings on distal portion of wire guide may also be utilized to determine wire guide position. Markings occur in 20 cm increments from 40 to 140 cm (See fig. 1)
5. Generously lubricate dilator with water-soluble lubricant and advance it, under fluoroscopic monitoring, over pre-positioned wire guide through strictured area. Note: When multiple dilator sizes are used, repeat this step for each.
6. Upon completion of procedure, remove dilator and wire guide from patient.
7. Clean and prepare wire guide for future use following cleaning instructions included in this booklet. During cleaning, inspect integrity and function of device to determine advisability of reuse. If kinks, bends or breaks exist, do not use. Dispose of device per institutional guidelines for biohazardous medical waste.

CLEANING INSTRUCTIONS

1. Immediately after use, soak wire guide in a solution of warm water and enzymatic cleaning solution. **Note:** Refer to enzymatic cleaning solution instructions for recommended duration of cleaning.
2. Wipe surface of wire guide with a soap sponge.
3. Rinse wire guide thoroughly with clean, running water.
4. Wipe wire guide dry with a lint-free cloth.
5. Proceed with either high-level disinfection or sterilize device using the following sterilization set points.

HIGH-LEVEL DISINFECTION

1. Following disinfectant manufacturer's guidelines, completely immerse wire guide in a disinfectant.
2. Rinse wire guide thoroughly with clean, running water. **Note:** Proper rinsing with clean water and complete drying of wire guide is necessary following use of disinfectant.
3. Dry wire guide completely with a lint-free cloth.

ETO CYCLE PARAMETERS

PARAMETER SET POINT

Prevacuum Pressure 2.0 psia (25.9" Hg Vac)

Relative Humidity 60% RH

Humidity Dwell 20 minutes

Chamber Temperature 54.4 °C (130 °F)

Exposure Pressure To Be Determined¹

Exposure Dwell 2 hours

ETO Concentration 600 mg/L

Post Vacuums 3

Post Vacuum Pressure 2.0 psia (25.9" Hg Vac)

Air Wash² 5 minutes

Aeration Temperature 54.4 °C (130 °F)

Aeration Dwell 12 hours

Air Changes During Aeration 90 per hour

1 To be calculated by the Institution to achieve an ETO concentration of 600 mg/L.

2 Repetitive vacuums from ambient pressure to 13.7 psia over 5 minutes.

Note: ETO sterilizer chamber loading should not exceed 11 products per cubic foot of chamber space or exceed the usable chamber space. If any damage to a product is noted prior to sterilization, the product should be taken out of service immediately.

AUTOClave CYCLE PARAMETERS

PARAMETER SET POINT

Cycle Type Prevacuum

Prevacuum Pressure 2.5 psia (24.8" Hg Vac)

Chamber Exposure Temperature 132.2 °C (270 °F)

Chamber Exposure Pressure 41.8 psia (27.1 psig)

Exposure Dwell 5 minutes

Note: Autoclave chamber loading should not exceed 10 products per cubic foot of chamber space or exceed the usable chamber space. If any damage to a product is noted prior to sterilization, the product should be taken out of service immediately.

MATERIALS

Component	Material	Patient Contacting
Dilator	Polyvinylchloride (PVC)	Yes
Dilator Marker	PVC/ black tungsten powder	Yes
Wire Guide w/Markings	Stainless Steel	Yes
Washing adapter	Stainless Steel /Nickel and Chrome Plated Brass	No

COMPLIANCE WITH RELEVANT REGULATIONS AND TECHNICAL STANDARDS

Please notify Cook for return authorization.

Cook Medical Europe Ltd

O'HALLORAN ROAD

NATIONAL TECHNOLOGY PARK

LIMERICK, IRELAND

TEL: +353 61 334440 FAX: +353 61 334441

WWW.COOKMEDICAL.COM

SHELF LIFE

The "use by" date as denoted by symbols on the labeling. The shelf life is shown by indication the time of manufacture (factory symbol) to the stated expiration date (hourglass symbol). The difference between both dates is the shelf life.

The aforementioned devices have a three (3) year shelf life.

PACKAGING AND SUPPLY DETAILS

Savary-Gilliard Dilator Sets SGD-100-1, SGD-100-2, SGD-70-1, SGD-70-2 is supplied with each component being put in individual packaging, which is sealed. Sets are supplied in plastic cases together with the instructions for use for Savary-Gilliard Dilators and Savary-Gilliard Wire Guides.

STERILIZATION DETAILS

Devices described in this Technical Data Summary are supplied non-sterile and can be disposable or reusable. Reusability of a device depends, in large part, on care of the device by user (reference IFU).

STORAGE RULES

These devices should be stored in a dark, dry, cool place; avoid extended exposure to light.

TRANSPORTATION RULES

Transportation shall account for the type of transportation employed, the destination, the vendor used for transportation and an estimation of the frequency and harshness of any handling required during transportation.

DISPOSAL RULES

Upon completion of procedure, the devices are to be disposed of as per institutional guidelines for biohazardous medical waste. There is no environmentally harmful effect as the result of the device disposal in accordance with the relevant national guidelines.

Regulatory Affairs Manager

COOK MEDICAL EUROPE LTD.

O'Halloran Road,

National Technology Park

Limerick, Ireland.

Company Registration No. 492972

VAT Registration No. 777626447