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PROPLATE

Electrosurgical Grounding Plate

Size / Size	Model	Specifications	
☒ ☐	9571C	Adult / Cloth / Solid / Uncorded / 180x120mm	
☐ ☐	9571F	Adult / Foam / Solid / Uncorded / 180x120mm	
☐ ☐	9572C	Adult / Cloth / Split / Uncorded / 180x120mm	
☐ ☐	9572F	Adult / Foam / Split / Uncorded / 180x120mm	

CAUTION | Federal Law (U.S.A) restricts this device to sale by or on the order of a physician.

www.protectate.com

LOT
180418
APR 2022

The PROPLATE (E) by Protecta is a disposable single use medical electrode which provides a return path for high frequency electrical current to the ESI device.

Instructions for use

- For grounding plate placement, choose a flat, hairless, or well-shaved area of patient's skin near the surgical site but not closer than 20cm (8 inches) proximal to the arm or torso or thigh or chest; preferably the back (center) of the back.
- Apply gel liner with electrode to the adipose tissue, or to the patient's back directly from the unsterilized area of the back. Do not use the gel liner for the patient's weight during surgery.
- Shape the electrode to the thorax (a pleated & contoured pad).
- Place the electrode on the patient's back so that the gel liner is in contact with the skin.
- After applying the electrode, do not move the grounding plate or the patient's back.
- After use, place the electrode in the disposal bag and dispose of the plate by using it as a sharps container.

Caution

- Do not use additional use of the PROPLATE grounding plate.
- Do not overlap.
- Do not reposition the grounding plate.
- Place ESI electrode as far as possible from the surgical area and grounding plate site.
- Do not allow the patient's body to touch conductive objects.
- Spill type must be used to activate this type of monitoring system.
- Do not use plate on adult patients weighing 15 kg, pediatric patients weighing 5 kg to 15 kg.
- Correct application and proper site selection can prevent the possible burn.
- Avoid removing and/or reapplying plates that skin site frequently.
- Do not use a plate if the gel is dry or dislodged from the aluminum foil of the plate. Otherwise, burns could result at the skin site.
- Avoid skin-to-skin contact (for example between the arms and body of the patient).
- Whenever the ESI fails to produce the desired effect, make sure that the plate is in full contact on the skin site before turning up power.
- Observe the safety instructions provided in the instructions for use of the diathermy device.
- Do not reuse the grounding plate.
- Re-use may cause a risk of cross-contamination, electrical burn. And the electrodes may not be firmly attached from skin to the patient's skin.
- The transport plastic liner has to be disposed to trash can immediately after the operation of the electrodes in order to prevent any possible slippery damage.

Warning

Not following these instructions may lead to skin burns during use.

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Bio Protect Inc. (in BDR)