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CERTIFICATE

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China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of HUNAN YANYANG LATEX TECHNOLOGY INDUSTRIAL CO.,LTD on the annexed DOCUMENT is genuine.

China Council for the Promotion of International Trade

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证明网址 Website for verifying the certificate: <http://www.rzcepit.com/validate.html>

Hunan Yunyang Latex Technology
Industrial Co., Ltd.

Director General Tang Liping

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(Signature)

Date 2022.11.28

INSTRUCTION FOR USE
medical device

Natural rubber latex male condoms ELASUN

Version 1 (2022)

1. Medical device name

Natural rubber latex male condoms **Elasun**

Versions:

1. Natural rubber latex male condoms **Elasun «G-spot air bag X5»**: especially thin, stimulating (with spikes), flavored, with lubricant - 2 pcs or 6 pcs/pack.
2. Natural rubber latex male condoms **Elasun «G-spot stimulation»**: especially thin, stimulating (with ribs and large dots), flavored, with lubricant - 10 pcs/pack.
3. Natural rubber latex male condoms **Elasun «Ribbed and dotted»**: especially thin, stimulating (with ribs and dots), flavored, with lubricant - 10 pcs/pack.
4. Natural rubber latex male condoms **Elasun «Big dotted»**: especially thin, stimulating (with large dots), flavored, with lubricant - 10 pcs/pack.
5. Natural rubber latex male condoms **Elasun «Orgasm and dotted»**: especially thin, stimulating (with large dots), flavored, with an increased amount of lubricant - 10 pcs/pack.
6. Natural rubber latex male condoms **Elasun «Slim fit»**: ultra-thin, smooth, tight-fitting, flavored, with an increased amount of lubricant - 10 pcs or 60 pcs/pack.
7. Natural rubber latex male condoms **Elasun «Ultra-thin and smooth»**: ultra-thin, smooth, flavored, with lubricant - 3 pcs or 10 pcs/pack.
8. Natural rubber latex male condoms **Elasun «Queen»**: ultra-thin, smooth, flavored, with an increased amount of lubricant - 12 pcs/pack.
9. Natural rubber latex male condoms **Elasun «Iron Man»** with physical delay of ejaculation: thickened, smooth, flavored, with lubricant - 2 pcs/pack.
10. Natural rubber latex male condoms **Elasun «Go Bananas»** for non-bAnal sex: ultra-thin, smooth, flavored, with an increased amount of lubricant - 2 pcs or 10 pcs/pack.
11. Natural rubber latex male condoms **Elasun «Strawberry»** for oral sex: ultra-thin, smooth, flavored, without lubricant - 10 pcs/pack.
12. Natural rubber latex male condoms **Elasun «Ice»**: ultra-thin, smooth, tight-fitting, flavored, with lubricant - 10 pcs/pack.

Abbreviations used in the text of the document:

1. **Elasun «G-spot air bag X5»** - Natural rubber latex male condoms **Elasun «G-spot air bag X5»**: extra thin, stimulating (with spikes), flavored, with lubricant - 2 pcs or 6 pcs/pack.
2. **Elasun «G-spot stimulation»** - Natural rubber latex male condoms **Elasun «G-spot stimulation»**: extra thin, stimulating (with ribs and large dots), flavored, with lubricant - 10 pcs/pack.
3. **Elasun «Ribbed and dotted»** - Natural rubber latex male condoms **Elasun «Ribbed and dotted»**: extra thin, stimulating (with ribs and dots), flavored, with lubricant - 10 pcs/pack.
4. **Elasun «Big dotted»** - Natural rubber latex male condoms **Elasun «Big dotted»**: extra thin, stimulating (with large dots), flavored, with lubricant - 10 pcs/pack.
5. **Elasun «Orgasm and dotted»** - Natural rubber latex male condoms **Elasun «Orgasm and dotted»**: extra thin, stimulating (with large dots), flavored, with an increased amount of lubricant - 10 pcs/pack.
6. **Elasun «Slim fit»** - Natural rubber latex male condoms **Elasun «Slim fit»**: ultra-thin, smooth, tight-fitting, flavored, with an increased amount of lubricant - 10 pcs or 60 pcs/pack.
7. **Elasun «Ultra-thin and smooth»** - Natural rubber latex male condoms **Elasun «Ultra-thin and smooth»**: ultra-thin, smooth, flavored, with lubricant - 3 pcs or 10 pcs/pack.
8. **Elasun «Queen»** - Natural rubber latex male condoms **Elasun «Queen»**: ultra-thin, smooth, flavored, with an increased amount of lubricant - 12 pcs/pack.
9. **Elasun «Iron Man»** - Natural rubber latex male condoms **Elasun «Iron Man»** with physical delay

- of ejaculation: thickened, smooth, flavored, with lubricant - 2 pcs/pack.
10. **Elasun «Go Bananas»** - Natural rubber latex male condoms Elasun «Go Bananas» for non-bAnal sex: ultra-thin, smooth, flavored, with an increased amount of lubricant - 2 pcs or 10 pcs/pack.
 11. **Elasun «Strawberry»** - Natural rubber latex male condoms Elasun «Strawberry» for oral sex: ultra-thin, smooth, flavored, without lubricant - 10 pcs/pack.
 12. **Elasun «Ice»** - Natural rubber latex male condoms Elasun «Ice»: ultra-thin, smooth, tight-fitting, flavored, with lubricant - 10 pcs/pack.

2. Information about the manufacturer, developer and authorized representative of the manufacturer of a medical device in the territory of the Russian Federation

Manufacturer and developer: Hunan Yunyang Latex Technology Industrial Co., Ltd.

Legal and actual address, production address: No.10 Wangwang East Road, Wangcheng Economic & Technological Development Zone, Wangcheng District, 410200 Changsha City, Hunan, P.R. China
Phone/Fax: 0731-88052518

The representative of the manufacturer in the territory of China and the holder of the certificate of registration of the trademark Elasun: Guangzhou Licheng Trading Co., Ltd

Office 5 1803, No. 1 Huizhan-nan-ulu Street, Haizhu District, Guangzhou City District

Phone: 020-89206393, fax: 020-89236105

www.elasun.cn

Authorized representative of the manufacturer of a medical device in the territory of the Russian Federation:

«ALIANCE-M» Limited Liability Company

443090, Samara region, Samara city, Antonov-Ovseenko street, house 44a, office 712

Phone: +7(937)206-86-26, e-mail: info@elasun.ru

3. Purpose of the medical device with indication of the potential user

3.1. Purpose of the medical device and the principle of operation

The device is intended for contraception and/or prevention of sexually transmitted infections.

The device is manufactured as a natural rubber latex sheath completely covering the penis during intercourse to preclude sperm from entering the female genital tract and/or to prevent sexual transmission of infections. This is a non-sterile single use medical device.

3.2. Potential consumer

The medical device is intended for non-professional use in accordance with the instructions for use.

3.3. Scope of application

For personal use including at home.

4. Indications, contraindications and possible side effects

Indications: For the purposes of contraception and/or prevention of sexually transmitted infections

Counterindications: Individual intolerance (hypersensitivity to device components)

Possible side effects: Allergic reactions caused by natural rubber latex are possible\

5. Description of the major functional elements of the medical device.

Description of the components of the medical device (if available).

5.1. Design and dimensional and mass specifications.

Depending on the version, condoms have the following texture and design: seamless, with parallel or non-parallel (contoured) sides, flat-ended or with reservoir tip (collector) on the end, smooth or textured (spiked, ribbed and/or dotted), transparent/translucent/colored, flavored, lubricated or non-lubricated. The open end of the condom is rolled into a rim.

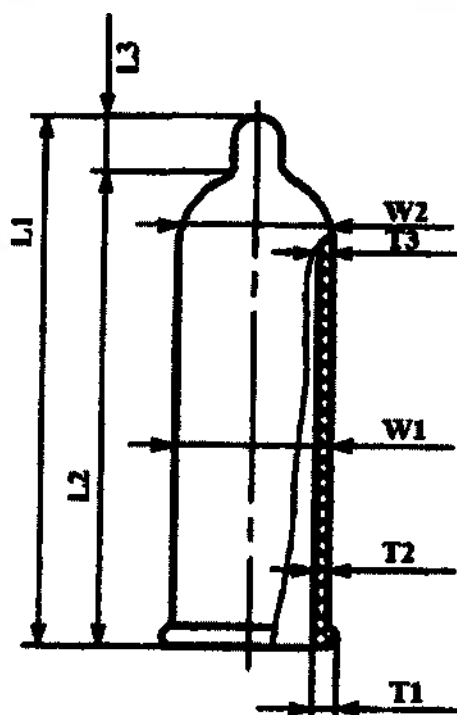
Fig. 1. Dimensions of the medical device

(L1 - total length; L2 - length excluding the reservoir tip; L3 - length of the reservoir tip;

W1 - width in the middle part; W2 - width in the upper part;

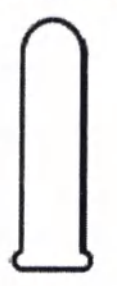
T1 - thickness of the rim; T2 - thickness in the middle part; T3 - thickness in the upper part)

Table 1. Design features of a medical device

Name	Image	Design features	Dimensional and mass specifications
1. Elasun «G-spot air bag X5»		Seamless condoms, with parallel sides, with reservoir tip, extra thin*, with spikes, translucent pink, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.05 ± 0.01 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 3 ± 0.3 g Width of textured surface (spikes): 30 ± 2 mm Length of textured surface (spikes): 58 ± 2 mm at the distance of 96 ± 2 mm from the open end. Spike height: 5 ± 0.5 mm Total number of spikes: 32 pcs. (4 pcs. in a horizontal row and 8 pcs. in a vertical row).

2. Elasan «G-spot stimulation»		Seamless condoms, contoured, flat-ended, extra thin*, with ribs and large dots, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.05 ± 0.01 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1.5 ± 0.3 g Textured surface across the entire width of the device. Length of textured surface D1 (dots): 85 ± 2 mm at the distance of 37 ± 2 mm from the open end. Distance between textured surfaces D1 and R: 4 ± 2 mm Length of textured surface R (ribs): 9 ± 2 mm Distance between textured surfaces R and D2: 4 ± 2 mm Length of textured surface D2 (dots): 12 ± 2 mm Distance between textured surfaces D2 and D3: 10 ± 2 mm Length of textured surface D3 (dots): 7 ± 2 mm Textured surface height: no more than 1 mm
3. Elasan «Ribbed and dotted»		Seamless condoms, contoured, with reservoir tip, extra thin*, with ribs and dots, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.05 ± 0.01 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1.5 ± 0.3 g Textured surface across the entire width of the device. Length of textured surface R (ribs): 55 ± 2 mm at the distance of 68 ± 2 mm from the open end. Distance between textured surfaces R and D: 30 ± 2 mm Length of textured surface D (dots): 18 ± 2 mm Textured surface height: no more than 1 mm
4. Elasan «Big dotted»		Seamless condoms, with parallel sides, with reservoir tip, extra thin*, with large dots, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.05 ± 0.01 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1.5 ± 0.3 g Textured surface across the entire width of the device. Length of textured surface (dots): 100 ± 2 mm at the distance of 70 ± 2 mm from the open end. Textured surface height: no more than 1 mm

5. Elasun «Orgasm and dotted»			Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.05 ± 0.01 mm Amount of lubricant: $1000 \pm 5\%$ mg Net weight of a single product: 1.5 ± 0.3 g Textured surface across the entire width of the device. Length of textured surface (dots): 98 ± 2 mm at the distance of 80 ± 2 mm from the open end. Textured surface height: no more than 1 mm
6. Elasun «Slim fit»		Seamless condoms, contoured (tapered in the middle part — tight-fitting), with reservoir tip, ultra-thin*, smooth, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 49 ± 2 mm Width in the upper part W2: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Amount of lubricant: $1000 \pm 5\%$ mg Net weight of a single product: 1 ± 0.3 g
12. Elasun «Ice»			Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 49 ± 2 mm Width in the upper part W2: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1 ± 0.3 g
7. Elasun «Ultra- thin and smooth»		Seamless condoms, with parallel sides, with reservoir tip, ultra-thin*, smooth, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1 ± 0.3 g
8. Elasun «Queen»			Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Amount of lubricant: $2000 \pm 5\%$ mg Net weight of a single product: 1 ± 0.3 g
9. Elasun «Iron Man»		Seamless condoms, contoured (extended at the top), with reservoir tip, thickened* at the top (for physical delay	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 52 ± 2 mm Width in the upper part W2: 60 ± 2 mm

		of ejaculation), smooth, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.15 ± 0.01 mm Thickness in the upper part T3: 0.2 ± 0.01 mm Amount of lubricant: $900 \pm 5\%$ mg Net weight of a single product: 1.5 ± 0.3 g
10. Elasun «Go Bananas»		Seamless condoms, contoured, with reservoir tip, ultra-thin*, smooth, transparent, natural color, flavored, with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Length excluding the reservoir tip L2: ≥ 160 mm Length of the reservoir tip L3: 12 ± 2 mm Width in the middle part W1: 49 ± 2 mm Width in the upper part W2: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Amount of lubricant: $1200 \pm 5\%$ mg Net weight of a single product: 1 ± 0.3 g
11. Elasun «Strawberry»		Seamless condoms, with parallel sides, flat-ended, ultra-thin*, smooth, transparent, natural color, flavored, non-with lubricant. The open end of the condom is rolled into a rim.	Total length L1: ≥ 180 mm Width in the middle part W1: 52 ± 2 mm Thickness of the rim T1: 1.6 ± 0.5 mm Thickness in the middle part T2: 0.04 ± 0.008 mm Net weight of a single product: 1 ± 0.3 g

*Extra thin are products with the thickness of 0.05 ± 0.01 mm; ultra-thin are products with the thickness of 0.04 ± 0.008 mm; thickened are products with the thickness of more than 0.1 mm.

5.2. Materials

The devices contain no materials of animal or human origin. The materials selected for manufacture of a medical device meet all the requirements (possess all chemical and physical properties) ensuring safety and effectiveness of device use: nontoxicity and biocompatibility with soft tissues.

Type of contact with the human body: short-term contact (less than 24 hours) with intact skin and mucous membrane

Table 2. Materials used for the manufacture of medical device

Main material: Natural rubber latex manufactured by Tavnor Rubber Industry 1982 Co., Ltd, Thailand. Free of spermicides. Flavor, lubricant and dye depending on the version:	
1. Elasun «G-spot air bag X5»	Bloomage water-based lubricant (with hyaluronic acid) manufactured by Bloomage Biotech Ltd. Flower (floral) food flavor Pink food dye
2. Elasun «G-spot stimulation» 3. Elasun «Ribbed and dotted» 4. Elasun «Big dotted» 5. Elasun «Orgasm and dotted» 6. Elasun «Slim fit» 7. Elasun «Ultra-thin and smooth» 8. Elasun «Queen»	Bloomage water-based lubricant (with hyaluronic acid) manufactured by Bloomage Biotech Ltd. Flower (floral) food flavor Does not contain any dyes

9. Elasun «Iron Man»	
10. Elasun «Go Bananas»	
11. Elasun «Strawberry»	Without lubrication Strawberry food flavor Does not contain any dyes
12. Elasun «Ice»	Lubricant: Dow corning "Ice" silicone oil manufactured by Dow Inc. Flower (floral) food flavor Does not contain any dyes

6. Nomenclature and scope of supply

Table 3. Nomenclature of medical device

Name	Quantity in the package	REF number
1. Elasun «G-spot air bag X5»	2 pcs	6921137700646
	6 pcs	6921137700554
2. Elasun «G-spot stimulation»	10 pcs	6921137700165
3. Elasun «Ribbed and dotted»	10 pcs	6921137760978
4. Elasun «Big dotted»	10 pcs	6921137761005
5. Elasun «Orgasm and dotted»	10 pcs	6921137700233
6. Elasun «Slim fit»	10 pcs	6921137700691
	60 pcs	6921137700677
7. Elasun «Ultra-thin and smooth»	3 pcs	6921137761036
	10 pcs	6921137760961
8. Elasun «Queen»	12 pcs	6921137760336
9. Elasun «Iron Man»	2 pcs	6921137760992
10. Elasun «Go Bananas»	2 pcs	6921137700493
	10 pcs	6921137700479
11. Elasun «Strawberry»	10 pcs	6921137700028
12. Elasun «Ice»	10 pcs	6921137700806

Scope of supply includes:

Name	Quantity
Medical device of one versions in individual packaging	According to table 3
Group consumer packaging (box)	1 pc
Instructions for use*	1 pc per consumer package

*Instructions for use in abbreviated form are provided on the outer and/or inner surface of the consumer packaging, and for products with the reference numbers 6921137700646 and 6921137700554 — in an additional leaflet inserted in the consumer packaging. The complete version of the instructions is provided at the request of the customer.

7. Description of accessories, medical devices or non-medical devices intended for use in combination with the claimed medical device (if available).

Not applicable. Accessories, medical devices or non-medical devices intended for use in combination with the claimed medical device are not provided.

8. Information on medicines in the medical device

Not applicable. The device does not contain medicines.

9. Materials of animal and (or) human origin

The devices do not contain materials of animal or human origin.

10. Functional and technical characteristics

- Non-sterile single use device.
- There is no contraceptive method, which is 100% effective in protecting against STIs and HIV infection; however, correct use of the device can help reduce the risk of pregnancy and some STIs.

The medical device complies with the following technical characteristics:

Characteristic	Value
Tensile strength, MPa	Not less than 20
Tensile strength (after aging), MPa	Not less than 17
Elongation at break, %	Not less than 700
Elongation at break (after aging), %	Not less than 600
Break volume, dm ³	Not less than 16.0 — for devices with the central part width of more than or equal to 45.0 mm and less than 50.0 mm; Not less than 18.0 — for devices with the central part width of more than or equal to 50.0 mm and less than 56.0 mm;
Break pressure, kPa	Not less than 1.00

- On the surface of the condom, there should be no cracks, holes, bubbles, foreign particles, coagulum, streaks and sags exceeding the thickness tolerance.
- The rim should be free of tears and pinches.
- On the surface of the condom, there should be not more than 3 folds longer than 10 mm and wider than 1 mm at the distance of more than 1/3 from the rim.
- Condoms must be air-tight.
- The length of each condom must be within the limits specified by the manufacturer, but not less than 160 mm.
- The width of each condom must be within the nominal range specified by the manufacturer, the tolerance is ± 2 mm.
- The thickness of each condom must be within the nominal range specified by the manufacturer. For the nominal value of less than 0.05 mm, the tolerance is ± 0.008 mm. For the nominal value greater than or equal to 0.05 mm, the tolerance is ± 0.01 mm.
- The amount of lubricant in individual packaging should be within the range specified by the manufacturer, but not less than 500 mg (except for the article 6921137700028 Elasan "Strawberry" without lubricant).

11. Precautions


1. Before use, please read the instructions for use.
2. Before use, check the integrity of the device and its packaging. Do not use the device if it has visible defects and/or its packaging is damaged.
3. Check the expiration date before use. Do not use the device after the expiry date specified on the package.

4. The condom is made of natural latex. Allergic reactions from natural rubber latex, including anaphylactic shock, are possible. If you are allergic to natural rubber latex, consult your doctor before use, and use with caution.
5. The condom should be put on the erected penis before any physical contact. There is no contraceptive method, which is 100% effective in protecting against STIs and HIV infection; however, correct use of the device can help reduce the risk of pregnancy and some STIs.
6. The device is intended for single use only. Reuse may break the integrity of the condom or increase the risk of contracting sexually transmitted infections. If a condom is used first for oral sex, use a new condom for other subsequent types of sex.
7. Handle the condom with care, and specifically remove it from the package carefully so as not to damage it with fingernails, jewelry, etc.
8. If you feel that the condom is slipping or getting too tight, stop and check it, as this problem may cause the condom to break.
9. If the condom leaks or breaks during use, contact a health facility as soon as possible, within 72 hours.
10. Use of condoms for non-vaginal intercourse increases the chance of breaking or slipping. For anal sex, or where necessary use extra lubricant on the outside of the condom.
11. As an additional lubricant, choose the appropriate recommended types. Avoid petroleum-based lubricants such as petroleum jelly, baby oils, shower gels, massage oils, butter, margarine, etc. as these can break the integrity of the condom.
12. Consult your doctor about compatibility of drugs coming into contact with the condom during its use.
13. Condoms can pose a choking hazard. Keep out of reach of children.
14. The device must be protected from direct sunlight and not be exposed to petroleum jelly, oils, organic solvents, petroleum products, phenols, acids and alkalis, copper, manganese and other metals and chemicals, which are harmful to rubber. Ozone and ultraviolet radiation have a damaging effect on rubber latex.

12. Usage procedure

1. Carefully open the package and remove the condom.
2. With your thumb and forefinger, lightly squeeze the tip of the condom to squeeze air out of it.
3. Put the condom on the erected penis.
4. Make sure that the twisted part of the condom is on the outer edge, fix the tip of the condom and at the same time pull the condom up to the base of the penis with the other hand.
5. After ejaculation, hold the condom tightly at the base and remove it as soon as possible.
6. After use, roll up the used condom and place it in a disposable bag, tie up the bag, or close it with zip ties or other aids to prevent spillage. Do not flush the device down the toilet.

[Schematic illustration of condom use]

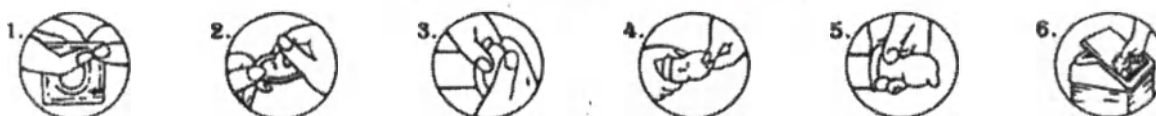Additional information (only for the Elasan "G-spot air bag X5" version): The condom should be worn in accordance with the position chosen for sex so that when penetrating, the stimulating part with spikes is located at the location of the partner's G-spot (note: the G-spot is located on the upper wall of the vagina 3-5 cm from the entrance). When penetrating from the front, the condom should be worn with a textured dot surface up; when penetrating from behind - with a textured dot surface down).


13. Packaging of the medical device

The packaging of the medical device includes:


- Individual packaging, which ensures the tightness and protection of the medical device against the adverse effects of light and other environmental conditions, and the integrity of the condom when the package is opened, and also contains labelling information. The design of individual packaging ensures its easy opening.

The packaging of the medical device ensures the shelf life of 5 years. The material used does not impair the quality of the medical device during its use and storage.

Articles 6921137700646 and 6921137700554 (Elasun "G-spot air bag X5") are packed in individual packaging — a blister of opaque plastic and aluminized protective film. The shape is ovoid (egg-shaped). Dimensions, mm: $61 \times 61 \times 21 \pm 1$ mm.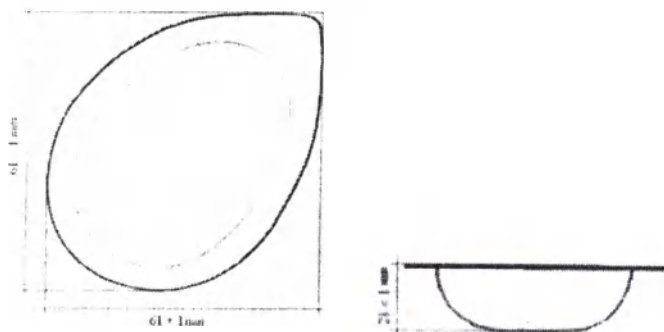

Articles 6921137700493 and 6921137700479 (Elasun "Go Bananas"), 6921137700233 (Elasun "Orgasm and dotted") are packed in individual packaging — a blister of opaque plastic and aluminized protective film. The shape is round. Dimensions, mm: $53 \times 53 \times 7 \pm 1$ mm.

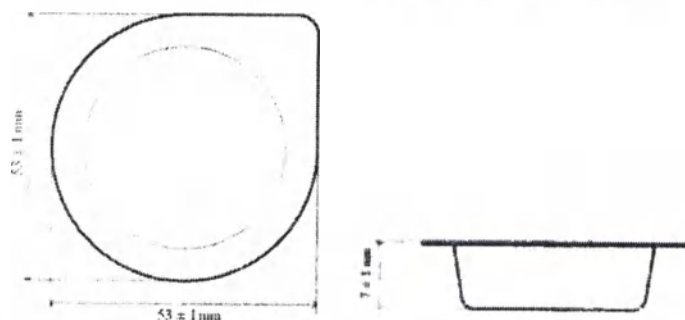All other articles are packed in individual opaque packaging made of multilayer foil material. The shape is square. Dimensions, mm: $60 \times 60 \pm 1$ mm.

- Group consumer packaging, which provides protection for individual packaging against external impacts during transportation and storage, and contains labelling information.

Articles 6921137700646 and 6921137700554 (Elasun "G-spot air bag X5") in individual packaging are placed into group consumer packaging — a transparent PVC box. A paper leaflet is also placed into the box.

All other articles in individual packaging are placed into group consumer packaging — a folding carton wrapped with transparent film. Instructions for use are printed on the outside and/or inside of the box.

- Transport packaging, in which products in consumer packaging are placed.

Heavy-duty shipping boxes made of corrugated cardboard ensure sufficient safety of the medical device during long-term transportation. Dimensions, mm (HxWxD): $385 \times 305 \times 545 \pm 2$ mm.

14. Labeling

Individual package labelling must contain the following information:

- Manufacturer's name and/or registered brand;
- Medical device name (version/model);
- A symbol prohibiting reuse;
- Manufacture date;
- Expiration date (year/month or month/year) in the following format: the year is indicated by four digits, the month — by two digits;
- Batch code.

Consumer package labelling must contain the following information:

- Medical device name;
- Registered brand;
- Name and address of the manufacturer;
- Device description (its design features);
- Key features such as width, length, color, presence/absence of lubricant, absence of spermicides;
- Warning of a potential allergic reaction to latex;
- Number of devices per consumer packaging;
- Bar code;
- Batch code;
- Manufacture date;
- Expiry date (year/month or month/year);
- Storage conditions;
- Instructions for use;
- Precautions including a single use statement;
- A symbol contains natural latex;
- A symbol prohibiting reuse;
- A symbol refer to the instructions for use;
- A sign calling observance the sanitary state of the environment;
- Storage conditions;
- Information on disposal.

Transport package labelling must contain the following information:

- Registered brand;
- Medical device name;
- A symbol warning of the need to protect against moisture;
- A symbol indicating careful handling of the goods (fragile goods);
- A symbol indicating the correct upright position of the goods;
- A symbol warning of the need to protect against sunlight.;
- Storage conditions;
- Name and address of the manufacturer;
- Manufacture date;
- Expiry date;
- Batch code;
- Number of packages;
- Weight and dimensions of the package.

Marks and symbols:

Symbol	Decoding	Symbol	Decoding
	Contains natural latex		Refer to the instructions for use
	Do not reuse		Observe the sanitary state of the environment
	Date of manufacture		Best before...
	Batch code		Protect from moisture
	The fragility of the goods. Careful handling of goods		Correct vertical position of the goods
	Keep away from sunlight		Manufacturer

15. Sterilization and disinfection

Non-sterile single use device.

Not subject to sterilization and disinfection.

16. Maintenance and repair

Single-use device.

Reuse is not allowed.

The devices are not subject to repair and maintenance.

17. Usage, transportation and storage conditions

The medical device is transported using all modes of transport in covered vehicles in accordance with the transportation rules applicable for this mode of transport, at the temperature from minus 50 to plus 50 °C. For transportation at sub-zero temperatures, the medical device should be unpacked after exposure for 1 day from the date of receipt under normal climatic conditions.

The medical device should be stored in a packaged form, in a closed warehouse at the temperature of 0 to +25 °C and the relative air humidity of not more than 80% at the distance of at least 1 m from heating devices. Products must be protected against direct sunlight and not exposed to petroleum jelly, oils, organic solvents, petroleum products, phenols, acids and alkalis, copper, manganese and other metals and chemicals, which are harmful to rubber. Ozone and ultraviolet radiation have a damaging effect on rubber latex. Keep out of reach of children.

There are no special operating conditions, when used according to the intended purpose, indications, contraindications, method of use, taking into account possible side effects and precautions. During the period of operation, the condom is in the human body. Operating temperature from +32 to +42 °C.

18. Shelf-life and manufacturer's warranty

Shelf-life is 5 years subject to storage conditions.

The manufacturer guarantees conformity of devices to the set requirements if all transportation, storage and usage conditions are met by the consumer.

The devices are non-toxic during the entire shelf-life and storage period.

Do not use after the expiry date.

19. Procedure and conditions of medical device disposal or destruction

After use, roll up the used condom and place it in a disposable bag, tie up the bag, or close it with zip ties or other aids to prevent spillage. Do not flush the device down the toilet.

The devices must be disposed of in accordance with the requirements of legislation and regulations concerning the disposal of products, adopted in the country of application of the medical device.

After using the medical device according to the intended purpose, it shall be disposed of as potentially hazardous wastes of class B in accordance with the requirements of SanPiN 2.1.3684-21.

In case of expiry and/or violation of package integrity, the medical device shall be disposed of as medical wastes of class A in accordance with the requirements of SanPiN 2.1.3684-21.

20. Environmental protection requirements

The devices neither release any toxic substances into the atmosphere nor produce (in direct contact) any harmful effect on the human body during manufacture, storage, transportation and usage. Work with devices does not require any special precautions.

21. Warranty Obligations

Before using the device, you should carefully read the instructions for use. The manufacturer guarantees the sterility of the goods, as well as its conformity with the requirements of international and national standards throughout the product shelf-life, provided that the transportation, storage rules, as well as usage conditions, established by the manufacturer are observed. The manufacturer shall not be liable for the use of the medical device for purposes other than as intended.

If the package integrity is damaged, or if there are other defects in the package or the device itself, it must be disposed of and replaced with a new one.

Concerning product quality, please contact the manufacturer or his authorized representative in the territory of the Russian Federation: Limited Liability Company "Alliance-M" 44a Antonova-Ovseenko Street, office 712, Samara region, 443090, Russian Federation, phone: +7(937)206-86-26, e-mail: info@elasun.ru

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