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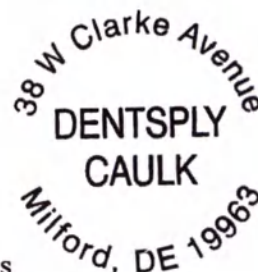
М.П. / Stamp

Instruction for Use

Dental composite cement Calibra® in design variants with accessories
Dental composite dual cure cement Calibra® Universal

Sworn to and subscribed before me
this 7th day of September, 2023.

[Signature]

Dawn Renea Kenton, Notary Public

2023

1. INTRODUCTION

ATTENTION! To ensure safety when using Dental composite dual cure cement Calibra® Universal as intended, all users should carefully read this Instructions for Use prior to using the product. The instructions for use are written to familiarize the consumer with the design of the medical device, its functions, and also describe the conditions and rules of operation. Instructions for use are intended for medical professionals.

- Always follow these instructions for use.
- Instructions for use should be kept in the immediate vicinity of the product.

2. INFORMATION ABOUT MANUFACTURER OF MEDICAL DEVICE

2.1. Developer

Dentsply LLC also dba Dentsply Caulk, 38 West Clarke Avenue, Milford, DE 19963, USA

2.2. Manufacturer

Dentsply LLC also dba Dentsply Caulk, 38 West Clarke Avenue, Milford, DE 19963, USA

2.3. Manufacturing site

Dentsply LLC also dba Dentsply Caulk, 38 W. Clarke Avenue, Milford, Delaware 19963, USA

3. INFORMATION ABOUT THE AUTHORIZED REPRESENTATIVE

Dentsply Sirona LLC

Address: premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035

Tel: +7 (495) 725-10-87

E-Mail: CIS-Info@dentsplysirona.com

4. NAME OF THE MEDICAL DEVICE

Dental composite cement Calibra® in design variants with accessories (Further in the text "cement," "material," "device"); Dental composite dual cure cement Calibra® Universal

5. PURPOSE OF THE MEDICAL DEVICE

The device is intended for fixation of indirect restorations: porcelain, ceramic, composite veneers, inlays, onlays, ceramic and metal-ceramic crowns, bridges; pins.

6. INDICATIONS

Cementation of indirect restoratives:

- porcelain, ceramic, composite veneers, inlays, onlays, ceramic and metal-ceramic crowns, bridges; pins;

7. CONTRAINDICATIONS

Dental composite dual cure cement Calibra® Universal is contraindicated:

- for use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components;
- for direct application to dental pulp tissue (direct pulp capping).

8. ADVERSE REACTIONS

- Product may irritate the eyes and skin. Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucous membranes: inflammation (see Warnings)
- Product may cause pulpal effects. (See Contraindications)
- Residual hardened excess cement may lead to soft tissue injury or irritation. (See Interactions and Step-by-Step instructions)

9. CONTENT OF DELIVERY

Dental composite cement Calibra® in design variants with accessories:

1. Dental composite dual cure cement Calibra® Universal (Syringe Refill) in syringes 4.5 g, shades: Light Shade, Medium Shade, Translucent Shade, Opaque Shade, Bleach Shade, consisting of:

1) Dental composite dual cure cement Calibra® Universal (Dual Cure AutoMix Syringe) in syringes 4.5 g of one shade – 2 pcs;

2) Mixing tips – 20 pcs;

3) Instructions for use on paper and/or in electronic form – 1 pce;

4) Technique guide – 1 pce (if necessary);

5) Reorder information – 1 pce (if necessary);

2. Dental composite dual cure cement Calibra® Universal (Syringe Introductory Kit), consisting of:

1) Dental composite dual cure cement Calibra® Universal (Dual Cure AutoMix Syringe) in syringes 4.5 g shades: Light Shade (1 pce), Medium Shade (1 pce), Translucent Shade (1 pce), Opaque Shade (1 pce), Bleach Shade (1 pce) – 5 pcs;

2) Mixing tips – 50 pcs;

3) Instructions for use on paper and/or in electronic form – 1 pce;

4) Technique guide – 1 pce (if necessary);

5) Reorder information – 1 pce (if necessary);

Accessories:

1. Mixing Tips Refill 50 pcs in package – no more than 100 packages.

10. POTENTIAL CONSUMERS

The medical device belongs to the group of products for professional use in dentistry.

Potential consumers of a medical device are qualified personnel of a medical institution. Before using the product, personnel must read the instructions for use.

11. WARNINGS AND PRECAUTIONS

Warnings

Calibra Universal Cement is acidic in nature and contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa, and may cause allergic contact dermatitis in susceptible persons.

• Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

- Eugenol containing materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.
- Contact with some astringent solutions may interfere with hardening of the polymeric components of the material.
- Calibra® Universal Cement contains an optimized photoinitiator combination. Use of high-power, broad-spectrum output curing lights during cleanup may produce unexpected results, rapid hardening, and shortened cleanup times during which the material must be in a semi-hardened state (gel phase). Monowave output LED lights with a single peak output around 470 nm are recommended. Check curing light effect on mixed cement in the laboratory prior to clinical use.

12. STEP-BY-STEP INSTRUCTIONS

The following preparatory steps apply to cementation of all types of indirect restorations:

1. Following removal of the temporary restoration and any remaining temporary cement, clean enamel and dentin as directed by the adhesive manufacturer's directions.
2. Check the fit and esthetics of the restoration. Technique Tip: Occlusal adjustment of inlays/onlays is best accomplished after final cementation.

The final shade of cured material is color stable. For color identification, please refer to the Try-In Paste section below.

Try-In Paste (optional)

- Dispense the appropriate shade of Calibra® Try-In Paste (available separately) from the syringe onto a clean mixing pad or glass slab. Load paste onto internal surfaces of the restoration and gently seat onto preparation. Clean excess with a cotton pellet and/or blunt explorer. Shades may be blended to achieve optimum esthetics. Important Technique Tip: Factors such as restoration shape, texture, thickness, and shade(s) of the underlying and surrounding tooth structure contribute to final perceived shade. Try-in paste is a guide for cement shade range selection only. NOTE: The try-in paste will not polymerize, thus offers unlimited work time.
- Once fit and esthetics are verified, thoroughly rinse try-in paste from restoration and preparation surfaces using water.

12.1 PREPARATION OF THE RESTORATION

• Metal Restorations

Internal surfaces of restorations should be clean and dry prior to cementation. Internal surface microetching (sandblasting with 50 μ alumina) of metal surfaces of the restoration is recommended. Calibra® Universal Cement has a chemical affinity for metal.

Technique Tips For Maryland Bridges: The use of 180° wrap of wings, rest seats, parallelism and slots or grooves is necessary for appropriate retention. The metal wings of the Maryland Bridge should be perforated, electrolytically etched, laboratory chemically etched, or mechanically sandblasted with 50 μ alumina.

• Porcelaine/Ceramic/Composite Restorations

Follow the dental laboratory or restoration manufacturer's instructions for pre-treatment, if required. Restorations designed to be silanated or if the internal silanated surface has been disturbed during try-in, apply Calibra® Silane Coupling Agent (available separately) according to the manufacturer's instructions.

12.2. APPLICATION WITH ADHESIVE (ADHESIVE FIXING)

12.2.1. TOOTH CONDITIONING/DENTIN PRETREATMENT/ADHESIVE APPLICATION

- Following restoration try-in, rinse preparation thoroughly with water spray and air dry.
- Proceed with adhesive application as directed in the complete directions for use.

CAUTION Delayed or Reduced Adhesion due to adhesive incompatibility

- Select compatible light cure or dual cure adhesive
- Always refer to and follow adhesive manufacturer's complete directions for use for selection and application of the appropriate light cure or dual cure adhesive. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately following adhesive application to cementation with Calibra Universal Cement, dual or self-cured dual barrel mix as indicated below.
- Calibra Universal Cement has been formulated to be compatible with specific* light cured Dentsply Sirona Adhesives without the need for addition of Self Cure Activator (see Complete Directions For Use).

*** Prime&Bond universal™ Adhesive**

Only Prime&Bond universal Adhesive are indicated for use without addition of Self Cure Activator. Insufficient data exist to support use of other Dentsply Sirona adhesives without Self Cure Activator. Follow selected adhesive's Dual Cured Cementation procedures.

- Follow adhesive application, air drying (solvent evaporation) and light curing instructions of selected adhesive prior to restoration cementation with Calibra Universal Cement for all types of restorations.

Special Instructions for Use with Light Transmissible restorations, including most* in-office CAD/CAM milled restorations:

- Final restoration must be light transmissible, $\leq 2.5\text{mm}$ thickness, with light curing of Calibra Universal Cement through the restoration after seating and excess cement cleanup.
- Select and apply Prime&Bond universal™ Adhesive. Light curing of applied adhesive may be accomplished before or after seating restoration with Calibra Universal Cement (i.e. the adhesive can be cured simultaneously with the cement immediately after the restoration has been placed on the treated surface).
- Following application and air drying (solvent evaporation), proceed immediately to cementation, dual cure cleanup and Light Transmissible Visible light curing through the restoration.
- * Zirconium-based restorations are generally not light-transmissible. Cure adhesive on the preparation prior to cementation.

Technique Tip: The adjacent teeth and/or the external surfaces of the restoration may be lubricated with a water-soluble medium to ease clean up of excess cement.

12.2.2. CROWN/BRIDGE/INLAY/ONLAY/POST CEMENTATION

Dentin Pretreatment/Adhesive Application. Refer to manufacturer's Directions for Use for selected adhesive. Adhesives (plus activators, if applicable) compatible with both light-cured, CQ initiated methacrylates and self-cured methacrylates are recommended. Apply adhesive (plus activator if applicable) to the internal bonding surface of the restoration if directed by adhesive manufacturer's directions for use.

12.2.3. CEMENTATION TECHNIQUE

CAUTION Danger of injury due to excessive force

- Apply slow and steady pressure on the syringe
- Do not use excessive force – dual barrel syringe rupture may result
- Remove syringe cap. Dispense and discard a small amount of material from the dual-barreled syringe. Be sure material is flowing freely from both ports. Holding syringe vertically, carefully wipe away excess so base and catalyst do not cross contaminate and cause obstruction of the ports. Save syringe cap for replacement following use.

- Install a mixing tip on the cartridge by lining up the v-shaped notch on the outside of the mixing tip with the v-shape notch on the syringe flange. Turn colored mixing tip cap 90 degrees in a clockwise direction to lock in place on syringe.

- Gently depress syringe plungers to begin the flow of material. **DO NOT USE EXCESSIVE FORCE.** If force is encountered, remove syringe from operating field, remove and discard mixing tip. Check for obstruction and confirm material is flowing from both syringe barrels. Wipe barrels and install new mixing tip as outlined above. Dispense a small amount through the mixing tip onto a mixing pad and discard.

- Apply a uniform layer of cement on the entire internal surface of the restoration directly from the mixing tip. For inlays/onlays, it may be helpful to apply a thin layer of cement to the internal portions of the tooth preparation to avoid any porosity or voids. **Technique Tip:** The mixing tip may be bent slightly to allow direct intraoral access for placement of cement into preparations with internal anatomy. For endodontic post spaces, use of a Lentulo Spiral or metal file to aid placement in the post space is recommended. At room temperature, Calibra Ceram Cement offers a minimum work time of 2 minutes when protected from ambient light. If loaded into restoration lined with adhesive or adhesive/activator mixture, room temperature work time (restoration seating time) will be less (see Precautions).

- Immediately seat the restoration in the mouth with gradual pressure. Verify complete seating. A gentle rocking or vibratory motion may be helpful to insure optimal seating.

CAUTION Incomplete cementation

- Intraoral work time is reduced due to warmth and contact with adhesive
- Assure restoration is fully seated within 1 minute
- Placement of multiple units may require multiple mixes of cement
- Protect restoration from contamination and movement until the final set of the cement (5 minutes from start of mix or in the case of light-transmissible restorations, completion of light curing).

CAUTION Inadequate retention due to insufficient stabilization

- Stabilize restoration during cleanup
- Protect restoration from movement during setting
- Allow cement to self-cure without disturbing for 5 minutes

12.3. APPLICATION WITHOUT USE OF ADHESIVE (SELF-ADHESIVE FIXATION)

12.3.1 TOOTH PRE-TREATMENT

Remove temporary restoration and excess temporary cement using an explorer, a rubber cup, and a prophylactic paste or water/floor of pumice.

- Rinse thoroughly and remove rinsing water completely by blowing gently with an air syringe or by blot drying with a dry cotton pellet.

- After try-in of the restoration (see the Try-In Paste section), thoroughly rinse the prepared surface with water jet and dry.

12.3.2. PREPARATION OF THE TOOTH

Dentin should be dried until there is no pooling of water, leaving a moist surface. Do not desiccate dentin. Avoid contamination. Etching of tooth surfaces is NOT recommended.

CAUTION Delayed adhesion due to improper tooth structure treatment.

- Tooth structure should be moist; Gently dry with air or blot with dry cotton.
- Remove all pooled moisture.
- Do not desiccate

12.3.3. PREPARATION OF AN IMPLANT ABUTMENT (METAL OR ZIRCONIA)

Follow the manufacturer's or lab's instructions for abutment pretreatment, including the following:

- recommended microetching/sandblasting with 50 μ alumina
- rinsing and drying
- priming (optional)

Avoid contamination after treating surfaces. Clean with alcohol and rinse thoroughly if contamination occurs.

12.3.4. ATLANTIS® CONUS SYNCONe® CAPS AND CONUS SUPERSTRUCTURE

See complete Directions for Use for Atlantis Conus abutment placement, Conus Structure fabrication and cementation instructions.

- Trial fitting of SynCone caps and Conus Structure is recommended to ensure proper clearance.
- Microetching (sandblasting with 50 μ alumina) of external surfaces of SynCone caps and internal surfaces of Conus Structure cavities followed by cleaning and drying is recommended.
- Place caps firmly onto abutments, and silicone sleeves over caps to prevent excess cement flowing into undercuts.

Technique Tip: The adjacent teeth and/or the external surfaces of the restoration may be lubricated with a water soluble medium to ease clean up of excess cement.

Cementation technique

Table 1

	Self-cure	Dual Cure – Non-Light Transmissible Restorations	Light Cure – Light Transmissible Restorations
Type of Restoration	All	PFM, Zirconia, Alumina, opaque ceramics, porcelain, composites	Translucent ceramics & composites
1. Fill and seat crown	Up to 2 min	Up to 2 min	Up to 2 min
Protect restoration from movement during gel phase cleanup through final set			
2. Remove Excess			
Gel stage from seating:	1-2 min	Up to 5 sec per surface ¹ light cure	Up to 5 sec per surface ¹ light cure
Gel Duration:	1 min	45 sec	45 sec
3. Stabilize	Constant Occlusal Pressure	Light cure margins 20-40 sec	Light cure margins 20-40 sec
4. Final Set	6 min from start of mix	6 min from start of mix	Upon completion of light curing each surface ² 10 sec

1. Buccal and Lingual 2. Buccal, Lingual and Occlusal

CAUTION Danger of injury due to excessive force. Apply slow and steady pressure on the syringe. Do not use excessive force – syringe rupture may result.

12.3.5. DUAL BARRELED SYRINGE DISPENSED

- Remove syringe cap. Dispense and discard a small amount of material from the dual-barreled syringe. Be sure material is flowing freely from both ports. Holding syringe vertically, carefully wipe away excess so base and catalyst do not cross contaminate and cause obstruction of the ports. Save syringe cap for replacement following use.
- Install a mixing tip on the cartridge by lining up the v-shaped notch on the outside of the mixing tip with the v-shape notch on the syringe flange. Turn colored mixing tip cap 90 degrees in a clockwise direction to lock in place on syringe.
- Gently depress syringe plungers to begin the flow of material. **DO NOT USE EXCESSIVE FORCE.** If force is encountered, remove syringe from operating field, remove and discard mixing tip. Check for obstruction and confirm material is flowing from both syringe barrels. Wipe barrels and install new mixing tip as outlined above. Dispense a small amount through the mixing tip onto a mixing pad and discard.
- Without delay, using gentle pressure, apply a thin, uniform layer of cement to the entire internal surface of the restoration directly from the mixing tip. At room temperature, Calibra® Universal Cement offers a minimum work time of 2 minutes. Technique Tip: The mixing tip may be bent slightly to allow direct intraoral access for placement of cement into preparations with internal anatomy. For endodontic post spaces, use of a Lentulo Spiral or metal file to aid placement in the post space is recommended.
- Immediately seat the restoration in the mouth. Verify complete seating. A gentle rocking or vibratory motion may be helpful to insure optimal seating.
- Protect restoration from contamination and movement until the final set of the cement (6 minutes from start of mix or in the case of light-transmissible restorations, completion of light curing).

12.4. REMOVING EXCESS CEMENT CALIBRA® UNIVERSAL

Cleaning Marginal Excess Calibra Universal Cement. Self-cure cleanup

The excess cement will reach the “gelled” state after approximately 1-2 minutes in the mouth, allowing easy removal. Excess cement will remain in the “gelled” state for approximately 1 minute. If exposed to directed operatory light, “gel” state may be reached sooner and remain “gelled” for a shorter period. Immediately after reaching the “gelled” state, floss interproximally, only in the direction of restoration seating, to remove excess cement. Complete excess cement removal using an instrument such as a rubber tip, a scaler, or an explorer. NOTE: Cement within the crown has not yet set. Do not move, torque, or disturb the crown during cleanup. Technique Tip: Following all excess removal, exposed margins may be light cured 20-40 seconds to assist restoration stabilization.

Optional dual cure cleanup

Due to the dual cure property of Calibra® Universal Cement, the operator has the option of utilizing a curing light to facilitate cleanup. Light curing to facilitate cleanup must be accomplished within the first minute following intraoral insertion. Excess cement cleanup may begin immediately following a brief exposure with the curing light. Conventional powered quartz tungsten halogen or LED lights producing only one peak wavelength around 470nm are recommended. Use of narrow spectrum output lights or high power, broad spectrum output halogen or LED lights may produce unexpected results. Light curing mixed cement after one minute, or continuously at one spot for more than 5 seconds, at any time, will cause cement to adhesively set, making cleanup difficult.

Immediately after seating is verified, briefly light cure excess cement at the margins by constantly moving the curing light tip around the margins for no more than 5 seconds per surface. Excess cement will reach a “gelled” state after this brief (< 5 second) cure.

CAUTION Residual excess cement – overcure during cleanup. Monowave (470 nm) LED or conventional halogen lights are recommended. High power, dual or broad spectrum lights may cause premature hardening of excess cement. Evaluate curing light/cement interaction in laboratory prior to clinical use.

The excess cement will remain in the “gelled” state for approximately 45 seconds following light exposure. All excess cement must be removed before final self-cure set is achieved, as outlined above. NOTE: Cement within the crown has not yet gelled or set. Do not move, torque, or disturb the crown during cleanup. Following all excess removal, exposed margins may be light cured 20-40 seconds to assist restoration stabilization.

Important Technique Tips:

- When simultaneously cementing multiple single units or bridgework, it is recommended to employ the light-cure cleanup on one or two adjacent units only, allowing other units' excess to self-cure, providing ample cleanup time.
- Ensure all excess cement has been removed from gingival sulcus and/or undercuts before final curing. Material is adhesive when set and will be difficult to remove.
- Clean excess cement from metal instruments immediately as set cement will adhere to the instrument.

12.5. CURING

• Non-Light-Transmissible restorations (Self-Cure)

For metallic, thick, or heavily opaqued ceramic or composite, or restorations that otherwise impede the transmission of light, once cleanup is completed and restoration is stabilized, allow Calibra® Universal Cement to self cure without disturbing for 6 minutes from start of mix. Protect restoration from contamination and movement during the setting time. Following the self cure set, check and adjust occlusion and polish as necessary. Patient may then be dismissed.

CAUTION Inadequate retention due to insufficient stabilization. Stabilize restoration during cleanup. Protect restoration from movement during setting. Allow cement to self-cure without disturbing for 6 minutes

• Light-Transmissible restorations (Light-Cure)

For most non-metallic, light-transmissible ceramic or composite restorations, Calibra® Universal Cement may be visible light cured. Once cleanup is completed and restoration is stabilized, light cure all areas of the restoration using a visible light, curing unit designed to cure CQ initiated methacrylates (spectral output including 470 nm), with a minimum output of 550 mW/cm² for 10 seconds from each direction – buccal, lingual, and occlusal. Following the light curing, check and adjust occlusion and polish as necessary.

CAUTION Inadequate polymerization due to insufficient curing. Check compatibility of curing light. Check curing cycle. Check curing output before each procedure

12.6. FINISHING AND POLISHING

- Removal of resin flash is best accomplished with the Enhance® Finishing System of points, cups, and discs. The Enhance System will remove flash and finish restoration margins without removal of or trauma to the enamel.
- Polish final restoration using Enhance® PoGo® Polishing system or Prisma® Gloss™ Polishing Paste and Prisma® Gloss™ Extra Fine Polishing Pastes. (See complete Directions for Use supplied with polishing product chosen).
- All shades of Calibra® Universal Cement are radio-opaque, with a 1 mm radio-opacity equivalent to 1.8 mm radio-opacity of aluminum. Aluminum has a radio-opacity equivalent to that of dentin. Thus 1 mm of material having a radio-opacity equivalent to 1 mm of aluminum has a radio-opacity equivalent to that of dentin.

12.7. HYGIENE

CAUTION Cross-contamination. Infection. Do not reuse single use products. Dispose of in accordance with local regulations. To prevent the syringes from exposure to spatter or spray of body fluids or contaminated

hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated. Syringe cannot be reprocessed. Dispose of contaminated syringe in accordance with local regulations.

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination.

Incidental contact of the syringe with water, soap, or a water-based hospital-level disinfection solution will not damage syringe body. Do not allow any solution contact with contained material. Discard composite material that has been in contact with any fluid or non-sterile instrument.

NOTE: Destruction of the label by vigorous wiping. Wipe syringe gently.

13. APPLICATION AREA

Scope - dentistry.

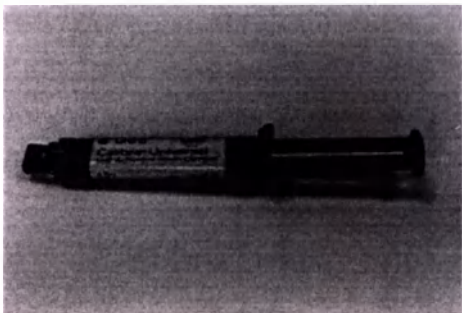
The device is intended for use by dentists and is used in dental clinics and dental laboratories.

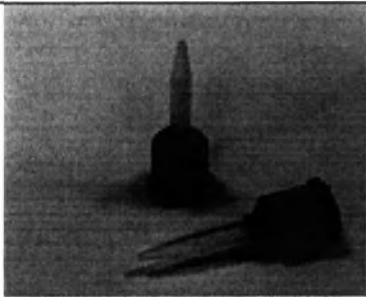
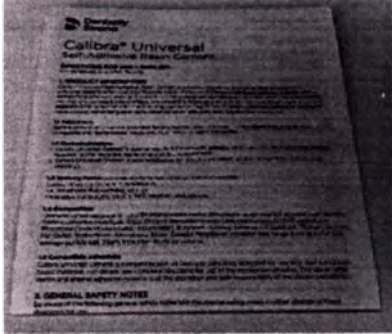
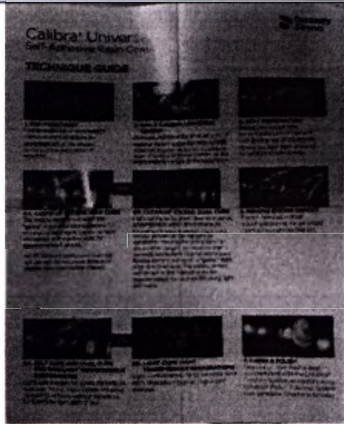
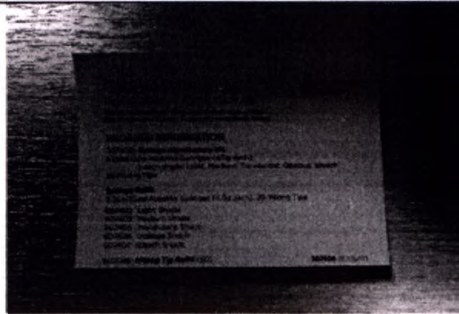
14. DESCRIPTION OF THE MEDICAL DEVICE

Dental composite cement Calibra® in design variants with accessories:

Dental composite dual cure cement Calibra® Universal

Table 1

General form	Dimensions, mm/Weight, g	Photo
Dental composite dual cure cement Calibra® Universal (Syringe Refill) in syringes 4.5 g, shades: Light Shade, Medium Shade, Translucent Shade, Opaque Shade, Bleach Shade	<u>Overall syringe size:</u> Length in open position: 125 mm $\pm$ 1 mm Length in closed position: 80 mm $\pm$ 1 mm Width: 24,9 mm $\pm$ 1 mm Height: 18,4 mm $\pm$ 1 mm Weight: 10 g $\pm$ 0,5 g Recoverable volume: 4,5 g $\pm$ 0,1 g <u>Cylinders body dimensions:</u> Length: 70 mm $\pm$ 1 mm Width: 24,9 mm $\pm$ 1 mm Height: 18,4 mm $\pm$ 1 mm <u>Cylinders plunger dimensions:</u> Diameter: 13,6 mm $\pm$ 1 mm Height: 14,5 mm $\pm$ 1 mm <u>Syringe handle dimensions:</u> Length: 67,3 mm $\pm$ 1 mm Width: 15 mm $\pm$ 1 mm Height: 18,4 mm $\pm$ 1 mm	

Mixing tips	Length: 39,7 mm ± 1 mm Width: 13,6 mm ± 1 mm Connecting part diameter: 11,8 mm ± 1 mm Outlet inner diameter: 0,8 mm ± 0,05 mm Weight: 1 g ± 0,5 g	
Instruction for use on paper and/or in electronic form	Length: 572 mm ± 5 mm Width: 355 mm ± 5 mm	
Technique guide (if necessary)	Length: 203 mm ± 2 mm Width: 165mm ± 2 mm	
Reorder information (if necessary)	Length: 79 mm ± 2 mm Width: 105 mm ± 2 mm	

15 TECHNICAL CHARACTERISTICS OF THE MEDICAL DEVICE

Dental composite dual cure cement Calibra® Universal

Table 2

Characteristic	Value ($\pm 10\%$)
Flexural strength, MPa	120
Water absorption, $\mu\text{g}/\text{mm}^3$	36
Water solubility, $\mu\text{g}/\text{mm}^3$	2,2
Working time at a temperature 23 ± 2 °C, s, no less than	90
Curing time at a temperature 23 ± 2 °C, s, no more than	360
Adhesion strength in the connection of structures with a base made of titanium, titanium alloys and zirconium oxide, MPa, no less than	15.7 (for titanium) 5 (for zirconium oxide)
Fluidity, mm	20-45
Sensitivity to ambient light, s, no less than	55
Compressive strength, MPa	336
Radiopacity	Should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 1 mm lower than the value declared by the manufacturer
The numerical value of the difference in color, denoted as ΔE , between the irradiated sample of the cured material or the sample subjected to water absorption and the control sample	$\leq 2,0$
Curing depth, mm, no less than	1
Film thickness, μm , no more than	19,6
Weight, g	4,5

16 LIST AND DESCRIPTION OF MEDICAL DEVICE MATERIALS FOR DIRECT OR SUBMITTED CONTACT WITH THE PATIENT'S BODY (HUMAN BODY)

16.1 CHEMICAL COMPOSITION

Dental composite dual cure cement Calibra® Universal has a different composition, depending on the shade. Inorganic filler particles ranging in size from 16 nm to 7 μm , the average particle size is 3.8 μm , the total filling volume is 48.7%.

Translucent Shade: EBPADMA Urethane Modified Bis-GMA Dimethacrylate resin, Urethane Dimethacrylate resin, Triethylene glycol dimethacrylate, Trimethylolpropane trimethacrylate, 2-Hydroxyethyl methacrylate, 3-(Acryloyloxy)-2-hydroxypropyl methacrylate, Benzoyl thiourea, Camphorquinone, 4-Dimethylamino Benzonitrile, Diphenyl (2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-4-methoxy benzophenone, Butylated Hydroxy Toluene, Silanated Barium Boron Fluoro alumino silicate glass, Hydrophobic amorphous fumed silica

Light Shade: EBPADMA Urethane Modified Bis-GMA Dimethacrylate resin, Urethane Dimethacrylate resin, Triethylene glycol dimethacrylate, Trimethylolpropane trimethacrylate, 2-Hydroxyethyl methacrylate, 3-(Acryloyloxy)-

2-hydroxypropyl methacrylate, Benzoyl thiourea, Camphorquinone, 4-Dimethylamino Benzonitrile, Diphenyl (2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-4-methoxy benzophenone, Butylated Hydroxy Toluene, Silanated Barium Boron Fluoro alumino silicate glass, Hydrophobic amorphous fumed silica, Colorant, Iron Oxide, Yellow, Colorant, Iron Oxide, Red, Colorant, Iron Oxide, Black, Titanium Dioxide

Medium Shade: EBPADMA Urethane Modified Bis-GMA Dimethacrylate resin, Urethane Dimethacrylate resin, Triethylene glycol dimethacrylate, Trimethylolpropane trimethacrylate, 2-Hydroxyethyl methacrylate, 3-(Acryloyloxy)-2-hydroxypropyl methacrylate, Benzoyl thiourea, Camphorquinone, 4-Dimethylamino Benzonitrile, Diphenyl (2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-4-methoxy benzophenone, Butylated Hydroxy Toluene, Silanated Barium Boron Fluoro alumino silicate glass, Hydrophobic amorphous fumed silica, Colorant, Iron Oxide, Yellow, Colorant, Iron Oxide, Red, Colorant, Iron Oxide, Black, Titanium Dioxide

Opaque Shade: EBPADMA Urethane Modified Bis-GMA Dimethacrylate resin, Urethane Dimethacrylate resin, Triethylene glycol dimethacrylate, Trimethylolpropane trimethacrylate, 2-Hydroxyethyl methacrylate, 3-(Acryloyloxy)-2-hydroxypropyl methacrylate, Benzoyl thiourea, Camphorquinone, 4-Dimethylamino Benzonitrile, Diphenyl (2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-4-methoxy benzophenone, Butylated Hydroxy Toluene, Silanated Barium Boron Fluoro alumino silicate glass, Hydrophobic amorphous fumed silica, Colorant, Iron Oxide, Yellow, Colorant, Iron Oxide, Red, Titanium Dioxide

Bleach Shade: EBPADMA Urethane Modified Bis-GMA Dimethacrylate resin, Urethane Dimethacrylate resin, Triethylene glycol dimethacrylate, Trimethylolpropane trimethacrylate, 2-Hydroxyethyl methacrylate, 3-(Acryloyloxy)-2-hydroxypropyl methacrylate, Benzoyl thiourea, Camphorquinone, 4-Dimethylamino Benzonitrile, Diphenyl (2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-4-methoxy benzophenone, Butylated Hydroxy Toluene, Silanated Barium Boron Fluoro alumino silicate glass, Hydrophobic amorphous fumed silica, Titanium Dioxide

16.2 SYRINGE MATERIALS

Polypropylene, low density polyethylene, pigment

16.3 MIXING TIPS MATERIALS

Natural polypropylene, Brown polypropylene

16.4 CONSUMERS PACKAGING MATERIALS

Low density polyethylene, solid bleached sulfate

17. ENVIRONMENTAL REQUIREMENTS

The medical device "Dental composite cement Calibra® in design variants with accessories" manufactured by Dentsply LLC also dba Dentsply Caulk, is safe for the environment if the rules for its use are observed

18. METHODS AND MEANS OF DISINFECTION AND PRE-STERILIZATION CLEANING

The product is non-sterile, intended for single use.

19. TRANSPORTATION, STORAGE AND OPERATION

Operating conditions

Temperature from +5°C to +42 °C

Humidity up to 98% relative humidity at temperature 25°C

Atmosphere pressure from 70 kPa to 106 kPa

Products are properly operated inside the oral cavity, in the presence of biological fluids in it at nominal temperatures: from +32°C to +42°C, for accessories not used in the oral cavity: from + 10°C to + 35°C.

Storage conditions

Temperature from +2°C to +24 °C

Humidity No more than 70%

Atmosphere pressure from 70 kPa to 106 kPa

Transportation conditions

Temperature from +5°C to +30°C

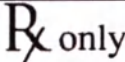
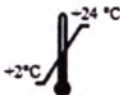
Humidity No more than 70%

Atmosphere pressure from 70 kPa to 106 kPa

20. PACKAGING AND LABELING

Dental dual cure composite cement Calibra® Universal delivers in consumer box

Table 3. Symbols used on the packaging

Symbol	Description
	Directive 93/42/EEC compliance symbol
	For dental use only
	Information about the manufacturer
	Storage temperature
	Avoid exposure to sunlight
	Caution! Refer to instructions for use
	The need to read the instructions for use / instructions in electronic form
	Keep away from moisture
	Reuse prohibition
	Authorized Representative in the European Community

	Green dot sign
---	----------------

21. STORAGE AND SHELF LIFE

Dental dual cure composite cement Calibra® Universal must be stored at a temperature of +2°C to +24°C

- Do not use dental dual cure composite cement Calibra® Universal after the stated expiration date.
- Do not disinfect syringes with oxidizing disinfectants.
- Leave the mixing tips on the syringe after use to seal it.

Keep out of the reach of children.

The shelf life of the kits is 2 years from the date of production

22. PROCEDURE FOR DISPOSAL AND DESTRUCTION

This product belongs to waste risk class B and must be disposed of in accordance with the Sanitary and Epidemiological Requirements for the Treatment of Medical Waste 2.1.3684-21.

Disposal must be carried out at a designated landfill or incinerator in accordance with regulations approved by local authorities. Do not dispose of with normal household waste.

23. WARRANTY

The medical device has been designed for dental use only. The application must be carried out in strict accordance with the "Instructions for use." The manufacturer accepts no liability for damage resulting from non-compliance with the instructions or the prescribed area of application. It is the user's responsibility to test the product for suitability and suitability for purposes other than those expressly stated in the instructions.

On the territory of the Russian Federation, claims for warranty and product quality are sent to the authorized representative of the manufacturer: Dentsply Sirona LLC, premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035.

24. REQUIREMENTS FOR THE MAINTENANCE AND REPAIR OF A MEDICAL DEVICE

Service and maintenance are not required. The medical device cannot be repaired.

25. ADVERTISEMENT

An organization authorized by the manufacturer to accept claims for product quality in the territory of the Russian Federation:

Dentsply Sirona LLC

Address: premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035. Tel.: +7 (495) 725 10 87

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М.П. / Stamp

Instruction for Use

Dental composite cement Calibra® in design variants with accessories
Dental composite cement Calibra® VeneerSworn to and subscribed before me
this 7th day of September, 2023.

Dawn Renea Kenton, Notary Public

2023

1. INTRODUCTION

ATTENTION! To ensure safety when using Dental composite cement Calibra® in design variants with accessories as intended, all users should carefully read this Instructions for Use prior to using the product. The instructions for use are written to familiarize the consumer with the design of the medical device, its functions, and also describe the conditions and rules of operation. Instructions for use are intended for medical professionals.

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3. AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

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E-Mail: CIS-Info@dentsplysirona.com

4. NAME OF THE MEDICAL DEVICE

Dental composite cement Calibra® in design variants with accessories (Further in the text "cement," "material," "device"): Dental composite cement Calibra® Veneer

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- porcelain, ceramic, composite translucent veneers.

7. CONTRAINDICATIONS

Dental composite cement Calibra® is contraindicated:

- for use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components.
- for direct application to dental pulp tissue (direct pulp capping).

8. ADVERSE REACTIONS

Product may irritate the eyes and skin. Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucous membranes: inflammation (see Warnings).

Product may cause pulpal effects. (See Contraindications)

Residual hardened excess cement may lead to soft tissue injury or irritation. (See Interactions and Step-by-Step instructions).

9. CONTENT OF THE MEDICAL DEVICE

Dental composite cement Calibra® in design variants with accessories

Dental composite cement Calibra® Veneer (Syringe Refill) in syringes 2 g, shades: Light Shade, Medium Shade, Translucent Shade, Opaque Shade, Bleach Shade, consisting of:

- 1) Dental composite cement Calibra® Veneer (Syringe Refill) in syringes 2 g of one shade – 1 pce;
- 2) Instructions for use on paper and/or in electronic form – 1 pce;

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11. WARNINGS AND PRECAUTIONS

Warnings

1. Calibra Veneer Cement contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
 - Avoid Eye Contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
 - Avoid Skin Contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and alcohol and wash well thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
 - Avoid Contact with Oral Soft Tissues/Mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
2. The bonding agent Calibra Silane contains acetone. Do not breathe vapours.

Precautions

- This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
- Insufficient data exists to support the use of Calibra Veneer Esthetic Resin Cement as a core build up, filling material, base, or liner.

- Insufficient data exists to support the use of Xeno® Self-Etch Adhesives (all versions) with Calibra Veneer Esthetic Resin Cement for veneer cementation light cured.

- Wear suitable protective eyewear, clothing, and gloves. Protective eyewear is recommended for patients.
- Contact with saliva, blood, and/or some astringent solutions during adhesive procedures may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.

- Devices marked “single use” on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

- The syringe cannot be reprocessed. To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the syringe is handled with clean/disinfected gloves. As an additional precautionary measure, the syringe may be protected from gross contamination but not from all contamination by applying a protective barrier.

- Containers should be tightly closed immediately after use. The syringe should be tightly closed by replacing the original cap immediately after use.

- Calibra Veneer Cement should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may result in unanticipated extrusion of the material or cause syringe rupture.

- Use only in well ventilated areas.

- Flammable: Calibra Silane Coupling Agent contains acetone. Keep away from sources of ignition.

INTERACTIONS

- Eugenol containing materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.

- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining, and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

- The Calibra Veneer Cement is light cured material; therefore, it should be protected from ambient light. Proceed immediately once the material has been placed.

12. STEP-BY-STEP INSTRUCTIONS

- Following removal of the temporary restoration and any remaining temporary cement, clean enamel and dentin as directed by the adhesive manufacturer’s directions.

- Check the fit and esthetics of the restoration. Technique Tip: Occlusal adjustment of veneers is best accomplished after final cementation.

Try-in Paste (optional)

- Dispense the appropriate shade of try-in paste from the syringe onto a clean mixing pad or glass slab. Load paste onto internal surfaces of the restoration and gently seat onto preparation. Clean excess with a cotton pellet and/or blunt explorer. Shades may be blended to achieve optimum esthetics. Important Technique Tip: Factors such as restoration shape, texture, thickness, and shade(s) of the underlying and surrounding tooth structure contribute to final perceived shade. Try-in paste is a guide for cement shade range selection only. NOTE: The try-in paste will not polymerize, thus offers unlimited work time.

- Once fit and esthetics are verified, thoroughly rinse try-in paste from restoration and preparation surfaces using water.

12.1. TREATMENT OF CERAMIC/COMPOSITE RESTORATIONS

Follow the dental laboratory or restoration manufacturer's instructions for pre-treatment, if required. Restorations designed to be silanated or if the internal silanated surface has been disturbed during try-in, apply Calibra Silane Coupling Agent as outlined below:

- Prior to applying the silane, clean the internal surface of the restoration with Caulk® 34% Tooth Conditioner Gel (34% phosphoric acid) (see complete Directions for Use). Apply for 30 seconds and rinse thoroughly with water for 20 seconds. Air-dry the internal surface of the restoration with clean, dry air from a dental syringe.

CAUTION Danger of injury due to excessive force. Apply slow and steady pressure on the syringe plunger. Do not use excessive force – syringe rupture may result

- Attach supplied needle tip to end of the Calibra Silane Coupler syringe. Gently apply pressure to syringe plunger. **DO NOT USE EXCESSIVE FORCE.** Silane agent should express easily, one drop at a time. Directly apply the silane to the clean internal surface of the restoration. Air-dry the silanated surface of the restoration with clean, dry air from a dental syringe or allow to air dry. **NOTE:** Avoid application of the silane to the external surface of the restoration. Contact with the external surface will cause bonding of the cement to the external surface, complicating clean-up and necessitating removal of the glazed portion of the external surface of the restoration.

- If silanated surface becomes contaminated, clean surface with alcohol or acetone, allow to air dry, and repeat application as outlined above.

12.2. TOOTH CONDITIONING/DENTIN PRETREATMENT/ADHESIVE APPLICATION

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CAUTION Delayed or reduced adhesion due to adhesive incompatibility. Select compatible light cure adhesive. Follow selected adhesive directions for use.

- Always refer to and follow adhesive manufacturer's complete directions for use for selection and application of the appropriate light cure adhesive. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately following adhesive application to cementation with Calibra Veneer Cement, light cured base as indicated in the appropriate section below.

12.3. VENEER CEMENTATION – LIGHT CURED

Before proceeding, refer to above sections for Treatment of the Restoration and Tooth Conditioning/Dentin Pretreatment/Adhesive Application.

- Adhesives compatible with light cured, CQ initiated methacrylates are recommended.
- If veneer restorations exceed 1.0mm thickness, are heavily opaqued, or otherwise impede the transmission of light, operator is urged to select Calibra Esthetic Resin Cement and follow the Dual Cured cementation technique, mixing base with catalyst (available separately).
- Apply adhesive to the internal bonding surface of the restoration if directed by adhesive manufacturer's Directions for Use.

12.4. CEMENTATION TECHNIQUE

Dispense the desired shade of Calibra Veneer Esthetic Resin Cement base paste from the syringe directly onto the veneer. Protect cement from exposure to light.

- Seat the loaded veneer in place. Remove excess from the gingival margin with a blunt instrument. Briefly (10 seconds or less) light cure the gingival portion only to tack restoration in place. Remove any excess from proximal and lingual margins. Technique Tip: Placement of mylar strips between preparation and adjacent teeth prior to seating veneer aids in isolation and in excess cement clean up. After removing gingival excess and tacking veneer in place, remove interproximal excess cement by pulling mylar strip towards the facial, i.e., from tooth to restoration surface.

12.5. CURING

CAUTION Inadequate polymerization due to insufficient curing. Check compatibility of curing light. Check curing cycle. Check curing output before each procedure.

- Using a visible light* curing unit designed to cure CQ initiated methacrylates (spectral output including 470nm), with a minimum output of 550mW/cm², light cure 20 seconds each from the buccal, lingual, and interproximal aspects.

- Following the light cure check and adjust occlusion as necessary. Proceed to Finishing and Polishing section.

- * Refer to curing light manufacturer's recommendations for compatibility and curing recommendations.

12.6. FINISHING AND POLISHING

- Removal of resin flash is best accomplished with the Enhance® Finishing System of points, cups, and discs. The Enhance System will remove flash and finish restoration margins without removal of or trauma to the enamel.

- Polish final restoration using Enhance® PoGo® Polishing system or Prisma® Gloss™ Polishing Paste and Prisma Gloss Extra Fine Polishing Paste. (See complete Directions for Use supplied with polishing product chosen).

All shades of Calibra Veneer Cement are radio-opaque, with a 1mm radiopacity equivalent to 1.77 mm radiopacity of aluminum. Aluminum has a radiopacity equivalent to that of dentin. Thus 1 mm of material having a radiopacity equivalent to 1mm of aluminum has a radiopacity equivalent to that of dentin.

12.7. HYGIENE

CAUTION! Cross-contamination.

- Do not reuse single use products. Dispose in accordance with local regulations.

- Syringe cannot be reprocessed. Dispose of contaminated syringe in accordance with local regulations.

12.8. CLEANING AND DISINFECTION

- To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid syringe contamination. The use of protective barriers is an additional precautionary measure against gross contamination but not against all contamination.

- Handling with clean/disinfected gloves, syringes may be disinfected with a water-based hospital-level disinfection solution according to national/local regulations.

- Repeated disinfection may damage label. **NOTE:** Destruction of the label by vigorous wiping. Wipe syringe gently.

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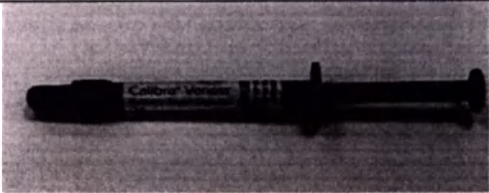
Dental composite cement Calibra® Veneer

Table 1

General form	Dimensions, mm/Weight, g	Photo
Dental composite cement Calibra® Veneer (Syringe Refill) in syringes 2 g, shades: Light Shade, Medium Shade, Translucent Shade, Opaque Shade, Bleach Shade	<u>Overall syringe size:</u> Length in open position: 130 mm $\pm$ 1 mm Length in closed position: 75,5 mm $\pm$ 1 mm Width: 12,8 mm $\pm$ 1 mm Height: 17,6 mm $\pm$ 1 mm Weight: 5 g $\pm$ 0,5 g Recoverable volume: 2 g $\pm$ 0,1 g <u>Cylinders body dimensions:</u> Length: 65 mm $\pm$ 1 mm Width: 12,8 mm $\pm$ 1 mm Height: 17,6 mm $\pm$ 1 mm <u>Cylinders plunger dimensions with Luer-lock:</u> Diameter: 9,4 mm $\pm$ 1 mm Height: 19,5 mm $\pm$ 1 mm <u>Plunger handle dimensions:</u> Length: 71,4 mm $\pm$ 1 mm Diameter: 12 mm $\pm$ 1 mm	
Instruction for use on paper and/or in electronic form	Length: 572 mm $\pm$ 5 mm Width: 355 mm $\pm$ 5 mm	

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Table 2

Characteristic	Value ($\pm 10\%$)
Flexural strength, MPa	101
Water absorption, $\mu\text{g}/\text{mm}^3$	15
Water solubility, $\mu\text{g}/\text{mm}^3$	<5
Working time, s	165-360
Polymerization time with min power 550 mW/cm ² , s	20
Curing depth, mm, no less than	1
Adhesion strength in the connection of structures with a base made of titanium, titanium alloys and zirconium oxide, MPa, no less than	$\geq 13,5$
Film thickness, μm , no more than	18,1
Fluidity, mm	22-28
Compressive strength, MPa	272
Sensitivity to ambient light, s, no less than	55
Radiopacity	Should be equal to or greater than the radiopacity of an aluminum sample of the same thickness and not less than 1,77 mm lower than the value declared by the manufacturer
The numerical value of the difference in color, denoted as ΔE , between the irradiated sample of the cured material or the sample subjected to water absorption and the control sample	$\leq 2,0$
Weight, g	2

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16.1 CHEMICAL COMPOSITION

Dental composite cement Calibra® Veneer has a different composition, depending on the shade. Particles of inorganic filler with a size of 0.02 to 1.3 microns, the total filling volume is 44.9%.

Translucent Shade: Bis-GMA, Ethoxylated Bisphenol A Dimethacrylate, Triethylene Glycol Dimethacrylate, 2-Hydroxy 4- methoxy benzophenone, p-Tolyl Diethanolamine, Ethyl-4-Dimethylamino Benzoate,

Camphorquinone, Butylated Hydroxy Toluene, Silanated BariumBoronFluoroAluminoSilicate, Hydrophobic Silicon Dioxide

Light Shade: Bis-GMA, Ethoxylated Bisphenol A Dimethacrylate, Triethylene Glycol Dimethacrylate, 2-Hydroxy 4- methoxy benzophenone, p-Tolyl Diethanolamine, Ethyl-4-Dimethylamino Benzoate, Camphorquinone, Butylated Hydroxy Toluene, Silanated BariumBoronFluoroAluminoSilicate, Hydrophobic Silicon Dioxide, Yellow Iron Oxide, Red Iron Oxide, Black Iron Oxide.

Medium Shade: Bis-GMA, Ethoxylated Bisphenol A Dimethacrylate, Triethylene Glycol Dimethacrylate, 2-Hydroxy 4- methoxy benzophenone, p-Tolyl Diethanolamine, Ethyl-4-Dimethylamino Benzoate, Camphorquinone, Butylated Hydroxy Toluene, Silanated BariumBoronFluoroAluminoSilicate, Hydrophobic Silicon Dioxide, Yellow Iron Oxide, Red Iron Oxide, Black Iron Oxide

Opaque Shade: Bis-GMA, Ethoxylated Bisphenol A Dimethacrylate, Triethylene Glycol Dimethacrylate, 2-Hydroxy 4- methoxy benzophenone, p-Tolyl Diethanolamine, Ethyl-4-Dimethylamino Benzoate, Camphorquinone, Butylated Hydroxy Toluene, Silanated BariumBoronFluoroAluminoSilicate, Hydrophobic Silicon Dioxide, Titanium Dioxide, Yellow Iron Oxide, Red Iron Oxide, Black Iron Oxide

Bleach Shade: Bis-GMA, Ethoxylated Bisphenol A Dimethacrylate, Triethylene Glycol Dimethacrylate, 2-Hydroxy 4- methoxy benzophenone, p-Tolyl Diethanolamine, Ethyl-4-Dimethylamino Benzoate, Camphorquinone, Butylated Hydroxy Toluene, Silanated BariumBoronFluoroAluminoSilicate, Hydrophobic Silicon Dioxide, Titanium Dioxide

16.2 SYRINGE MATERIALS:

Polypropylene, polipropylene copolymer, pigment

16.3 CONSUMERS PACKAGING MATERIALS

Low density polyethylene, Solid Bleached Sulfate

17. ENVIRONMENTAL REQUIREMENTS

The medical device " Dental composite cement Calibra® in design variants with accessories " manufactured by Dentsply LLC also dba Dentsply Caulk, is safe for the environment if the rules for its use are observed.

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The product is non-sterile, intended for single use

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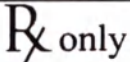
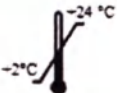
20. PACKAGING AND LABELING

Dental composite cement Calibra® Veneer delivers in consumer box.

Table 3. Symbols used on the packaging

Symbol	Description
	Directive 93/42/EEC compliance symbol
	For dental use only
	Information about the manufacturer
	Storage temperature
	Avoid exposure to sunlight
	Caution! Refer to instructions for use
	The need to read the instructions for use / instructions in electronic form
	Keep away from moisture
	Reuse prohibition
	Authorized Representative in the European Community
	Medical device
	Best before date

21. STORAGE AND SHELF LIFE

Dental composite cement Calibra® Veneer must be stored at a temperature of +2°C to +24°C

- Do not use Dental composite cement Calibra® Veneer after the stated expiration date.
- Do not disinfect syringes with oxidizing disinfectants.
- Leave the mixing tips on the syringe after use to seal it.

Keep out of the reach of children.

The shelf life of the kits is 2 years from the date of production.

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On the territory of the Russian Federation, claims for warranty and product quality are sent to the authorized representative of the manufacturer: Dentsply Sirona LLC, 115432, Russia, Moscow, Andropov prospect, 18, bldg. 6, offices 1 39 40.

24. REQUIREMENTS FOR THE MAINTENANCE AND REPAIR OF A MEDICAL DEVICE.

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Dentsply Sirona LLC

Address: premises 3N, bld.2, Ovchinnikovskaya embankment, 18/1, ZamoscvoRechie municipal district, intra-city area, Moscow, 115035.

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