JD

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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade & China Chamber of International Commerce

01455766

中国国际贸易促进委员会
China Council for the Promotion of International Trade
China Chamber of International Commerce

证明书
CERTIFICATE

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THIS IS TO CERTIFY THAT: the seal of Taizhou Kangjian MEDICAL Equipment Co., Ltd. on the annexed DOCUMENT is genuine.

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APPROVED
Taizhou Kangjian Medical Equipments Co., Ltd.

The Machine Electricity Zone (Hang Ni Kant)
of Yuhuan County, 317600 Zhejiang,
P.R. China

General Director

Lan Hepin

03.12.2022

INSTRUCTION FOR USE

for medical device

Non-rebreathing face oxygen mask, trade mark Yuki group

According to the remarks, specified in the Notification of the Federal Service for Surveillance in
Healthcare No. 10-75335/22 dated 29.11.2022 on the provision of the materials and data

2022

Contents

1. Name of medical device	5
2. Purpose of medical device and functional characteristics	5
3. Conditions/ scope of use	5
4. Indications for use	5
5. Contraindications	5
6. Side effects	5
7. Potential user	5
8. Warnings and precautionary measures	5
9. MODE OF APPLICATION	5
10. Description of the device	6
11. CLASSIFICATION OF MEDICAL PRODUCT	6
12. Technical description of medical device	6
13. Care and maintenance	8
14. Contact with human organism	8
15. Description of raw materials, materials and component parts	8
16. List of materials of animal and/or human origin	8
17. Data of state register of pharmaceuticals	8
18. Packaging	8
19. Scope of supply	8
20. Labeling	9
Warnings and precautionary measures	8
21. Warranty	12
22. Sterility	12
23. Conditions of operation, storage and transportation	12
24. SHELF LIFE	12
25. Disposal	12
26. Protection of environment	12
27. List of international standards to which medical device corresponds	12
28. Organization – manufacturer	13
29. Authorized representative of the manufacturer	13

Name of medical device

Non-rebreathing face oxygen mask, trade mark Yuki group

1. Oxygen face mask, Non-rebreathing, size XL, consisting of:
 - mask – 1 pcs;
 - instruction for use.
2. Oxygen face mask, Non-rebreathing, size L, consisting of:
 - mask – 1 pcs;
 - instruction for use.
3. Oxygen face mask, Non-rebreathing, size M, consisting of:
 - mask – 1 pcs;
 - instruction for use.
4. Oxygen face mask, Non-rebreathing, size S, consisting of:
 - mask – 1 pcs;
 - instruction for use.

2. Purpose of medical device and functional characteristics

To deliver air with a high concentration of oxygen (O₂) for patients with dyspnea or hypoxia.

3. Conditions/ scope of use

Used in hospitals and clinics.

4. Indications for use

For oxygen therapy of patients with dyspnea or hypoxia.

5. Contraindications

There are no clinical contraindications.

6. Side effects

Not detected.

7. Potential user

Face oxygen masks are intended for use by professionals such as doctors or nurses in clinics or hospitals.

8. Warnings and precautionary measures

- Single-use. Do not reuse.
- Do not use if packaging is opened or damaged.
- Do not store at extreme temperature and humidity. Store in dry and cool place.
- Make sure that all connections are reliable and air freely circulates in the tube.

9. MODE OF APPLICATION

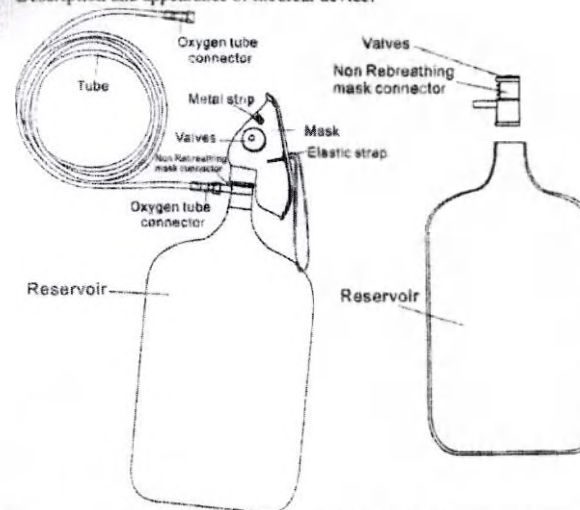
1. Connect the oxygen supply tube to the gas source.

Set the desired oxygen flow.

3. Check the gas flow through the device. Always check that the valves are working properly. The valves between the mask and the reservoir should rise on inspiration and down on expiration.
4. Place the mask on the patient's face using the elastic strap until it locks in place.
5. For a tighter fit, shape the metal strip on the mask according to the shape and size of the patient's nose.

10. Description of the device

Description and appearance of medical device:

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
A Non-rebreathing mask consists of a mask, reservoir and oxygen tube. Mask with metal band and elastic strap, oxygen tube consists of oxygen tube connector and a tube.

11. CLASSIFICATION OF MEDICAL PRODUCT

Depending on potential risk of use of the device, in accordance to the requirements of the Order of the Ministry of Healthcare of Russia No.4n of 06.06.2012 "On approval of nomenclature classification of medical devices", classified as class 2a.

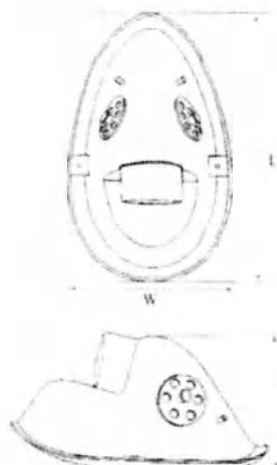12. Technical description of medical device

Specification	Main size (mm)			Oxygen tube			Резервуар
	L	In	W	Length (with connector)	External diameter	Internal diameter	Capacity
Extra large (XL)	145±5	68±3	83±3	2000±50	6±0.5	4±0.5	1000±100
Large (L)	110±5	56±3	77±3				
Medium (M)	100±5	48±3	70±3				
Small (S)	90±5	52±3	63±3				

No.	Characteristic	Requirements
1	Appearance	Surface of the Non-rebreathing mask must be smooth, soft and medium hard, free of burrs, dirt, impurities and kinks. Face parts of mask must be smooth, without burrs and rough edges. There should be no foreign bodies on the outside of the tube, the surface of the tube should be smooth, and there should be a small amount of wall crystals. The upper part of the tube should be round, free from burrs and defects. Cavity of the pipe must be clean and give free access.
2	Reservoir size	250*143mm
3	Mass of mask	26±3 g (XL), 18±3 g (L), 16±3 g (M), 12±3 g (S)
4	Mass of pipe	30±3 g
5	Mass of reservoir	76±2 g
6	Dimensions of oxygen tube connector	Length: 35.5 mm ± 0.5 mm External diameter: 8 mm ± 0.5 mm Internal diameter: 5.5 mm ± 0.5 mm Tapering: 1:1.86
7	Strength characteristics	Each connector must not break or weaken under the influence of a static voltage of 15 N for 15 s. The minimum breaking force of the oxygen tube must be at least 35 N.

Care and maintenance

Devices have completely finished structure and do not demand any maintenance and repair during the process of operation. Devices are single-use.

14. Contact with human organism

Long-term (less than 7 days) contact with skin.

15. Description of raw materials, materials and component parts

Material	Grade	Manufacturer
Polyvinyl chloride	TEAM	«Jiangsu Yabang Teamlong Polymer Co., Ltd.» (China)
Polypropylene	LONG	«Jiangsu Yabang Teamlong Polymer Co., Ltd.» (China)

16. List of materials of animal and/or human origin

Does not contain materials of animal and/or human origin.

17. Data of state register of pharmaceuticals

This medical device does not contain pharmaceuticals and pharmaceutical substances.

18. Packaging

Name	Material	Dimension
Transport packaging	Corrugated paper	490 x 360 x 300 ± 5 mm
Individual packaging	Polyethylene film grade RS, manufactured by company «Jiangsu Yabang Teamlong Polymer Co., Ltd.» (China)	210 x 170 mm ± 3 mm
Label	100 g Art paper	135 x 95 ± 3 mm

19. Scope of supply

Non-rebreathing face oxygen mask, trade mark Yuki group

- Oxygen face mask, Non-rebreathing, size XL, consisting of:
 - mask - 1 pcs;
 - instruction for use.
- Oxygen face mask, Non-rebreathing, size L, consisting of:
 - mask - 1 pcs;
 - instruction for use.
- Oxygen face mask, Non-rebreathing, size M, consisting of:
 - mask - 1 pcs;
 - instruction for use.

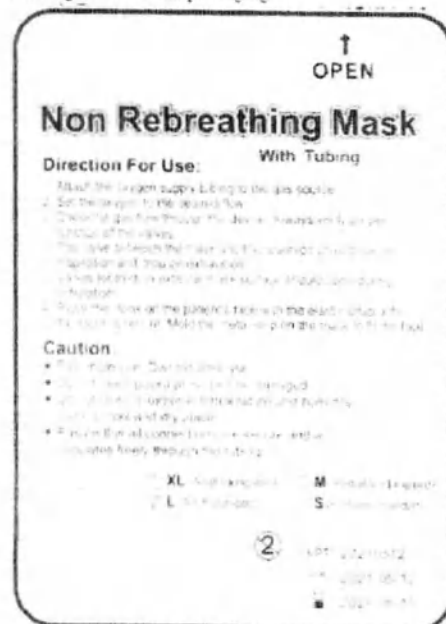
Oxygen face mask, Non-rebreathing, size S, consisting of:
 - 1 pos.
 - instruction for use.

20. Labeling

Labeling of individual packaging includes:

- name and address of manufacturer;
- name of device;
- dimension of device;
- method of application;
- warnings and precautionary measures;
- lot number;
- date of manufacture;
- shelf life;
- symbol "Single-use";
- CE marking.

Labeling of individual packaging:

Label of labeling of individual packaging in Russian language:

Face Non-rebreathing oxygen mask, trade mark Yuki group
 Size: XL

Method of application

- 1 Connect the oxygen supply tube to the gas source.
- 2 Set the desired oxygen flow.
- 3 Check the gas flow through the device. Always check that the valves are working properly. The valves between the mask and the reservoir should rise on inspiration and down on expiration.
- 4 Place the mask on the patient's face using the elastic strap until it locks in place.
- 5 For a tighter fit, shape the metal strip on the mask according to the shape and size of the patient's nose.

Warnings and precautionary measures

- Single-use. Do not reuse.
- Do not use if packaging is opened or damaged.
- Do not store at extreme temperature and humidity. Store in dry and cool place.
- Make sure that all connections are reliable and air freely circulates in the tube

Taizhou Kangjian Medical Equipments Co., Ltd.
 The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County,
 Zhejiang Province 317600, China
 Tel.: 0086-576-87299799 Fax: 0086-576-87239799
 E-mail: webmaster@kangjian.com

STERILE

LOT XXXXXXXX
 XXXXXXXX
 XXXXXXXX

Authorized representative:
 "ARMEDICA" LLC
 Shevchenko street, house 20, premises 7, the city of Yekaterinburg,
 Sverdlovsk region, 620075
 Tel: 8(343) 286 42 73, 8(343) 286 42 74
 E-mail: org@yukigroup.com

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Taizhou Kangjian Medical Equipments Co., Ltd.

The Machine Electricity Zone (Hang Ni Kan) of Yuhuan

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XXXXXX

CE 0197

Labeling of transport packaging includes:

- name and address of manufacturer;
- name and address of authorized representative;
- name of device;
- dimension of device;
- numbers according to the manufacturer's system;

- number of devices (pcs);
- weight of packaging (net/ gross);
- symbol "Single-use";
- CE marking;
- Symbol "Top";
- Symbol: "Warning, fragile";
- Symbol "Protect from moisture";
- Symbol "Avoid sunlight"

Labeling of transport packaging

NON REBREATHING MASK

With Tubing

- ☐ XL (Adult Elongated) ☐ M (Pediatric Elongated)
☐ L (Adult Standard) ☐ S (Pediatric Standard)

LOT: 20210512
MFG: 2021-05-12
EXP: 2026-05-11
QTY: 100 PCS
G.W: 8.55 KGS
N.W: 7.65 KGS
MEAS: 50X36X35 CM

Model of labeling of transport packaging in Russian language:

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LOT:
MFG:
EXP:
QTY: 100 PCS
G.W: 7.65 KGS
N.W: 6.7 KGS
MEAS: 50X36X34 CM

Sterile EO CE0197

**Face Non-rebreathing oxygen mask, trade mark Yuki group
Size: XL**

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LOT:
MFG:
EXP:
QTY: 100 PCS
G.W: 7.65 KGS
N.W: 6.7 KGS
MEAS: 50X36X34 CM

Street, house 20, premises 7, the city of Yekaterinburg,
Region, 620075
Tel: 286 42 73, 8(343) 286 42 74
E-mail: yekigroup.com

No. _____ of _____

Symbol	Value
	Single-use
	Batch code
	Use before...
	Date of manufacture
	Warning! Refer to the instructions for use
	Sterilization with ethylene oxide
	Manufacturer
	Authorized representative in European Community
	Do not use if packaging is damaged
	Top
	Warning, fragile
	Keep away from moisture
	Do not expose to sunlight

197

CE marking

Warranty

The manufacturer guarantees that the devices were made in accordance with its technical characteristics, applied industry standards and normative requirements.
Warranty period is 5 years.

22. Sterility

Sterile medical device. Sterilization with ethylene oxide (EO)

23. Conditions of operation, storage and transportation

Subject to operation at temperature from -5°C to +35°C and relative air humidity to 95% at 25°C.
Face oxygen masks are stored at temperature from -20°C to +40°C and relative humidity 80% at 25°C.

It is permitted to transport face oxygen masks by any kind of transport of closed type with observance of transportation rules established for this kind of transport, at the following environmental conditions – temperature from -20 to +35°C, humidity not more than 85% at 25°C.

24. SHELF LIFE

Shelf life is 5 years.

25. Disposal

Face oxygen masks with expired shelf life shall be disposed in accordance with the requirements of SanPiN 2.1.3684-21, as waste of Class A.

Face oxygen masks after use shall be disposed in accordance with the requirements of SanPiN 2.1.3684-21, as waste of Class B.

26. Protection of environment

Face oxygen masks in use, transportation and storage do not have a negative impact on human and the environment.

27. List of international standards to which medical device corresponds

No.	Doc. No.	Standard name
1	ISO 11135-1: 2014	Sterilization of health care products. Ethylene oxide. Requirements for development, validation and routine control of a sterilization process for medical devices.
2	EN 556-1: 2001	Sterilization of medical devices. Requirements for medical devices to be designated "Sterile" – Part 1. Requirements for terminally sterilized medical devices.
3	ISO 14971: 2019	Medical devices. Application of risk management to medical devices.
4	EN 980: 2016	Symbols for use in labeling of medical devices.
5	EN 1041: 2016	Information supplied by the manufacturer of medical devices.
6	ISO 11607-1: 2019	Packaging for terminally sterilized medical devices – Part 1. Requirements for materials, sterile barrier systems and packaging systems.

ISO 13485:2016	Packaging for terminally sterilized medical devices
ISO 14644-1:2015	Clean rooms and associated controlled environments Part 1: Classification of air cleanliness by particle concentration
ISO 14644-2:2015	Clean rooms and associated controlled environments Part 2: Monitoring to provide evidence of clean room performance related to air cleanliness by particle concentration
ISO 11737-1:2018	Sterilization of health care products Microbiological methods Part 1: Determination of a population of microorganisms on products
ISO 11737-2:2019	Sterilization of health care products Microbiological methods Part 2: Tests of sterility performed in the determination, validation and maintaining of a sterilization process
12 ISO 10993-1:2018	Biological evaluation of medical devices Part 1: Evaluation and testing with a risk management process
13 ISO 10993-5:2016	Biological evaluation of medical devices Part 5: Tests for in vitro cytotoxicity
14 ISO 10993-7:2016	Biological evaluation of medical devices Part 7: Ethylene oxide sterilization residuals
15 ISO 10993-10:2016	Biological evaluation of medical devices Part 10: Tests for skin sensitization
16 MEDDEV 2.7/1 version 4 (2016)	Evaluation of clinical data: a guide for manufacturers and notified bodies under directives 90/269/EEC and 90/269/EEC
17 Q/TKME HN001-2016	Single-use oxygen mask

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28. Organization – manufacturer

Taizhou Kangjian Medical Equipments Co., Ltd.

The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, 317600 Zhejiang, P.R. China

Tel: 0086-576-87299799

Fax: 0086-576-87239799

E-mail: webmaster@kangjian.com

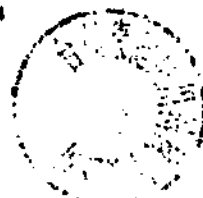
29. Authorized representative of the manufacturer

Limited liability company "ARMEDICA" ("ARMEDICA" LLC)

Snatchevskaya street, house 20, premises 7, Sverdlovsk region, 620075

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