20

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CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

01455767

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of Taizhou Kangjian MEDICAL Equipment Co., Ltd. on the annexed DOCUMENT is genuine.

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APPROVED
Taizhou Kangnan Medical Equipment Co., Ltd.

The Machine Electricity Zone (Hang Ni Kan)
of Yuhuan County, 317600 Zhenjiang,
P.R. China

General Director

Lan Hepin

30.11.2022

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INSTRUCTION FOR USE

for medical device

Nasal oxygen cannulas, trade mark Yuki group

According to the remarks, specified in the Notification of the Federal Service for Surveillance in Healthcare No. 70-7/G85/22 dated 22.11.2022 on the provision of the materials and data

Contents

Name of medical device.....	5
Purpose of medical device and functional characteristics.....	5
3. Conditions/ scope of use.....	5
4. Indications for use.....	5
5. Contraindications.....	5
6. Side effects.....	6
7. Potential user.....	6
8. Warnings and precautionary measures.....	6
9. MODE OF APPLICATION.....	6
10. Description of the device.....	6
11. CLASSIFICATION OF MEDICAL PRODUCT.....	7
12. Technical description of medical device.....	7
13. Care and maintenance.....	9
14. Contact with human organism.....	9
15. Description of raw materials, materials and component parts.....	9
16. List of materials of animal and/or human origin.....	9
17. Data of state register of pharmaceuticals.....	9
18. Packaging.....	9
19. Labeling.....	10
20. Warranty.....	15
21. Sterility.....	16
22. Conditions of operation, storage and transportation.....	16
23. SHELF LIFE.....	16
24. Disposal.....	16
25. Protection of environment.....	16
26. List of international standards to which medical device corresponds.....	16
27. Organization – manufacturer.....	17
28. Authorized representative of the manufacturer.....	17

1. Name of medical device

Nasal oxygen cannula, size L, consisting of:

- cannula - 1 pcs.

- instruction for use.

Nasal oxygen cannula, size M, consisting of:

- cannula - 1 pcs.

- instruction for use.

Nasal oxygen cannula, size S, consisting of:

- cannula - 1 pcs.

- instruction for use.

Nasal oxygen cannula, size XS, consisting of:

- cannula - 1 pcs.

- instruction for use.

2. Purpose of medical device and functional characteristics

To provide oxygen (O₂) to urgent and critically ill patients.

3. Conditions/ scope of use

Used in hospitals and clinics.

4. Indications for use

a) Patients with heart failure have difficulty breathing due to congestion in the lungs.

b) Patients with diseases of the respiratory system that affect the vital capacity of lungs.

c) Patients with a lack of oxygen caused by toxicosis, oxygen cannot penetrate into the tissue and the capillary vessel.

d) In some patients, after surgery, hemorrhagic shock or craniocerebral disease, including birth asphyxia, irregular for a long time or the fetal heartbeat is poor, and so on.

e) Patients in a coma, for example, with impaired cerebral circulation, etc.

5. Contraindications

6. Warnings

Medical product is dangerous as it can cause symptoms such as bacterial / microbial infection.

7. Potential user

Nasal oxygen cannulas are intended for use by professional people such as doctors or nurses in clinics or hospitals.

8. Warnings and precautionary measures

- Single use. Dispose after use.
- Do not use if packaging is opened or damaged.
- Do not store at extreme temperature and humidity. Store in dry and cool place.
- Make sure that all connections are reliable and air freely circulates in the tube.

9. MODE OF APPLICATION

1. Connect the oxygen supply tube to the gas source and set the required oxygen flow.

2. Check the gas flow through the device.

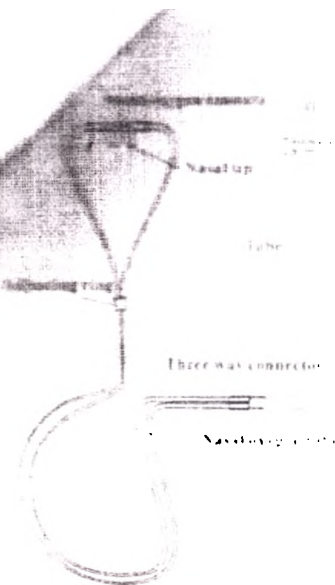
3. Insert the nasal tips into the nostrils with two plastic tubes above the ears and below the chin.

4. Carefully adjust the plastic shutter until the cannula locks into place.

NOTE: The nasal tips can be trimmed with scissors for use on small patients.

10. Description of the device

Description and appearance of medical device:

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The nasal oxygen cannula consists of a nasal oxygen cannula and an oxygen tube. The cannula consists of a nasal cannula, a nasal oxygen tube, an adjusting ring, and a three-way oxygen tube connector. The oxygen tube consists of oxygen tube connectors and tube itself.

11 CLASSIFICATION OF MEDICAL PRODUCT

Classification of potential risk of use of the device, in accordance to the requirements of the Federal Law of the Russian Federation No 40 of 06.06.2012 "On approval of technical regulations" (No. 188-ФЗ), classified as class 2a

12 Technical description of medical device

№	№	220	Oxygen tube	Nasal tip (mm)	Nasal cannula (mm)
1	1	18			

№	№	220	Oxygen tube	Nasal tip (mm)	Nasal cannula (mm)
1	1	18			
2	2	18			
3	3	18			
4	4	18			
5	5	18			
6	6	18			
7	7	18			
8	8	18			
9	9	18			
10	10	18			
11	11	18			
12	12	18			
13	13	18			
14	14	18			
15	15	18			
16	16	18			
17	17	18			
18	18	18			
19	19	18			
20	20	18			
21	21	18			
22	22	18			
23	23	18			
24	24	18			
25	25	18			
26	26	18			
27	27	18			
28	28	18			
29	29	18			
30	30	18			
31	31	18			
32	32	18			
33	33	18			
34	34	18			
35	35	18			
36	36	18			
37	37	18			
38	38	18			
39	39	18			
40	40	18			
41	41	18			
42	42	18			
43	43	18			
44	44	18			
45	45	18			
46	46	18			
47	47	18			
48	48	18			
49	49	18			
50	50	18			
51	51	18			
52	52	18			
53	53	18			
54	54	18			
55	55	18			
56	56	18			
57	57	18			
58	58	18			
59	59	18			
60	60	18			
61	61	18			
62	62	18			
63	63	18			
64	64	18			
65	65	18			
66	66	18			
67	67	18			
68	68	18			
69	69	18			
70	70	18			
71	71	18			
72	72	18			
73	73	18			
74	74	18			
75	75	18			
76	76	18			
77	77	18			
78	78	18			
79	79	18			
80	80	18			
81	81	18			
82	82	18			
83	83	18			
84	84	18			
85	85	18			
86	86	18			
87	87	18			
88	88	18			
89	89	18			
90	90	18			
91	91	18			
92	92	18			
93	93	18			
94	94	18			
95	95	18			
96	96	18			
97	97	18			
98	98	18			
99	99	18			
100	100	18			

No	Characteristic	Requirements
1	Appearance of the nasal oxygen cannula	The appearance of the nasal oxygen cannula must be free of foreign bodies; the tube surface must be smooth and free of visible dots on the tube wall are acceptable. The upper part of the tube should not be free from burrs and defects. The nasal oxygen cannula should be soft and flexible without kinking.
2	Weight of the nasal oxygen cannula	34 ± 2g
3	Dimensions and characteristics of tube and details	
4	Minimum strength of the tube with oxygen connector	Minimum 15 N
5	Minimum strength of the nasal oxygen tube	Minimum 15 N
6	Minimum strength of the nasal oxygen tube	Minimum 15 N
7	Minimum strength of the nasal oxygen tube	Minimum 15 N
8	Minimum strength of the nasal oxygen tube	Minimum 15 N
9	Minimum strength of the nasal oxygen tube	Minimum 15 N
10	Minimum strength of the nasal oxygen tube	Minimum 15 N
11	Minimum strength of the nasal oxygen tube	Minimum 15 N
12	Minimum strength of the nasal oxygen tube	Minimum 15 N
13	Minimum strength of the nasal oxygen tube	Minimum 15 N
14	Minimum strength of the nasal oxygen tube	Minimum 15 N
15	Minimum strength of the nasal oxygen tube	Minimum 15 N
16	Minimum strength of the nasal oxygen tube	Minimum 15 N
17	Minimum strength of the nasal oxygen tube	Minimum 15 N
18	Minimum strength of the nasal oxygen tube	Minimum 15 N
19	Minimum strength of the nasal oxygen tube	Minimum 15 N
20	Minimum strength of the nasal oxygen tube	Minimum 15 N
21	Minimum strength of the nasal oxygen tube	Minimum 15 N
22	Minimum strength of the nasal oxygen tube	Minimum 15 N
23	Minimum strength of the nasal oxygen tube	Minimum 15 N
24	Minimum strength of the nasal oxygen tube	Minimum 15 N
25	Minimum strength of the nasal oxygen tube	Minimum 15 N
26	Minimum strength of the nasal oxygen tube	Minimum 15 N
27	Minimum strength of the nasal oxygen tube	Minimum 15 N
28	Minimum strength of the nasal oxygen tube	Minimum 15 N
29	Minimum strength of the nasal oxygen tube	Minimum 15 N
30	Minimum strength of the nasal oxygen tube	Minimum 15 N
31	Minimum strength of the nasal oxygen tube	Minimum 15 N
32	Minimum strength of the nasal oxygen tube	Minimum 15 N
33	Minimum strength of the nasal oxygen tube	Minimum 15 N
34	Minimum strength of the nasal oxygen tube	Minimum 15 N
35	Minimum strength of the nasal oxygen tube	Minimum 15 N
36	Minimum strength of the nasal oxygen tube	Minimum 15 N
37	Minimum strength of the nasal oxygen tube	Minimum 15 N
38	Minimum strength of the nasal oxygen tube	Minimum 15 N
39	Minimum strength of the nasal oxygen tube	Minimum 15 N
40	Minimum strength of the nasal oxygen tube	Minimum 15 N
41	Minimum strength of the nasal oxygen tube	Minimum 15 N
42	Minimum strength of the nasal oxygen tube	Minimum 15 N
43	Minimum strength of the nasal oxygen tube	Minimum 15 N
44	Minimum strength of the nasal oxygen tube	Minimum 15 N
45	Minimum strength of the nasal oxygen tube	Minimum 15 N
46	Minimum strength of the nasal oxygen tube	Minimum 15 N
47	Minimum strength of the nasal oxygen tube	Minimum 15 N
48	Minimum strength of the nasal oxygen tube	Minimum 15 N
49	Minimum strength of the nasal oxygen tube	Minimum 15 N
50	Minimum strength of the nasal oxygen tube	Minimum 15 N
51	Minimum strength of the nasal oxygen tube	Minimum 15 N
52	Minimum strength of the nasal oxygen tube	Minimum 15 N
53	Minimum strength of the nasal oxygen tube	Minimum 15 N
54	Minimum strength of the nasal oxygen tube	Minimum 15 N
55	Minimum strength of the nasal oxygen tube	Minimum 15 N
56	Minimum strength of the nasal oxygen tube	Minimum 15 N
57	Minimum strength of the nasal oxygen tube	Minimum 15 N
58	Minimum strength of the nasal oxygen tube	Minimum 15 N
59	Minimum strength of the nasal oxygen tube	Minimum 15 N
60	Minimum strength of the nasal oxygen tube	Minimum 15 N
61	Minimum strength of the nasal oxygen tube	Minimum 15 N
62	Minimum strength of the nasal oxygen tube	Minimum 15 N
63	Minimum strength of the nasal oxygen tube	Minimum 15 N
64	Minimum strength of the nasal oxygen tube	Minimum 15 N
65	Minimum strength of the nasal oxygen tube	Minimum 15 N
66	Minimum strength of the nasal oxygen tube	Minimum 15 N
67	Minimum strength of the nasal oxygen tube	Minimum 15 N
68	Minimum strength of the nasal oxygen tube	Minimum 15 N
69	Minimum strength of the nasal oxygen tube	Minimum 15 N
70	Minimum strength of the nasal oxygen tube	Minimum 15 N
71	Minimum strength of the nasal oxygen tube	Minimum 15 N
72	Minimum strength of the nasal oxygen tube	Minimum 15 N
73	Minimum strength of the nasal oxygen tube	Minimum 15 N
74	Minimum strength of the nasal oxygen tube	Minimum 15 N
75	Minimum strength of the nasal oxygen tube	Minimum 15 N
76	Minimum strength of the nasal oxygen tube	Minimum 15 N
77	Minimum strength of the nasal oxygen tube	Minimum 15 N
78	Minimum strength of the nasal oxygen tube	Minimum 15 N
79	Minimum strength of the nasal oxygen tube	Minimum 15 N
80	Minimum strength of the nasal oxygen tube	Minimum 15 N
81	Minimum strength of the nasal oxygen tube	Minimum 15 N
82	Minimum strength of the nasal oxygen tube	Minimum 15 N
83	Minimum strength of the nasal oxygen tube	Minimum 15 N
84	Minimum strength of the nasal oxygen tube	Minimum 15 N
85	Minimum strength of the nasal oxygen tube	Minimum 15 N
86	Minimum strength of the nasal oxygen tube	Minimum 15 N
87	Minimum strength of the nasal oxygen tube	Minimum 15 N
88	Minimum strength of the nasal oxygen tube	Minimum 15 N
89	Minimum strength of the nasal oxygen tube	Minimum 15 N
90	Minimum strength of the nasal oxygen tube	Minimum 15 N
91	Minimum strength of the nasal oxygen tube	Minimum 15 N
92	Minimum strength of the nasal oxygen tube	Minimum 15 N
93	Minimum strength of the nasal oxygen tube	Minimum 15 N
94	Minimum strength of the nasal oxygen tube	Minimum 15 N
95	Minimum strength of the nasal oxygen tube	Minimum 15 N
96	Minimum strength of the nasal oxygen tube	Minimum 15 N
97	Minimum strength of the nasal oxygen tube	Minimum 15 N
98	Minimum strength of the nasal oxygen tube	Minimum 15 N
99	Minimum strength of the nasal oxygen tube	Minimum 15 N
100	Minimum strength of the nasal oxygen tube	Minimum 15 N

13. Care and maintenance

14. Contact with human organism

15. Description of raw materials, materials and component parts

16. List of materials of animal and/or human origin

17. Data of state register of pharmaceuticals

18. Packaging

Disposable Nasal Oxygen Cannula

Directions for use

1. Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow.
 2. Check for gas flow through the nasal cannula.
 3. Insert the nasal tips into the nostrils with the pad resting over the ears and under the chin.
 4. Gently adjust the plastic slide until the cannula is secure.
- NOTE: Nasal tips may be trimmed with scissors to fit smaller patients.

Caution

- For single use. Discard after use.
 - Don't use if package is open or damaged.
 - Don't store at extreme temperature and humidity.
- Store in cool and dry place.
- Ensure that all connections are gas tight and are lubricated freely through the tubing.

A-121 $d = 0.1$ $\tau_{100} = 1.0$ $\tau_{100} = 1.0$ $\tau_{100} = 1.0$

LOT
MFG
EXP

Taizhou Kanglian Medical Equipment Co., Ltd.

Shanghai Hongyan Medical Equipment Co., Ltd.

Shanghai International Holding Corp. GmbH (Europe)

Nasal Oxygen Cannula carton box

NASAL OXYGEN CANNULA

SIZE: L

Taizhou Kangjian Medical Equipment Co., Ltd.
The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, Zhejiang Province 317600, China
Tel.: 0086-576-87299799 Fax: 0086-576-87239799
E-mail: webmaster@kangjian.com

LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

NASAL OXYGEN CANNULA

SIZE: M

Taizhou Kangjian Medical Equipment Co., Ltd.
The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, Zhejiang Province 317600, China
Tel.: 0086-576-87299799 Fax: 0086-576-87239799
E-mail: webmaster@kangjian.com

LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

NASAL OXYGEN CANNULA

SIZE: S

Taizhou Kangjian Medical Equipment Co., Ltd.
The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, Zhejiang Province 317600, China
Tel.: 0086-576-87299799 Fax: 0086-576-87239799
E-mail: webmaster@kangjian.com

LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

NASAL OXYGEN CANNULA

SIZE: XS

Taizhou Kangjian Medical Equipment Co., Ltd.
The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, Zhejiang Province 317600, China
Tel.: 0086-576-87299799 Fax: 0086-576-87239799
E-mail: webmaster@kangjian.com

LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

Model of labeling of transport packaging in Russian language:

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LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

RU No _____ of _____

Nasal oxygen cannulas, trade mark Yuki group

Size: L

Quantity: _____ pcs.

Taizhou Kangjian Medical Equipments Co., Ltd.
The Machine Electricity Zone (Hang Ni Kan) of Yuhuan County, Zhejiang Province 317600, China
Tel.: 0086-576-87299799 Fax: 0086-576-87239799
E-mail: webmaster@kangjian.com

LOT
MFG
EXP
QTY: 100 PCS
G.W: 4.51 KGS
N.W: 4 KGS
MEAS: 34X28X20 CM

Authorized representative:

"ARMEDICA" LLC

Shevchenko street, house 20, premises 7, the city of

Yekaterinburg, Sverdlovsk region, 620075

Tel.: 8(343) 286 42 73, 8(343) 286 42 74

E-mail: tor@yukigroup.com

RU No _____ of _____

Symbol

Value

Single-use

Batch code

Use before...

Date of manufacture

Warning! Refer to the instructions for use

STERILE EO

Sterilization with ethylene oxide

Manufacturer

EC REP

Authorized representative in European Community

Do not use if packaging is damaged

Top

Warning, fragile

Keep away from moisture

Do not expose to sunlight

CE 0197

CE marking

Legend

Definition

QTY

Number of products (pcs) in a package

G W

Mass of transport package (gross)

N W

Mass of transport package (net)

20. Warranty

Manufacturer guarantees that the devices were made in accordance with its technical specifications and applied industry standards and normative requirements.

Warranty period is 5 years

21. Sterility

Sterile medical device: Sterilization with ethylene oxide (EO)

22. Conditions of operation, storage and transportation

Subject to operation at temperature from -5°C to $+35^{\circ}\text{C}$ and relative air humidity to 95% at 25°C .

Nasal oxygen cannulas are stored at temperature from $+20^{\circ}\text{C}$ to $+40^{\circ}\text{C}$ and relative humidity 80% at 25°C .

It is permitted to transport nasal oxygen cannulas by any kind of transport of closed type with observance of transportation rules established for this kind of transport, at the following environmental conditions – temperature from $+20^{\circ}\text{C}$ to $+35^{\circ}\text{C}$, humidity not more than 85% at 25°C .

23. SHELF LIFE

Shelf life is 5 years.

24. Disposal

Nasal oxygen cannulas with expired shelf life shall be disposed in accordance with the requirements of SanPiN 2.1.3684-21, as waste of Class A.

Nasal oxygen cannulas after use shall be disposed in accordance with the requirements of SanPiN 2.1.3684-21, as waste of Class B.

25. Protection of environment

Nasal oxygen cannulas in use, transportation and storage do not have a negative impact on humans and the environment.

26. List of international standards to which medical device corresponds

No	Doc. No	Standard name
1	ISO 11135-1: 2014	Sterilization of health care products Ethylene oxide. Requirements for development, validation and routine control of a sterilization process for medical devices.
2	EN 556-1: 2001	Sterilization of medical devices- Requirements for medical devices to be designated "Sterile" Part 1: Requirements for terminally sterilized medical devices
3	ISO 14971: 2019	Medical devices Application of risk management to medical devices
4	EN 980: 2016	Symbols for use in labeling of medical devices
5	EN 1041: 2016	Information supplied by the manufacturer of medical device

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Disposable Nasal Oxygen Cannula

Direction for use

1. Attach the oxygen supply tubing to the gas source and set the oxygen to the prescribed flow.
 2. Check for gas flow through the device.
 3. Insert the nasal tips into the nostrils with the two plastic tubes over the ears and under the chin.
 4. Gently adjust the plastic slide until the cannula is secure.
- NOTE:** Nasal tips may be trimmed with scissors to fit smaller patients.

Caution

- For single use. Discard after use.
 - Don't use if package is open or damaged.
 - Don't store at extreme temperature and humidity.
- Store in cool and dry place.
- Ensure that all connections are secure and air circulates freely through the tubing.

Adult (18-64)	Child (6-17)	Infant (1-5)	Neonate (<5)
---------------	--------------	--------------	--------------

LOT 20160825
MFG 2016-09-2
EXP 2019-09-2

[continued] (C) 1997

Teizhou Kanglian Medical Equipment Co., Ltd.

Call for the latest on the economy, the environment, and more. Monday 1-800-270-9276. Fax 1-800-270-9276. E-mail: info@18002709276.com

© 2004 Blackwell Publishing Ltd, *Journal of Internal Medicine* 255: 105–112

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(TAIZHOU KANGJIAN MEDICAL EQUIPMENT CO., LTD.)