

Фотографические изображения
медицинского изделия

PHILIPS

***«Обеспечение программное медицинское Vue PACS
для получения, хранения, архивирования, обмена
и обработки медицинских изображений и данных
на виртуальных и/или электронных носителях с
принадлежностями, в составе»***

УТВЕРЖДАЮ

Руководитель по нормативному
регулированию ООО «ФИЛИПС»



Горкуша Д.В.

«19» сентября 2022 г.





PHILIPS

«Обеспечение программное медицинское Vae PACS для получения,
хранения, архивирования, обмена и обработки медицинских
изображений и данных на виртуальных и/или электронных носителях с
принадлежностями, в составе»

Руководство пользователя

№ по каталогу 3G7607

«УТВЕРЖДАЮ»

Филипп Медикал Системс Нидерланд Б.В.

Хелена Хейбун
(ИМЯ)

Специалист отдела
(должность)

Исследования и разработки

4 августа 2022г.
(дата)

(подпись)
(подпись)

PHILIPS

«Обеспечение программное медицинское View PACS для получения, хранения, архивирования, обмена и обработки медицинских изображений и данных на виртуальных и/или электронных носителях с принадлежностями, в составе»

Руководство пользователя Portal

№ по каталогу 6M3344

«УТВЕРЖДАЮ»

Филипп Медикал Системс Интернационал Б.В.

Хасяна Халийбун
(подпись)

Специалист отдела
(должность)

информационно-правового регулирования

4 августа 2022г.
(дата)

Подпись:
(подпись)

PHILIPS

«Обеспечение программное медицинское Vue PACS для получения,
хранения, архивирования, обмена и обработки медицинских
изображений и данных на виртуальных и/или электронных носителях с
принадлежностями, в составе»

Руководство пользователя Vue Reporting

№ по каталогу 6K9436

«УТВЕРЖДАЮ»

Филикс Медикал Системс Нидерланд Б.В.

Хелена Хейнбуш
(имя)

Специалист отдела
(должность)

РЕГЛАМЕНТИРОВАННОЕ ПРАВОВОЕ РЕГУЛИРОВАНИЕ

4 августа 2022 г.
(дата)

(подпись)
(подпись)

PHILIPS

«Обеспечение программное медицинское Vue PACS для получения,
хранения, архивирования, обмена и обработки медицинских
изображений и данных на виртуальных и/или электронных носителях с
принадлежностями, в составе»

Руководство пользователя СТ Perfusion

№ по каталогу 918543

«УТВЕРЖДАЮ»

Финанс Медикал Системс Нидерланд Б.В.

Хелена Хейнбоу
(подпись)

Специалист отдела
(должность)

нормативно-правового регулирования

4 августа 2024г.
(дата)

(подпись)
(подпись)

PHILIPS

«Обеспечение программное медицинское Vue PACS для получения,
хранения, архивирования, обмена и обработки медицинских
изображений и данных на виртуальных и/или электронных носителях с
принадлежностями, в составе»

Руководство пользователя MR Diffusion и MR Perfusion

№ по каталогу AE9562

«УТВЕРЖДАЮ»

Филипп Медицина Системе Нидерланд Б.В.

Хелена Хинейбун
(имя)

Специалист отдела
(должность)

Нормативно-правового регулирования

4 августа 2022г.
(дата)

Инициалы
(подпись)

PHILIPS

«Обеспечение программное медицинское Vue PACS для получения, хранения, архивирования, обмена и обработки медицинских изображений и данных на виртуальных и/или электронных носителях с принадлежностями, в составе»

Руководство пользователя Лечение опухолей

№ по каталогу 6K9E34

«УТВЕРЖДАЮ»

Филипп Мединав Системе Напарланд Б.В.

Хелена Хисейбу
(имя)

Согласован отделе
(должность)

нормативно-правового регулирования

4 августа 2021г.
(дата)

Ивановский
(подпись)

PHILIPS

«Обеспечение программное медицинское Vue PACS для получения,
хранения, архивирования, обмена и обработки медицинских
изображений и данных на виртуальных и/или электронных носителях с
принадлежностями, в составе»

Руководство пользователя Vue Motion

№ по каталогу 6K9272

«УТВЕРЖДАЮ»

Филипп Медиал Системс Нидерланд Б.В.

Хелена Хиндбуш
(Имя)

Специалист отдела
(должность)

Нормативно-правового регулирования

4 августа 2022г.
(дата)

(подпись)
(подпись)

«УТВЕРЖДАЮ»

Филипп Медикал Системе Нидерланд Б.В.

Хелена Хейнбун
(имя)

Специалист отдела
(должность)

нормативно-правового регулирования

4 августа 2022г.
(дата)

(подпись)
(подпись)


Дополнение к Руководствам пользователя медицинского изделия


«Обеспечение программное медицинское Vue PACS для получения, хранения, архивирования, обмена и обработки медицинских изображений и данных на виртуальных и/или электронных носителях с принадлежностями, в составе»

Производитель:


Филипп Медикал Системе Нидерланд Б.В., Нидерланды
Нидерланды, г. Бест, Веемплюйс, 6, индекс 5684 PC

Archive Explorer											
Diagnose View Manage Share											
Auto Next Create Report View Testimony File											
Unread											
Referral Name	Patient ID	Date	# Images	Priority	Accession	Mod	ID	Body Part	Date of Birth	Ref. Physician	
Hindwig, Duane	202643	5/3/2021	56	Medium	629839	CT	5011	ABD	9/22/1934		
Jackson, Jenkins Sarah	268457	12/31/2007	1	Medium	3474230607	OT		OPERATION	3/1/2021		
Jackson, Jenkins Sarah	268457	5/3/2021	110	Medium	185600008	MR	12345		3/1/2021		
Hatcher, Amy	63635	5/3/2021	107	Low	2244704114	MR	123456-1	LUMBAR	9/11/1932		
Hernandez, Miller Robert	887766	5/3/2021	305	Low	1575896254	CT	WE23434	RUNOFF	3/9/2021		
Hernandez, Miller Robert	887766	5/3/2021	1002	High	1480630439	CT		RUNOFF	3/9/2021		
Lugo, Robert	55576864905	5/3/2021	2774	Medium		CT		HEART	7/14/1996		
Wilson, Ashood Jack	321321	12/6/2011	1	Low	153EC894	OT			9/19/1962	ROTSHEIN	
Brasgood, Carle A	DT2411865505	5/3/2021	7	Low	DT1226803063	XA	5229600	EMBOLIZATION	12/14/1961	CKEEFE REGIS	
Wilson, Darlene	11243242434	5/3/2021	61	Low	YAE1655720	CT	14120		7/25/1961	DI	
Wilson, Darlene P	NLP2804	5/3/2021	56	Medium	DEMO1	US	262620120995		7/25/1961	DEMO DR	
Miller, John	2411081105	5/3/2021	237	Low		CT	20665	HEAD	3/23/1970		
Lee, Hopkins Mary	456	5/3/2021	139	Low	1817089582	MR		HEAD	3/4/2021		
Lee, Hopkins Mary	456	5/3/2021	432	Low	1063216348	CT		ABDOMEN	3/4/2021		
Jackson, Jenkins Sarah	268457	12/31/2007	1	Medium	1056230070	OT		PATHOLOGY	3/1/2021	ROTSHEIN	
Jackson, Jenkins Sarah	268457	12/1/2011	1	Low	153EC410	OT		ABDOMEN	3/1/2021	ROTSHEIN	
Baker, Story Susan	85412	5/3/2021	156	High		MR		LUMBAR	3/2/2021		
Smith, Edward	201034346666	5/3/2021	520	Low	9275000040588	MR	3053	ABDOMEN	2/17/1937	RAMBAM	
Brown, Smith Edward	5346	12/4/2011	1	Low	153EC450	OT			12/5/2011	ROTSHEIN	
Stewart, Lugo Jimmy	586578	12/4/2011	1	Low	153EC455	OT		ABDOMEN	12/5/2011	ROTSHEIN	
Torres, Piper Paula	677321	12/5/2011	1	Low	153EC833	OT			11/9/1941	ROTSHEIN	

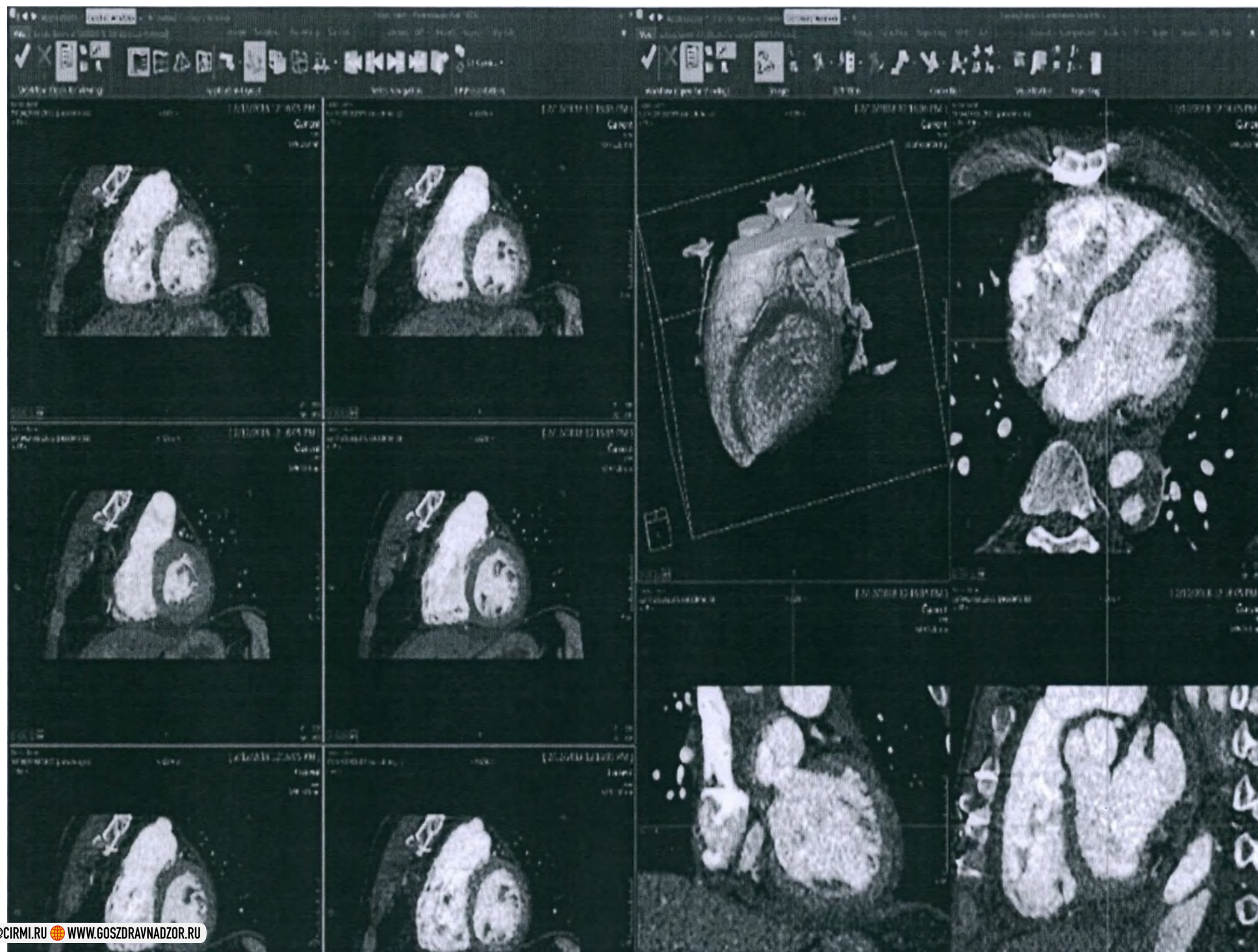

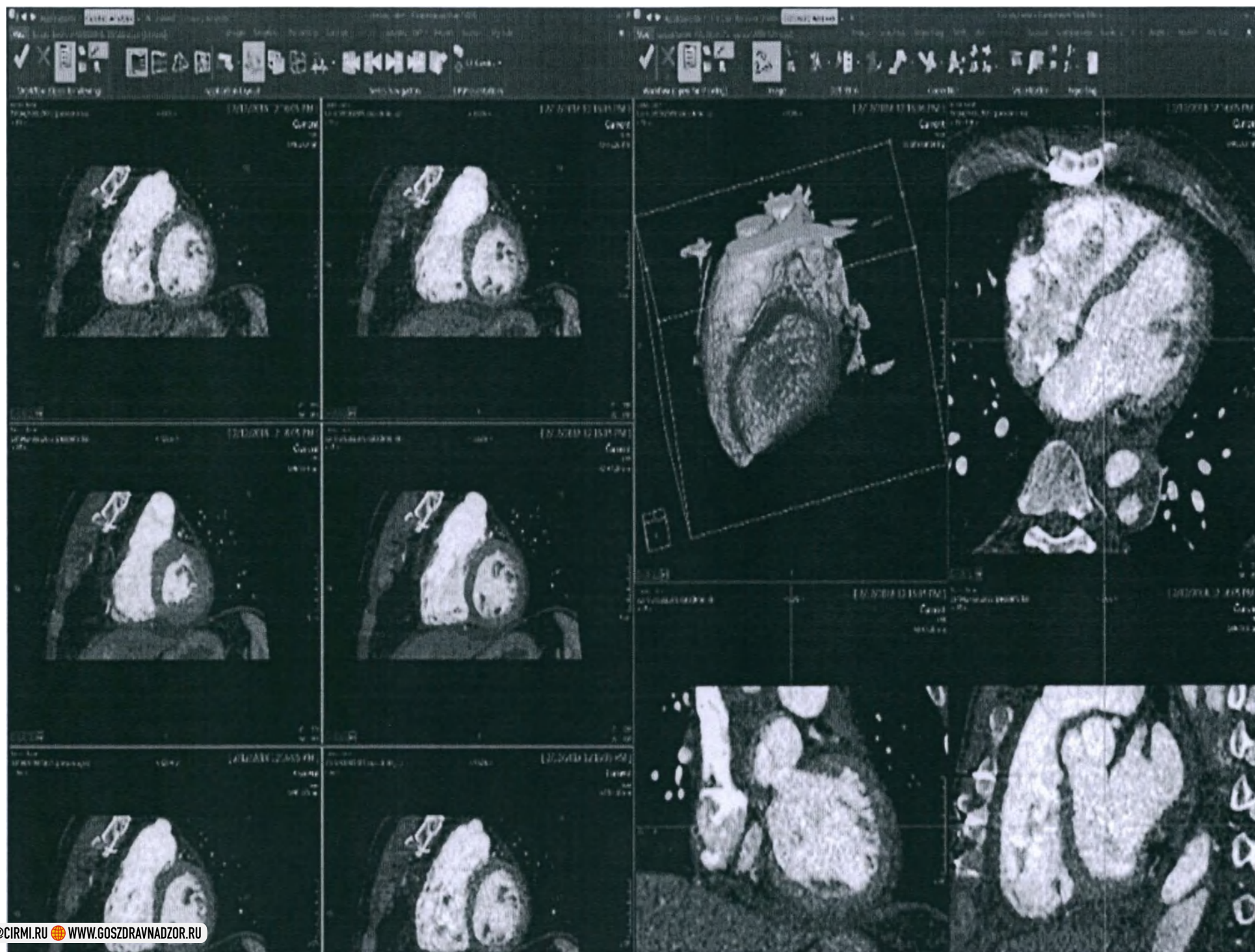

4. Ключ лицензионный для активации программного модуля 3D BASE PKG UNLIMITED в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований

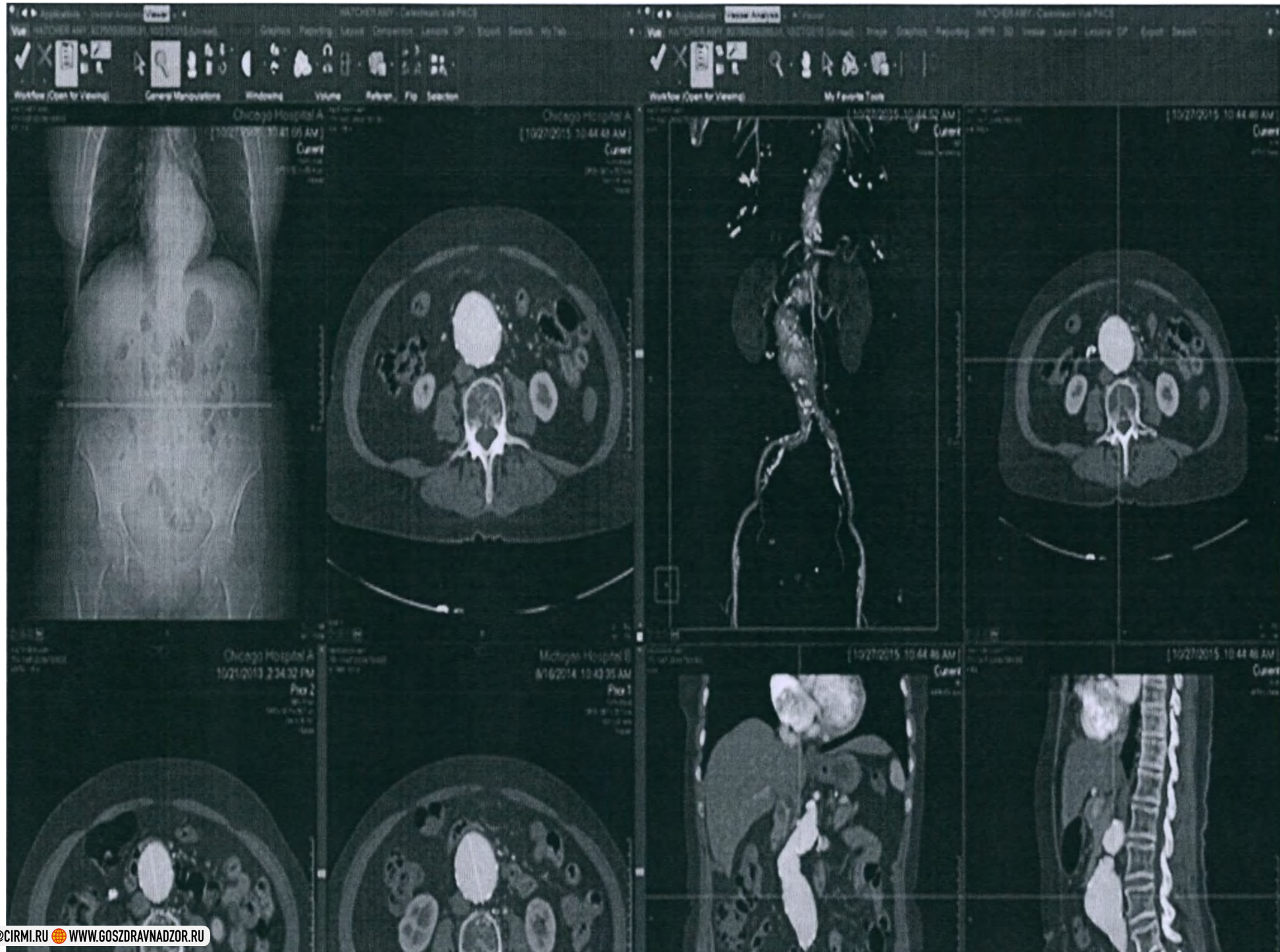

10.94.195.200 portal:compare=1&user_name=rona&password=9CADD95 837D 496D 9D42 18265115F135&study_instance_uid=1.2.840.113704.7.1.0.11.820926179105.144362251005&series_instance_uid=1.2.840.113704.7.1.0.11.820926179105.144362251005

Carestream DAVIS DOROTHY

Patient ID: 201201051940 Date of Birth: 6/1/1943 (71Y 0M) Gender: F

Patient History

Filter studies by

1/3/2015 7:05 AM
ECG CARDIAC

9/28/2014 2:47 PM
CT

6/23/2014 3:10 PM
CT CHEST

3/31/2014 2:30 PM
CT

1/26/2014 2:27 PM
OT PATHOLOGY REPORT
(Pathology Report.pdf)

1/15/2014 7:26 AM
OT BIOPSY VIDEO
Lung Biopsy (Lung Biopsy.mp4)

12/25/2013 9:31 AM
OT LOBECTOMY VIDEO
Lobectomy Procedure (Left Upper Lobes)

9/20/2011 2:20 PM
CR SKILL

DAVIS DOROTHY, 201201051940
Acc: 9275100123456
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: BODYAX
1 - 53
Lobby (1.5)

2/2/2015 11:56:56 AM
REMOVED
Pos: 35.00 mm
SW: 1.00 mm
C: 600 W: 1600
Zoom: 84%

DAVIS DOROTHY, 201201051940
Acc: 9275100234567
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: Recon 2: BODYAX
3 - 58
Lobby (1.9)

9/28/2014 2:51:32 PM
REMOVED
Pos: 39.25 mm
SW: 1.25 mm
C: 600 W: 1600
Zoom: 109%

DAVIS DOROTHY, 201201051940
Acc: 9275100345678
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: BODYAX
1 - 55
Lobby (2.1)

6/23/2014 3:22:13 PM
REMOVED
Pos: 50.00 mm
SW: 1.00 mm
C: 600 W: 1600
Zoom: 86%

DAVIS DOROTHY, 201201051940
Acc: 9275100456789
Study Desc: CT CHEST
Series Desc: Recon 2: BODYAX
3 - 54
Lobby (1.8)

3/31/2014 2:33:57 PM
REMOVED

Patient Reports

1/1 John Jennings
2/2/2015
(Report matches study selected on Patient History pane)

PROCEDURE: CT Chest

CLINICAL INDICATION: Follow-up of a known left-sided squamous cell carcinoma of the lung post-surgery with suspected lung metastasis

TECHNIQUE: CT scan of the chest without contrast was performed on the GE volumetric 64 slice CT scanner. 3-D coronal reformatted images were obtained from the axial source images.

COMPARISON: CT March 31 2014, CT June 23 2014, CT September 28 2014

FINDINGS:
Status post left upper lobectomy. Several nodules are seen in the left lower lobe in the upper posterior with a diameter of (1.1 mm), another upper anterior nodule (2 mm) (larger than in the previous scan) and mid-posterior nodule (12 mm) (not larger than in the previous scan). There is a very small nodule in the right upper lobe with no change compared with the previous study. Some emphysematic bulbs are seen in the right lung. There is no acute infiltrate. The mediastinum is shifted to the left (due to the lobectomy) without evidence for mass or lymphadenopathy. The vascular structures of the mediastinum are normal in course and caliber. Aortic vascular

Bookmark Information

Name	SW	C	W	Zoom
B01	1.00	600	1600	84%

Follow Up Information

Exam Date	Long Diameter (cm)	Trend*
2/2/2015	1.4	+5.8% ↑
9/28/2014	1.3	-1.1% ↓
6/23/2014	1.4	+4.5% ↑
3/31/2014	1.3	0.0%

* Trend according to baseline 3/31/2014

Notes

1/1 John Jennings

Purpose: Critical Results - Notification

Existing lung nodules have mildly increased in size.

2/2/2015 2:17 PM

10.94.195.200/portal/?compare=1&user_name=sna&password=&bmld=SCAD0095.BJ7D-496D-9D92-1826111F511&study_instance_id=1.2.840.113704.7.1.0.11.98.0926179105.1443622510.0&series_instance_id=1.2.840.113704.7.1.0.11.98.0926179105.1443622510.0

Carestream DAVIS DOROTHY

Patient ID: 201201051940 Date of Birth: 6/1/1943 (71Y 8M) Gender: F

Patient History

Filter studies by

Last Month (0) Last Year (2) Last 3 Years (8) All (12)

All studies (12)

2/2/2015 11:54 AM
CT CHEST
CT 2D REFORM STATISTIC CODE

1/5/2015 7:05 AM
BOB CARDIAC

9/28/2014 2:47 PM
CT
CT 2D REFORM STATISTIC CODE

6/23/2014 3:10 PM
CT CHEST
CT 2D REFORM STATISTIC CODE

3/31/2014 2:30 PM
CT
CT CHEST

1/26/2014 2:27 PM
QT PATHOLOGY REPORT
(Pathology Report.pdf)

1/15/2014 7:26 AM
QT BIOPSY VIDEO
Lung Biopsy (Lung Biopsy.m4)

12/25/2013 9:31 AM
QT LOBECTOMY VIDEO
Lobectomy Procedure (Left Upper Lobe)

9/20/2011 2:20 PM
CR SKULL

DAVIS DOROTHY, 201201051940
Acc: 9275100123456
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: BODYAX
1 - 53
Lossy (1.5)

2/7/2015, 11:56:56 AM
REMOVED
Pos: 35.00 mm
SW: 1.00 mm
C-600 W:1600
Zoom: 84%

Name: B01 (F01)
Vol: 413.0 mm³
Dia1: 13.7 mm
Dia2: 8.2 mm
Description: Lesion (Lung)

DAVIS DOROTHY, 201201051940
Acc: 9275100234567
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: Recon 2: BODYAX
3 - 58
Lossy (1.5)

9/28/2014, 2:51:32 PM
REMOVED
Pos: -39.25 mm
SW: 1.25 mm
C-600 W:1600
Zoom: 109%

DAVIS DOROTHY, 201201051940
Acc: 9275100345678
Study Desc: CT 2D REFORM STATISTIC CODE
Series Desc: BODYAX
1 - 55
Lossy (1.5)

6/23/2014, 3:22:13 PM
REMOVED
Pos: 50.00 mm
SW: 1.00 mm
C-600 W:1600
Zoom: 86%

Name: B03 (F01)
Vol: 478.0 mm³
Description: Lesion (Lung)

DAVIS DOROTHY, 201201051940
Acc: 9275100456789
Study Desc: CT CHEST
Series Desc: Recon 2: BODYAX
3 - 54
Lossy (1.5)

3/31/2014, 2:33:57 PM
REMOVED

Name: B01 (F01)
Vol: 388.3 mm³
Description: Lesion (Lung)

Patient Reports

3/1 John Jennings
2/2/2015
(Report matches study selected on Patient History pane)

PROCEDURE: CT Chest

CLINICAL INDICATION: Follow-up of a known left-sided squamous cell carcinoma of the lung post-surgery with suspected lung metastasis

TECHNIQUE: CT scan of the chest without contrast was performed on the GE volumetric 64 slice CT scanner. 3D coronal reformatted images were obtained from the axial source images.

COMPARISON: CT March 31 2014, CT June 23 2014, CT September 28 2014

FINDINGS:
Status post left upper lobectomy. Several nodules are seen in the left lower lobe: in the upper posterior with a diameter of (11 mm), another upper anterior nodule (9 mm) (larger than in the previous scan) and mid-posterior nodule (12 mm) (not larger than in the previous scan). There is a very small nodule in the right upper lobe with no change compared with the previous study. Some emphysematic bulbs are seen in the right lung. There is no acute infiltrate. The mediastinum is shifted to the left (due to the lobectomy) without evidence for mass or lymphadenopathy. The vascular structures of the mediastinum are normal in course and caliber. Aortic vascular

Bookmark Information

Name: B01
Description: Lesion (Lung)

Volume (cm³): 0.4
Volume Error (%): +/-25%
Long Diameter (cm): 1.4
Short Diameter (cm): 0.6
HU Mean (HU): -159

Follow Up Information

Exam Date	Long Diameter (cm)	Trend*
2/2/2015	1.4	+5.8%
9/28/2014	1.3	-1.1%
6/23/2014	1.4	+4.5%
3/31/2014	1.3	0.0%

* Trend according to baseline 3/31/2014

Notes

3/1 John Jennings
Purpose: Critical Results - Notification
Existing lung nodules have mildly increased in size.

2/2/2015 2:17 PM

LTSM Configuration

Wakeup period to check for new studies that need to be backed up: 15 minutes

Maximum pending studies waiting to be backed up (new studies): 50

Maximum pending studies waiting to be re-backed up (retries): 100

Period to wait before the backup of a new study will be executed: 15 minutes

☐ Continue to re-back studies that previously failed to be backed up

☒ Backup fragmented studies

File System / External backup Device configuration

Automatically mark studies as "backed up" after: 0.1 hours

Wakeup period to check completion of studies backed up: 0.1 hours

Wakeup period to insert studies into backup table: 12 hours

Storage Commitment

☐ Enable Storage Commitment feature

Wakeup period to check for studies that require storage commit request to be sent: 0.1 hours

Waiting time after backup is completed before sending storage commitment request: 0.1 hours

8. Ключ лицензионный для активации программного модуля BASE WFM SW в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований, DICOM DATA IMPORT

Archive Explorer

Worklist Home View Manage Acquire Organize Share

Weekday Shift

Stroke/Trauma (3/3)

ED/STAT (8/23)

Inpatient (7/24)

Flags	Patient Name	Age	Gender	Ref. Physician	Procedure Text	Mod
	Edwards, Elizabeth	57Y 6M	F	Evans David	CT Head w/o Contrast	CT
	Ross, Fred	44Y 1M	M	Blankenship Nathaniel	CR Leg. 2 Views	CR
	Valdez, Francisco	34Y 11M	M	Richardson Justin	CR Leg. 2 Views	CR
	Brown, Louis	72Y 6M	M	Keaton Robert	CT Head w/o Contrast	CT
	Green, Mary	69Y 10M	F	Blankenship Nathaniel	CT Face/Neck/Chest w/ Contrast	CT
	Moore, Emily	13Y 4M	F	Evans David	CT Head w/o Contrast	CT
	Wright, Chloe	19Y 6M	F	Blankenship Nathaniel	XR Cervical Spine, 2-3 VIEWS	DX
	Lewis, Sofia	43Y 7M	F	Blankenship Nathaniel	CT Head/Brain w/o IV Contrast	CT
	Taylor, James	72Y 2M	M	Morris Lawrence	XR Neck, Soft Tissue	DX
	Harris, Benjamin	59Y 3M	M	Morris Lawrence	CT Soft Tissue Neck w/ IV Cont.	CT
	Lopez, Matthew	54Y 8M	M	Keaton Robert	CT Brain w/o IV Contrast	CT
	Parker, Pamela	27Y 5M	F	Evans David	PET Whole Body w/ concurrent	PT CT
	Green, Gary	65Y 9M	M	Evans David	PET-CT Head	PT CT
	Jenkins, Sarah	64Y 10M	F	EVANS DAVID	CT Head w/ Contrast	CT
	Turner, Aimee	65Y 10M	F	Blankenship Nathaniel	CT ABD/Pelvis w/o CONT	CT
	Bates, Robin	55Y 8M	F	Blankenship Nathaniel	PET-CT Cervical w/ CT Thx w/	PT CT
	Lee, Noah	66Y 3M	M	Richardson Justin	CT Brain w/o IV Contrast	CT


Search


No conversation history. Search contacts to start a conversation.


Online

Notify admin


Artelli, Dario
Chicago Medical Center

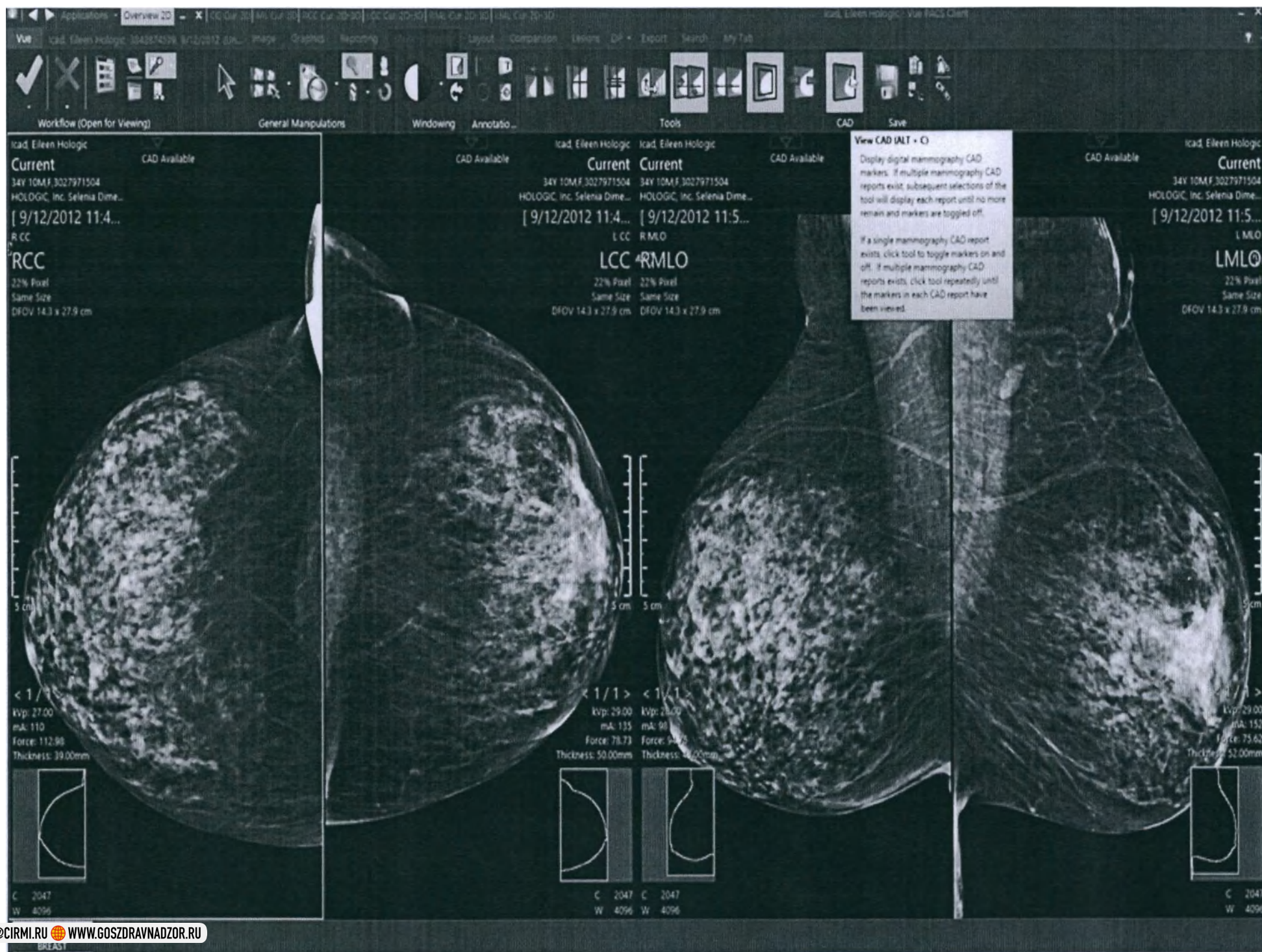

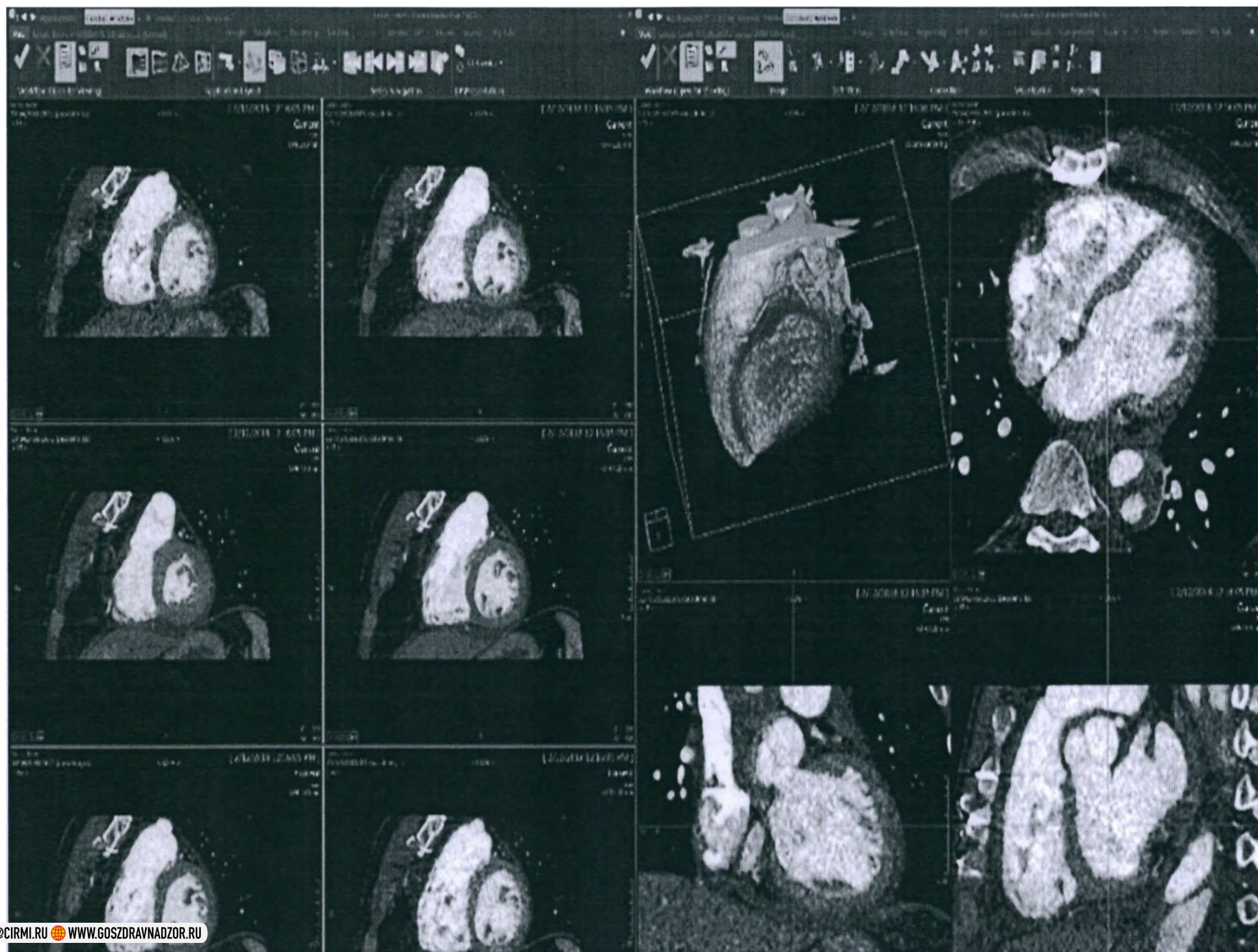

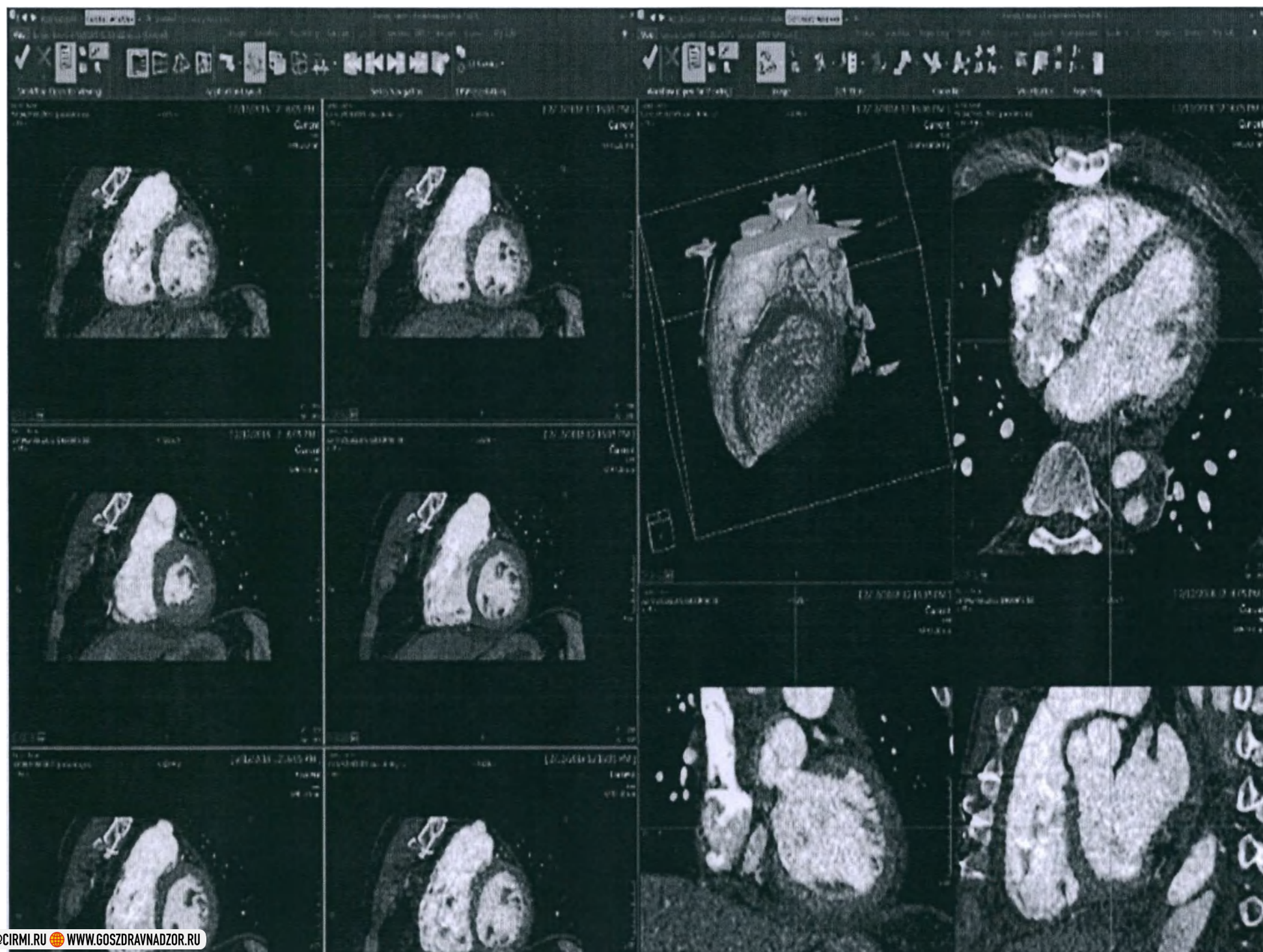


12. Ключ лицензионный для активации программного модуля CARDIAC CT UNLIMITED LIC в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований 23

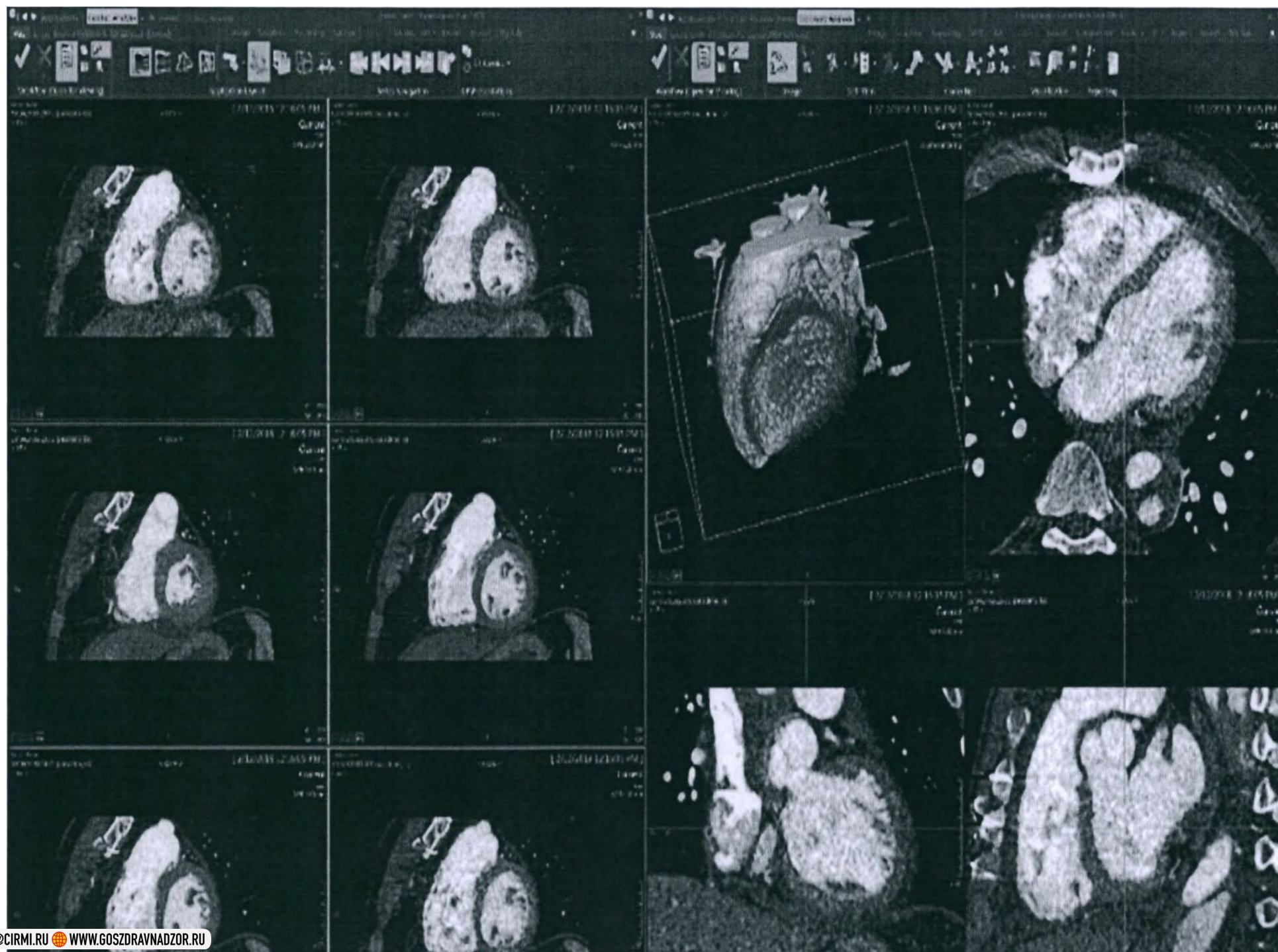





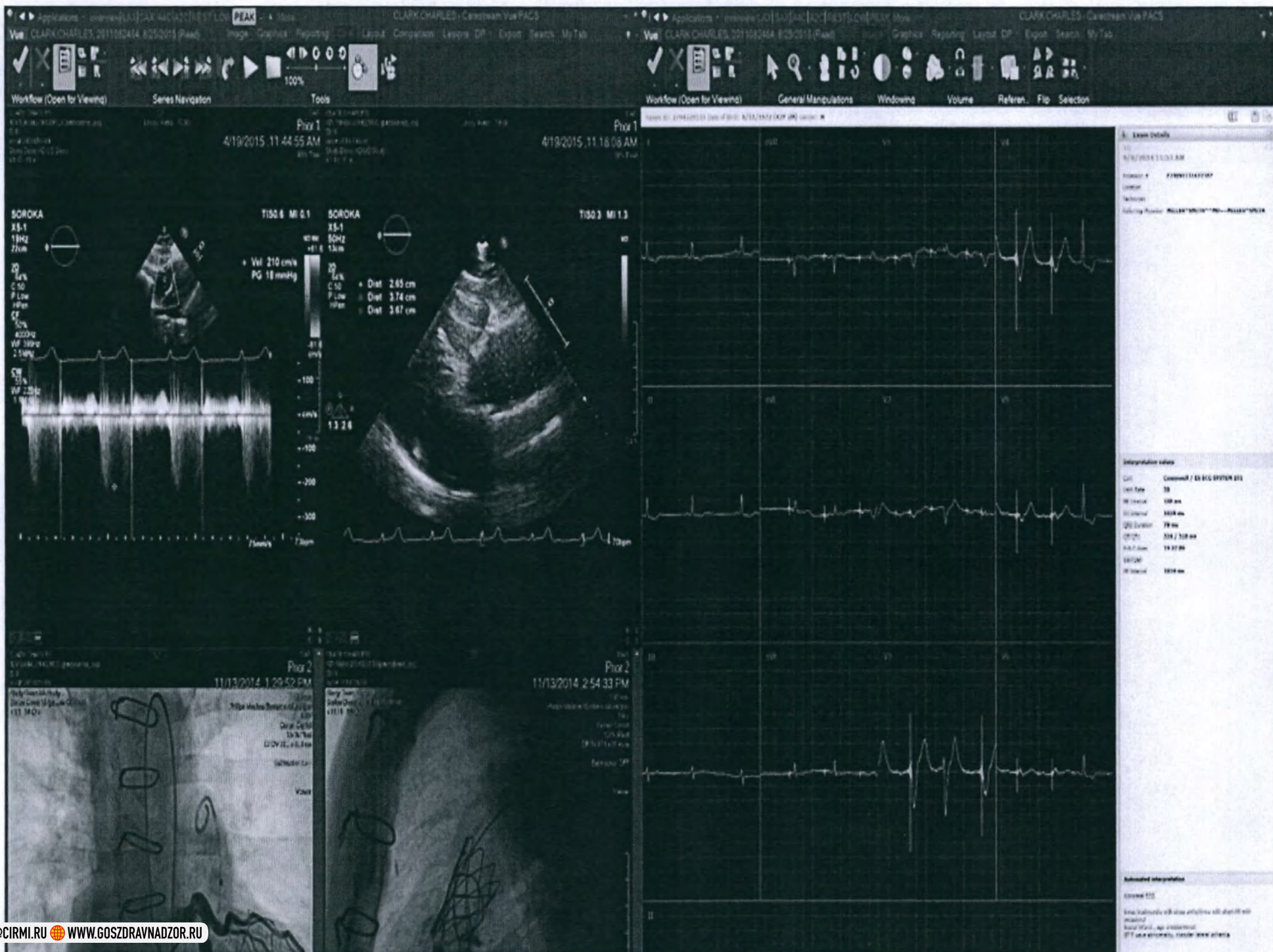

Password Syntax Settings			
Minimum password length.	8	Maximum password length.	8
Minimum numeric chars.	1	Maximum consecutive chars.	2
Minimum upper case chars.	0	Minimum special chars.	0

General Settings	
Identical password check.	3
Expiration options:	
☐ Password never expires	☒ Password expires after (days): 100

Password Failures Settings	
Allowed number of password failures.	10
When number exceeds limit:	
☐ Lock user	☒ Suspend user. Duration in minutes: 100

Application Settings	
CD-Direct	Access Control

Permission Settings	
Group Permissions	

Password Syntax Settings

Minimum password length:

8

Maximum password length:

8

Minimum numeric chars:

1

Maximum consecutive chars:

2

Minimum upper case chars:

0

Minimum special chars:

0

General Settings

Identical password check:

3

Expiration options:

☐ Password never expires☒ Password expires after (days):

100

Password Failures Settings

Allowed number of password failures:

10

When number exceeds limit:

☐ Lock user☒ Suspend user. Duration in minutes:

100


Application Settings


CD-Direct

Access Control

Permission Settings

Group Permissions

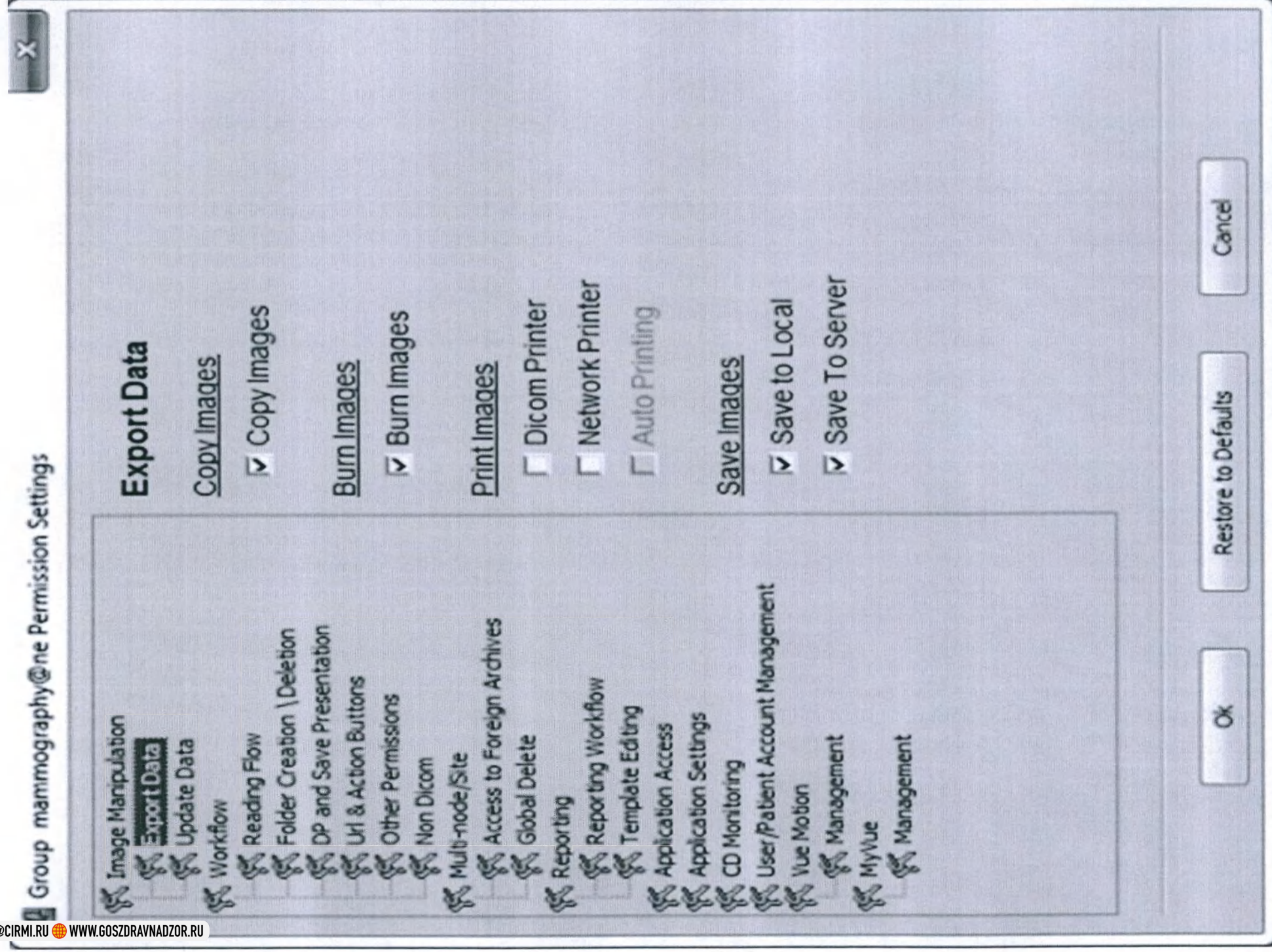

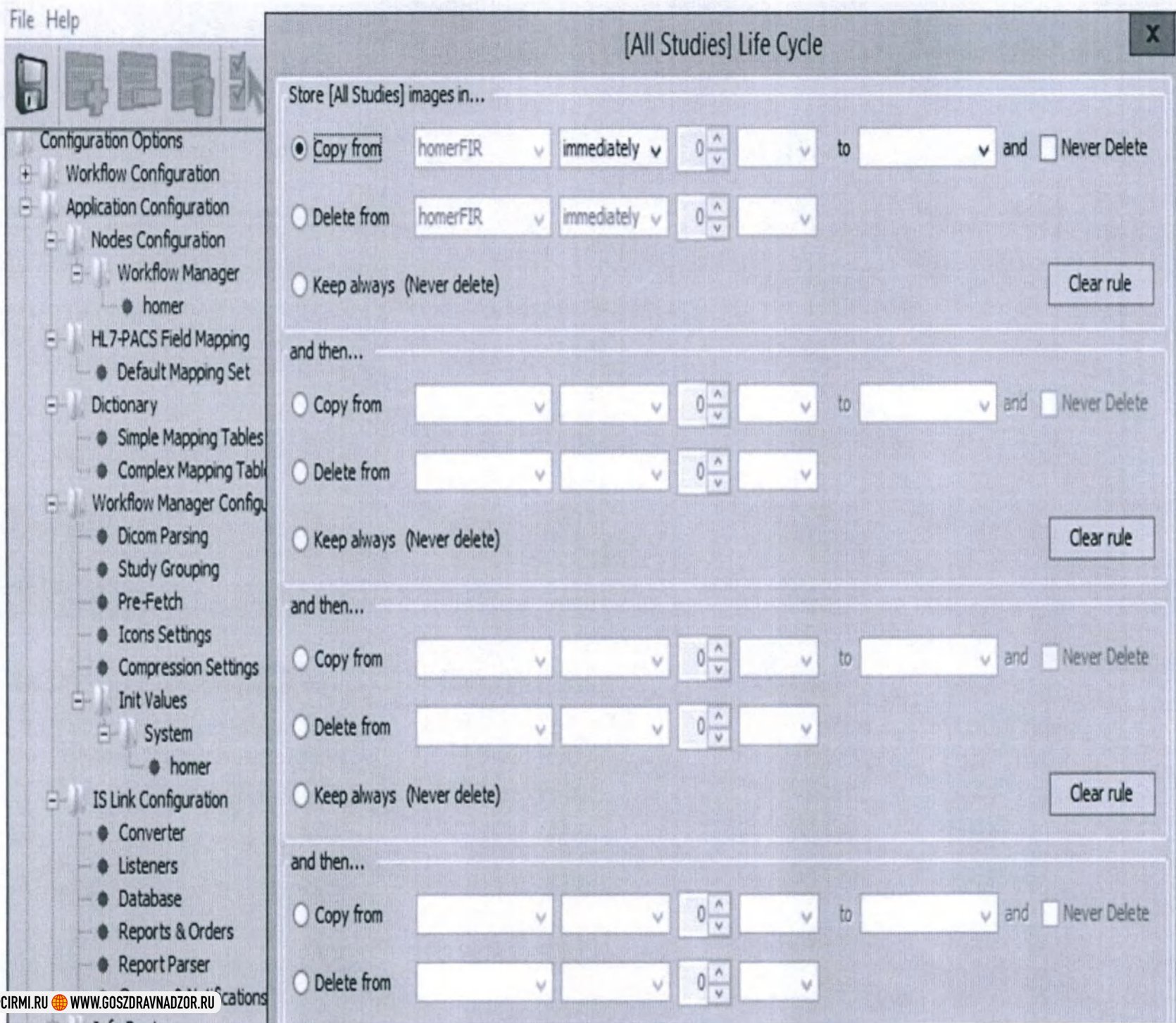

Archive Explorer

My Lists

Reported Patients

Performed Procs.


Local Patients


Create Patient Merge Patient Link Patient Link Patient Show Linked Patients

Local Patients

Patient Name A	Patient ID	Date of Birth	Age	Gender	Address	Last Order	Home Phone	SSN	Email	Issuer
Ashwood, Jack	276672673	9/13/1966	50Y 3M	M	237 E ERIE ST. CHICAGO, US, IL	Not Known	(312)-423-		ash@home.com	pacsdemo
Austin, Anthony	275671623	6/1/1975	41Y 6M	M	234 WALKER DR. CHICAGO	Not Known	(312)-432-		austin@home.com	pacsdemo
Baker, Blake	201345600	6/12/1963	53Y 1M	M	23 JAMES ST. BALTIMORE, U	10/13/2015	(312)-432-		baker@home.com	pacsdemo
Cearley, Wilma	200990000	10/30/1927	88Y 1M	F	902 E COLORADO ST. LOS A	Not Known	(312)-582-		cearley@home.com	pacsdemo
Clark, Charles	2794329133	6/11/1973	43Y 1M	M	543 TORNADA BLVD. SAN DI	Not Known	(312)-582-		clark@home.com	pacsdemo
Davis, Dorothy	201201061	6/1/1944	72Y 6M	F	1515 W 18TH ST. CHICAGO, U	10/11/2015	(312)-580-	761-11-6271	davis@home.com	pacsdemo
Edwards, Elizabeth	201215021	2/15/1958	58Y 9M	F	2964 N SHEFFIELD AVE. CHC	Not Known			edwards@home.com	pacsdemo
Fox, Frances	1111022575	10/10/1969	47Y 1M	F	1578 N CLYBOURN AVE. CHC	Not Known	(312) 255 0		fox@home.com	pacsdemo
Garcia, Gloria	2000160919	9/3/1968	48Y 3M	F	2900 W BELMONT AVE. CHC	Not Known	(773) 604 8		garcia@home.com	pacsdemo
Green, Gary	201217101	8/19/1949	67Y 3M	M	65 STATE AV. BALTIMORE, US	11/8/2015 6	(312)-540-		green@home.com	pacsdemo
Hatcher, Amy	200967964	9/12/1939	77Y 2M	F	635 WEST JONES AV. CHICA	11/11/2015	(312)-433-		hatcher@home.com	pacsdemo
Homan, Candace	200990000	6/1/2002	14Y 6M	F	849 S SEPULVEDA BLVD. CH	Not Known	(312)-324-		homan@home.com	pacsdemo
Hopkins, Mary	200935486	10/19/1952	64Y 1M	F	65 HAMPTONS DRIVE, CHCA	10/15/2015	(312)-343-		hopkins@home.com	pacsdemo
Jenkins, Sarah	200934654	2/6/1990	26Y 9M	F	2342 W MAPLE, CHICAGO, U	10/15/2015	(312)-231-		jenkins@home.com	pacsdemo
Jones, Coleen	200954654	12/19/1972	43Y 11M	F	2312 LONG BEACH BLVD. LO	Not Known	(311)-230 0		jones@home.com	pacsdemo
March, Allan	200923756	10/19/1940	76Y 1M	M	1028 WALKER AV. BALTIMOR	11/11/2015	(312)-580-		march@home.com	pacsdemo
Miller, Bob	017258581	6/1/1967	50Y 6M	F	3123 N GEORGE ST. CHICAG	Not Known	(312)-543-		mler@home.com	pacsdemo
Nugent, Wanda A	200937878	10/28/1933	83Y 1M	F	123 LAKE ST. CHICAGO, US, IL	Not Known	(312)-344-		nugent@home.com	pacsdemo
Peterson, Andrea	201077874	1/1/2004	12Y 11M	F	34 LAKE ST. CHICAGO, US, IL	Not Known	(312)-432-		peterson@home.com	pacsdemo


All Lists

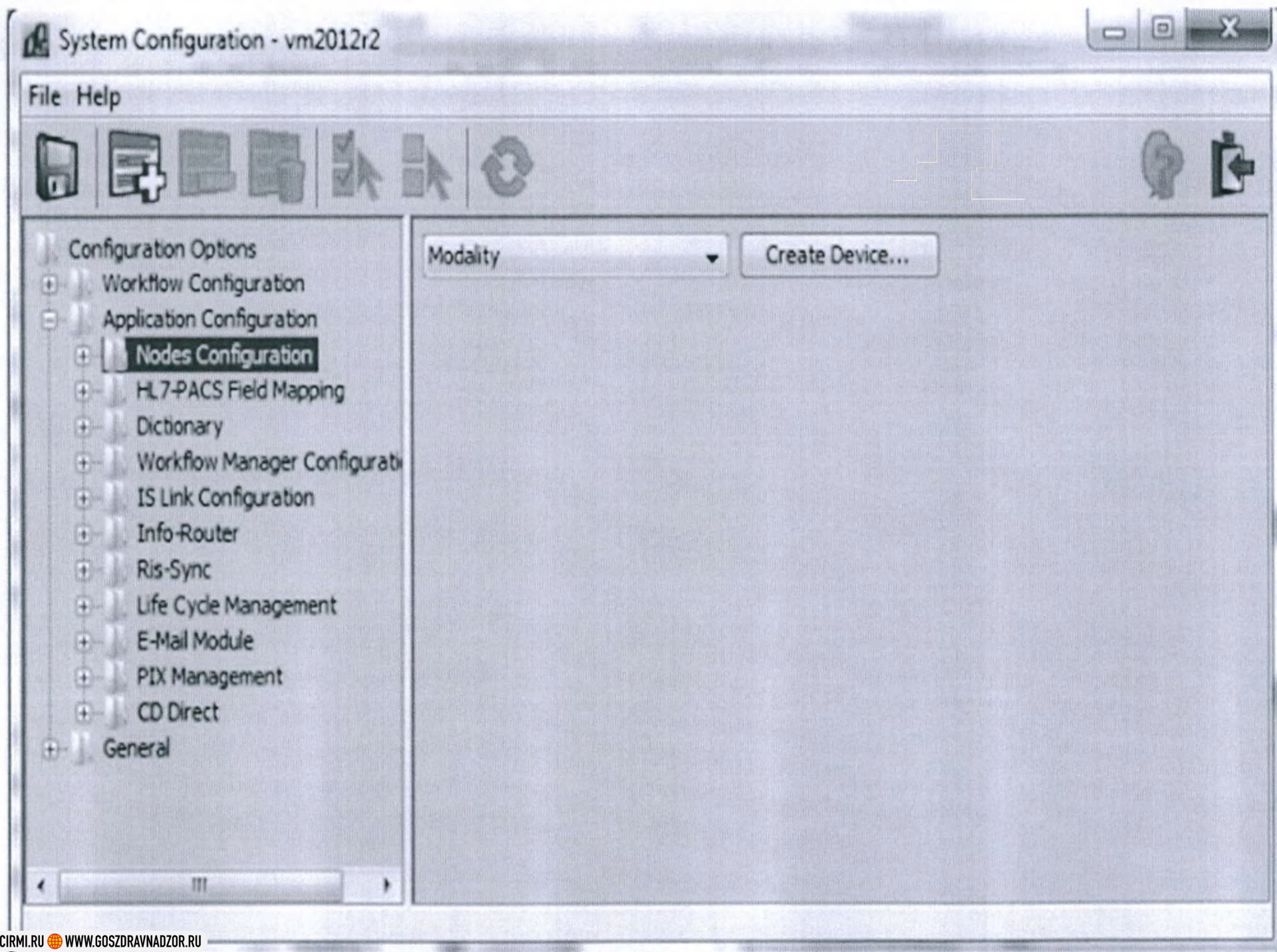


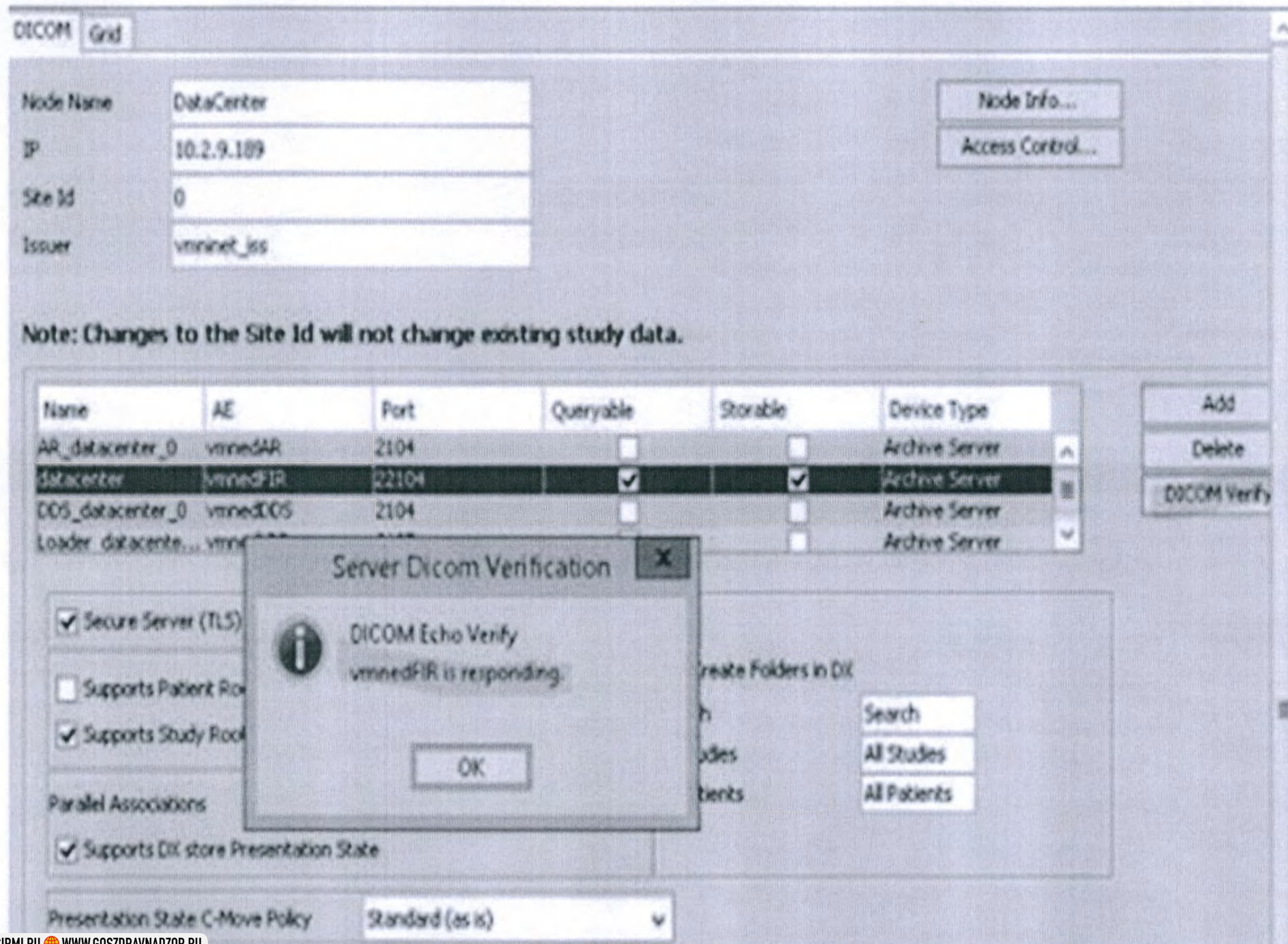

23. Ключ лицензионный для активации программного модуля DB1 F/UNLIMITED MAMMO LIC в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований

34

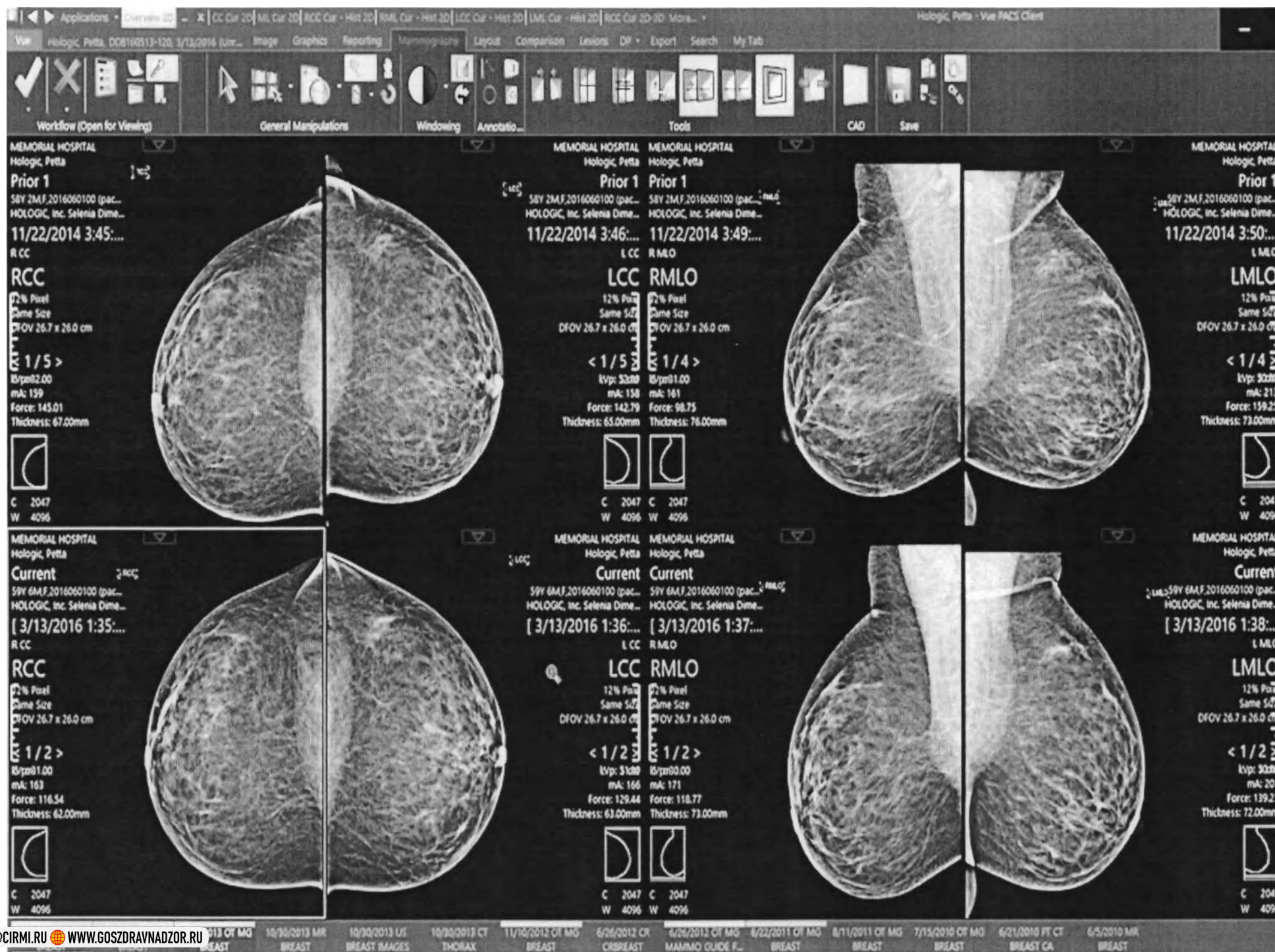

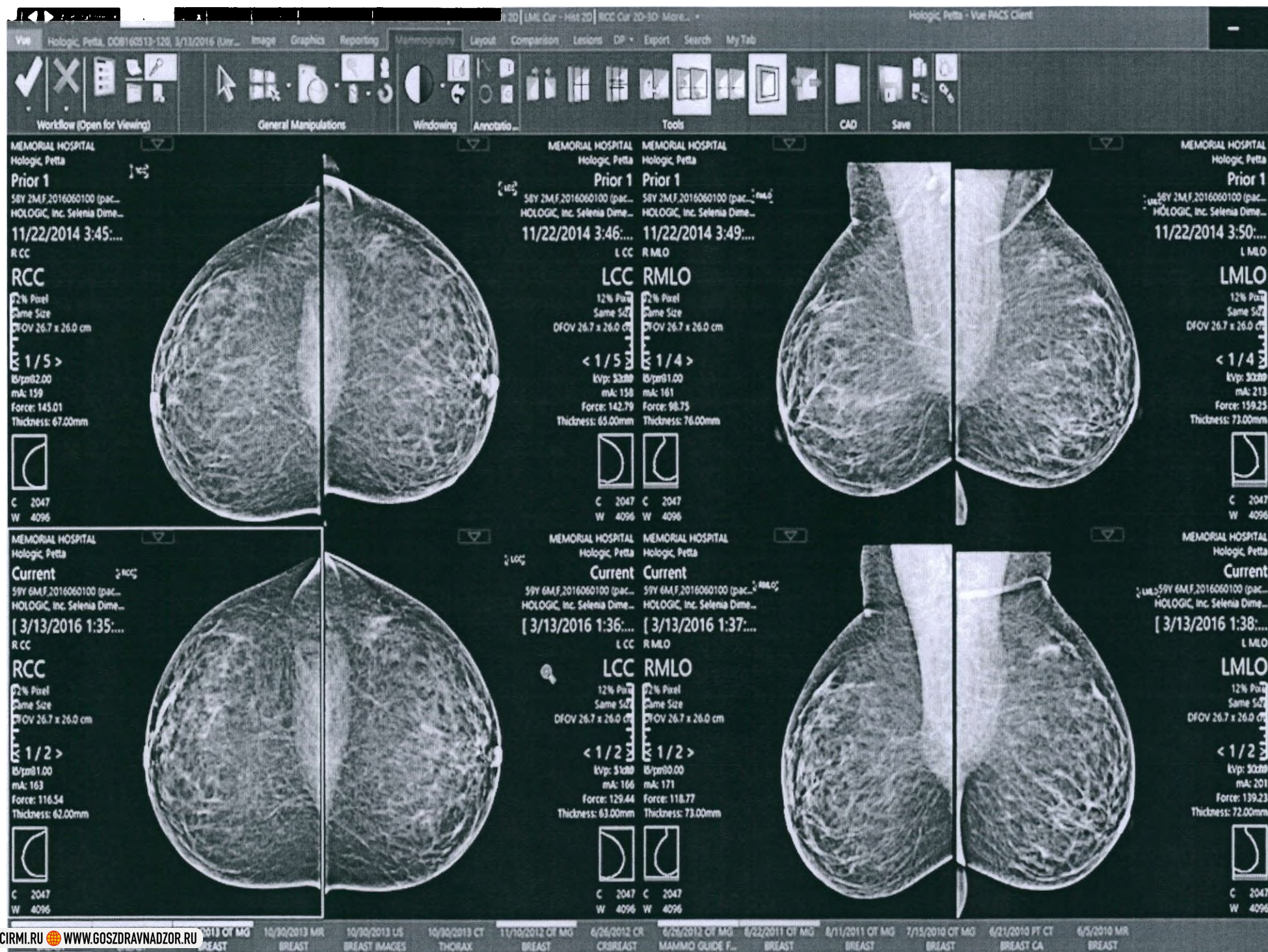







Patient chart
EM90-000465
Demo Patient

New order

Leigh Dymant
SGH Senior PSC
Ministry Of Health

Demo Patient
EM90-000465
 (Medical...)
 6/26/1990 (40y 10m)
 Male

(null) 0

N/A
 N/A
 N/A
 N/A
 N/A
 N/A
 N/A
 N/A

1160
 0
 4
 4461
 59
 28
 38
 1309
 762

Patient Chart

Exams

Future (1)

Order Signed

May 19 2021
 AC Joints X-ray, AP Non-weight Bearing

History (18/46) ☒ Show only performed exams

Completed Feb 6 2018 CT Abdomen	Approved Aug 21 2017 CT Abdomen and Pelvis	Dictated Aug 21 2017 Thoraco Lumbar Spine X-ray, AP and Lateral	Completed Aug 21 2017 CT 3D Head
Approved Aug 21 2017 MRA (Contrast Enhanced) Thoracic Aorta	Signed Aug 21 2017 CT 3D Pelvis	Started Jul 11 2017 CT Angio Liver (Pre/Post-LTX Work-Up)	Completed Jul 11 2017 CT Abdominal Wall

Clinical Indications

Weight
N/A

Height
N/A

Common Actions

Update PID

Reschedule

[EditScheduleArrive Act]

[NewOrderandSchedule]

[ScheduleOrderWalk-In.Action]

Edit patient

Audit log
 [StatusSuspend.Acti...]
 [Patient Letters.Action]
 View report
 [StatusPreparation.A...

User Management Admin

File Tools Help

Users

Groups

Profiles

Login Name	First Name	Last Name	Group Name	Profile Name	E-Mail	Last Login Date	Password Age	Status
administrator	administrator	administrator	administrators	Diagnostic		19-Feb-2015 17:50:48	63	ACTIVE
k	kate	r	administrators	Diagnostic		25-Feb-2015 18:41:40	46	ACTIVE
x	x	x	radiologists	Diagnostic		07-Apr-2015 15:10:46	63	ACTIVE

IS Link Administration Tool - vmlondon

File View Tools Help











Patients Visits Orders Reports

PATIENT DBID	ZID	PATIENT ID	ISSUER	NAME FAMILY	NAME GIVEN	NAME MIDDLE	NAME PREFIX	NAME SUFFIX	NAME DEGREE	BIRTH DATETI
39	n... 11111	vmilon... 22222	33333							06-Sep-2015
24	n... 111	vmilon... NATI	111							06-Sep-2015
45	n... 123	vmilon... 123	123							06-Sep-2015

Number of rows : 3 out of 3

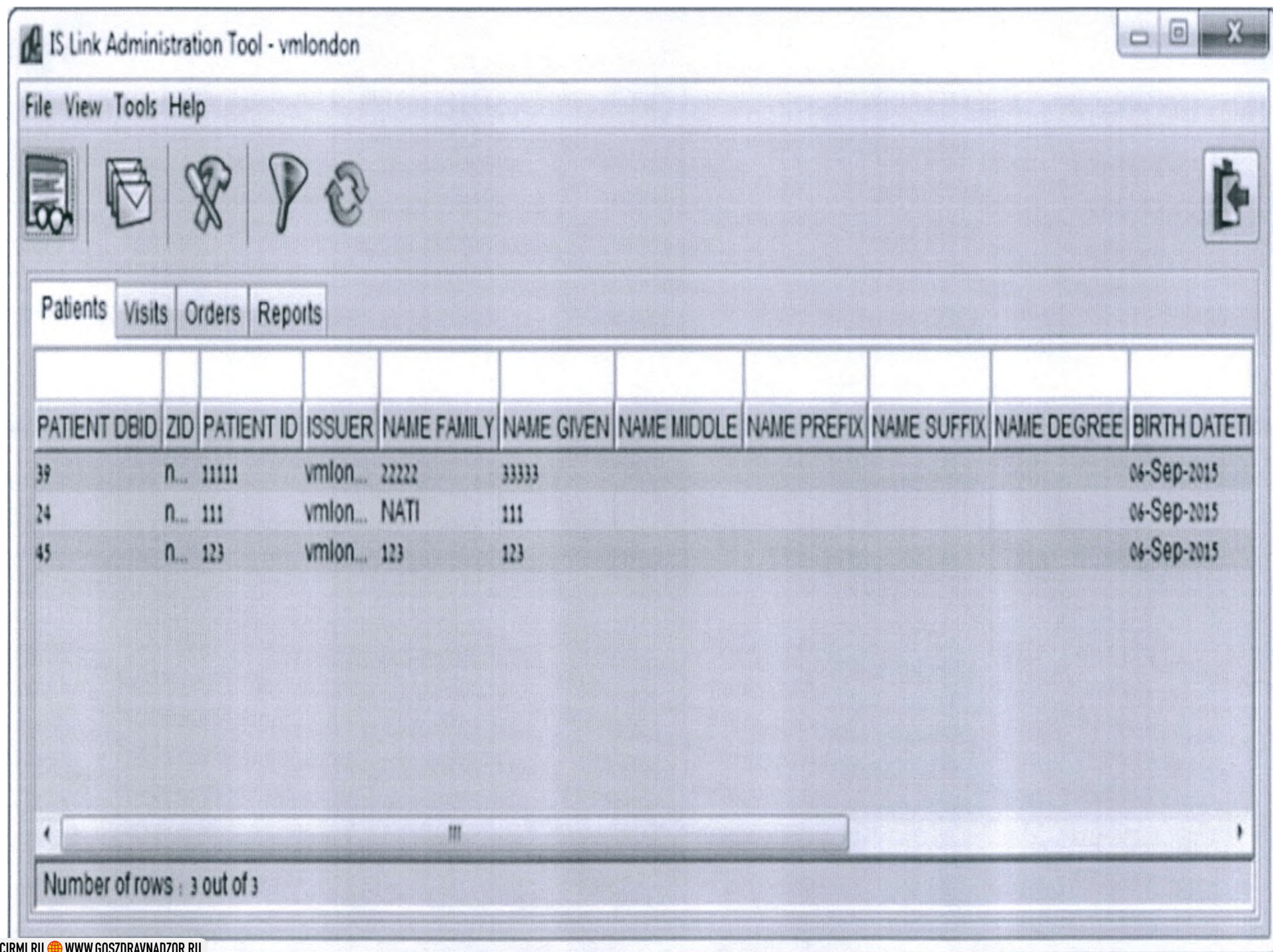

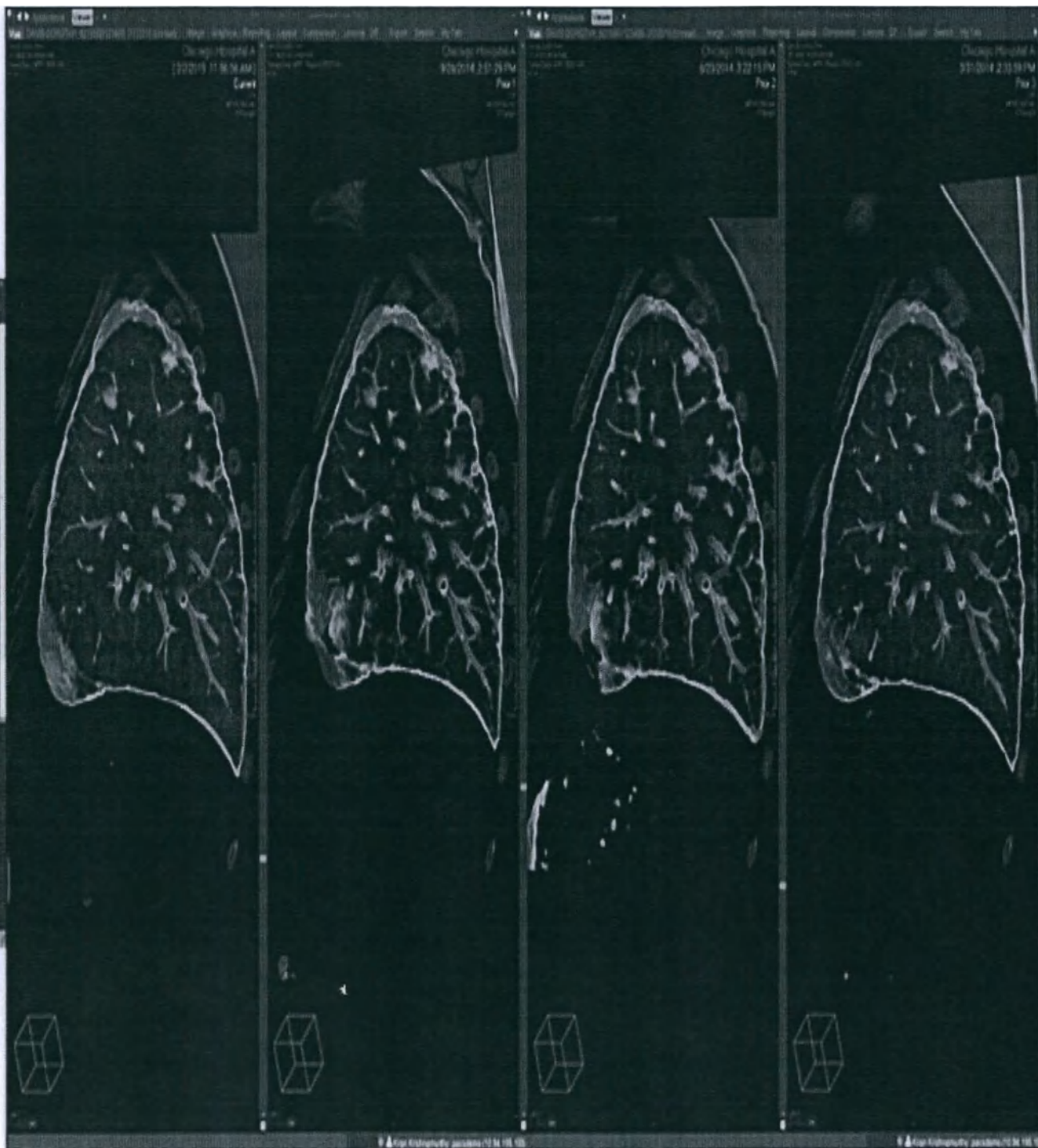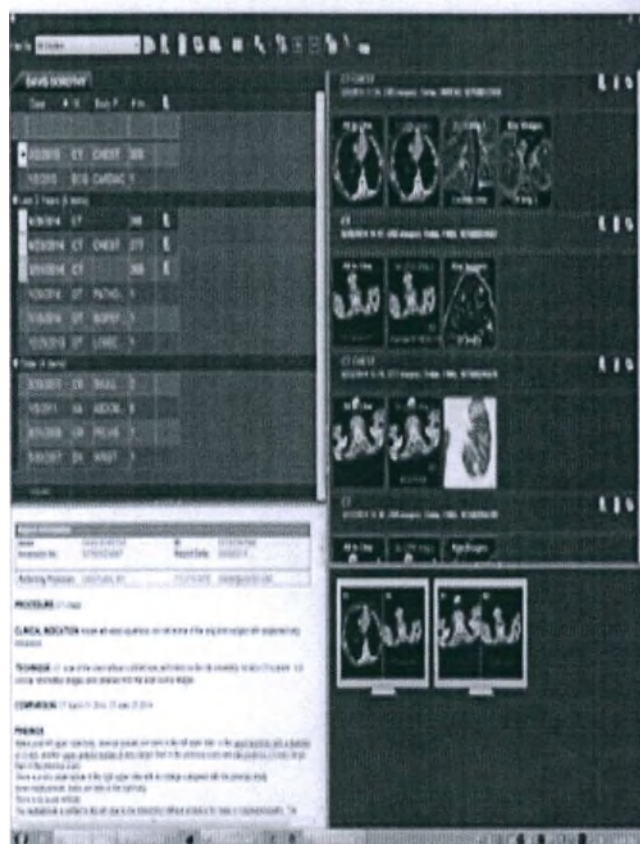

Carestream Vue Explorer X Carestream Vue Explorer X

10.94.195.222/archive/default.aspx/

Archive Explorer Green, Gary

Green, Gary Gender M Age 67Y, 3M ID 201217101972 Edit Exam Details

Dermatolog... Browse Y Search

☐	Name	Created	Size
☐	Thumbs.db	26/11/16 15:49:08	16.5K
☐	Herpes Zo...	27/11/15 09:33:57	590.0K
☐	O-Sm-m-Me...	21/10/15 06:15:54	1.61M
☐	Asymmetri...	21/10/15 06:15:54	3.02M
☒	Melanoma...	21/10/15 06:15:54	736.0K
☐	Mole.jpg	21/10/15 06:15:54	8.31M
☐	Normal re...	21/10/15 06:15:54	2.84M
☐	Pigmentati...	21/10/15 06:15:54	172.1K
☐	Sun pigme...	21/10/15 06:15:54	490.0K



Enter comment here

Descent

Body part

Mole Shape

Mole border type

Mole color


Mole size


Mole surface


Type

70% [Icons]

Remove all files upon storing Create Exam (0 Files)

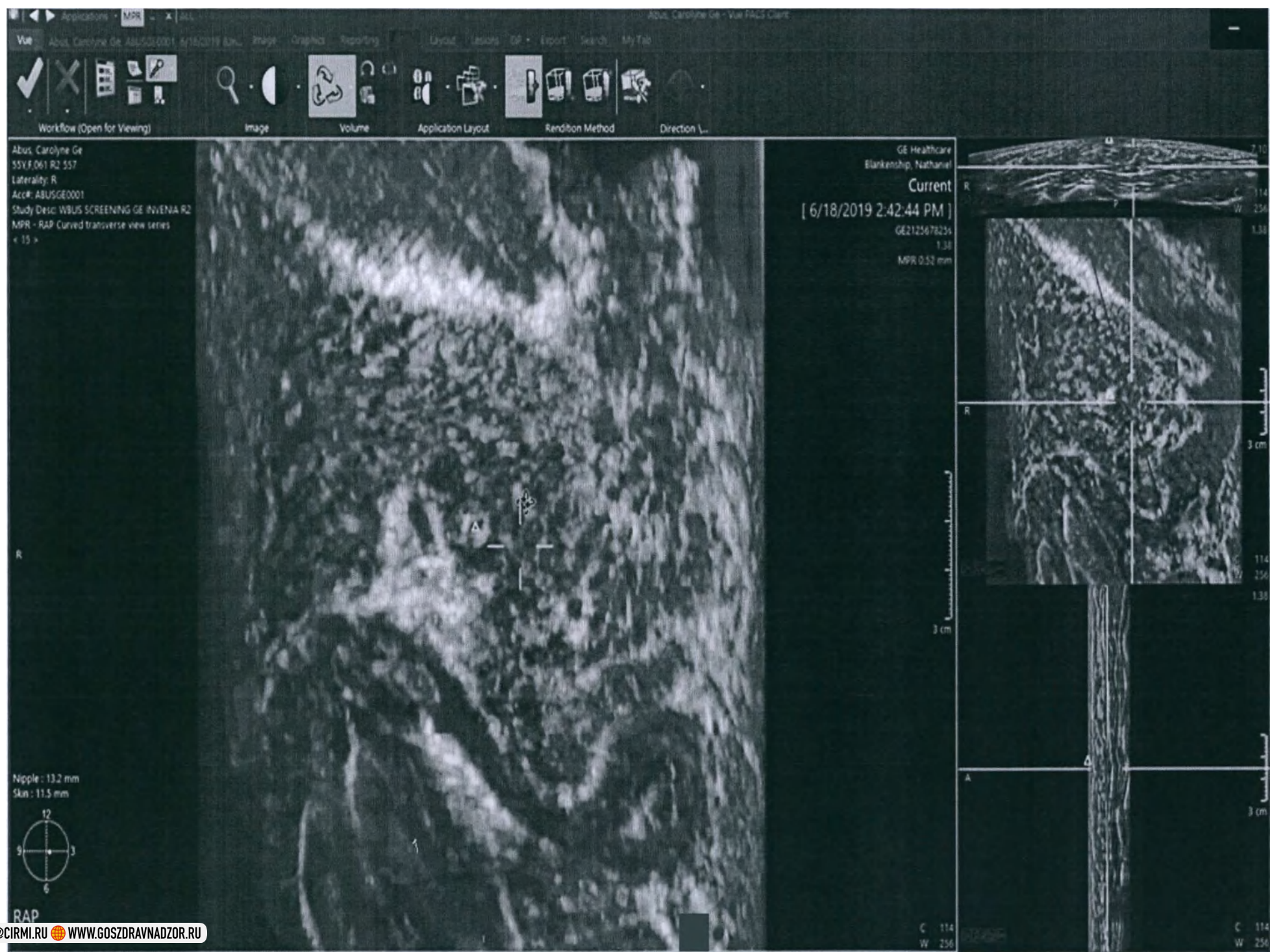

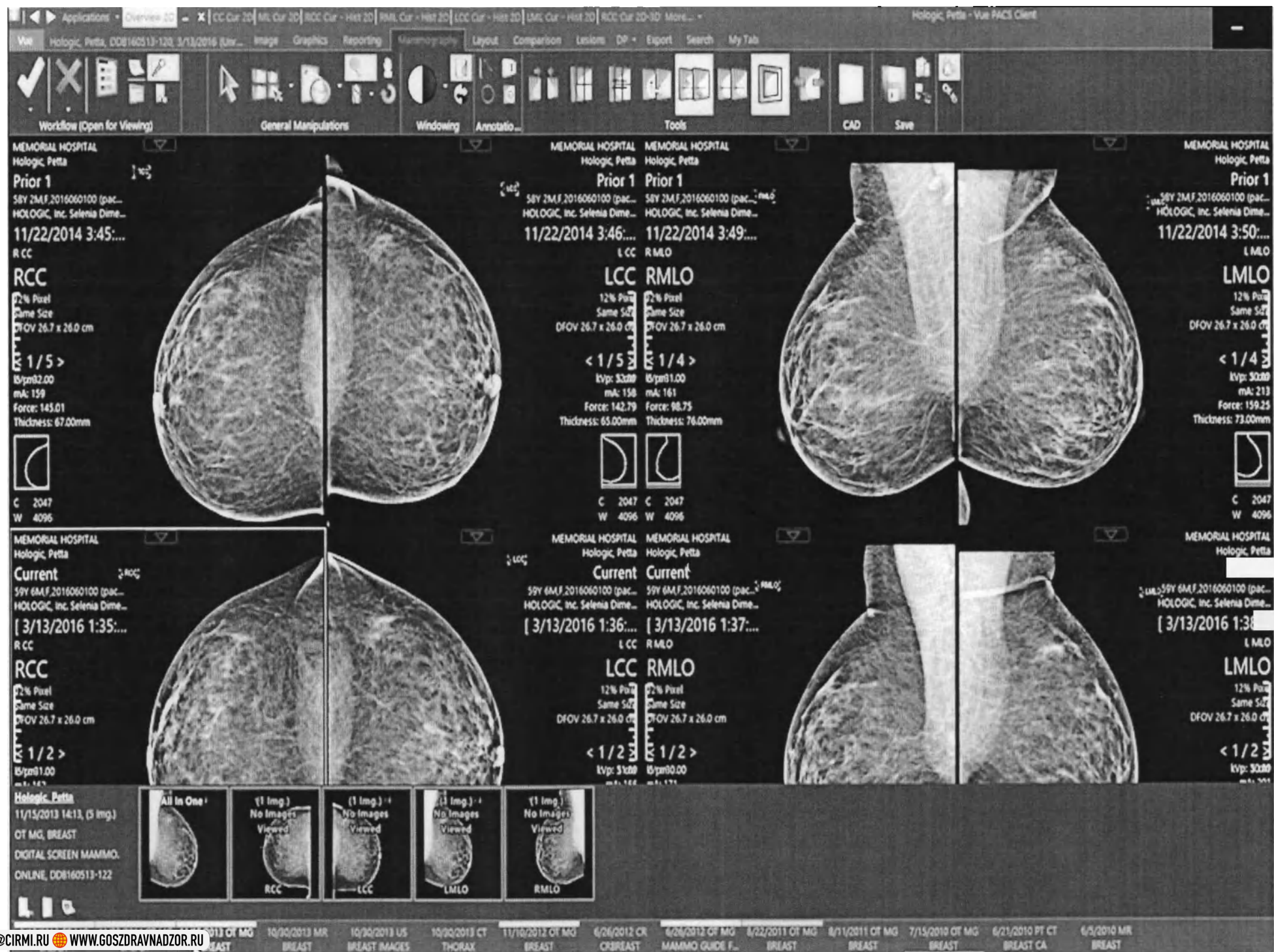

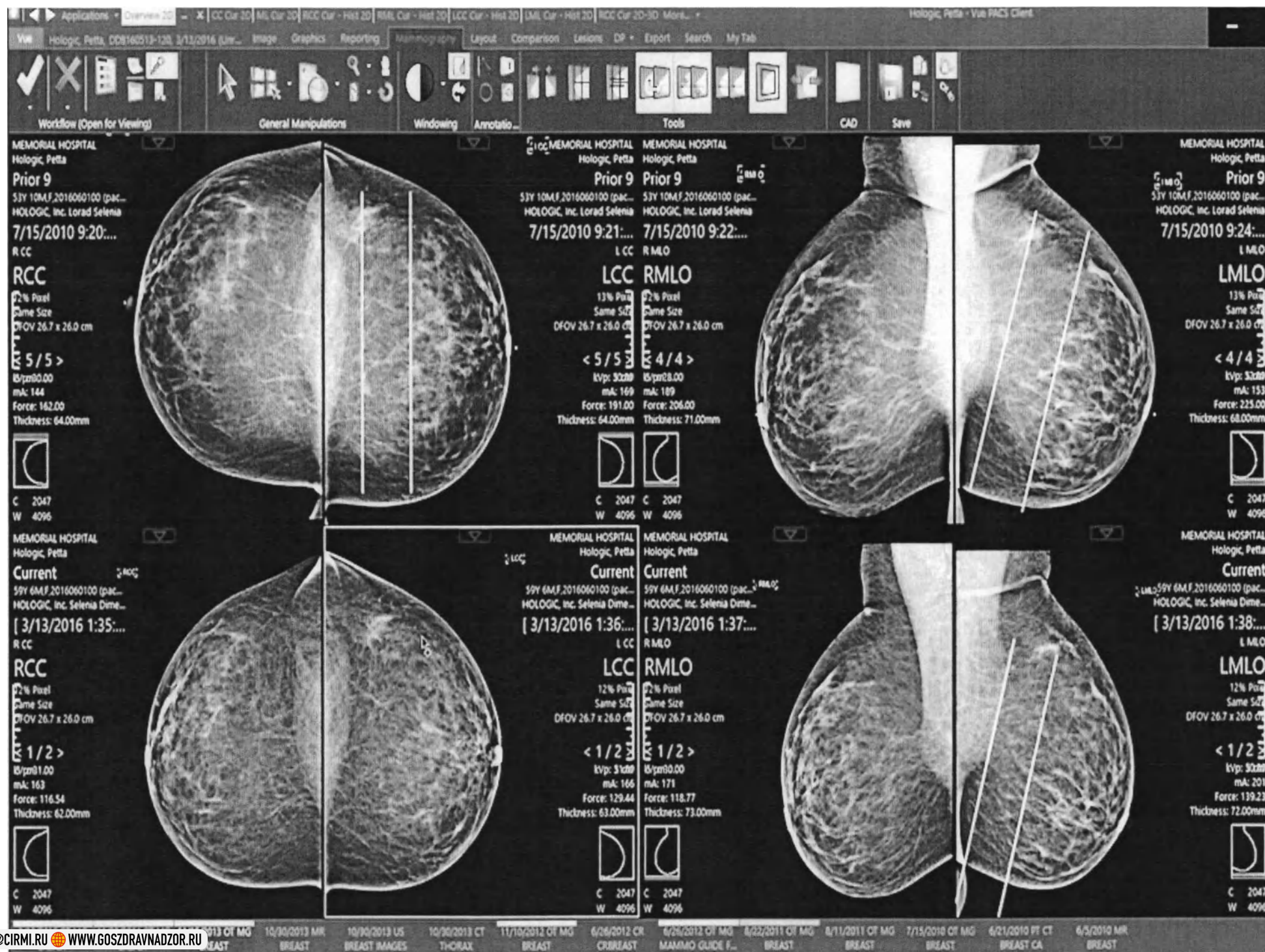

Add New Device

DICOM

Node Name

IP

Node Info...

Access Control...

Name	AE	Port
PRINTER_13		104

Add

Delete

DICOM Verify

Select printer kind:

☐ Set as Default Printer


Printer configuration

☐ The AE Titles configured above are Virtual AES

Outgoing Dicom AE

Transfer Syntax Policy

Add Cancel

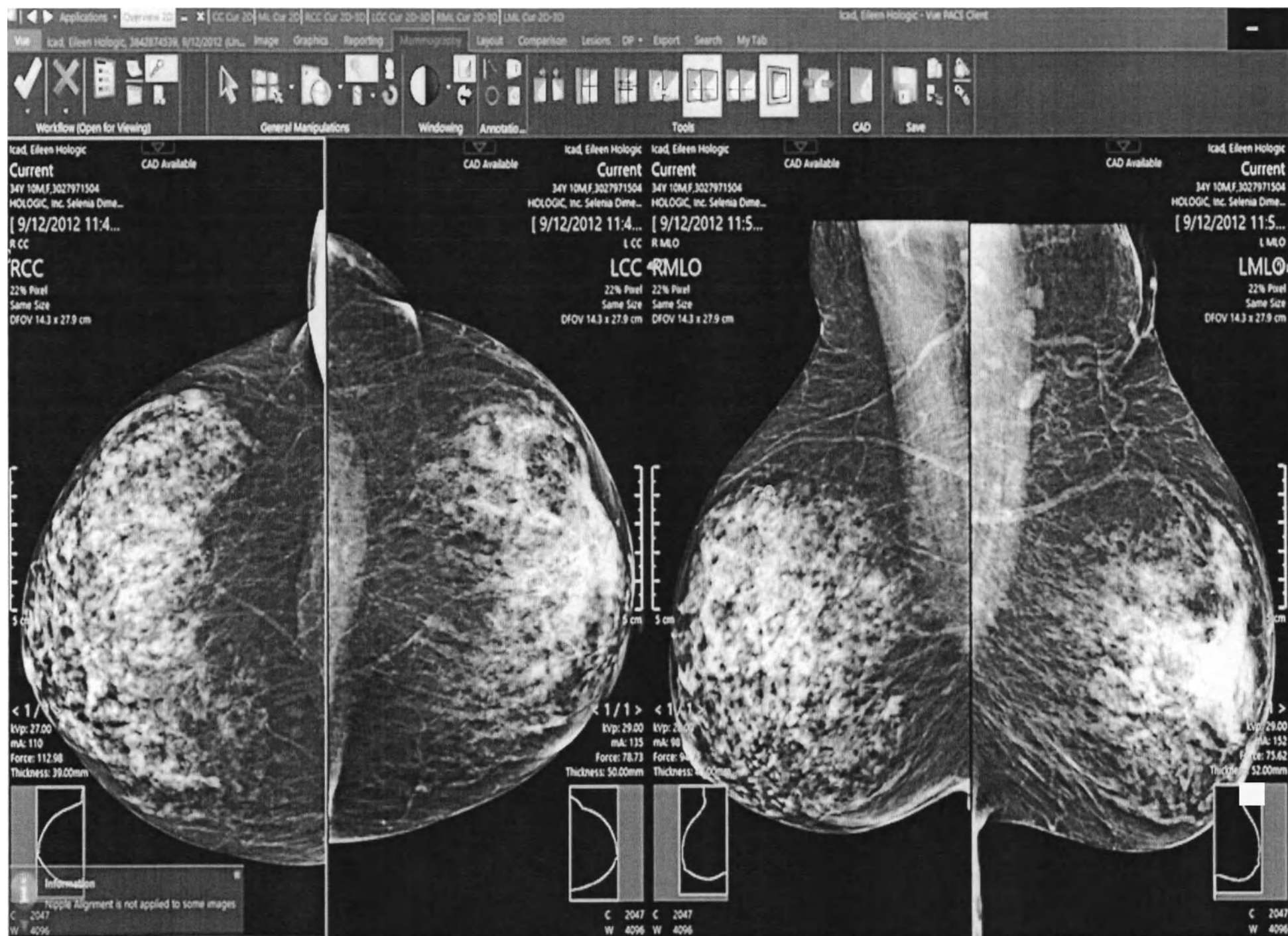

Archive Explorer

Search Console

Nottingham, Belle Michigan Medical Center

Worklist

Home View Manage Acquire Organize Share

My Lists

Diagnose View Motion Batch Sign View Report Print Report Order

Mammography

Reported Patients

Performed Procedures

Local Patients

ALL InProgress

Prio	Date	Patient Name	Age	Mod	Body Part	Devices	Status	Custom Workflow	Images
Low	6/16/2019	Abus, Carolyn Ge	55Y	US	BREAST	WBUS SCREENING GE INVENIA R2	Unread	mammo i	6
Low	1/16/2017	Diagnose, Diane	89Y 5M	OT MG	BREAST		Unread	mammo diag	10
Low	2/9/2014	Gerhart, Teresa		OT MG	BREAST		Unread	mammo c MG	5
Low	7/17/2017	Hohelomo, Kim		OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo f MG	19
Low	1/1/2016	Holdway, Irene	56Y 9M	OT MG	BREAST	BI, MLOs and CCs + TOMO	Unread	mammo f MG	10
Low	12/14/2013	Hollimplant, Leann	56Y 9M	OT MG	BREAST		Unread	mammo d MG	9
Low	4/19/2016	Hologic, Dora	59Y 7M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo MG	33
Low	3/13/2016	Hologic, Patsy	59Y 6M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo 1c	13
Low	11/23/2019	Hologic_3dquorum, Comb	55Y 10M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo j	19
Low	11/27/2019	Holomo, Renee	49Y 10M	OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo g MG	38
Low	9/12/2012	Icad, Eileen Hologic	34Y 10M	OT MG	BREAST	MG SCREENING ICAD TOMO	Unread	mammo aa	12
Low	7/10/2019	Ickett, Jolene	62Y 3M	OT MG	BREAST	SCREENING MAMMO BILAT WICAD	Unread	mammo a MG	7
Low	7/12/2019	Linton, Isabella	70Y 10M	US		US BREAST COMPLETE BILATERAL	Unread	mammo gb	46
Low	1/2/2020	Linton, Isabella	71Y 3M	MR	BREAST	MR BREAST WITHOUT AND WITH IV CONTRAST	Unread	mammo ga	3188
Low	1/14/2020	Mcdonald, Clair	57Y 3M	OT MG	BREAST	MA MAMMOGRAM SCREENING BIL W TOMO CO	Unread	mammo 1a MG	13
Low	12/10/2017	Selesia, Lilly Hologic	69Y 11M	OT MG	BREAST	MG SCREENING HD TOMO HOLOGIC	Unread	mammo f MG	11
Low	7/25/2010	Tilley, Fran	73Y 7M	OT MG	BREAST	MA Mammogram Digital Bilateral	Unread	mammo 1b MG	9

All Lists

Quality

Recently Viewed

Local Drive

Research

40. Ключ лицензионным для активации программного модуля MX WEB SVK W/3 MX-U LIC в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, не более 75 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 иссле-

51

Archive Explorer										
Worklist My Lists Mammography Reported Patients Performed Procedures Local Patients ALL InProgress All Lists Quality Recently Viewed Local Drive Research Home View Manage Acquire Organize Share										
Diagnose View Motion Batch Sign View Studies Print Report Order										
★ Mammography										
Priority	Date	Patient Name	Age	Mod	Body Part	Description	Status	Customs Worklist	Images	
Low	6/18/2019	Abus, Carolyn Ge	55Y	US	BREAST	WBUS SCREENING GE INVENIA R2	Unread	mammo i	6	
Low	1/16/2017	Diagnose, Diane	69Y 5M	OT MG	BREAST		Unread	mammo diag	10	
Low	2/9/2014	Gerhart, Teresa		OT MG	BREAST		Unread	mammo c MG	5	
Low	7/17/2017	Holstomo, Kim		OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo f MG	19	
Low	1/1/2016	Holdway, Irene	56Y 9M	OT MG	BREAST	BL MLOs and CCs + TOMO	Unread	mammo f MG	10	
Low	12/14/2013	Hollimplant, Leann	56Y 9M	OT MG	BREAST		Unread	mammo d MG	9	
Low	4/19/2016	Hologic, Delta	59Y 7M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo MG	33	
Low	3/13/2016	Hologic, Peta	50Y 6M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo 1c	13	
Low	11/23/2019	Hologic_3dquorum, Comb	55Y 10M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo j	19	
Low	11/27/2019	Holomo, Renee	45Y 10M	OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo g MG	38	
Low	9/12/2012	Icad, Eileen Hologic	34Y 10M	OT MG	BREAST	MG SCREENING ICAD TOMO	Unread	mammo aa	12	
Low	7/10/2019	Ickert, Jolene	62Y 5M	OT MG	BREAST	SCREENING MAMMO BILAT WICAD	Unread	mammo a MG	7	
Low	7/12/2019	Unlon, Isabella	70Y 10M	US		US BREAST COMPLETE BILATERAL	Unread	mammo gb	46	
Low	1/2/2020	Unlon, Isabella	71Y 3M	MR	BREAST	MR BREAST WITHOUT AND WITH IV CONTRAST	Unread	mammo ga	3168	
Low	1/14/2020	McDonald, Clair	57Y 3M	OT MG	BREAST	MA MAMMOGRAM SCREENING BIL W TOMO CO	Unread	mammo 1a MG	13	
Low	12/10/2017	Selenia, Lily Hologic	69Y 11M	OT MG	BREAST	MG SCREENING HD TOMO HOLOGIC	Unread	mammo f MG	11	
Low	7/25/2010	Tiley, Fran	73Y 7M	OT MG	BREAST	MA Mammogram Digital Bilateral	Unread	mammo 1b MG	9	

Archive Explorer

Home View Manage Actions Organize Share

★ My Lists

★ Mammography

Reported Patients

Performed Procedures

Local Patients

ALL In Progress

Pro	Date	Patient Name	Age	Mod	Body Part	Description	Status	Custom Workflow	# Images
Low	5/15/2019	Abus, Carolyn Ge	55Y	US	BREAST	WBUS SCREENING GE INVENIA R2	Unread	mammo i	6
Low	1/16/2017	Diagnose, Diane	59Y 5M	OT MG	BREAST		Unread	mammo diag	10
Low	2/9/2014	Gerhart, Teresa		OT MG	BREAST		Unread	mammo c MG	5
Low	7/17/2017	Hofelomo, Kim		OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo f MG	19
Low	1/1/2016	Holdway, Irene	56Y 9M	OT MG	BREAST	BI, MLOs and CCs + TOMO	Unread	mammo f MG	10
Low	12/14/2013	Holmplant, Leann	56Y 9M	OT MG	BREAST		Unread	mammo d MG	9
Low	4/19/2016	Hologic, Delta	59Y 7M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo MG	33
Low	3/13/2016	Hologic, Delta	59Y 6M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo f c	13
Low	11/23/2019	Hologic_3dquorum, Comb	55Y 10M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo j	19
Low	11/27/2019	Holomo, Renee	49Y 10M	OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo g MG	38
Low	3/12/2012	Icad, Eileen Hologic	34Y 10M	OT MG	BREAST	MG SCREENING ICAD TOMO	Unread	mammo aa	12
Low	7/10/2019	Ickert, Jolene	62Y 5M	OT MG	BREAST	SCREENING MAMMO BIAT WICAD	Unread	mammo a MG	7
Low	7/12/2019	Linon, Isabella	70Y 10M	US		US BREAST COMPLETE BILATERAL	Unread	mammo gb	46
Low	1/2/2020	Linon, Isabella	71Y 3M	MR	BREAST	MR BREAST WITHOUT AND WITH IV CONTRAST	Unread	mammo ga	3188
Low	1/14/2020	Mcdonald, Clair	57Y 3M	OT MG	BREAST	MA MAMMOGRAM SCREENING BIL W TOMO CO	Unread	mammo 1a MG	13
Low	12/10/2017	Selena, Lily Hologic	69Y 11M	OT MG	BREAST	MG SCREENING HD TOMO HOLOGIC	Unread	mammo f MG	11
Low	7/25/2010	Tilley, Fran	73Y 7M	OT MG	BREAST	MA Mammogram Digital Bilateral	Unread	mammo 1b MG	9

All Lists

Quality

Recently Viewed

Local Drive

Research

1 - 17 of 17 items

Archive Explorer

Nottingham, Belle Michigan Medical Center

Home View Montage Acquire Organize Share

Diagnose View Motion Batch Sign View Report Print Report Order

Mammography

Reported Patients

Performed Procedures

Local Patients

All In Progress

Find	Date	Patient Name	Age	Mod	Body Part	Description	Status	Custom Workflow	T	Images
Low	6/16/2019	Abus, Caroline Ge	55Y	US	BREAST	WBUS SCREENING GE INVENIA R2	Unread	mammo i		6
Low	1/16/2017	Diagnose, Diane	59Y 5M	OT MG	BREAST		Unread	mammo diag		10
Low	2/9/2014	Gehart, Teresa		OT MG	BREAST		Unread	mammo c MG		5
Low	7/17/2017	Hohelomo, Kim		OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo f MG		19
Low	1/1/2016	Holdway, Irene	56Y 5M	OT MG	BREAST	B/L MLOs and CCs + TOMO	Unread	mammo f MG		10
Low	12/14/2013	Hollenplast, Leann	56Y 5M	OT MG	BREAST		Unread	mammo d MG		9
Low	4/19/2016	Hologic, Delta	59Y 7M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo MG		33
Low	3/13/2016	Hologic, Peta	59Y 6M	OT MG	BREAST	Standard Screening-Combo HD	Unread	mammo f c		13
Low	11/23/2019	Hologic_3dquorum, Comb	55Y 10M	OT MG	BREAST	Standard Screening - Combo HD	Unread	mammo j		19
Low	11/27/2019	Holomo, Renee	49Y 10M	OT MG	BREAST	MG SCREENING WITOMO	Unread	mammo g MG		38
Low	9/12/2012	Icad, Eileen Hologic	34Y 10M	OT MG	BREAST	MG SCREENING ICAD TOMO	Unread	mammo aa		12
Low	7/10/2019	Ickert, Jolene	62Y 5M	OT MG	BREAST	SCREENING MAMMO BILAT W/ICAD	Unread	mammo a MG		7
Low	7/12/2019	Linton, Isabella	70Y 10M	US	BREAST	US BREAST COMPLETE BILATERAL	Unread	mammo gb		46
Low	1/2/2020	Linton, Isabella	71Y 3M	MR	BREAST	MR BREAST WITHOUT AND WITH IV CONTRAST	Unread	mammo ga		3188
Low	1/14/2020	McDonald, Clair	57Y 3M	OT MG	BREAST	MA MAMMOGRAM SCREENING BIL W TOMO CO	Unread	mammo fa MG		13
Low	12/10/2017	Selenia, Lily Hologic	69Y 11M	OT MG	BREAST	MG SCREENING HD TOMO HOLOGIC	Unread	mammo f MG		11
Low	7/25/2010	Tilley, Fran	73Y 7M	OT MG	BREAST	MA Mammogram Digital Bilateral	Unread	mammo fb MG		9

All Units


Quality

Recently Viewed

Local Drive

Research

1 - 17 of 17 items

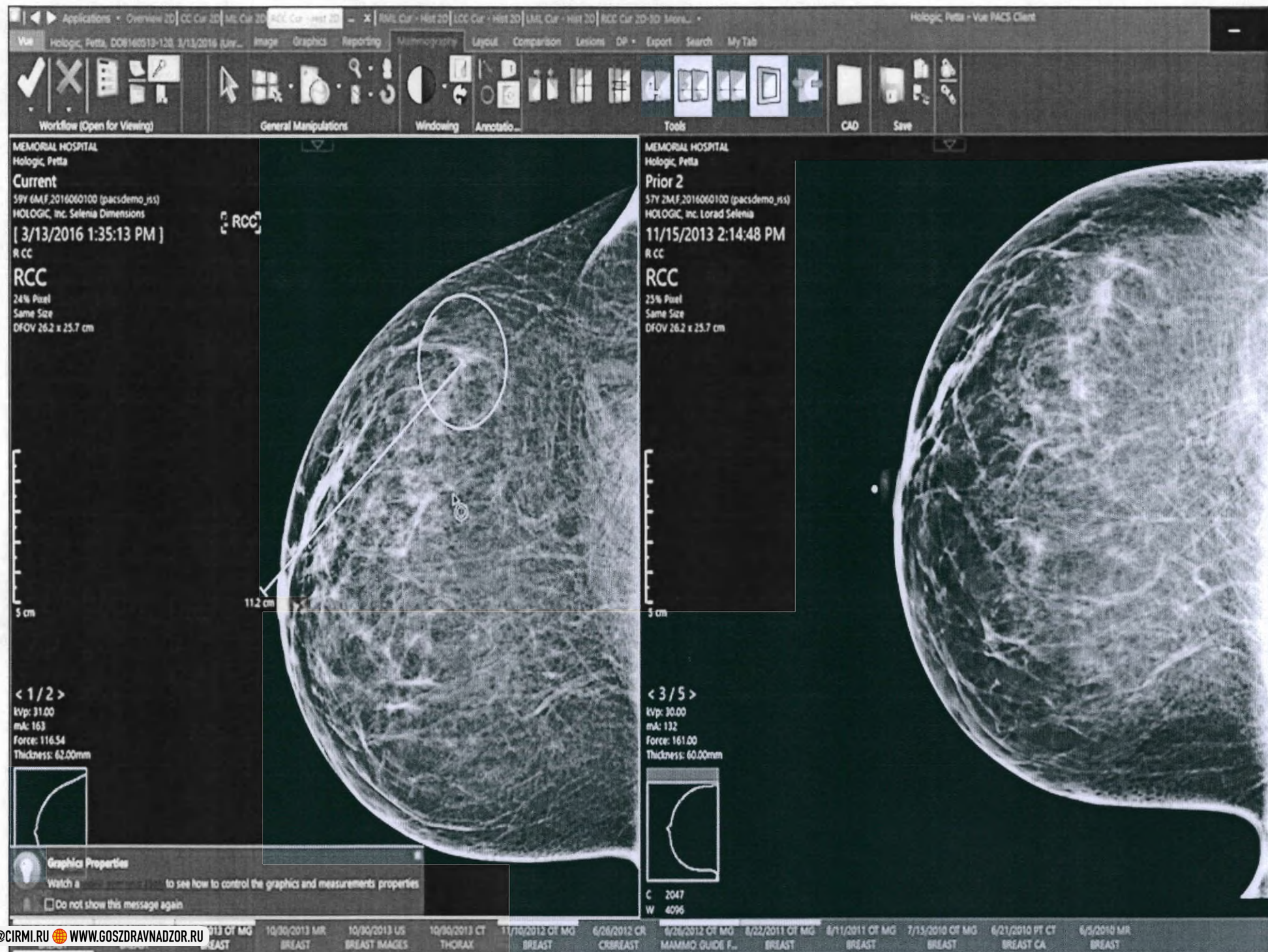

 ООО ЦИПМИ  INFO@CIRMI.RU  WWW.GOSZDRAVNADZOR.RU

LTSM Configuration

Wakeup period to check for new studies that need to be backed up: 15 minutes

Maximum pending studies waiting to be backed up (new studies): 50

Maximum pending studies waiting to be re-backed up (retries): 100

Period to wait before the backup of a new study will be executed: 15 minutes

☐ Continue to re-back studies that previously failed to be backed up

☒ Backup fragmented studies

File System / External backup Device configuration

Automatically mark studies as "backed up" after: 0.1 hours

Wakeup period to check completion of studies backed up: 0.1 hours

Wakeup period to insert studies into backup table: 12 hours

Storage Commitment

☐ Enable Storage Commitment feature

Wakeup period to check for studies that require storage commit request to be sent: 0.1 hours

Waiting time after backup is completed before sending storage commitment request: 0.1 hours

 ООО ЦИПМИ  INFO@CIRMI.RU  WWW.GOSZDRAVNADZOR.RU

ООО ЦИРМИ  INFO@CIRMI.RU  WWW.GOSZDRAVNADZOR.RU

March, Allan - Explorer Portal

Archive Explorer March, Allan

Name: March, Allan Date of Birth: 10/19/1942 (76Y 11M) Current exam Modality: CT Body Part: CHEST Description: CT Chest w/ Contrast
 Patient ID: 200932796456 Gender: M Exam Date: 10/5/2019 Accession Number: 9275000039533 Referring Physician: Evans David New Item

Exams

Date	M	Body Part	# Ss	Has
10/5/2019	CT	CHEST	278	false
1/30/2019	CT	THX	275	true
1/5/2019	E	CARDIAC	1	false
4/7/2018	OT	PATHO	3	false

CHEST CT 9275000039533
 10/5/2019 3:21:00 PM (278 Images) ONLINE UNREAD

Thumbnails Report Order Bookmarks Exam Data Documents Critical Findings (1)
 Study Notes (1)

Exam

CT Chest 12:15 05/10/19
 CT Chest with contrast

Technologist	Modality	Notes for radiologist	Protocol name
Tse'ela Mida	CT room 1	follow up case	CT Chest/Abd/Pel - Oncology F/U

Order Information

Reason for order	Relevant history	Question for radiologist	Notes
Prostate cancer with metastases - follow-up	prostate cancer with metastases. Received radiation therapy	metastases follow up	Patient is in pains

Ordering organization	Referring physician	Patient type
Oncology	Aviad Danziger	Out Patient

March, Allan - Explorer Portal

Active Explorer March, Allan

Name: March, Allan Date of Birth: 10/19/1942 (76Y 11M) Current exam Modality: CT Body Part: CHEST Description: CT Chest w/ Contrast
 Patient ID: 200932798456 Gender: M Exam Date: 10/5/2019 Accession Number: 9275000039533 Referring Physician: Evans David New Item

Exams

Date	Modality	Body Part	# Images	Has
10/5/2019	CT	CHEST	278	None
1/30/2019	CT	THX	275	True
1/5/2019	E	CARDIAC	1	False
4/7/2018	OT	PATHO	3	False

CHEST CT 9275000039533
 10/5/2019 3:21:00 PM (278 images) ONLINE UNREAD

Thumbnails Report Order Bookmarks Exam Data Documents Critical Findings (1)

Study Notes (1)

Documents (1)

Scanned order
 1 Scanned order

Scanned order - Scanned order 1 / 1


Philips Healthcare
 Imaging Department


Diagnostic Exam ORDER

1 ☐ Male 2 ☐ Female	☐ Routine ☐ Urgent	Scheduled Exam Date	
☐ Allergic to Drugs	☐ On isolation LMP	☐ Pregnant	
Ordering Provider (Print)		Attending Physician (Print)	
Ordering Provider (Signature)		Provider's ID Code	Phone or Beeper
Insurance authorization number if applicable:			

STAMP PATIENT'S IDENTIFICATION OR PRINT CLEARLY


Patient's Name (LAST, FIRST, MI)

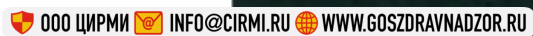


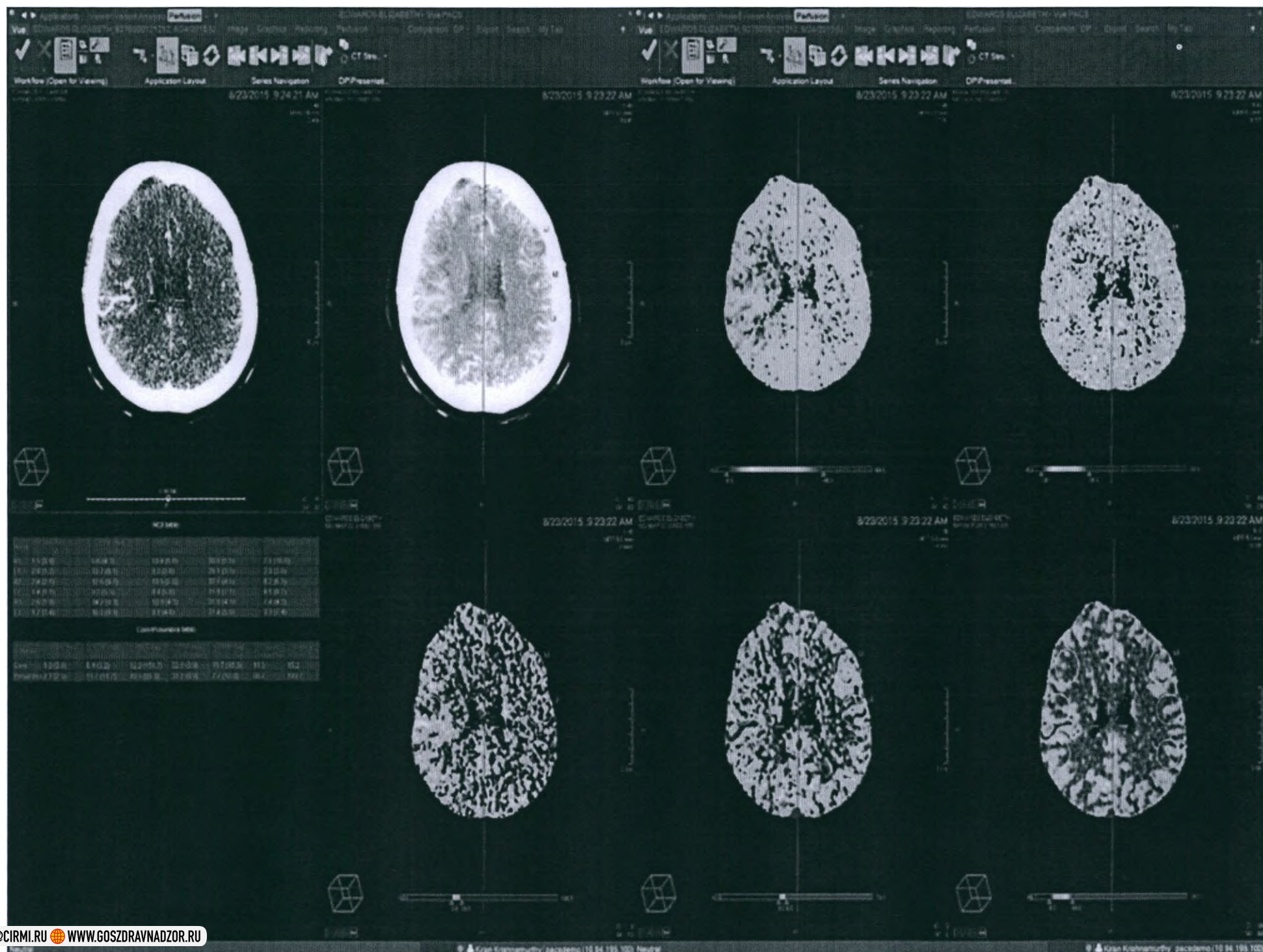



32. Ключ лицензионный для активации программного модуля PERFUSION / DIFFUSION UNLIMITED LIC в вариантах исполнения: не более 7 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований


63

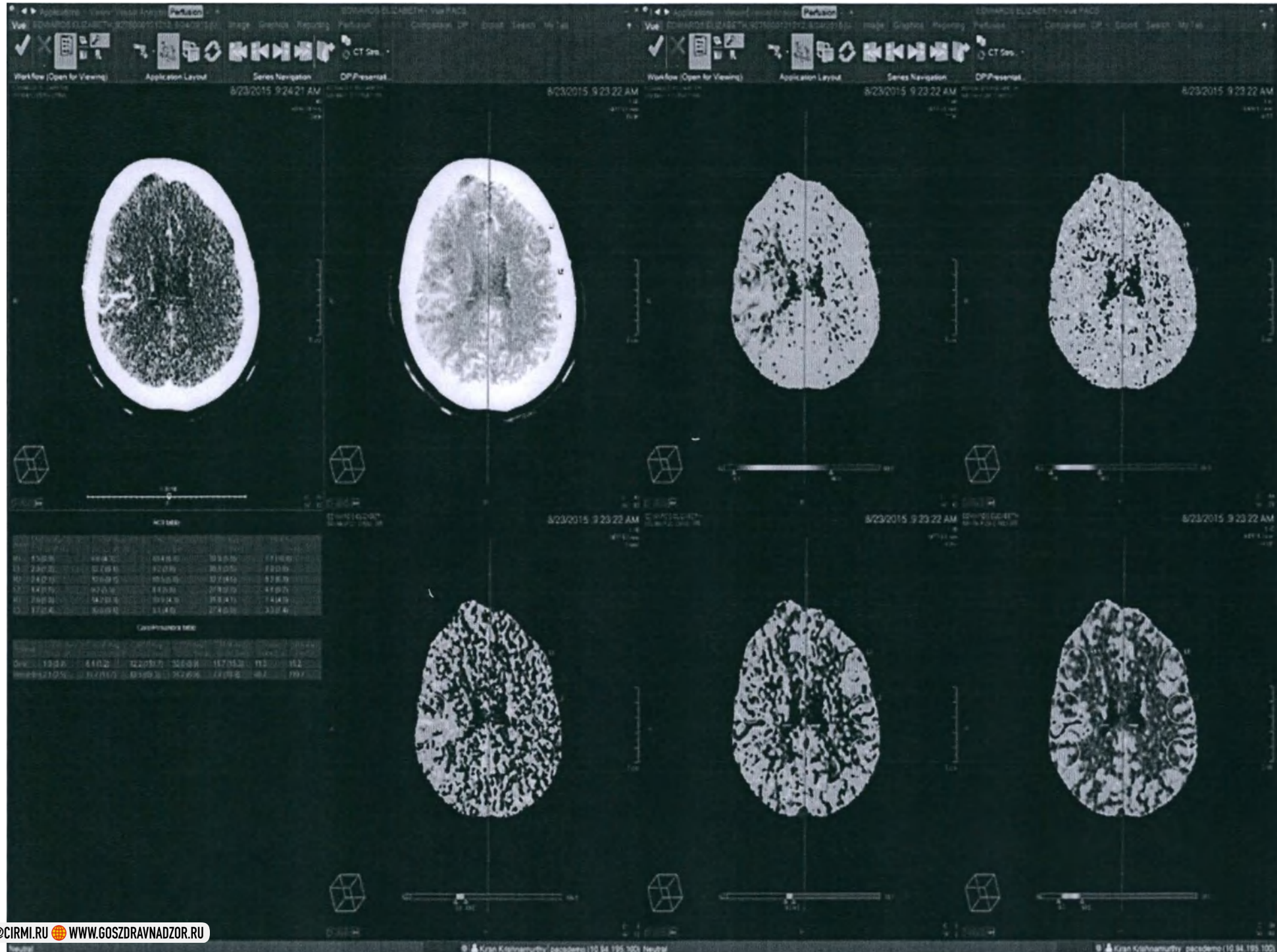

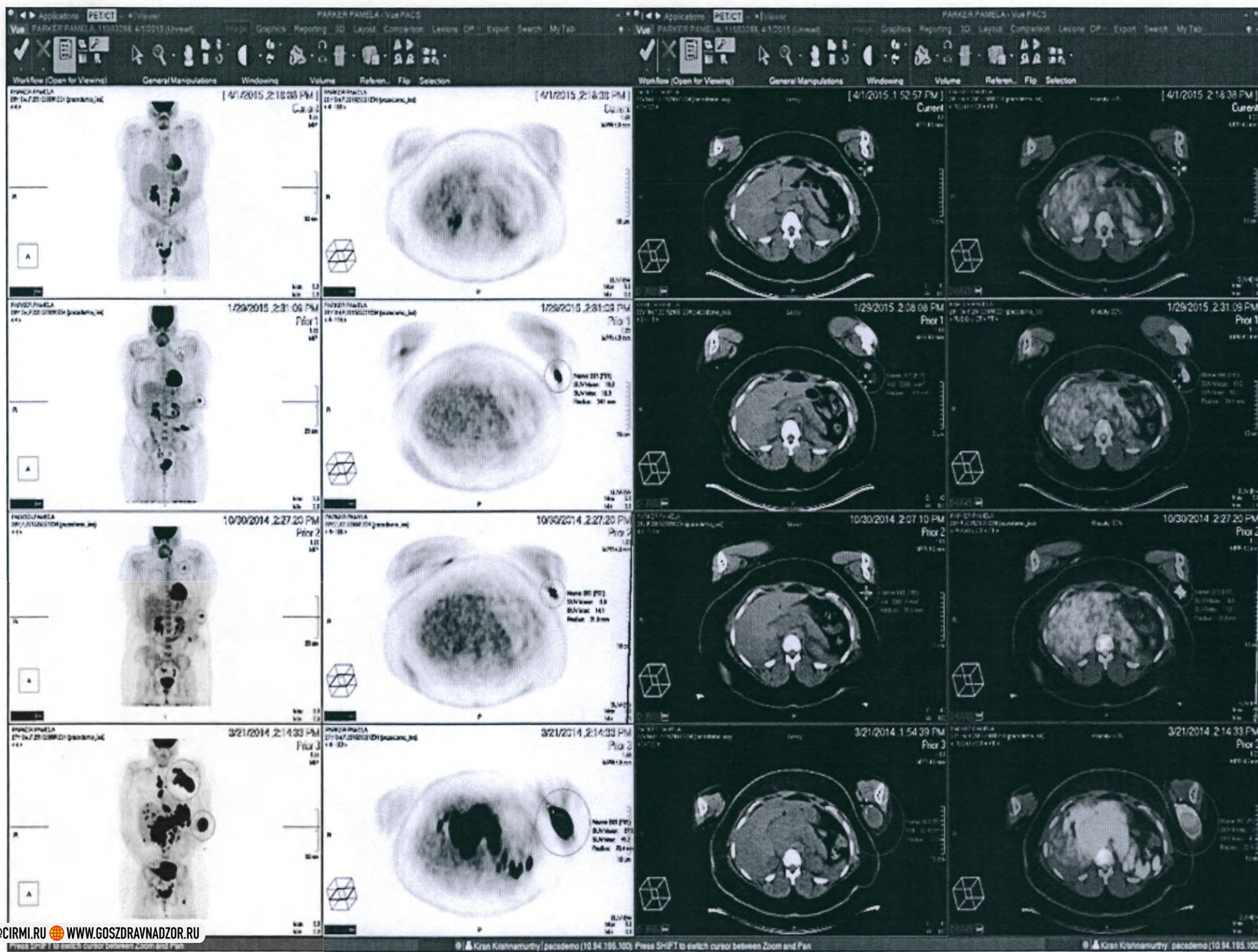

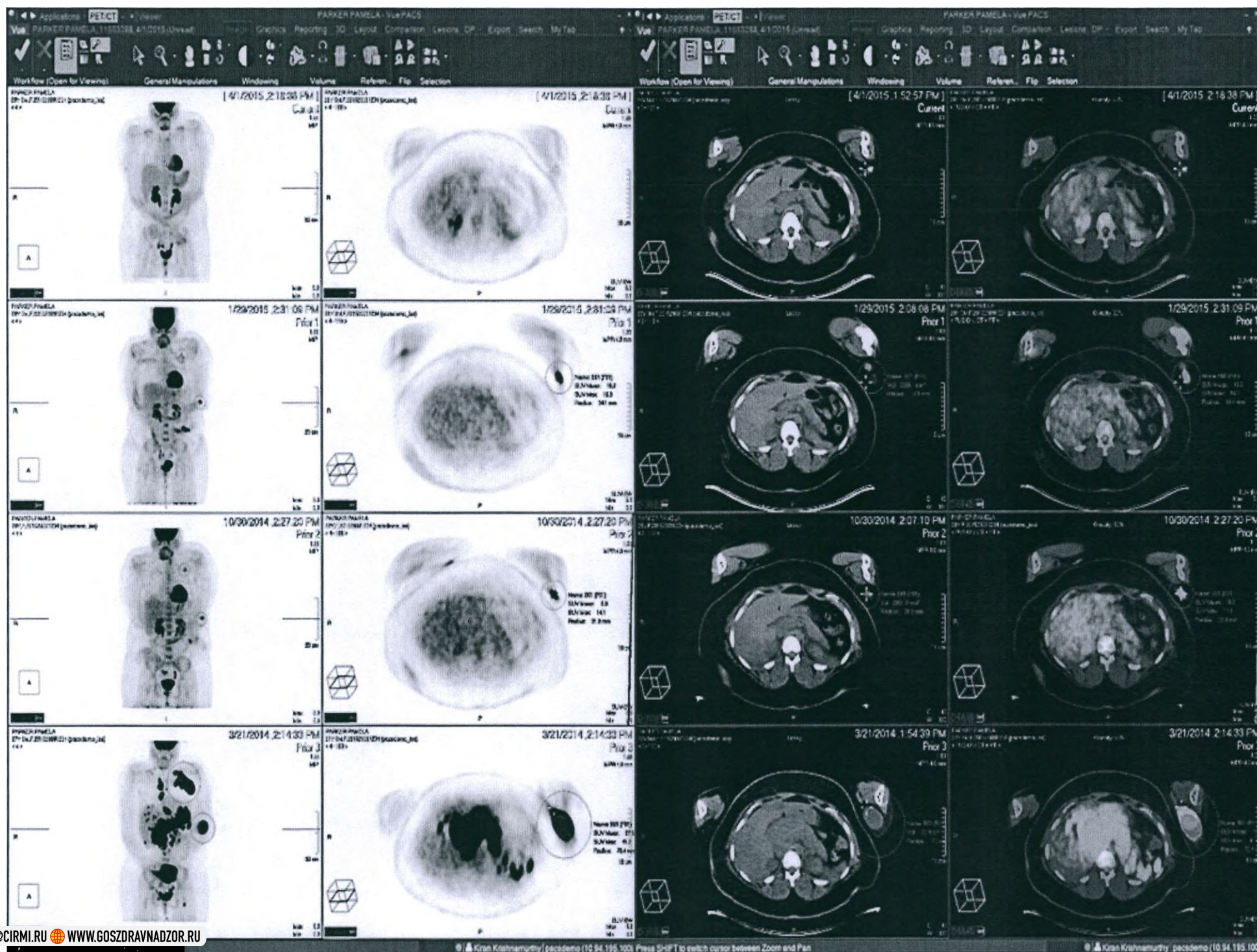

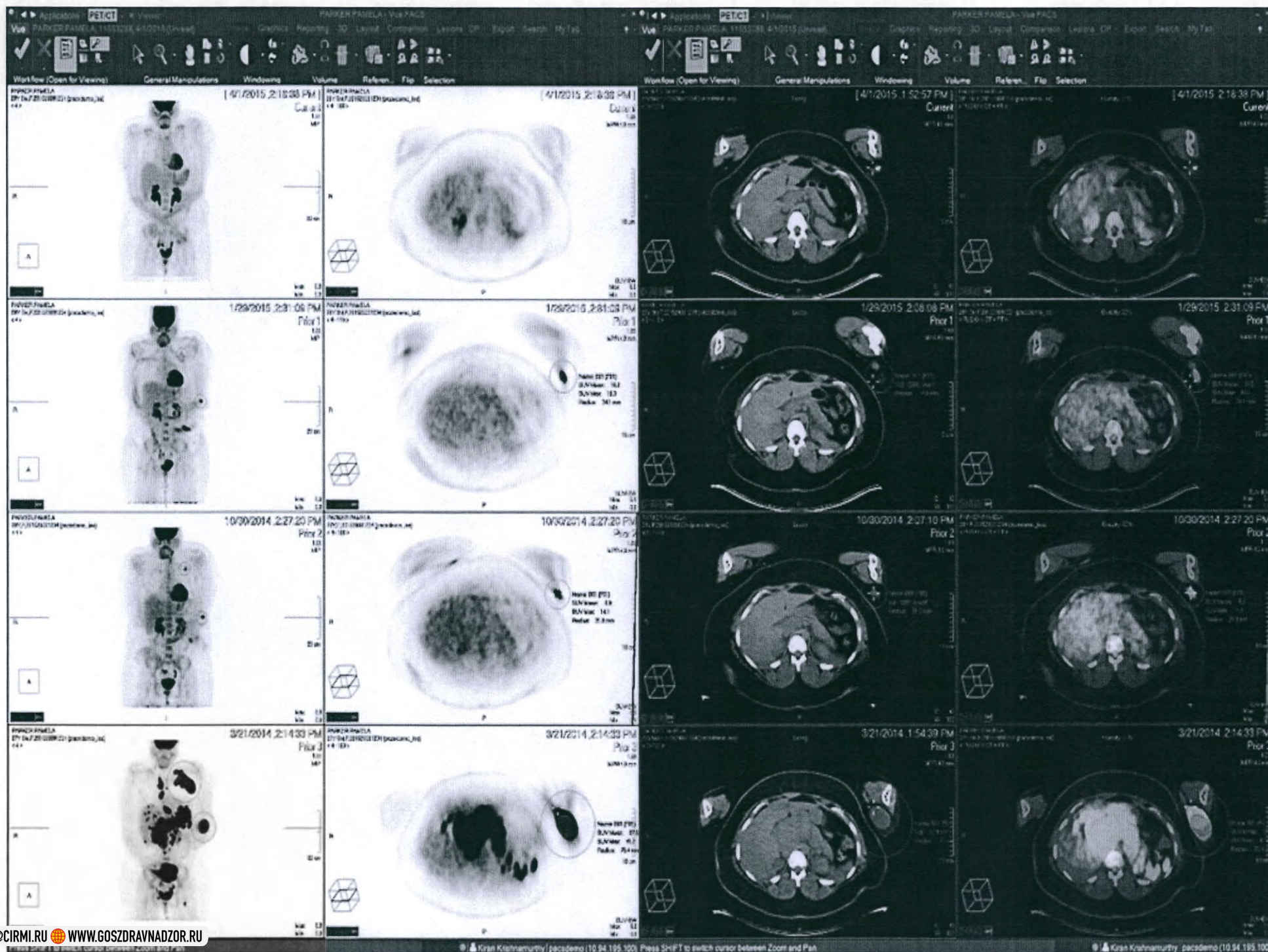

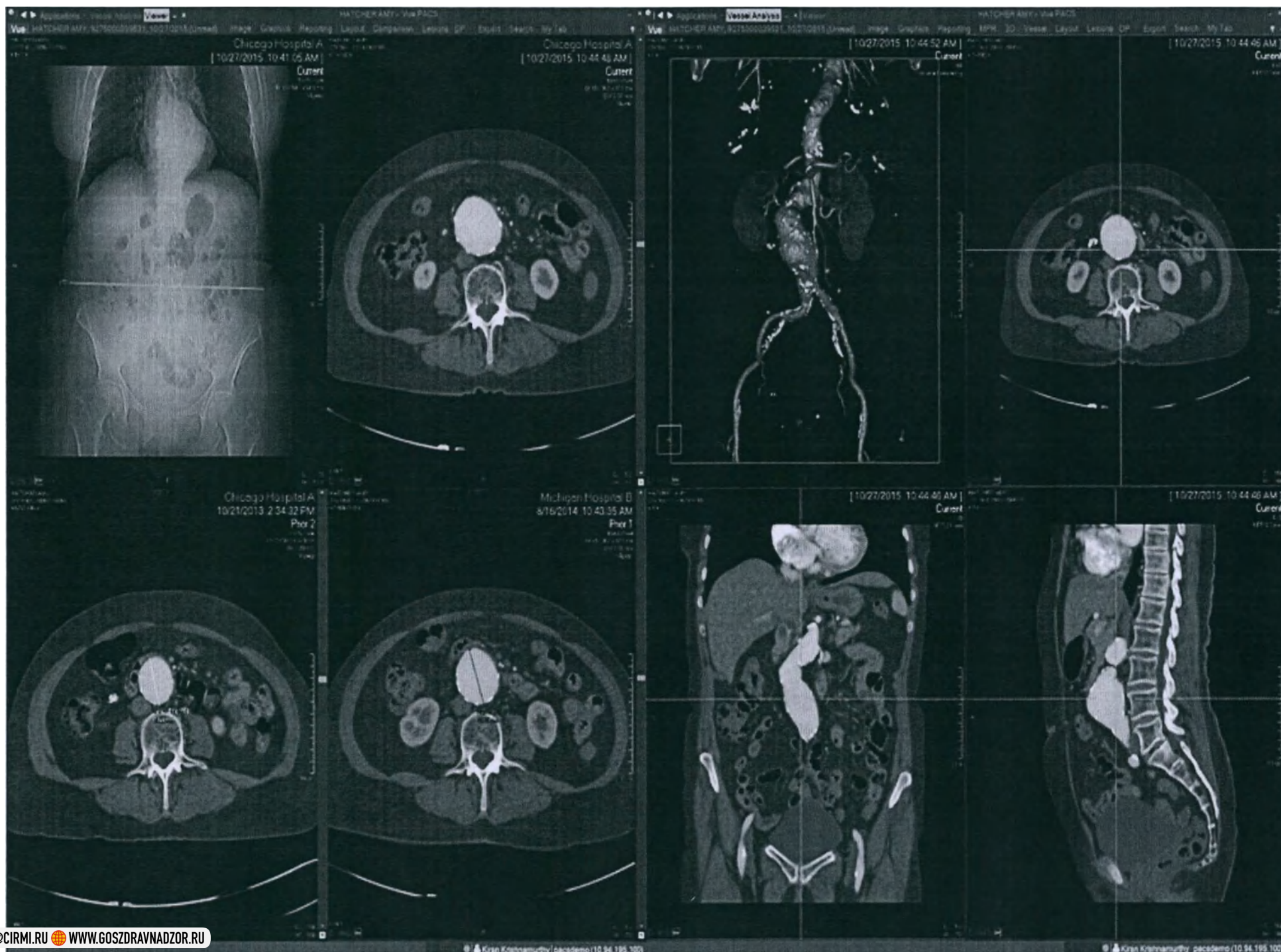

57. Ключ лицензионный для активации программного модуля PRODUCTIVITY ENGINE в вариантах исполнения: не более 25 000 исследований, не более 50 000 исследований, не более 100 000 исследований, не более 200 000 исследований, не более 300 000 исследований, не более 400 000 исследований, не более 500 000 исследований, дополнительно не более 50 000 исследований, дополнительно не более 100 000 исследований, дополнительно не более 500 000 исследований 68

Archive Explorer

10.14.195.272/archive/default.aspx

Kac, Christine
Chicago Medical Center - Corporate

My Lists Home View Acquire Organize Patient Share

Reported Patients

Share Exams Enter Access Code Storage: Pending Sharing

Reported Patients

Patient Name	Patient ID	Date	# Images	Priority	Accession	Mod.	Body Part	Date of Birth	Ref. Physician	Gender	Age	Status	Site
Diers, Dorothy	201201001940	2/2/2016	312	Medium	9275100123456	CT	CHEST	6/1/1944	EVANS DAVID	F	71Y 0M	Final	
Hatcher, Amy	200979684965	10/27/2016	426	High	9275100039551	OT CT	ABDOMEN	9/15/1955	EVANS DAVID	F	77Y 1M	Final	
Haker, Rose	20154580035267	7/6/2016	515	Medium	20151019187008	PT CT CT	OTHER	8/13/1963	PUMAS DAVID	M	67Y 7M	Final	
March, Alan	200920796456	10/5/2016	272	Medium	9275100039553	CT	CHEST	10/19/1940	EVANS DAVID	M	75Y 11M	Final	
Green, Cary	201271161372	6/13/2016	162	Medium	9275100404546	PT CT	HEAD	8/15/1946	EVANS DAVID	M	66Y 9M	Final	
Martin, Matthew	5/20396429	3/16/2016	62	Medium	896778616	US		1/1/1957		M	79Y 2M	Final	
Scott, Stella	401034450006	10/19/2016	321	Medium	505443322	CT	ABDOMEN	1/1/1994		F	22Y 9M	Final	
Casidy, Olivia	881100486	5/13/2016	32	Medium	666524438	US	PELVIS	1/1/1997	EVANS DAVID	F	18Y 8M	Final	
Harker, Vanele	201520001234	4/1/2016	429	High	111031288	PT CT	WHOLE...	10/13/1947	EVANS DAVID	F	28Y 9M	Final	
Edwards, Elizabeth	201200121950	6/24/2016	62	High	9275001121212	CT	CT DRAIN	2/15/1908		F	88Y 9M	Final	
Fox, Frances	111162576	7/9/2016	470	High	11625613	MR	BDN	10/10/1969		F	46Y 8M	Final	
Garcia, Gloria	2001100318	7/26/2016	471	Medium	7001451070	MR	MR (BRAIN)	8/3/1935		F	81Y 11M	Final	

All Lists Recently Viewed Archives

Powered by Carestream

1 - 17 of 17 items

000 ЦРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

38. Ключ лицензионный для активации программного модуля QUALITY ENGINE в вариантах исполнения: не более 25 000 исследований, не более 50 000 исследований, не более 100 000 исследований, не более 200 000 исследований, не более 300 000 исследований, не более 400 000 исследований, не более 500 000 исследований, дополнительно не более 50 000 исследований, дополнительно не более 100 000 исследований, дополнительно не более 500 000 исследований 69

Home > Quality > Peer Review > General

General

Exam Selection Criteria

ModalitySubspecialtyAdvanced

Subspecialty	Ratio (%)	Days Back	Windows (days)		
Body	10	60	10		

Quality

Entering Quality Exams

Peer Review

General

Scores

000 ЦИРМИ

INFO@CIRMI.RU

WWW.GOSZDRAVNADZOR.RU

March, Allan - Explorer Portal

Archive Explorer

March, Allan

Name: March, Allan

Date of Birth: 10/19/1942 (76Y 11M)

Current exam

Modality: CT

Body Part: CHEST

Description: CT Chest w/ Contrast

Patient ID: 200932798456

Gender: M

Exam Date: 10/5/2019

Accession Number: 9275000039533

Referring Physician: Evans David

New Item

Exams

	Date	M...	Body Part	# Im	Ha...
	10/5/2019	CT	CHEST	278	false
	1/30/2019	CT	THX	275	true
	1/5/2019	E...	CARDIAC	1	false
	4/7/2018	OT	PATHO...	3	false

CHEST CT 9275000039533

10/5/2019 3:21:00 PM (278 Images) ONLINE UNREAD

Thumbnails

Report

Order

Bookmarks

Exam Data

Documents

Critical Findings (1)

Sticky Notes (1)

Exam

Exam

CT Chest

12:15 05/10/19

CT Chest with contrast

1 0

Technologist

Tse'ela Mida

Modality

CT room 1

Notes for radiologist

follow up case

Protocol name

CT Chest/Abd/Pel - Oncology F/U

Order Information

Reason for order

Prostate cancer with metastases - follow-up

Relevant history

prostate cancer with metastases. Received radiation therapy

Question for radiologist

metastases follow up

Notes

Patient is in pains

Ordering organization

Oncology

Referring physician

Aviad Danziger

Patient type

Out Patient



Archive Explorer

Worklist

Home View Manage Acquire Organize Share

Weekday Shift

Stroke/Trauma (3/3)

ED/STAT (8/23)

Inpatient (7/24)

Flags Patient Name Age Gender Ref. Physician Procedure Text Mod

Edwards, Elizabeth 57Y 5M F Evans David CT Head w/o Contrast CT

Ross, Fred 44Y 1M M Blankenship Nathaniel CR Leg. 2 Views CR

Valdez, Francisco 34Y 11M M Richardson Justin CR Leg. 2 Views CR

Brown, Louis 72Y 6M M Keaton Robert CT Head w/o Contrast CT

Green, Mary 83Y 10M F Blankenship Nathaniel CT Face/Neck/Chest w/ Contrast CT

Moore, Emily 13Y 4M F Evans David CT Head w/o Contrast CT

Wright, Chloe 19Y 6M F Blankenship Nathaniel XR Cervical Spine, 2-3 VIEWS DX

Lewis, Sofia 49Y 7M F Blankenship Nathaniel CT Head/Brain w/ IV Contrast CT

Taylor, James 72Y 2M M Morris Lawrence XR Neck Soft Tissue DX

Harris, Benjamin 55Y 3M M Morris Lawrence CT Soft Tissue Neck w/ IV Contrast CT

Lopez, Matthew 84Y 4M M Keaton Robert CT Brain w/ IV Contrast CT

Parker, Pamela 27Y 5M F Evans David PET Whole Body w/ concurrent PT CT

Green, Gary 65Y 9M M Evans David PET-CT Head PT CT

Jenkins, Sarah 64Y 10M F EVANS DAVID CT Head w/ Contrast CT

Turner, Amee 85Y 10M F Blankenship Nathaniel CT ABD/Pelvis w/o CONTR CT

Bates, Robin 55Y 8M F Blankenship Nathaniel PET-CT Cervical w/ CT Thx w/ PT CT

Lee, Noah 86Y 3M M Richardson Justin CT Brain w/ IV Contrast CT

Search

No conversation history

Search contacts to start a conversation

Akertblom Lars-Goran

Altal David

Ansalone Rogelio

Aradi Yinnon

Arletti Dario

Aschner Eitan

Belgium User

Ben-Yossef Shaul

Bennet Kyle

Beyer Andrea

bi bi

Bibi Erez

Canepa Daniel

Cardoso Alejandro

Cohen Meir


Notify admin

RVU

DAY 22

My Lists

000 ЦРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

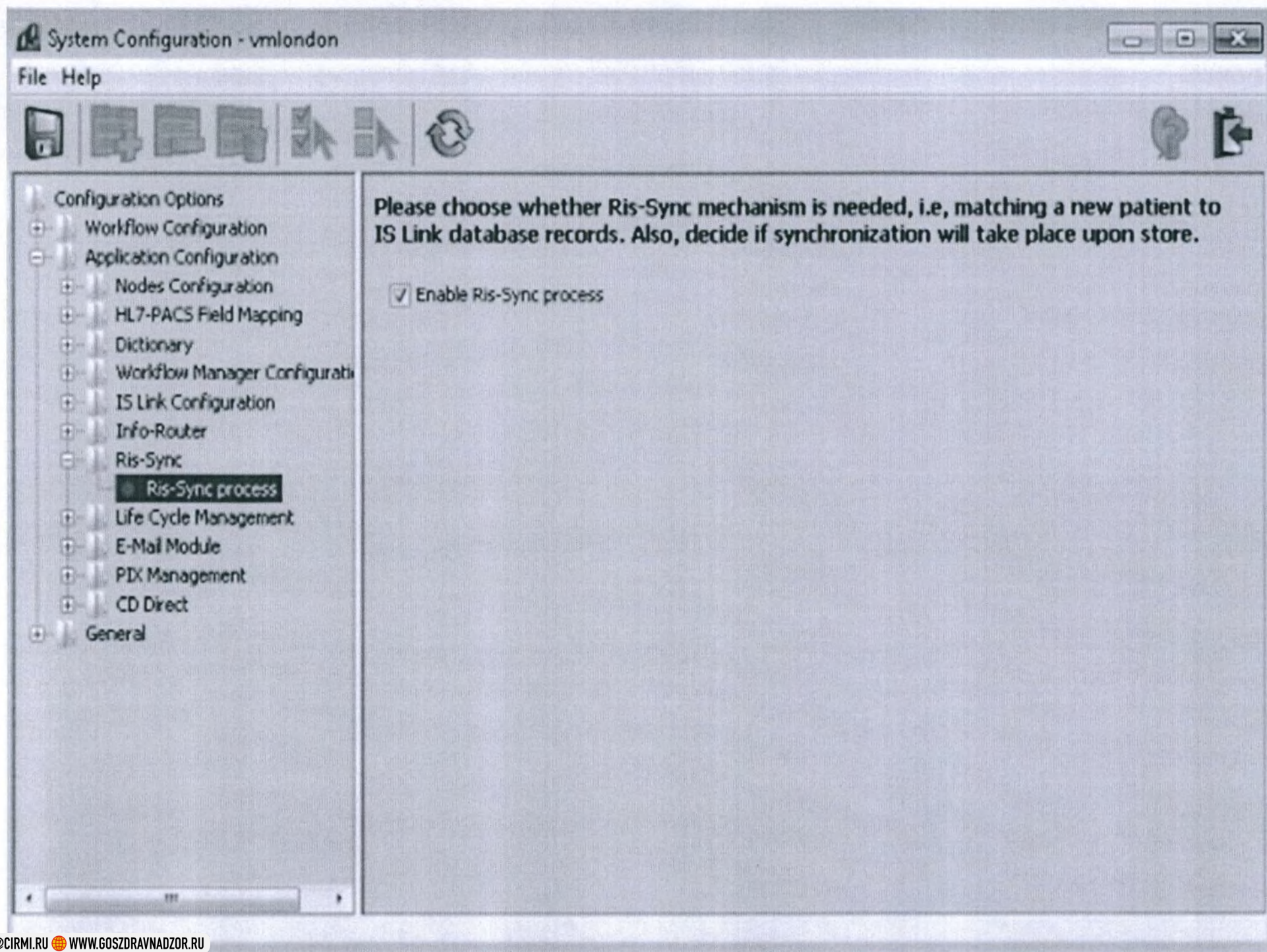

Archive Explorer

Worklist Home View Manage Acquire Organize Share

Weekday Shift

View Motion View Report Print Report Order

Weekday Shift

Flags Patient Name Age Gender Ref. Physician Procedure Text Mod

Stroke/Trauma (3/3)

Edwards, Elizabeth 57Y 6M F Evans David CT Head w/o Contrast CT

Ross, Fred 44Y 1M M Blankenship Nathaniel CR Leg. 2 Views CR

Valdez, Francisco 34Y 11M M Richardson Justin CR Leg. 2 Views CR

ED/STAT (8/23)

Brown, Louis 72Y 6M M Keaton Robert CT Head w/o Contrast CT

Green, Mary 89Y 10M F Blankenship Nathaniel CT Face/Neck/Chest w/ Contrast CT

Moore, Emily 13Y 4M F Evans David CT Head w/o Contrast CT

Wright, Chloe 19Y 6M F Blankenship Nathaniel XR Cervical Spine, 2-3 VIEWS DX

Lewis, Sofia 49Y 7M F Blankenship Nathaniel CT Head/Brain w/o IV Contrast CT

Taylor, James 72Y 2M M Morris Lawrence XR Neck, Soft Tissue DX

Harris, Benjamin 59Y 3M M Morris Lawrence CT Soft Tissue Neck w/ IV Cont. CT

Lopez, Matthew 84Y 4M M Keaton Robert CT Brain w/o IV Contrast CT

Inpatient (7/24)

Parker, Pamela 27Y 5M F Evans David PET Whole Body w/ concurrent PT CT

Green, Gary 65Y 9M M Evans David PET-CT Head PT CT

Jenkins, Sarah 64Y 10M F EVANS DAVID CT Head w/ Contrast CT

Turner, Aimee 65Y 10M F Blankenship Nathaniel CT ABD/Pelvis w/o CONT CT

Bates, Robin 55Y 9M F Blankenship Nathaniel PET-CT Cervical w/ CT Throat PT CT

Lee, Noah 86Y 5M M Richardson Justin CT Brain w/o IV Contrast CT

RVU DAY 22

My Lists

Arfelli, Dario Chicago Medical Center

< Back to All Conversations Online

Search

No conversation history. Search contacts to start a conversation.

Albertoni Lars-Goran

Altal David

Ansalone Rogerio

Aradi Yinnon

Arfelli Dario

Aschner Eitan

Belgium User

Ben-Yossef Shaul

Bennet Kyle

Beyer Andrea

bi bi

Bibi Erez

Canepa Daniel

Cardoso Alejandro

Cohen Meri

Notify admin

System Configuration - vmlondon

File Help

Configuration Options

- Workflow Configuration
- Application Configuration
- Nodes Configuration
- HL7-PACS Field Mapping
 - Default Mapping Set**
- Dictionary
- Workflow Manager Config
- IS Link Configuration
- Info-Router
- Ris-Sync
- Life Cycle Management
- E-Mail Module
- PDX Management
- General

Carestream Field	HL7 Field ID	HL7 Field Name	Dictionary Table	RIS-Synced	Update Condition	Normalized
Accession Number	OBR-3.1	Filer Order Number...				
Admission Date	PV1-44.1	Admit Date/Time - ...				
Admission Type	PV1-4	Admission Type				
Body Part			Procedures Table	Yes	Not Null In IS Link	No
Clinical Info	OBR-13	Relevant Clinical Info				
Consent Code	PR1-3.1	Procedure Code - I...				
Discharge Date	PV1-45.1	Discharge Date/Tim...				
Exam End Date	OBR-27.5.1	Quantity/Timing - E...				
Exam Start Date	OBR-27.4.1	Quantity/Timing - S...				
Hospital Service	PV1-10	Hospital Service				
Modality			Procedures Table			
Modifier	OBR-15.5	Specimen Source - ...				
Old Patient ID	MRG-1.1	Prior Patient ID - In...				
Order Status	ORC-5	Order Status		Yes	Not Null In IS Link	No
Patient Birthdate	PID-7.1	Date/Time of Birth - ...		Yes	Not Null In IS Link	No
Patient Class	PV1-2	Patient Class		Yes	Not Null In IS Link	No
Patient Custom Fiel...	PID-17	Religion		No		
Patient Custom Fiel...	PID-18.1	Patient Account Nu...		No		
Patient Custom Fiel...	PID-20.1	Driver's License Nu...		No		
Patient Custom Fiel...	PID-23	Birth Place		No		
Patient Encrypted ...	ZEB-1	Zeb _eMS_		Yes	Not Null In IS Link	No
Patient ID	PID-3.1	Patient ID (Internal...		No		
Patient Location POC	PV1-3.1	Assigned Patient Lo...		Yes	Not Null In IS Link	No
Patient Last Name	PID-5.1	Patient Name - Fam...		Yes	Not Null In IS Link	Yes
Patient First Name	PID-5.2	Patient Name - Give...		Yes	Not Null In IS Link	Yes
Patient Middle Name	PID-5.3	Patient Name - Mid...		Yes	Not Null In IS Link	Yes
Patient Suffix	PID-5.4	Patient Name - Suffix		Yes	Not Null In IS Link	Yes
Patient Prefix	PID-5.5	Patient Name - Prefix		Yes	Not Null In IS Link	Yes
Patient Degree	PID-5.6	Patient Name - Deg...				
Patient Sex	PID-8	Sex		Yes	Not Null In IS Link	No
Patient Segment	PID-0	Pid Segment _eMS_		No		
Pre-Fetch Code	OBR-21	Filer Field 2				
Priority	OBR-27.6	Quantity/Timing - P...	Priorities Table	Yes	Not Null In IS Link	No
Procedure Code	OBR-4.1	Universal Service I...		Yes	Not Null In IS Link	No
Reading Physician ID	OBR-32.1.1	Principal Result Inte...		Yes	Not Null In IS Link	No

Update Field Mapping

March, Allan - Explorer Portal

March, Allan

Arif, Dano
Chicago Medical Center

Name: March, Allan Date of Birth: 10/19/1942 (76Y 11M) Modality: CT Body Part: CHEST Description: CT Chest w/ Contrast
Patient ID: 200932798456 Gender: M Exam Date: 10/5/2019 Accession Number: 9275000009533 Referring Physician: Evans David New Item

Exams

CHEST CT 9275000009533
10/5/2019 3:21:00 PM (278 images) ONLINE UNREAD

Thumbnail
Report
Order
Bookmarks
Exam Data
Documents
Critical Findings (1)

Sticky Notes (1)

Exam

CT Chest
08/10/19

CT Chest with contrast

Technologist
Tse'ela Mida

Modality
CT room 1

Notes for radiologist
follow up case

Protocol name
CT Chest/Abd/Pel - Oncology
F/U

Order Information

Reason for order
Relevant history
Question for radiologist
Notes

Date	M	Body Part	# Images	Has
10/5/2019	CT	CHEST	278	false
1/30/2019	CT	THX	275	true
1/5/2019	E	CARDIAC	1	false
4/7/2018	OT	PATHO	3	false

000 ЦРПМ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

E Y C L

Manual Procedure Selection

Select a exam from the following list

RP Criteria	Group	Exams	Procedure
XR CT MR US Angio Fluoro Mamm NM BMD	CT Abdomen CT Chest CT Head and Neck CT Lower Limb CT Others CT Pelvis and Hip CT Spine CT Upper Limb Others	Subile Studies CT Abdomen CT Abdomen and Pelvis CT Abdominal Wall CT Adrenal Glands CT Kidneys CT Liver CT Pancreas CT Urography	Recommended Procedure CT Abdomen and Pelvis

Selected Exams (2) Remove all

CT Abdomen	CT Abdomen	CT Abdomen	CT Abdomen and Pelvis
CT Abdomen with contrast		CT Abdomen and Pelvis	

Cancel

OK

March, Allan - Explorer Portal

Archive Explorer March, Allan

Name: March, Allan Date of Birth: 10/19/1942 (76Y 11M) Modality: CT Body Part: CHEST Description: CT Chest w/ Contrast
 Patient ID: 200932798456 Gender: M Exam Date: 10/5/2019 Accession Number: 9275000039533 Referring Physician: Evans David

Current exam

Exams

Date	M	Body Part	# Im	Ha
10/5/2019	CT	CHEST	278	false
1/30/2019	CT	THX	275	true
1/5/2019	E...	CARDIAC	1	false
4/7/2018	OT	PATHO...	3	false

CHEST CT 9275000039533
 10/5/2019 3:21:00 PM (278 images) ONLINE UNREAD

Thumbnails Report Order Bookmarks Exam Data Documents Critical Findings (1)

Sticky Notes (1)

Exam

CT Chest 12:15 05/10/19
 CT Chest with contrast

Technologist	Modality	Notes for radiologist	Protocol name
Tse'ela Mida	CT room 1	follow up case	CT Chest/Abd/Pel - Oncology F/U

Order Information

Reason for order	Relevant history	Question for radiologist	Notes
Prostate cancer with metastases - follow-up	prostate cancer with metastases. Received radiation therapy	metastases follow up	Patient is in pains

Ordering organization	Referring physician	Patient type
Oncology	Aviad Danziger	Out Patient

Search patient

Documentation

3551500211

Albert, Austin

Walk in

Yaniv, Goldenberg

Administrator

Main

John, Doe

3551500211

4/5/1962 (56y 9m)

Male

[Exam PatientSum... CT Abdomen

Patient type Out Patient

Status Exam Done

Accession # 9270001414 9270001414

Attachments (2)

Scanned order

Consent form

Performed Exam

CT Abdomen	CT Abdomen	16:30
		10/27/19
CT Abdomen with contrast		10

Exam Information

Patient identified by

Start time

Modality

Notes for radiologist

Armband

Verbal

ID card

10/28/2019

14:53

CT room 1

follow up case

Notes

Priority

Reason for order

Patient is in pain, can't walk

Stat

Routine

Urgent

Followup of a melanoma metastases after chemotherapy

[New Box.Form]

[LoggedInUser.Form]

YG

Cancel

New exam

Discontinue

In progress

Save

Complete

Search patient

New order
8877665544
Shai, Aharoni

New order

shahar valdman
Scheduler
Main

John, Doe
8877665544
(Passport)
1/1/2000 (21y)
Male

[Exam Patient Summary]
CT Abdomen | CT Abdomen and Pelvis
Patient type
Out Patient
Status
Accession #

(0)

(20)

Attachments (0)

Quick scan

1 Questionnaire

2 Others

3 CD required form

Patient Order Clinical Information

[Patient Form]
9000030438 (New Visit)

Ordered Exams (2) Search

CT Abdomen
CT Abdomen with contrast

CT Abdomen
CT Abdomen and Pelvis

Order Information

Ordering organization
Emergency

Patient type
Out Patient

Requested by *
[Select...]

Referring specialty
[Select...]

Reason for order

Notes

Justification status
Not Required Required

Priority
Routine Urgent Stat

Additional Information

Question for radiologist

Relevant history

Created on date
01/12/2021, 16:21

Created by
shahar valdman

Cancel Complete

ООО ЦРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

Ordered Exams *	ct	Q		
7 results found for ct 				
^ Order Information	v CT Abdomen			
	v CT Abdomen and Pelvis			
	v CT Abdominal Wall			
	v CT Brain			
	v CT Chest			
	v CT Chest Wall			
	v CT Chest and Abdomen			
Ordering organization				
Neurology				
Reason for order				

Search patient

Documentation

3551500211

Albert, Austin

Walk in

Yaniv, Goldenberg

Administrator

Main

John, Doe

3551500211

4/5/1952 (58y 9m)

Male

Exam PatientSum... CT Abdomen

Patient type Out Patient

Status Exam Done

Accession # 9270001414 9270001414

Attachments (2)

Scanned order

Consent form

Performed Exam

CT Abdomen	CT Abdomen	16:30 10/27/19
CT Abdomen with contrast		10

Exam Information

Patient identified by	Start time	Modality	Notes for radiologist
Armband Verbal ID card	10/28/2019 14:53	CT room 1	follow up case

Notes	Priority	Reason for order
Patient is in pain, can't walk	Stat Routine Urgent	Followup of a melanoma metastases after chemotherapy

[New Box.Form]

[LoggedInUser.Form]

YG

Cancel

New exam Discontinue In progress Save Complete

Search patient

New appointment

15:02

Mona Steve

New appointment

shahar valdman

Receptionist

White River Medical Center

Doe, John

1234555

8/2/1947 (73y 5m)

Male

Exam: CT Abdomen

Patient type: Out Patient

Status: N/A

Accession #: N/A

(2)

0

(192)

055111114

oferpc@gmail.com

Rekanati

Tel aviv, Gosh Dan

Israel

94949

Financial classification

N/A

Referring physician

N/A

Payer

N/A

Ordering organization

Emergency

Patient

Order

Scheduling

Clinical Information

Exams (1)*

CT Abdomen

CT Abdomen

20 min

CT Abdomen with contrast

Filter

Location

Preferred Time

Days

Mon

Tue

Wed

Thu

Fri

Sat

Sun

Time of day

☀ 06:00 - 1...

🌙 12:00 - 1...

🌙 16:00 - 0...

Wednesday, January 13, 2021

Suggested

Agenda

Summary

Outpatient Clinic

08:30

CT room 1

08:45

CT room 1

09:00

CT room 1

09:15

CT room 1

09:30

CT room 1

[More results](#)

Pediatric Clinic

13 January

Wednesday

14 January

Thursday

15 January

Friday

16 January

Saturday

17 January

Sunday

Cancel

Schedule

ООО ЦРММ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

File Edit View Favorites Tools Help

Web Slice Gallery Philips Access

Search patient 8

New appointment 7

Tse'elo Mids Scheduler Main 1

Ordered (last 30 days)

Schedule order Edit order Cancel order View exam Notes 6

5 Edwards, Elizabeth
201215021955
2/15/1960 (59y 8m)
Female


Exam MRI Cardiac
Patient type In Patient
Status Ordered
Accession # N/A


Details Attachments (0) Exams (10)

Ordered (1/5) Ordered Cancelled Justified Outpatients 4

Search

Flags	Last name	First name	Patient ID	Exam description	Status	Justification status	Patient type	Order create
					Ordered			
	Yunker	Dave	CB900007474	CT Shoulders	Ordered	Not Required	In Patient	10/27/2019
	Edwards	Elizabeth	201215021955	MRI Cardiac	Ordered	Not Required	In Patient	10/27/2019
	March	Allan	200932798456	MRI Abdomen	Ordered	Not Required	Out Patient	10/27/2019
	Davis	Dorothy	201201061940	MRI Cervical Spine	Ordered	Required	Out Patient	10/27/2019
	Hatcher	Amy	200967984365	Chest X-ray	Ordered	Not Required	Emergency	10/27/2019

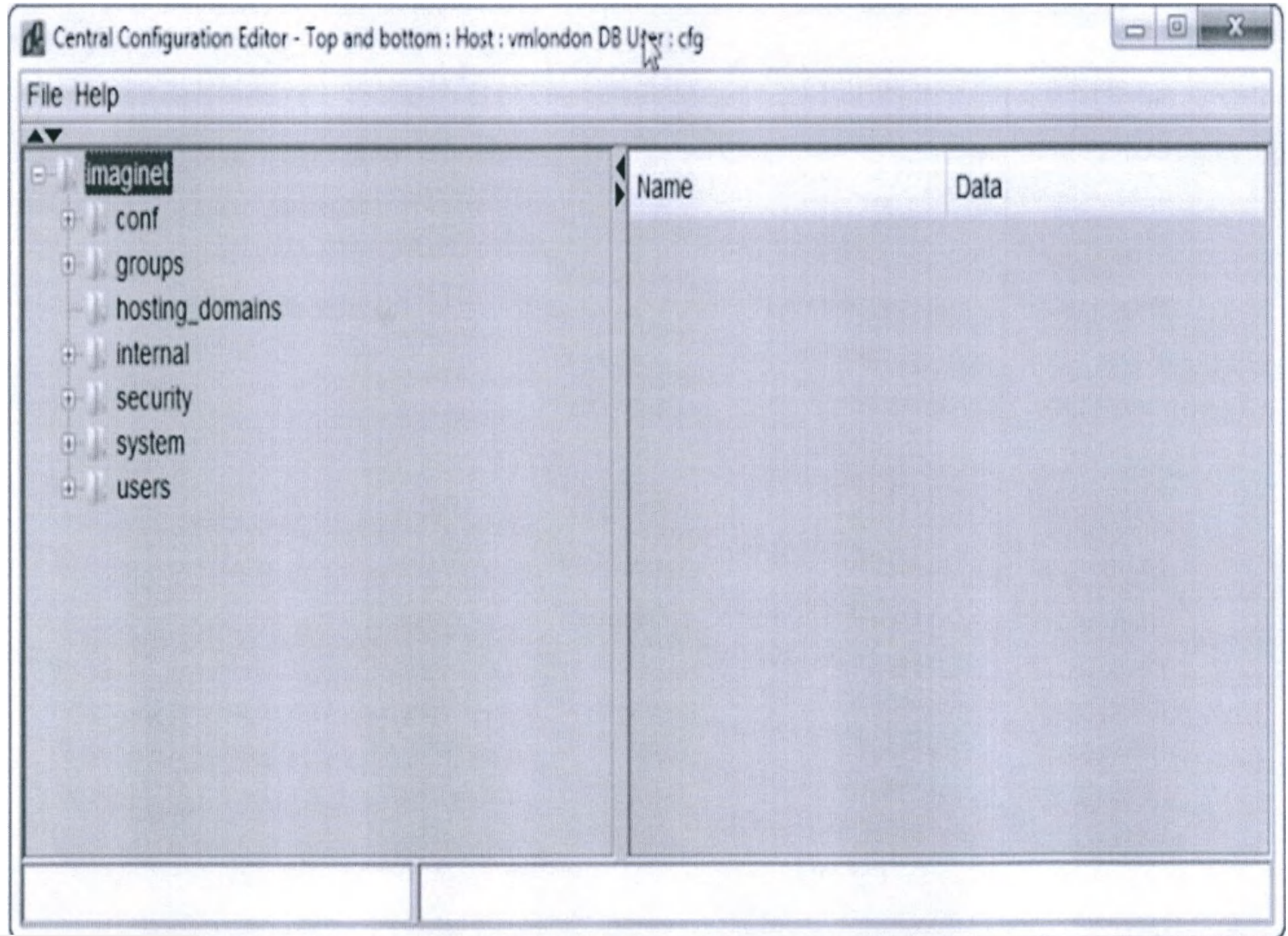

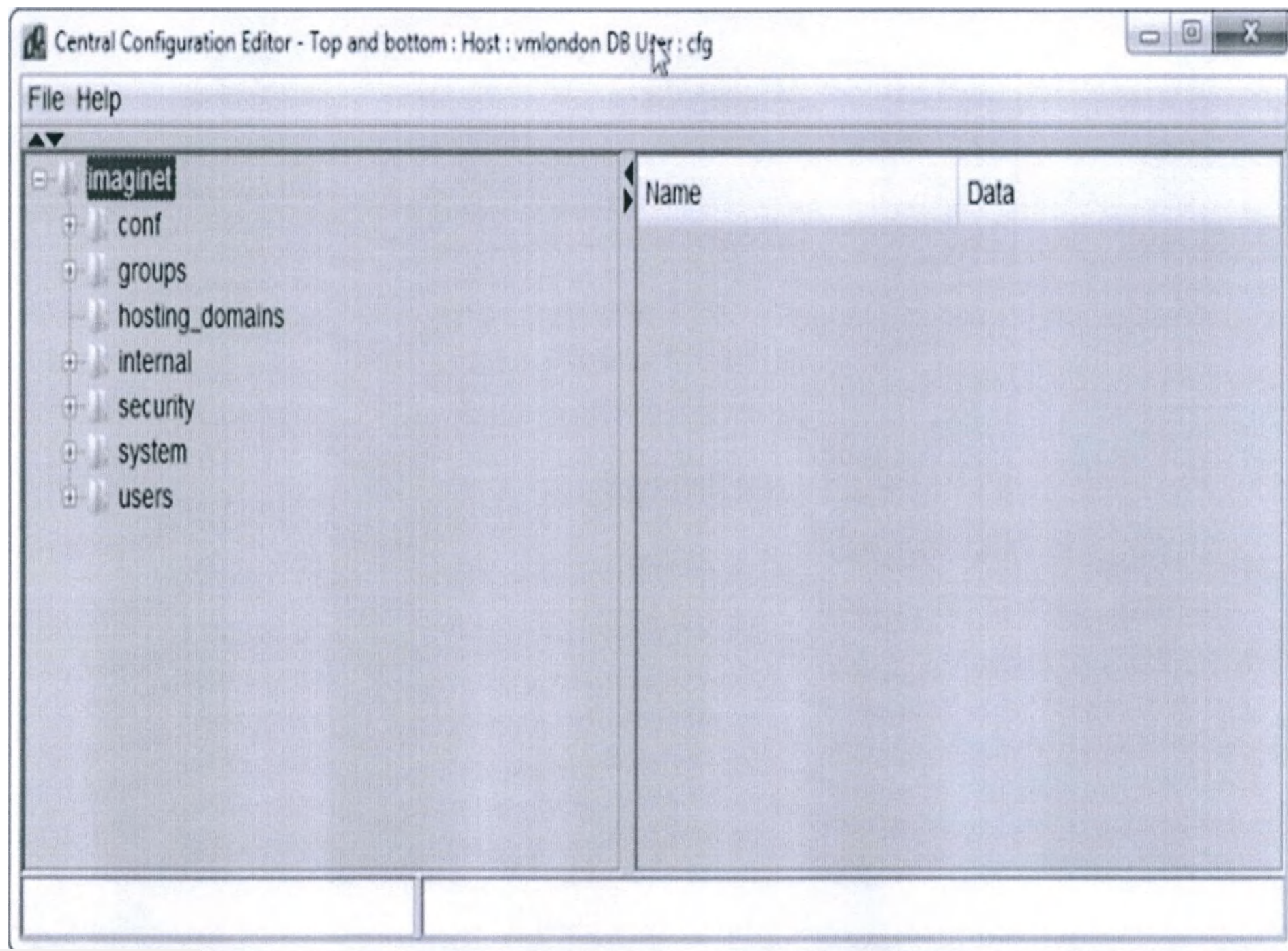

Archive Explorer

Search Reports

Worklist

Admin Worklist

My Lists

All Lists

Quality

Recently Viewed

Research

Search Reports

Search Options

☒ Filter out negative results

Saved Queries

Recent Queries

Patient Age	Patient Sex	Gender	Body Part	Description	Modality	Procedure
79.2	7/22/1936	M	BREASTRT	US BREAS	US	IMG2569
BILATERAL SEVEN VIEW DIGITAL DIAGNOSTIC MAMMOGRAM WITH CAD AND RIGHT BREAST ULTRASOUND 10/20/2014 10/20/2014 CLINICAL INDICATION: 79-year old male with reported palpable lump right upper outer quadrant and left lateral perisubareolar pain presents for further evaluation. TECHNIQUE: Bilateral CC, MLO, right ML, right XCCL and magnification right CC views were obtained and analyzed with CA						
47.1	9/9/1967	F	BREASTB	MAMMOG	MG	IMG1283
Procedure: MAMMOGRAM DIAGNOSTIC DIGITAL BILATERAL WICAD Visit Date: 10/20/2014 Accession No.: 14035981 Location: MAMMOGRAPHY 310 Phone: (310)667-5650 ADDENDUM SECOND READING ON A BILATERAL 6 DIAGNOSTIC MAMMOGRAM AND BILATERAL BREAST ULTRASOUND 10/20/2014 Per patient request, study being reviewed. Examination is compared to prior studies 07/23/2013, 07/15/2013, 07/06/2013, 04/21/2012, 04/05/2011.						
49.9	11/29/1964	F	HUMERUS	XRAY HUM	CR	IMG2463
Procedure: XRAY HUMERUS RIGHT 2 VIEWS Visit Date: 10/16/2014 Accession No.: 140398168 Location: DIAGNOSTIC RAD MGB Phone: (310)423-9975 X-RAY OF THE RIGHT HUMERUS 10/16/2014 CLINICAL INDICATION: Fracture right humerus. COMPARISON: Multiple x-rays of the right humerus are available for comparison dating back date 06/22/2014. FINDINGS: There is a spiral fracture of the proximal right humerus invol						
38.1	8/29/1976	F	FOOTLT	XRAY FOO	CR	IMG5450
Procedure: XRAY FOOT LEFT 3 VIEWS MORE Visit Date: 10/16/2014 Accession No.: 140399200 Location: DIAGNOSTIC RAD MGB Phone: (310)423-4566 LEFT FOOT 10/16/2014 AT 15:28 CLINICAL INDICATION: Left 5th toe pain. REFERENCE: None FINDINGS: Three views of the left foot were obtained. The patient is status post remote hammer toe surgery about the 5th toe. The remainder of the osseous structures and joint						
31.5	3/24/1993	M	BRAIN	MRI BRAIN	MR	IMG1810
Procedure: MRI BRAIN W/O CONTRAST Visit Date: 10/16/2014 Accession No.: 140375678 Location: MRI TAPER Phone: (310)385-6016 MRI OF BRAIN WITH AND WITHOUT CONTRAST 10/16/2014 AT 09:45 CLINICAL INDICATION: Exertional headaches. TECHNICAL FACTORS: Sagittal and axial sequences were obtained with and without contrast. FINDINGS: The ventricles and cortical sulci appear normal in size and configuration.						
68.5	5/6/1926	F	HIPLT	XRAY HIP	CR	IMG5866
Procedure: XRAY HIP LEFT 2-3 VIEWS Visit Date: 10/20/2014 Accession No.: 140403954 Location: ASAP EMERGENCY DEPT Phone: (310)423-7248 FX						
59.4	6/10/1955	M	LOWEREX	VASC US L	US	IMG2685
21-Oct-2014 13:52 Venous Lower Extremity CSN Number: 29126858858 Exam location: CSMC Performed Portable PROCEDURE: Venous lower extremity bilateral (93370) DESCRIPTION: Color flow duplex sonography of the lower extremity venous system was performed utilizing both						
59	11/5/1956	M	NOTSPECI	IR EMBOLI	XA US	IMG2792

Procedure: XRAY HUMERUS RIGHT 2 VIEWS Visit Date: 10/16/2014 Accession No.: 140398168 Location: DIAGNOSTIC RAD MGB Phone: (310)423-9975 X-RAY OF THE RIGHT HUMERUS 10/16/2014 CLINICAL INDICATION: Fracture right humerus. COMPARISON: Multiple x-rays of the right humerus are available for comparison dating back date 06/22/2014. FINDINGS: There is a spiral fracture of the proximal right humerus involving the greater tuberosity, surgical neck proximal shaft. There is stable alignment and callus formation. The visualized joint spaces are normal. IMPRESSION: Significant healing and callus formation of a right proximal humerus fracture in near-anatomic alignment.

Procedure: XRAY FOOT LEFT 3 VIEWS MORE Visit Date: 10/16/2014 Accession No.: 140399200 Location: DIAGNOSTIC RAD MGB Phone: (310)423-4566 LEFT FOOT 10/16/2014 AT 15:28 CLINICAL INDICATION: Left 5th toe pain. REFERENCE: None FINDINGS: Three views of the left foot were obtained. The patient is status post remote hammer toe surgery about the 5th toe. The remainder of the osseous structures and joint

Procedure: MRI BRAIN W/O CONTRAST Visit Date: 10/16/2014 Accession No.: 140375678 Location: MRI TAPER Phone: (310)385-6016 MRI OF BRAIN WITH AND WITHOUT CONTRAST 10/16/2014 AT 09:45 CLINICAL INDICATION: Exertional headaches. TECHNICAL FACTORS: Sagittal and axial sequences were obtained with and without contrast. FINDINGS: The ventricles and cortical sulci appear normal in size and configuration.

Procedure: XRAY HIP LEFT 2-3 VIEWS Visit Date: 10/20/2014 Accession No.: 140403954 Location: ASAP EMERGENCY DEPT Phone: (310)423-7248 FX

21-Oct-2014 13:52 Venous Lower Extremity CSN Number: 29126858858 Exam location: CSMC Performed Portable PROCEDURE: Venous lower extremity bilateral (93370) DESCRIPTION: Color flow duplex sonography of the lower extremity venous system was performed utilizing both

11 of 12604 items

View Images

Archive Explorer


Worklist Admin Worklist My Lists All Lists Quality Recently Viewed Research Search Reports Search Options Filter out negative results Saved Queries Recent Queries Archives

Search Reports

Patient Age	Patient ID	Gender	Body Part	Description	Modality	Procedure
79.2	7/22/1935	M	BREASTRT	US BREAS	US	IMG2559
BILATERAL SEVEN VIEW DIGITAL DIAGNOSTIC MAMMOGRAM WITH CAD AND RIGHT BREAST ULTRASOUND 10/20/2014 10/20/2014 CLINICAL INDICATION: 73-year old male with reported palpable lump right upper outer quadrant and left lateral perisubareolar pain presents for further evaluation. TECHNIQUE: Bilateral CC, MLO, right ML, right XCCL and magnification right CC views were obtained and analyzed with CA						
47.1	5/5/1967	F	BREASTBI	MAMMOG	MG	IMG1283
Procedure: MAMMOGRAM DIAGNOSTIC DIGITAL BILATERAL WICAD Visit Date: 10/20/2014 Accession No.: 140390961 Location: MAMMOGRAPHY 310 Phone: (310)967-5650 ADDENDUM SECOND READING ON A BILATERAL 6 DIAGNOSTIC MAMMOGRAM AND BILATERAL BREAST ULTRASOUND 10/20/2014 Per patient request, study being reviewed. Examination is compared to prior studies 07/23/2013, 07/15/2013, 07/06/2013, 04/21/2012, 04/05/2011.						
49.9	11/25/1964	F	HUMERUS	XRAY HUM	CR	IMG5469
Procedure: XRAY HUMERUS RIGHT 2 VIEWS Visit Date: 10/16/2014 Accession No.: 140396958 Location: DIAGNOSTIC RAD MGB Phone: (310)423-9975 X-RAY OF THE RIGHT HUMERUS 10/16/2014 CLINICAL INDICATION: Fracture right humerus. COMPARISON: Multiple x-rays of the right humerus are available for comparison dating back date 08/22/2014. FINDINGS: There is a spiral fracture of the proximal right humerus invol						
38.1	5/29/1975	F	FOOTLT	XRAY FOO	CR	IMG5450
Procedure: XRAY FOOT LEFT 3 VIEWSMORE Visit Date: 10/16/2014 Accession No.: 140399200 Location: DIAGNOSTIC RAD MGB Phone: (310)423-4566 LEFT FOOT 10/16/2014 AT 15:28 CLINICAL INDICATION: Left 5th toe pain. REFERENCE: None FINDINGS: Three views of the left foot were obtained. The patient is status post remote hammer toe surgery about the 5th toe. The remainder of the osseous structures and joint						
31.5	3/24/1963	M	BRAIN	MRI BRAIN	MR	IMG1810
Procedure: MRI BRAIN WOW CONTRAST Visit Date: 10/16/2014 Accession No.: 140375675 Location: MRI TAPER Phone: (310)685-6016 MRI OF BRAIN WITH AND WITHOUT CONTRAST 10/16/2014 AT 09:45 CLINICAL INDICATION: Exertional headaches. TECHNICAL FACTORS: Sagittal and axial sequences were obtained with and without contrast. FINDINGS: The ventricles and cortical sulci appear normal in size and configuration.						
38.5	5/6/1926	F	HIPLT	XRAY HIP	CR	IMG5466
Procedure: XRAY HIP LEFT 2-3 VIEWS Visit Date: 10/20/2014 Accession No.: 140403094 Location: ASAP EMERGENCY DEPT Phone: (310)423-7248 FX						
59.4	6/13/1955	M	LOWEREX	VASC US L	US	IMG6685
21-Oct-2014 13:52 Venous Lower Extremity CSN Number: 29126858856 Exam location: CSMC Performed Portable PROCEDURE: Venous lower extremity bilateral (53170). DESCRIPTION: Color flow duplex sonography of the lower extremity venous system was performed utilizing both						
59	11/5/1955	M	NOTSPECI	IR EMBOLI	XA US	IMG2792


Procedure: XRAY HUMERUS RIGHT 2 VIEWS Visit Date: 10/16/2014 Accession No.: 140396958 Location: DIAGNOSTIC RAD MGB Phone: (310)423-9975 X-RAY OF THE RIGHT HUMERUS 10/16/2014 CLINICAL INDICATION: Fracture right humerus. COMPARISON: Multiple x-rays of the right humerus are available for comparison dating back date 08/22/2014. FINDINGS: There is a spiral fracture of the proximal right humerus involving the greater tuberosity, surgical neck proximal shaft. There is stable alignment and callus formation. The visualized joint spaces are normal. IMPRESSION: Significant healing and callus formation of a right proximal humerus fracture in near anatomic alignment.


1 - 10 of 129924 items

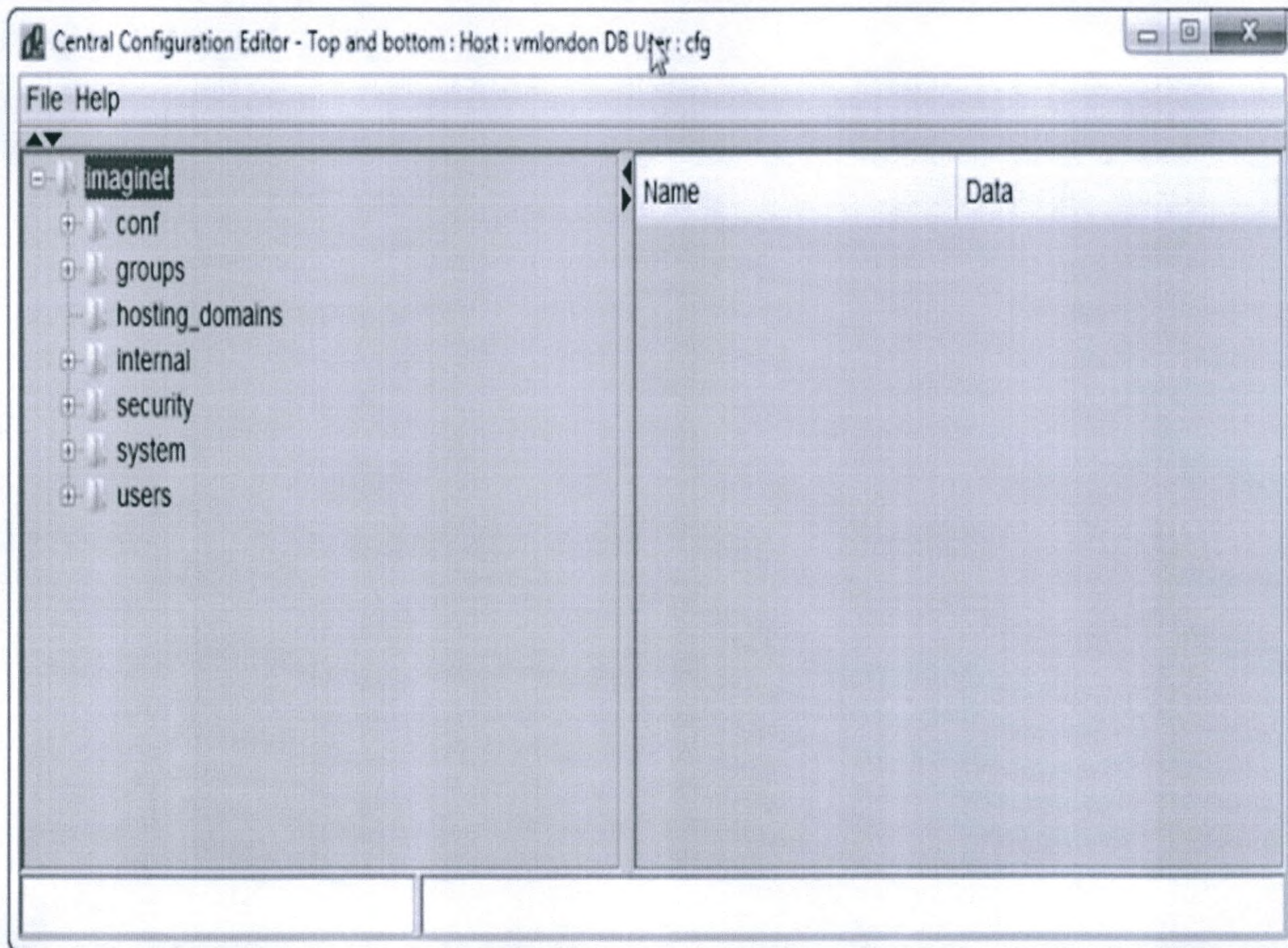




{
{
}

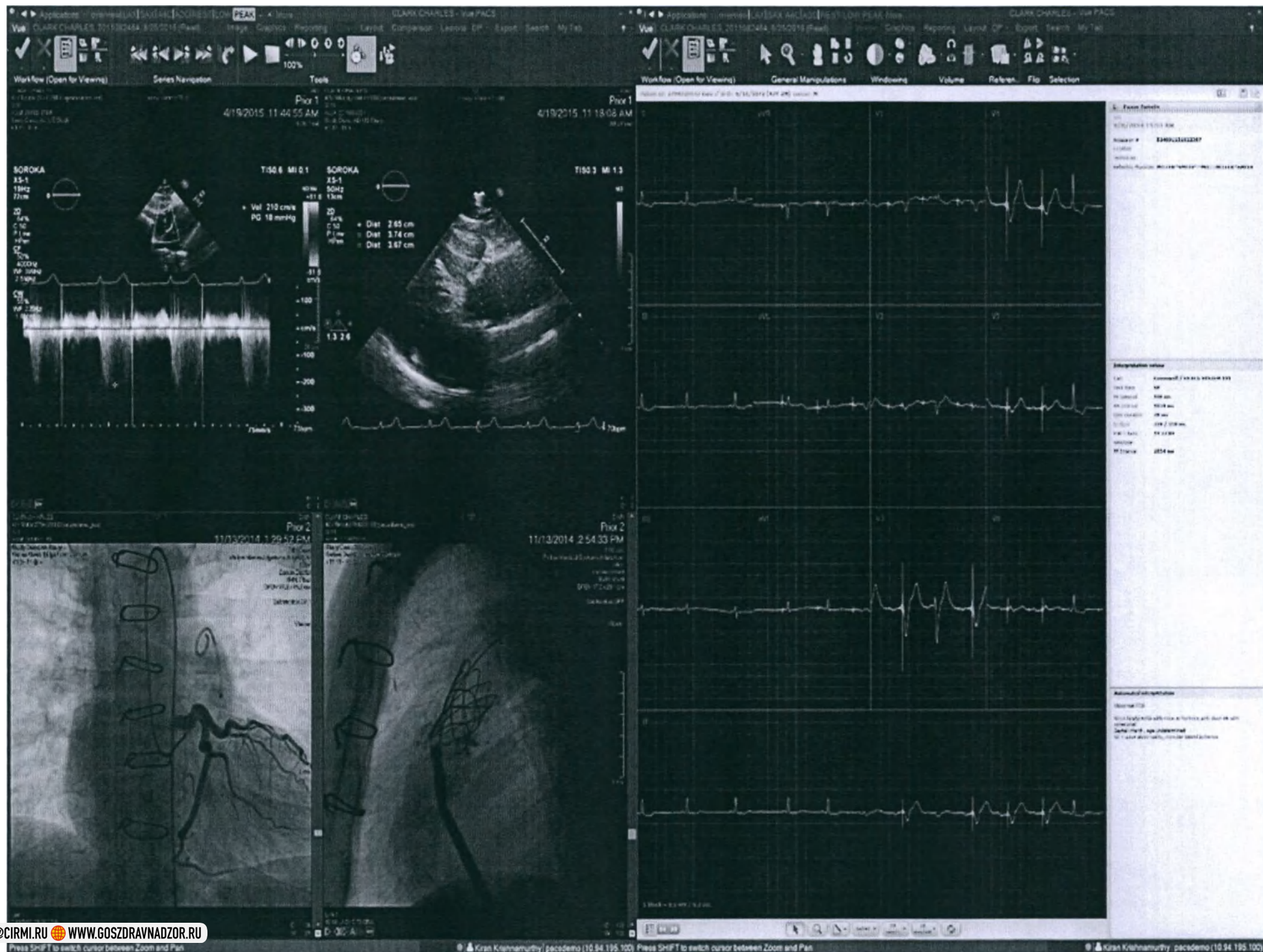

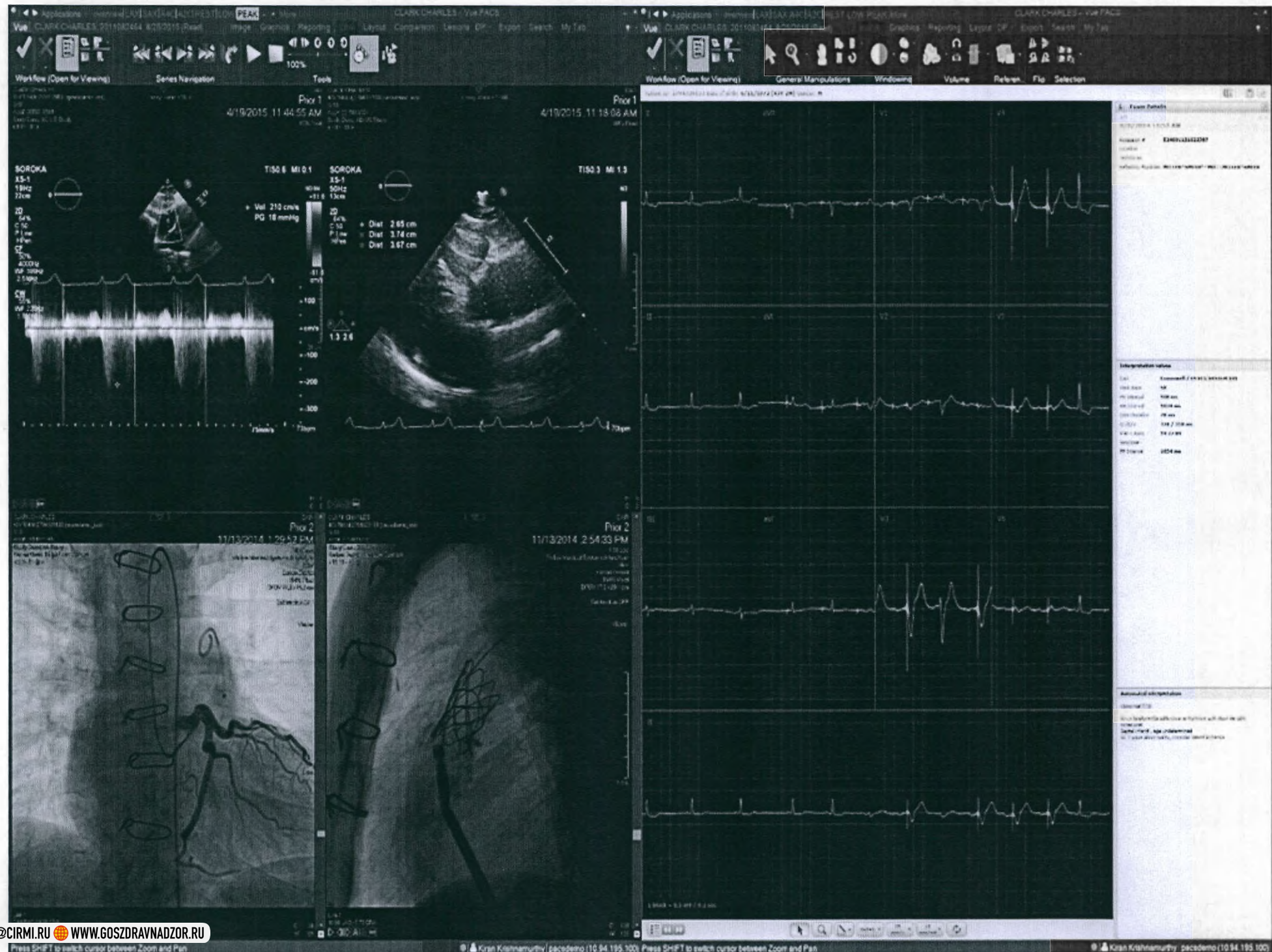

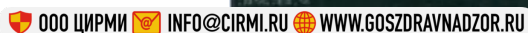

March, Allan - Explorer Portal

Archive Explorer March, Allan

Name: March, Allan Date of Birth: 10/19/1942 (76Y 11M) Patient ID: 200932798456 Gender: M

Current exam Modality: CT Body Part: CHEST Description: CT Chest w/ Contrast Exam Date: 10/5/2019 Accession Number: 9275000039533 Referring Physician: Evans David New Exam

Exams

Date	Mod	Body Part	# Images	Pa
10/5/2019	CT	CHEST	278	False
1/30/2019	CT	THX	275	True
1/5/2019	E	CARDIAC	1	False
4/7/2018	OT	PATHO	3	False

CHEST CT 9275000039533
10/5/2019 3:21:00 PM (278 images) ONLINE UNREAD

Thumbnails Report Order Bookmarks Exam Data Documents Critical Findings (1)

Sticky Notes (1)

Documents (1)

Scanned order
Scanned order

Scanned order - Scanned order 1 / 1

Philips Healthcare
Imaging Department

Diagnostic Exam ORDER

1 ☐ Male
2 ☐ Female

☐ Routine ☐ Urgent

Scheduled Exam Date

☐ Allergic to Drugs ☐ On isolation LMP ☐ Pregnant

Ordering Provider (Print) Attending Physician (Print)

Ordering Provider (Signature) Provider's ID Code Phone or Beeper Insurance authorization number if applicable

STAMP PATIENT'S IDENTIFICATION OR PRINT CLEARLY

Patient's Name (LAST, FIRST, MI.)

Exams ▾

HATCHER AMY QA FLOW ▾

Create Order - QA FLOW HATCHER AMY

Order Created By: secsec, 3/26/2015 Order Last Modified By: sec sec, 3/26/2015

Patient Information

Name: QA FLOW HATCHER AMY ID: 63635 Date of Birth: 8/11/1932 Gender: M

Order Information

Status

Cancelled ▾

Department

Institution Name

Confidentiality Code

Restricted ▾

Accession Number1934411292

Priority

Routine ▾

Reason for Exam *

Referring Physician Notes/Interpretation

Tech Notes

Clinical Info

Referring Physician

Choose a Referring Physician...

Manage Referring Physicians

Technician

Choose a Technician...

Exam Details:

☐ Order without imaging study

Scheduled Date16/21/2014

Device

No. of Images*

Attach To Study

Modality

Choose a Modality...

Body Part Group

Choose Body Part Group...

Body Part

Choose Body Part...

Procedure Code & Text

Choose a Procedure Code & Text...

Result Distribution Details:

☒ Send Results to Primary Referring Physician

Additional Physicians CCE

Choose a Referring Physician

Reprint Via Fax

Result Via Email Notification

ООО ЦРМИ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

File Tools Help

Users Groups Profiles

Name	Conc. Users	Current Users	Expiration Date	Version
Diagnostic	Unlimited	0	permanent	12.1
MyVue	Unlimited	0	permanent	12.1
Print Station	Unlimited	0	permanent	12.1
Referring	Unlimited	0	permanent	12.1
Super User	1	0	permanent	12.1
Transcription	Unlimited	0	permanent	12.1
VA+	Unlimited	0	permanent	12.1
Vue Motion	Unlimited	0	permanent	12.1

Profile Content

Features

- 3d package**
- Action buttons
- Advanced annotations and measurements
- Advanced CR Measurements
- Advanced layout and DP package
- Advanced printing package
- Application Settings UI
- Basic annotations and measurements


Feature Properties Dependent Permissions

Feature Name: 3d package

Feature number of licenses: Unlimited


Profile Status


OK

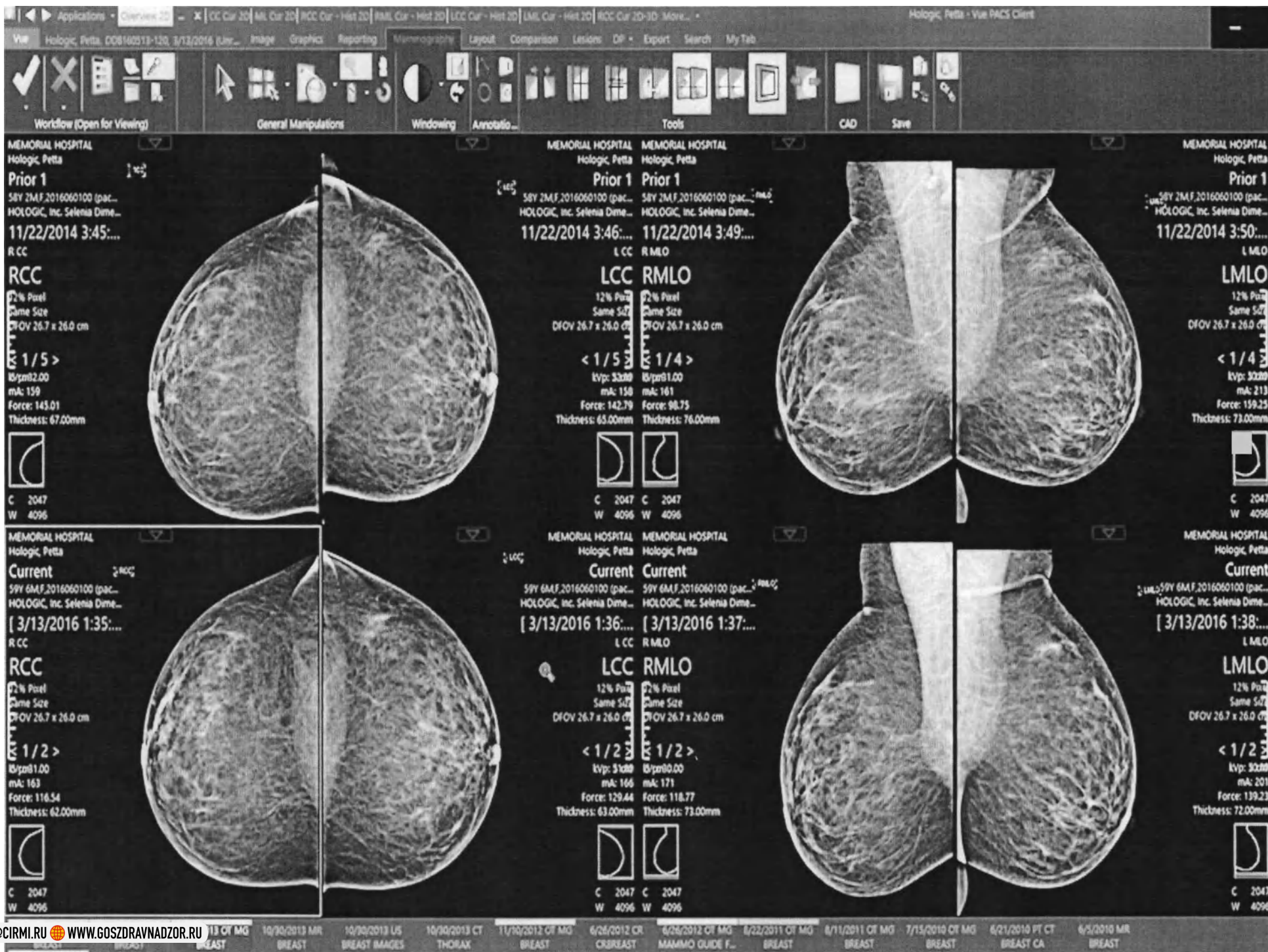


88. Ключ лицензионный для активации программного модуля UNLIMITED READING LIC в вариантах исполнения: не более 1 000 исследований, не более 10 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований 99



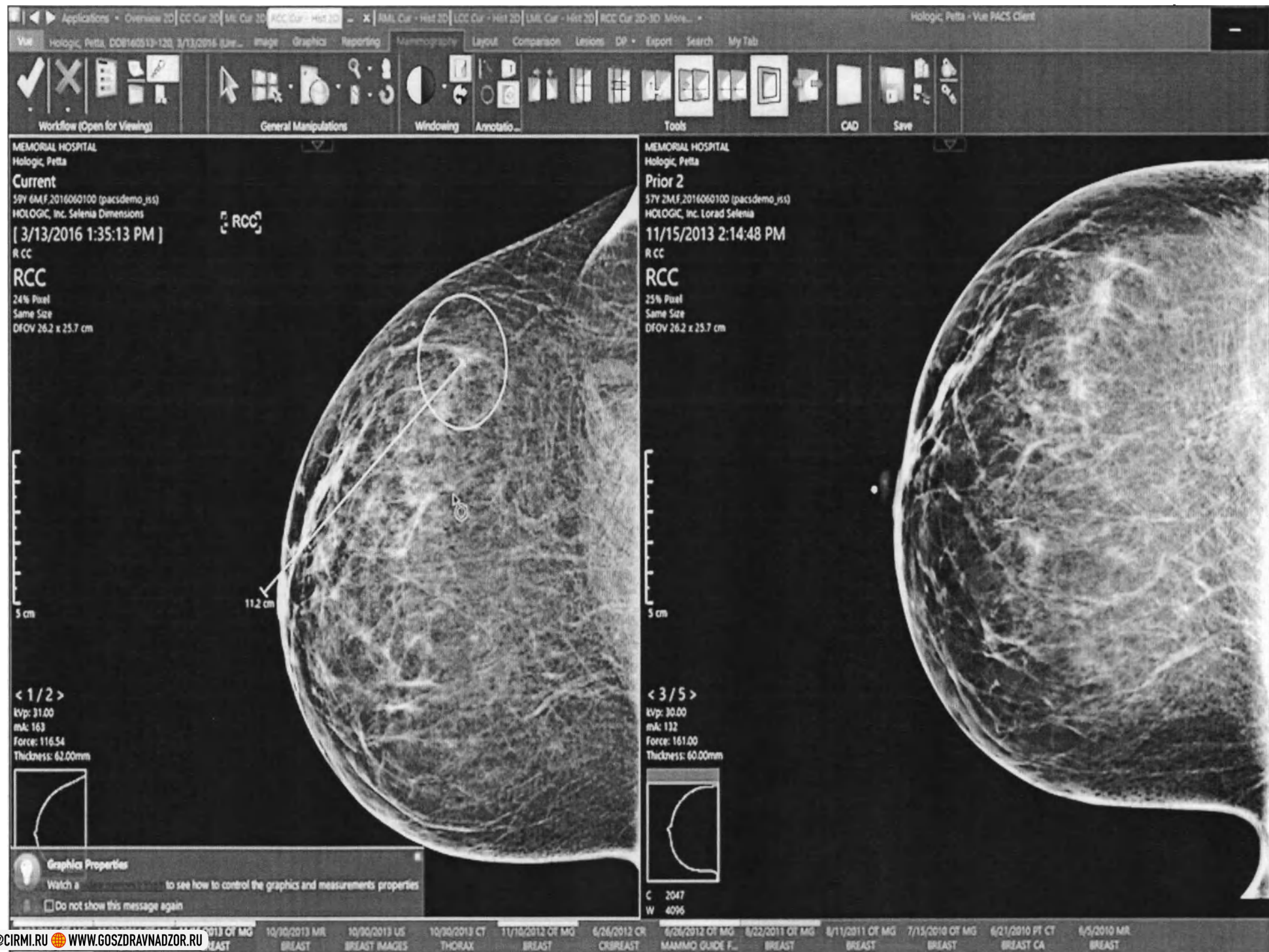


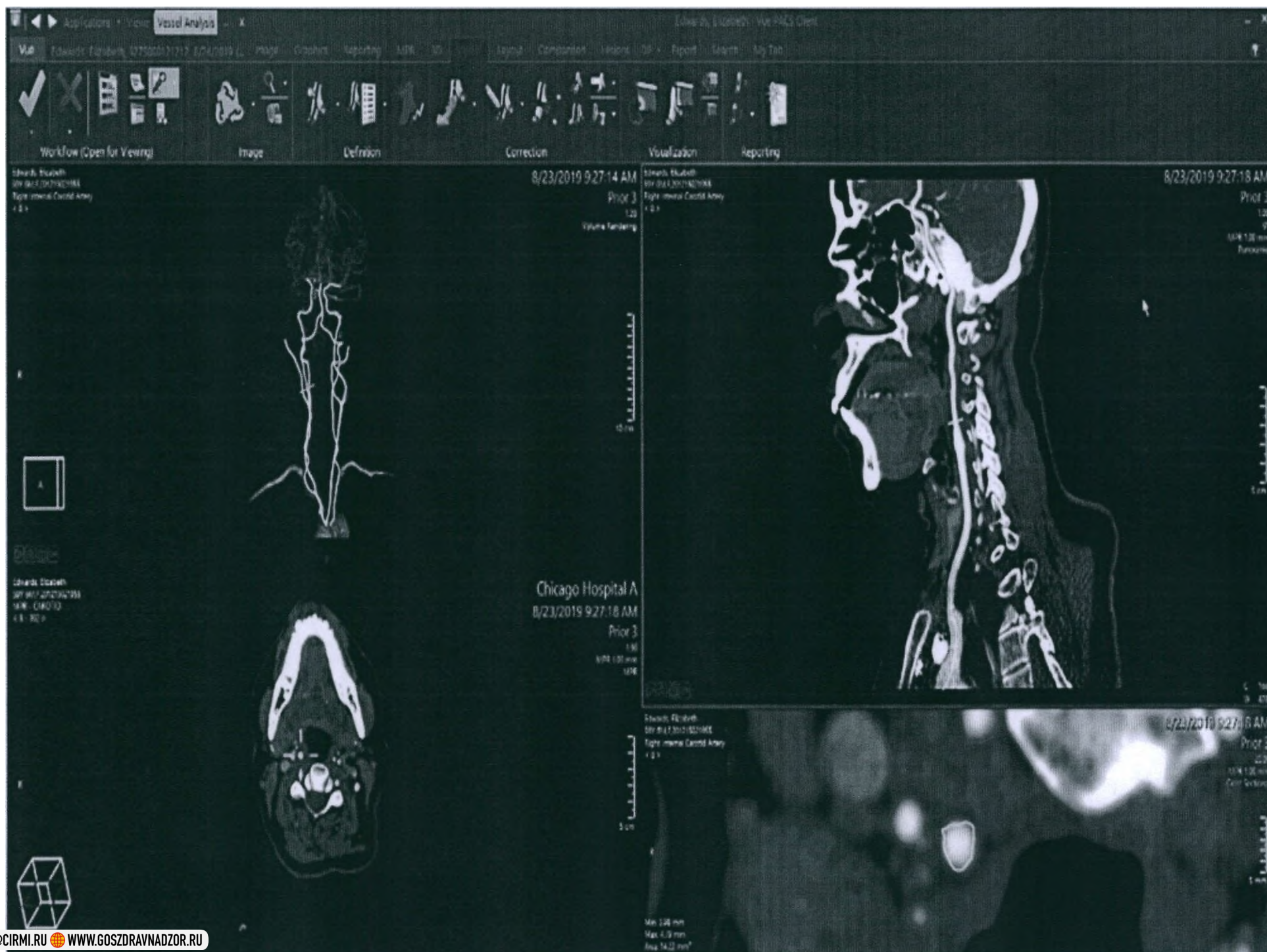

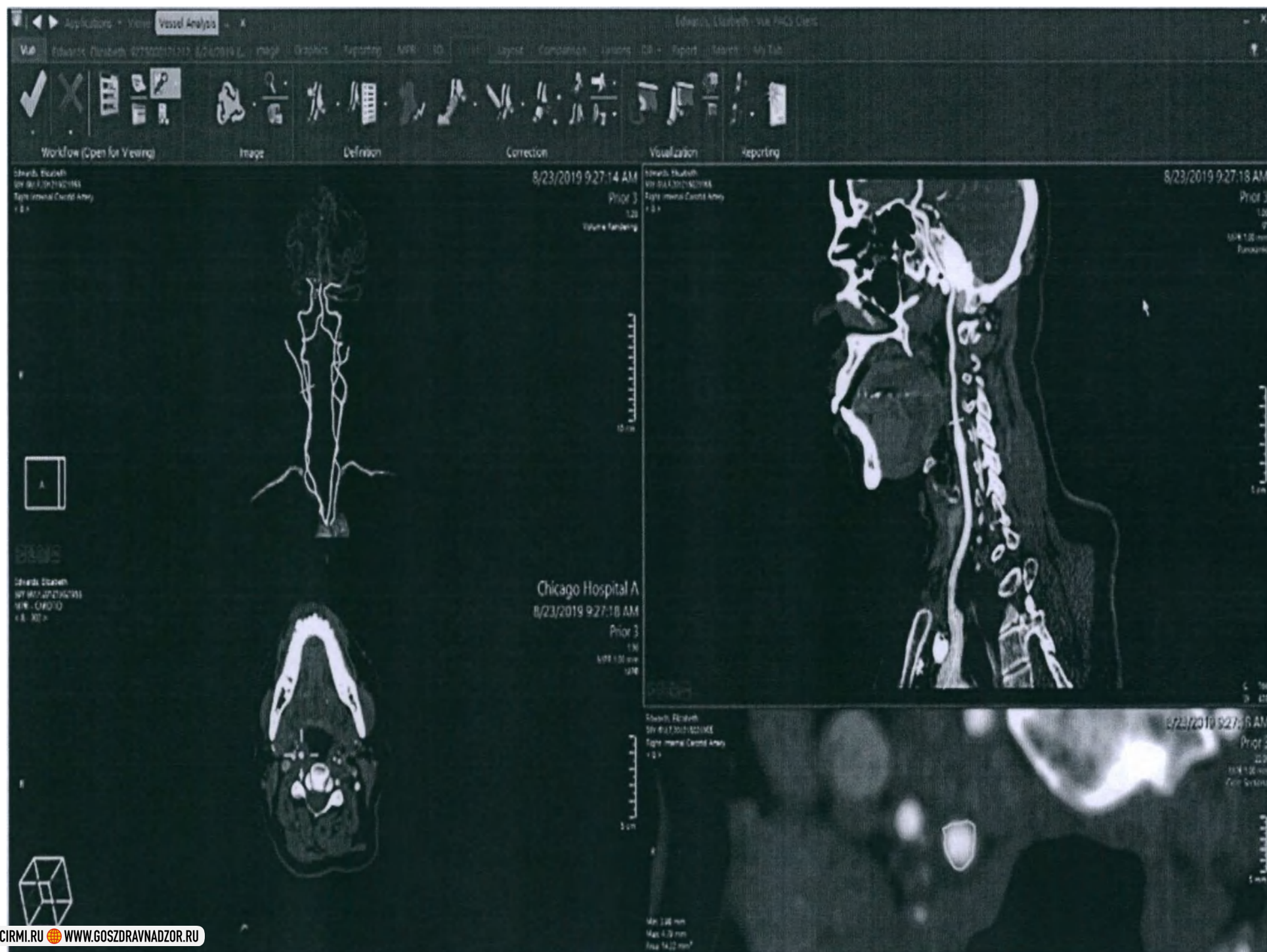

У2. Ключ лицензионный для активации программного модуля VESSEL ANALYSIS UNLIMITED в вариантах исполнения: не более 1 500 исследований, не более 15 000 исследований, не более 25 000 исследований, не более 50 000 исследований, дополнительно не более 10 000 исследований, дополнительно не более 25 000 исследований, дополнительно не более 100 000 исследований



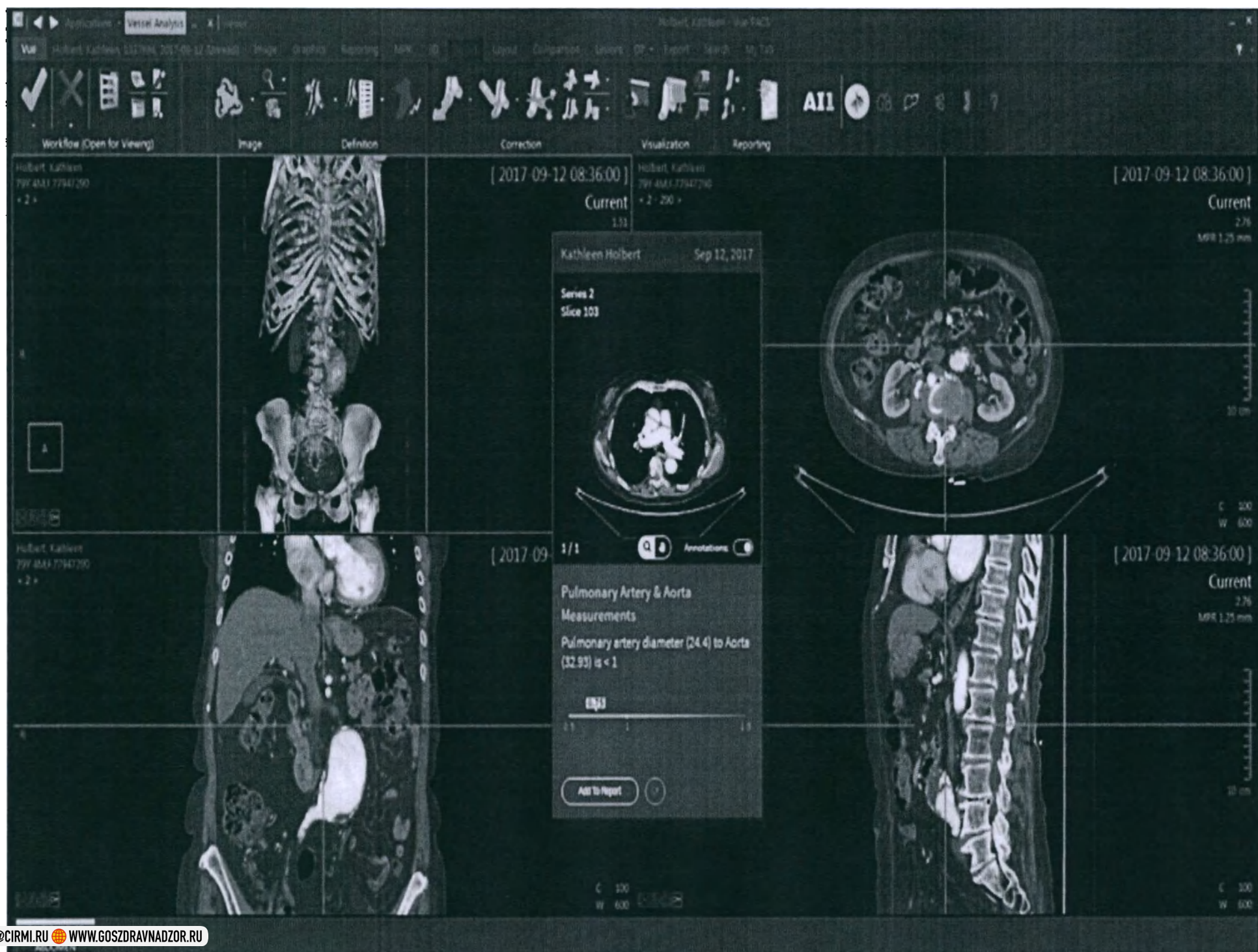

File Tools Help

Users Groups Profiles

Name	Conc. Users	Current Users	Expiration Date	Version
Diagnostic	Unlimited	0	permanent	12.1
MyVue	Unlimited	0	permanent	12.1
Print Station	Unlimited	0	permanent	12.1
Referring	Unlimited	0	permanent	12.1
Super User	1	0	permanent	12.1
Transcription	Unlimited	0	permanent	12.1
VA+	Unlimited	0	permanent	12.1
Vue Motion	Unlimited	0	permanent	12.1

Profile Content

Features

- 3d package
- Action buttons
- Advanced annotations and measurements
- Advanced CR Measurements
- Advanced layout and DP package
- Advanced printing package
- Application Settings UI
- Basic annotations and measurements


Feature Properties Dependent Permissions

Feature Name: 3d package

Feature number of licenses: Unlimited

Profile Status

OK

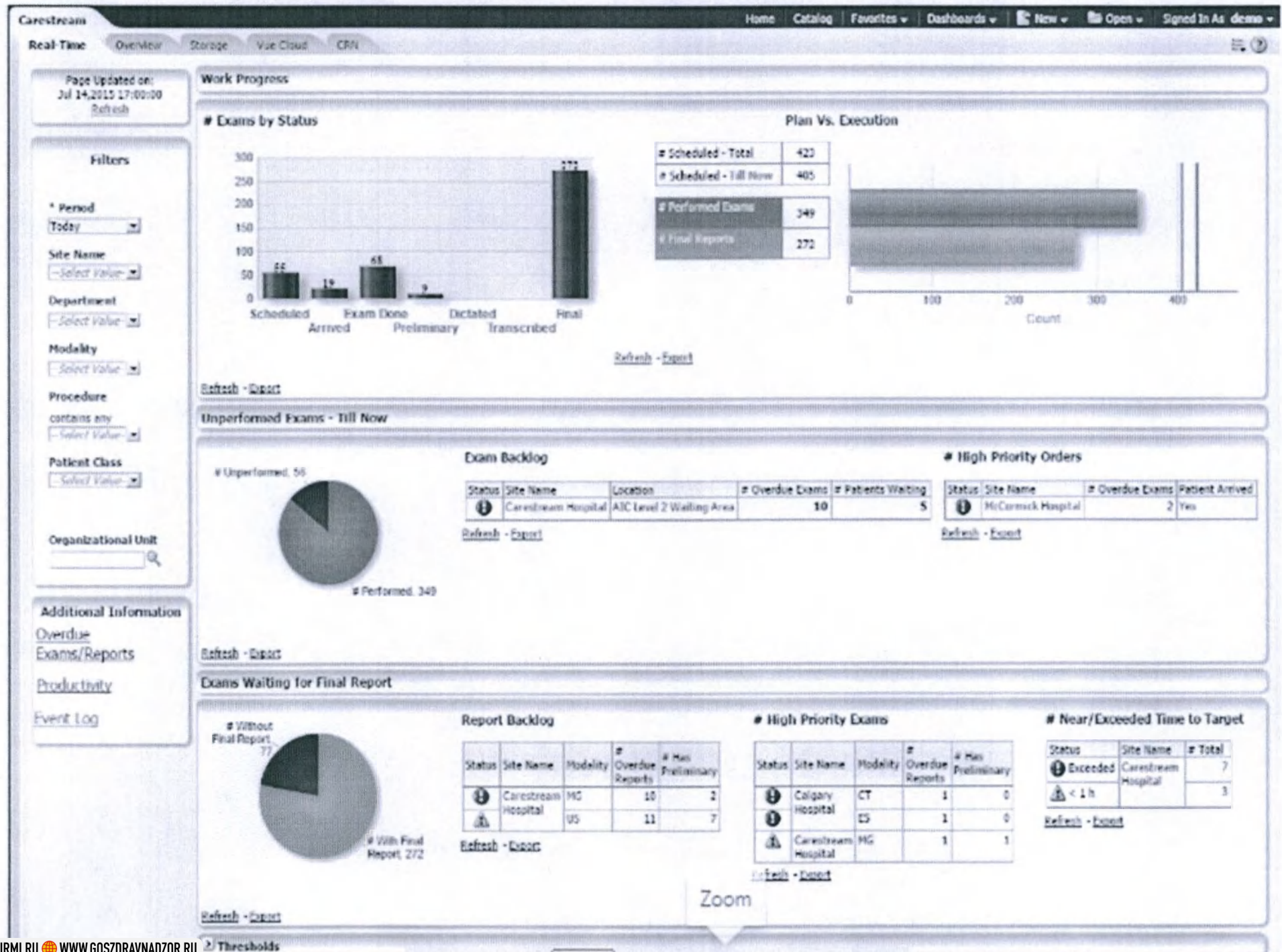

File Edit View Tools



Command execution status, no filter applied

Total: 1368 Running: 1 Failed: 0 Succeeded: 1367 Hold: 0 Scheduled: 0 Skipped: 0 Waiting: 0 Not started: 0 Delayed: 0

Atom Only v

Command execution status, according to filter criteria

Total: 9 Running: 1 Failed: 0 Succeeded: 8 Hold: 0 Scheduled: 0 Skipped: 0 Waiting: 0 Not started: 0 Delayed: 0

Atom Only
All
Group Only

Status	P...	ID	Type	Patient Name	Patient ID	Issuer	Progress	Rule ID	Host	Event Time	Scheduled Time	Date Started	Date Compl
Running	HL...	3936	Dynamic Copy	QA FLOW WOLF SARAH	85471	vmnero_iss	50.0%	1 - Enable Copy	vmnero	24-Jun-2020 08:48:27	24-Jun-2020 08:48:27	24-Jun-2020 08:48:31	
✓ Succeeded	HL...	3935	Consolidate SR	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:52	24-Jun-2020 08:47:55	24-Jun-2020 08:47:55	24-Jun-2020
✓ Succeeded	HL...	3934	Copy SR	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:52	24-Jun-2020 08:47:55	24-Jun-2020 08:47:55	24-Jun-2020
✓ Succeeded	HL...	3933	Copy Key Images	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:52	24-Jun-2020 08:47:55	24-Jun-2020 08:47:55	24-Jun-2020
✓ Succeeded	HL...	3931	Consolidate SR	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:42	24-Jun-2020 08:47:47	24-Jun-2020 08:47:47	24-Jun-2020
✓ Succeeded	HL...	3930	Copy SR	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:42	24-Jun-2020 08:47:46	24-Jun-2020 08:47:46	24-Jun-2020
✓ Succeeded	HL...	3929	Copy Key Images	LAST200 FIRST	PID200	vmnero_iss	100.0%	2 - Create Struc...	vmnero	24-Jun-2020 08:47:42	24-Jun-2020 08:47:46	24-Jun-2020 08:47:46	24-Jun-2020
✓ Succeeded	HL...	3927	Dynamic Copy	QA FLOW ASHOOD JACK	321321	vmnero_iss	100.0%	1 - Enable Copy	vmnero	24-Jun-2020 08:47:23	24-Jun-2020 08:47:23	24-Jun-2020 08:47:26	24-Jun-2020
✓ Succeeded	ME...	3926	Run An Execut...				100.0%	141 - Update MV...	vmnero	24-Jun-2020 08:00:00	24-Jun-2020 08:00:00	24-Jun-2020 08:00:05	24-Jun-2020

Selected commands: 1/9

Command information log (ID: 3936)

Jun 24, 2020 08:48:31, execution number 1: Running in vmnero...

Jun 24, 2020 08:48:31, execution number 1: STUDY_INSTANCE_UID: 1.2.840.113704.7.1.0.10253159195236207.1323154593.0

Jun 24, 2020 08:48:31, execution number 1: Copy requested by ADMIN TOOL

Jun 24, 2020 08:48:31, execution number 1: Association requested to: vmneroFIR

Jun 24, 2020 08:48:31, execution number 1: Association confirmed.

Jun 24, 2020 08:48:31, execution number 1: Copy requested for patient: QA FLOW WOLF SARAH

Jun 24, 2020 08:48:31, execution number 1: Copy requested for patient id: 85471

Jun 24, 2020 08:48:31, execution number 1: Copy requested for accession number:

Jun 24, 2020 08:48:31, execution number 1: Requesting copy for 1 STUDY

Jun 24, 2020 08:48:31, execution number 1: PR Move Policy: Move with referenced image

Jun 24, 2020 08:48:31, execution number 1: KOS Move Policy: Move with referenced image

Jun 24, 2020 08:48:31, execution number 1: Copy request for study_instance_uid: 1.2.840.113704.7.1.0.10253159195236207.1323154593.0

Jun 24, 2020 08:48:31, execution number 1: Patient details will be de-identified

Jun 24, 2020 08:48:53, execution number 1: Copy to vmneroFIR in progress, 450 completed, 0 failed.



10.94.195.200/portal/compare=1&user_name=...&password=...&bmkid=9CAD0095-837D-496D-9092-1826511E5F13&study_instance_uid=1.2.840.113704.7.1.0.113820926179105.1443622510.0&series_instance_uid=1.2.840.113704.7.1.0.113820926179105.1443622510.0

Carisstream DAVIS DOROTHY

Patient ID: 201201051940 Date of Birth: 6/1/1943 (71Y BM) Gender: F

Patient History

Filter studies by

Unit Months Unit Years All (0) (2) (8) (12)

All studies (12)

2/2/2015 11:54 AM CT CHEST

1/5/2015 7:05 AM ECG CARDIAC

9/28/2014 2:47 PM CT

6/23/2014 3:10 PM CT CHEST

3/31/2014 2:30 PM CT

1/26/2014 2:27 PM OT PATHOLOGY REPORT (Pathology Report.pdf)

1/15/2014 7:26 AM OT BIOPSY VIDEO Lung Biopsy (Lung Biopsy.m4)

12/25/2013 9:31 AM OT LOBECTOMY VIDEO Lobectomy Procedure (Left Upper Lobec

9/20/2011 2:20 PM CR SKULL

Patient Reports

1/1 John Jennings 2/2/2015 (Report matches study selected in Patient History pane)

PROCEDURE: CT Chest

CLINICAL INDICATION: Follow-up of a known left-sided squamous cell carcinoma of the lung post-surgery with suspected lung metastasis

TECHNIQUE: CT scan of the chest without contrast was performed on the GE volumetric 64 slice CT scanner. 3-D coronal reformatted images were obtained from the axial source images.

COMPARISON: CT March 31 2014, CT June 23 2014, CT September 28 2014

FINDINGS: Status post left upper lobectomy. Several nodules are seen in the left lower lobe: in the upper posterior with a diameter of (14 mm), another upper anterior nodule (2 mm) (larger than in the previous scan) and mid-posterior nodule (12 mm) (not larger than in the previous scan). There is a very small nodule in the right upper lobe with no change compared with the previous study. Some emphysematic bulbs are seen in the right lung. There is no acute infiltrate. The mediastinum is shifted to the left (due to the lobectomy) without evidence for mass or lymphadenopathy. The vascular structures of the mediastinum are normal in course and caliber. Aortic vascular

Bookmark Information

Name	801
Description	Lesion (Lung)
Volume (cm ³)	0.4
Volume Error (%)	+/-25%
Long Diameter (cm)	1.4
Short Diameter (cm)	0.6
HU Mean (HU)	-159

Follow Up Information

Exam Date	Lung Diameter (cm)	Trend
2/2/2015	1.4	+5.8% ↑
9/28/2014	1.3	-1.1% ↓
6/23/2014	1.4	+4.5% ↑
3/31/2014	1.3	0.0%

↑ Trend according to baseline 801/001

Notes

1/1 John Jennings

Purpose: Critical Results - Notification

Existing lung nodules have mildly increased in size.

2/2/2015 2:17 PM

Vue Report Editor

Workflow (Open for Viewing) Workflow Options Advanced Settings

Accession Number: 927500023456
Relation for Study: Followup of a lesion
Procedure Code: 71250
Description: CT thorax; w/o cont
Expected # of Images: 305
Clinical Info

Unread: 10/19/2017, 8:46 PM | Unread: 10/19/2018, 8:46 PM

TECHNIQUE: CT scan of the chest without contrast was performed on the GE volumetric 64 slice CT scanner.
3-D coronal reformatted images were obtained from the axial source images.

Total DLP: 798.75 mGycm Total CTDI: 17.21 mGy

COMPARISON: [None]

FINDINGS:
The lungs are clear. No focal opacification, effusion, pneumothorax, edema, or nodules are seen. There is no acute infiltrate.
The mediastinum is unremarkable without evidence for mass or lymphadenopathy. The vascular structures of the mediastinum are normal in course and caliber. Aortic vascular calcifications and coronary artery calcifications are present. The heart size is normal without evidence for pericardial thickening or effusion.
The axillary regions, subpectoral regions, and supraclavicular regions are all unremarkable. Imaging obtained through the upper abdomen reveals no acute abnormality. The surrounding osseous structures are remarkable for mild degenerative spondylosis of the spine. No osteolytic or osteoblastic lesion is detected.

Pat. Pat. D. A. Descri...
Davi... 201... 1/5... 00... E...
Davi... 201... 9/28... 92... CT 2D R... C...

CMC
CHICAGO MEDICAL CENTER
9/28/2017

Patient
Name: Davis, Dorothy
ID: 201201061940
Accession Number: 9275000234567

PROCEDURE: CT Chest
CLINICAL INDICATION: Known left-sided squamous cell carcinoma of the lung post-surgery with suspected lung metastasis
TECHNIQUE: CT scan of the chest without contrast was performed on the GE volumetric 64 slice CT scanner. 3-D coronal reformatted images were obtained from the axial source images.
COMPARISON: CT 23/06/2017 : CT

Field Options

000 ЦРММ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU

Configuration Options		Status	ID	Description	Function	Event	Command	Filter	Da ^
+	Workflow Configuration	☒ Active	1	Enable Copy	Enable Copy	Copy Request	Dynamic Copy		10-
-	Application Configuration	☒ Active	2	Create Structured Report When RIS R...	Create Structure...	RIS	Group SR	((MESSAGE_TYPE contains ...	10-
-	Nodes Configuration	☒ Inactive	3	Create Structured Report When Study...	Create Structure...	Whole Stud...	Create SR ...	((IMAGE_SOURCE is not "h...	10-
-	Workflow Manager	☒ Active	4	Create Structured Report When MV S...	Create Structure...	Buffered Me...	Create SR	((messageType is "SR") A...	10-
-	homer	☒ Active	5	Set HasSR When RIS PACS Msg Arrives	Set HasSR When ...	RIS	Set tag HasSR	((MESSAGE_TYPE contains "...	10-
-	HL7-PACS Field Mapping	☒ Active	6	Enable Generate SR	Enable Generate SR	Generic Probe	Create SR ...	((ORIGINATOR contains "Ge...	10-
-	Default Mapping Set	☒ Active	7	Migrate	Migrate	Internal Cop...	Perform Mig...	((DESTINATION_LIST is not...	10-
-	Dictionary	☒ Inactive	8	Send SCN Message over HL7 - Sample	Custom Function	Whole Stud...	Run An Exe...		10-
-	Simple Mapping Tables	☒ Inactive	9	E-Mail Report To Referring Physicians	E-Mail Report	Buffered Me...	Send E-Report	((TAMAR_REPORT_STATUS ...	10-
-	Complex Mapping Tables	☒ Active	10	Series Icons Generator	Series Icons Gen...	Whole Stud...	Create serie...		10-
-	Workflow Manager Configuration	☒ Active	11	Create Series Icons For Existing Studies	Create Series Ico...	Generic Probe	Create serie...	((ORIGINATOR is "CREATE_...	10-
-	Dicom Parsing	☒ Active	12	Create HCFF	Create HCFF	Generic Probe	Create HCFF	((ORIGINATOR is "CREATE_...	10-
-	Study Grouping	☒ Active	13	Copy to CD-Direct	Copy to CD-Direct	Internal Cop...	Copy to CD-...	((INITIATOR is "CDSurf")	10-
-	Pre-Fetch	☒ Active	14	Create Duplicate Study	duplicate_study	Duplicate St...	Duplicate St...		10-
-	Icons Settings	☒ Inactive	15	Patient Portal - publish images/reports	Patient Portal - p...	SQL Query	Update DIC...		10-
-	Compression Settings	☒ Inactive	16	Patient Portal - send email on publish	Patient Portal - s...	Tags Changed	Send patien...	((NEW_TAMAR_STUDY_PUB...	10-
-	Init Values	☒ Inactive	17	Patient Portal - send email on addendum	Patient Portal - s...	RIS	Send patien...	((MESSAGE_TYPE contains "...	10-
-	System	☒ Inactive	18	Patient Consent Flow	Patient Consent ...	RIS	Patient user...	((CONSENT_TEXT is "Opt-in"...	10-
-	homer	☒ Inactive	19	Referring Consent Flow	Referring Consen...	rissync	Patient user...		10-
-	IS Link Configuration	☒ Active	20	HL7 Manual Distribution	Manual Distribute...	Manual Dist ...	Report Distri...	((TAMAR_HAS_DRAFT_REP...	10-
-	Converter	☒ Active	21	Fax And Email Manual Distribution	Manual Distribute...	Manual Dist ...	Report Distri...	((TAMAR_HAS_DRAFT_REP...	10-
-	Listeners	☒ Inactive	41	FAX distribution	Distribute PACS R...	Buffered Me...	Report Distri...	((messageType is "SR") AN...	10-
-	Database	☒ Inactive	42	Email distribution	Distribute PACS R...	Buffered Me...	Report Distri...	((messageType is "SR") AN...	10-
-	Reports & Orders	☒ Inactive	61	Create Work List Item for Order With ...	Create Work List ...	RIS	Create Empt...	TAMAR_STUDY_CREATION...	10-
-	Report Parser	☒ Inactive	62	Create Work List Item for Linked Orde...	Create Work List ...	RIS	Create Empt...	TAMAR_STUDY_CREATION...	10-
-	Queues & Notifications	☒ Inactive	63	Create Work List Item for Linked Orders	Create Work List ...	rissync	Create Empt...	TAMAR_STUDY_CREATION...	10-
-	Info-Router	☒ Inactive	64	Acquisition Started	Order Status Man...	First/Additio...	Order Statu...		10-
-	Rules								

Rule Information

Configuration Options	Status	ID	Description	Function	Event	Command	Filter	Da ^
Workflow Configuration	Active	1	Enable Copy	Enable Copy	Copy Request	Dynamic Copy		10-
Application Configuration	Active	2	Create Structured Report When RIS R...	Create Structure...	RIS	Group SR	((MESSAGE_TYPE contains ...	10-
Nodes Configuration	Inactive	3	Create Structured Report When Study...	Create Structure...	Whole Stud...	Create SR ...	((IMAGE_SOURCE is not "h...	10-
Workflow Manager	Active	4	Create Structured Report When MV S...	Create Structure...	Buffered Me...	Create SR	((messageType is "SR") A...	10-
homer	Active	5	Set HasSR When RIS PACS Msg Arrives	Set HasSR When ...	RIS	Set tag HasSR	((MESSAGE_TYPE contains "...	10-
HL7-PACS Field Mapping	Active	6	Enable Generate SR	Enable Generate SR	Generic Probe	Create SR ...	((ORIGINATOR contains "Ge...	10-
Default Mapping Set	Active	7	Migrate	Migrate	Internal Cop...	Perform Mig...	((DESTINATION_LIST is not...	10-
Dictionary	Inactive	8	Send SCN Message over HL7 - Sample	Custom Function	Whole Stud...	Run An Exe...		10-
Simple Mapping Tables	Inactive	9	E-Mail Report To Referring Physicians	E-Mail Report	Buffered Me...	Send E-Report	((TAMAR_REPORT_STATUS ...	10-
Complex Mapping Tables	Active	10	Series Icons Generator	Series Icons Gen...	Whole Stud...	Create serie...		10-
Workflow Manager Configuration	Active	11	Create Series Icons For Existing Studies	Create Series Ico...	Generic Probe	Create serie...	((ORIGINATOR is "CREATE_...	10-
Dicom Parsing	Active	12	Create HCFF	Create HCFF	Generic Probe	Create HCFF	((ORIGINATOR is "CREATE_...	10-
Study Grouping	Active	13	Copy to CD-Direct	Copy to CD-Direct	Internal Cop...	Copy to CD-...	((INITIATOR is "CDSurf")	10-
Pre-Fetch	Active	14	Create Duplicate Study	duplicate_study	Duplicate St...	Duplicate St...		10-
Icons Settings	Inactive	15	Patient Portal - publish images/reports	Patient Portal - p...	SQL Query	Update DIC...		10-
Compression Settings	Inactive	16	Patient Portal - send email on publish	Patient Portal - s...	Tags Changed	Send patien...	((NEW_TAMAR_STUDY_PUB...	10-
Init Values	Inactive	17	Patient Portal - send email on addendum	Patient Portal - s...	RIS	Send patien...	((MESSAGE_TYPE contains "...	10-
System	Inactive	18	Patient Consent Flow	Patient Consent ...	RIS	Patient user...	((CONSENT_TEXT is "Opt-in"...	10-
homer	Inactive	19	Referring Consent Flow	Referring Consen...	rissync	Patient user...		10-
IS Link Configuration	Active	20	HL7 Manual Distribution	Manual Distribute...	Manual Dist ...	Report Distri...	((TAMAR_HAS_DRAFT_REP...	10-
Converter	Active	21	Fax And Email Manual Distribution	Manual Distribute...	Manual Dist ...	Report Distri...	((TAMAR_HAS_DRAFT_REP...	10-
Listeners	Inactive	41	FAX distribution	Distribute PACS R...	Buffered Me...	Report Distri...	((messageType is "SR") AN...	10-
Database	Inactive	42	Email distribution	Distribute PACS R...	Buffered Me...	Report Distri...	((messageType is "SR") AN...	10-
Reports & Orders	Inactive	61	Create Work List Item for Order With ...	Create Work List ...	RIS	Create Empt...	TAMAR_STUDY_CREATION...	10-
Report Parser	Inactive	62	Create Work List Item for Linked Orde...	Create Work List ...	RIS	Create Empt...	TAMAR_STUDY_CREATION...	10-
Queues & Notifications	Inactive	63	Create Work List Item for Linked Orders	Create Work List ...	rissync	Create Empt...	TAMAR_STUDY_CREATION...	10-
Info-Router	Inactive	64	Acquisition Started	Order Status Man...	First/Additio...	Order Statu...		10-

Rule Information



Vue PACS

Picture Archiving and Communication System
Version 12

LOT 12.2.5 REF 8966424

Rx only

Quantity 1

© Koninklijke Philips N.V. 2019. All rights are reserved. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical or otherwise, is prohibited without the prior written consent of the copyright owner.

Copyrights and all other proprietary rights in any software and related documentation ("Software") made available to you rest exclusively with Philips or its licensors. No title or ownership in the Software is conferred to you. Use of the Software is subject to the end user license conditions as are available on request.

To the maximum extent permitted by law, you shall not decompile and/or reverse engineer the software or any part thereof.

Build: 12.2.51021



Philips Medical Systems Nederland B.V.
Veenpluis 6, 5684 PC, Best, The Netherlands



<https://www.philips.com/IFU>



2020-04-23



0197

(01)00664838100336
(11)200423 (10)12.2.5

Обеспечение программное медицинское View PACS для получения, хранения, архивирования, обмена и обработки медицинских изображений и данных на виртуальных и/или электронных носителях с принадлежностями, в составе

Версия: ...

Номер по каталогу: ...

Серийный номер:

Дата производства:

**Производитель: Philips Medical Systems Nederland B.V.
(«Филипс Медикал Системс Нидерланд Б.В.»), Нидерланды
Veenpluis 6, 5684 PC Best, The Netherlands**

Расшифровку графических символов: см. руководство пользователя

Уполномоченный представитель производителя:

ООО «ФИЛИПС», Россия,

123022, г. Москва, ул. Сергея Макеева, 13

Телефон/факс: + 7 (495) 937 93 00/ + 7 (495) 937 93 59,

medservice.russia@philips.com

Регистрационное удостоверение № _____ от _____ г.

ВСЕГО ПРОШИТО.
ПРОНУМЕРОВАННО И
СКРЕПЛЕНО ПЕЧАТЬЮ
114 ЛИСТА / ОВ /

