Approved by China Tianjin Recare CO., LTD
General Manager

CHINA TIANJIN RECARE CO., LTD
signature/stamp

Chang Yajing

20" September 2023

USER MANUAL

MALE CONDOMS WITH NATURAL LATEX IN OPTIONS LIGHT, CLASSIC,
LONGER, DOTTED&RIBBED, POWER, XXL, ORGANIC

CONTENTS

1. PURPOSE AND INDICATIONS	4
2. TECHNICAL REQUIREMENTS	4
3. SAFETY REQUIREMENTS	10
4. INTENDED USE	10
5. USE	11
6. TRANSPORTATION AND STORAGE	11
7. DISPOSAL AND ENVIRONMENTAL REQUIREMENTS	11
8. INSTRUCTIONS FOR USE	12
9. INFORMATION ABOUT STUDY IN ANIMALS	12
10. INFORMATION ABOUT MATERIALS OF HUMAN AND ANIMAL ORIGIN CONTAINED IN THE MEDICAL DEVICE	12
11. INFORMATION ABOUT THE PRESENCE OF MEDICAL PRODUCTS AS PART OF THE COMPOSITION OF THE MEDICAL DEVICE	12
12. INFORMATION ABOUT STERILIZATION	12
13. REQUIREMENTS FOR MAINTENANCE AND REPAIR	10
14. INFORMATION ABOUT COMPLAINTS	12
15. MANUFACTURER'S WARRANTY	13
ATTACHMENT A	14

Present manual applies to male condoms made with natural latex (hereinafter referred to as condoms, product), and is a consolidated document containing information about the purpose, principle of operation and characteristics of the product necessary for proper operation, transportation, storage, maintenance and disposal, as well as manufacturer's warranty information.

 Information important for the safe use of the product is marked with a warning triangle with an exclamation mark. These instructions should be given special attention.

For all questions regarding the use of the product, please contact the manufacturer or their authorized representative:

China Tianjin Recare CO., LTD, China

Address: B106, No.12 Kaihua Street, Huayuan Industrial Area, Tianjin, China

Production location: Building 13-3, Shengjinyuan, Nancheng street, Nankai District, 300101

Tianjin, PEOPLE'S REPUBLIC OF CHINA

Tel.: 0086-22-27285791

E-mail: market8@recare.com.cn

Limited Liability Company OOO "BTK"

Address: 194156, Saint Petersburg, Engelsa prospekt, building No. 34, letter V, premises 14-N

Tel.: 8-911-726-27-72

E-mail: sidek@mail.ru

1. PURPOSE AND INDICATIONS

1.1. Purpose and conditions of use

Male condoms made with natural latex (hereinafter referred to as condoms, product) are disposable means for male contraception intended to prevent AIDS, infections, sexually transmitted diseases and prevent the risk of unwanted pregnancy.

Type of contact with the human body - short-term contact with mucous membrane and intact skin.

No special knowledge is required to use a condom. It is enough to follow the instructions for use.

Consumers are men and women who have reached puberty and are sexually active.

1.2. Indications and contraindications

Indications:

The condom is a contraceptive that protects against unwanted pregnancy and sexually transmitted diseases.

Contraindications:

Hypersensitivity or allergy to any of the components of the condom.

Side effects:

Condoms are made with natural latex, which can cause an allergic reaction, including anaphylactic shock, if the user is allergic to latex.

2. TECHNICAL REQUIREMENTS

2.1 Main parameters and characteristics

2.1.1 The condom must comply with the requirements of present extract from the technical file.

2.1.2 The parameters and characteristics of the product are shown in Table 1.

Table 1.

#	Type	Size, LxW, mm	Thickness, mm	Thickness of the rim, mm	Color	Weight, g	Texture	Pattern number
1	Light Natural Latex Condoms (Ultrathin)	185x52±2	0,06±0,3	1,5±0,3	Natural	1,8±0,3	Smooth	Pattern 1
2	Classic Natural Latex Condoms (Classic)	190x52±2	0,07±0,3	1,5±0,3	Natural	1,8±0,3		Pattern 2
3	Longer Natural Latex Condoms (Prolonging)	195x52±2	0,07±0,3	1,5±0,3	Natural	1,8±0,3		Pattern 3

4	Dotted&Ribbed Natural Latex Condoms (Dotted&Ribbed)	200x52±2	0,07±0,3	1,5±0,3	Natural	1,8±0,3	Dots and ribbed	Pattern 4
5	Power Natural Latex Condoms (Strong)	185x52±2	0,08±0,3	1,5±0,3	Natural	1,8±0,3	Smooth	Pattern 5
6	XXL Natural Latex Condoms (XXL)	190x54±2	0,06±0,3	1,5±0,3	Natural	1,8±0,3		Pattern 6
7	Organic Natural Latex Condoms (Vegan)	185x52±2	0,06±0,3	1,5±0,3	Natural	1,8±0,3		Pattern 7

2.1.3 The amount of lubricant 550 ± 150 mg.

2.1.4 The physical and mechanical properties of the condom correspond to Table 2.

Table 2

Indicator name	Indicator value	
	Before thermal ageing	After thermal ageing
Conditional tensile strength, MPa (kgf/cm ²), not less	20 (200)	17(170)
Relative elongation at break, %, not less	700	600

2.1.5 Condoms must be leakproof.

2.1.6 The appearance of the condom should be the same as in Figures 1-7.

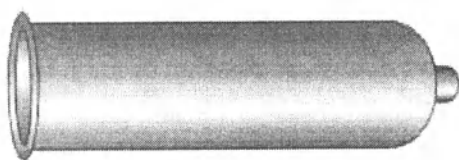
Figure 1. Light condoms (Ultrathin)

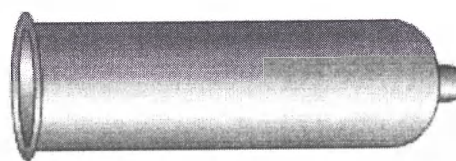
Figure 2. Classic condoms (Classic)

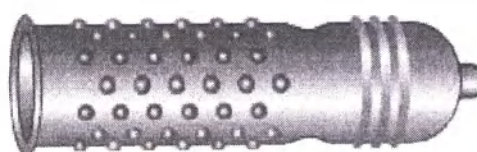
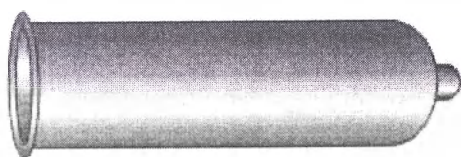
Figure 3. Longer condoms (Prolonging)

Figure 4. Dotted&Ribbed condoms (Dotted&Ribbed)

Figure 5. Power condoms (Power)

Figure 6. XXL condoms (XXL)

Figure 7. Organic condoms (Vegan)

2.1.7 The open end of the condom should be rolled to form a rim.

2.1.8 Shelf life - 5 years from the date of manufacture.

2.2 Requirements for raw materials, supplies and purchased products

2.2.1 The materials used in the manufacture of the product comply with the requirements of the current standards or regulatory and technical documentation for these materials, provided for by the current legislation, and have accompanying documents confirming the quality of the supplier.

2.2.2 Incoming control of raw materials and materials used in the manufacture of the product is carried out in accordance with the requirements of the quality management system implemented by the manufacturer.

2.2.3 For the manufacture of condoms, the manufacturer uses raw materials and supplies according to table 3.

Table 3. Raw materials and supplies

Description	Material, brand, manufacturer	Content percentage
Light Natural Latex Condoms (Ultrathin)	Natural Latex Manufacturer: Tavorn Rubber Industrial (1982) co., ltd, Thailand	96.69%
Classic Natural Latex Condoms (Classic)	Potassium Hydroxide Manufacturer: Chengdu Huarong Chemical Limited Liability Company, China	0.15%
Dotted&Ribbed Natural Latex Condoms (Dotted&Ribbed)	Casein Manufacturer: Gansu Hualing Casein co,ltd, China	0.28%
Power Natural Latex Condoms (Power)	Sulphur Manufacturer: Changzhi Sheng Chemical Co., Ltd, China	0.95%
XXL Natural Latex Condoms (XXL)	Zinc Oxide Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	1.15%

	Accelerator Px Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	0.78%
	Silicone oil Manufacturer: Beijing Dingye Industry Trade Co Ltd	18.0%
	Cornstarch powder Manufacturer: Dezhou Runde Powder Co.,ltd, China	2%
	Individual packaging: Polypropylene with aluminium foil PET 12/PE15/ALU6/PE20/LLDPE30 Micron; PET 12/ALU6/LLDPE30 Micron	-
Organic Natural Latex Condoms (Vegan)	Natural Latex Manufacturer: Tavorn Rubber Industrial (1982) co.,ltd, Thailand	96.69%
	Potassium Hydroxide Manufacturer: Chengdu Huarong Chemical Limited Liability Company, China	0.15%
	Sulphur Manufacturer: Changzhi Sheng Chemical Co., Ltd, China	0.95%
	Zinc Oxide Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	1.15%
	Accelerator Px Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	0.78%
	Silicone oil Manufacturer: Beijing Dingye Industry Trade Co Ltd	18.0%
	Cornstarch powder Manufacturer: Dezhou Runde Powder Co., ltd, China	2%
	Individual packaging: Polypropylene with aluminium foil PET 12/PE15/ALU6/PE20/LLDPE30 Micron; PET 12/ALU6/LLDPE30 Micron	-
Longer Natural Latex Condoms (Prolonging)	Natural Latex Manufacturer: Tavorn Rubber Industrial (1982) co.,ltd, Thailand	90.69%
	Potassium Hydroxide	0.15%

	Manufacturer: Chengdu Huarong Chemical Limited Liability Company, China	
	Casein Manufacturer: Gansu Hualing Casein co.,ltd, China	0.28%
	Sulphur Manufacturer: Changzhi Sheng Chemical Co., Ltd, China	0.95%
	Zinc Oxide Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	1.15%
	Accelerator Px Manufacturer: Gaoyi Province Zhengtong auxiliary factory, China	0.78%
	Benzocaine	6%
	Silicone oil Manufacturer: Beijing Dingye Industry Trade Co Ltd	18.0%
	Cornstarch powder Manufacturer: Dezhou Runde Powder Co. ,ltd, China	2%
	Individual packaging: Polypropylene with aluminium foil PET 12/PE15/ALU6/PE20/LLDPE30 Micron; PET 12/ALU6/LLDPE30 Micron	-

2.3 Package contents

Male condoms with natural latex in option light, classic, longer, dotted&ribbed, power, xxl, organic.

1. Options:

1. Light Natural Latex Condoms (Ultrathin)-3, 6, 12 pcs
- Instructions for use
2. Classic Natural Latex Condoms (Classic) -3, 6, 12 pcs
- Instructions for use
3. Longer Natural Latex Condoms (Prolonging) -3, 6, 12 pcs
- Instructions for use
4. Dotted&Ribbed Natural Latex Condoms (Dotted&Ribbed) -3, 6, 12 pcs
- Instructions for use
5. Power Natural Latex Condoms (Power) -3, 6, 12 pcs
- Instructions for use
6. XXL Natural Latex Condoms (XXL.)-3, 6, 12 pcs
- Instructions for use

7. Organic Natural Latex Condoms (Vegan)-3, 6, 12 pcs

- Instructions for use

2.4 Marking

2.4.1 Marking shall be in accordance with ISO 15223-1.

2.4.2. The following information is printed on the primary packaging: batch number, production date and expiration date.

2.4.3. One side of the consumer packaging is marked with the following symbols and information:

- product name;
- product dimensions and wall thickness;
- purpose;
- the words "follow the attached instructions for use (see insert)", "read the full text of the instructions on the website btk-med.ru"; "Consumer claims are accepted by the customer organization b-t-k@bk.ru"
- the words "electronically tested";
- expiration date;
- name of the manufacturer; 
- manufacturer's address;
- authorized person and importer in the Russian Federation;
- registration certificate;
- barcode;

- symbol "Contains natural rubber latex"; 

- symbol "Caution!"; 

- symbol "non sterile"; 

- symbol packaging recycling 

- symbol "prohibition of re-use"; 

- symbol recycling 

- symbol "do not use if packaging is damaged"; 

- temperature limit; 

- humidity range; 

- batch number symbol; 

- date of manufacture (year and month); 
- number of products.

2.4.4 Transport marking - according to GOST 14192 with additional designations:

- product name;
- name of the manufacturer;
- number of boxes.

2.4.4 Transport marking - according to GOST 14192 with additional designations:

- product name;
- name of the manufacturer;
- number of boxes.

2.5 Packaging

2.5.1 Individual packaging: each condom is individually sealed in polypropylene foil.

2.5.2 Consumer package: the product is packed in 3, 6, 12 pieces of the same type in a cardboard box wrapped in a transparent film. Instructions for use are included in each package.

2.5.3 Transport container contains up to 144 consumer packages.

3. SAFETY REQUIREMENTS

In terms of sanitary-chemical, toxicological and biological indicators, condoms must comply with the requirements of ISO 10993-1, as well as regulatory documents approved in the prescribed manner.

4. INTENDED USE


Warning!

Check the condom expiration date before using. If the individual package is damaged, discard the condom and use a new one from the undamaged package.

Warning!

Do not open the package with teeth or sharp objects.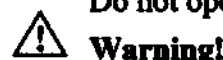

Warning!

Stop using a condom if you experience any discomfort or irritation during use.

Consult with a specialist:

- compatibility of condoms with other contraceptives;
- when using a topical medicine that is applied to the penis or inserted into the vagina;
- if irritation/discomfort persists after using a condom;
- if you are not sure if you are allergic to latex;
- if the condom leaks or breaks during use (do this as soon as possible within 72 hours).

If necessary, use only a water-based lubricant with condoms.

Warning!

Condoms are for single use only. Reuse may compromise the integrity of the product or increase the risk of transmission of sexually transmitted diseases.

Warning!

No method of contraception can guarantee 100% protection against unwanted pregnancy, HIV or other sexually transmitted diseases.

5. USE

5.1 Open the package from the serrated edge and carefully remove the condom.

5.2 Check that the condom is ready to roll on the right way.

5.3 Squeeze air from the tip of the condom, put the condom on the erect penis.

5.4 Roll the condom down the base of the penis. If the product does not roll, then it is folded in the opposite direction. Throw away this condom and use a new one.

5.5 If during intercourse you feel the condom slipping or excessive squeezing of the penis, stop and check the integrity of the product.

5.6 Immediately after ejaculation, withdraw the penis, holding the condom tightly at its base, and carefully remove the product.

5.7 Throw in the trash. Don't flush the condom down the drain.

6. TRANSPORTATION AND STORAGE

6.1 Transportation of the product can be carried out in any covered means of transport. Fastening of the transport packaging in vehicles and transportation of products in vehicles must be carried out in accordance with the rules applicable to the respective modes of transport.

6.2 The product is stored at:

- temperature from 0° to plus 25°C;

- relative air humidity up to 80%.

Transportation must be carried out in the following conditions:

- temperature from minus 50°C to plus 50°C;

- relative air humidity up to 80%.

Under these conditions, the product must retain its operational properties.

6.3 Keep the product away from direct sunlight and heat sources.

7. DISPOSAL AND ENVIRONMENTAL REQUIREMENTS

7.1 Disposal must be carried out in accordance with the rules of collection, accounting and disposal established by the authorized federal executive body, provided for by SanPiN 2.1.3684.

7.2 According to SanPiN 2.1.3684, the used product belongs to class B medical waste - epidemiologically hazardous waste.

7.3 The product is disposed of as household waste in accordance with sanitary and hygienic standards. It is forbidden to throw into the sewer.

7.4 The product must be disposed of in the event of:

- expired expiration date;

- damaged packaging;

- detection of defects.

7.5 The product has no negative impact on the environment.

8. INSTRUCTIONS FOR USE

The product must be used in accordance with the instructions for use.

Operating conditions:

- temperature from +32° to +42°C;
- relative air humidity from to 98%.
- pressure 760 mmHg

9. INFORMATION ABOUT STUDY IN ANIMALS

Studies in animals have not been conducted.

10. INFORMATION ABOUT MATERIALS OF HUMAN AND ANIMAL ORIGIN CONTAINED IN THE MEDICAL DEVICE

The product contains casein, which acts as a stabilizer. It can be considered as a derivative of tissues/materials of animal origin, because is it derived from milk. Casein does not have a toxic effect on the human body, but it can cause an individual allergic reaction.

11. INFORMATION ABOUT THE PRESENCE OF MEDICAL PRODUCTS AS PART OF THE COMPOSITION OF THE MEDICAL DEVICE

The product contains a drug for local anesthesia with a short duration of action - benzocaine.

12. INFORMATION ABOUT STERILIZATION

Product for single use, not sterile.

13. REQUIREMENTS FOR MAINTENANCE AND REPAIR

The product of single use, during operation is not subject to maintenance, restoration and repair. After use, the product must be disposed of. Reuse is prohibited.

14. INFORMATION ABOUT COMPLAINTS

Claims against the manufacturer are submitted in case of non-compliance of the supplied product, its packaging, marking and the packaging contents with the requirements of the accompanying documentation.

All complaints submitted must be registered by the manufacturer and contain information on the measures taken.

The consumer shall be notified in writing of the measures taken.

To determine the causes of damage, it is necessary to draw up an act in which the following must be indicated:

name and batch number of the product;

date of receipt of the product from the manufacturer and the number of the document by which it was received;

detailed description of the damage;

suspected cause of damage, etc.

The manufacturer accepts a complaint if the fault of the recipient in the occurrence of a defect in the product is not established.

15. MANUFACTURER'S WARRANTY

15.1 The manufacturer guarantees the compliance of the product with the requirements given in present document, provided that the consumer observes the conditions and rules for transportation, storage and operation during the entire shelf life. Shelf life - 5 years from the date of manufacture.

ATTACHMENT A

LIST OF INTERNATIONAL REGULATORY DOCUMENTS

Designation of the national (or international) standard (document) in the country of manufacture

Directive 93/42/EEC dated 14 June 1993 Medical devices, facilities and equipment.
Directive 93/42/EEC dated 14 June 1993 concerning medical devices.

EN ISO 13485:2017 Medical devices - Quality management system - Requirements for regulatory purposes.

ISO 14971:2021 Medical devices -- "Application of risk management to medical devices", IDT.

ISO 10993-1:2021 "Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process", IDT.

ISO 15223-1:2020 "Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements", IDT.

ISO 4074:2015 Natural rubber latex male condoms. Requirements and test methods.

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