17, Krylatskaya str., bld. 2, Moscow, 121614
Tel.: (495) 580-77-77 Fax: (495) 580-78-78

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EN

DEVICE DESCRIPTION:

The TECNIS Synergy™ OptiBlue® foldable acrylic 1-piece lens, Model ZFR00V, is an ultraviolet (UV) and violet light-filtering posterior chamber intraocular lens (IOL), which is intended to provide high-quality vision over a full range of distances, including far, intermediate and near to increase spectacle independence. The lens optic is biconvex and consists of a soft acrylic material with UVAM (a supplemental UV-light absorbing material) and proprietary violet light-filtering chromophore which reduces transmittance of violet light wavelengths.

The lens is designed to be positioned in the lens capsule to replace the optical function of the natural crystalline lens. The lens model ZFR00V incorporates a proprietary wavefront-designed aspheric optic with a squared posterior edge that provides a 360-degree barrier. The edge of the optic has a frosted design to reduce potential edge glare effects. The posterior optic of the lens has a proprietary diffractive surface designed to correct chromatic aberration for enhanced contrast sensitivity and provide high-quality vision over a full range of distances. The diffractive design of this lens leads to strong visual acuities across all distances and a continuous range of vision from intermediate through near, resulting in increased spectacle independence. Accommodation will not be restored.

INDICATIONS FOR USE:

The lens model ZFR00V is indicated for primary implantation for the visual correction of: [1] aphakia in adults in whom a cataractous lens has been removed, and [2] aphakia following refractive lensectomy in presbyopic adults who may benefit from useful near vision and reduced spectacle dependence across a range of distances. The lens is intended to be placed in the capsular bag.

Haptics:

1. Material: Soft foldable acrylic with UVAM and proprietary violet light-filtering chromophore
2. One-piece lens
3. Configuration: TRI-FIX design, Modified C, integral with optic
4. Haptic Thickness: 0.46 mm

DIRECTIONS FOR USE:

1. Prior to implanting, examine the lens package for proper lens model, dioptric power, and expiration date.
2. Open the peel pouches and remove the lens in a sterile environment. Verify the dioptric power of the lens.
3. Examine the lens thoroughly to ensure particles have not become attached to it and examine the lens optical surfaces for other defects.
4. The lens may be soaked or rinsed in sterile balanced salt solution until ready for implantation.
5. Handle the lens by the haptic portion. Do not grasp the optical area with forceps.
6. Transfer the lens, using a sterile technique, to an appropriate loading device.
7. The physician should consider the following points:
 - The surgeon should target emmetropia, as this lens is designed for optimum visual performance when emmetropia is achieved.
 - Care should be taken to achieve centration of the intraocular lens.
8. Johnson & Johnson Surgical Vision, Inc. recommends using The UNFOLDER Platinum 1 Series implantation system or an equivalent insertion system or instrument to insert the lens model ZFR00V. Only insertion instruments that have been validated and approved for use with this lens should be used.

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