DURAMEDICAL SDN. BHD.	INSTRUCTION FOR USE	Version	00
		Date	06/2023


APPROVED BY

Director of DURAMEDICAL SDN.
BHD.

(seal/signature)

Mr. Lam Yat Hing

«06» June 2023
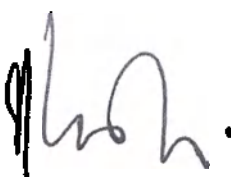
CHOUNG YOUN SIN
NOTARY PUBLIC
KLANG SELANGOR MALAYSIA

03 NOV 2023

INSTRUCTION
FOR USE OF THE MEDICAL DEVICE:
Dental Dam DENTAL FORMULA®

2023

CONTENTS

1 Information on the Medical Device	3
1.1 Name of the Medical Device.....	3
1.2 Manufacturer of the Medical Device	5
1.3 Designation of the Medical Device	5
1.4 Scope of Application of the Medical Device	5
1.5 Expected Users of the Medical Device	5
1.6 Indications the Use of the Medical Device	5
1.7 Contraindications for the Use of the Medical Device	5
1.8. Adverse Effects	5
1.9 Principle of Operation of the Medical Device.....	5
1.10 Instruction for Use (Application) of the Medical Device	5
1.11 Conditions of Use (Application) of the Medical Device	6
1.12 Warnings and Precautions	6
1.13 Shelf Life of the Medical Device	6
2 Conditions for Transportation, Storage and Disposal of the Medical Device.....	6
2.1 Conditions for Transportation of the Medical Device	6
2.2 Conditions for Storage of the Medical Device	6
2.3 Conditions for Disposal of the Medical Device	6
3 Methods of Cleaning and Disinfecting the Medical Device	6
4 Technical Description of the Medical Device.....	6
5 Packaging of the Medical Device	8
6 Scope of Delivery of the Medical Device	9
7 List of National Standards Applied by the Manufacturer of the Medical Device	9
8 Circumstances under which a consumer should consult a healthcare professional.....	9
9 Warranties of the Manufacturer	9
10 Information about the Authorized Representative of the Manufacturer	10

Handwritten signature

1 Information on the Medical Device

1.1 Name of the Medical Device

Dental Dam DENTAL FORMULA®

Designs:

- 1) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: light blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 2) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 3) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: green, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 4) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: black, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 5) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: pink, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 6) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: light blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 7) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 8) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: green, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 9) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: black, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 10) Latex Dental Dam DENTAL FORMULA® thin plain / mint flavor, color: pink, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 11) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: light blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 12) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 13) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: green, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 14) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: black, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 15) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: pink, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 16) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: light blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 17) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 18) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: green, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 19) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: black, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 20) Latex Dental Dam DENTAL FORMULA® medium plain / mint flavor, color: pink, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.

Handwritten signature.

- 21) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: light blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 22) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 23) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: green, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 24) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: black, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 25) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: pink, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 26) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: light blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 27) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 28) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: green, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 29) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: black, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 30) Latex Dental Dam DENTAL FORMULA® heavy plain / mint flavor, color: pink, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 31) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: light blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 32) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: blue, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 33) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: green, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 34) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: black, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 35) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: pink, size: 5" x 5", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 36) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: light blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 37) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: blue, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 38) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: green, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 39) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: black, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.
- 40) Latex Dental Dam DENTAL FORMULA® extra heavy plain / mint flavor, color: pink, size: 6" x 6", in a package of 10/20/30/36/52 pcs., with instruction on the packaging.

The following terms and designations "medical device", "device" and "dental dam" may be used to refer to the *Dental Dam DENTAL FORMULA®* medical device.

Handwritten signature

1.2 Manufacturer of the Medical Device

Manufacturer: DURAMEDICAL SDN. BHD., Malaysia

Manufacturing site: Lot 4962, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang, Selangor Darul Ehsan, Malaysia

1.3 Designation of the Medical Device

The dental dam is intended for one-time use during dental procedures in order to isolate a patient's certain tooth from the rest of the oral cavity and create a dry operational field.

1.4 Scope of Application of the Medical Device

The Dental Dam is used in dental practice.

1.5 Expected Users of the Medical Device

Medical personnel involved in dental procedures.

1.6 Indications the Use of the Medical Device

Indications for the use of the medical device is the need to isolate a patient's certain tooth from the rest of the oral cavity and create a dry operational field.

1.7 Contraindications for the Use of the Medical Device

- Allergy to natural latex;
- Disturbed nose breathing.

1.8. Adverse Effects

- Allergic reaction to natural latex;

1.9 Principle of Operation of the Medical Device

The dental dam is an elastic sheet made of natural latex that should be pierced for placement on patient's tooth during dental procedures.

The principle of the dental dam's operation is to isolate a patient's tooth from the rest of the oral cavity in order to create a dry operational field. In this way, the patient is prevented from swallowing liquids, materials and medicines used, soft tissues are protected from dangerous liquids and the risk of infection is reduced.

1.10 Instruction for Use (Application) of the Medical Device

1. Determine the future location of the hole for a patient's tooth to be treated on the dental dam.
2. Pierce a hole of the required diameter with a special Dental Dam Punch.
3. Put the dental dam on the patient's tooth, secure it with a clamp using forceps.
4. Fix the dental dam on the outside of the mouth with a special frame.
5. Dispose of the used dental dam according to the medical waste disposal rules.

A handwritten signature in black ink, appearing to be "thl".

1.11 Conditions of Use (Application) of the Medical Device

The device is used in dental or preventive medical institutions in an enclosed space in a dry place, away from direct sunlight, at temperature from +10°C to +30°C and relative humidity from 45% to 85%, in accordance with instruction for use of the medical device.

1.12 Warnings and Precautions

- The device is intended for single use;
- For individual use only;
- The medical device is non-sterile and should not be sterilized;
- Do not wet and keep away from moisture;
- Avoid exposure to direct sunlight.
- The device contains natural latex, which may cause allergic reaction.

1.13 Shelf Life of the Medical Device

The established shelf life of dental dam is 3 years.

2 Conditions for Transportation, Storage and Disposal of the Medical Device

2.1 Conditions for Transportation of the Medical Device

The device can be transported by all modes of transport (sea, road, rail or air transport) in accordance with the transportation rules in force for this type of transport. It is not allowed to violate the storage conditions during transportation.

2.2 Conditions for Storage of the Medical Device

Store devices in a dry place protected against direct sunlight, heat sources, ignition, ultraviolet radiation, as well as ozone, at temperatures from 10°C to + 30°C and relative humidity from 45% to 85%.

2.3 Conditions for Disposal of the Medical Device

Any used medical devices shall be disposed of in the Russian Federation by an organization carrying out medical activities as class "B" waste in accordance with SanPiN 2.1.3684-21.

Any unused dental dams that have come into disrepair, or dental dams with expired shelf life, shall be disposed of as "A" class waste in accordance with SanPiN 2.1.3684-21.

3 Methods of Cleaning and Disinfecting the Medical Device


The Dental Dam is a non-sterile single-use medical device and shall not be cleaned. Reuse of the medical device is a gross violation of the instruction for use (application) of the medical device.

4 Technical Description of the Medical Device


Description and dimensions of the medical device. The Dental dam is a latex sheet that has a smooth surface on one side and a matte surface on the other. The smooth side of the device shall face the patient and ensure sliding on the insulated tooth's surface. The matte side shall face the doctor in order to avoid glares from the dental lamp.

Image of the medical device's design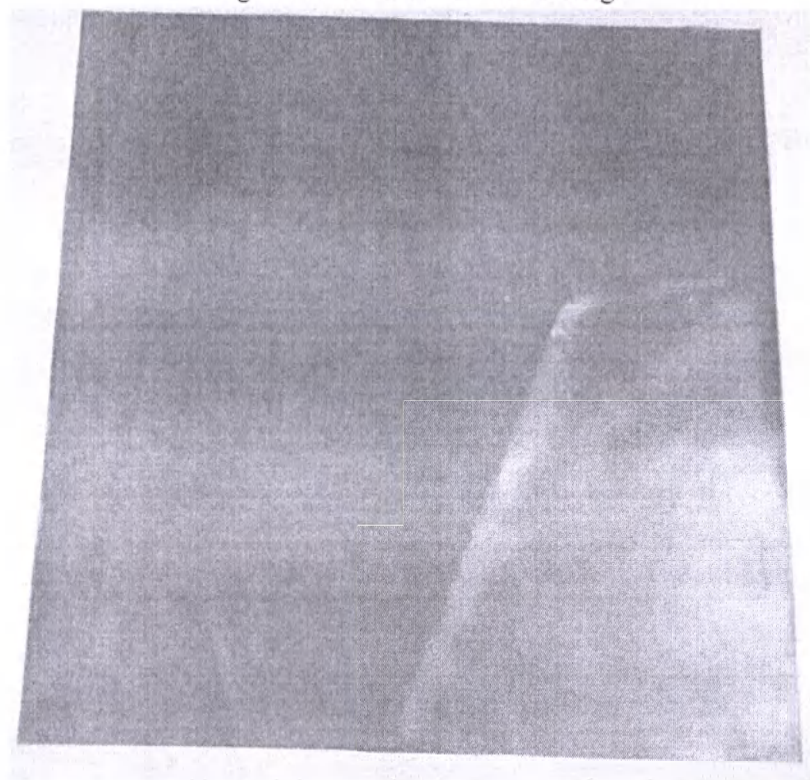

Representation of the medical device's usage on jaw model

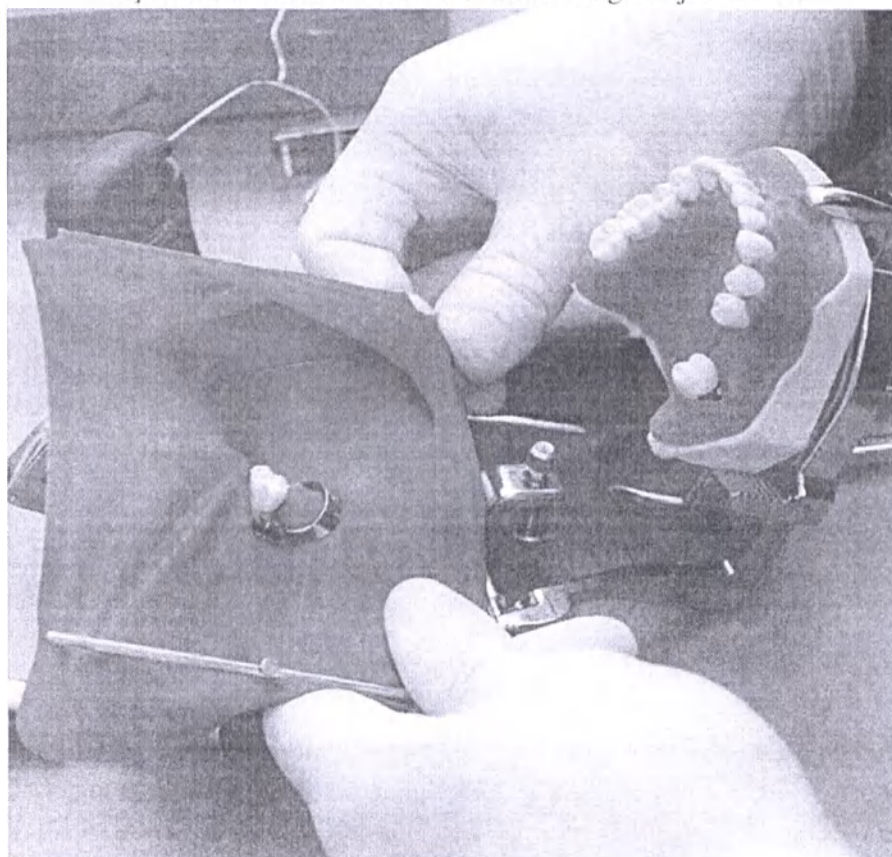Dimensions of the medical device, with regard to its design option, are shown in Table No.1, while Table No.2 shows physical and mechanical indicators.

Table No.1 - Dimensions of the medical device:

DESIGN OPTION	SIZE		THICKNESS
Thin	5" x 5" (127 mm x 127 mm) ± 0,03 mm	6" x 6" (152 mm x 152 mm) ± 0,03 mm	0,14 ± 0,02 mm
Medium			0,18 ± 0,02 mm
Heavy			0,22 ± 0,02 mm
Extra heavy			0,25 ± 0,02 mm

Table No.2 Physical and mechanical indicators of medical device:

CHARACTERISTIC	SPECIFICATION
<u>1. Before aging:</u>	
- Tensile strength, MPa, min	24
- Elongation at break, %, min	700
<u>2. After aging:</u>	
- Tensile strength, MPa, min	19
- Elongation at break, %, min	630

Levels of Control and Acceptable Quality Levels. For quality control purposes, the medical device is sampled according to ISO 2859-1. The control levels and acceptable quality level (AQL) of the dental dams are shown in Table No. 3.

Table No.3 Levels of control and acceptable quality levels (AQL):

Characteristic		Control level	Acceptable quality level (AQL)
Watertightness		G-1	1.5
Dimensions (length, height, width)		S-2	4.0
Physical characteristics	- Tensile strength (before and after aging)	S-2	4.0
	- Elongation at break (before and after aging)		
Visual defects	Major visual defects	G-1	2.5
	Minor visual defects	G-1	4.0

5 Packaging of the Medical Device

The packaging of Dental Dams provides protection against external mechanical damage and climatic factors during transportation and storage.

Consumer packaging. Dental Dams are packed in boxes of lined cardboard of durable material (manufactured by: NINGBO ASIA PULP AND PAPER CO. LTD., China).

Shipping box. Dental Dams in consumer packaging are packaged in cardboard shipping boxes, (manufactured by Muda Packaging Industries Sdn Bhd, Malaysia). The cardboard boxes are sturdy enough to keep devices safe during transportation and storage.

Depending on the scope of delivery, the shipping box contains a certain number of Dental Dams of one or both sizes within one design option. It is not allowed to pack Dental Dams of different designs in a shipping box.

6 Scope of Delivery of the Medical Device

Dental Dams are available in the following options:

- 10 pcs. in a consumer package, with instruction on the packaging.
- 20 pcs. in a consumer package, with instruction on the packaging.
- 30 pcs. in a consumer package, with instruction on the packaging.
- 36 pcs. in a consumer package, with instruction on the packaging.
- 52 pcs. in a consumer package, with instruction on the packaging.

7 List of National Standards Applied by the Manufacturer of the Medical Device

Table No. 5 - List of standards used by the manufacturer:

Designation	Name
ISO 13485	Medical devices. Quality management systems. Requirements for regulatory purposes.
ISO 14971	Medical devices. Application of risk management to medical devices
ISO 2859-1	Statistical methods. Procedures for sampling control by Alternative attribute. Part 1. Sampling control plans Consecutive batches based on an acceptable level of quality
ISO 15223-1	Medical device. Symbols, used in marking of the medical devices, labels and accompanying documents. Part 1. Basic requirements.

8 Circumstances under which a consumer should consult a healthcare professional

If allergic reaction to natural latex occurs, contact a doctor immediately.

9 Warranties of the Manufacturer

The manufacturer guarantees the preservation of all parameters and characteristics of dental dams for 3 years from manufacturing date, provided that the conditions for transportation, storage and use are in accordance with the instructions.

10 Information about the Authorized Representative of the Manufacturer

In case of any questions related to the circulation of the medical device in the Russian Federation, please, contact the Authorized Representative of the manufacturer:

“BE Plus” Company Limited (Be Plus Co. Ltd.), 6th Verkhny per. 12, liter A, part of the office 5.3.1, Sergievskoe municipality, Saint Petersburg, Russia, 194292. +7 (812) 309-36-62, benovydental.ru, beplus@beplus.spb.ru

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