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1. Appearance Description

2. Daily Operating Process


- STEP 1 Preparation
- STEP 2 Startup
- STEP 3 Daily L-J QC
- STEP 4 Sample Preparation
- STEP 5 Sample Analysis
- STEP 6 Result Review
- STEP 7 Shutdown

2.1 Preparation

- Check the waste bucket and make sure it is not full.
- Check the reagents to see if the reagents are expired, frozen, or enough.
- Check the thermal printer and make sure enough printer paper is installed.

2.2 Startup

1. Press power switch at the back of the analyzer.
2. Input **username** and **password**. (Default: Username: admin Password: admin)

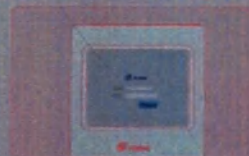2.3 Daily L-J QC

1. Tap the menu option "QC" to "L-J QC" to "Setting".
2. Enter the "L-J QC setting" interface. Tap "New", or select a QC file without QC results.
3. Enter the expiration date of the lot.
4. Select "control level", "control type" and "QC mode" and set "QC sample ID".
5. Enter the target and limits in the edit boxes according to the package insert of the lot of controls.

6. Take out one level of control from the refrigerator and handle it as the control insert.

7. Run control under the "QC" interface and review the "QC graph" or "QC table".

2.4 Sample Preparation

- The analyzer can run 3 types of sample: venous whole blood, capillary whole blood and preformed samples.
- Venous whole blood: collect at least 0.5 mL EDTA-K₂ or EDTA-K₃ whole blood and run it within 15 min to 8 hours after being collected to ensure the accuracy of the results.
- Capillary whole blood: collect at least 100 μ L whole blood and run it within 5 min to 2 hours after being collected to ensure the accuracy of the results.
- Blood samples shall be mixed as shown in the following figure.


Figure 1: Mix the blood sample for at least 3 times.


Figure 2: Mix and mix well (used for capillary whole blood).

Hematology Analyzer Installation Guide

1. Take out the instrument from the cargo box. Remove these screws in the red circle like following picture shows.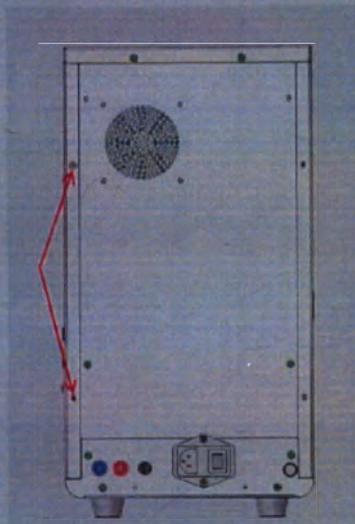

2. Remove right side cover. (Like following picture shows.)

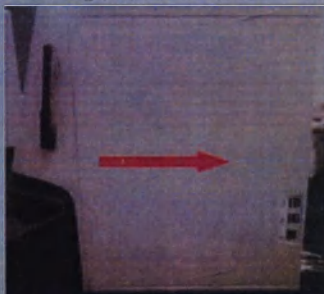3. Cut the fixing belt, which fixes sampling system. (Don't hurt the black motor belt.) Then, remove the shielding paper on chambers shielding box.

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No.	Name	Qty.	Unit	Confirmed Result
1.	Hematology Analyzer	1	PCS	✓
2.	Power Cable	1	PCS	✓
3.	Fuse	2	PCS	✓
4.	Waste Container	1	PCS	✓
5.	Diluent Adapter Tube	1	PCS	✓
6.	Waste Float Adapter Tube	1	PCS	✓
7.	Thermal Printer Paper (57*50 mm)	1	PCS	✓
8.	Maintenance Accessories	1	PCS	✓
9.	Ground Lead	1	PCS	✓
10.	Stylus Pen	1	PCS	✓
11.	RFID Reader	1	PCS	✓
12.	Operation Manual	1	PCS	✓
13.	Operation Guide	1	PCS	✓
14.	Certificate of Quality	1	PCS	✓
15.	Installation Guide	1	PCS	✓
16.	Packing List	1	PCS	✓

Packer: PR001
Check: PR002

QC: 08
Date: 2023-02

P/N: 02-70-08-1008-01[01]

FileName: Packing List ID: S0P400-F61 Version: 01 Effective date: 2022.01.30 Page: 4 / 4

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