orochemie

Managing Director
Rüdiger Eppler

August 13, 2021

TABLE OF CONTENTS

1.	Description	4
2.	Manufacturer	4
3.	Applied normative references	4
4.	Precautions	4
5.	Intended use	5
6.	Technical specifications	5
7.	Manufacturing flow-chart	7
8.	Scope of delivery	7
9.	Labeling	8
10.	Storage and transportation conditions	10
11.	Disposal	10
12.	Shelf-life	10
13.	Responsible supplier	10
	Annex 1	11
	Annex 2	12
	Annex 3	13
	Annex 4	14
	Annex 5	15

Tooth surface polishing and cleaning paste Lunos Polierpaste

Instruction for use

For product registration in the territory of the Russian Federation

AP

5. Intended use

5.1 Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One

1. Take a sufficient amount of the polishing paste and fill it into an application ring, a glass or other suitable vessel.
2. Use a suitable polishing instrument to take up polishing paste and start the polishing process. Please note that the self-reducing effect starts after around approx. 10 seconds.
3. Keep polishing until the polishing particles have broken down into fine particles, so that a smoothing effect can be achieved once the plaque has been removed.
4. Please note that there must always be enough of the polishing paste on the polishing instrument and on the tooth to prevent excessive build-up of heat on the tooth.
5. Please, also make sure that the maximum recommended rpm setting recommended by the polishing instrument manufacturer is not exceeded. Observe the operating instructions provided by the manufacturer.
6. On completing the process wash away the residual paste with water.

5.2 Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft

1. Take a sufficient amount of the polishing paste and fill it into an application ring, a glass or other suitable vessel.
2. Use a suitable polishing instrument to take up polishing paste and start the polishing process.
3. Please note that there must always be enough of the polishing paste on the polishing instrument and on the tooth to prevent excessive build-up of heat on the tooth.
4. Please, also make sure that the maximum recommended rpm setting recommended by the polishing instrument manufacturer is not exceeded. Observe the operating instructions provided by the manufacturer.
5. On completing the process wash away the residual paste with water.

6. Technical specifications

6.1 The device and its components are manufactured from materials presented in table 1.

Table 1

No.	Name	Used material
1	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Mint	see Annex 1
2	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Neutral	see Annex 2
3	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Orange	see Annex 3
4	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Orange	see Annex 4
5	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Neutral	see Annex 5
6	Tube 15 g	Polyethylen, e.g. Hostalen GF 4750
7	Tube 50 g, 100 g	Plastic Barrier Laminate, e.g. 350/1- 7 S plastic

6.2 Paste specification

6.2.1 When being visually inspected, Lunos Polierpaste must be homogeneous and free from foreign particles.

6.2.2 The paste is available in the following flavor variants:

- Lunos Polierpaste Two in One: Neutral, Mint, Orange;

- Lunos Polierpaste Super Soft: Neutral, Orange.

6.2.3 Paste weight must be:

- 15±1.35 g;
- 50±4.5 g;
- 100±4.5 g.

6.2.4 Radioactive dentine abrasion (RDA) must be:

- Lunos Polierpaste Two in One: 28;
- Lunos Polierpaste Super Soft: <5.

6.2.5 Lunos Polierpaste Two in One, Mint contains 1500 ppm of sodium fluoride.

6.2.6 Paste acidity pH must be: 7.5-9.0.

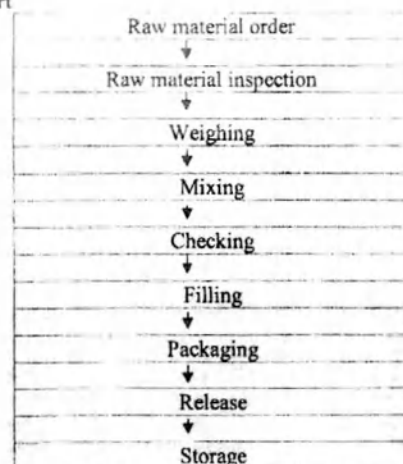
6.2.7 Paste yield point must be: 150-600 Pa.

6.2.8 Tube dimensions must be:

- For 15 g of paste:
 - length: 81±1 mm;
 - width: 31±1 mm;
 - cap diameter: 18±1 mm;
- For 50 g of paste:
 - length: 122±1 mm;
 - width: 48±1 mm;
 - cap diameter: 28±1 mm;
- For 100 g of paste:
 - length: 143±1 mm;
 - width: 55±1 mm;
 - cap diameter: 33±1 mm.

6.3 Presentation: Paste (Tube) 15 g; 50 g; 100 g.

7. Manufacturing flow-chart

Flow-chart description

- 1. Raw material order:** purchase of raw materials depending on the specification.
- 2. Raw material inspection:** incoming inspection of certified raw materials and auxiliary materials.
- 3. Weighing and mixing:** control amount of excipients and mix according to manufacturing description.
- 4. Checking:** Examination before release for filling according specification
- 5. Filling:** powder is filled into bottles in accordance with the order.
- 6. Packaging:** Packaging of bottles into folding carton. Product remain in quarantine until released.
- 7. Release:** Release for market according to specification.
- 8. Storage:** After final release the product it is stored in the ware house.

8. Scope of delivery

The device scope of delivery

No.	Name	Quantity, pcs.
1	Tooth surface polishing and cleaning paste Lunos Polierpaste, hardware versions:	
1	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Mint (weight 15 g)	1
2	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Neutral (weight 15 g)	1
3	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Orange (weight 15 g)	1
4	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Mint (weight 100 g)	1
5	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Neutral (weight 100 g)	1
6	Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Orange (weight 100 g)	1
7	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Orange (weight 15 g)	1

No.	Name	Quantity, pcs.
8	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Neutral (weight 15 g)	1
9	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Orange (weight 50 g)	1
10	Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Neutral (weight 50 g)	1
II	Instruction for use	1

9. Labeling

9.1 Paste labeling according to GOST R ISO 15223-1.

- Lot number:
- Medical Product
- CE marking
- "Expiry date..."
- Manufacturer
- Indication of Storage temperature
- Attention symbol
- Refer to accompanying documents symbol

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Sticker layout of the 15 g package

10. Storage and transportation conditions

The device is to be transported by all kinds of transport in covered vehicles, in accordance with the shipping rules applicable for such transport.

With respect to climatic exposure the following transportation conditions are to be met:

Transportation temperature from 1 to +30 °C.

Store in a dry place protected from light at temperature from 0 to +25 °C;

Avoid exposure to high temperatures and direct sunlight, sparks and open flame.

DO NOT EXCEED the expiry date indicated on the package.

11. Disposal

The device must be collected and handed to specialized licensed services disposing this type device. Disposal must be carried out in accordance with norms and regulations of the legislation currently in force. According to SanPiN 2.1.3.684-21 the device belongs to Class A, which means epidemiologically safe waste.

12. Shelf-life

12.1 Guaranteed shelf life at temperature from 0 to +25°C is 30 months from the date manufacture.

13. Responsible supplier

Limited Liability Company "Hariko Denta Med" ("Hariko Denta Med" LLC), 115230 Moscow, Kashirskoe шоссе, housing 1. Tel: +7 (495) 913-66-06, E-mail: info@hariko.ru

Annex 1

Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Mint

Ingredient	Function	CAS No.	Content
Glycerol	Surface-active agent	56-81-5	10-30%
2-phenoxyethanol	Preservative	122-99-6	<0.8%
Mixture of hydroxybenzoates	Preservative	99-76-3	<0.4%
Sodium fluoride	Fluorinating agent	7681-49-4	<0.2%
Carboxymethylcellulose sodium	Thickening agent	9004-32-4	<0.5%
Xanthan gum	Thickening agent	11138-66-2	<0.4%
Sodium olefin sulfonate C14-16	Emulsifying agent	68439-57-6	<0.2%
Tricalcium phosphate	Gloss agent	12167-74-7	1-10%
Feldspar	Abrasive	68476-25-5	30-40%
Wollastonite	Abrasive	13983-17-0	10-20%
Citric acid	Solution acidity regulator	5949-29-1	<1.5%
CI 42090 / CI 47005	Coloring agent	-	<1%
CI 77891	Coloring agent	-	<1%
Peppermint oil	Flavoring agent	8006-90-4	<0.1%
Water	Solvent	7732-18-5	add. 100%

Annex 2

Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Neutral

Ingredient	Function	CAS No.	Content
Glycerol	Surface-active agent	56-81-5	10-30%
2-phenoxyethanol	Preservative	122-99-6	<0.8%
Mixture of hydroxybenzoates	Preservative	99-76-3	<0.4%
Carboxymethylcellulose sodium	Thickening agent	9004-32-4	<0.5%
Xanthan gum	Thickening agent	11138-66-2	<0.4%
Sodium olefin sulfonate C14-16	Emulsifying agent	68439-57-6	<0.2%
Tricalcium phosphate	Gloss agent	12167-74-7	1-10%
Feldspar	Abrasive	68476-25-5	30-40%
Wollastonite	Abrasive	13983-17-0	10-20%
Citric acid	Solution acidity regulator	5949-29-1	<1.5%
CI 77891	Coloring agent	-	<0.5%
Water	Solvent	7732-18-5	add. 100%

K 11

Annex 3

Tooth surface polishing and cleaning paste Lunos Polierpaste Two in One, Orange

Ingredient	Function	CAS No.	Content
Glycerol	Surface-active agent	56-81-5	10-30%
2-phenoxyethanol	Preservative	122-99-6	<0.8%
Mixture of hydroxybenzoates	Preservative	99-76-3	<0.4%
Carboxymethylcellulose sodium	Thickening agent	9004-32-4	<0.5%
Xanthan gum	Thickening agent	11138-66-2	<0.4%
Sodium olefin sulfonate C14-16	Emulsifying agent	68439-57-6	<0.2%
Tricalcium phosphate	Gloss agent	12167-74-7	1-10%
Feldspar	Abrasive	68476-25-5	30-40%
Wollastonite	Abrasive	13983-17-0	10-20%
Citric acid	Solution acidity regulator	5949-29-1	<1.5%
CI 47005	Coloring agent	-	<1%
CI 77891	Coloring agent	-	<1%
Orange flavoring	Flavoring agent	-	<0.2%
Water	Solvent	7732-18-5	add. 100%

Annex 4

Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Orange

Ingredient	Function	CAS No.	Content
Glycerol	Surface-active agent	56-81-5	30-40%
2-phenoxyethanol	Preservative	122-99-6	<0.8%
Mixture of hydroxybenzoates	Preservative	99-76-3	<0.4%
Carboxymethylcellulose sodium	Thickening agent	9004-32-4	1-2 %
Sodium olefin sulfonate C14-16	Emulsifying agent	68439-57-6	<0.5%
Tricalcium phosphate	Gloss agent	12167-74-7	35-40%
CI 15985/CI 16255	Coloring agent	-	<2%
Orange flavoring	Flavoring agent	-	<0.2%
Water	Solvent	7732-18-5	add. 100%

Annex 5

Tooth surface polishing and cleaning paste Lunos Polierpaste Super Soft, Neutral

Ingredient	Function	CAS No.	Content
Glycerol	Surface-active agent	56-81-5	30-40%
2-phenoxyethanol	Preservative	122-99-6	<0.8%
Mixture of hydroxybenzoates	Preservative	99-76-3	<0.4%
Carboxymethylcellulose sodium	Thickening agent	9004-32-4	1-2 %
Sodium olefin sulfonate C14-16	Emulsifying agent	68439-57-6	<0.5%
Tricalcium phosphate	Gloss agent	12167-74-7	35-40%
Water	Solvent	7732-18-5	add. 100%

Urkundenrolle UR O 6092 / 2021

UZ 11334/2021

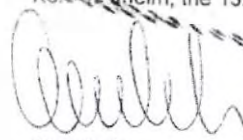
Certificate verifying signature

I hereby certify that this document was signed in my presence by

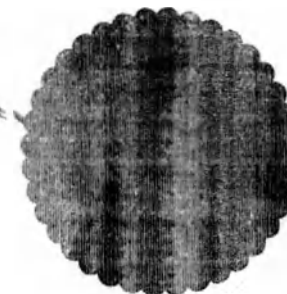
Mr. Rüdiger Fritz Eppler,
born on 31.03.1958,
business address 70806 Kornwestheim, Max-Planck-Straße 27,

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