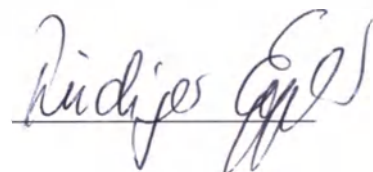
20

orochemie

Managing Director
Rüdiger Eppler

April 29, 2022

Tooth polishing powder Lunos ProphylaxePulver

Instruction for use

For product registration in the territory of the Russian Federation

TABLE OF CONTENTS

1.	Description	4
2.	Manufacturer	4
3.	Applied normative references	4
4.	Precautions	4
5.	Intended use	5
6.	Technical specifications	7
7.	Manufacturing flow-chart	7
8.	Scope of delivery	8
9.	Labeling	8
10.	Storage and transportation conditions	9
11.	Disposal	10
12.	Shelf-life	10
13.	Responsible supplier	10
	Annex 1	11
	Annex 2	12
	Annex 3	13
	Annex 4	14

No.	Change history	Approved by:	Date
0	First issue	Rüdiger Eppler	May 19, 2021
1	Addition of MD symbol	Rüdiger Eppler	August 13, 2021

RP

1. Description

Tooth polishing powder Lunos ProphylaxePulver (hereinafter referred to as powder, device) is intended for removal of mucous plaque, pigment, fissure cleaning (supra- and sub-gingival application), preparation of teeth for bonding application, cleaning of orthodontic fixtures (rings, brackets), removal of residual adhesive, cleaning of implants before installation using the air abrasion method.

Application field – for professional use in dentistry.

Indications for use:

- supra-gingival tooth cleaning;
- sub-gingival tooth cleaning (Lunos ProphylaxePulver Perio Combi).

Contraindications:

- airway diseases (e.g. bronchitis, asthma);
- hypomineralised tooth enamel;
- spots on enamel;
- hypersensitivity to one of the material ingredients
- Lunos Prophylaxe Pulver Perio Combi: the treatment of deep periodontal pockets can lead to bacteraemia. For this reason, please take into consideration that necessary restrictions or suitable measures must be undertaken for the treatment of risk patients (endocarditis, pregnant and lactating women, infectious diseases, immunodeficiency (neutropenia, agranulocytosis, diabetes, haemophilia), patients on treatment (radiation therapy, chemotherapy, antibiotic therapy)).

Side effects: not known

According to the potential risk of application, the device belongs to Class I (regulation (EU) 2017/745)

2. Manufacturer

orochemie GmbH + Co. KG

Address: Max-Planck-Straße 27, D-70806 Kornwestheim, GERMANY

Tel.: +49 71 54 13 08-0 Fax: +049 71 54 13 08-0 www.orochemie.de

3. Applied normative references

Applied standard	Standard name
REGULATION (EU) 2017/745	Medical device regulation
EN ISO 14971	Medical devices - application of risk management to medical devices
EN ISO 780	Packaging. Distribution packaging. Graphical symbols for handling and storage of packages
EN ISO 15322-1	Graphical symbols for use in the labelling of medical devices.
EN 1041	Information supplied by the manufacturer of medical devices

4. Precautions

1. Patient and physician need to wear suitable eye protection, e.g. protective goggles. The physician needs to wear suitable protective equipment in the form of mouth and nose protection and suitable protective gloves.
2. Use a suitable aspiration cannula.
3. Do not direct the jet at the gingiva during the treatment. Direct exposure of the soft tissue may damage the gingiva. Moreover, there is a risk of emphysema arising in the tissue and/or of embolism from air being introduced into blood circulation.

4. To protect sensitive skin, please apply Vaseline to the patient's lips before commencing the treatment.
5. After completion of the treatment, clean the patient's face with a moist disposable cloth (not included in the scope of delivery).
6. The powder cannot be used to remove tartar and calculus.
7. Do not use an air-powder abrasive device without water because it will harden air-powder mixture absorption and can damage the treated area.
6. DO NOT USE the device if the protective seals on the package are broken or their authenticity is doubtful.
7. If the product cause an allergy or hypersensitivity to the patient, it is necessary to stop using the product immediately.
8. Do not exceed the expiry date.
9. Please make sure not to direct the jet of the powder jet device directly at fixed dental restorations and fillings as this may damage the restorations.

5. Intended use

Tooth polishing powder Lunos ProphylaxePulver Perio Combi is recommended to be used in the following cases:

- Removal of periodontal biofilm in the scope of periodontal maintenance therapy (recall therapy) after completed initial therapy;
- Implant cleaning and treatment of peri-implantitis after completed initial therapy;
- Gentle removal of plaque at the sulcus and sensitive tooth surfaces;
- Removal of slight discolorations in supragingival region;
- Removal of plaque on orthodontic appliances (brackets).

Tooth polishing powder Lunos ProphylaxePulver Gentle Clean is recommended to be used in the following cases:

- Supragingival cleaning of the tooth surface:
 - Removal of discoloration and plaque;
- Preparation and cleaning of the tooth surface before:
 - Adhesive attachment of orthodontic brackets;
 - Application of fluoride and bleaching preparations;
 - Insertion of composite restorations, fixed dentures, e.g. inlays, onlays, crowns, veneers etc.
 - Application of sealing materials.

5.1 Filling the powder

5.1.1 Please remove all residual powder from the powder chamber of the powder jet device before refilling it with Prophylaxis Powder. Observe Operational Instructions of the powder jet device.

- Do not clean the device with water or moist air since the moist powder can block the spray nozzle.

5.1.2 Shake the closed bottle before filling the powder into the powder chamber.

5.1.3 Please remove the seal on the bottle before filling the powder. For this purpose, take the lid off the bottle, pull off the seal and then screw the lid back on the bottle.

5.1.4 Before filling the dental powder chamber, make sure that the chamber is completely dry. There must be no residual moisture in the chamber, e.g. from preparatory measures.

5.1.5 Filling the powder into the powder chamber, make sure that the maximum filling level of the powder chamber is not exceeded.

5.1.6 Fold the "rocker" all the way up or unscrew the lid from the bottle. To minimize the amount of dust generated when you fill the chamber, it is recommended to use the "rocker" for this purpose and to slightly compress the bottle.

5.2 Setting the device for air-powder abrasive polishing and cleaning.

The prophylaxis powders are used in a suitable powder jet handpiece.

Efficacy of the cleaning with the prophylaxis powder depends on several factors including an air-powder abrasive device being utilized and preset values for compressed air and water flow.

5.2.1 Follow the guidelines given in the device operating instructions and test the settings to find optimal settings prior to start the treatment. Regarding the settings for the air-powder abrasive device or the dental chair consult the respective manufacturer.

5.2.2 Direct the spray nozzle, for example, into the wet sink and spray the water-powder mixture for 10 sec. until it becomes homogenous. Adjust the water flow and air pressure according to the requirements.

5.3 Patient preparation

5.3.1 Apply vaseline to the corners of the patient's lips to prevent these areas from dehydration and cracking.

5.3.2 Place the aspiration cannula in the corner of the patient's mouth so that the area under the tongue remains dry.

5.3.3 Use a dental aspirator to suck the tooth-reflected mixture of water and powder.

5.4 Patient's tooth processing

5.4.1 Supra-gingival application

5.4.1.1 Hold the spray nozzle near the tooth surface at the distance of 3-5 mm and at an angle of 30° to 60° to the tooth surface; process each side of the tooth for max. 5 seconds constantly moving the nozzle in circular motions.

5.4.2 Sub-gingival application (for Perio Combi powder only)

5.4.2.1 Hold the spray nozzle near the gingival margin at the distance of 3-5 mm and at an angle of 30° to 60° to the tooth surface. Process the tooth along the gingival margin moving smoothly so that the powder cleans the sub-gingival area. The smaller the angle, the deeper the powder jet will enter the gingival sulcus and the less the jet of water-powder mixture will be refracted.

5.4.2.2 Avoid directing the jet at the same place for a long time. Move along the surface to provide the homogeneous cleaning.

5.4.3 During the processing hold the dental aspirator near the tooth to be treated.

5.4.4 Please note the operating instructions of the powder jet device before use of the prophylaxis powder. The distance from the tooth surface, the angle to the tooth and the duration of use must be set individually depending on the powder jet device and the needs and safety of the patient.

5.5 Device preparation for the further treatment

5.5.1 At the end of the day clean the powder chamber of the air-powder abrasive device and remove residual powder with dry air. Then close the powder chamber again and turn on for approx. 5 seconds to remove all residual powder inside the device.

5.5.2 In case of using different powders, before each powder change it is necessary to empty the powder chamber and to blow it out with dry air to remove all residual powder. Avoid the contact of the prophylaxis powder and moisture; the moist powder can block the spray nozzle.

5.5.3 Clean the device in accordance with the information given in the operating instructions.

5.6 Notes for the patient

Please advise your patient that the ingestion of food or beverages (tea, coffee or other food items) within the first two to three hours after completion of the treatment can cause discoloration of the teeth.

6. Technical specifications

6.1 The device and its components are manufactured from materials presented in table 1.
Table 1

No.	Name	Used material
1	Tooth polishing powder Lunos ProphylaxePulver Perio Combi, Neutral	see Annex 1
2	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Neutral	see Annex 2
3	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Orange	see Annex 3
4	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Spearmint	see Annex 4
5	Bottle	Polyethylen, e.g. TIPELIN 6000B

6.2 Powder specification

6.2.1 When being visually inspected, Tooth polishing powder Lunos ProphylaxePulver must be homogeneous and free from foreign particles.

6.2.2 The powder is available in the following flavor variants:

- Lunos ProphylaxePulver Gentle Clean: Neutral, Spearmint, Orange;
- Lunos ProphylaxePulver Perio Combi: Neutral.

6.2.3 Powder weight must be:

- 20 ± 1.8 g;
- 100 ± 4.5 g;
- 180 ± 4.5 g.

6.2.4 Particle size must be:

- Lunos ProphylaxePulver Gentle Clean D 50: 50-75 μ m;
- Lunos ProphylaxePulver Perio Combi D 50: 20-35 μ m.

6.2.5 Powder moisture must be max. 11.5%.

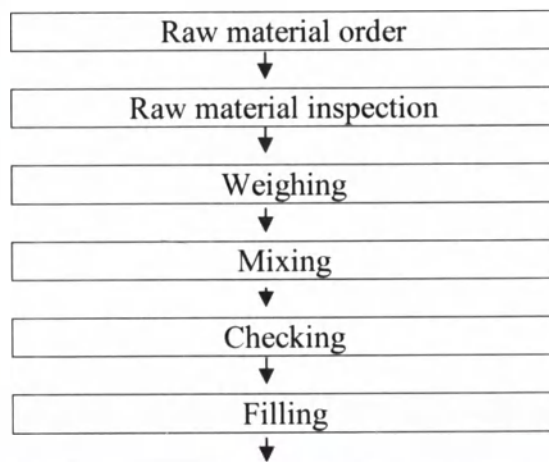
6.2.6 Powder acidity, pH must be 4.5-6.6.

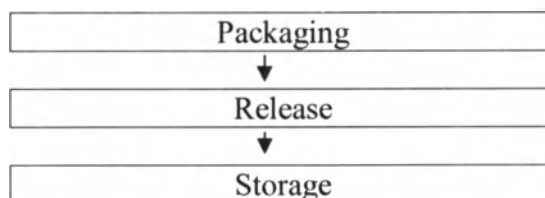
6.2.6 Bottle dimensions must be:

- For 20 g of powder:
 - length: 85 ± 2 mm;
 - width: 35 ± 2 mm;
 - cap diameter: 24 ± 2 mm;
- For 100 g and 180 g of paste:
 - length: 131 ± 2 mm;
 - width: 59 ± 2 mm;
 - cap diameter: 24 ± 2 mm.

6.3 Presentation: Powder (Bottle) 20 g; 100 g; 180 g.

7. Manufacturing flow-chart

Flow-chart description

- 1. Raw material order:** purchase of raw materials depending on the specification.
- 2. Raw material inspection:** incoming inspection of certified raw materials and auxiliary materials.
- 3. Weighing and mixing:** control amount of excipients and mix according to manufacturing description.
- 4. Checking:** Examination before release for filling according specification
- 5. Filling:** powder is filled into bottles in accordance with the order.
- 6. Packaging:** Packaging of bottles into folding carton. Product remain in quarantine until released.
- 7. Release:** Release for market according to specification.
- 8. Storage:** After final release the product it is stored in the ware house.

8. Scope of delivery

The device scope of delivery

No.	Name	Quantity, pcs.
I	Tooth polishing powder Lunos ProphylaxePulver, hardware versions:	
1	Tooth polishing powder Lunos ProphylaxePulver Perio Combi, Neutral (weight 20 g)	1
2	Tooth polishing powder Lunos ProphylaxePulver Perio Combi, Neutral (weight 100 g)	4 pcs./ pkg
3	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Neutral (weight 20 g)	1
4	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Spearmint (weight 20 g)	1
5	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Neutral (weight 180 g)	4 pcs./ pkg
6	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Orange (weight 180 g)	4 pcs./ pkg
7	Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Spearmint (weight 180 g)	4 pcs./ pkg
II	Instruction for use	1

9. Labeling

9.1 Dental set labeling according to GOST R ISO 15223-1.

- Lot number:
- Medical Product
- CE marking
- "Expiry date..."

LOT

MD

CE

- Manufacturer
- Indication of Storage temperature
- Indication of Humidity
- Protect from moisture
- Shake before use
- Attention symbol

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Sticker layout of the 20 g bottle

10. Storage and transportation conditions

The device is to be transported by all kinds of transport in covered vehicles, in accordance with the shipping rules applicable for such transport.

With respect to climatic exposure the following transportation conditions are to be met:

Transportation temperature from 1 to +30 °C.

Store in a dry place protected from light at temperature from 0 to +25 °C;

Air humidity from 10% to 75%.

Avoid exposure to high temperatures and direct sunlight, sparks and open flame.

DO NOT EXCEED the expiry date indicated on the package.

11. Disposal

The device must be collected and handed to specialized licensed services disposing this type of device. Disposal must be carried out in accordance with norms and regulations of the legislation currently in force.

According to SanPiN 2.1.3684-21 the device belongs to Class A, which means epidemiologically safe waste.

12. Shelf-life

12.1 Guaranteed shelf life of the device at temperature from 0 to +25°C is 30 months from the date of manufacture.

13. Responsible supplier

Limited Liability Company "Hariko Denta Med" ("Hariko Denta Med" LLC), 115230 Moscow, 5, Kashirskoe shosse, housing 1. Tel: +7 (495) 913-66-06, E-mail: info@hariko.ru

Annex 1

Tooth polishing powder Lunos ProphylaxePulver Perio Combi, Neutral

Ingredient	Function	CAS No.	Content
Trehalose	Abrasive	6138-23-4	95-100%
Colloidal hydrophobic silica	Anticaking agent	68611-44-9	<1%

PC

Annex 2

Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Neutral

Ingredient	Function	CAS No.	Content
Trehalose	Abrasive	6138-23-4	95-100%
Colloidal hydrophobic silica	Anticaking agent	68611-44-9	<1%

Handwritten signature

Annex 3

Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Orange

Ingredient	Function	CAS No.	Content
Trehalose	Abrasive	6138-23-4	95-100%
Colloidal hydrophobic silica	Anticaking agent	68611-44-9	<1%
Orange flavoring	Flavoring agent	-	<0.6%

Annex 4

Tooth polishing powder Lunos ProphylaxePulver Gentle Clean, Spearmint

Ingredient	Function	CAS No.	Content
Trehalose	Abrasive	6138-23-4	95-100%
Colloidal hydrophobic silica	Anticaking agent	68611-44-9	<1%
Spearmint flavoring	Flavoring agent	-	<0.6%

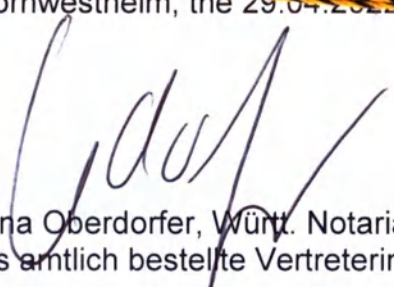
Certificate verifying signature

I hereby certify that this document was signed in my presence by

Mr. Rüdiger Fritz Eppler,
born on 31.03.1958,
business address 70806 Kornwestheim, Max-Planck-Straße 27,

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