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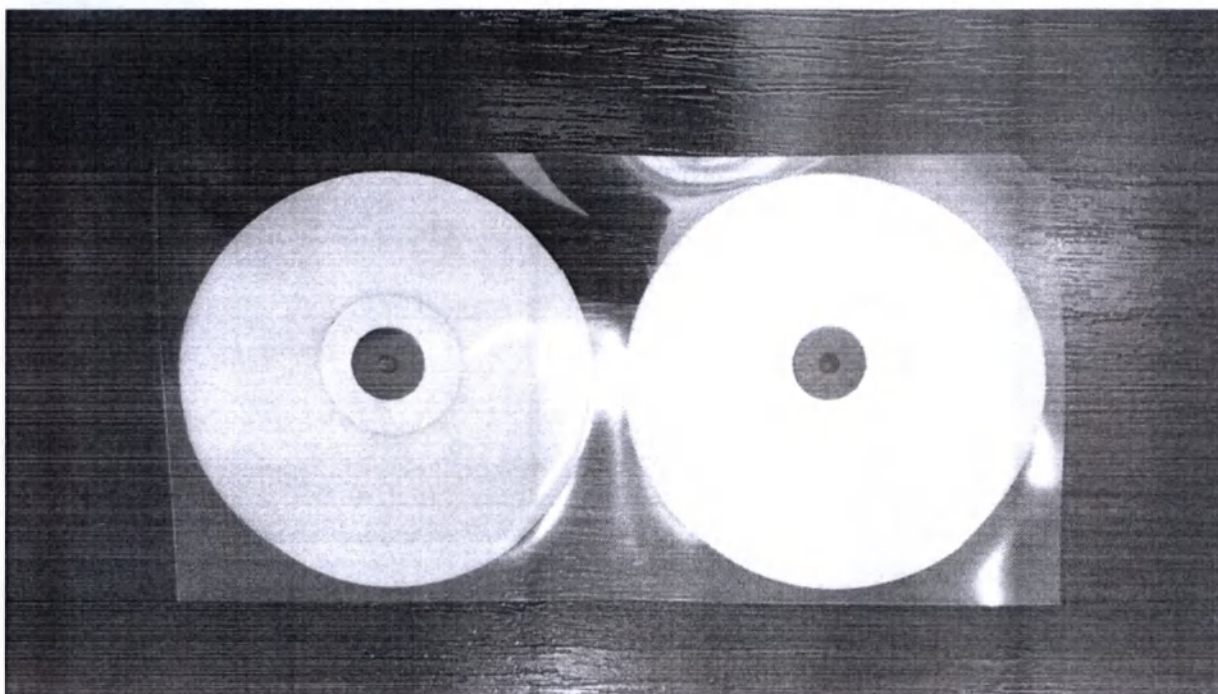
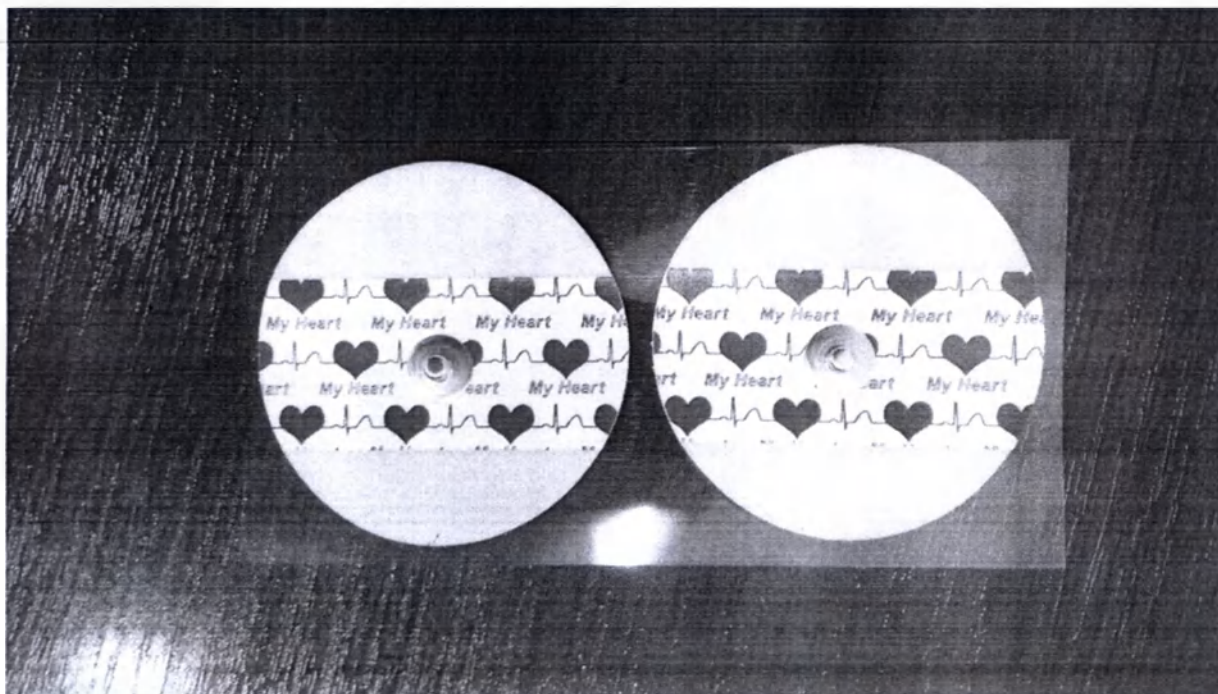
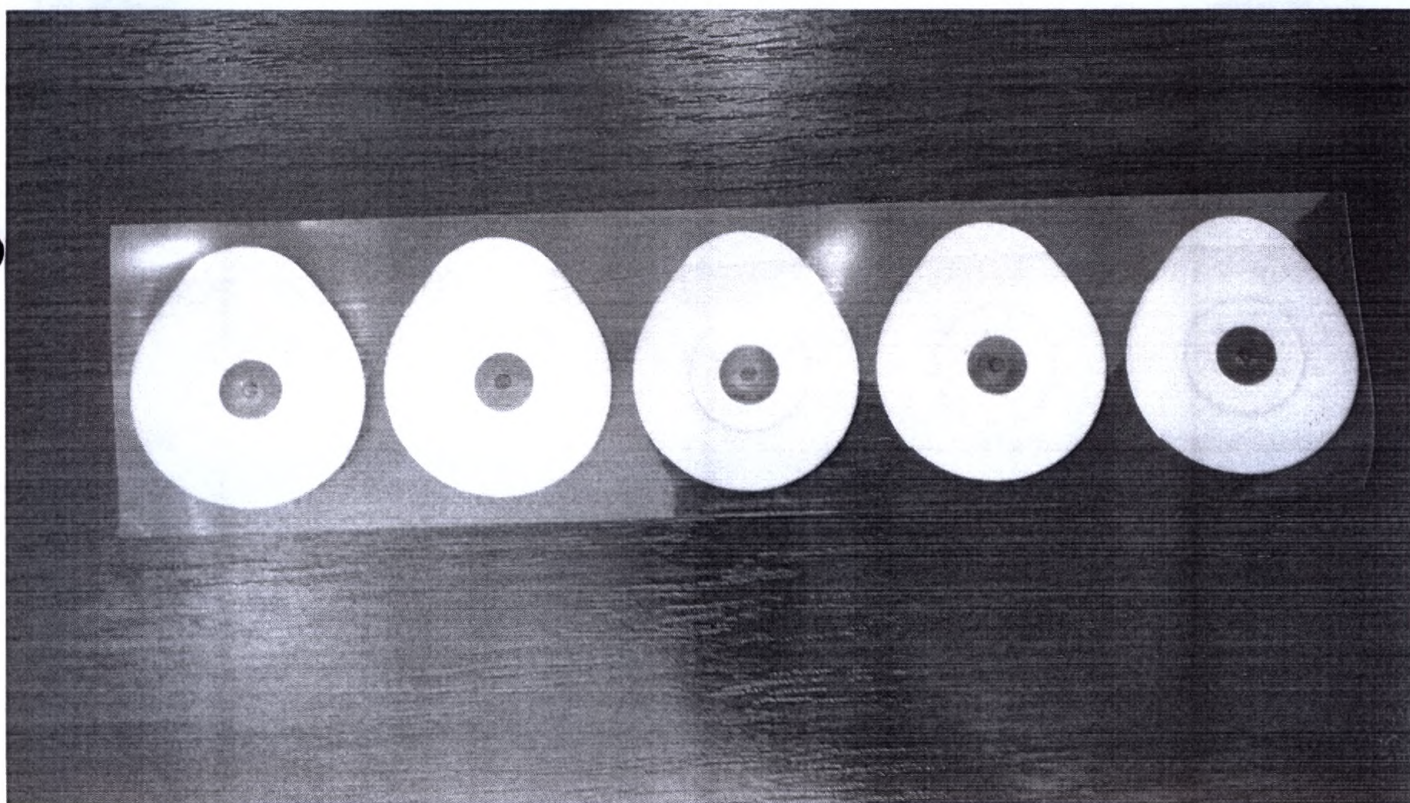
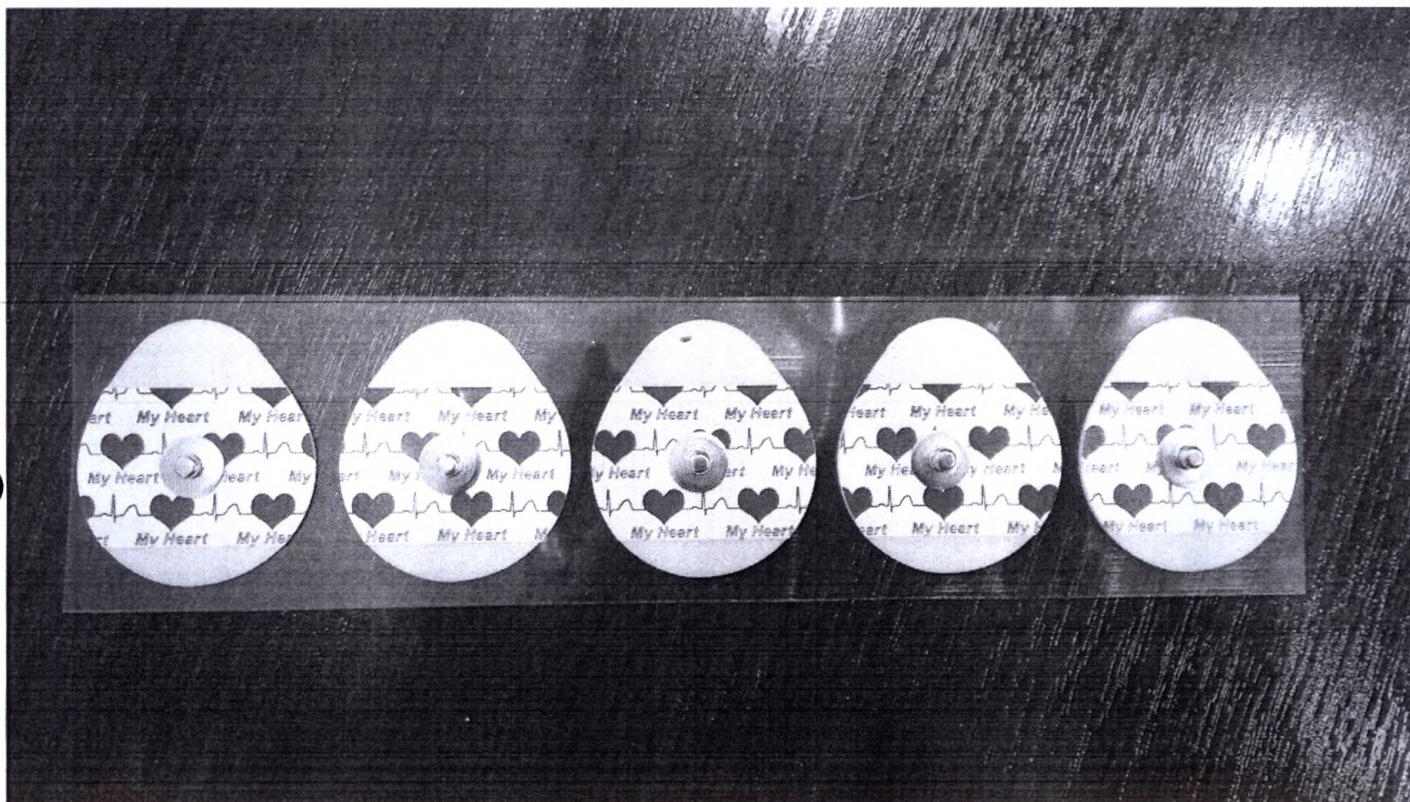
China Qingdao Bright Medical Manufacturing Co., Ltd

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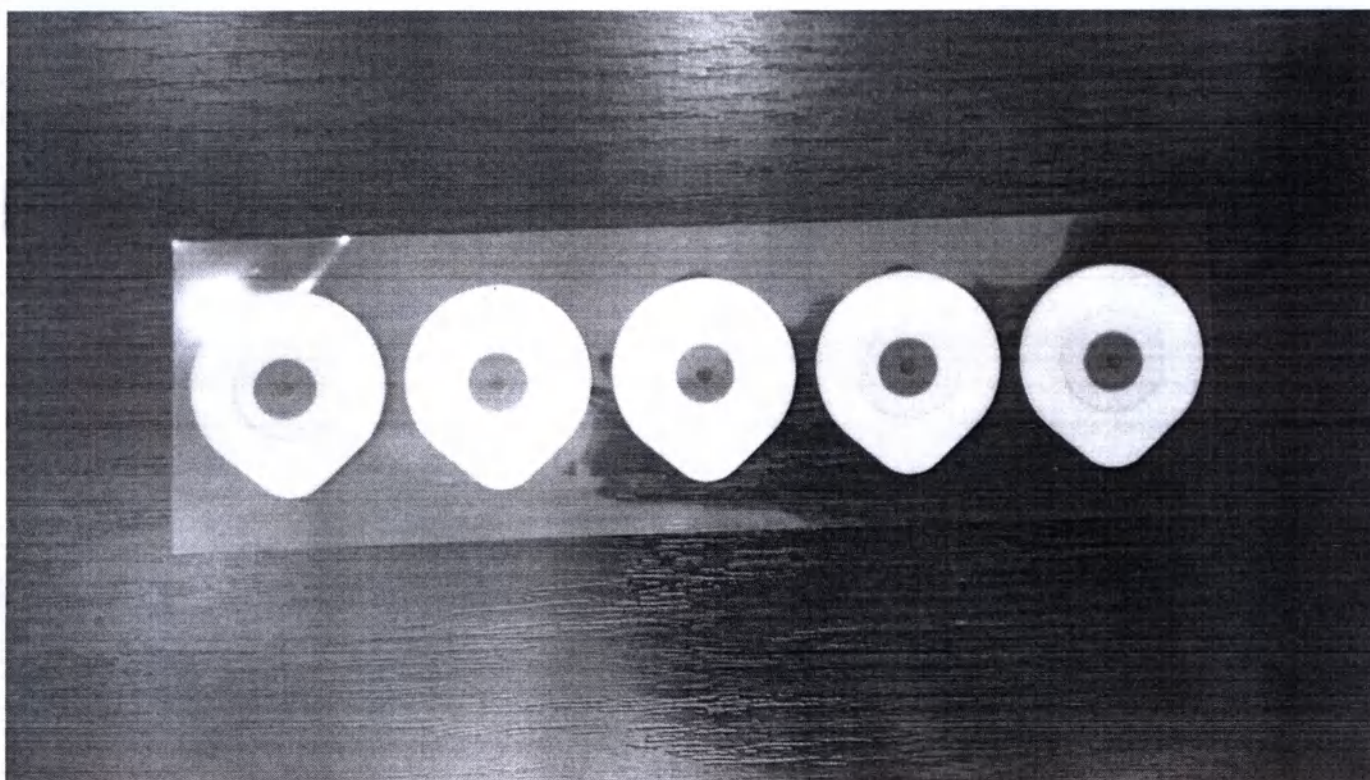
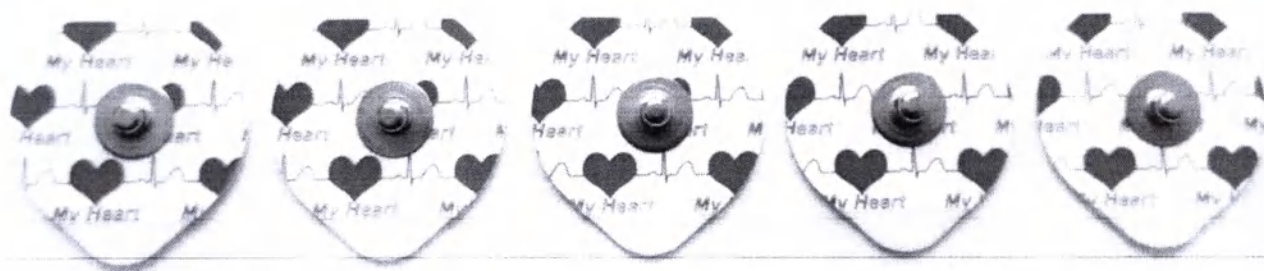
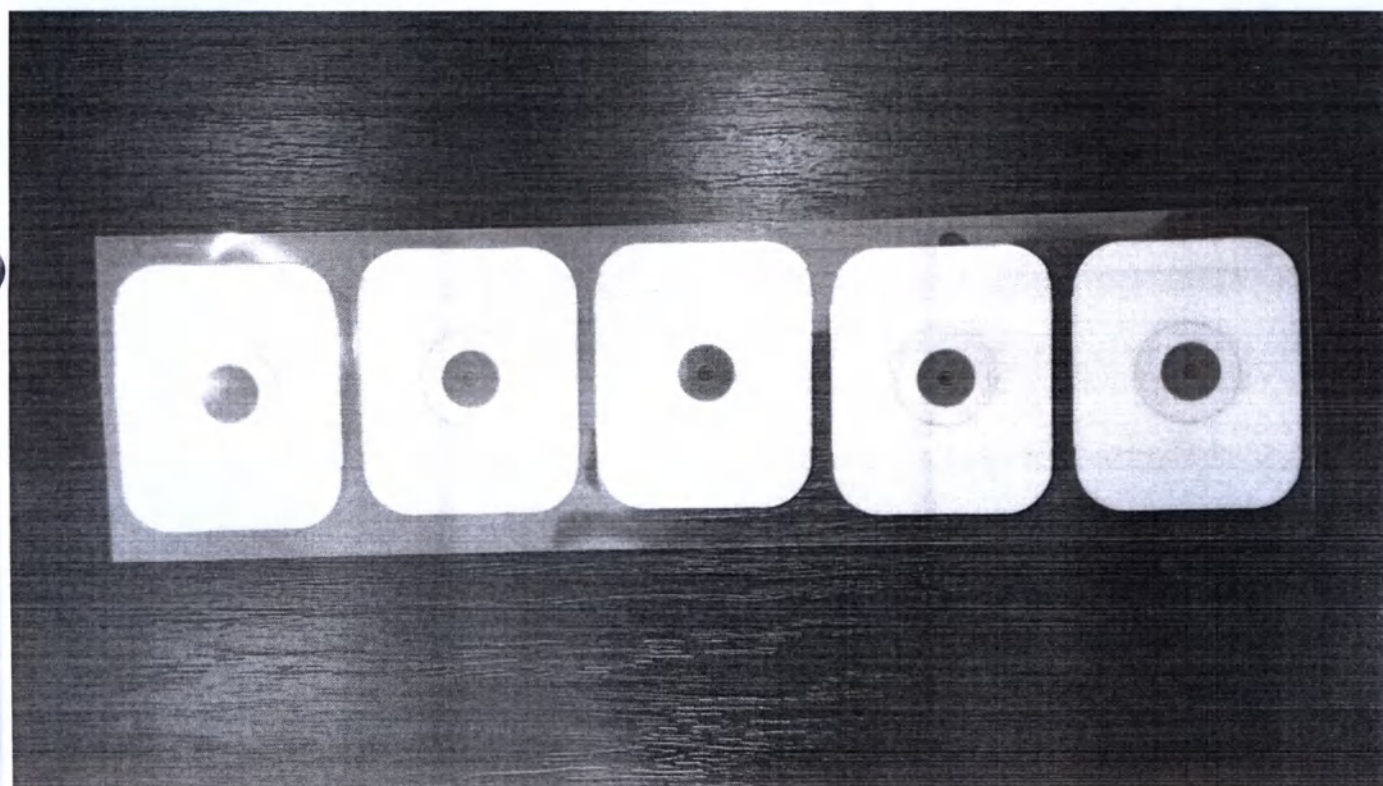
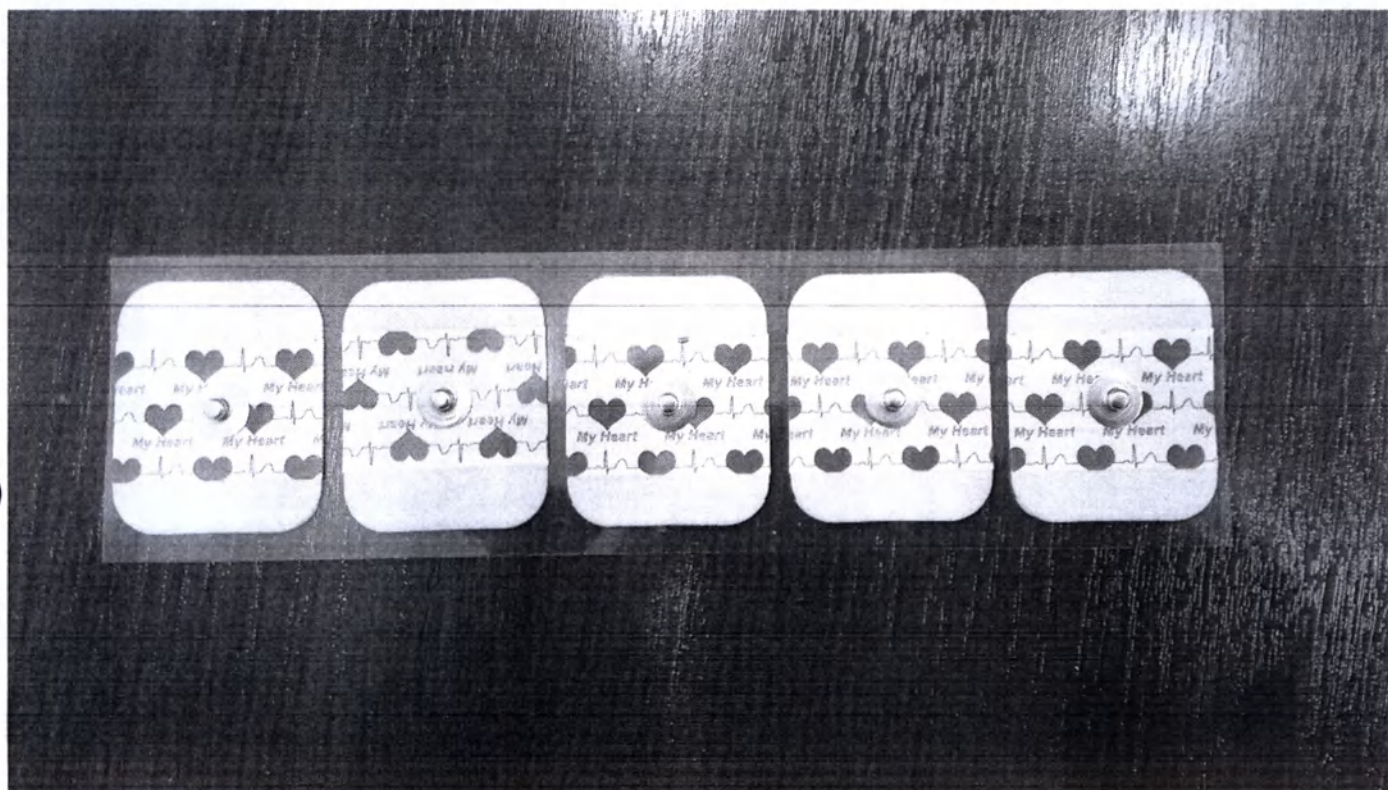
Application

- ① Determine application sites. Remove any excessive hair, calluses and ensure the skin is clear of any oils, creams, fluids, dirt or other impediments to good electric contact. If necessary, clean the skin with appropriate preparation material. Always ensure the application site is dry prior to application of electrodes.
- ② Connect the lead wires to the electrodes.
- ③ Remove the release liner and position the electrodes on the skin. Smooth the backing surface with your finger as necessary.


Instructions for use

- ① The product is not for internal use. In case of accidental injection of the electrode consult a physician immediately. Not suitable for use on newborn or premature infants.
- ② Due to high adhesion strength of the adhesive material, this product is not suitable for use on patients with fragile skin.
- ③ Use only in combination with approved patient cables and adapters. Never allow contact with an external electrical source.

DO NOT USE IF THE GEL IS DRY OR DISCOLORED

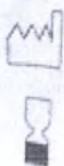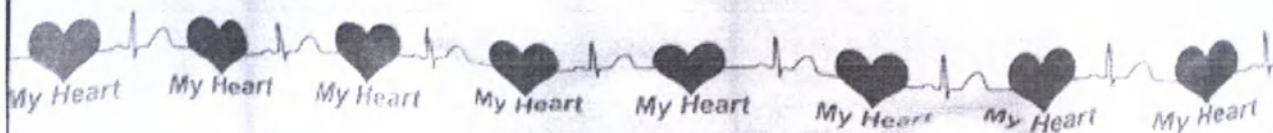
SUGGEST TO USE OUT WITHIN 15 DAYS AFTER OPENING

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