certified copy

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Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

Tel.: +49 (0)8152 700-0
Fax: +49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To whom it may concern

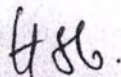
October 30, 2019

Registration documents for Imprint 4 and Intraoral Syringe - Green - IFU

We, 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that following pages are an excerpt of the Information for Use.

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3M ESPE

Instructions for use

Imprint™ 4 Vinylpolysiloxane impression materials

2019

3M ESPE

Imprint™ 4

VPS Impression Material

Instructions for Use

Product Description

Imprint™ 4 impression material is a new generation of addition type silicones with improved hydrophilicity and peppermint flavor. Imprint™ 4 products provide very quick intraoral setting times. They are used for one-step and two-step impression techniques. The Imprint™ 4 portfolio consists of the following products:

Super Quick Setting Products:	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Imprint™ 4 Penta	Type 1 heavy-bodied	yellow	5:1	all Pentamix devices
Super Quick Heavy Imprint™ 4	Type 1 heavy-bodied	yellow	1:1	Garant Dispenser
Super Quick Heavy** Imprint™ 4	Type 3 light-bodied	blue	1:1	Garant Dispenser
Super Quick Regular Imprint™ 4	Type 3 light-bodied	lavender	1:1	Garant Dispenser
Super Quick Light Imprint™ 4	Type 3 light-bodied	lavender	1:1	Garant Dispenser
Super Quick Light Imprint™ 4 Super Quick Ultra-Light**	Type 3 light-bodied	orange	1:1	Garant Dispenser
Regular Setting Products:	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Imprint™ 4	Type 0 putty	pine green	5:1	all Pentamix devices after 1999.*
Penta Putty	Type 1 heavy-bodied	blue	5:1	all Pentamix devices
Imprint™ 4 Penta Heavy	Type 1 heavy-bodied	blue	1:1	Garant Dispenser
Imprint™ 4 Heavy**	Type 1 heavy-bodied	blue	1:1	Garant Dispenser
Imprint™ 4 Regular	Type 3 light-bodied	apple green	1:1	Garant Dispenser
Imprint™ 4 Light	Type 3 light-bodied	pink	1:1	Garant Dispenser
Imprint™ 4 Penta Monophase**	Type 2 medium-bodied	lavender	5:1	all Pentamix devices
Imprint™ 4 Monophase**	Type 2 medium-bodied	lavender	1:1	Garant Dispenser

* Pentamix devices before September 1999 extrude the paste too slowly.

** This product is not available in all countries.

☞ These Instructions for Use should be kept for the duration of product use. For all additionally mentioned products, please refer to the corresponding Instructions for Use.

Intended Use

Indications

All types of impressions, e. g.:

- Crown, bridge, inlay, and onlay preparations
- Implants
- Orthodontic impressions

The super quick setting products are especially suited for single-unit and two-unit impressions.

Contraindications

Contraindications are not available

Application area

Dentistry. For professional use only.

Precautionary Measures

For Patients

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known mint oil allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

For Dental Personnel

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.

Use of protective gloves and a no-touch technique is recommended. Mint oil may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Block out any undercuts or interdental spaces sufficiently to aid impression removal from the teeth after the impression material has set. Removal from the mouth may be difficult or could lead to the extraction of natural teeth or prostheses. Blocking-out may be necessary when using Imprint™ 4 Penta Putty or any other impression material with increased shore hardness.

Impression Tray:

All impression trays generally used for precision impressions are suitable. Rigid impression trays are recommended for two-step impression technique and when using Imprint™ 4 Penta Putty.

- For sufficient adhesion, apply VPS Tray Adhesive on the tray and allow it to dry completely. The ideal drying time is 5–15 minutes.

Penta Material:

- Place the Penta foil bags into the designated Penta cartridge.
- Insert the loaded cartridge into the Pentamix mixing unit.
- Attach a Penta Mixing Tip – Red.

- Turn the plungers of the mixing unit downward using the hand wheel until resistance occurs. Do not force further movement.
- When using new foil bags, start mixing and extrude a small amount until the paste is homogeneous in color. Discard this initial amount of paste.
- Please follow the section "Impression Taking" for tray filling procedure.

Garant Dispenser:

- Place the cartridge of the impression material to be used in a Garant dispenser.
- Prior to application, remove the cap and check that both cartridge openings are not clogged.
- When using new cartridges, extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
- To mix heavy-bodied or medium-bodied materials, attach a green Garant mixing tip to the cartridge.
- There are two options to mix and apply light-bodied or medium-bodied materials:
 - 1.1 For light-bodied materials attach a yellow Garant mixing tip to the cartridge and a yellow Garant intraoral tip to the mixing tip. To enlarge the exit opening of the intraoral tip, cut it with a scalpel if required.
 - 1.2 For medium-bodied materials attach a green Garant mixing tip to the cartridge and a white Garant intraoral tip to the mixing tip. To enlarge the exit opening of the intraoral tip, cut it with a scalpel if required.
- 2. Use a 3M ESPE Intra-oral Syringe – Green. See next section.
- Please follow the section "Impression Taking" for the further procedure.

3M ESPE Intra-oral Syringe – Green

The Intra-oral Syringe – Green can be used for syringing preparations with VPS precision impression materials of ISO 4823 Type 3 and Type 2 consistencies, provided the VPS impression materials are intended for syringing.

- Have the Garant dispenser with the inserted cartridge ready for use (see Preparation Garant Dispenser).
- Have an Intra-oral Syringe – Green ready for use and remove the plungers. Leave the mixing tip of the Intra-oral Syringe in an angled position; do not fold open.
- Place the Intra-oral Syringe on the openings of the cartridge and hold firmly in place.
- Use the Garant dispenser to fill the Intra-oral Syringe with the required amount of impression material, but do not exceed the marking line (about 1 cm below the mixing tip joint).
- Remove the Intra-oral Syringe from the cartridge and close the cartridge, using the original cartridge cap.
 - When the cap is used more than once, make sure that the catalyst paste does not come into contact with the base paste to prevent paste carryover.
- Place the plunger in the Intra-oral Syringe, pushing the paste in the syringe barrel no further than to the joint of the mixing tip.
 - Any paste extruded on the sides of the mixing tip joint must be wiped off before the application in the mouth.
 - The content of an Intra-oral Syringe is sufficient for syringing about 2–4 preparations. It may be necessary to prepare a number of Intraoral Syringes in advance in the case of larger prosthetic work.
 - An Intraoral Syringe must only be used on one patient!
 - The filled Intra-oral Syringe can be stored for as long as 12 hours.
- Immediately before application, fold out the mixing tip of the Intra-oral Syringe at an angle of 180° (until resistance is felt).
 - The mixing tip must be folded out completely to prevent leakages at the mixing tip joint.

- Press out a small quantity of paste onto a mixing block until the base paste and catalyst flow out in an even, homogeneous mixture. Immediately apply the impression material around the preparation (see Impression Taking).
 - Refilling an Intra-oral Syringe which has been emptied or partially emptied is prohibited.

Retraction

Retraction cords as well as solutions and pastes based on aluminum chloride, aluminum sulfate, or iron sulfate are suitable retraction agents.

Recommendation: 3M ESPE Astringent Retraction Paste/Retraction Capsule. The paste containing aluminum chloride is directly applied from the retraction capsule into the sulcus. The paste can be used as an alternative to, or in combination with, retraction cords or other retraction methods, depending on the clinical situation and the practitioner's application technique.

- Areas from which impressions are to be taken should be kept dry.
- Prior to taking the impression, completely remove all residual retraction agents by rinsing and drying.

Times

	Working Time at 23 °C / 74 °F from start of mixing* min:sec	Intraoral Syringe Time min:sec	Intraoral Setting Time min:sec
Symbols on Product			
Super Quick Setting			
Imprint TM 4 Penta Super Quick Heavy	1:15	-	1:15
Imprint TM 4 Super Quick Heavy	1:15	-	1:15
Imprint TM 4 Super Quick Regular	1:15	0:35	1:15
Imprint TM 4 Super Quick Light	1:15	0:35	1:15
Imprint TM 4 Super Quick Ultra-Light	1:15	0:35	1:15
Regular Setting			
Imprint TM 4 Penta Putty	1:30	-	2:30
Imprint TM 4 Penta Heavy	2:00	-	2:00
Imprint TM 4 Heavy	2:00	-	2:00
Imprint TM 4 Regular	2:00	1:00	2:00
Imprint TM 4 Light	2:00	1:00	2:00

* Start of mixing = Entry of paste into the mixing tip.

These are the processing times for products that have been stored at a temperature of 23 °C / 74 °F and 20 % – 80 % relative humidity. These times are reduced due to higher temperatures of the product or prolonged if the temperature of the product is lower.

Impression Taking

When applying any kind of paste be sure:

- the impression tray is prepared with adhesive.
- the preparation is clean and dry.
- that the base paste and catalyst paste flow evenly into the mixing tip and extrude homogeneously in color and free of streaks (paste color: see "Product Description") and to discard any unevenly mixed paste.
- to keep the mixing tip permanently immersed in the paste to avoid the formation of air bubbles.
- to apply the wash material in the sulcus from the bottom up.
- not to exceed the processing times of tray and syringe material to avoid incomplete and inaccurate impressions.
- not to extrude set material by force. This may damage the cartridge and mixing tip resulting in the formation of leaks.

One-step Impression Technique:

Recommended product combinations using Penta tray Material

Imprint TM 4 Penta Super Quick Heavy	Imprint TM 4 Super Quick Regular or Imprint TM 4 Super Quick Light
Imprint TM 4 Penta Putty	Imprint TM 4 Regular
Imprint TM 4 Penta Heavy	Imprint TM 4 Regular or Imprint TM 4 Light
Imprint TM 4 Penta Monophase	Imprint TM 4 Regular or Imprint TM 4 Light

Recommended product combinations using 50 ml cartridges

Imprint TM 4 Super Quick Heavy	Imprint TM 4 Super Quick Regular or Imprint TM 4 Super Quick Light
Imprint TM 4 Heavy	Imprint TM 4 Regular or Imprint TM 4 Light
Imprint TM 4 Monophase	Imprint TM 4 Regular or Imprint TM 4 Light

- Load the impression tray with the heavy-bodied or Putty impression material.
- While the tray is being loaded, apply the wash material around the preparation using a wiggle and stir motion.
 - Depending on the number of units to be syringed, initiate application so that the tray loading and syringing around the preparation are completed simultaneously.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after the paste has set completely.

Monophase technique

- Initially fill the Penta Elastomer Syringe with Imprint TM 4 Penta Monophase or have Imprint TM 4 Monophase in the Garant Dispenser at hand.
- Load the impression tray with Imprint TM 4 Penta Monophase or Imprint TM 4 Monophase impression material.
- While the tray is being loaded, apply Imprint TM 4 Penta Monophase from the Penta Elastomer Syringe or Imprint TM 4 Monophase from the Garant Dispenser around the preparation using a wiggle and stir motion.
 - Depending on the number of units to be syringed, initiate application so that the tray loading and syringing around the preparation are completed simultaneously.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.

- Remove the impression from the mouth after the paste has set completely.

Two-step Impression Technique:

Recommended product combinations

Imprint™ 4 Penta Putty Imprint™ 4 Super Quick Ultra-Light or
Imprint™ 4 Light

- Load the impression tray with the heavy-bodied or Putty impression material.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after the paste has set.
- Excise undercuts and interdental septa, etc., and cut spillways if required.
- Clean and dry the impression thoroughly to provide for optimal coalescence of the two materials.
- Insert the wash impression material into the preliminary impression at the appropriate sites and/or apply around the preparations.
- Replace the preliminary impression in the mouth and hold it in place.
- Remove the impression from the mouth after the paste has set completely.

After Impression Taking

- Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual impression material that may have been left in the mouth.

Hygiene

- Place the impression in a standard disinfectant solution and follow the manufacturer's instructions for handling. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
- After disinfection, rinse the impression of any remaining disinfection solution with running water for approximately 15 seconds.

Model Preparation

- The impression may be poured at the earliest 30 minutes after removal from the mouth.
- In order to attain a bubble-free model, rinse the impression with water shortly before pouring; remove excess water. A silicone surfactant can be used if required.
- Pour the impression with a standard dental die or model stone. The impressions can be silver- or copper-plated.

Cleaning

- Garant Dispenser: remove paste that has not set using a cloth soaked in alcohol. The dispenser handle and plunger can be autoclaved up to a temperature of 135 °C / 275 °F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- Paste that has not set may be removed with ethanol or by rinsing with water and soap.

Notes

- Completely remove the oxygen inhibition layer of composite materials, e.g., fillings or core buildups; it may impair or retard the setting of silicone impression materials.
- Keep the filled mixing tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent mixing tip may lead to carry-over of paste and ensuing formation of clogs.
- After filling a 3M ESPE Intra-oral Syringe reclose the cartridge with the original cartridge cap.
- Vinyl polysiloxane impression materials must not be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.

- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

Technical characteristics

Imprint™ 4 Penta Heavy Base Paste cartridge: Length 130 ± 10 mm, Diameter 60 ± 6 mm; Imprint™ 4 Penta Putty Base Paste cartridge: Length 130 ± 10 mm, Diameter 60 ± 6 mm; Imprint™ 4 Penta Super Quick Base Paste cartridge: Length 130 ± 10 mm, Diameter 60 ± 6 mm; Base Paste/Catalyst Paste Imprint™ 4 Light cartridge: Length 124 ± 2 mm, Diameter 50 ± 2 mm; Base Paste/Catalyst Paste Imprint™ 4 Regular cartridge: Length 124 ± 2 mm, Diameter 50 ± 2 mm; Base Paste/Catalyst Paste Imprint™ 4 Super Quick ultra-light cartridge: Length 124 ± 2 mm, Diameter 50 ± 2 mm; Base paste/Catalyst Paste Imprint™ 4 Super Quick Light cartridge: Length 124 ± 2 mm, Diameter 50 ± 2 mm; Imprint™ 4 Penta Heavy Catalyst Paste cartridge: Length 130 ± 10 mm, Diameter 27 ± 3 mm; Imprint™ 4 Penta Putty Catalyst Paste cartridge: Length 130 ± 10 mm, Diameter 27 ± 3 mm; Imprint™ 4 Penta Super Quick Heavy Catalyst Paste cartridge: Length 30 ± 10 mm, Diameter 27 ± 3 mm.

Completeness of kits:

Material Imprint™ 4 Penta Heavy Trial Kit (71497): Base paste Imprint™ 4 Penta Heavy (1 pcs. 300 ml); Catalyst paste Imprint™ 4 Penta Heavy (1 pcs. 60 ml); mixing tips Penta (10 pcs.); Imprint™ 4 Light (1 pcs. 50 ml.); Garant mixing tips, yellow (5 pcs.); Garant Intraoral Tips, yellow (5 pcs.); Instruction for use (1 pcs), Step-by-step card (1 pcs). Material Imprint™ 4 Penta Heavy (71484) in kit: Base paste Imprint™ 4 Penta Heavy (2 pcs. 300 ml); Catalyst paste Imprint™ 4 Penta Heavy (2 pcs. 60 ml); Instruction for use (1 pcs). Material Imprint™ 4 Penta Putty (71486) in kit: Base paste Imprint™ 4 Penta Putty (2 pcs. 300 ml); Catalyst paste Imprint™ 4 Penta Putty (2 pcs. 60 ml); Instruction for use (1 pcs). Material Imprint™ 4 Light (71488) in kit: Imprint™ 4 Light (4 pcs. 50 ml.); mixing tips, yellow (5 pcs.); intra-oral syringe, green (5 pcs.); Plug of cartridge (2 pcs., if necessary); Instruction for use (1 pcs). Material Imprint™ 4 Penta Super Quick Heavy (71485) in kit: Base paste Imprint™ 4 Penta Super Quick Heavy (2 pcs. 300 ml); Catalyst paste Imprint™ 4 Penta Super Quick Heavy (2 pcs. 60 ml); Instruction for use (1 pcs). Material Imprint™ 4 Regular (71487) in kit: Imprint™ 4 Regular (4 pcs. 50 ml.); mixing tips, yellow (5 pcs.); intra-oral syringe, green (5 pcs.); Plug of cartridge (2 pcs., if necessary); Instruction for use (2 pcs). Material Imprint™ 4 Super Quick Ultra-Light (71491) in kit: Imprint™ 4 Super Quick Ultra-Light (4 pcs. 50 ml.); mixing tips, yellow (5 pcs.); intra-oral syringe, green (5 pcs.); Plug of cartridge (2 pcs., if necessary); Instruction for use (2 pcs). Material Imprint™ 4 Super Quick Light (71490) in kit: Imprint™ 4 Super Quick Light (4 pcs. 50 ml.); mixing tips, yellow (5 pcs.); intra-oral syringe, green (5 pcs.); Plug of cartridge (2 pcs., if necessary); Instruction for use (2 pcs).

Storage, use and Stability

Store the product at $15-25^{\circ}\text{C} / 59-77^{\circ}\text{F}$.

Do not use after the expiration date.

Store set impressions at temperatures below $30^{\circ}\text{C} / 86^{\circ}\text{F}$.

Operating conditions: This product is designed to be used at room temperature. The humidity and pressure are not regulated.

Transportation conditions: Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance.

Transportation of medical device is carried out by all type of transport.

Maintenance and repair. This medical device isn't subject to maintenance and repair.

Disposal

Disposal methods: Dispose of content/container in accordance with the local/regional/national/international/regulations. Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Medicinal substances: Medical devices do not contain or deliver medicinal substances.

Human Blood Derivatives: Medical devices do not contain or deliver human blood derivatives.

Animal-Derived Substances: No materials of human origin are intentionally being added to the products.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Information valid as of October 2019

Customer Service

Authorized representative of the manufacturer in the Russian Federation:

AO "3M Russia", Moscow, Poselenie Moskovskiy, Kievskoe shosse, 22 km,6, bld.1

phone +7 (495) 578-74-74

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Dental Products

Carl-Schurz-Str. 1

41453 Neuss – Germany

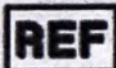
3M, ESPE, Imprint, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH.

© 2014, 3M. All rights reserved.

44000766517/04

(44000766905/04)

Labels and packaging symbols

	Use by date		Do not reuse
	Catalogue number		Lot number
	Caution	CONTENTS	Contained items
	Recycling code (Low-density polyethylene)		Product meets the requirements of the applicable EC directives
	Manufacturer		Temperature limit (storage conditions)
	Recycling code (Non-corrugated fiberboard)		Green dot

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Instructions for Use

Product Description

Intra-oral Syringes - Green are used to apply VPS impression materials around preparations. They have a maximum capacity of 1.5 ml of impression material and are filled individually with the aid of the hand dispenser (e.g., Garant) right from 50 ml cartridges.

The Intra-oral Syringes are available in the colors purple for 3M ESPE polyether impression materials (mixing ratio 2:1) and green for addition-type silicone impression materials (VPS) (mixing ratio 1:1).

☞ These Instructions for Use must be kept for reference for the duration of product use. Please refer to the relevant instructions for use for details about the impression materials used in the application syringe.

Indications

- Intra-oral Syringes - Green: Syringing preparations with VPS precision impression materials in accordance with ISO 4823 Type 3 and Type 2, provided the VPS impression materials are intended for syringing.

The Intra-oral Syringes may not be used for impression materials with a higher consistency or for other products in paste form (e.g., impression materials for filling the tray in accordance with ISO 4823 Type 1 and Type 0 or pastes for bite registration).

Application

The mixing spirals and plungers are color-coded to ensure clear correlation of the Intra-oral Syringe to the correct material class (polyether or addition-type silicone impression materials):

Purple = Mixing ratio 2:1 = 3M ESPE polyether impression materials (e.g., Impregum, Permadyne).

Green = Mixing ratio 1:1 = addition-type silicone impression materials (VPS).

The content of an Intra-oral Syringe is sufficient for syringing about 2-4 preparations, depending on the quantity applied around each unit. An Intra-oral Syringe must only be used on one patient!

- Have the hand dispenser with the inserted 50 ml cartridge ready for use (see Garant installation instructions, for example).
- Remove the original cap or the mixing tip left on the cartridge from the previous use from the 50 ml cartridge. **Retain the original cap!**
- Check the cartridge openings for possible blockages in accordance with the instructions for use for the impression material; press out a small quantity of base paste and catalyst to equal the pastes.
- Remove the plunger of the Intra-oral Syringe.
- Place the Intra-oral Syringe on the openings of the Garant cartridge and hold firmly in place while filling. Leave the mixing tip in an angled position; do not fold open.
- Use the hand dispenser to fill the Intra-oral Syringe with the required amount of impression material, but do not exceed the marking line (about 1 cm below the mixing tip joint).
- Remove the Intra-oral Syringe from the 50 ml cartridge and close the 50 ml cartridge, using the original cartridge cap.
 - When the cap is used more than once, make sure that the catalyst paste does not come into contact with the base paste to prevent paste carryover.

- Place the plunger in the Intra-oral Syringe, pushing the paste in the syringe barrel no further than to the joint of the mixing tip.
 - Any paste extruded on the sides of the mixing tip joint must be wiped off before the application in the mouth.
 - Once prepared as described above, the Intra-oral Syringe can be stored for as long as 12 hours. It may be necessary to prepare a number of Intraoral Syringes in advance, depending on the number of preparations to be syringed, in the case of larger prosthetic work.
- Immediately before application, fold out the mixing tip of the Intra-oral Syringe at an angle of 180° (until resistance is felt).
 - The mixing tip must be folded out completely to prevent leakages at the mixing tip joint.
- Press out a small quantity of paste onto a mixing block until the base paste and catalyst flow out in an even, homogeneous mixture.
- Immediately apply the impression material around the preparation in accordance with the instructions for use.
 - Refilling an Intra-oral Syringe which has been emptied or partially emptied is prohibited.

Notes

- Intra-oral Syringes cannot be autoclaved.
- The Intra-oral Syringe - Purple with the mixing ratio 2:1 may be used solely for 3M ESPE polyether impression materials, and the Intra-oral Syringe - Green with the mixing ratio 1:1 may be used solely for addition-type silicone impression materials (VPS). Filling the Intra-oral Syringe with the wrong paste will cause incorrect mixing results.

Technical characteristics

Length (mm): 104 ±10; Diameter (mm): 27 ±3.

Supply configuration: Intra-oral Syringe, (20 pcs.).

Operating Transportation and Storage Conditions

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Usage: Product is designed to be used at room temperature, the humidity and pressure are not regulated

Transportation: During transportation the products may experience short term temperature variations without affecting product safety and performance.

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Maintenance and Repair

This medical device isn't subject to maintenance and repair

Disposal

Disposal methods: Dispose of content/container in accordance with the local/ regional/national/ international/ regulations

Medical devices in Russian Federation must be disposed of in the local public entities in accordance with SanPiN 2.1.7.2790-10 "Sanitary-epidemiological requirements for the handling of medical waste" to the waste class B).

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Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture, 3M Deutschland GmbH MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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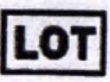
00000000000/01 (aro18112014)

Labels and packaging symbols

	Use by date		Do not reuse
	Catalogue number		Lot number
	Caution	CONTENTS	Contained items
			Product meets the requirements of the applicable EC directives
	Manufacturer		Temperature limit (storage conditions)

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3M, ESPE, Garant, Impregum and Permadyne are trademarks of 3M or 3M Deutschland GmbH.
© 2015, 3M. All rights reserved.