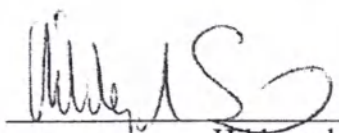
APPROVED

Hildegard Saar
Deputy Head of Regulatory Affairs of
URSAPHARM Arzneimittel GmbH
URSAPHARM
Arzneimittel GmbH
Industriest. 35 D-66129 Saarbrücken
Telefon +49 (0) 6805 9292-0

INSTRUCTION FOR USE

*Please, read this instruction carefully before the start of application/use of
this medical device*

Ophthalmological moisturizing solution

HYLOMAX-COMOD®

Preservative-free!

Sterile

For instillation into conjunctival sac

Sodium hyaluronate 2 mg/ml solution

Composition

1 ml of HYLOMAX-COMOD® solution contains:

Sodium hyaluronate– 2,0 mg

Citric acid anhydrous, sodium citrate dihydrate, sorbitol, water for injections.

Intended purpose for medical device

- For intensive moistening, lubricating and protection of the front surface of the eye (cornea and conjunctiva) at expressed and prolonged feeling of "dryness" of the eyes, foreign body sensation, burning in the eyes, including those which are being caused by climatic factors and intense visual load;
- For intensive moistening, lubricating and accelerating of healing of the front surface of the eye after ophthalmic surgeries, as well as at after cornea's injuries and damages;
- To eliminate discomfort when wearing contact lenses.

General properties

Medical device Ophthalmological moisturizing solution HYLOMAX-COMOD® is sterile isotonic 0,2% aqueous sodium hyaluronate solution without preservatives in an original container (COMOD®-system).

Hyaluronic acid (in the form of sodium salt) in HYLOMAX-COMOD® solution is a natural substance. Sodium hyaluronate is a physiological polysaccharide compound found both in the tissues of the eye and in other tissues and fluids of the human body.

A special physical-chemical property of sodium hyaluronate molecules is their strong ability to bind water molecules. Aqueous solution of sodium hyaluronate has a necessary viscosity and high adhesive

properties to the anterior surface of the eye, due to which HYLOMAX-COMOD® solution forms a uniform, long-lasting tear film on the corneal surface, which is not washed off while winking and does not cause visual acuity decline. This protects the eye permanently against dryness and irritation that often arise through contact with the environment and when during wearing contact lenses.

HYLOMAX-COMOD® solution contains a significant concentration of sodium hyaluronate, and therefore provides a high viscosity. Prolonged and intense discomfort disappears, resulting in permanent eye protection against irritation.

At the same time? Constant and sufficient hydration of the eye surface contributes to a normal physiological course and accelerates the healing of eye injuries and wounds after injuries and surgical interventions.

HYLOMAX-COMOD® solution makes wearing of any soft or rigid contact lenses more comfortable and HYLOMAX-COMOD® solution does not deposit on the surface of the lenses.

HYLOMAX-COMOD® solution is stored in the original COMOD® container, which is an elaborate system of reservoirs and valves to ensure that no outside air enters and that uniform droplets are ejected regardless of the degree of force exerted.

Metal parts and valves of the container which are in contact with the HYLOMAX-COMOD® solution are partially silver coated, which together with the absolute air-tightness of the system makes the HYLOMAX-COMOD® solution sterile as long as no preservatives are present.

This prevents possible undesirable effects of preservatives on the eye tissue and ensures good tolerance of the HYLOMAX-COMOD® solution, even with prolonged use.

Contraindications

Hypersensitivity to any of the components of the HYLOMAX-COMOD® solution.

Dosage, frequency, and duration of application

It is recommended to apply 1 drop of HYLOMAX-COMOD® solution 3 times per day into conjunctival sac of each eye. HYLOMAX-COMOD® solution can be used more frequently if necessary. The instillation frequency of HYLOMAX-COMOD® solution must be determined individually in accordance with how you feel and the recommendation of your ophthalmologist or contact lenses specialist. If you use HYLOMAX-COMOD® solution very frequently (for example, more than 10 times per day), please consult with your ophthalmologist. You should also consult an ophthalmologist if you continue to have discomfort or complaints several days after the application of HYLOMAX-COMOD®.

The COMOD®-system contains and allows the extraction of 10 ml of HYLOMAX-COMOD® solution, which corresponds to approximately 300 drops of solution.

A new filled container of HYLOMAX-COMOD® solution must be purchased after the contents have been completely used, as the system is not designed for reuse.

There is no limit to the duration of use of HYLOMAX-COMOD® solution.

Mode of application and recommendations for contact lenses users

The cap must be removed prior to each application (see Fig. 1 at the end of the text).

Before using HYLOMAX-COMOD® solution for the first time, invert the container with drip tip facing down and press the base several times until the first drop appears at the tip of the drip tip (see Fig. 2). Now the system is ready for further use.

When injecting the HYLOMAX-COMOD® solution, hold the container with the drip tip downwards, pushing rapidly and vigorously on its base. This activates the drip mechanism and only one drop

of solution is extracted. Due to the special design of the valves in the COMOD® system, the droplet size and extraction speed remain uniform even when applying very high pressure on the base of the container.

With the head slightly tilted back, gently retract the lower eyelid with your finger and drop a single drop into the conjunctival sac as described above (see Fig.3). Slowly close the eye, allowing the liquid to spread evenly over the surface of the eye.

Once the procedure is complete, place the cap tightly over the dropper (see Fig.4).

Avoid contact between the tip of the dropper and the eye and skin during the drip.

When wearing soft and rigid contact lenses, HYLOMAX-COMOD® solution can be injected into conjunctival sac without lens removal.

The COMOD-system® is intended for individual use only.

Side effects

In very rare cases hypersensitive reactions like burning or excessive tearing could occur, all effects stop immediately when the use of HYLOMAX-COMOD® solution is to be discontinued.

Precautions for use and interactions with other eye products

There is no limitation of the duration of application of HYLOMAX-COMOD® solution. However – as in all persisting symptoms – it is recommended to consult with ophthalmologist if complaints do not improve e.g. within two weeks.

HYLOMAX-COMOD® solution should not be applied concomitantly with other ophthalmological preparations. In these cases, HYLOMAX-COMOD® solution should be applied about 30 minutes after the use of the other ophthalmic products. In case of application of eye ointments, they shall be applied after the HYLOMAX-COMOD® instillations.

Storage conditions

HYLOMAX-COMOD® solution should be stored at temperature below +25°C.

HYLOMAX-COMOD® solution should be stored out of reach of children.

Shelf-life 3 years in closed original package.

After first opening of a container the HYLOMAX-COMOD® solution could be used for 12 weeks.

Mark the date when you have started to use HYLOMAX-COMOD® solution. On the container's label a special field "Date of first application" for filling by patient is provided.

Do not use beyond expiration date printed on the outer carton.

Disposal instructions

After finishing using the HYLOMAX-COMOD® solution, the container can be disposed of together with solid household waste (class A). Unused containers that have expired should be disposed of in the same way.

Packaging

10 ml of ophthalmological moisturizing solution HYLOMAX-COMOD® is filled into a multi-dose container (COMOD®-system), which is placed into an outer carton together with Instruction for use. 280 units of finished products are placed into carton transport box. The dimension of transport box: 60 x 40 x 25 cm; weight: approx. 11.2 kg.

Transport conditions

All types of covered transport at temperatures from +5°C to +25°C in carton transport boxes.

Maintenance instructions and repair of medical device

Not applicable for the medical device Ophthalmological moisturizing solution HYLOMAX-COMOD®.

Guarantee obligations

The manufacturer guarantees the quality of medical device for the declared shelf life when specified conditions of transportation, storage, applications are duly respected.

Manufacturer

URSAPHARM Arzneimittel GmbH
Industriestrasse 35, 66129 Saarbruecken, Germany

Representative office in Moscow:

107996, Moscow, Gilyarovskogo str. 57, bld. 4

Tel/fax: +7 (495) 684-34-43

e-mail: ursapharm@ursapharm.ru

<http://www.ursapharm.ru>

Authorized Representative of URSAPHARM Arzneimittel GmbH in Russia in all matters related to medical device circulation

ZAO ITEMS WAREHOUSE


Legal Address: 121059, Moscow, Bryanskaya str. 12

Actual Address: 109428, Moscow, Ryazansky prospect 16

Tel/fax: +7 (495) 729 40 04

Explanation by drawings for the section

“Mode of application and recommendations for contact lenses users”

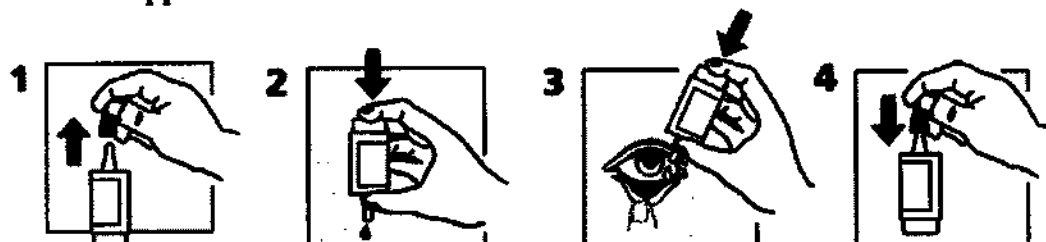Medical device “Ophthalmological moisturizing solution HYLOMAX-COMOD®” is registered in the Russian Federation by Federal Service on surveillance in healthcare under the No. ΦС3 2010/07271, 25.06.2010. and included in the State Register of medical devices and Organizations engaged in the manufacturing of medical devices.

Data of information:

December 2023

Indicative icons:

Manufacturer:	
Lot number:	
Shelf life:	

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