78630


Collagen Matrix, Inc.

15 Thornton Road, Oakland, New Jersey 07436 • Tel: 201-405-1477 • Fax: 201-405-1355

MANUFACTURER'S STATEMENT

To whom it may concern;

Collagen Matrix, Inc. with a business address of 15 Thornton Road, Oakland, New Jersey, 07436, USA hereby confirms that the attached Instructions for Use (IFU) for SYMBIOS® Xenograft Granules is a draft copy of the current Instructions for Use (MS512, Rev 0) containing specific information for registration in the country of Russia.

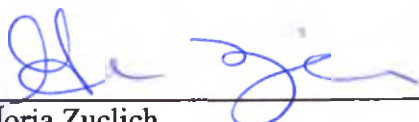Gloria Zuclich
Director, Regulatory Affairs

March 30, 2018

Date

Subscribed and sworn to before me this 30 day of March month, 2018 year.

SYMBIOS® Xenograft Granules

INSTRUCTIONS FOR USE

Description: SYMBIOS® *Xenograft Granules* is a sterile, biocompatible anorganic porous porcine bone mineral matrix consisting predominantly of calcium phosphate for use in periodontal, oral and maxillofacial surgery. It is produced by removal of the organic components from porcine cancellous bone. SYMBIOS® *Xenograft Granules* is sterilized by gamma irradiation. The product is available in granular form, non-pyrogenic and for single use only.

The anorganic bone matrix of SYMBIOS® *Xenograft Granules* has interconnecting macro- and microscopic porous structure that supports the formation and ingrowth of new bone at the implantation site. The use of SYMBIOS® *Xenograft Granules* may be considered when autogenous bone is not indicated, or insufficient in quantity to fulfill the needs of the proposed surgical procedure.

Indications for Use: SYMBIOS® *Xenograft Granules* is indicated for:

- Augmentation or reconstructive treatment of the alveolar ridge.
- Filling of infrabony periodontal defects.
- Filling of defects after root resection, apicoectomy, and cystectomy.
- Filling of extraction sockets to enhance preservation of the alveolar ridge.
- Elevation of the maxillary sinus floor.
- Filling of periodontal defects in conjunction with products intended for Guided Tissue Regeneration (GTR) and Guided Bone Regeneration (GBR).
- Filling of peri-implant defects in conjunction with products intended for Guided Bone Regeneration (GBR).

Contraindications: Contraindications customary to the use of bone grafts should be observed. SYMBIOS® *Xenograft Granules* should not be used in patients with:

- Acute or chronic infection (osteomyelitis) at the surgical site
- Metabolic diseases (diabetes, hyperparathyroidism, osteomalacia)
- Severe renal dysfunction, severe liver disease
- High dose corticosteroid therapy
- Vascular impairment at the implant site
- Osteoporosis
- Known allergy or hypersensitivity to porcine-derived implantable materials

Warnings: The device should be secured to prevent motion and migration; use in areas where the graft can be adequately contained. Do not use if package is opened or damaged or if expiration date has been exceeded.

SYMBIOS® *Xenograft Granules* cannot be re-sterilized or re-used. Open, unused product must be discarded. In vivo stability may be adversely affected if re-sterilized. Cross-contamination and infection may occur if re-used.

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist or physician.

Precautions: In order to facilitate the formation of new bone SYMBIOS® *Xenograft Granules* should only be implanted in direct contact with a well vascularized bony tissue. Drilling may be recommended to facilitate bleeding from cortical bone.

In larger defects a mixture of autogenous bone or bone marrow may improve the formation of new bone.

Implantology

Generally, in augmented areas, the placement of titanium fixtures should take place once the bone has sufficient strength and integrity for dental implant placement, which is typically greater than 6 months after implantation of a bone graft material. For sinus floor elevation, typically 9-12 months should be allowed after implantation of bone graft material before placement of the titanium fixtures. X-rays should be taken to confirm the bone integrity prior to dental implant placement.

Periodontology

The filling of periodontal defects with SYMBIOS® *Xenograft Granules* requires (along with plaque control) the successful local treatment of the periodontal lesion (e.g. root planing, debridement of granular tissue) prior to implantation.

Adverse Reactions: Possible complications that may occur with any dental surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection or pain.

Application:

- After exposure of the bony defect with a mucoperiosteal flap, all granulation tissue must be carefully removed.

DRAFT, 03-29-2018

mix SYMBIOS® Xenograft Granules with autogenous bone, osseous coagulum, patient's blood or sterile normal saline. If large maxillofacial defects are present, SYMBIOS® Xenograft Granules should be mixed with autogenous bone in a ratio of approximately 1:1.

In order to assure the formation of new bone, SYMBIOS® Xenograft Granules should only be placed in direct contact with well vascularized bone. Cortical bone should be mechanically perforated.

Loosely pack SYMBIOS® Xenograft Granules into osseous defect using a sterile instrument. Use of excessive force will result in compression of the particles and loss of trabecular architecture.

- Overfilling of the defects should be avoided.
- The mucoperiosteal flaps should be sutured to achieve primary closure, if possible. A surgical dressing may be placed over the wound for one to two weeks.
- If primary wound closure cannot be achieved completely, further immobilization of the flap (e.g., by incision through the periosteum) should be performed and/or a bioabsorbable membrane (e.g. Resorbable Collagen Membrane) should be placed over the bone graft site.

How Supplied: SYMBIOS® Xenograft Granules is supplied sterile, non-pyrogenic, and for single use only.

Storage: Store at room temperature (15 °C/59 °F – 30 °C/86 °F).

Available Sizes:

Catalogue Number	Particle Size	Volume
32310000	0.25 – 1.0 mm	0.5 cc (0.5 ml)
32310001		1.0 cc (1.0 ml)
32310002		2.0 cc (2.0 ml)
32310003		4.0 cc (4.0 ml)
32310004	1.0 – 2.0 mm	1.0 cc (1.0 ml)
32310005		2.0 cc (2.0 ml)

Additional Country Specific Storage and Product Information:

SYMBIOS® Xenograft Granules do not contain any dangerous hazardous substances that would require special handling. SYMBIOS® Xenograft Granules can be disposed of in accordance with all local, state, or federal regulations.

Store at room temperature (15 °C/59 °F – 30 °C/86 °F) in a dry place (relative humidity about 10% to 80%). The shelf life of the device is 36 months from the date of manufacture. The manufacturer guarantees the operability and maintenance of the sterility of the products during the entire shelf life, subject to storage conditions. The product can be transported by any means of transport, such as covered railway wagons, containers, freight vehicles, ship holds, aviation modules in accordance with the rules for these to modes of transport, in compliance with the conditions of transportation without risk of damage to the package. The product does not affect the environment during operation. Minor excursions from the recommended temperature range during storage, shipping, and handling are acceptable. Product cannot be guaranteed to meet specifications, if stored at temperatures below -10°C / 14°F or above 49°C / 120°F for more than 10 days.

Labeling Symbols:

Manufacturer

Use-by date

Do not re-use

Caution: See instructions for use

Batch number

Catalogue number

Do not use if package is damaged

Temperature limitation

Sterilized using irradiation

Do not re-sterilize

R_x Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist or physician.

Authorized representative in the European Community

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Distributor:

www.dentsplyimplants.com

Authorized representative of the manufacturer in the territory of the Russian Federation:

LLC "Dentsply Sirona", Russian Federation, 115432 Moscow, Andropov, Prospekt 18, Building 6 tel.: +7 (495) 725 1087

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see on the
package

Only for use by dentists **Rx Only**

Information on safety, indications and contraindications:
see on the package

STERILE R

2
STERILIZE

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477 , fax: 201.405.1355 ,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:

LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Prospekt 18,
Building 6 tel.: +7 (495) 725 1087

Xenograft Granules 0.25 - 1.0 mm, 1.0 cc (1.0 ml)

Dental matrix Implant for bone tissue SYMBIOS Xenograft Granules

Reg. Cert. № d.d. Particle Size: 0,25-1,0 mm
Volume: 1.0 cc (1.0 ml)

REF 32310000

LOT see on the package

see on the package

Only for use by dentists **Rx Only**

Information on safety, indications and contraindications:
see on the package

STERILE R

2
STERILIZE

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477 , fax: 201.405.1355 ,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:

LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Propsect 18,
Building 6 tel.: +7 (495) 725 1087

Xenograft Granules 0.25 - 1.0 mm, 2.0 cc (2.0 ml)

Dental matrix Implant for bone tissue SYMBIOS Xenograft Granules

Reg. Cert. № d.d. Particle Size: 0,25-1,0 mm
Volume: 2.0 cc (2.0 ml)

REF 32310000

LOT see on the package

 see on the package

Only for use by dentists **Only**

Information on safety, indications and contraindications:
see on the package

STERILE R

2
STERILIZE

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477 , fax: 201.405.1355 ,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:

LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Propsect 18,
Building 6 tel.: +7 (495) 725 1087

Xenograft Granules 0.25 - 1.0 mm, 4.0 cc (4.0 ml)

Dental matrix Implant for bone tissue SYMBIOS Xenograft Granules

Reg. Cert. № d.d. Particle Size: 0,25-1,0 mm
Volume: 4.0 cc (4.0 ml)

REF 32310000

LOT see on the package

see on the package

Only for use by dentists **Rx Only**

Information on safety, indications and contraindications:
see on the package

STERILE R

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477 , fax: 201.405.1355 ,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:

LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Propsect 18,
Building 6 tel.: +7 (495) 725 1087

Xenograft Granules 1.0 - 2.0 mm, 1.0 cc (1.0 ml)

Dental matrix Implant for bone tissue SYMBIOS Xenograft Granules

Reg. Cert. № d.d. Particle Size: 1.0 - 2.0 mm
Volume: 1.0 cc (1.0 ml)

REF 32310000

LOT see on the package

see on the package

Only for use by dentists **Rx Only**

Information on safety, indications and contraindications:
see on the package

STERILE R

2
STERILIZE

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477 , fax: 201.405.1355 ,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:
LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Propsect 18,
Building 6 tel.: +7 (495) 725 1087

Xenograft Granules 1.0 - 2.0 mm, 2.0 cc (2.0 ml)

Dental matrix Implant for bone tissue SYMBIOS Xenograft Granules

Reg. Cert. № d.d. Particle Size: 1.0 - 2.0 mm
Volume: 2.0 cc (2.0 ml)

REF 32310000

LOT see on the package

see on the package

Only for use by dentists **Rx Only**

Information on safety, indications and contraindications:
see on the package

STERILE

R

2
STERILIZE

keep the packaging and instructions until the product is used

Manufacturer:

«Collagen Matrix, Inc., 15 Thornton Road,
Oakland, New Jersey 07436, USA/CША,
tel: + 201.405.1477, fax: 201.405.1355,
e-mail:
CMIcontact@collagenmatrix.com

Country of origin: USA

Authorized representative:

LLC "Dentsply Sirona,
Russian Federation, 115432
Moscow, Andropov, Propsect 18,
Building 6 tel.: +7 (495) 725 1087

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