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HOLLYWOOD SPECTRA

* The laser operator and all staff in the laser treatment room should wear protective eyewear.

Laser Peel

3-8 mm
Zoom-Collimated

Hollywood Peel

Indication	Wavelength (nm)	Tx. Step	Carbon Lotion	Mode	Spot Size (mm)	Fluence (J/cm ²)	Pulse Rate (Hz)	Pass	Session	Interval (week)
• Skin Rejuvenation • Enlarged Pores • Inflammatory Acne & Redness	1064	1	○	Spectra	8	0.9~	5-10	1	3~	2-3
		2	○	QS	8	0.9~	5-10	2-4		
		3	X	Spectra	8	2.4~	5-10	Partial multi-pass		

◇ Endpoint: Mild erythema, slight heat sensation
 - Note: Avoid treating melasma patients or skip the melasma area as the thermal effect could aggravate the condition.

Hollywood Laser Peel® is a 3-step laser peel that uses our carbon-based Spectra Lotion™ with two different operation modes in 1064 nm: Spectra Mode (300 μs) and the Q-switched Mode (5 ns). The Spectra Mode stimulates the skin through gentle heating, and the Q-switched mode delivers mild epidermal exfoliation and resurfacing.

PREPARATION: Spectra Lotion

Lightly apply carbon-based Spectra lotion and leave for 15-20 minutes to allow seeping well into the pores.

① Spectra Mode (300 μs)

Gently heat up the carbon applied skin and induce mild epidermal turnover.

② QS Mode (5 ns)

Explode carbon particles to exfoliate stratum corneum and induce pore contraction.

HyperSurge beam speeds wound healing!

③ Spectra Mode (300 μs)

Repeatedly irradiate the target lesions to induce sebaceous gland contraction and increase skin elasticity.

Soft Peel

Indication	Wavelength (nm)	Carbon Lotion	Mode	Spot Size (mm)	Fluence (J/cm ²)	Pulse Rate (Hz)	Pass	Session	Interval (week)
• Epidermal Melasma • Uneven Skin Tone & Texture	1064	○	QS	8	0.9~	5-10	1	3~	2-3

◇ Endpoint: Mild erythema

Soft Peel™ is a 1-step laser peel that uses our carbon-based Spectra Lotion™ and Q-switched mode (5 ns) in 1064 nm. It effectively targets epidermal melanin that is abnormally dispersed, such as in epidermal melasma. This treatment may be combined with laser toning to treat dermal melasma condition.

PREPARATION: Spectra Lotion

Lightly apply carbon-based Spectra lotion and leave for 15-20 minutes to allow seeping well into the pores.

① QS Mode (5 ns)

Explode carbon particles to target excess pigment without damaging the skin cell.

HyperSurge beam speeds wound healing!

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HOLLYWOOD LASER PEEL

FACE IT. YOU'RE A STAR.

IMPROVED SKIN TONE • GLOWING, YOUTHFUL LOOKING SKIN
SMOOTHER, SOFTER FEEL • REDUCTION OF FINE LINES AND ACNE SCARS
NO DOWNTIME • SAFE FOR ALL SKIN TONES • STIMULATES COLLAGEN

Ask today about the Hollywood Laser Peel that is known to be used by stars before walking the red carpet!

Hollywood Laser Peel is also known as Spectra Laser Peel

© 2017, LUTRONIC, a division of Cosmova, All rights reserved. Hollywood Laser Peel and all logos are trademarks of the LUTRONIC, a division of Cosmova. LUTRONIC, a division of Cosmova.

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Unwanted skin pigmentation?
BE CAMERA READY

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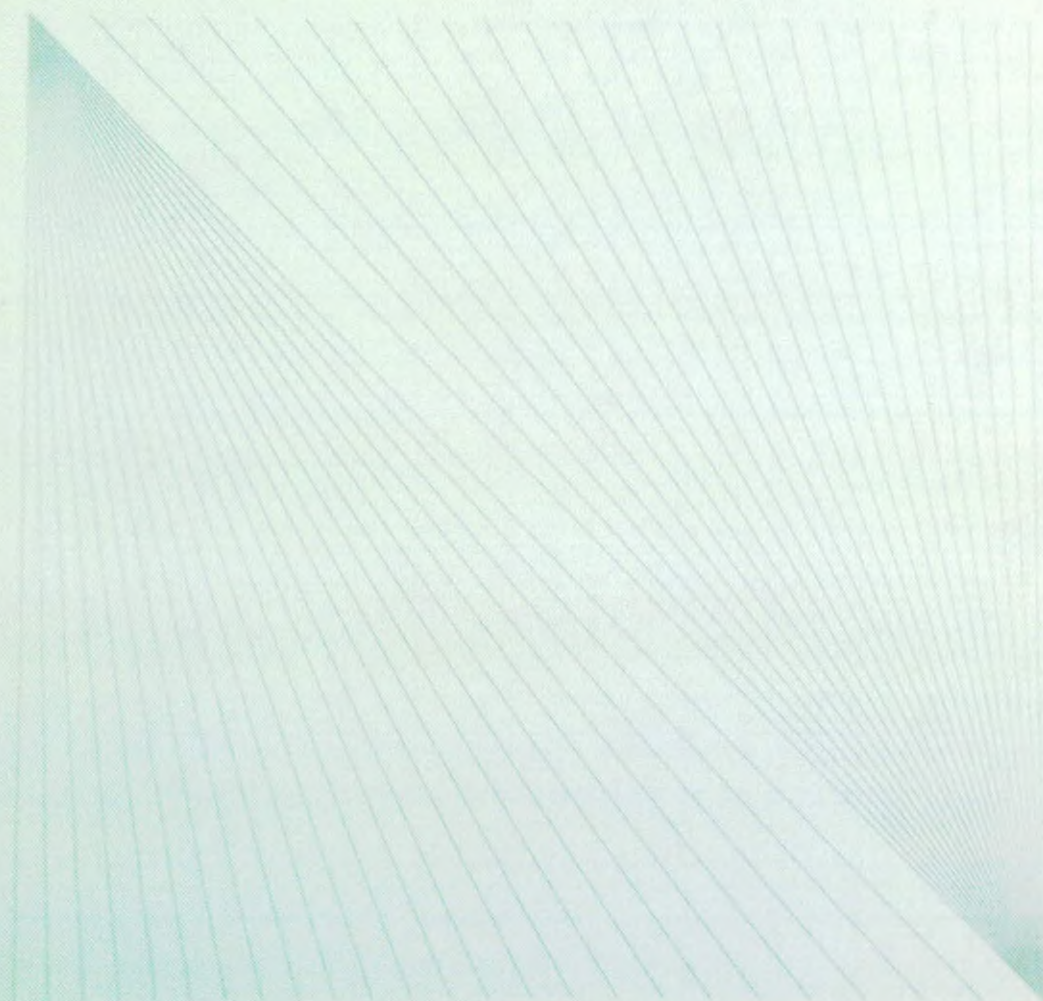
INTELLIGENT DESIGN FOR
INTELLIGENT CARE

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