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CERTIFICATE

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«02» May 2023 г.

INSTRUCTIONS FOR USE OF THE MEDICAL DEVICE

Orthodontic adhesives for fixing orthodontic brackets in sets

Version 001

CONTENT

Instructions for use Orthodontic adhesives for fixing orthodontic brackets in sets, in versions

Orthodontic adhesives for fixing light-cured brackets in sets..... 3

Instructions for use Orthodontic adhesives for fixing orthodontic brackets in sets, in versions

Orthodontic adhesives for fixing chemically cured brackets in a set..... 6

INSTRUCTIONS FOR USE

Orthodontic adhesives for fixation of orthodontic brackets in sets, in versions

Light-cure orthodontic adhesive for fixing brackets in sets

PY № _____ OT _____

These instructions for use apply to orthodontic adhesives for fixing orthodontic brackets in sets in the following versions:

1. Light-cure orthodontic adhesive for fixing brackets in a set consisting of:

- paste (syringe 3.5 g) - 3 pcs.;
- primer (bottle 8 ml) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 3 pcs.;
- brushes - 40 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc.

2. Light-cure adhesive for fixing brackets in set Mini, consisting of:

- paste (syringe 3.5 g) - 1 pc.;
- primer (bottle 3 ml) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 1 pc.;
- brushes - 20 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc.

3. Light cure orthodontic adhesive for fixing rings Blue glue band cement Series I in a set consisting of:

- paste (syringe 5 g) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 3 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc.

4. Light cure orthodontic adhesive for fixing brackets colored Green glue in a set consisting of:

- paste (syringe 4 g) - 2 pcs.;
- primer (bottle 8 ml) - 1 pc.;
- etching gel(syringe 2.5 ml) - 1 pc.;
- spatula - 1 pc.;
- brushes - 20 pcs.

- syringe tips - 1 pc.
- instructions for use - 1 pc.

5. Light cure orthodontic adhesive for fixing brackets colored Green glue in a set Mini consisting of:

- paste (syringe 4 g) - 1 pc.;
- primer (bottle 3 ml) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 1 pc.;
- brushes - 7 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc

DESCRIPTION

Light-curing orthodontic adhesive is a kind of methacrylic acid modified epoxy resin as the main raw material of orthodontic adhesive. This material consists of a liquid (primer), a paste and an etching agent - an acid solution (etching gel), auxiliary devices are mixing spatulas, brushes.

Appearance: the liquid (primer) must be transparent, without impurities, the color must be uniform; the paste should not contain impurities, clots, the filler should be evenly distributed; The etching agent is a translucent colloid containing phosphoric acid, the content of phosphoric acid in it is in the range of $(37 \pm 2)\%$.

PURPOSE

The medical product is intended for orthodontic bonding of all types of orthodontic brackets.

INDICATIONS FOR USE

Orthodontic adhesive is a material intended for fixation (bonding) of all types of orthodontic brackets for orthodontics.

CONTRAINDICATIONS

Hypersensitivity to adhesive components.

OPERATING PRINCIPLE

Orthodontic adhesive allows you to hold all types of brackets and other orthodontic devices on the enamel for a long period of time under the influence of various loads associated with tooth movement and chewing.

The principle of operation is to fix (glue) the surface of orthodontic devices (brackets) to the tooth enamel.

MODE OF APPLICATION

- 16
- 1) Tooth preparation: clean the surface of the tooth enamel, remove dirt from the surface of the teeth, rinse with water; Isolate the teeth, no oil and moisture, dry the surface of the teeth.
 - 2) Acid etching of the tooth surface: apply the etching gel to the tooth surface for about 50-60 seconds, rinse thoroughly with water, dry, acid etching is performed if, after drying, the tooth surface becomes chalky. If there is no chalk deposit, increase the acid etching time.
 - 3) Applying Primer: Using a small brush, apply a thin layer of primer to the surface of the tooth, and then apply a thin layer to the base of the bracket. (when using the Orthodontic Adhesive kit for fixing Blue glue band cement Series rings, the primer can be taken from other kits of the manufacturer)
 - 4) Applying adhesive: apply a small amount of paste to the base of the bracket, forming a thin layer. Glue the bracket to the desired location on the tooth. Adjust the position of the bracket, applying gentle pressure for 5-10 seconds to prevent the bracket from slipping. Remove excess liquid and paste.
 - 5) Light curing: for metal brackets, light curing with a curing lamp is carried out for 10 seconds on each side of the bracket. For ceramic brackets, illumination is carried out with a polymerization lamp from above for 10 seconds. After polymerization, an arch can be applied. Standard curing lamps can be used.

POSSIBLE SIDE EFFECTS

Not identified

WARNINGS

Before using a medical device, the dentist must read the warning information given in the instructions for use.

Orthodontic adhesives are supplied in non-sterile form and cannot be sterilized.

If the packaging is damaged, medical products cannot be used.

The dose of orthodontic adhesive is intended for single use and cannot be reused.

The safety and effectiveness of orthodontic adhesives in pregnant women has not been established.

It is necessary to avoid prolonged contact of the adhesive with the skin and mucous membrane of the oral cavity. Rinse thoroughly with water after contact.

PRECAUTIONARY MEASURES

- 1) Avoid contact of pickling agent containing phosphoric acid with eyes or skin. In case of contact, wash immediately with soap and water.
- 2) The tooth surface is covered with liquid (primer); exposure to light for 2-3 seconds can increase the consistency of the liquid to prevent the bracket from positioning after moving.
- 3) Some patients may experience allergic reactions to uncured resin; contact with gums or other soft tissue should be avoided. If contact occurs with skin, wash immediately with soap and water and consult a physician for treatment if necessary.
- 4) This material is sensitive to light and should be kept away from the light source.

- 17
- 5) The bonded orthodontic bracket must be of high quality and must be handled by a qualified dentist.
 - 6) The material is transported closed and away from sunlight.

STORAGE AND TRANSPORTATION

Orthodontic adhesive should be stored in its original packaging at a temperature between 0 and 28 °C.

Materials should not be exposed to elevated temperatures or intense light.

Do not damage the integrity of the packaging.

The medical product is transported by all types of covered transport, under conditions that ensure the safety of packaging and products, in accordance with the cargo transportation rules in force for this type of transport, at a temperature from 0 to 28 °C.

During transportation and storage, medical devices do not have a negative impact on humans and the environment.

MANUFACTURER'S WARRANTY

Orthodontic adhesives are not subject to periodic maintenance and routine repairs.

The shelf life of the medical device is 24 months from the date of manufacture.

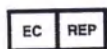
The expiration date is indicated on the packaging of the medical product; It is not allowed to use the product after the expiration date.

The manufacturer guarantees the quality, effectiveness and safety of orthodontic adhesives provided that the consumer complies with the conditions of storage, transportation, disposal and instructions for use of the medical product during the shelf life.

SYMBOLS USED IN MARKING

CE 0197

CE marking (in accordance with EU Directive 93/42/EEC) with identification of the notified body

Representative in the EU

Manufacturer

Batch number

Catalog number

Use before

Avoid exposure to sunlight

Refer to instructions for use

Temperature limit

Keep away from moisture

Warning, please refer to the instructions in the app

Last revision date: May 2023

INSTRUCTIONS FOR USE

Orthodontic adhesives for fixation of orthodontic brackets in sets, in versions

Non-mix orthodontic adhesive for fixing brackets in sets

PY № _____ OT _____

These instructions for use cover orthodontic adhesives for fixing orthodontic brackets in sets in the following versions:

1. Non-mix orthodontic adhesive for fixing brackets in a set consisting of:

- paste (syringe 3.5 g) - 3 pcs.;
- primer (bottle 10 ml) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 3 pcs.;
- brushes - 20 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc.

2. Non-mix orthodontic adhesive for fixing brackets in a set Mini consisting of:

- paste (syringe 3.5 g) - 1 pc.;
- primer (bottle 3 ml) - 1 pc.;
- etching gel (syringe 2.5 ml) - 1 pc.;
- spatula - 1 pc.;
- brushes - 7 pcs.;
- syringe tips - 1 pc.
- instructions for use - 1 pc.

DESCRIPTION

Light-curing orthodontic adhesive is a kind of methacrylic acid modified epoxy resin as the main raw material of orthodontic adhesive. This material consists of a liquid (primer), a paste and an etching agent - an acid solution (etching gel), auxiliary devices are mixing spatulas, brushes.

Appearance: the liquid (primer) must be transparent, without impurities, the color must be uniform; the paste should not contain impurities, clots, the filler should be evenly distributed; The etching agent is a translucent colloid containing phosphoric acid, the content of phosphoric acid in it is in the range of $(37 \pm 2)\%$.

PURPOSE

The medical product is intended for orthodontic bonding of all types of orthodontic brackets.

INDICATIONS FOR USE

10

Orthodontic adhesive is a material intended for fixation (bonding) of all types of orthodontic brackets for orthodontics.

CONTRAINDICATIONS

Hypersensitivity to adhesive components.

OPERATING PRINCIPLE

Orthodontic adhesive allows you to hold all types of brackets and other orthodontic devices on the enamel for a long period of time under the influence of various loads associated with tooth movement and chewing.

The principle of operation is to fix (glue) the surface of orthodontic devices (brackets) to the tooth enamel.

MODE OF APPLICATION

- 1) Tooth preparation: clean the surface of the tooth enamel, remove dirt from the surface of the teeth, rinse with water; Isolate the teeth, no oil and moisture, dry the surface of the teeth.
- 2) Acid etching of the tooth surface: apply the etching gel to the tooth surface for about 50-60 seconds, rinse thoroughly with water, dry, acid etching is performed if, after drying, the tooth surface becomes chalky. If there is no chalk deposit, increase the acid etching time.
- 3) Applying Primer: Using a small brush, apply a thin layer of primer to the surface of the tooth, and then apply a thin layer to the base of the bracket. (when using the Orthodontic Adhesive kit for fixing Blue glue band cement Series rings, the primer can be taken from other kits of the manufacturer)
- 4) Applying adhesive: apply a small amount of paste to the base of the bracket, forming a thin layer. Glue the bracket to the desired location on the tooth. Adjust the position of the bracket, applying gentle pressure for 5-10 seconds to prevent the bracket from slipping. Remove excess liquid and paste.
- 5) Light curing: for metal brackets, light curing with a curing lamp is carried out for 10 seconds on each side of the bracket. For ceramic brackets, illumination is carried out with a polymerization lamp from above for 10 seconds. After polymerization, an arch can be applied. Standard curing lamps can be used.

POSSIBLE SIDE EFFECTS

Not identified

WARNINGS

Before using a medical device, the dentist must read the warning information given in the instructions for use.

Orthodontic adhesives are supplied in non-sterile form and cannot be sterilized.

If the packaging is damaged, medical products cannot be used.

The dose of orthodontic adhesive is intended for single use and cannot be reused.

The safety and effectiveness of orthodontic adhesives in pregnant women has not been established.

It is **necessary** to avoid prolonged contact of the adhesive with the skin and mucous membrane of the oral cavity. Rinse thoroughly with water after contact.

PRECAUTIONARY MEASURES

- 1) Avoid contact of pickling agent containing phosphoric acid with eyes or skin. In case of contact, wash immediately with soap and water.
- 2) The tooth surface is covered with liquid (primer), exposure to light for 2-3 seconds can increase the consistency of the liquid to prevent the bracket from positioning after moving.
- 3) Some patients may experience allergic reactions to uncured resin; contact with gums or other soft tissue should be avoided. If contact occurs with skin, wash immediately with soap and water and consult a physician for treatment if necessary.
- 4) This material is sensitive to light and should be kept away from the light source.
- 5) The bonded orthodontic bracket must be of high quality and must be handled by a qualified dentist.
- 6) The material is transported closed and away from sunlight.

STORAGE AND TRANSPORTATION

Orthodontic adhesive should be stored in its original packaging at a temperature between 0 and 28 °C.

Materials should not be exposed to elevated temperatures or intense light.

Do not damage the integrity of the packaging.

The medical product is transported by all types of covered transport, under conditions that ensure the safety of packaging and products, in accordance with the cargo transportation rules in force for this type of transport, at a temperature from 0 to 28 °C.

During transportation and storage, medical devices do not have a negative impact on humans and the environment.

MANUFACTURER'S WARRANTY

Orthodontic adhesives are not subject to periodic maintenance and routine repairs.

The shelf life of the medical device is 24 months from the date of manufacture.

The expiration date is indicated on the packaging of the medical product; It is not allowed to use the product after the expiration date.

The manufacturer guarantees the quality, effectiveness and safety of orthodontic adhesives provided that the consumer complies with the conditions of storage, transportation, disposal and instructions for use of the medical product during the shelf life.

SYMBOLS USED IN MARKING

CE 0197

CE marking (in accordance with EU Directive 93/42/EEC) with identification of the notified body

Representative in the EU

Manufacturer

LOT
REF

Batch number

Catalog number

Use before

Avoid exposure to sunlight

Refer to instructions for use

Temperature limit

Keep away from moisture

Warning, please refer to the instructions in the app

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