SEELER & DR. FELLMETH

NOTARE

NOTARE SEELER & DR. FELLMETH · Q7, 23 · 68161 MANNHEIM

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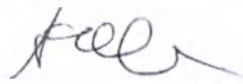
i.v.

Andrea Weber

Manager Global Regulatory Affairs

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For USA: Elecsys CleanCell M

English

Reagent handling: Ready for use. Avoid foam formation.

Storage and stability: on the MODULAR ANALYTICS E170 and **cobas e** analyzers: 21 days

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Warning

H290 May be corrosive to metals.

H315 Causes skin irritation.

H319 Causes serious eye irritation.

Prevention:

P234 Keep only in original packaging.

P264 Wash skin thoroughly after handling.

P280 Wear protective gloves/ eye protection/ face protection.

Response:

P332 + P313 If skin irritation occurs: Get medical advice/attention.

P337 + P313 If eye irritation persists: Get medical advice/attention.

P390 Absorb spillage to prevent material damage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

For US users: Warning. Bottles contain potassium hydroxide solution; corrosive. In case of contact, flush areas with copious amounts of water. Get immediate medical attention for eyes, or if ingested.

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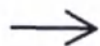
FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS, COBAS E and ELECSYS are trademarks of Roche

Additions, deletions or changes are indicated by a change bar in the margin.

© 2018, Roche Diagnostics

Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

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