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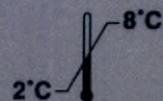
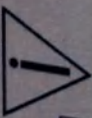
EN
NSE
713-330_R01
S1R190

Safety Precautions

CAUTION: This product contains human serum and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection.

Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens, Biosafety Level 2 or other appropriate biohazard practices should be used for materials that contain or are suspected of containing infectious agents.

The human-sourced material used in the CAL A, CAL B, CAL C, CAL D, CAL E, and CAL F is derived from donor units tested and found nonreactive to HBsAg, anti-HBc, anti-HDV, and anti-HIV.

The following warnings and precautions apply to CAL A, CAL B, CAL C, CAL D, CAL E, and CAL F:

Contains sodium azide
EL-H032
PS01

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this Package Insert.

For laboratory procedures, see only.

NAME

NSE Calibrators (also referred to as NSE Cals)

INTENDED USE

The NSE Calibrators are for the calibration of the Alinity i system when used for the qualitative determination of neuron-specific enolase (NSE) in human serum.

For additional information, refer to the NSE reagent package insert and the Alinity i-series Operations Manual.

CONTENTS

The CAL A contains phosphate buffer with protein (bovine) stabilizer.
The CAL B, CAL C, CAL D, CAL E, and CAL F contain human NSE in phosphate buffer with protein (bovine) stabilizer.

Preservatives: sodium azide and antimicrobial agent.

The calibrators are at the following target concentrations:

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