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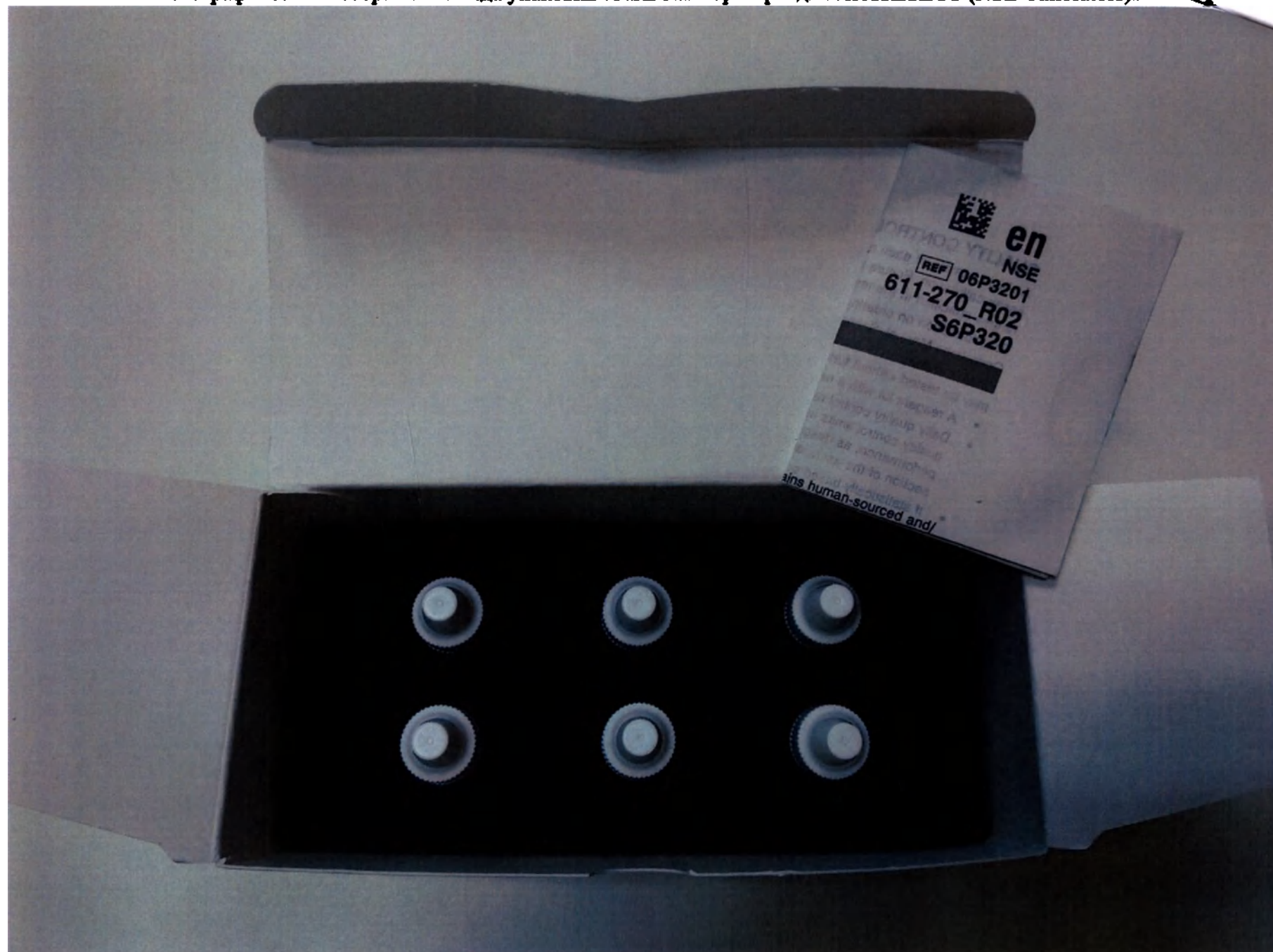
NSE Calibrators FOR USE WITH ARCHITECT

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this Package insert. For laboratory professional use only.

NAME
NSE Calibrators (also referred to as NSE Cals)

INTENDED USE
The NSE Calibrators are for the calibration of the ARCHITECT System when used for the quantitative determination of neuron-specific enolase (NSE) in human serum. For additional information, refer to the NSE reagent package insert and the ARCHITECT System Operations Manual.

CONTENTS

Safety Precautions

CAUTION: This product contains human-sourced and/or potentially infectious components. Refer to the CONTENTS section of this package insert. No known test method can offer complete assurance that products derived from human sources or inactivated microorganisms will not transmit infection. Therefore, all human-sourced materials should be considered potentially infectious. It is recommended that this product and human specimens be handled in accordance with the OSHA Standard on Bloodborne Pathogens. Biosafety Level 2 or other appropriate biosafety practices should be used for materials that contain or are suspected of containing infectious agents.¹⁻⁴

The human-sourced material used in the CAL B - CAL F was derived from donor units tested and found to be nonreactive for

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611285F02

www.corelaboratory.abbott/IFU
REF 06P3201 NSE Cals

NSE Calibrators

FOR USE WITH
ARCHITECT

			ng/mL µg/L
CAL A	1 x	8.0 mL	0.0
CAL B	1 x	4.0 mL	4.0
CAL C	1 x	4.0 mL	20.0
CAL D	1 x	4.0 mL	80.0
CAL E	1 x	4.0 mL	200.0
CAL F	1 x	4.0 mL	400.0

CONTAINS: AZIDE

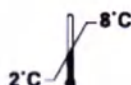
Abbott GmbH
Max-Planck-Ring 2
65205 Wiesbaden
Germany
+49-6122-580

PRODUCED FOR ABBOTT BY

Fujirebio Diagnostics, Inc.
201 Great Valley Parkway
Malvern, Pennsylvania 19355, USA

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
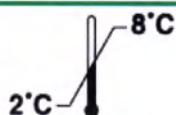
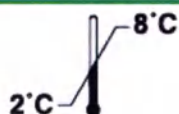
IVD REF 06P32C
4.0 mL
20.0 ng/mL

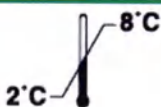
Калибратор D (ARCHITECT NSE Calibrator D)

NSE**FOR USE WITH ARCHITECT****CAL D****CONTAINS: AZIDE****IVD****REF** 06P32D
4.0 mL
80.0 ng/mL320-415_R01
ABBOTT
65205 Wiesbaden, Germany**LOT**

Калибратор E (ARCHITECT NSE Calibrator E)

NSE**FOR USE WITH ARCHITECT****CAL E****CONTAINS: AZIDE****IVD****REF** 06P32E
4.0 mL
200.0 ng/mL320-420_R01
ABBOTT
65205 Wiesbaden, Germany**LOT**

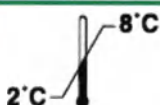Калибратор F (ARCHITECT NSE Calibrator F)

NSE**FOR USE WITH ARCHITECT****CAL F****CONTAINS: AZIDE****IVD****REF** 06P32F
4.0 mL
400.0 ng/mL320-425_R01
ABBOTT
65205 Wiesbaden, Germany**LOT**

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