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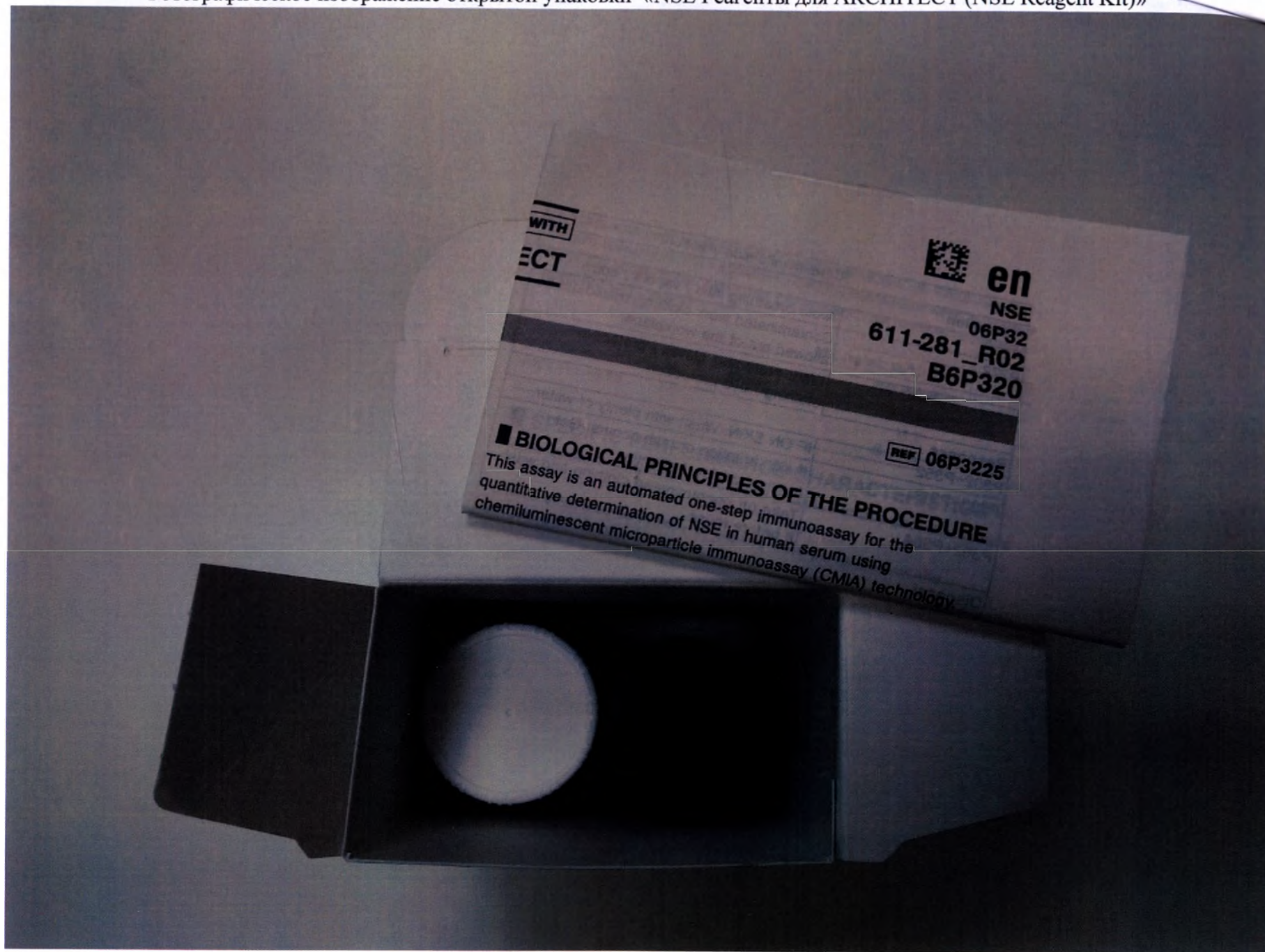
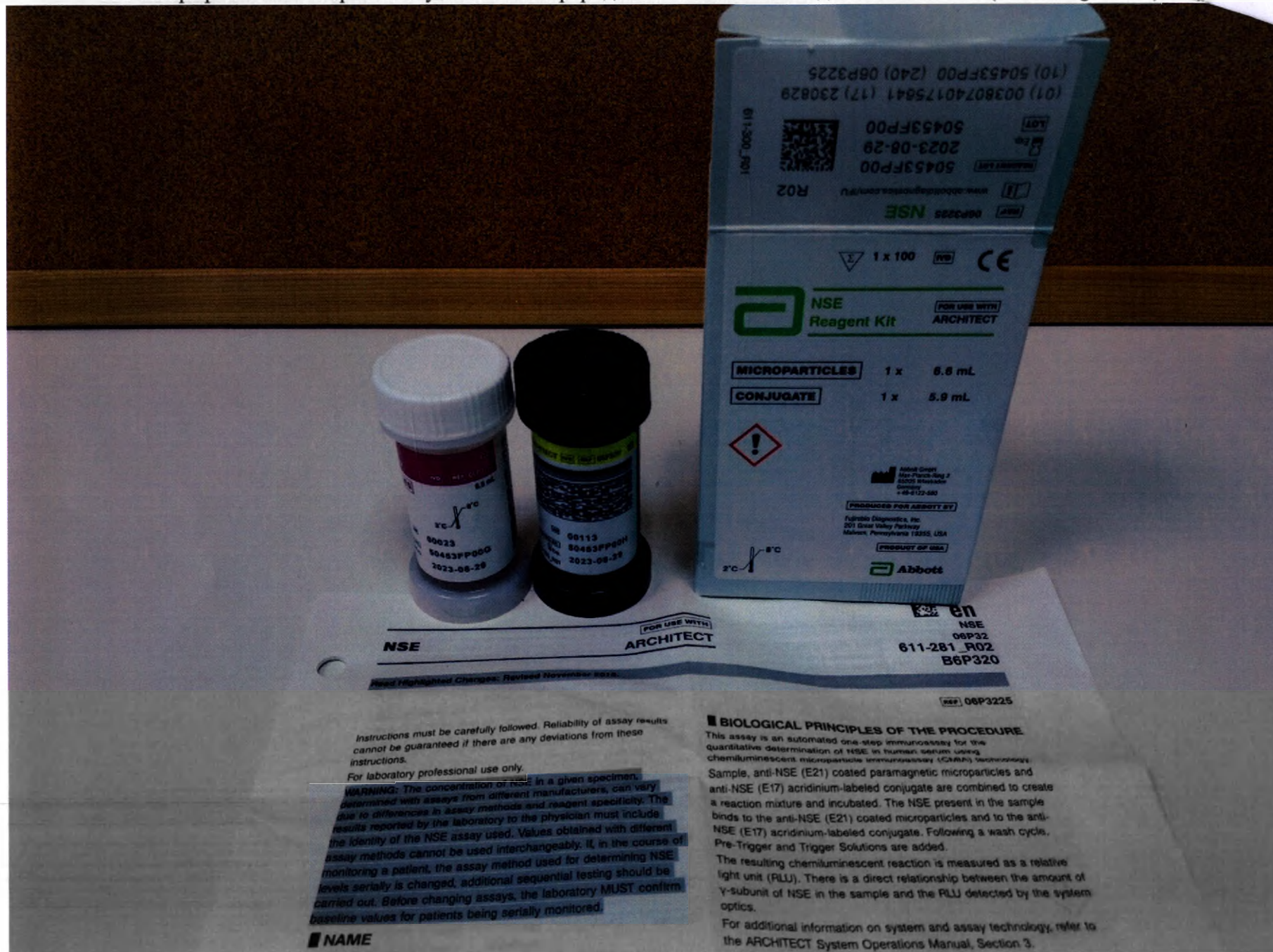
Instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from these instructions.

For laboratory professional use only.

WARNING: The concentration of NSE in a given specimen, determined with assays from different manufacturers, can vary due to differences in assay methods and reagent specificity. The results reported by the laboratory to the physician must include the identity of the NSE assay used. Values obtained with different assay methods cannot be used interchangeably. If, in the course of monitoring a patient, the assay method used for determining NSE levels serially is changed, additional sequential testing should be carried out. Before changing assays, the laboratory MUST confirm baseline values for patients being serially monitored.

NAME

BIOLOGICAL PRINCIPLES OF THE PROCEDURE

This assay is an automated one-step immunoassay for the quantitative determination of NSE in human serum using chemiluminescent microparticle immunoassay (CMIA) technology. Sample, anti-NSE (E21) coated paramagnetic microparticles and anti-NSE (E17) acridinium-labeled conjugate are combined to create a reaction mixture and incubated. The NSE present in the sample binds to the anti-NSE (E21) coated microparticles and to the anti-NSE (E17) acridinium-labeled conjugate. Following a wash cycle, Pre-Trigger and Trigger Solutions are added. The resulting chemiluminescent reaction is measured as a relative light unit (RLU). There is a direct relationship between the amount of γ -subunit of NSE in the sample and the RLU detected by the system optics.

For additional information on system and assay technology, refer to the ARCHITECT System Operations Manual, Section 3.

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