Consultronix SA
ul. Racławicka 58, 30-017 Kraków
NIP: 676-01-15-479
Raiffeisen Bank Polska SA Kraków
27 1750 1048 0000 0000 0107 9597

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Consultronix S.A. (Poland)

TABLE OPHTHALMOLOGIC ELECTRIC the TAVO-xyz series with accessories

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www.cxsa.pl

CONTENT

- | | |
|-----|---------------------------------|
| 1. | General instructions |
| 2. | Appointment |
| 3. | Technical characteristics |
| 4. | Completeness |
| 5. | Device and principle of work |
| 6. | Indication of security measures |
| 7. | Installation order |
| 8. | Preparation for work |
| 9. | Operating procedure |
| 10. | Maintenance |
| 11. | Manufacturer's guarantees |

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www.cxsa.pl

1 . GENERAL INSTRUCTIONS

1.1 . Operation of TABLES OPHTHALMOLOGIC ELECTRIC the TAVO-xyz series (table-tops TAVO-x, TAVO-s, TAVO-b, TAVO-u, TAVO-v, TAVO-c) with accessories before acquaintance with the present instruction isn't allowed.

1.2 . The present instruction is intended for acquaintance with service regulations, and also for the management at maintenance, repair, transportations and storage of Ophthalmologic tables (a table-top of TAVO-s, TAVO-b, TAVO-u, TAVO-v, TAVO-c) with accessories.

1.3 . The volume of data and the illustrations, given in this instruction, provides the correct operation of tables and all of them knots.

2 . APPOINTMENT

TABLE OPHTHALMOLOGIC ELECTRIC the TAVO-xyz series (table-tops TAVO-x, TAVO-s, TAVO-b, TAVO-u, TAVO-v, TAVO-c) with accessories are intended for placement on them various devices and providing convenient conditions during the work with them.

3 . TECHNICAL CHARACTERISTICS

Voltage of a network	230~ ±10% 50 Gts
Safety locks	5x20 2 x 6.3. AT
Power	600BA
Max. loading	F max=1200N according to ca.120 kg
Minimum overlapping	495 mm
Movement	250mm
Speed	10-16 mm/sec.

CLASSIFICATION BY the UT60601-STANDARD from 1/08/90

Protection type at direct and indirect contact the	Class I
Extent of protection pripyamy and indirect contact	Type B
Extent of protection against humidity	Usual (unprotected case)
The sterilization method	Disinfection isn't required
Extent of protection against combustible substances	Without protection
Working conditions	Continuous work

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www.cxsa.pl

4. COMPLETENES

I TABLE OPHTHALMOLOGIC ELECTRIC the TAVO-xyz series with accessories:

- 1 Elevator the electromechanical telescopic.
- 2 Basis panel the metal.
- 3 Plate metal under a table-top.
- 4 Cable network.
- 5 Control panel.
- 6 Screws for control panel fixture – 2 pieces
- 7 M6x35 screws with a washer for fastening of the telescopic elevator to the basis – 4шт.
- 8 M6x20 screws for table-top fastening – 4 pieces.
- 9 Safety locks – 2 pieces.
- 10 Key six-sided.
- 11 Maintenance instruction.
- 12 Wheels of 4 pieces.
- 13 Table-top.

II Additional accessories

- 1 Table-top TAVO-x – for one device.
- 2 TAVO-s table-top – for one device.
- 3 TAVO-b– table-top for two devices.
- 4 TAVO-u– table-top for two devices, in shape "banana".
- 5 TAVO-v– table-top for three devices.
- 6 TAVO-c– table-top threefold table-top.
- 7 Legs under the basis.
- 8 Wheels under the basis.
- 9 The holder for the computer.
- 10 The holder for the printer.
- 11 The holder for the keyboard.

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biuro@cxsa.pl

www.cxsa.pl

TABLE OPHTHALMOLOGIC ELECTRIC the TAVO-xyz series (table-tops TAVO-x, TAVO-s, TAVO-b, TAVO-u, TAVO-v, TAVO-c) with accessories .

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www.cxsa.pl

5 . DEVICE AND PRINCIPLE OF WORK

TABLES the OPHTHALMOLOGIC ELECTRIC TAVO-xyz series (table-tops TAVO-x, TAVO-s, TAVO-b, TAVO-u, TAVO-v, TAVO-c) with accessories have the metal basis on which the telescopic electromechanical elevator executed in the form of a column fastens (TAVO-c I establish on two telescopic electromechanical elevators with supports). To the lower part of basis to wheels fasten. The case of a column double also is made from different in section the aluminum profiles. On the case of a column sockets are fixed, and inside there is the electric motor setting in linear motion on a vertical a metal rod that leads to column lengthening. In the top part of a column the horizontal plate for table-top fastening is established. The control panel change of height of a table fastens to a table-top in a convenient place and has two buttons "up" and "down".

6. CLASSIFICATION BY the UT60601-STANDARD from 1/08/90

Protection type at direct and indirect contact the	Class I
Extent of protection pripyamy and indirect contact	Type B
Extent of protection against humidity	Usual (unprotected case)
The sterilization method	Disinfection isn't required
Extent of protection against combustible substances	Without protection
Working conditions	Continuous work

7 . INSTALLATION ORDER

- 7.1 The telescopic elevator, establish vertically, a bottom up, on a pure equal place so that the metal plate of fastening of a table-top was below.
- 7.2 The panel of basis put from above, on the telescopic elevator also upside down. Openings have to be from above for fastening of wheels. The long party of basis has to settle down parallel to the short party of a plate for table-top fastening.
- 7.3 Openings of the telescopic elevator combine with openings in the basis. The basis fix by means of screws, using a key six-sided.
- 7.4 Establish wheels.
- 7.5 Table turn. Table put on wheels.
- 7.6 The control panel attach on a table-top.
- 7.7 Table-top establish on a plate of the telescopic elevator and fix screws.
- 7.8 Пульт managements connect to the socket on the telescopic elevator.
- 7.9 The device install on a table. The device connect to the network socket in the top part of the telescopic elevator by means of the cable which is included in the package of the device.
- 7.10 Cable network Connect to the bottom socket of the telescopic elevator.

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www.cxsa.pl

8 . PREPARATION FOR WORK

Fork of a network cable insert into the socket.

9 . OPERATING PROCEDURE

The control panel establish the necessary height of a table.

10 . MAINTENANCE

Procedure of service is carried out after a table detachment from a network. You address to specialists in product installation if malfunctions arose.

Replacement of safety locks

You execute the following actions for replacement of safety locks. Safety locks are in the holder located in the block of tension of a telescopic rack.

You extend the regulator in the block of tension and safety locks replace the fused.

New safety locks establish. The regulator return into place.

Table connect to a power supply network.

WARRANTY CARD

MARK ABOUT SALE

Category of a product _____

Producer _____

Model _____

Serial number/article _____

Sale date _____

Organization seller _____

Signature of the seller _____

The product is checked, no damages has.

I am acquainted and agrees with service regulations.

Signature of the buyer _____

Guarantee period – 12 months.

Concerning a guarantee to address:

JSC Dzhapan Medikal Produkts (JSC DZHAMP), legal/actual address: 117981, Moscow, Vernadsky Ave 41, ph./fax +7(495) 432-3800, 430-8488

ATTENTION! You watch the correct filling of the warranty card. Serial number the product/article has to correspond in accuracy to number in the coupon. All columns of the coupon (except for columns for the guarantee center) have to be filled.

At inexact or incomplete filling of the coupon the guarantee on a product is considered lost.

The guarantee of the producer is carried out according to the Civil Code of the Russian Federation in accordance with general practice.

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biuro@cxsa.pl

www.cxsa.pl

The product shouldn't bear traces of mechanical or thermal influences, and also traces of influence of various chemical agents. Operation of a product is allowed only according to the operation manual to a product.

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