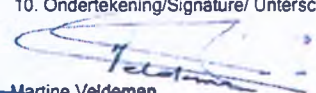
B 00196121

B 00196121

APOSTILLE

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"APPROVED"

Ontex BVBA

Group Sustainability & Scientific Affairs Manager

(position)

Bart Waterschoot

(name)

(signature)

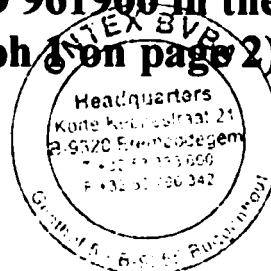
« 29 » December 2017

«day» month (numerals)

Exploitation documentation

DIRECTIONS FOR USE OF THE MEDICAL PRODUCT

**Medical absorbing products for ND 961900 in the following
dimensions (see Paragraph 1 on page 2)**


**«Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium
2018**

I, the undersigned, Henri Van den Bossche, notary holding residence at Buggenhout, confirm that the signature on this document is the signature of Mr. Bart Waterschoot and is well known by me.

This confirmation of the signature only concerns its authenticity and does not imply a certification by the notary of the capacity or the powers of the aforementioned signatory.

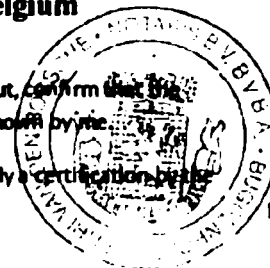Done on the 17th of April 2018 at Buggenhout.

1. Name of the medical product (hereinafter "MP")

Medical absorbing products for ND 961900 in the following dimensions (hereinafter "Products"):

1. Adult pants, size XS (Extra Small).
2. Adult pants, size S (Small).
3. Adult pants, size M (Medium).
4. Adult pants, size L (Large).
5. Adult pants, size XL (Extra Large).
6. Adult diapers, size XS (Extra Small).
7. Adult diapers, size S (Small).
8. Adult diapers, size M (Medium).
9. Adult diapers, size L (Large).
10. Adult diapers, size XL (Extra Large).

2. Information on the medical product

Name of the operating company: Ontex BVBA

Registered address of the company: «Ontex BVBA», Genthof 5, 9255 Buggenhout, Belgium.

Name of the manufacturing companies:

1. «Ontex BVBA», Belgium.
2. «Ontex BVBA Eeklo», Belgium.
3. «Ontex Tüketim Ürünleri Sanayi Ve Ticaret Anonim Sirketi», Turkey.
4. «Ontex Sante France SAS», France.
5. «Ontex CZ s.r.o.», Czech Republic.
6. LLC «Ontex RU», Russia.

Registered address of the company:

1. Genthof 5, 9255 Buggenhout, Belgium.
2. Korte Moeie 53, 9900 Eeklo, Belgium.
3. Selimpasa Mah. 6093 Sokak No. 78 Silivri, Istanbul, Turkey.
4. Quai du Rivage, 62119 Dourges, France.
5. Turnov, Vesecko cp. 491, PSC 51101, Czech Republic.
6. LLC «Ontex RU», 142400, Moscow Region, Noginsk district, ~56km+900m of Highway M-7 „Volga“, 1, bld. 3, Russia.

Company profile of the manufacturer:

Production, sale and distribution of hygienic and medical products both under own brands and retail brands.

nsions (see Paragraph

Medical absorbing products for ND 961900 in the following dimensions (see Paragraph 1 on page 2) – single use.


Photo of a diaper pant in the expanded and collapsed views.


A black and white photograph of a folded white cloth or towel, showing its texture and folds, set against a dark background.


The image shows a top-down view of a single-use adult diaper. The central area, which corresponds to the perineal region, is brightly illuminated with a white, glowing light, indicating the presence of moisture. The rest of the diaper, including the leg cuffs and side panels, appears in a dark gray tone against a black background.

Schematic representation of a diaper pant for adults

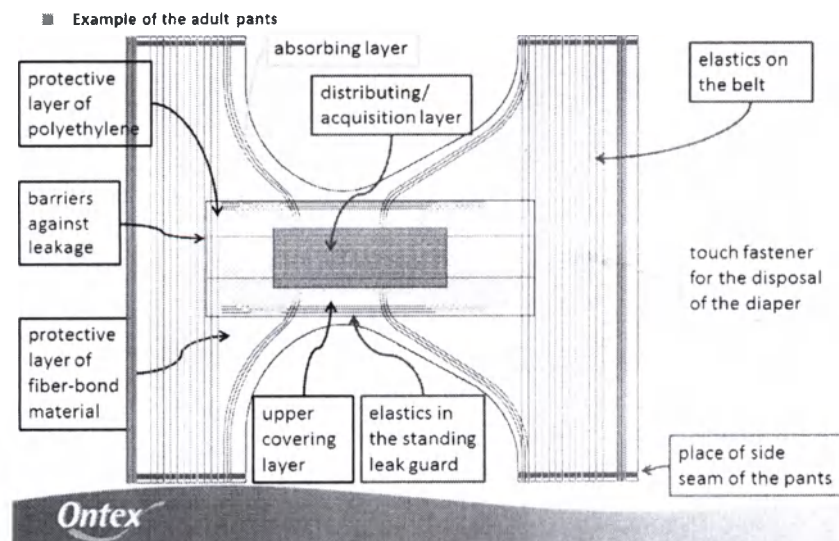Schematic representation of adult diaper

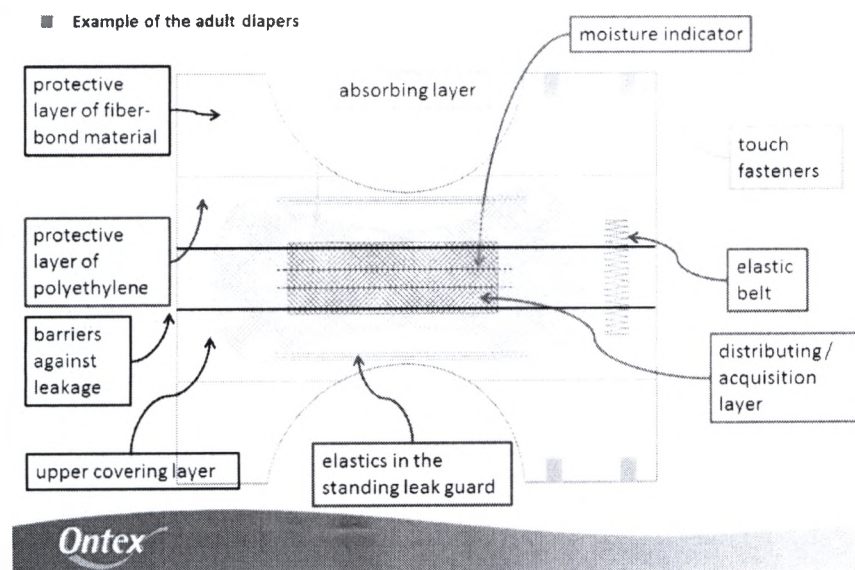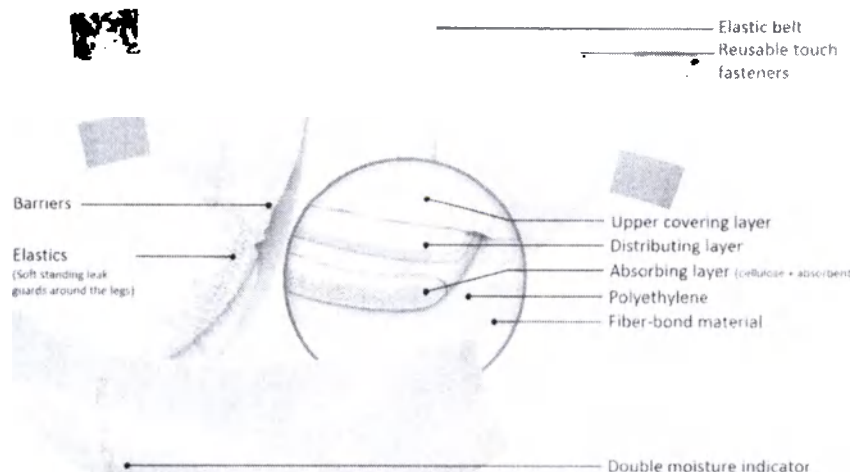All variants of products (adult diapers) have a double indicator of saturation.

3.2 Designation of the product given by the producer

The products are designed for patients suffering from incontinence moderate and severe. Products protect the clothing and pants from soaking and contamination by bio effluents. The products are designed for use in conditions of therapeutic and preventive medical institutions and at home.

3.3 Indication for use of the products

Products are used in cases of urinary incontinence of various degrees of incontinence. Depending on the amount of absorb the fluid products of the following types:

- products (adult diapers, adult pants) average degree of incontinence (sizes XS (Extra Small), S (Small), M (Medium));
- products (adult diapers, adult pants) for severe incontinence (sizes L (Large) XL (Extra Large)).

3.4 Contra indications for use of the products:

- dermatopathy of externalia;
- idiosyncrasy for the materials of the products, expressed hypersensitivity reactions.

3.5 Side effects of the products

Side effects of the product were not identified by the manufacturer.

3.6 Scope of application of the product

Products are used at general healthcare institutions as well as domiciliary (in everyday life).

3.7 Use instructions of the product

Before application of the product one should:

- wash hands before and after the change of the product;
- change the product every 3-4 hours or more often, if the product soaks quicker;
- touch fasteners should not disturb the belly and legs, that is normal blood circulation should not be disturbed;
- do not tear the used product, don't wash it, don't throw to the sewage line;
- do not cut or change the form of the product because it may influence its effectiveness;
- wash genitals or clean the intimate zone with the help of special napkins for intimate hygiene at each change of the product.

3.8 Methods of product application:

- Unwrap the product;
- Turn the lying patient on the side, put the narrower part of the diaper between the patient's legs. Then lay the patient on the back in such way that the rare wider part of the diaper was under the clunis.
- Fix the product with the help of touch fasteners from the both sides of the diaper or wear like the underwear (as pants);
- Only intended for single use;
- Wrap the used product into the paper napkin and throw to the dustbin.

3.9 Completeness

Products are supplied completed in accordance with the engineering and technical documents and order requirements. Completeness of the product delivery should comply with the information specified in the Table No 1.

Table No 1. Completeness of the delivery

No	Name	Number (pcs.)
1.	Medical absorbing products for ND 961900 in the following dimensions (see Paragraph 1 on page 2)	from 8 to 30

Products are manufactured under private labels on request. The list of trademarks covered by this manual is listed in APPENDIX 1.

All products are produced under the control of the company Ontex BVBA unified design documentation, technology, using the same materials, regardless of brand.

The list of products of private brands can be augmented while maintaining the technology of manufacture of products and its compliance with the operational and technical documentation of the manufacturer.

4. Technical characteristics of the medical product

4.1 Size of the products

Size of the products (diapers and pants) should comply with the sizes specified in the Table No 2 and Table No 3.

Table No 2. Sizes of diapers for adults

Name of the size	Sizes in accordance with the identifying signs				
	Extra Small (XS)	Small (S)	Medium (M)	Large (L)	Extra Large (XL)
Waist line*, cm	40-70	50-90	70-130	100-160	120-170
Length/Width of the diaper*, mm	610X390	700X530	800X680	980X800	980X850
Length of the absorbing layer*, mm	510	560	630	725	725
Width of the absorbing layer in the front part*, mm	225	300	280	280	305
Width of the absorbing layer in the rare part*, mm	250	330	300	300	305
Width of the absorbing layer in the middle part*, mm	150	150	160	160	190
Total width of the touch fasteners*, mm	60	60	63	63	70
Distance between the upper and lower touch fastener*, mm	78	70	78	97	97
Width of the inner frills*, mm	35	40	40	40	40
Length/width of the elastic belt*, cm	19X4	19X4	19X4	19X4	19X4

*acceptable variations on target indicators specified in the table are of $\pm 10\%$

Table No 3. Sizes of pants for adults

Name of the size of the products	Sizes					Width of the inner frills*, mm
	Waist line*, cm	Length/Width *, mm	Absorbing layer (length/width) *, mm	Width of absorbing layer in the middle*, mm	Width of absorbing layer in front and rare parts*, mm	
XS	50-70	630X175	510X100	150	225	35
S	60-90	800X582	480X190	165	300	40
M	80-120	830X728	480X190	160	290	35
L	100-145	900X811	480X190	170	280	40
XL	130-170	900x925	400x190	190	305	45

*acceptable variations on target indicators specified in the table are of $\pm 10\%$

4.2 Materials of the products

Materials specified in the Table No 4 are used for manufacturing of the products.

Table No 4. Materials of the product

Layer of the products	Materials of the products
Upper covering layer	Hydrophilic fiber-bonded material of polypropylene under trademark «Regent», LLC «First Chemical Corporation», Russia
Distributing layer	Fiber-bonded material of polypropylene under trademarks «DRY WEB TDL2», «DRY WEB T28», «DRY WEB TDL3 G», «DRY WEB T19», «DRY WEB T12G» produced by «TWE Meulebeke BVBA», Belgium
Absorbing layer	Cellulose bleached without chlorine under trademark «GP Cellulose» produced by «GP Cellulose LLP», USA Polymer superabsorbent under trademark «LG Chem GS-3070HD» produced «LG Chem, Ltd», the Republic of Korea
Protective layer	Polyethylene film under trademark «Hyfol PP Plus», produced by «RKW SE», Germany. Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol Ros», Russia.
Touch fastener	Self-adhesive plastic band (polypropylene) under trademark «AVERY DENNISON F5 2400 Wave C» produced by «AVERY DENNISON Belgie b.v.b.a», Belgium
Standing leak guard	Hydrophobic fiber-bonded material of polypropylene under trademark «Avgol» produced by LLC «Avgol Ros», Russia.
Elastic belt	Plastic band (polystirol) under trademark «3M EBL Soft» produced by «3M Deutschland GmbH», Germany.
Elastics	Elastomer polyurethane thread under trademark "Spandex" produced by «HYOSUNG SPANDEX (JIAXING) CO. LTD», the Republic of Korea.
Double moisture indicator	Hot melt glue (moisture indicator) under trademark «Swift Melt 1525» produced by «H.B. Fuller Deutschland GmbH», Germany
Glue	Hot-melt adhesive: Hot melt glue «Bostik» on the base of styrole copolymer under trademarks «H2465-04», «H24256» produced by «Bostik Nederland B.V.», Netherlands.

Layers (seams) of the products should be fastened tightly with the help of thermal processing or hot-melt glue, or other method providing the tightness of the glued layers and seams of the diapers. Seams should be smooth.

4.3 Weight and functional performance of the product

Weight and functional performance of the product (diapers or pants) are specified in the Table No 5, No 6

Table No 5. Weight and absorptive capacity of the products

Name of the size of the products	Specification of the diapers for adults			Specification of the pants for adults		
	Total weight, not less than, g	Full absorption, g, not less than	Weight of the absorbent, g, not less than	Total weight, not less than, g	Full absorption, g, not less than	Weight of the absorbent, g, not less than
XS	50	1000	5	70.6	1000	5
S	79.5	1000	5	66	1000	5
M	86	1450	10	70.5	1450	10
L	96	2000	8	75	2000	8
XL	134	2800	6	80.5	2800	6

Table No 6. Functional performance of the products

Name of the size of the Product	Specification of the diapers for adults			Specification of the pants for adults		
	Rewet, no more, g	Absorptive capacity after centrifuge, not less than, g	Absorption time, not less than, cm ³ /s	Rewet, no more, g	Absorptive capacity after centrifuge, not less than, g	Absorption time, not less than, cm ³ /s
XS	4,4	120	2,3	4,4	120	2,3
S		120			120	
M		220			220	
L		500			500	
XL		560			560	

5. Classification of the medical product

In accordance with the degree of potential risk of the application in medical purposes medical product " Medical absorbing products for ND 961900 in the following dimensions (see Paragraph 1 on page 2)" belong to the 1 class according to Annex 2 of the Order No 4n "On approval of nomenclature classification of the medical products" as

of June 06, 2012 of the Ministry of Health of the Russian Federation with subsequent amendments.

Type of contact with human body: brief contact with intact skin and mucous membranes of the human body.

6. Marking of the products

Ready-made products are marked on the package or on the labels; place of marking application is specified in the design documents.

Marking of individual packing contains:

- The name of the manufacturing company and (or) its trademark;
- The address of the manufacturing company, including the country of origin;
- The name of the products;
- The size of the product;
- The number of products in the package;
- The materials used in the manufacture of products;
- The batch number on the side of the packaging in the format: LOT Line: hour/day/year/minute/month;
- The date of manufacture (refer to batch number);
- The usage instruction;
- The date of the expiration date (see batch number: EXP month/year);
- Method of application (drawings and/or words);
- The purpose of the product;
- The contraindications to the use of the product;
- The storage conditions;
- The information on recycling products;
- The bar code of product (if available);
- The information about the registration of the product;
- The information about what the product is intended for single use;
- The name of the importing company on the territory of the Russian Federation;
- The address of the importing company on the territory of the Russian Federation;

The of signs and pictograms on the label:

- "Do not reuse"

- "Protect from moisture"

- Protect from sunlight

- Disposal: do not dispose into the sewers

- The Loop

- the level of absorbance

It is allowed to mark other data including information about consumer properties and the use of the products.

Data are printed or embossed that ensure their readability and preservation (stability) during the storage, transportation, handling and intended usage of the products.

Marking of the transportation tare with the following information:

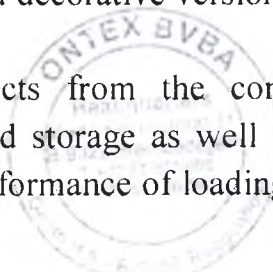
- the name of the products;
- the article of the products;
- the number of the products in the transportation tare;
- the name and the address of the manufacturing company;
- the name and the address of the company authorized by the manufacturer, of the importer;
- the bar code of the transportation tare (if available);
- the date of manufacturing of the products (month, year);
- the expiry date;
- conditions of storage, transportation, recycling possibility;
- icons and symbols according to the legislation requirements of the consignee country.

7. Packaging

Several pieces of products are packed into the bags made of polyethylene which protects the products in transportation and storage. Seams in the bags of polyethylene should be welded.

Products of one group, structure, linear sizes, technical and decorative version are packed in one bag.

Package should provide the protection of the products from the contamination, mechanical damage, weather impact in transportation and storage as well as the most complete usage of vehicle capacity (tonnage) and easy performance of loading-unloading operations.

Boxes of corrugated cardboard and other packing materials ensuring the protection of the products may be used as transportation tare.

It is allowed to use the other packing materials including those that are manufactured by the manufacturing company which provide the necessary durability and protection of the products in transportation and storage.

8. Transportation and storage

Products are transported by all kinds of vehicles in accordance with the cargo transportation rules operating on the transport of given type at a temperature of from -50 °C to +50 °C, a relative humidity of 80%. Material is not hazardous at any kind of transportation.

Ready-made products should be stored in sheltered storage facilities. Storage conditions should exclude the impact of moisture and aggressive environment on the products.

To preserve the quality product should be preserved in original package at the room temperature from +5 °C to +25 °C and relative humidity no more than 70 % in childproof and moisture protected sites. Don't expose the products to direct sunlight.

The temperature of using from +10 °C up to +45 °C.

9. Safety requirements

Work with products does not require special safety measures.

Structure of the products does not contain materials which are hazardous for the health of the person during the operation.

Products are explosive safe, inflammable when exposed to the open fire without explosions.

In case of fire materials of the products and package should be extinguished with foam, extinguishing powder, carbon dioxide, water, sand.

10. Requirements for environment protection

Requirements for environment protection during the application of the product

Products do not release toxic substances to the environment and do not impact (at direct contact) the health of the person and the environment during the transportation, storage and operation. Products are explosive safe, inflammable when exposed to the open fire without explosions. Special environment protective measures from harmful influence are not required during the operation.

Requirements for environment protection and disposal of the products

The main possible harmful impact on the environment is the contamination of the atmosphere, soils and waters of the inhabited areas as a result of:

- unorganized waste landfilling on the territory of manufacturing company or outside its territory;
- random landfilling in unspecified places.

Products and materials used for their manufacture do not impose risk for the health of people and for the environment both during the operation and after its completion.

11. Disposal

It is allowed to dispose waste and materials during the manufacture by contract with company which has the required licence. (To dispose with the help of the method approved by the local legislation).

The product can be disposed as household waste after its usage. It is not allowed to throw the used products into the toilet.

Products are classified under the waste class A – epidemiological waste safe according SanPiN 2.1.7.2790-10.

12. Maintenance and repair

Not applicable. Products are not objects of maintenance and repair.

13. Warranty liabilities

Manufacturer assures the compliance of the products with the requirements of the product specifications and guarantees the replacement of the products in case of nonconformity.

Shelf life: 5 years from the date of manufacture indicated on the package. Date of manufacture is specified with news ticker on the side of the package.

After the expiration date this medical product should be disposed.

14. Information about reclamations

If defects of the products appear during warranty period as well as at primary acceptance of the products, the owner of the product should send a reclamation to the address of the selling organization – request for the replacement according to the form No 1.

Form No 1.


The address where the representative of the company carrying out warranty liabilities should arrive, phone number;		
Date	Short description of the defect	Date of the sending of the reclamation

Send the reclamations to the address:

LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.


Postal address: LLC «Ontex RU», 105064, Moscow, Zemlyanoy val St, 9, Russia.

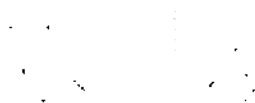Tel.: 8 495 646 92 21; website: www.ontex.ru

The range of private labels is given below:

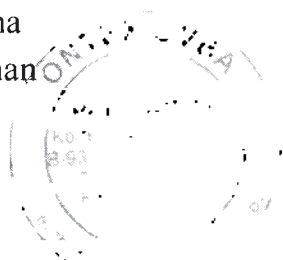- 365 days
- ARO
- BABIKO
- Every day
- First Aid
- Medisan
- Tereza Med
- Nevo
- Luxsan
- Pelena
- Auchan

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