BE IT KNOWN that I Laura Anne Pawley, Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG, a duly authorised Notary Public, duly admitted, sworn and practice throughout England and Wales.

CERTIFY ONLY that Brian Michael Howes, a United Kingdom Citizen, who is well known to me, who, is duly authorised by Andrew Brownhill of Reckitt Benckiser Healthcare (UK) Limited ('the Company') to represent them in this matter, has today caused the annexed Technical File for Contex Lights for use in Russia to be produced to me and that he has represented to me on behalf of the Company that the said document is a true electronic copy of the original document.

SIGNED and sealed at Mezzanine Floor, Epsom Square, Epsom, Surrey, KT19 8AG on the 5th October 2023.

Protocol No: 2023-6101

Laura Anne Pawley

Laura Anne Pawley
Notary Public

Dear Sir / Madam

I Andrew Brownhill of Reckitt Benckiser Healthcare (UK) Limited CERTIFY that the attached Technical File for Contex Lights is a true electronic document for use in the Russian Federation.

Name: Andrew Brownhill

Signature:

Andrew Brownhill
Electronically signed by:
Andrew Brownhill
Reason: I approve this document.
Date: Oct 3, 2023 08:07 GMT+1

Date: 03-Oct-2023

Reckitt Benckiser Healthcare (UK)
Limited

103-105 Bath Road
Slough, Berkshire SL1 3UH
United Kingdom
T +44 (0)1753 217800
F +44 (0)1753 217809

Registered in England No. 00261319. Registered office at 103-105 Bath Road, Slough, Berkshire, SL1 3UH, United Kingdom.

www.reckitt.com

Approved by:

Andrew Brownhill

Electronically signed by:
Andrew Brownhill
Reason: I approve this
document.
Date: Oct 2, 2023 15:17
GMT+1

02-Oct-2023

Andrew Brownhill
Senior Global Regulatory Manager, Intimate Wellness

MAINTENANCE DOCUMENTATION
(Instruction for use)
OF THE MEDICAL DEVICE
Contex® Lights natural rubber latex condoms

Reckitt Benckiser Healthcare (UK) Ltd., UK

Version 1.2

NAME OF THE MEDICAL DEVICE

Contex® Lights natural rubber latex condoms in variations:

1. Contex® Lights natural rubber latex condoms 3 pcs. per pack.
2. Contex® Lights natural rubber latex condoms 6 pcs. per pack.
3. Contex® Lights natural rubber latex condoms 12 pcs. per pack.
4. Contex® Lights natural rubber latex condoms 18 pcs. per pack.
5. Contex® Lights natural rubber latex condoms 30 pcs. per pack.

MANUFACTURER

Reckitt Benckiser Healthcare (UK) Ltd., 103-105 Bath Road, Slough, Berkshire, SL1 3 UH, UK.

MANUFACTURING SITE*

SSL Manufacturing (Thailand) Ltd., Wellgrow Industrial Estate 96 & 100, Moo 5, Bangna-Trad Road KM.36, Bangsamak, Bangpakong, Chachoengsao, 24180, Thailand.

**information about the place of manufacture is presented in the outer side of the secondary/consumer packaging in the format "The country of origin: [country]"*

AUTHORIZED REPRESENTATIVE OF THE MANUFACTURER

Reckitt Benckiser Healthcare LLC, Floor 3, 4 Shlyuzavaya emb., Moscow, 115114, Russia.

Phone: +7 (495) 961 25 65; fax: +7 (495) 961 25 66

PURPOSE OF THE MEDICAL DEVICE

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

POTENTIAL CONSUMERS OF THE MEDICAL DEVICE

Medical Device is intended to be used by laypersons.

MEDICAL DEVICE DESCRIPTION

Contex® Lights natural rubber latex condoms are colorless, smooth, thin, straight walled, teat ended, in silicone lubricant. Made from natural latex.

TECHNICAL CHARACTERISTICS**

Technical parameter	Specification
Condom length, mm	180 ± 10
Condom width, mm (measured at 30 mm from the bead)	52 ± 2
Teat length, mm	Min 11
Thickness, mm	$0.055+0.004/-0.008$
Bead thickness, mm	1-2
Condom weight (after removed from the primary package, W2), g	1.4 ± 0.5

Elongation, %	Median 700
Elongation, % (aged)	Median 600
Tensile strength, mPa	Median 20
Tensile strength, mPa (aged)	Median 17
Freedom from holes	Impermeable

****main technical characteristics are presented in the outer side of the secondary/consumer packaging as: nominal length – 180 mm; nominal width – 52 mm****

PACKAGE CONTENTS OF THE MEDICAL DEVICE***

The medical device is packed in a secondary / consumer package of 3 pcs. a pack, 6 pcs. a pack, 12 pcs. a pack, 18 pcs. a pack or 30 pcs. a pack. It is accompanied by the instruction for consumers that can be enclosed in the package or printed in the inside and / or outside of the package.

*****quantity of condoms is presented in the outer side of the secondary/consumer packaging, the instruction for consumers is printed in the inner side of the secondary/consumer packaging**

LABELLING INFORMATION

Each individual package is labelled with:

- trademark;
- lot number (LOT);
- expiry date (EXP).

Symbols used for labelling and what they mean:

	Use by/Expiry date
	Lot number

Additional information can be printed on the individual package including reference to international standards

The following information is applied to the outer surface of the secondary/ consumer package:

- condom description.
- quantity of condoms in consumer package;
- nominal condom length;
- nominal condom width;
- indication that the condom contains lubricant;
- indication that the lubricant does not contain spermicide.
- indication, that the condom contains natural latex;
- storage guidelines;
- name and address of the manufacturer;
- country of origin (if applicable).
- lot number (LOT);
- expiry date (EXP).

Symbols used for labelling and what they mean:

	Use by/Expiry date
	Lot number
	Contains natural latex
	For single use only
	Caution!

Additional information can be printed on the outer surface of the secondary/ consumer package including reference to international standards.

Outer or inner surface of consumer packages and/or package insert contains prescribing information, which includes:

- reminder to handle the condoms carefully, including when removing them out of the package;
- indications, contraindications, precautions for use, possible side effects;
- method of application;
- necessity to remove condom right after sexual intercourse;
- necessity to exclude petroleum- and oil-based lubricants;
- necessity to use safe additional lubricant if needed;
- necessity to consult a specialist on compatibility with topical medications;
- disposal instructions for used condoms;
- the condom is a disposable product.

INDICATIONS

- prevention of unwanted pregnancy;
- protection from HIV and other sexually transmitted diseases.

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition.

PRECAUTIONS FOR USE

Put a condom on before the penis touches the partner. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. Do not use the condom after the expiration date. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Use the lubricant recommended for use with condoms. Petroleum - and oil-based lubricants (petroleum jelly, baby oil), some pessaries and some topical medicines that are applied to the penis or vagina can damage condoms.

Stop use if you feel discomfort or irritation while using the condom.

Consult a specialist if:

- about the compatibility of condoms with other contraceptives;
- if you are using topical medicines applied to penis or vagina;
- if irritation/discomfort persists after use;
- if you are unsure if you are allergic to latex;
- if a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

For anal sex, if it is necessary use additional lubrication on the outside of the condom.

Always read the instructions for use on the pack of the lubricant before use.

Condoms may present a choking hazard. Keep out of reach of children.

If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows. Use a condom only once. Reuse may increase risk of failure or infection.

None of the contraception methods can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

POSSIBLE SIDE EFFECTS

Condoms are made of natural rubber latex, which, like any component of the condom, might cause allergic reactions including anaphylactic shock if user is allergic to latex.

APPLICATION

Tear the package at the toothed edge and handle the condom carefully as it can be damaged by fingernails and sharp objects.

Make sure that the condom is not inside out. Having squeezed the teat place the condom on the top of the erect penis. Roll down to its base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage.

Soon after ejaculation pull the penis out holding the condom firmly at the base of the penis and carefully take the condom off.

Dispose in a bin. Do not flush the condom down the toilet.

STORAGE AND SHELF LIFE

Store the product in a dry place at 0-25°C. Avoid mechanical impact and direct sunlight. The condom must not come into contact with petroleum products, phenols and their derivatives, oils of petroleum origin, gasoline, kerosene and other organic substances.

Shelf life - 5 years.

MEDICINAL OR PHARMACEUTICAL SUBSTANCE CONTAINED IN THE MEDICAL DEVICE

Not applicable.

LIST OF MATERIALS OF ANIMAL AND (OR) HUMAN ORIGIN****

Compounded natural rubber latex contains casein ($0.02\% \pm 10\%$ of the contents of it), functions as a stabilizer that is considered as a derivative from tissue/materials of animal origin as it is extracted from milk.

Casein (milk protein) is a natural by-product of milk. It has no known toxic effects although it may cause a reaction in individuals with an allergy to milk. The exposure from the condoms is considered not to pose any additional risk to users of the condoms.

The device does not contain any human blood derivatives.

*****warning about possible side effect in the form of the allergic reaction because of the natural latex contained in the medical device composition is presented in the inner side of the secondary/consumer package*

STERILIZATION

The medical device is presented as a non-sterile device and is not intended to be sterilized thereafter.

USAGE WITH OTHER MEDICAL DEVICES*****

Condoms can be used on their own to prevent from STIs and pregnancy and aren't intended to be used in conjunction with other medical devices.

However, consumers can use condoms with condom compatible lubricants if needed.

******recommendation to use compatible with condoms lubricant when it is necessary is presented in the inner side of the secondary/consumer packaging*

INSTALLATION, MAINTENANCE, CALIBRATION

Not applicable.

CLEANING, DISINFECTION, STERILISATION OF THE MEDICAL DEVICE

Not applicable.

INTERNATIONAL STANDARDS

ISO 13485

ISO 14971

ISO 4074

PRECAUTIONS TO BE TAKEN BY USER RELATED TO THE DISPOSAL OF THE DEVICE

Context® Lights natural rubber latex condoms should be disposed in a bin. The condoms should not be flushed the condom down the toilet.

DATE OF THE ISSUE / THE LATEST REVISION OF THE MAINTENANCE DOCUMENTATION

10/2022.

GUIDELIENSS FOR DESTRUCTION AND DISPOSAL

The devices should be used in accordance with the instructions for use.

In household use, the product is disposed of as household waste. Do not put it down the drain.


In all other cases, the condoms are disposed of in accordance with local requirements.

ENVIRONMENT SAFETY REQUIREMENTS

With proper usage, storage and disposal Natural latex condoms Context® Lights have no negative impact on the environment.

**Appendix to the Maintenance
documentation of the medical device
«Contex® Lights natural rubber latex condoms »**

*Information
to be printed in the inner side
of the secondary/ consumer package*

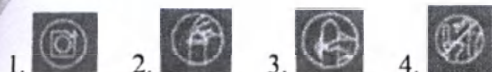Read the instruction carefully before use, especially if condoms are used for oral or anal sex.

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections.

ATTENTION: Check the expiry date on the condom wrapper before you use it. Do not use the condom after the expiration date. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

PURPOSE

Condoms are used as a barrier contraception to prevent unwanted pregnancy and to protect against HIV and other sexually transmitted diseases.

APPLICATION

1. Tear the package at the toothed edge and handle the condom carefully as it can be damaged by fingernails and sharp objects.

2. Make sure that the condom is not inside out. Having squeezed the teat place the condom on the top of the erect penis.

3. Roll down to its base. If the condom doesn't unroll, it may be on backwards, discard and use a new one. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage.

Soon after ejaculation pull the penis out holding the condom firmly at the base of the penis and carefully take the condom off.

4. Dispose in a bin. Do not flush the condom down the toilet.

For anal sex or if it is necessary use additional lubrication on the outside of the condom.

Condoms may present a choking hazard. Keep out of reach of children.

If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows.

Use a condom only once. Reuse may increase risk of failure or infection.

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the components in condom composition.

POSSIBLE SIDE EFFECTS

Condoms are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if user is allergic to latex.

PRECAUTIONS FOR USE

Use the lubricant recommended for use with condoms. Petroleum- and oil-based lubricants (petroleum jelly, baby oil), some pessaries and some topical medicines that are applied to the penis or vagina can damage condoms.

Stop use if you feel discomfort or irritation while using the condom.

Consult a specialist:

- about the compatibility of condoms with other contraceptives;
- if you are using topical medicines applied to penis or vagina;
- if irritation/discomfort persists after use;

- if you are unsure if you are allergic to latex;
- if a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

None of the contraception methods can guarantee 100% protection against unwanted pregnancy, HIV and other sexually transmitted diseases.

Date of the latest revision of the instruction: 10/2022.

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