
"Approved"

on behalf of

Richter Rubber Technology Sdn.Bhd., Malaysia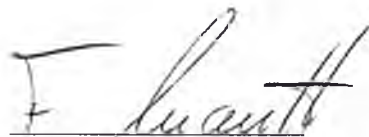

Fabian Knauth

Managing Director


29th June, 2016

MAINTENANCE DOCUMENTATION

"Condoms made of natural latex of trade mark "VIVA" produced as: Classic, Ultrathin, Ribbed, Dotted, Color aromatized"

manufactured by Richter Rubber Technology Sdn.Bhd., Plot 209 & 214, Kuala Ketil, Industrial Estate, 09300, Kuala Ketil, Kedah, Malaysia

Signed in the presence of.

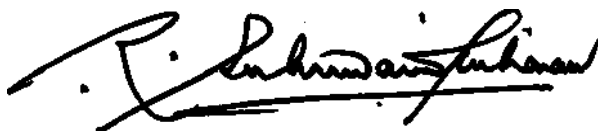R. SUBRAMANIAM KALIANAN
ADVOCATE & SOLICITOR
NOTARY PUBLIC
SUNGAI PETANI, KEDAH
MALAYSIA

1. Designation.

Condoms made of natural latex of trade mark VIVA produced: Classic, Ultrathin, Ribbed, Dotted, Color aromatized (hereinafter "Condoms") are designed for individual use as a contraceptive agents as well as for prophylactics and preservation from infectious diseases transmitted by the sexual way.

2. Scope.

Gynecology, urology.

3. Application conditions.

Disposable medical device for individual use.

4. General indications for use.

The Condoms are contraceptive agents and protective agents against infections and diseases transmitted by the sexual way.

5. Applied materials

The Condoms are manufactured in accordance with the Annex 1.

6. Main technical characteristics.

The Condoms for individual use are non-sterile disposable products. The open end of condom is rolled up into the crown (for the comfort of putting on). No visible defects (fold, slight tear or crown deformation, rubber or non-rubber impurities) shall be available. The Condoms are manufactured of the natural latex color.

The Condoms are of cylinder form in length and have a reservoir. The Condom surface is smooth and textured (dotted, ribbed), transparent, uncolored or colored (colored aromatized). The Condoms are made of the following dimensions stated in Table 1.

Table 1.

№	Properties	Limites		
		Classic, ribbed, dotted	Ultra thin	Colored aromatized
1	Length	Not less than 185 mm	Not less than 185 mm	Not less than 185 mm
2	Width	53 ± (2) mm	53 ± (2) mm	53 ± (2) mm
3	Thickness	0,065± (0,015) mm	Not more than 0,060	0,065± (0,015) mm
4	Weight	1,4-2,5 mg	1,4-2,5 mg	1,4-2,5 mg
5	Reservoir dimension	18 ± (2) mm	18 ± (2) mm	18 ± (2) mm
6	Crown width	1,5 (± 0,3) mm	1,5 (± 0,3) mm	1,5 (± 0,3) mm
7	Smell	Absent	Absent	Absent

Physical-mechanical indexes of Condoms are stated in Table 2.

Table 2

	before heat aging	after heat aging
Relative strength with tension	21.8 MPa	18.8 MPa
Relative extension with break	Not less 700%	Not less 600%

Every Condom has passed 100% electronic testing for holes, using current.

6. Contraindications.

Counter-indicated for persons susceptible to natural latex allergy.

7. Side effects.

Possible allergic reactions with people susceptible to natural latex allergy.

8. The principle of operation.

The Condoms made of natural latex are the mean of male barrier contraception and also are the barrier for diseases transmitted by the sexual way.

The procedure of using Condoms requires no special training, that's why, it may be fulfilled without the advice of medical specialist but under the strict meeting the instruction for use.

The Condom shall be put on before the coitus and after the erection.

With the correct use, the Condoms are the effective measure for the prevention of undesirable pregnancy and diseases transmitted by the sexual way, including HIV/AIDS, however, no contraception measure guarantees 100% protection.

9. Preparing for use.

To be sure that the Condom shelf life in foil packing is not expired. If the shelf life is expended use another condom. The packing shall be new and have no damage. While the packing is damaged use another condom.

10. A method of use.

Open very carefully.

Teeth, nails and other sharp subjects may damage the condom.

- Get the condom out of the packing and examine its opening. The Condom shall be exposed in one direction and, actually, not exposed in other direction.

- The Condom shall be put on the erected penis. Press on the end of condom for the removal of air. Put the condom to the end of penis and, with the other hand roll out along all length till its base.

- After the ejaculation, remove the condom as the penis is still erected. Take the condom with the base of penis and roll it up. Don't touch the vagina by the condom or penis after the condom has been removed.

11. Precautions.

- Every Condom may be used only once.
- Any gel on the water or glycerin basis may be used, if necessary.
- Don't use any vegetable, animal or culinary fats and also lotions and vaseline. Their use may lead to the destruction of latex structure and violation of the condom integrity.
- In the event of using medical products which may contact with the condom it is necessary to contact the physician.

12. Marking.

The marking is applied directly on the individual and consumer packing made of foil. The marking is applied by any means (printing, stamping, die), allowing its brightness, clarity and readability.

Marking of individual packing:

- manufacturing "Classic, Ribbed, Dotted, Strawberry, Ultra Thin";
- number of batch «LOT»;
- valid until «EXP:».

Marking of Condoms shall include the following data:

- name of the producer (manufacturer), the name of the country of origin, address;
- name of the importing company/authorized organization, address;
- product name, trademark;
- technical specification and quantity;

Factory: Plot 209 & 214, Kuala Ketil Industrial Estate, 09300 Kuala Ketil, Kedah Darul Aman, Malaysia.

Fax: +604-416 1667 Tel: +604-416 1668 ; +604-416 1669

- dimensions;
- basic raw materials used for the production.
- date of the product expiration;
- valid until «EXP: ...»;
- batch number «LOT: ...»;
- disposal instructions: "Do not throw down the drain";
- method of storage;
- product description;
- product barcode (if necessary);
- information about the certification (if any);
- registration number.

Transport marking:

- name of the producer (manufacturer), the name of the country of origin, address;
- name of the importing company/authorized organization, address;
- product name, trademark;
- quantity;
- shelf life;
- batch number «LOT: ...»;
- disposal instructions: "Do not throw down the drain";
- method of storage;
- product barcode.

13. Storage and shelf life.

Shelf life is 5 years since production. Don't use after the end of shelf time. Keep in the plant packing with the temperature of +0°C to + 25°C and relative air humidity not more than 80% at a distance of not less than 1 m far from heating devices protected from direct sun.

14. Transportation conditions.

Transported by any kind of transportation. During the transportation it is necessary to use clean means and meet the precautions; not to put a heavy cargo on the product, don't contact the production with sharp goods, protect the product from blows; not to expose the product to direct sun.

15. Warranty.

The manufacturer is not liable for any consequences resulting from the improper use. The warranty is 5 years.

16. Method of disposal.

It is forbidden to flush down the toilet. The used Condom should be disposed of as a household waste.

17. Producer (manufacturer).

Richter Rubber Technology Sdn.Bhd., Polot 209 & 214, Kuala Ketil, Industrial Estate, 09300, Kuala Ketil, Kedah, Malaysia

18. Authorized representative in Russia.

ООО "БОЛЕАР", legal address: 115230, Moscow, Kashirskoye Shosse. 12, bldg. 1; actual address: 115201, Moscow, Kashirskiy passage, 13, tel. : +7 (495)787-92-68, www.bolear.ru.

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Annex 1 to the maintenance document
"Approved"
on behalf of
Richter Rubber Technology Sdn.Bhd., Malaysia

Fabian Knauth
Managing Director


29th June, 2016

The materials used are

**"Condoms made of natural latex of trade mark VIVA produced as: Classic, Ultrathin, Ribbed,
Dotted, Color aromatized"**

Manufactured by Richter Rubber Technology Sdn.Bhd., Plot 209&214, Kuala Ketil, Industrial Estate,
09300, Kuala Ketil, Kedah, Malaysia

Signed in the presence of.

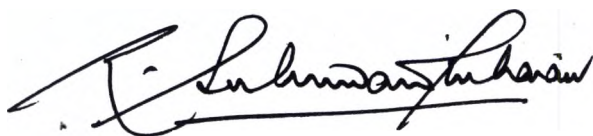R. SUBRAMANIAM KALLANAN
ADVOCATE & SOLICITOR
NOTARY PUBLIC
SUNGAI PETANI, KEDAH
MALAYSIA

Condoms made of natural latex of trade mark VIVA Classic

100 g of condoms manufactured from:

- 1) The natural latex mixture manufacturer of Richter Rubber Technology Sdn. Bhd., Malaysia.

The formulation natural latex compound.

No	Used components	Content in %
1	Dry latex	94.04
2	Vultamol	0.175
3	ZDEC	0.658
4	Wingstay-L	0.876
5	ZDBC	0.164
6	Zinc Oxide	1.736
7	Sulphur	0.987
8	Potassium Hydroxide	0.164
9	Emulan OG	0.044
10	Potassium Oleate Solution	1.151
Total		100

The amount natural latex mixture into the product 94.94%

- 2) Powdering - the food corn starch of trade mark Agenasorb 9020, manufacturer Agrana Starke GmbH, Germany.

The amount of corn starch into the product 3.716%

- 3)* Grease - silicone oil is of trade mark BRB Silicone Oil DM 350 EP Silicone Fluid, Density , the viscosity of 350 cps, manufacturer BRB International BV, The Netherlands.

The amount of grease into the product 1.349 %

Condoms made of natural latex of trade mark VIVA Ultrathin

100g of condoms manufactured from:

- 1) The natural latex mixture manufacturer of Richter Rubber Technology Sdn. Bhd., Malaysia.

The formulation natural latex compound.

No	Used components	Content in %
1	Dry latex	94.04
2	Vultamol	0.175
3	ZDEC	0.658
4	Wingstay-L	0.876
5	ZDBC	0.164
6	Zinc Oxide	1.736
7	Sulphur	0.987
8	Potassium Hydroxide	0.164
9	Emulan OG	0.044
10	Potassium Oleate Solution	1.151
Total		100

The amount natural latex mixture into the product 92.42%

- 2) Powdering - the food corn starch of trade mark Agenasorb 9020, manufacturer Agrana, Starke GmbH, Germany.

The amount of corn starch into the product 5.216%

- 3) Grease - silicone oil is of trade mark BRB Silicone Oil DM 350 EP Silicone Fluid, Density, the viscosity of 350 cps, manufacturer BRB International BV, The Netherlands.

The amount of grease into the product 2.379 %

Condoms made of natural latex of trade mark VIVA Ribbed

100g of condoms manufactured from:

1) The natural latex mixture manufacturer of Richter Rubber Technology Sdn. Bhd., Malaysia.

The formulation natural latex compound.

No	Used components	Content in %
1	Dry latex	94.04
2	Vultamol	0.175
3	ZDEC	0.658
4	Wingstay-L	0.876
5	ZDBC	0.164
6	Zinc Oxide	1.736
7	Sulphur	0.987
8	Potassium Hydroxide	0.164
9	Emulan OG	0.044
10	Potassium Oleate Solution	1.151
Total		100

The amount natural latex mixture into the product 94.94%

2) Powdering - the food corn starch of trade mark Agenasorb 9020, manufacturer Agrana Starke GmbH, Germany.

The amount of corn starch into the product 3.716%

3) Grease - silicone oil is of trade mark BRB Silicone Oil DM 350 EP Silicone Fluid, Density , the viscosity of 350 cps, manufacturer BRB International BV, The Netherlands.

The amount of grease into the product 1.349 %

Condoms made of natural latex of trade mark VIVA Dotted

100g of condoms manufactured from:

1) The natural latex mixture manufacturer of Richter Rubber Technology Sdn. Bhd., Malaysia.

The formulation natural latex compound.

No	Used components	Content in %
1	Dry latex	94.04
2	Vultamol	0.175
3	ZDEC	0.658
4	Wingstay-L	0.876
5	ZDBC	0.164
6	Zinc Oxide	1.736
7	Sulphur	0.987
8	Potassium Hydroxide	0.164
9	Emulan OG	0.044
10	Potassium Oleate Solution	1.151
Total		100

The amount natural latex mixture into the product 94.94%

2) Powdering - the food corn starch of trade mark Agenasorb 9020, manufacturer Agrana Starke GmbH, Germany.

The amount of corn starch into the product 3.716%

3) Grease - silicone oil is of trade mark BRB Silicone Oil DM 350 EP Silicone Fluid, Density , the viscosity of 350 cps, manufacturer BRB International BV, The Netherlands.

The amount of grease into the product 1.349 %

Condoms made of natural latex of trade mark VIVA Color aromatized

Color – red.

Flavor – strawberry.

100g of condoms manufactured from:

- 1) The natural latex mixture manufacturer of Richter Rubber Technology Sdn. Bhd., Malaysia.

The formulation natural latex compound.

No	Used components	Content in %
1	Dry latex	94.04
2	Vultamol	0.175
3	ZDEC	0.658
4	Wingstay-L	0.876
5	ZDBC	0.164
6	Zinc Oxide	1.736
7	Sulphur	0.987
8	Potassium Hydroxide	0.164
9	Emulan OG	0.044
10	Potassium Oleate Solution	1.151
Total		100

The amount natural latex mixture into the product 94.704%

- 2) Pigment - the food; red colour; color pantone No. Colanyl Carmine LTX 100-ID 0030, manufacturer of Clariant (Singapore) Pte. Ltd.

The amount of pigment into the product 0.14%

- 3) Powdering - the food corn starch of trade mark Agenasorb 9020, manufacturer Agrana Starke GmbH, Germany.

The amount of corn starch into the product 3.716%

- 4) Grease - silicone oil is of trade mark BRB Silicone Oil DM 350 EP Silicone Fluid, Density , the viscosity of 350 cps, manufacturer BRB International BV, The Netherlands.

The amount of grease into the product 1.35 %

- 5) Flavor strawberry – of trade mark Strawberry Flavor Oil 9698, manufacturer Matrix Flavours & Fragrances Sdn.Bhd., Malaysia.

The amount of flavor into the product 0.09%

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