The State of Maryland

Office of the Secretary of State

This Apostille is not valid for use anywhere within the United States of America, its territories or possessions.

This Apostille does not certify the content of the document for which it is issued.

Apostille

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1. Country: United States of America
This public document
2. has been signed by **Robert P. Duckworth**
3. acting in the capacity of **Clerk of the Circuit Court for Anne Arundel County**
4. bears the seal/stamp of the **Circuit Court for Anne Arundel County**

Certified

5. at Annapolis, Maryland
6. the **6th day of September, 2018**
7. by The Secretary of State of Maryland
8. **No. 463324**

9. Seal

10. Signature

Secretary of State

Affidavit

State of Maryland)
)SS:
County of Anne Arundel)

Before me, the undersigned notary public, this day, personally,
appeared to me known or proven, who being duly sworn according to law,
deposes the following:

Yoshiko Yano

I, _____, hereby certify that the attached
document is

- ☒ an original document
☐ a true copy of an original document

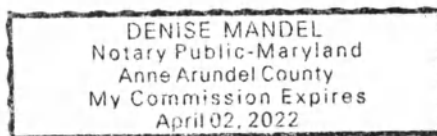
to the best of my knowledge and belief.

Signed _____

Subscribed and sworn to before me this

6 day of September, 2018.

Notary Public

State of Maryland, Anne Arundel County, Sgt.

I, **ROBERT P. DUCKWORTH**, Clerk of the Circuit Court for Anne Arundel County, Maryland, a court of record, do hereby certify that **Denise Mandel** was a commissioned/appointed and qualified Notary Public commencing on the 28th day of February, 2018.

In Testimony Whereof, I have hereunto set my hand and affixed the seal of the court September 6, 2018

Robert P. Duckworth

Robert P. Duckworth, Clerk
Circuit Court for Anne Arundel County, Maryland

1 Sept, 2018

To whom it may concern:

To be submitted to the Competent Health Authorities in Federal Republic of Russia
We, Bayer Medical Care Inc., USA (1 Bayer Drive Indianola Pennsylvania 15051-0780 USA) hereby state that attached document (Instruction for use) (Sterile High Pressure Connector Tubing) is true.

Bayer Medical Care Inc

1 Bayer Drive
Indianola, PA 15051-0780
U.S.A.

(412) 767-2400

www.radiology.bayer.com

Best regards,

William Hohn

Regulatory Affairs International Strategist – Europe

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©2017 Bayer. All Rights Reserved. Reproduction of this manual is strictly prohibited without express written consent of Bayer Medical Care Inc.

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Bayer, the Bayer Cross, MEDRAD, Twist & Go, and MEDRAD Twist & Go may be registered trademarks of Bayer in the US and other countries.

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©2013-2014, 2017 Bayer. All Rights Reserved. Reproduction of this manual is strictly prohibited without express written consent of Bayer Medical Care Inc.

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Bayer, the Bayer Cross and MEDRAD Twist & Go may be registered trademarks of Bayer in the US and other countries.

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UNITED STATES

Bayer Medical Care Inc.
1 Bayer Drive
Indianola, PA 15051-0780
U.S.A.
Phone: +1 (412) 767-2400
+1 (800) 633-7231
Fax: +1 (412) 767-4120

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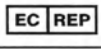
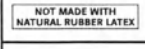
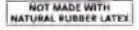
Introduction: Read the information contained in these instructions for use. Understanding the information will assist you in using the *MEDRAD® Twist & Go (Twist & Go)* High Pressure Connector Tubing properly.

Important Safety Notice: The *Twist & Go* High Pressure Connector Tubing is intended to be used by qualified medical professionals with proper training and experience in angiographic procedures and the use of injectors from Bayer. As with any injector, operator vigilance is required when using an Injection System. Visual inspections and good practice during set-up and use and compliance with the operation manual remain the primary methods to limit the risk of air being present during fluid delivery.

Intended Use: The *Twist & Go* High Pressure Connector Tubing is intended to be used in the delivery of contrast media and common flushing solutions. The device is indicated for single use only with injectors from Bayer with *Twist & Go* Syringes.

Contraindications: None known.

Restricted Sale: Rx Only.

▲ WARNING

Air Embolism Hazard. Serious patient injury or death may result.

- Ensure that one operator is designated as being responsible for purging the High Pressure Connector Tubing. Do not change operators during the procedure. If an operator change must occur, ensure that the new operator verifies that the fluid path is purged of air.
- Ensure that patient is not connected while purging air from High Pressure Connector Tubing.
- Remove all air from the High Pressure Connector Tubing.
- Use caution when removing air from the syringe or tubing connections. Component damage may result from the use of tools during air removal.
- Securely tighten the high pressure connector tubing to syringe tip connection. Component damage may result from the use of tools during tightening.
- Do not use tubing to manipulate the injector position.
- Avoid contact of and damage to syringe tip, tubing and connections when manipulating injector position.

▲ WARNING

Bloodborne Contamination Hazard. Serious patient and/or worker injury or death may result.

- Use caution when removing air from the syringe or tubing connections. Component damage may result from the use of tools during air removal.
- Do not use tools to securely tighten the high pressure connector tubing to syringe tip connection.
- Avoid contact of and damage to syringe tip, tubing and connections when manipulating injector position.
- For devices labeled for single use, please note: This product is intended for single use only. Do not resterilize, reprocess or reuse. The disposable devices have been designed and validated for single use only. Re-use of the single use disposable devices pose risks of device failure and risks to the patient. Potential device failure includes significant component deterioration with extended use, component malfunction, and system failure. Potential risks to the patient include injury due to device malfunction or infection as the device has not been validated to be cleaned or re-sterilized.
- Do not exceed 1200psi (8273kPa).
- Do not use tubing to manipulate injector position.

Biological Contamination Hazard. Serious patient and/or worker injury or death may result.

- Properly discard disposables after use if contamination may have occurred during setup or use.

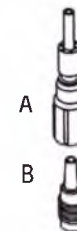
▲ CAUTION

Environmental Contamination Hazard. Patient and/or worker injury may result.

- Do not use if sterile package is opened or damaged. Patient or operator injury may result if package is opened or damaged, or if damaged components are used. Visually inspect contents and package before each use.
- Follow sterile technique principles, specifically, maintain sterility of the syringe tip and high pressure connector tubing.

INSTALLING, FILLING AND PURGING A HIGH PRESSURE CONNECTOR TUBING

1. Remove the dust cap from the syringe tip if attached.
2. Insert the *Twist & Go* High Pressure Connector Tubing onto the *Twist & Go* syringe tip.
3. Turn the *Twist & Go* (A) nut clockwise to securely tighten the high pressure connector tubing to the syringe tip (B).
4. Purge all air from the high pressure connector tubing. Use the injector to push contrast and flushing solutions out until all of the air has been removed from the High Pressure Connector Tubing. Gentle tapping at any connection point may be needed to facilitate air removal.

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