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EN

REF 1151300
CALiaGold®
pierceTube

Pierceable tube for human feces samples
collection

IVD

CE

100 COLLECTION TUBE (13 x 98 mm)

NOTE: This package insert must be read carefully prior to product use. Package insert instructions must be followed accordingly. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package insert.

INTENDED USE

CALiaGold® pierceTube is used only to collect and store the human feces. The tube must be used with the kit CALiaGold® reagent for the measurement of human calprotectin in feces. For *In Vitro* Diagnostic use only.

CHARACTERISTICS

Each CALiaGold® pierceTube contains 1.7 mL of extraction buffer.

COMPONENTS OF THE KIT

The CALiaGold® pierceTube, stored at 2-8 °C, is stable up to the expiry date indicated on the label.

CALiaGold® pierceTube

100 COLLECTION TUBE
diameter 13 mm, height 98 mm

Composition and concentration of active components in the Tube are:
Phosphate Buffer 100mM, Bovin Serum Albumin (BSA) 1.0%, sodium azide < 0.1% and stabilizers.

COLLECTION TUBE description

Colorless pierceable cap (1)

Extraction buffer

- 4) Reinsert the yellow cap and screw on tightly. Do not reopen.
- 5) Vortex the CALiaGold® pierceTube device with the yellow head down vigorously on a vortex mixer for 30 seconds and allow it to stand for 10 minutes on the yellow head. If the stool does not completely come off the grooves, repeat the procedure until the grooves are completely free of stool.
- 6) Place the tube on the sample tray of the analyzer with colorless pierceable cap (1) upward and barcode in the external space.

STABILITY AND STORAGE

Feces: 3 days at 20-25 °C or 6 days at 2-8 °C
Feces in CALiaGold® pierceTube: 3 days at room temperature up to 37 °C and 14 days at 2-8 °C.

INSTRUMENTATION AND MATERIALS REQUIRED

Materials provided

- CALiaGold® pierceTube

Materials not provided

- General laboratory equipment
- Clinical chemistry analyzers
- CALiaGold® reagent

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