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APPROVED BY

General Manager
Demophorus Limited

Signature
Sakis Papaspyrou

★ 01-Sep-2014

Seal: [DEMOPHORUS LIMITED * LIMASSOL]

**INSTRUCTION
by application**

Demotek Latex Surgical Gloves.
Demophorus Limited, Cyprus.

1. Application

The product is designed to isolate the hands of medical personnel during various medical procedures in surgeries.

Area of application: surgery.

Gloves shall be worn on hands to prevent infection.


Latex surgical gloves, sterile, powdered and powder-free, of anatomic shape, with a textured palmar side, smooth cuff and back side. Single use.

Ambidextrous.

Highly elastic, which minimizes the likelihood of rupture.

End of the cuff is beaded, providing the glove fits tightly to hand and does not roll.

Powdered gloves are treated with corn starch which prevents sticking and reduces the risk of allergic reactions of healthcare workers.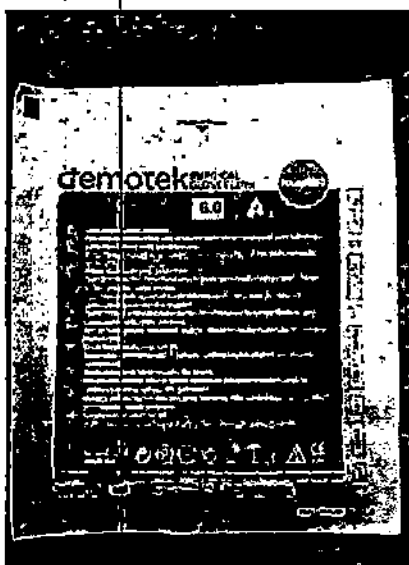

Range.

Demotek Latex Surgical Gloves.

I. Demotek Latex Surgical Gloves:

1. Demotek Latex Surgical Gloves, Powder Free: SG09, SG10, SG11, SG12, SG13, SG14, SG15, SG16.


2. Demotek Latex Surgical Gloves, Powdered: SG01, SG02, SG03, SG04, SG05, SG06, SG07, SG08.

2. Basic specifications

Main parameters and characteristics.

Name	REF	Material	Powdered/Powder free	Size	Latex thickness (mm)	Width/Length/Circumference of the hand (mm)
Demotek	SG09	Latex	Powder free	5.5	Finger:	72±4/≥250/150±5

Latex Surgical Gloves.	SG10			6.0	≥ 0,1 Palm: ≥ 0.13 Cuff: ≥ 0.1 Bead of the glove not more than 2.5 mm.	77±5/≥260/160±5
	SG11			6.5		83±5/≥260/170±5
	SG12			7.0		89±5/≥270/190±5
	SG13			7.5		95±5/≥270/200±5
	SG14			8.0		102±6/≥270/220±5
	SG15			8.5		108±6/≥280/230±5
	SG16			9.0		114±6/≥280/240±5
	SG01	Powdered.		5.5		72±4/≥250/150±5
	SG02			6.0		77±5/≥260/160±5
	SG03			6.5		83±5/≥260/170±5
	SG04			7.0		89±5/≥270/190±5
	SG05			7.5		95±5/≥270/200±5
	SG06			8.0		102±6/≥270/220±5
	SG07			8.5		108±6/≥280/230±5
	SG08			9.0		114±6/≥280/240±5

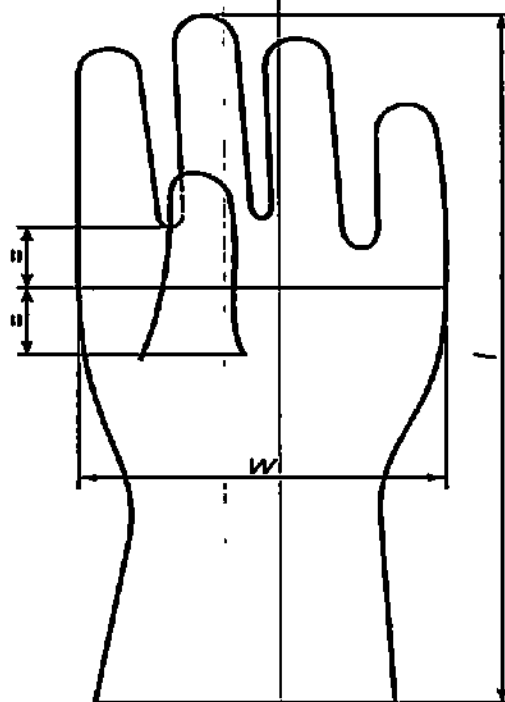Points to measure the width and length of the glove

Size XS corresponds to the size of 5-6,

Size S to the size 6-7,

Size M to the size 7-8,

Size L to the size 8-9,

Size XL to the size 9 or more.

Inspection levels and acceptable quality levels (AQL).

Parameter	Inspection level	AQL
Dimensions	S-2	4.0
Tightness	1	1.5
Tension load and elongation at break (before and after aging), force at an elongation of 300% (before aging)	S-2	4.0

Tensile characteristics.


Parameter	Value
Tension load at break before aging, N	≥ 12.5
Elongation at break before aging, %	≥ 700
Tension load required to achieve an elongation of 300% before aging, N	≤ 2.0
Tension load at break after aging, N	≥ 9.5
Elongation at break after aging, %	≥ 550

3. Contraindications.

Allergic reaction to latex.

4. Method of application.

How to put on gloves:

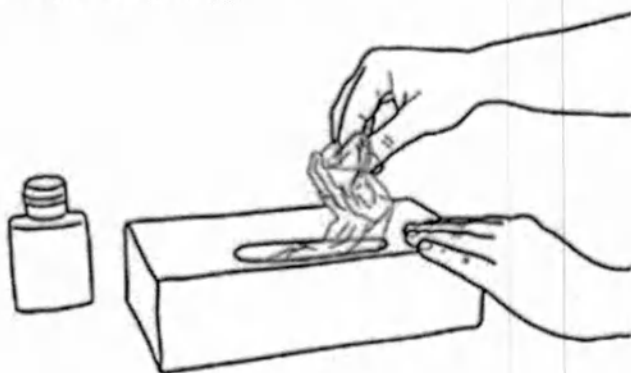
1. Get the glove out of the package (box)


2. Touch only the part of the glove that will be located on the wrist (upper edge of the


cuff).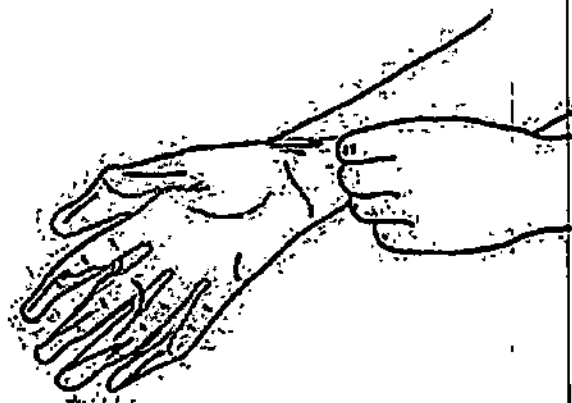

3. Put on the first glove.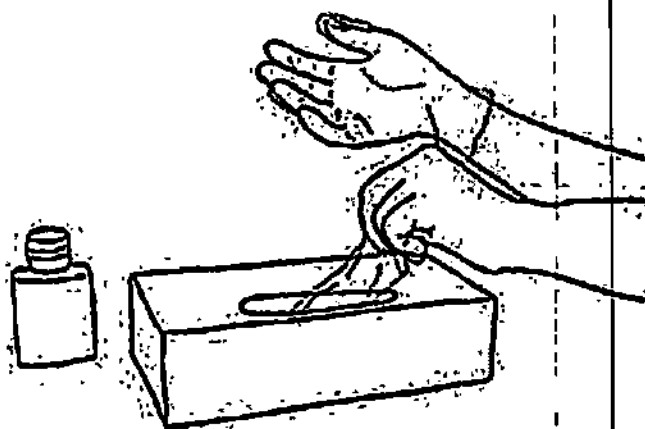

4. Take a second glove with the hand without gloves and touch the area of the glove which will be on your wrist.

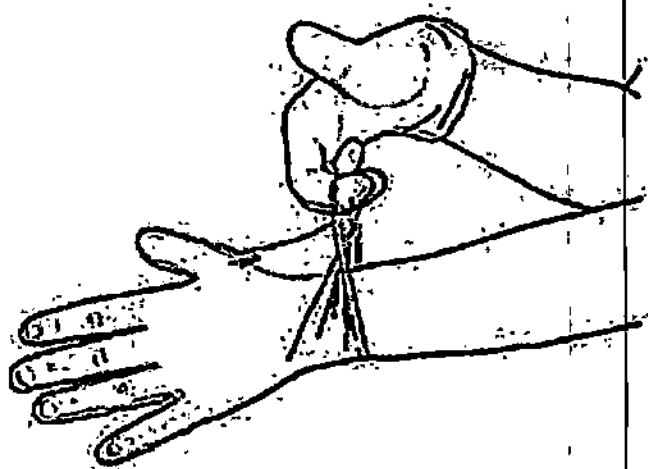
5. To avoid touching the skin of the forearm with the gloved hand, turn the outer surface of the glove out in such a way to put it on the bent fingers of the gloved hand, it will allow you to put on the second glove.


6. After the gloves are on, do not touch any ambient objects, contact with which is an indication to change gloves.

How to remove gloves:

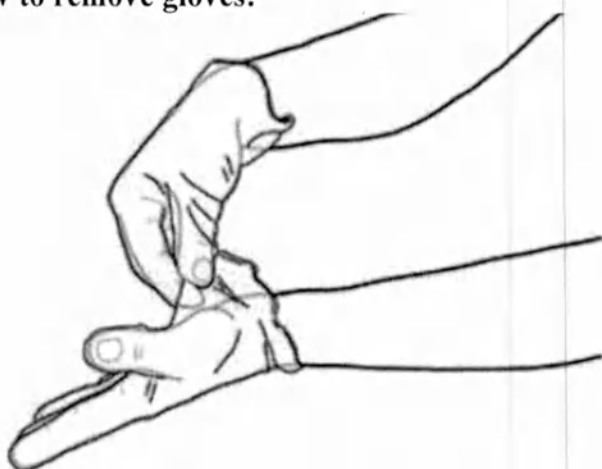
1. Pinch one glove at the wrist to pull it off without touching the skin of the forearm, and pull it from the hand in such a way that the glove would turn inside out.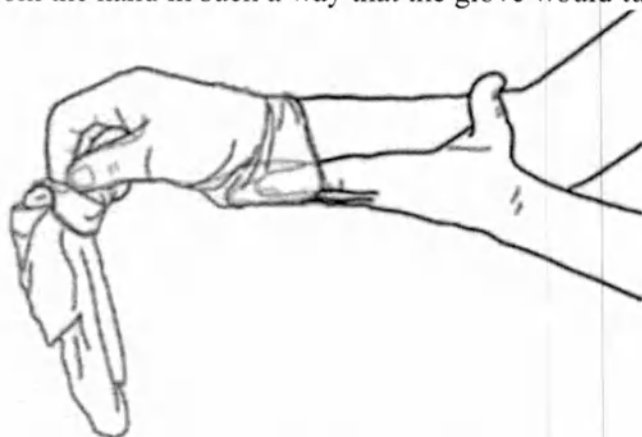

2. Holding the removed glove in the gloved hand, put the fingers of the hand from which the

glove has been removed inside the space between the second glove and wrist. Remove the second glove by rolling from the hands, and put it inside the first glove.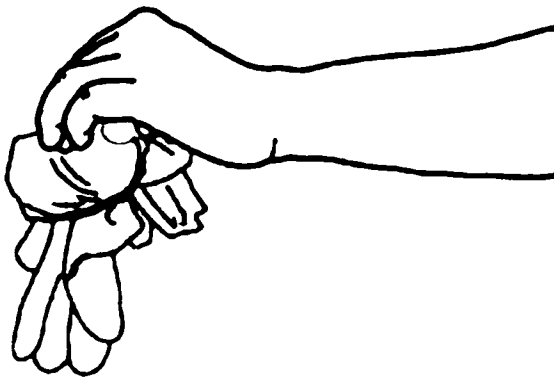

3. Dispose of the removed gloves.

5. Conditions of storage and use.

Temperature of storage, transport and use: +10 °C to + 35°C.

Relative humidity: no more than 80%.

Do not expose to direct sunlight.

6. Shelf life.

Warranty period for retention of sterility: 5 years

7. Disposal.

Demotek Latex Surgical Gloves must be disposed or destroyed in accordance with the rules and laws of the country of disposal.

8. Transportation.


Demotek Latex Surgical Gloves can be transported by all modes of transport in covered vehicles in accordance with the rules of transportation of cargo applicable to this mode of transport, at a temperature from +10°C to +35°C and at relative humidity: no more than 80%.

9. Packing.

The gloves are packaged in individual paper packages in which the right-hand glove is separate from the left-hand, and also in a paper bag, which has a protective function and maintains the sterility of the product.


Transportation package is a box made of corrugated cardboard, containing 50 units of the product in a paper bag.

10. Marking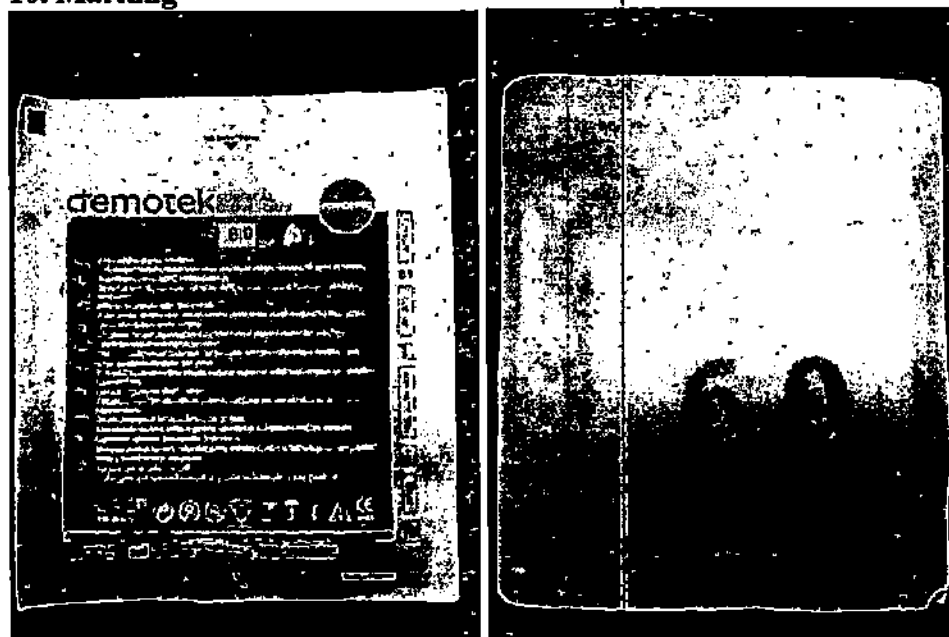


On the product, the following marks and symbols are used:

- product name;
- REF;
- release date (year, month);
- shelf life (year, month);
- CE Marking;
- re-use is prohibited;
- temperature range;
- size;
- mark of material;
- Sterile, sterilized by irradiation (minimum absorbed dose 25kGr);
- it is prohibited to violate the integrity of the packaging;
- protect from direct sunlight;
- protect from moisture;
- read the instructions before use;
- lot number;
- manufacturer;
- manufacturer country;
- «Powdered» and «Powder free»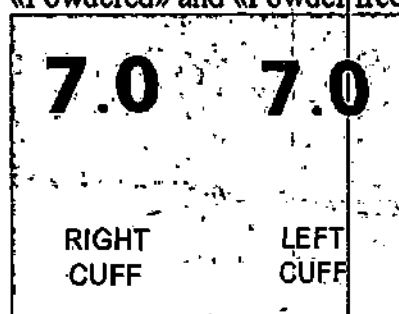

On the inside individual containers are marked:

- the size;
- the inscription in English "the left glove" or "right-hand glove".

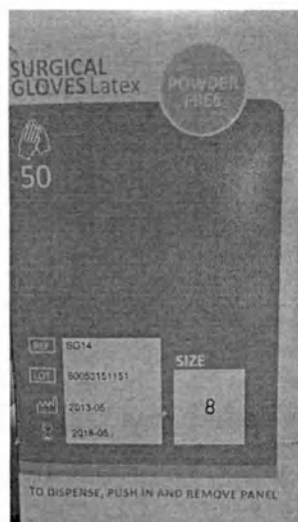Group package includes the following marking:

- the name and address of the developer,
- the country of manufacture;
- the Internet address;
- the name of the product;
- the size;
- the batch number LOT;
- the REF;
- the inscription "Powdered" or "powder free";
- the marks "for single use Only, Contains latex", "do Not use with damaged packaging", LO, "Refer to documentation", "Sterile. Method of sterilizing radiation";
- the storage conditions;
- the date of manufacture;
- the date of expiry;
- the number of pairs of gloves per pack;
- the barcode.

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