APPROVED BY

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General Manager
Demophorus Limited

Signature/
Sakis Papaspyrou
★ 01-Sep-2014

Seal: [DEMOPHORUS LIMITED * LIMASSOL]

INSTRUCTION FOR USE
OF A MEDICAL PRODUCT

Demotek Nitrile Examination Gloves.
Demophorus Limited, Cyprus.
Cyprus.

1. Application

The product is designed to isolate the hands of medical personnel during various medical procedures do not requiring sterile conditions.

(to reduce the risk of contamination of environmental objects by potentially pathogenic microorganisms and their transmission from health care workers to patients and vice versa, as well as from one patient to another).

Area of application: diagnosis, obstetrics, gynecology and urology. And where is a risk of an allergy to natural rubber latex.

Nitrile examination gloves, powdered or powder free, fully textured, non-sterile.

Are highly elastic, which minimizes the likelihood of rupture.

End of the cuff is beaded, providing the glove fits tightly to hand and does not roll.

Powdered gloves are treated with corn starch which prevents sticking.

Range of medical products.

I. Demotek Nitrile Examination Gloves

1. Demotek Nitrile Examination Gloves, Powder Free NGC38PF-XS, NGC38PF-S, NGC38PF-M, NGC38PF-L, NGC38PF-XL, NGC40PF-XS, NGC40PF-S, NGC40PF-M, NGC40PF-L, NGC40PF-XL.

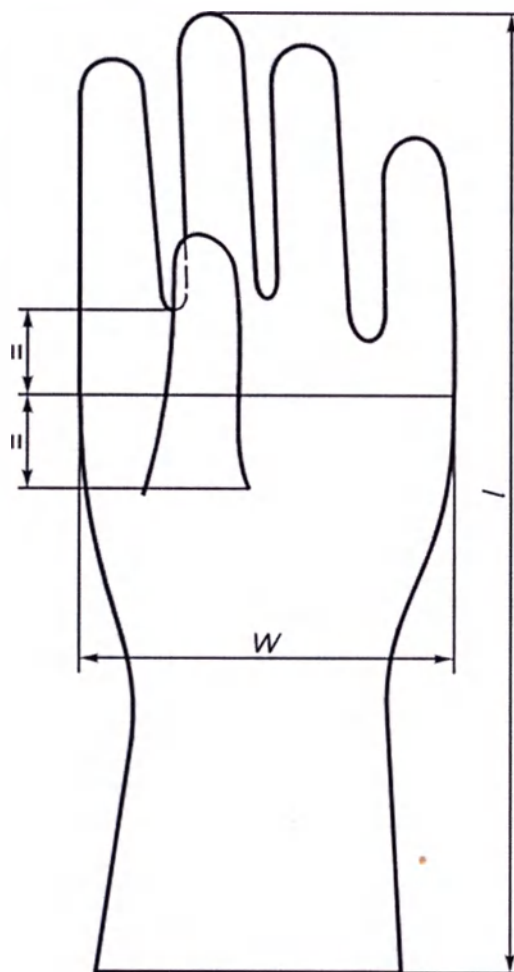
2. Demotek Nitrile Examination Gloves, Powdered: NGC38PP-XS, NGC38PP-S, NGC38PP-M, NGC38PP-L, NGC38PP-XL, NGC40PP-XS, NGC40PP-S, NGC40PP-M, NGC40PP-L, NGC40PP-XL.

2. Basic specifications

Main parameters and characteristics.

Name	REF	Material	Powdered/Powder free	Size	Width/Length/Circumference of the hand (mm)	Nitrile thickness (mm)
Demotek Nitrile Examination	NG38PF-XS	Nitrile	Powder free	XS (5)	76±3/≥220/ 145±5	Finger: ≥ 0.11
	NG38PF-			S (6)	84±3/≥220/	Palm:

on Gloves.	S				152±5	≤ 2.03
	NG38PF-M			M (7)	94±3/≥230/ 178±5	Cuff: ≥ 0.1
	NG38PF-L			L (8)	105±3/≥230/ 203±5	Bead of the glove not more than 2.5 mm.
	NG38PF-XL			XL (9)	113±3/≥230/ 229±5	
	NG38PP-XS			XS (5)	76±3/≥220/ 145±5	
	NG38PP-S		Powdered.	S (6)	84±3/≥220/ 152±5	
	NG38PP-M			M (7)	94±3/≥230/ 178±5	
	NG38PP-L			L (8)	105±3/≥230/ 203±5	
	NG38PP-XL			XL (9)	113±3/≥230/ 229±5	

Points to measure the width and length of the glove

Size **XS** corresponds to the size 5-6,

Size **S** to the size 6-7,

Size **M** to the size 7-8,

Size **L** to the size 8-9,

Size **XL** to the size 9 or more.

Inspection levels and acceptable quality levels (AQL).

Parameter	Inspection level	AQL
Dimensions	S-2	4.0
Tightness	G-1	2.5
Tension load and elongation at break (before and after aging)	S-2	4.0

Tensile characteristics.


Parameter	Value
Tension load at break before aging, N	≥ 7.0
Elongation at break before aging, %	≥ 500
Tension load at break after aging, N	≥ 7.0
Elongation at break after aging, %	≥ 400

3. Contraindications.


Allergic reaction to modified corn starch.

4. Method of application.

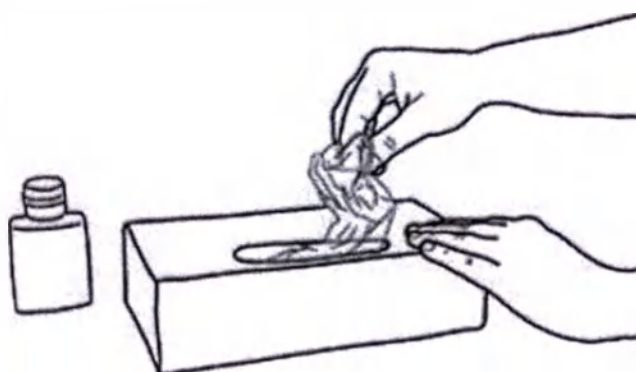
How to put on gloves:

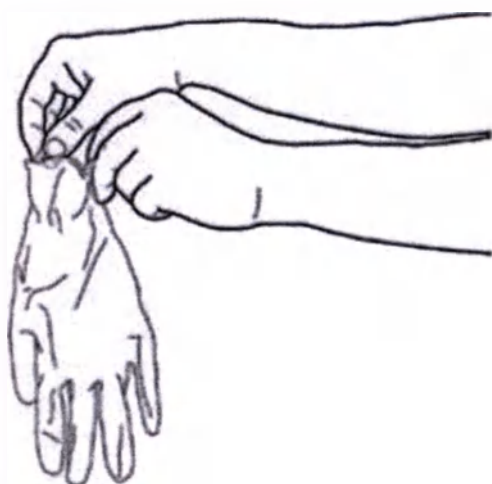
1. Get the glove out of the package (box)

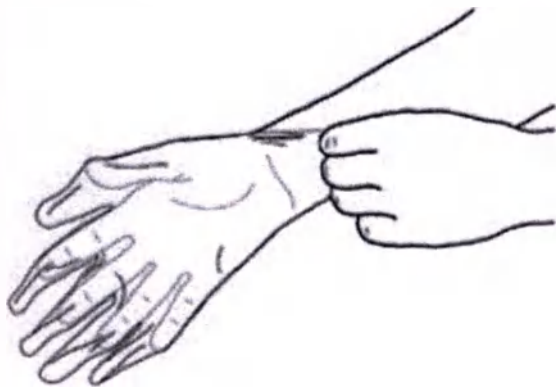
2. Touch only the part of the glove that will be located on the wrist (upper edge of the cuff).

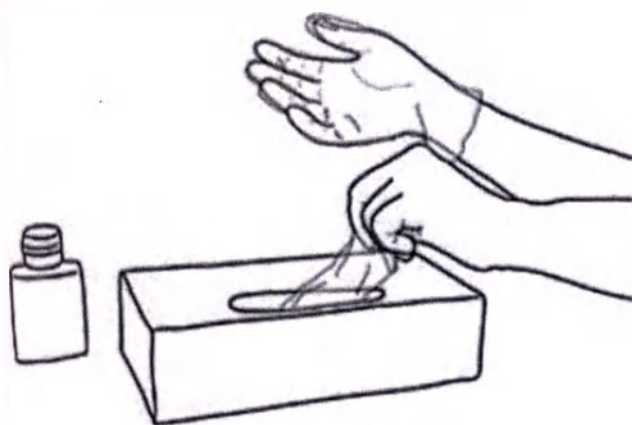3. Put on the first glove.

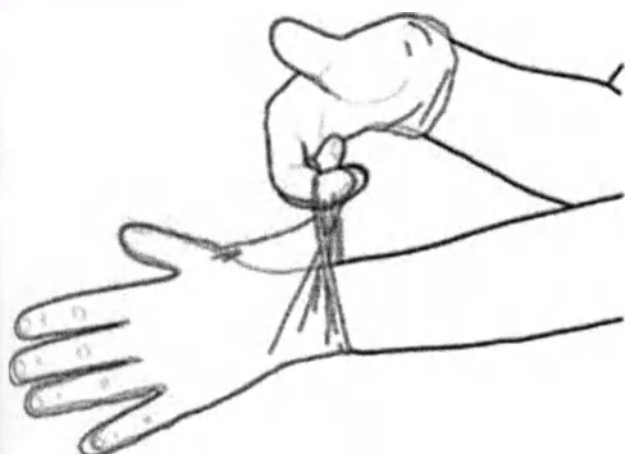4. Take a second glove with the hand without gloves and touch the area of the glove which will be on your wrist.


5. To avoid touching the skin of the forearm with the gloved hand, turn the outer surface of the glove out in such a way to put it on the bent fingers of the gloved hand, it will allow you to put on the second glove.


6. After the gloves are on, do not touch any ambient objects, contact with which is an indication to change gloves.

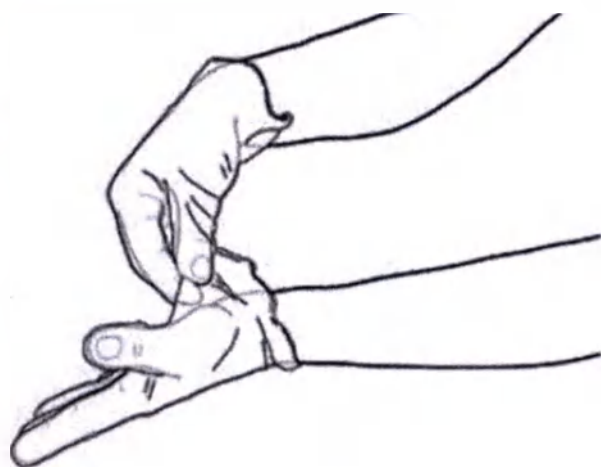
How to remove gloves:

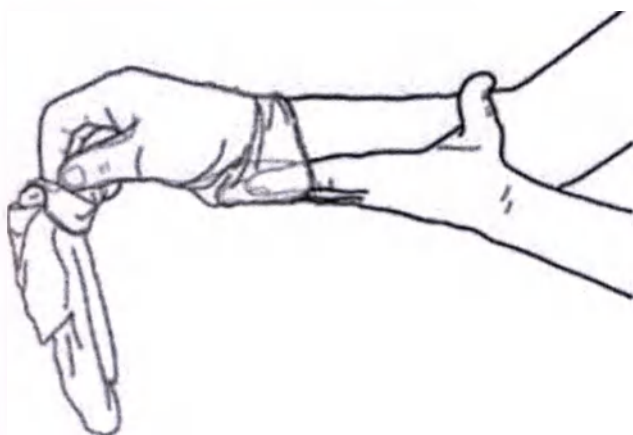1. Pinch one glove at the wrist to pull it off without touching the skin of the forearm, and pull it from the hand in such a way that the glove would turn inside out.

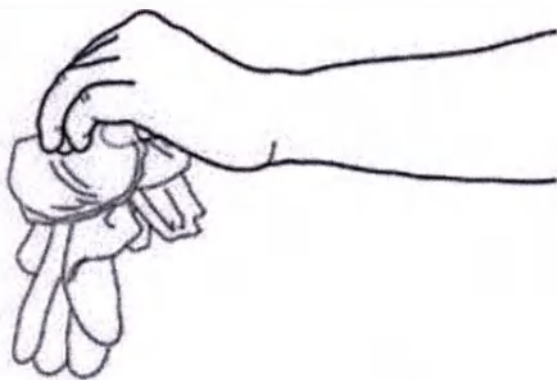2. Holding the removed glove in the gloved hand, put the fingers of the hand from which the glove has been removed inside the space between the second glove and wrist. Remove the second glove by rolling from the hands, and put it inside the first glove.

3. Dispose of the removed gloves.

5. Conditions of storage and use.

Storage temperature: +10 °C to +40°C.

Store in a dry place.

Relative humidity: no more than 80%

Do not expose to direct sunlight.

6. Shelf life.

5 years.

7. Disposal.

Demotek Nitrile Examination Gloves must be disposed or destroyed in accordance with the rules and laws of the country of disposal.

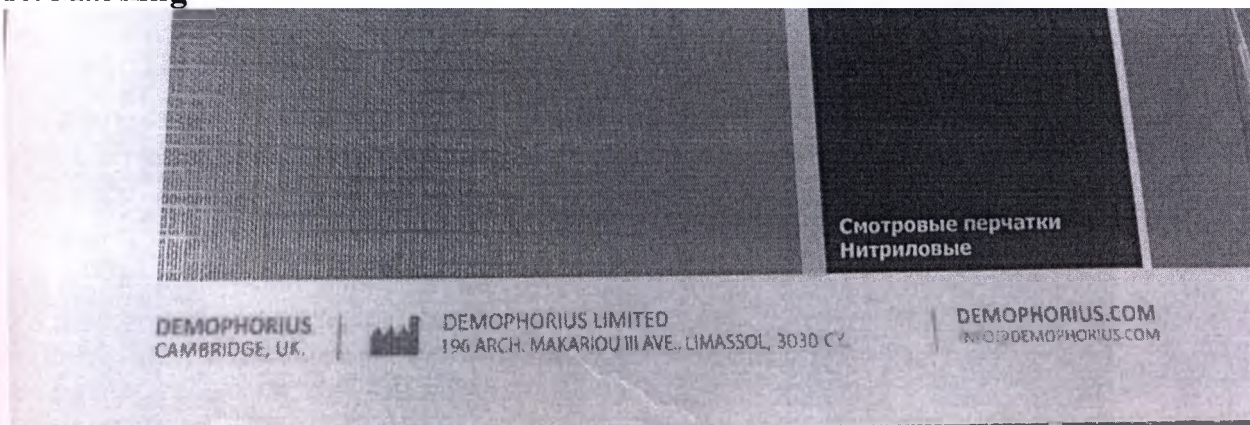
8. Transportation.

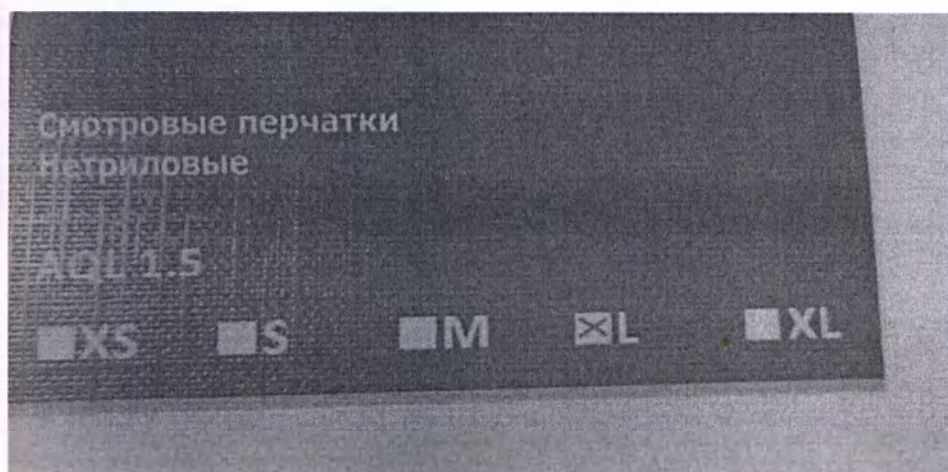
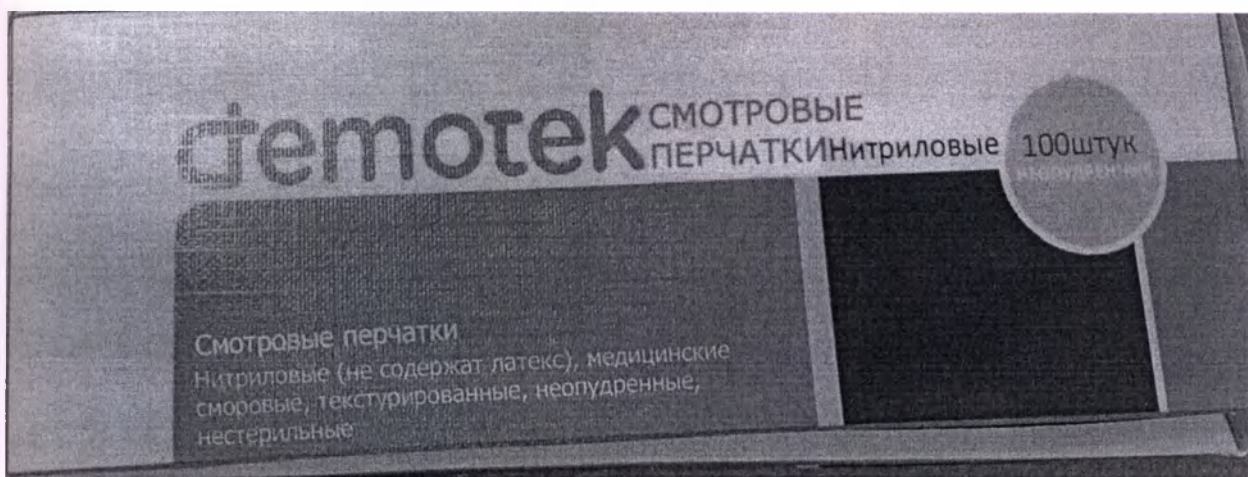
Demotek Nitrile Examination Gloves can be transported by all modes of transport in covered vehicles in accordance with the rules of transportation of cargo applicable to this mode of transport, at a temperature from +10°C to +40°C and at relative humidity no more than 80%.

9. Packing.


Demotek Nitrile Examination Gloves are packed in a cardboard box of 100 pcs.

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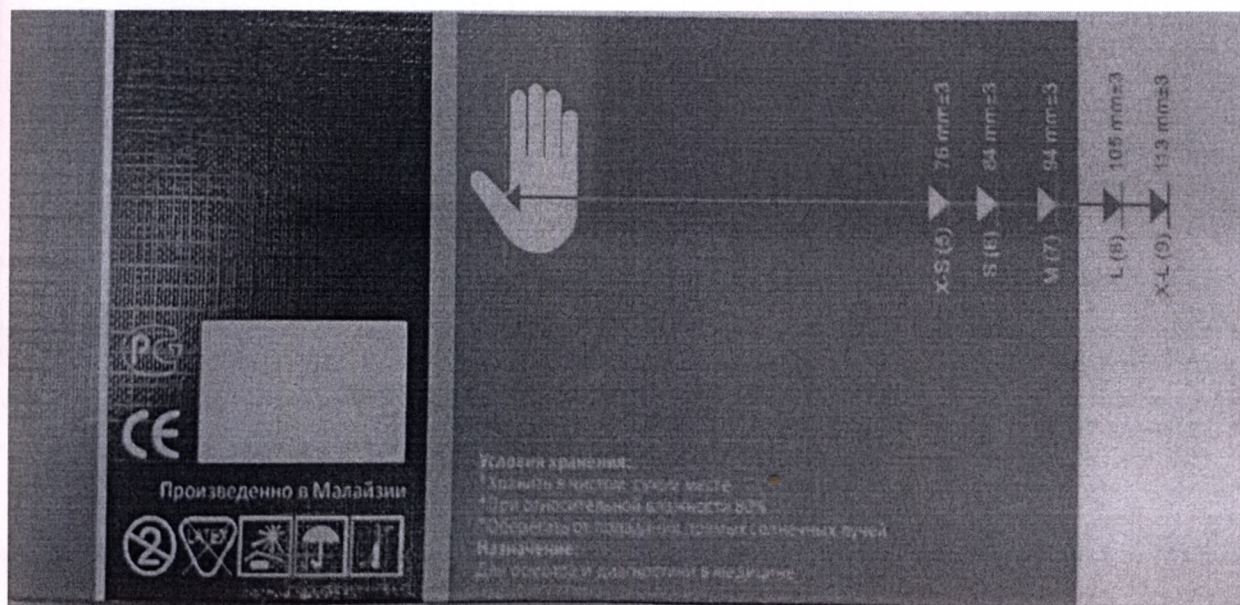On the product, the following marks and symbols are used:

- product name;
- REF;
- LOT;
- release date (year, month);
- shelf life (year, month);
- CE Conformity Marking;
- re-use is prohibited;
- size;
- without latex;
- protect from direct sunlight;
- protect from moisture;
- read the instructions before use;
- defect rate;
- manufacturer;
- manufacturer country;
- mark of conformity;
- storage conditions;
- importer to the Russian Federation;
- number of items in the package;
- «Powdered» and «Powder free»

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